
HOGGI[®]



BINGO EVOLUTION / BINGO SWITCH Spare parts catalogue

en

Index

1 Model Identification

1.1 Model identification	5
1.2 Basic Equipment	5

2 Spare parts

2.1 Mobility base swivelling front wheels	6-7
2.2 Mobility base for additional equipment	8
2.3 Mobility base trike	9
2.4 Mobility base Mini	10
2.5 Mobility base with steering wheel attachment BINGO SWITCH	11
2.6 Mobility base for additional devices BINGO SWITCH	12
2.7 Mobility base with steering wheel BINGO SWITCH CROSS	13
2.8 Mobility base for additional devices BINGO SWITCH CROSS	14
2.9 Mobility base with steering wheel Bingo Evolution CROSS	15
2.10 Mobility base for additional equipment Bingo Evolution CROSS	16
2.11 Seat unit	17
2.12 Seat Mini	18
2.13 Frame	19-20
2.14 Front frame for additional	21
2.15 Trike frame	22
2.16 Flame Mini	23
2.17 Holding Frame SWITCH	24
2.18 Holding Frame	25
2.19 Front bar	26
2.20 Frame suspension system	27
2.21 Spring set	28
2.22 Rear frame	29
2.23 Rear frame Evo CROSS	30
2.24 Brake	31
2.25 Push bar	32-33
2.26 Seat Bearing left	34-36
2.27 Side covers	37
2.28 Tie down kit	38
2.29 Tie down kit Mini	39
2.30 Castor fork with fasteners	40
2.31 Aluminium front forks	41
2.32 Swivel lock for two castor fork`s	42
2.33 Swivel lock	43
2.34 Steering wheels	44
2.35 Center steering wheel attachment	45
2.36 Center wheel	46
2.37 Pneumatic wheel	47
2.38 PU - wheel	48
2.39 Pneumatic wheel	49
2.40 Drum brake wheels	50-51
2.41 Pneumatic wheels black	52
2.42 Seat base Mini	53
2.43 Releaser, tilting mechanism	54
2.44 Releaser, tilting mechanism Mini	55
2.45 Seat surface	56
2.46 Seat surface Mini	57

Index

2 Spare parts

2.47 Hip support right Mini	58
2.48 Adaptive hight adjustable arm rests	59
2.49 Pelotte thigh support Mini	60
2.50 Foot rest hanger	61-63
2.51 Foot rest hanger Mini	64
2.52 Foot rest hanger angle adjustable	65-66
2.53 Foot rest hanger	67
2.54 Foot board assembly	68
2.55 Foot rest flip up lock	69
2.56 Foot board assembly angle adjustable and foldable	70
2.57 Evo foot rest with heel stop	71
2.58 Back base	72
2.59 Back base OT	73
2.60 Back base OT Plus	74
2.61 Lateral support left	75-76
2.62 Back insert	77
2.63 OT Back insert	78
2.64 Back insert Mini	79
2.65 Back base Mini	80
2.66 Upholstery Mini	81
2.67 Calf pad support plate	82
2.68 Anatomical neck-head	83
2.69 Trunk suport	84
2.70 Trunk suport Mini	85
2.71 Abduction block	86-87
2.72 Bingo Evo abduction block	88
2.73 Retainer for accessories	89
2.74 Retainer Mini	90
2.75 Bingo Evo retainer	91
2.76 Bingo Evo retainer Mini	92
2.77 Grab rail with upholstery Mini	93
2.78 Therapy tray Mini	94
2.79 Therapy tray Mini, transparent	95
2.80 Therapy tray Mini,clear, height adjustable	96
2.81 Therapy tray Mini, transparent, height adjustable	97
2.82 Attendant brake, mounted at the Push bar	98-99
2.83 Canopy incl. adapter and rain protection	100
2.84 Headrest pads Mini	101
2.85 Evo anti tip left and right	102
2.86 Evo CROSS anti tip left and right	103
2.87 Hip belt	104
2.88 Bingo Evo 4-point lap belt	105
2.89 Five point harness	106
2.90 Foot straps	107
2.91 Angle huggers	108
2.92 Groin strap	109
2.93 Fixation jacket	110
2.94 Chest-shoulder harness	111
2.95 Chest-shoulder harnes with 4 buckels	112
2.96 „Soft“ accessories bag	113


2.96 Large accessories bag	113-114
2.97 Tray for additional equipment	115
2.98 Evo CROSS tray for additional equipment	116
2.99 Oxygen bottle holder lang,right and hand attachment	117
2.100 Oxygen bottle holder lang, adjustable angle holders	118
2.101 Oxygen bottle holder lang,right and hand attachment	119

Model Identification

The mobility base type label is attached to the inside of the right-side strut.

The seat unit type label is located on the back of the seat base.


Please provide the serial number when ordering spare parts to ensure smooth delivery.

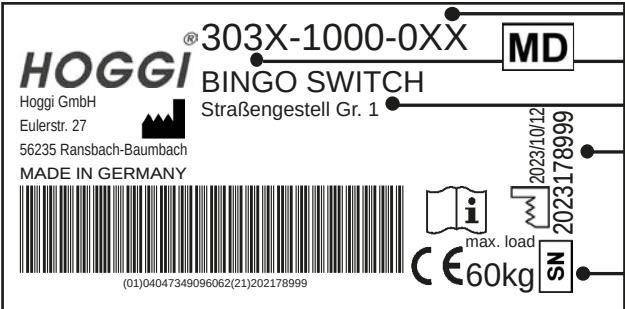	Color code
	Article number
	Size
	Serial number
	CE-marking
	Model designation / Product name
	Maximum user weight / Maximum load capacity
	Manufacturer
	Place of manufacture / Manufacturing location
	Year of manufacture / Manufacturing location

The basic equipment includes:


- BINGO EVOLUTION and BINGO SWITCH rehab stroller (mobility base and seat unit)
- Front and rear wheels with air or PU tires
- Upholstery
- Pelotte thigh support
- Foot rest hanger
- Footplate holder and footplate

Mobility base swivelling front wheels

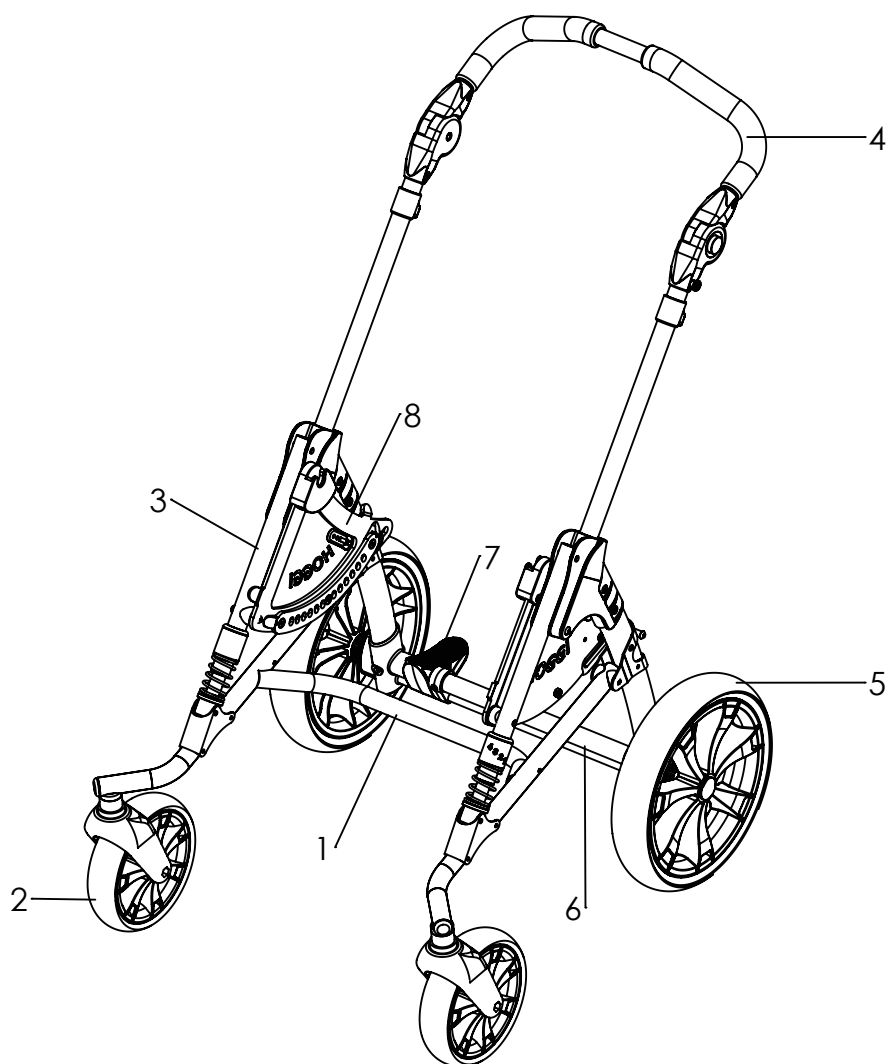


Pos	Article number	Articel description	Page
1-8	3232-1011-xxx	Mobility base Mini swivelling wheels	
	3232-1000-xxx	Mobility base size 1 swivelling wheels	
	3234-1000-xxx	Mobility base size 2 swivelling wheels	
	3235-1000-xxx	Mobility base size 2XL swivelling wheels	
1	3232-1100E-xxx	Steering wheel frame size 1	19
	3234-1100E-xxx	Steering wheel frame size 2	
2	12xx-	Front wheel	48
3	3231-1300E-xxx	Front bar	26
	3231-7400E-xxx	Frame suspension system	27
4	3231-1400E-xxx	Push handle size 1	32
	3233-1400E-xxx	Push handle size 2	
	3235-1400E-xxx	Push handle size 2XL	
5	12xx-	Rear wheel	48
6	3231-1200E-xxx	Rear Frame size 1	29
	3233-1200E-xxx	Rear Frame size 2	
7	3232-1300E-032	wheel lock assembly size 1	31
	3234-1300E-032	wheel lock assembly size 2	
8	3230-1510E-032	Bearing (seat), Mini left	36
	3230-1520E-032	Bearing (seat), Mini right	
	3231-1510E-032	Bearing (seat), size 1 and 2 left	34
	3231-1520E-032	Bearing (seat), size 1 and 2 right	
	3235-1510E-032	Bearing (seat), size 2XL left	35
	3235-1520E-032	Bearing (seat), size 2XL right	

Mobility base swivelling front wheels

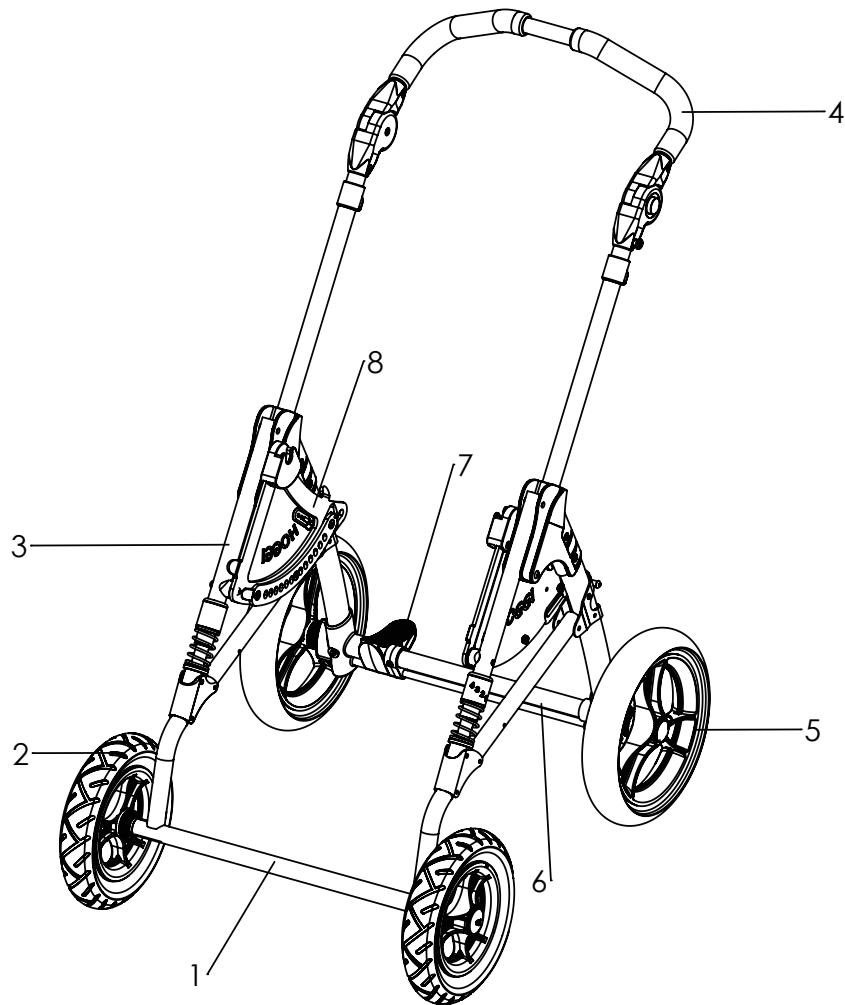


Pos	Article number	Articel description	Page
1-8	3231-1011-032	Mobility base Mini rigid front wheels	
	3231-1000-xxx	Mobility base size 1 rigid front wheels	
	3233-1000-xxx	Mobility base size 2 rigid front wheels	
1	3231-1100E-xxx	Rigid frame size 1 with fasteners	20
	3233-1100E-xxx	Rigid frame size 2 with fasteners	
2	12xx-	Front wheel	48
3	3231-1300E-xxx	Front bar	26
	3231-7400E-xxx	Frame suspension system	27
4	3231-1400E-xxx	Push handle size 1	32
	3233-1400E-xxx	Push handle size 2	
	3235-1400E-xxx	Push handle size 2XL	
5	12xx-	Rear wheel	48
6	3231-1200E-xxx	Rear Frame size 1	29
	3233-1200E-xxx	Rear Frame size 2	
7	3232-1300E-032	Wheel lock assembly size 1	31
	3234-1300E-032	Wheel lock assembly size 2	
8	3230-1510E-032	Bearing (seat), Mini left	36
	3230-1520E-032	Bearing (seat), Mini right	
	3231-1510E-032	Bearing (seat), size 1 and 2 left	34
	3231-1520E-032	Bearing (seat), size 1 and 2 right	

Mobility base for additional equipment

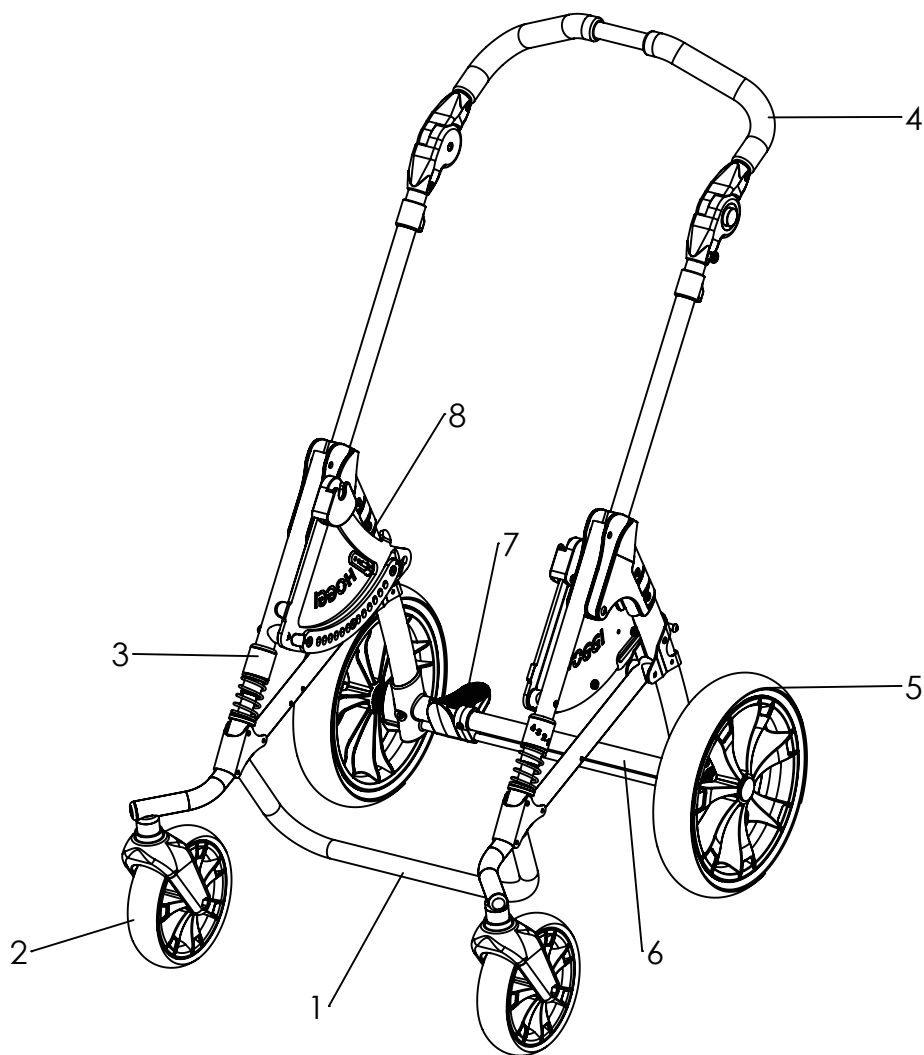


Pos	Article number	Articel description	Page
1-8	3238-1011-xxx	Mobility base for additional equipment Mini	
	3238-1000-xxx	Mobility base for additional equipment size 1	
	3239-1000-xxx	Mobility base for additional equipment size 2	
	3239-1006-xxx	Mobility base for additional equipment size 2XL	
1	3238-1100E-xxx	Front frame for additional equipment size 1	21
	3239-1100E-xxx	Front frame for additional equipment size 2	
2	12xx-	Front wheel	48
3	3231-1300E-xxx	Front bar	26
	3231-7400E-xxx	Frame suspension system	27
4	3231-1400E-xxx	Push handle size 1	32
	3233-1400E-xxx	Push handle size 2	
	3235-1400E-xxx	Push handle size 2XL	
5	12xx-	Rear wheel	48
6	3231-1200E-xxx	Rear Frame size 1	29
	3233-1200E-xxx	Rear Frame size 2	
7	3232-1300E-032	Wheel lock assembly size 1	31
	3234-1300E-032	Wheel lock assembly size 2	
8	3230-1510E-032	Bearing (seat), Mini left	36
	3230-1520E-032	Bearing (seat), Mini right	
	3231-1510E-032	Bearing (seat), size 1 and 2 left	34
	3231-1520E-032	Bearing (seat), size 1 and 2 right	
	3235-1510E-032	Bearing (seat), size 2XL left	35
	3235-1520E-032	Bearing (seat), size 2XL right	

Mobility base trike

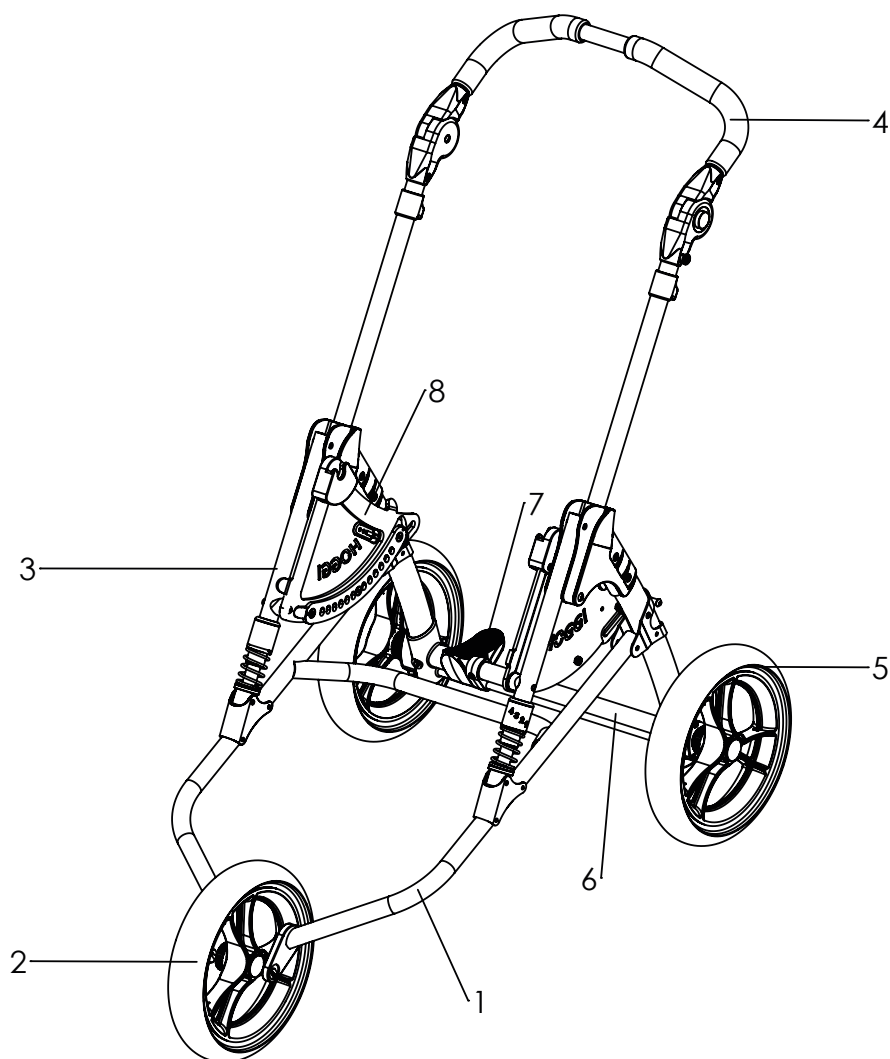


Pos	Article number	Articel description	Page
1-8	3236-1011-032	TRIKE mobility base Mini	
	3236-1000-xxx	TRIKE mobility base size 1	
	3237-1000-xxx	TRIKE mobility base size 2	
1	3236-1100E-xxx	TRIKE-front frame size 1	22
	3237-1100E-xxx	TRIKE-front frame size 2	
2	12xx-	Front wheel	48
3	3231-1300E-xxx	Front bar	26
	3231-7400E-xxx	Frame suspension system	27
4	3231-1400E-xxx	Push handle size 1	32
	3233-1400E-xxx	Push handle size 2	
	3235-1400E-xxx	Push handle size 2XL	
5	12xx-	Rear wheel	48
6	3231-1200E-xxx	Rear frame size 1	29
	3233-1200E-xxx	Rear frame size 2	
7	3232-1300E-032	wheel lock assembly size 1	31
	3234-1300E-032	wheel lock assembly size 2	
8	3230-1510E-032	Bearing (seat), Mini left	36
	3230-1520E-032	Bearing (seat), Mini right	
	3231-1510E-032	Bearing (seat), size 1 and 2 left	34
	3231-1520E-032	Bearing (seat), size 1 and 2 right	

Mobility base Mini

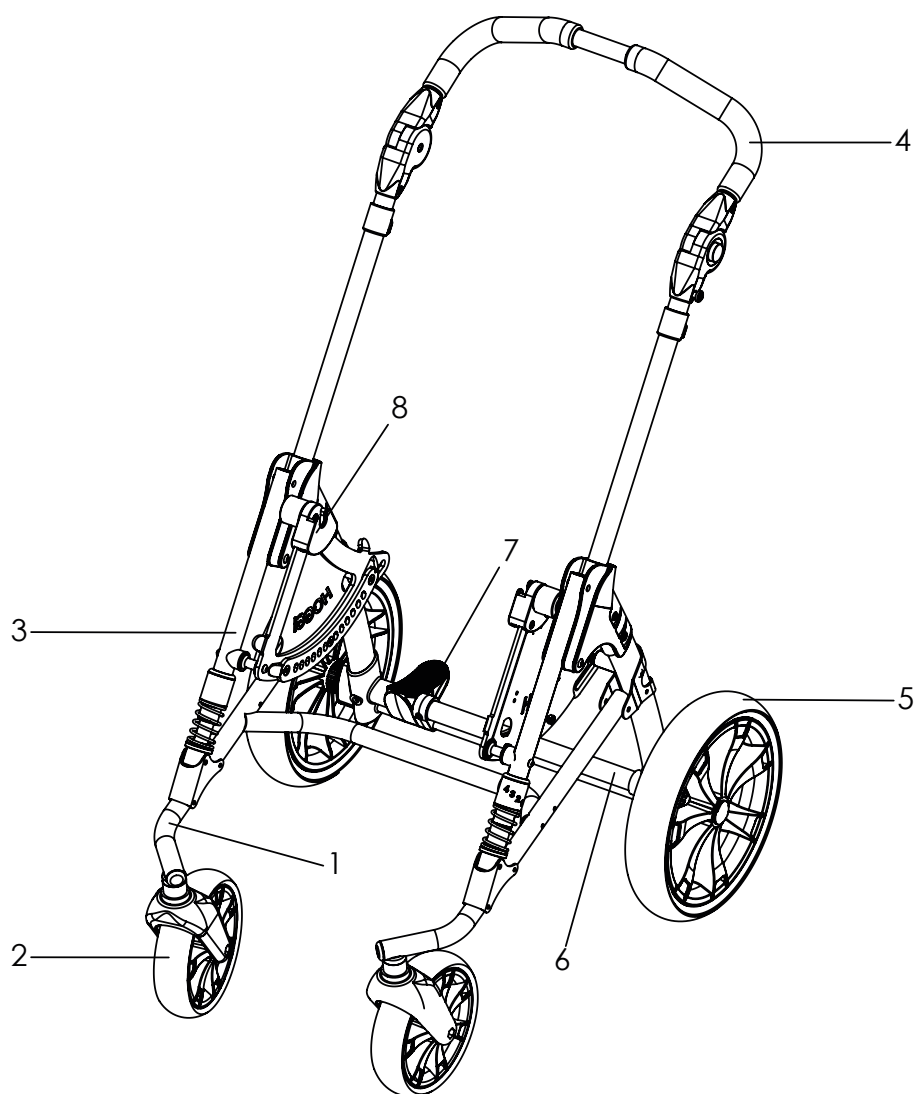


Pos	Article number	Articel description	Page
1-8	3230-1000-032	BINGO-Evolution mobility base Mini (invert), swivelling front wheels	
1	3230-1100E-032	Frame Mini invert (swivelling front wheels)	23
2	12xx-	Front wheel	48
3	3231-1300-032	Front bar	26
	3231-7400E-032	Frame suspension system	27
4	3231-1400-032	Push handle size 1	32
5	12xx-	Rear wheel	48
6	3231-1200E-032	Rear frame size 1	29
7	3232-1300E-032	Brake size 1	31
8	3230-1510E-032	Bearing (seat), Mini left	36
	3230-1520E-032	Bearing (seat), Mini right	

Mobility base with steering wheel attachment

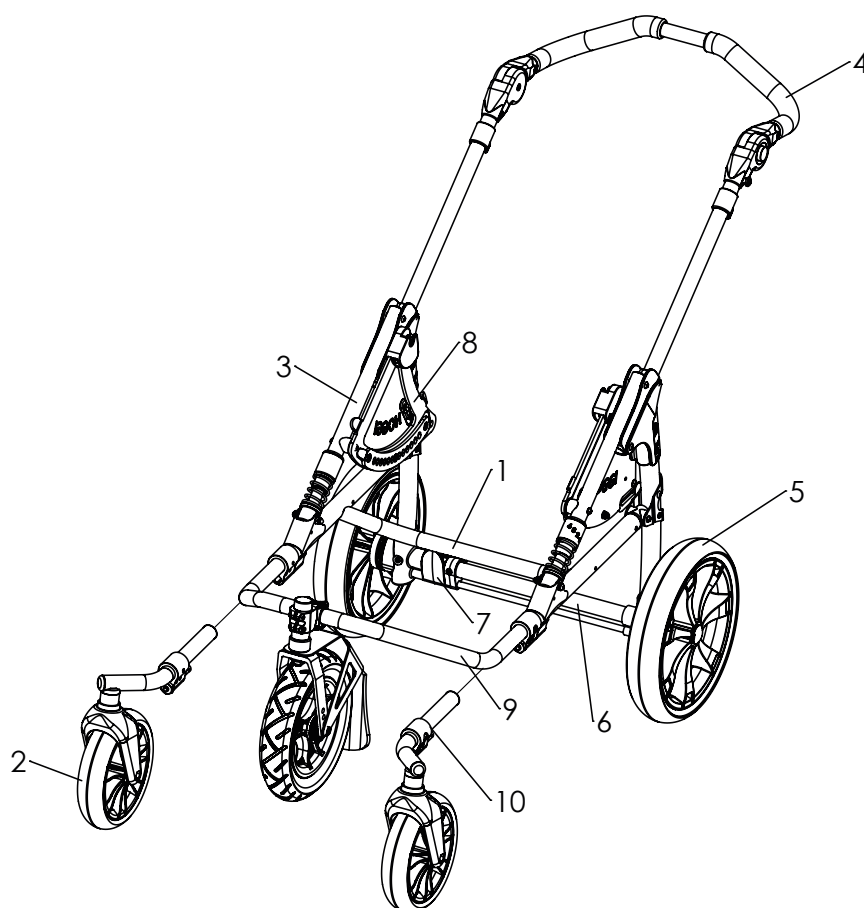


Pos	Article number	Articel description	Page
1-10	3030-1000-xxx	Mobility base mini with steering wheel attachment	
	3030-1020-xxx	Mobility base mini with center steering wheel attachment	
	3032-1000-xxx	Mobility base size 1 with steering wheel attachment	
	3032-1020-xxx	Mobility base size 1 with center steering wheel attachment	
	3034-1000-xxx	Mobility base size 2 with steering wheel attachment	
	3034-1020-xxx	Mobility base size 2 with center steering wheel attachment	
	3036-1000-xxx	Mobility base size 2XL with steering wheel attachment	
	3036-1020-xxx	Mobility base size 2XL with center steering wheel attachment	
1	3032-1100E-xxx	Holding Frame Switch size 1	24
	3034-1100E-xxx	Holding Frame Switch size 2	
2	12xx-	Front wheels	48
3	3031-1301E-xxx	Frame without suspension system	26
	3031-7400E-xxx	Frame suspension system, fourfold adjustable	27
4	3231-1400E-xxx	Push handle size 1 complete with fasteners	32
	3233-1400E-xxx	Push handle size 2 complete with fasteners	
	3235-1400E-xxx	Push handle size 2XL complete with fasteners	
5	12xx-	Rear wheels	48
6	3231-1200E-xxx	Rear Frame size 1 with fasteners	29
	3233-1200E-xxx	Rear Frame size 2 with fasteners	
7	3232-1300E-032	Wheel lock assembly with spring-loaded brake bars size 1	31
	3234-1300E-032	Wheel lock assembly with spring-loaded brake bars size 2	
8	3230-1510E-032	Seat Bearing Mini left with fasteners	36
	3230-1520E-032	Seat Bearing Mini right with fasteners	
	3231-1510E-032	Seat Bearing size 1 and 2 left with fasteners	34
	3231-1520E-032	Seat Bearing size 1 and 2 right with fasteners	
	3235-1510E-032	Seat Bearing left 2XL with fasteners	35
	3235-1520E-032	Seat Bearing right 2XL with fasteners	
9	3032-1500E-xxx	Center wheel attachment BINGO SWITCH size 1	45
	3034-1500E-xxx	Center wheel attachment BINGO SWITCH size 2	
10	3032-1300E-xxx	Steering wheels Interchangeable attachment Switch	44

Mobility base for additional devices BINGO SWITCH

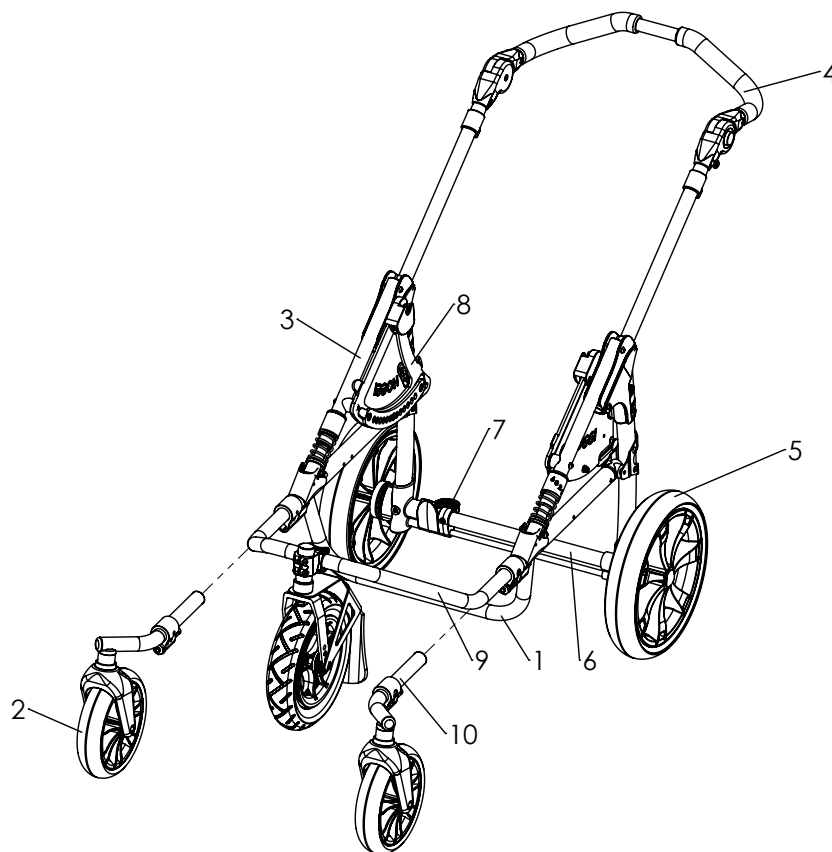


Pos	Article number	Articel description	Page
1-10	3031-1000-xxx	Mobility base for additional devices mini with steering wheel	
	3031-1020-xxx	Mobility base for additional devices mini with center steering wheel	
	3033-1000-xxx	Mobility base for additional devices size 1 with steering wheel	
	3033-1020-xxx	Mobility base for additional devices size 1 with center steering wheel	
	3035-1000-xxx	Mobility base for additional devices size 2 with steering wheel	
	3035-1020-xxx	Mobility base for additional devices size 2 with center steering wheel	
	3037-1000-xxx	Mobility base for additional devices size 2XL with steering wheel	
	3037-1020-xxx	Mobility base for additional devices size 2XL with center steering wheel	
1	3033-1100E-xxx	Holding Frame for additional equipment Switch size 1	25
	3035-1100E-xxx	Holding Frame for additional equipment Switch size 2	
2	12xx-	Front wheels	48
3	3031-1301E-xxx	Frame without suspension system	26
	3031-7400E-xxx	Frame suspension system, fourfold adjustable	27
4	3231-1400E-xxx	Push handle size 1 complete with fasteners	32
	3233-1400E-xxx	Push handle size 2 complete with fasteners	
	3235-1400E-xxx	Push handle size 2XL complete with fasteners	
5	12xx-	Rear wheels	48
6	3231-1200E-xxx	Rear Frame size 1 with fasteners	29
	3233-1200E-xxx	Rear Frame size 2 with fasteners	
7	3232-1300E-032	Wheel lock assembly with spring-loaded brake bars size 1	31
	3234-1300E-032	Wheel lock assembly with spring-loaded brake bars size 2	
8	3230-1510E-032	Seat Bearing Mini left with fasteners	36
	3230-1520E-032	Seat Bearing Mini right with fasteners	
	3231-1510E-032	Seat Bearing size 1 and 2 left with fasteners	34
	3231-1520E-032	Seat Bearing size 1 and 2 right with fasteners	
	3235-1510E-032	Seat Bearing left 2XL with fasteners	35
	3235-1520E-032	Seat Bearing right 2XL with fasteners	
9	3032-1500E-xxx	Center wheel attachment BINGO SWITCH size 1	45
	3034-1500E-xxx	Center wheel attachment BINGO SWITCH size 2	
10	3032-1300E-xxx	Steering wheels Interchangeable attachment Switch	44

Mobility base with steering wheel BINGO SWITCH CROSS

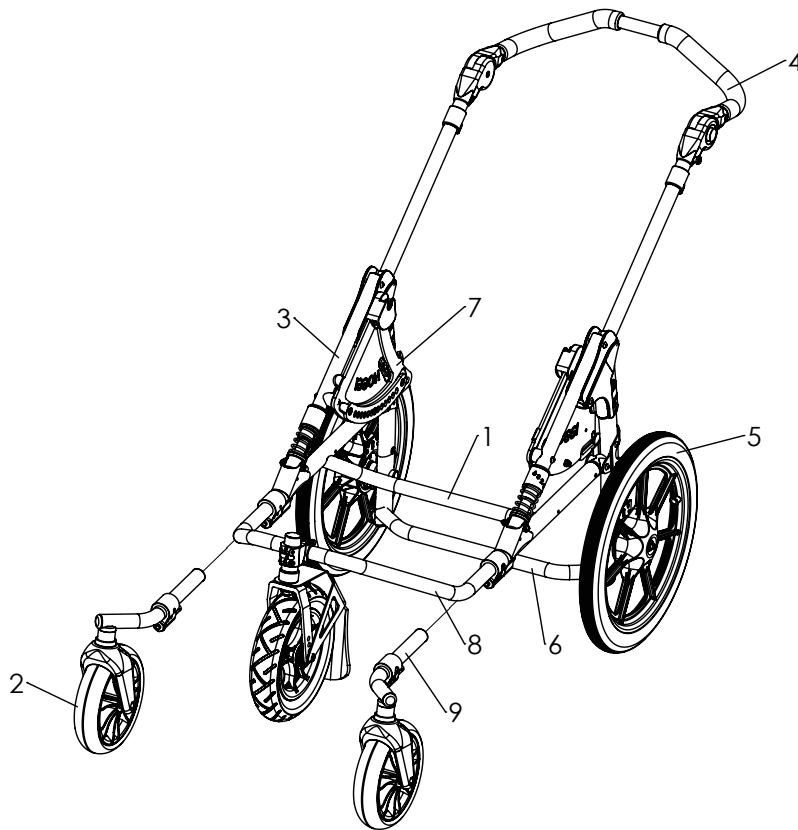


Pos	Article number	Articel description	Page
1-10	3030-1030-xxx	Mobility base mini with steering wheel	
	3030-1040-xxx	Mobility base mini with center steering wheel	
	3032-1030-xxx	Mobility base 1 with steering wheel	
	3032-1040-xxx	Mobility base 1 with center steering wheel	
	3034-1030-xxx	Mobility base 2 with steering wheel	
	3034-1040-xxx	Mobility base 2 with center steering wheel	
	3036-1030-xxx	Mobility base 2XL with steering wheel	
	3036-1040-xxx	Mobility base 2XL with center steering wheel	
1	3032-1100E-xxx	Holding Frame Switch size 1	24
	3034-1100E-xxx	Holding Frame Switch size 2	
2	12xx-	Front wheels	48
3	3031-1301E-xxx	Frame without suspension system	26
	3031-7400E-xxx	Frame suspension system, fourfold adjustable	27
4	3231-1400E-xxx	Push handle size 1 complete with fasteners	32
	3233-1400E-xxx	Push handle size 2 complete with fasteners	
	3235-1400E-xxx	Push handle size 2XL complete with fasteners	
5	12xx-	Rear wheels	48
6	3232-1200E-xxx	Rear Frame EVO CROSS size 1	30
	3234-1200E-xxx	Rear Frame EVO CROSS size 2	
7	3232-1290E	Brake components EVO CROSS	52
8	3230-1510E-032	Seat Bearing Mini left with fasteners	36
	3230-1520E-032	Seat Bearing Mini right with fasteners	
	3231-1510E-032	Seat Bearing size 1 and 2 left with fasteners	34
	3231-1520E-032	Seat Bearing size 1 and 2 right with fasteners	
	3235-1510E-032	Seat Bearing left 2XL with fasteners	35
	3235-1520E-032	Seat Bearing right 2XL with fasteners	
9	3032-1500E-xxx	Center wheel attachment BINGO SWITCH size 1	45
	3034-1500E-xxx	Center wheel attachment BINGO SWITCH size 2	
10	3032-1300E-xxx	Steering wheels Interchangeable attachment Switch	44

Mobility base for additional devices, BINGO SWITCH CROSS

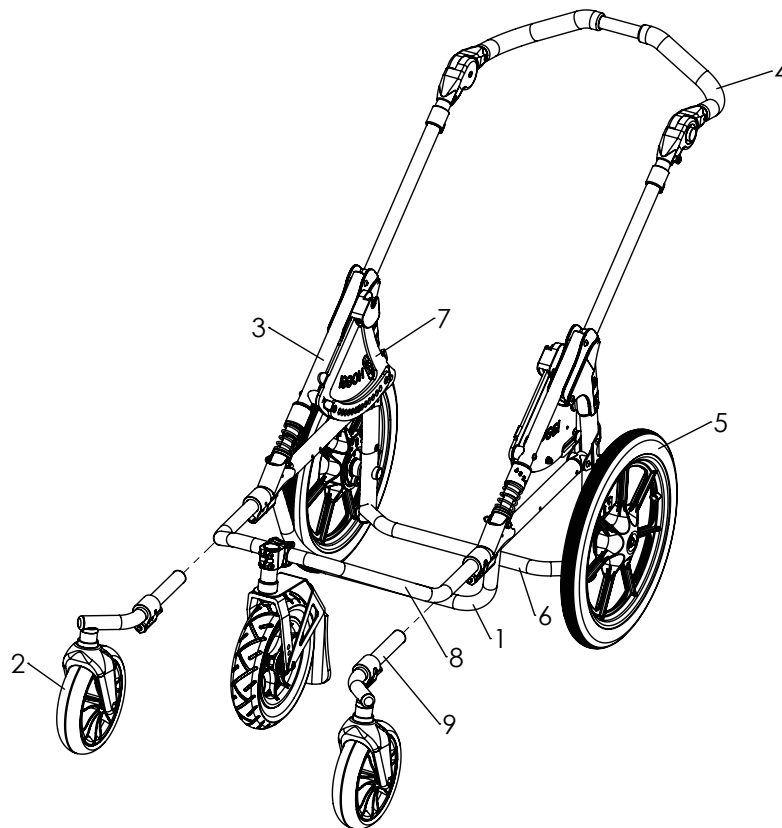


Pos	Article number	Articel description	Page
1-10	3031-1030-xxx	Mobility base mini for additional devices with steering wheel	
	3031-1040-xxx	Mobility base mini for additional devices with center steering wheel	
	3033-1030-xxx	Mobility base for additional devices size 1 with steering wheel	
	3033-1040-xxx	Mobility base for additional devices size 1 with center steering wheel	
	3035-1030-xxx	Mobility base for additional devices size 2 with steering wheel	
	3035-1040-xxx	Mobility base for additional devices size 2 with center steering wheel	
	3037-1030-xxx	Mobility base for additional devices size 2XL with steering wheel	
	3037-1040-xxx	Mobility base for additional devices size 2XL with center steering wheel	
1	3033-1100E-xxx	Holding Frame for additional equipment Switch size 1	25
	3035-1100E-xxx	Holding Frame for additional equipment Switch size 2	
2	12xx-	Front wheels	48
3	3031-1301E-xxx	Frame without suspension system	26
	3031-7400E-xxx	Frame suspension system, fourfold adjustable	27
4	3231-1400E-xxx	Push handle size 1 complete with fasteners	32
	3233-1400E-xxx	Push handle size 2 complete with fasteners	
	3235-1400E-xxx	Push handle size 2XL complete with fasteners	
5	12xx-	Rear wheels	48
6	3232-1200E-xxx	Rear Frame EVO CROSS size 1	30
	3234-1200E-xxx	Rear Frame EVO CROSS size 2	
	3232-1290E	Brake components EVO CROSS	52
8	3230-1510E-032	Seat Bearing Mini left with fasteners	36
	3230-1520E-032	Seat Bearing Mini right with fasteners	
	3231-1510E-032	Seat Bearing size 1 and 2 left with fasteners	34
	3231-1520E-032	Seat Bearing size 1 and 2 right with fasteners	
	3235-1510E-032	Seat Bearing left 2XL with fasteners	35
	3235-1520E-032	Seat Bearing right 2XL with fasteners	
9	3032-1500E-xxx	Center wheel attachment BINGO SWITCH size 1	45
	3034-1500E-xxx	Center wheel attachment BINGO SWITCH size 2	
10	3032-1300E-xxx	Steering wheels Interchangeable attachment Switch	44

Mobility base with swivelling wheels Bingo Evo CROSS

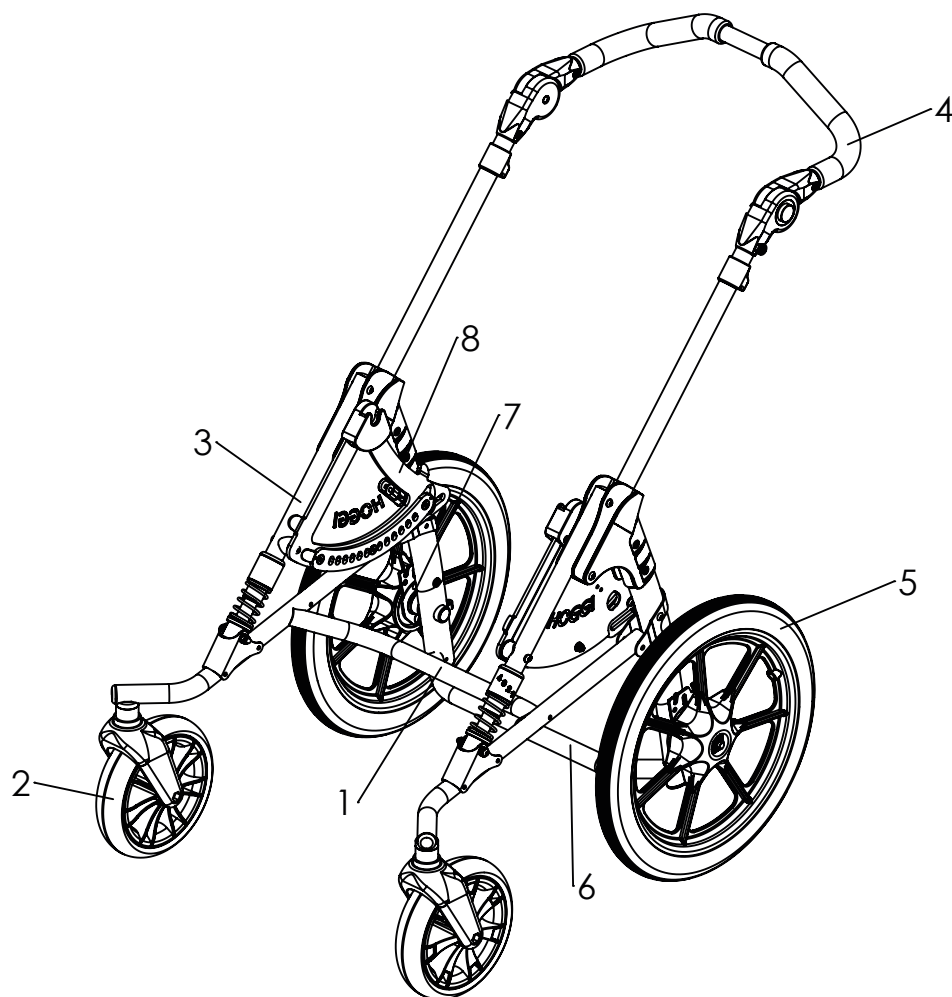


Pos	Article number	Articel description	Page
1-8	3232-1030-xxx	BINGO Evo CROSS mobility base 1 swivelling wheels	
	3234-1030-xxx	BINGO Evo CROSS mobility base 2 swivelling wheels	
	3235-1030-xxx	BINGO Evo CROSS mobility base 2-XL swivelling wheels	
1	3232-1100E-xxx	Frontframe for swivelling front wheels complete size 1	19
	3234-1100E-xxx	Frontframe for swivelling front wheels complete size 2	
2	12xx-	Front wheels	48
3	3031-1301E-xxx	Frame without suspension system	26
	3031-7400E-xxx	Frame suspension system, fourfold adjustable	27
4	3231-1400E-xxx	Push handle size 1 complete with fasteners	32
	3233-1400E-xxx	Push handle size 2 complete with fasteners	
	3235-1400E-xxx	Push handle size 2XL complete with fasteners	
5	12xx-	Rear wheels	48
6	3232-1200E-xxx	Rear Frame EVO CROSS size 1	30
	3234-1200E-xxx	Rear Frame EVO CROSS size 2	
7	3232-1290E	Brake components EVO CROSS	52
8	3230-1510E-032	Seat Bearing Mini left with fasteners	36
	3230-1520E-032	Seat Bearing Mini right with fasteners	
	3231-1510E-032	Seat Bearing size 1 and 2 left with fasteners	34
	3231-1520E-032	Seat Bearing size 1 and 2 right with fasteners	
	3235-1510E-032	Seat Bearing left 2XL with fasteners	35
	3235-1520E-032	Seat Bearing right 2XL with fasteners	

Mobility base for additional equipment, Bingo Evo CROSS

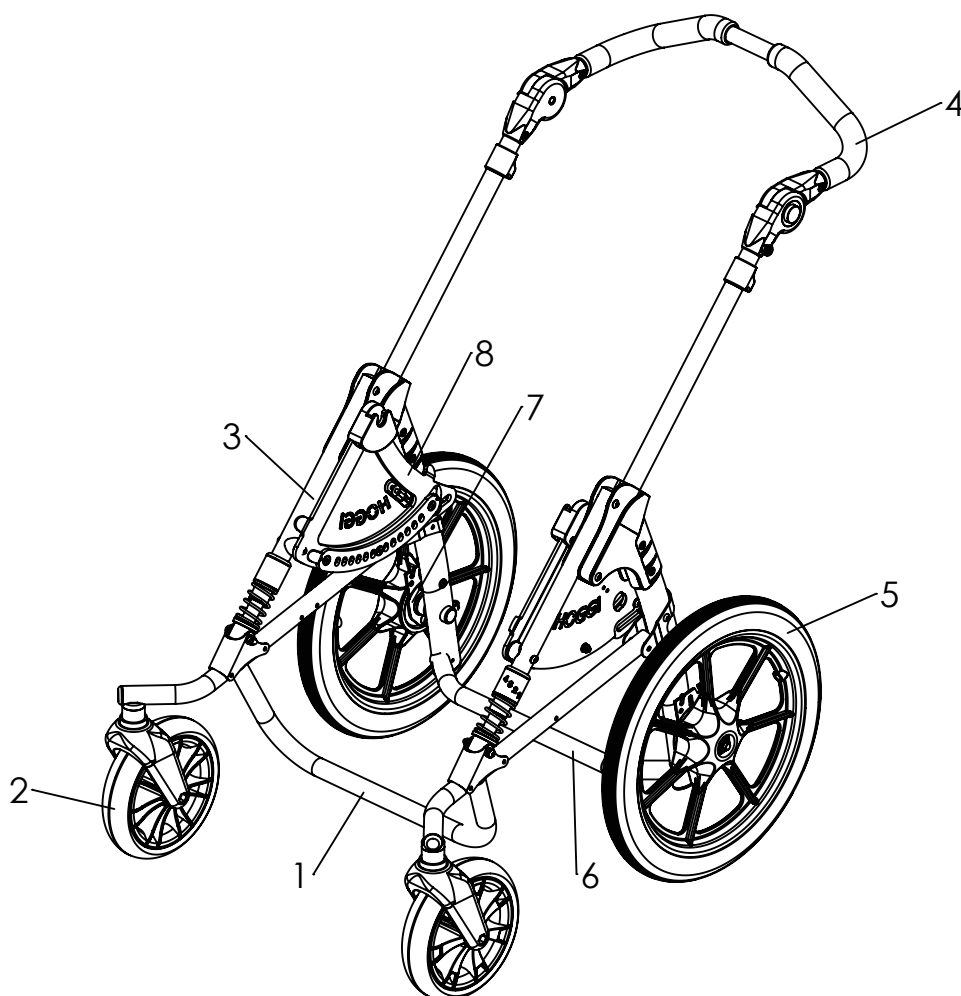


Pos	Article number	Articel description	Page
1-8	3238-1030-xxx	BINGO Evo CROSS mobility base for additional equipment size 1	
	3239-1030-xxx	BINGO Evo CROSS mobility base for additional equipment size 2	
	3239-1008-xxx	BINGO Evo CROSS mobility base for additional equipment size 2 XL	
1	3033-1100E-xxx	Holding Frame for additional equipment Switch size 1	21
	3035-1100E-xxx	Holding Frame for additional equipment Switch size 2	
2	12xx-	Front wheels	48
3	3031-1301E-xxx	Frame without suspension system	26
	3031-7400E-xxx	Frame suspension system, fourfold adjustable	27
4	3231-1400E-xxx	Push handle size 1 complete with fasteners	32
	3233-1400E-xxx	Push handle size 2 complete with fasteners	
	3235-1400E-xxx	Push handle size 2XL complete with fasteners	
5	12xx-	Rear wheels	48
6	3232-1200E-xxx	Rear Frame EVO CROSS size 1	30
	3234-1200E-xxx	Rear Frame EVO CROSS size 2	
7	3232-1290E	Brake components EVO CROSS	52
8	3230-1510E-032	Seat Bearing Mini left with fasteners	36
	3230-1520E-032	Seat Bearing Mini right with fasteners	
	3231-1510E-032	Seat Bearing size 1 and 2 left with fasteners	34
	3231-1520E-032	Seat Bearing size 1 and 2 right with fasteners	
	3235-1510E-032	Seat Bearing left 2XL with fasteners	35
	3235-1520E-032	Seat Bearing right 2XL with fasteners	

Spare parts

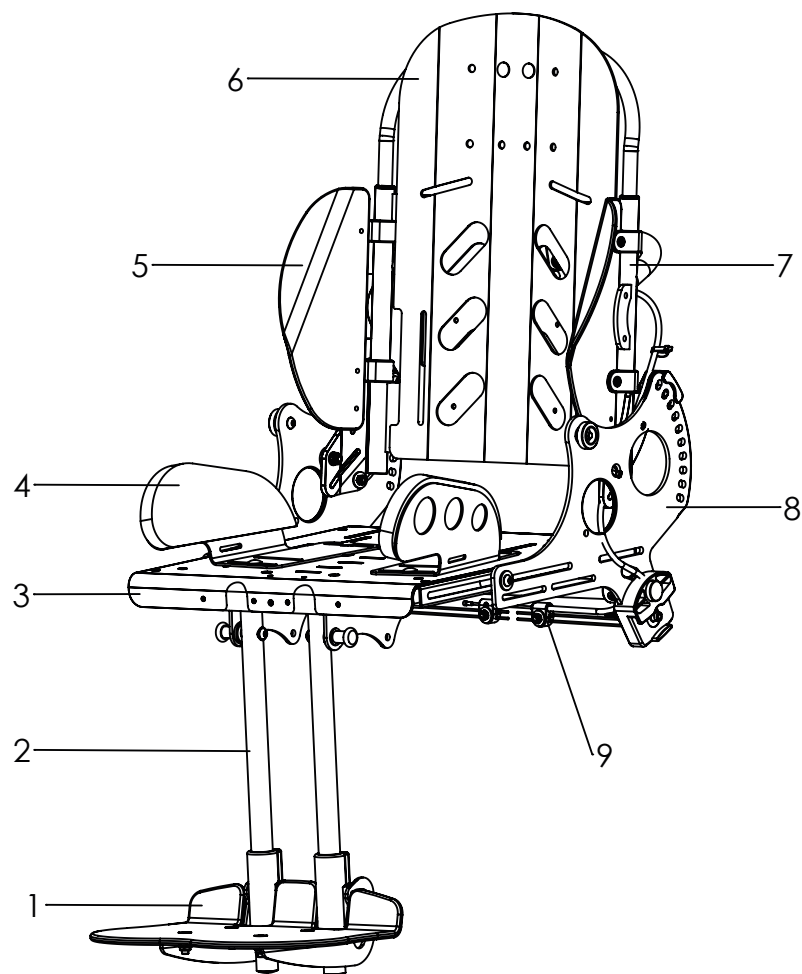


Pos	Article number	Articel description	Page
1-9	3232-2000-xxx	Seat unit, size 1	
	3234-2000-xxx	Seat unit, size 2	
	3235-2000-xxx	Seat unit, size 2XL	
1	323X-2XXXE	Footrest	67
2	323X-2X00E-xxx	Knee angle system	60
3	323X-2200-xxx	Seat surface	56
4	323X-2900E-xxx	Thigh support	59
	323X-2600E-xxx	Hip support	58
5	323X-27XX	Side pads	74
6	323X-2800E-xxx	Back insert	76
7	323X-2300E-xxx	Back base	71
8	323X-2100E-xxx	Seat base	53
9	3231-1900E	Tilt release mechanism	54

Spare parts

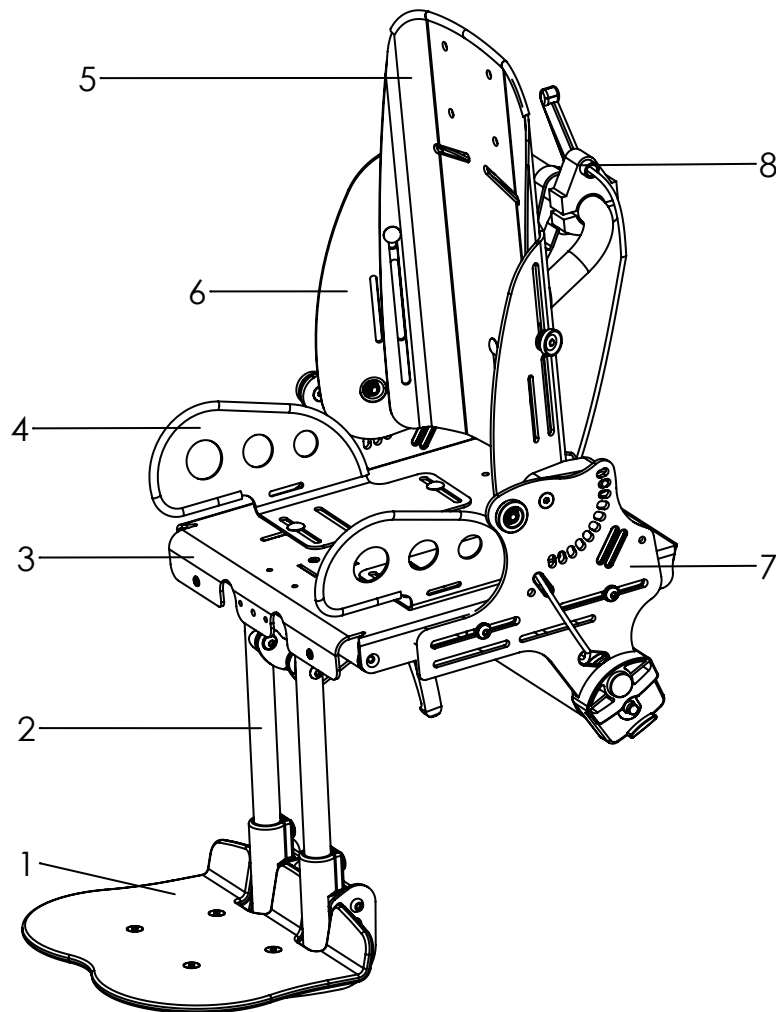


Pos	Article number	Articel description	Page
1-8	3230-2000-032	Seat Mini	
1	323X-2XXXE	Footrest	67
2	3230-2400-032	Knee angle system	60
3	3230-2200E-032	Seat surface size Mini	57
4	3230-290X-032	Thigh support mini	59
	3230-2600E-032	Hip supports mini	58
5	3230-2302E-032	Back insert Mini	79
6	3230-2301E-032	Back base mini	78
7	3230-2100E-032	Seat base Mini	53
8	3230-1900E	Tilt release mechanism Mini	54

Spare parts

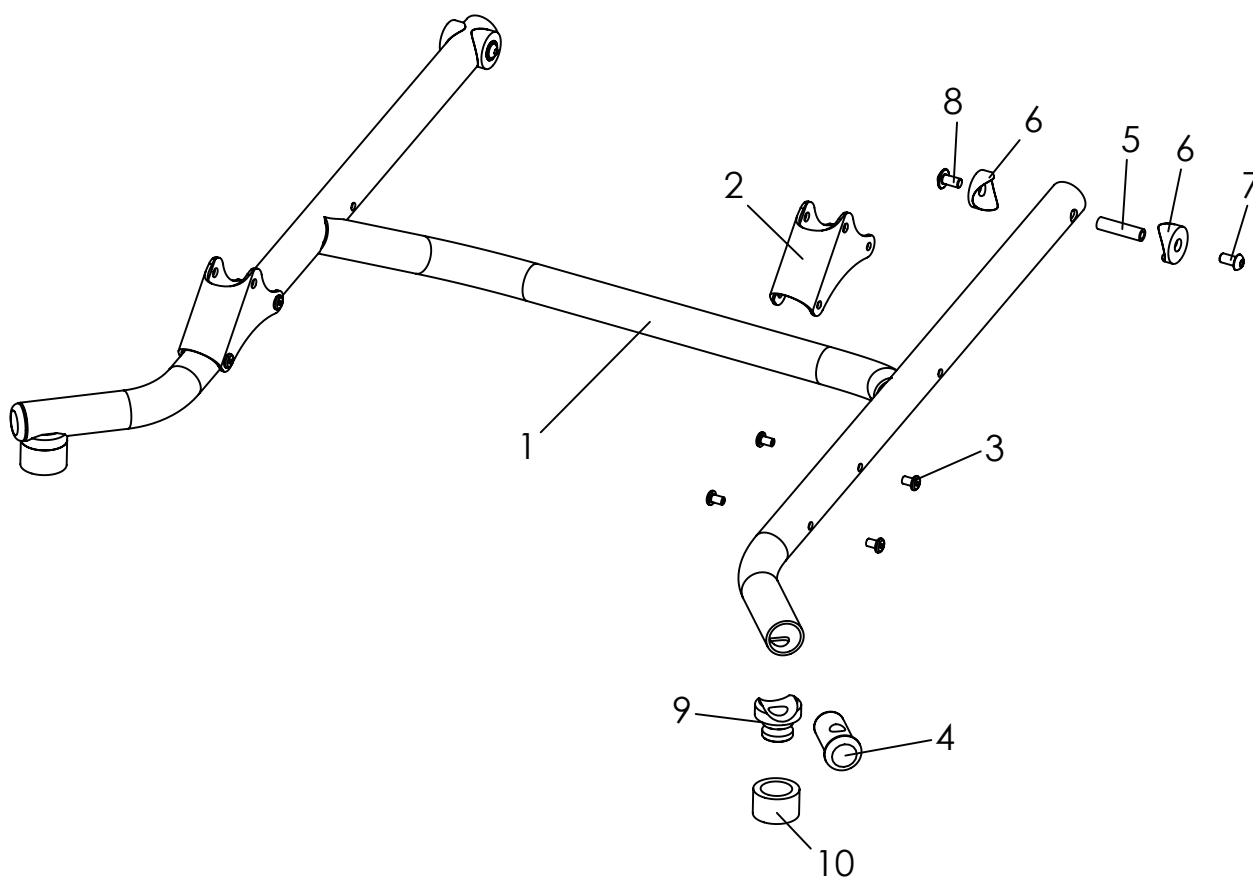


Pos	Article number	Articel description	Quantity	Unit
1-10	3232-1100E-xxx	Frame size 1 (swivel front wheels)	1	Pc.
	3234-1100E-xxx	Frame size 2 (swivel front wheels)	1	Pc.
1-4	3232-1110E-xxx	Frame size 1	1	Pc.
	3234-1110E-xxx	Frame size 2	1	Pc.
2	3231-1120-xxx	Front strut bearing	2	Pc.
3	1069-0004	Blind rivet F5x10 black	8	Pc.
4	3202-1130	Block	2	Pc.
5	1114-0027	Threaded bushing 8xM6x32	2	Pc.
6	1111-0012	Saddle washer PA black 24x8.4x3	4	Pc.
7	1029-6001-033	Button head socket screw black, Tufflock M6x12	2	Pc.
8	1030-0612	Button head socket screw with flange M6x12	2	Pc.
9	3202-1140	Saddle bushing for front caster	2	Pc.
10	3202-1151	Cover ring	2	Pc.

Spare parts

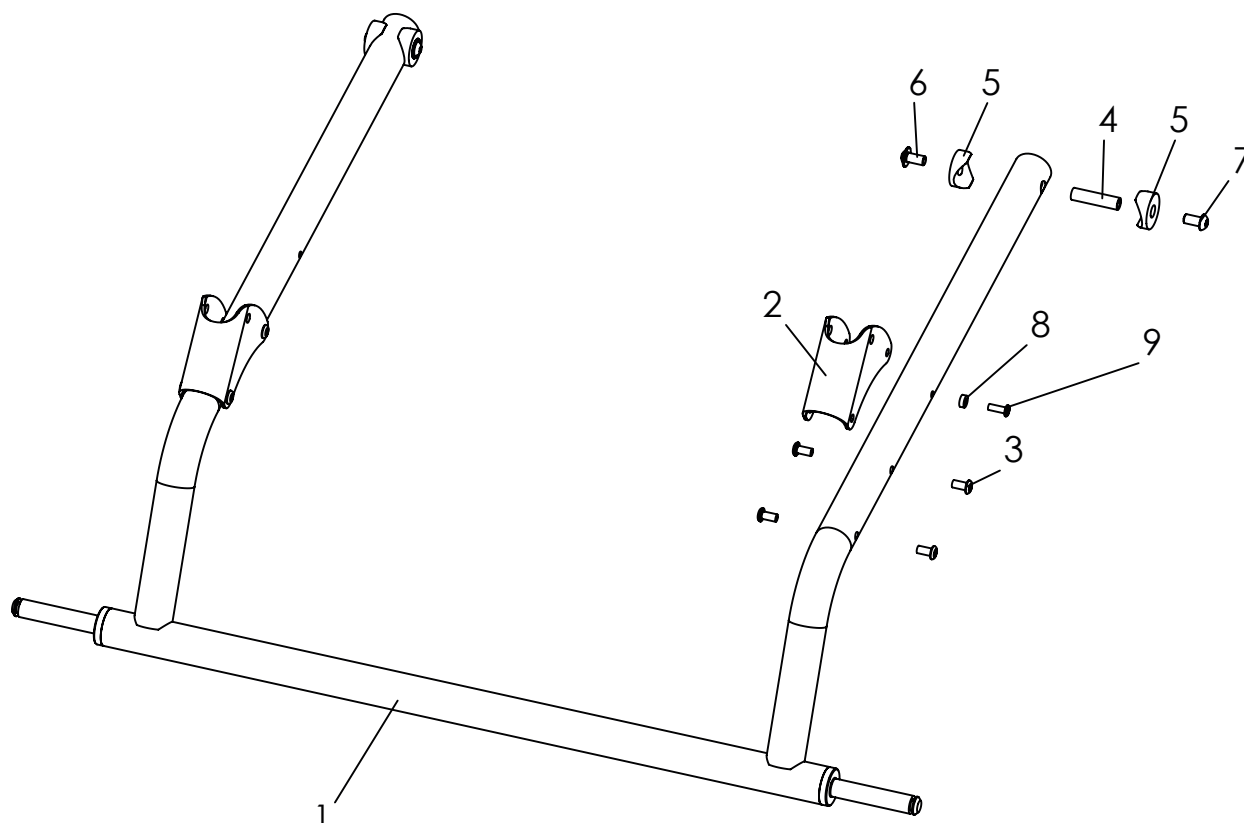


Pos	Article number	Articel description	Quantity	Unit
1-9	3231-1100E-xxx	Frame size 1	1	Pc.
	3233-1100E-xxx	Frame size 2	1	Pc.
1-3	3231-1110E-xxx	Frame size 1	1	Pc.
	3233-1110E-xxx	Frame size 2	1	Pc.
2	3231-1120-xxx	Front strut bearing	2	Pc.
3	1069-0004	Blind rivet F5x10	8	Pc.
4	1114-0027	Threaded bushing 8xM6x32	2	Pc.
5	1111-0012	Saddle washer PA black 24x8.4x3	4	Pc.
6	1030-0612	Button head socket screw with flange M6x12	2	Pc.
7	1029-6001-033	Buttonheadsocketscrew black, Tufflock M6x12	2	Pc.
8	1062-0412	Blind rivet F4x12	2	Pc.
9	1112-0010	Spacer PA black 8x4,3x3	2	Pc.

Spare parts

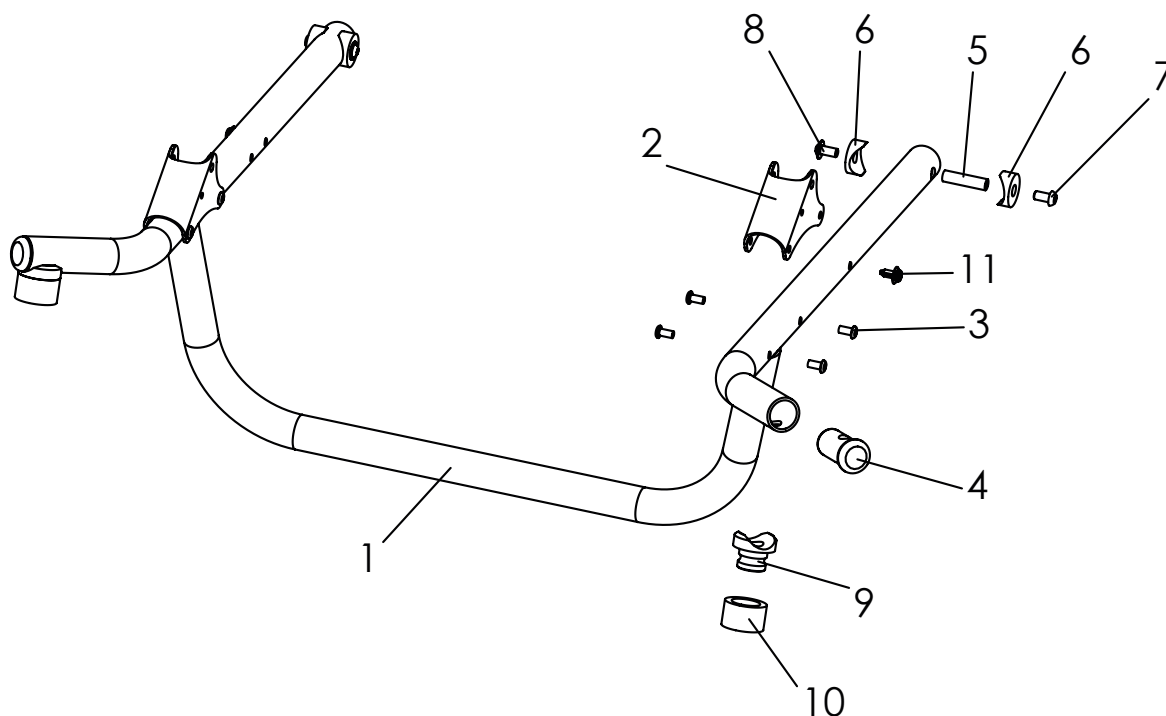


Pos	Article number	Articel description	Quantity	Unit
1-11	3238-1100E-xxx	Front frame for additional equipment size 1	1	Pc.
	3239-1100E-xxx	Front frame for additional equipment size 2	1	Pc.
1-4	3238-1110E-xxx	Front frame for additional equipment size 1	1	Pc.
	3239-1110E-xxx	Front frame for additional equipment size 2	1	Pc.
2	3231-1120-xxx	Front strut bearing	2	Pc.
3	1069-0004	Blind rivet F5x10 black	8	Pc.
4	3202-1130	Block	2	Pc.
5	1114-0027	Threaded bushing 8xM6x32	2	Pc.
6	1111-0012	Saddle washer PA black 24x8.4x3	4	Pc.
7	1029-6001-033	Buttonheadsocketscrew black, Tufflock M6x12	2	Pc.
8	1030-0612	Button head socket screw with flange M6x12	2	Pc.
9	3202-1140	Saddle bushing for front caster	2	Pc.
10	3202-1151	Cover ring	2	Pc.
11	1149-0008	Screw rivet PA	2	Pc.

Spare parts

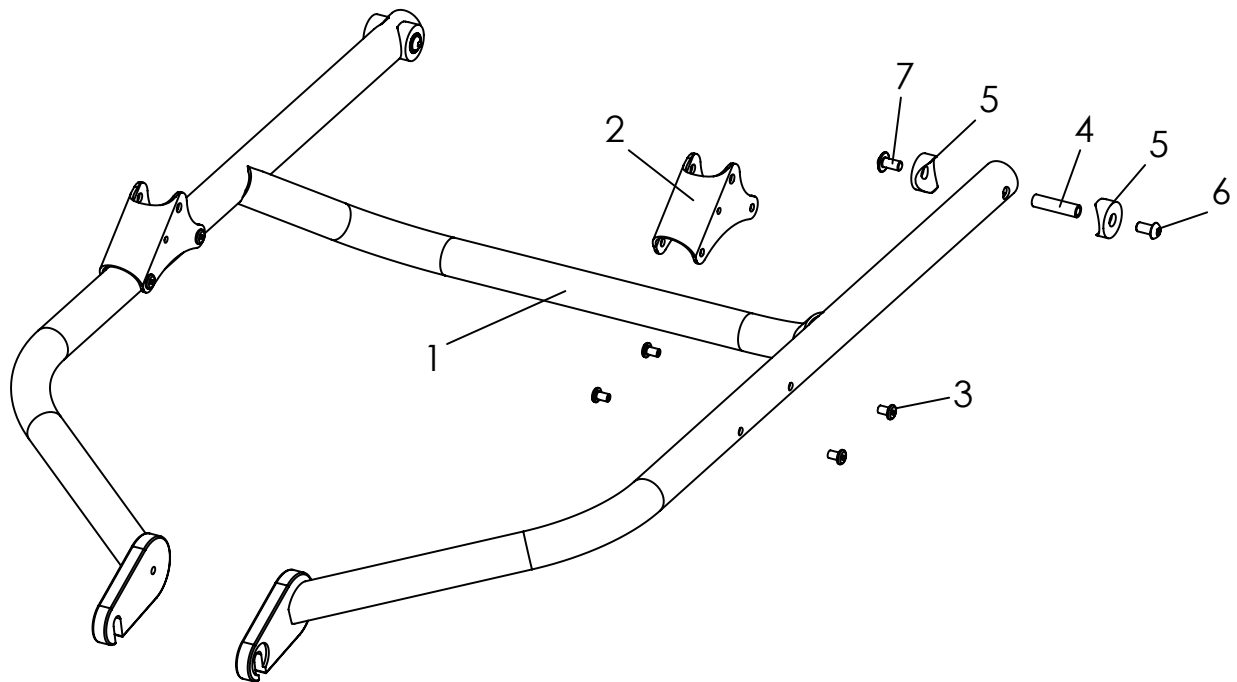


Pos	Article number	Articel description	Quantity	Unit
1-7	3236-1100E-xxx	Trike-Frame size 1	1	Pc.
	3237-1100E-xxx	Trike-Frame size 2	1	Pc.
1-3	3236-1110E-xxx	Trike-Frame size 1	1	Pc.
	3237-1110E-xxx	Trike-Frame size 2	1	Pc.
2	3231-1120	Front strut bearing	2	Pc.
3	1069-0004	Blind rivet F5x10 black	8	Pc.
4	1114-0027	Threaded bushing 8xM6x32	2	Pc.
5	1111-0012	Saddle washer PA black 24x8.4x3	4	Pc.
6	1029-6001-033	Buttonheadsocketscrew black, Tufflock M6x12	2	Pc.
7	1030-0612	Button head socket screw with flange, M6x12	2	Pc.

Spare parts

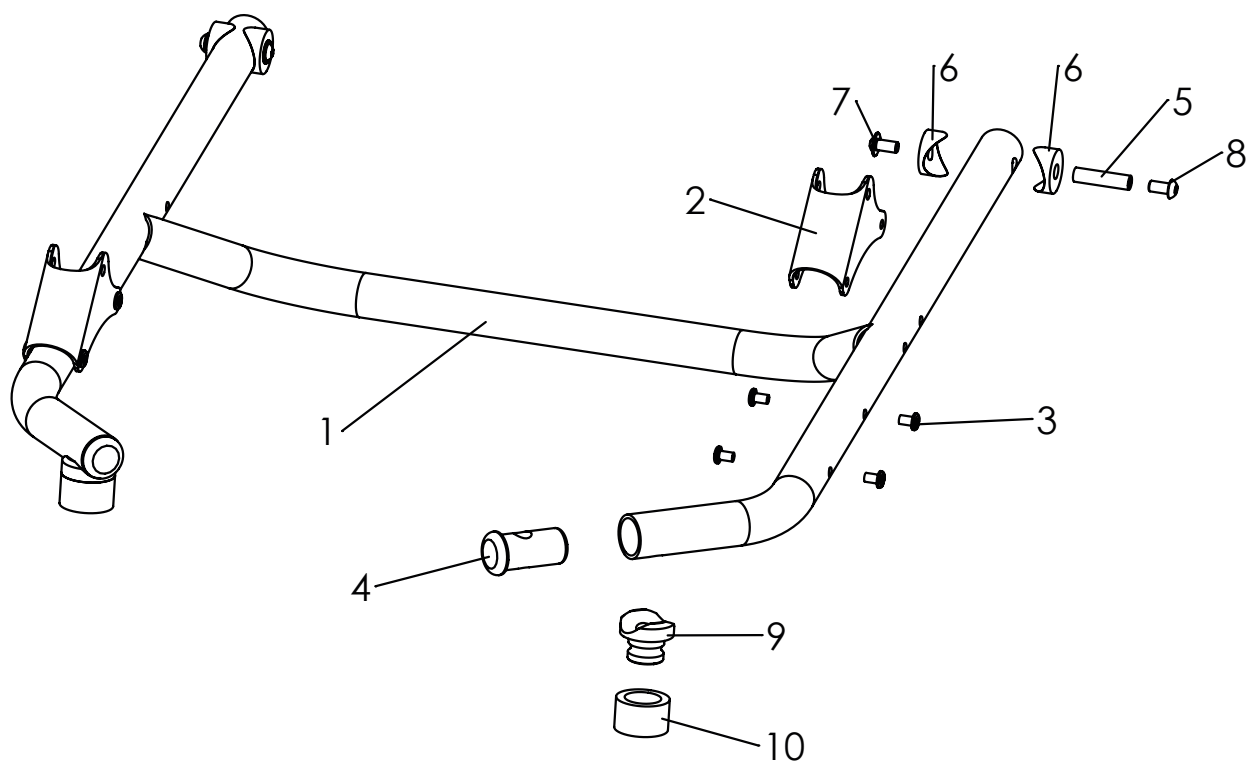


Pos	Article number	Articel description	Quantity	Unit
1-10	3230-1100E-032	Frame Mini (swivel front wheels)	1	Pc.
1-4	3230-1110E-032	Frame Mini (swivel front wheels)	1	Pc.
2	3231-1120-032	Front strut bearing	2	Pc.
3	1069-0004	Blind rivet F5x10 black	8	Pc.
4	3202-1130	Block	2	Pc.
5	1114-0027	Threaded bushing 8xM6x32	2	Pc.
6	1111-0012	Saddle washer PA black 24x8.4x3	4	Pc.
7	1030-0612	Button head socket screw with flange M6x12	2	Pc.
8	1029-6001-033	Buttonheadsocketscrew black, Tufflock M6x12	2	Pc.
9	3202-1140	Saddle bushing for front caster	2	Pc.
10	3202-1151	Cover ring	2	Pc.

Spare parts

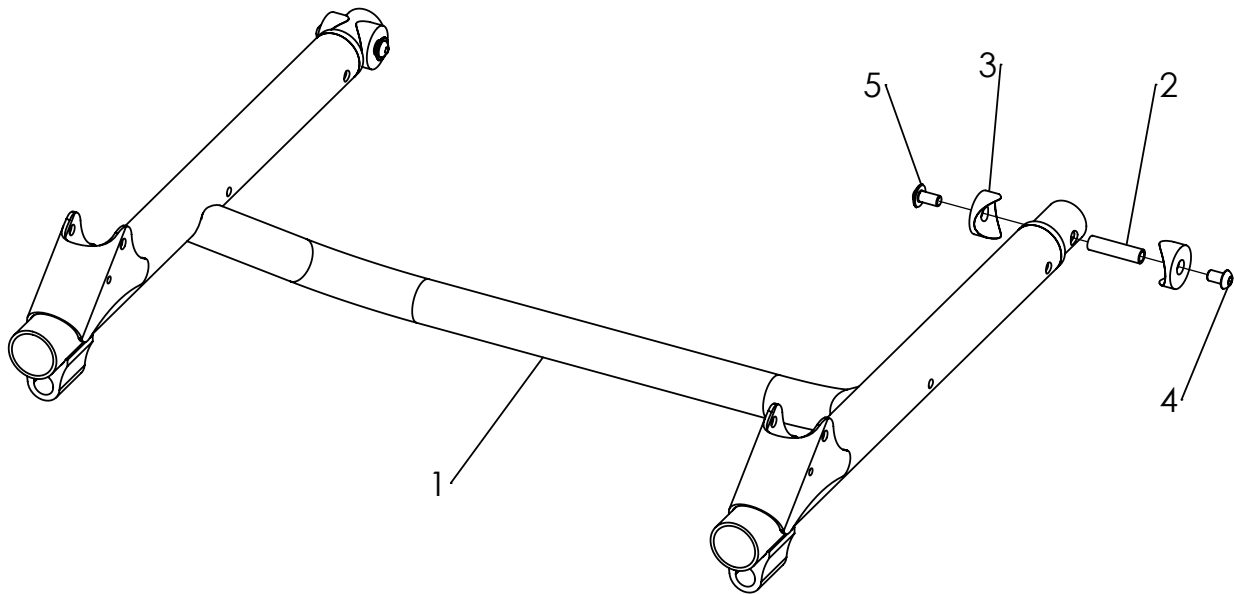


Pos	Article number	Articel description	Quantity	Unit
1-5	3032-1100E-xxx	Holding Frame Switch size 1	1	Pc.
	3034-1100E-xxx	Holding Frame Switch size 2	1	Pc.
1	3032-1110E-xxx	Holding Frame Switch size 1	1	Pc.
	3034-1110E-xxx	Holding Frame Switch size 2	1	Pc.
2	1114-0027	Threaded bushing 8xM6x32	2	Pc.
3	1111-0012	Saddle washer PA black 25x8.4x3	4	Pc.
4	1029-6001-033	Button head socket screw black, Tufflock M6x12	2	Pc.
5	1030-0612	Button head socket screw with flange M6x12	2	Pc.

Spare parts

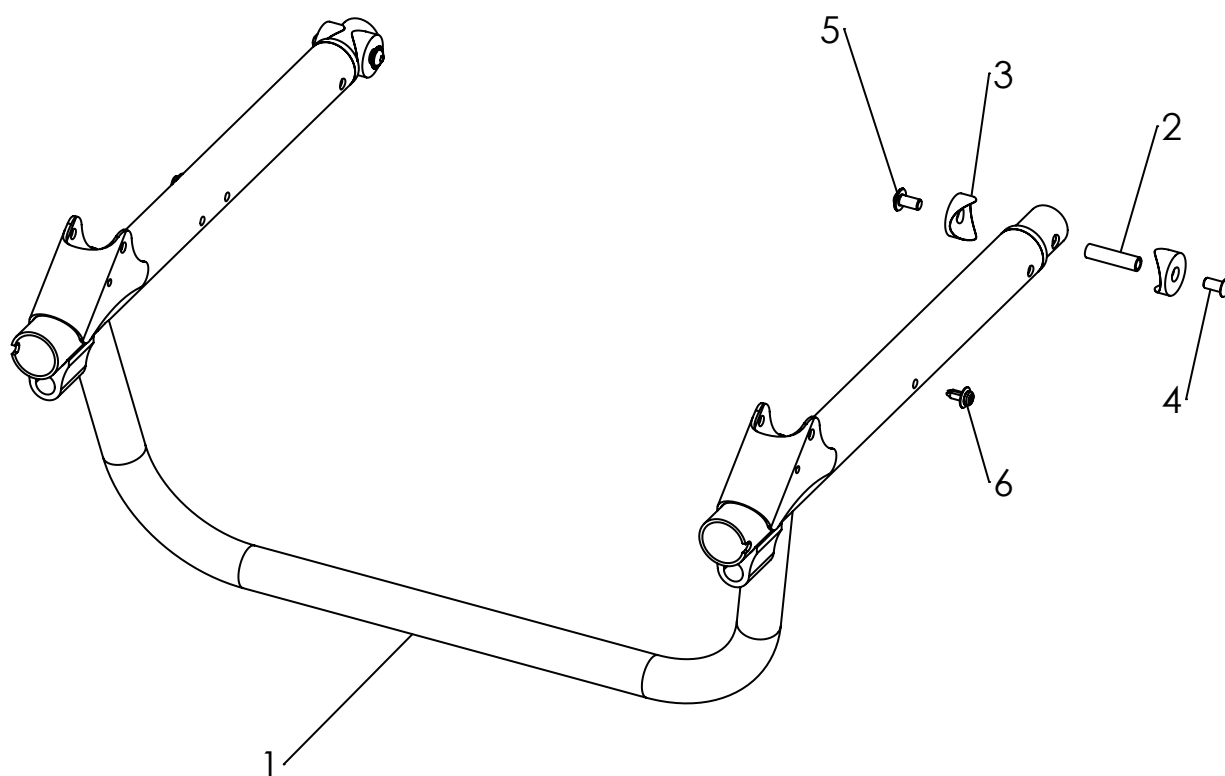


Pos	Article number	Articel description	Quantity	Unit
1-6	3033-1100E-xxx	Holding Frame for additional equipment Switch size 1	1	Pc.
	3035-1100E-xxx	Holding Frame for additional equipment Switch size 2	1	Pc.
1	3033-1110E-xxx	Holding Frame for additional equipment Switch size 1	1	Pc.
	3035-1110E-xxx	Holding Frame for additional equipment Switch size 2	1	Pc.
2	1114-0027	Threaded bushing 8xM6x32	2	Pc.
3	1111-0012	Saddle washer PA black 25x8.4x3	4	Pc.
4	1029-6001-033	Button head socket screw black, Tufflock M6x12	2	Pc.
5	1030-0612	Button head socket screw with flange M6x12	2	Pc.
6	1149-0008	Screw rivet PA	2	Pc.

Spare parts

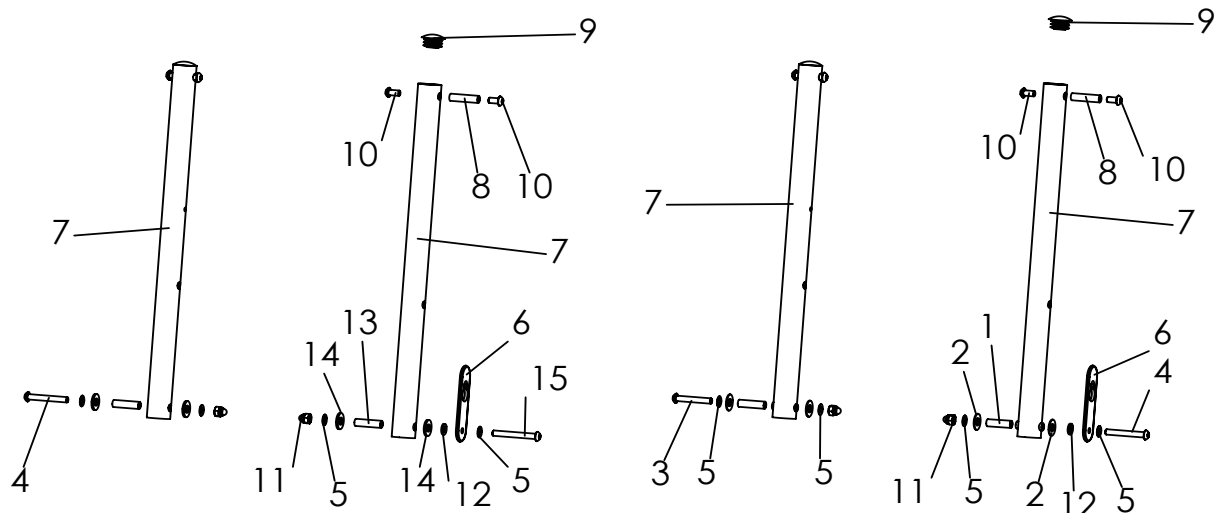


Pos	Article number	Articel description	Quantity	Unit
1-12	3231-1300E-xxx	Front bar, complete	1	Pc.
4-15	3031-1300E-xxx	Front bar Switch, complete	1	Pc.
1	1114-0025	Bushing 8x0,8x27	2	Pc.
2	1113-0003	Washer PA 18x8,4x1 black	4	Pc.
3	1029-0640-033	Button head socket screwv black M6x40	1	Pc.
4	1029-0645-033	Button head socket screw black M6x45	1	Pc.
5	1052-0006	Spring washer B6	4	Pc.
6	3231-1431	Catch strap	1	Pc.
	3235-1431	Catch strap (EVO 2XL)	1	Pc.
7	3231-1310-xxx	Front bar	2	Pc.
8	1114-0027	Threaded bushing 8xM6x32	2	Pc.
9	1141-0009	Ribbed insert SFL 25x1-2,5	2	Pc.
10	1029-0612-033	Button head socket screw M6x12	4	Pc.
11	1044-0006-033	Hexagonal domed cap nut self-locking black M6	2	Pc.
12	1051-0006-033	Washer A6,4	1	Pc.
13	1114-0037	Bushing steel 8x0,9x29,5	2	Pc.
14	1113-0035	Washer PA 18x8,4x2 black	4	Pc.
15	1029-0650	Button head socket screw M6x50	1	Pc.

Spare parts

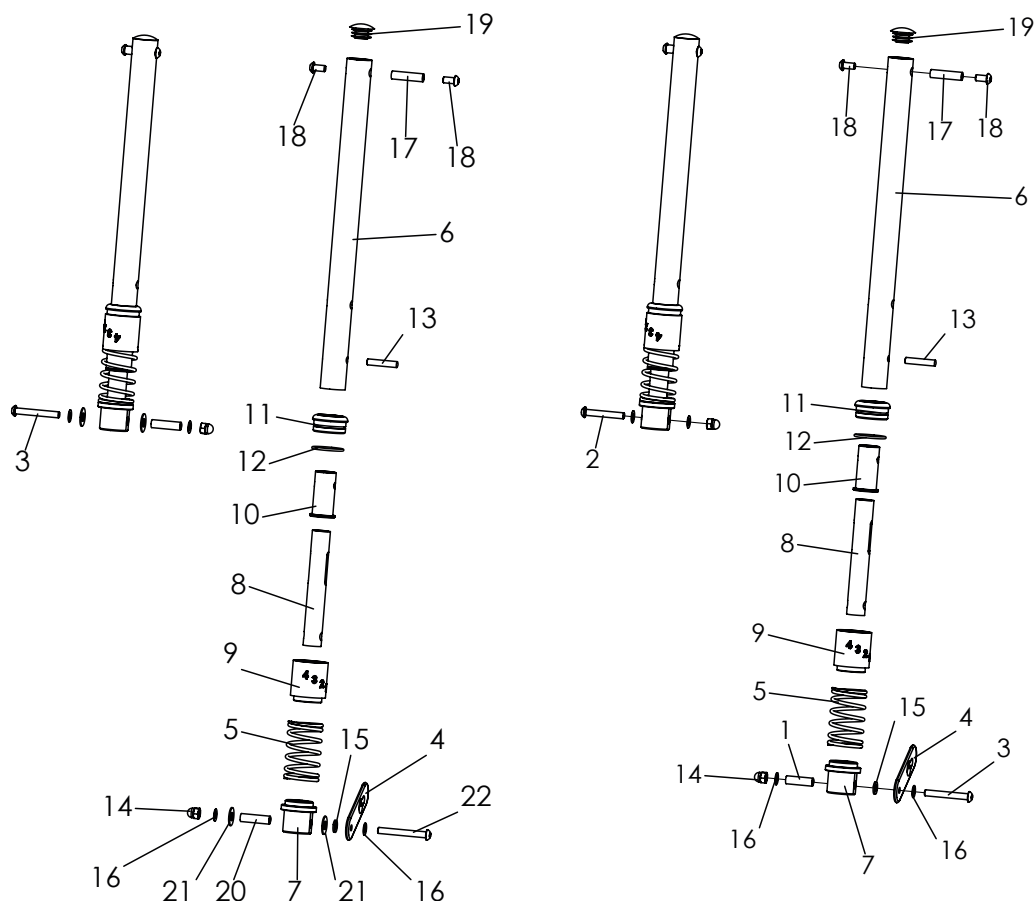

Pos	Article number	Articel description	Quantity	Unit
1-19	3231-7400E-xxx	Frame suspension system, fourfold adjustable, complete	1	Set
3-22	3031-7400E-xxx	Frame suspension system Switch, complete	1	Set
1	1114-0025	Bushing zp 8x0,8x27	2	Pc.
2	1029-0640-033	Button head socket screwv black M6x40	1	Pc.
3	1029-0645-033	Button head socket screw black M6x45	1	Pc.
4	3231-1431	Catch strap	1	Pc.
	3235-1431	Catch strap 2XL	1	Pc.
5	1131-901X	Compression spring set	1	Set
6	3231-7431-xxx	Front brace suspension	2	Pc.
7	3231-7432	Spring hanger (at the bottom)	2	Pc.
8	3231-7433-007	Guide tube suspension	2	Pc.
9	3231-7434	Spring hanger (at the top)	2	Pc.
10	3231-7435	Guiding bushing, suspension	2	Pc.
11	3231-7436	End bushing	2	Pc.
12	1150-0012	O-Ring NBR 70 28x2	2	Pc.
13	1071-0630	Parallel pin m6x30	2	Pc.
14	1044-0006-033	Hexagonal domed cap nut self-locking black M6	2	Pc.
15	1051-0006-033	Washer A6,4	1	Pc.
16	1052-0006	Spring washer B6	4	Pc.
17	1114-0027	Threaded bushing 8xM6x32	2	Pc.
18	1029-0612-033	Button head socket screw M6x12	4	Pc.
19	1141-0009	Ribbed insert SFL 25x1-2,5	2	Pc.
20	1114-0037	Bushing steel 8x0,9x29,5	2	Pc.
21	1113-0003	Washer PA 18x8,4x1 black	4	Pc.
22	1029-0650	Button head socket screw M6x50	1	Pc.

Spare parts

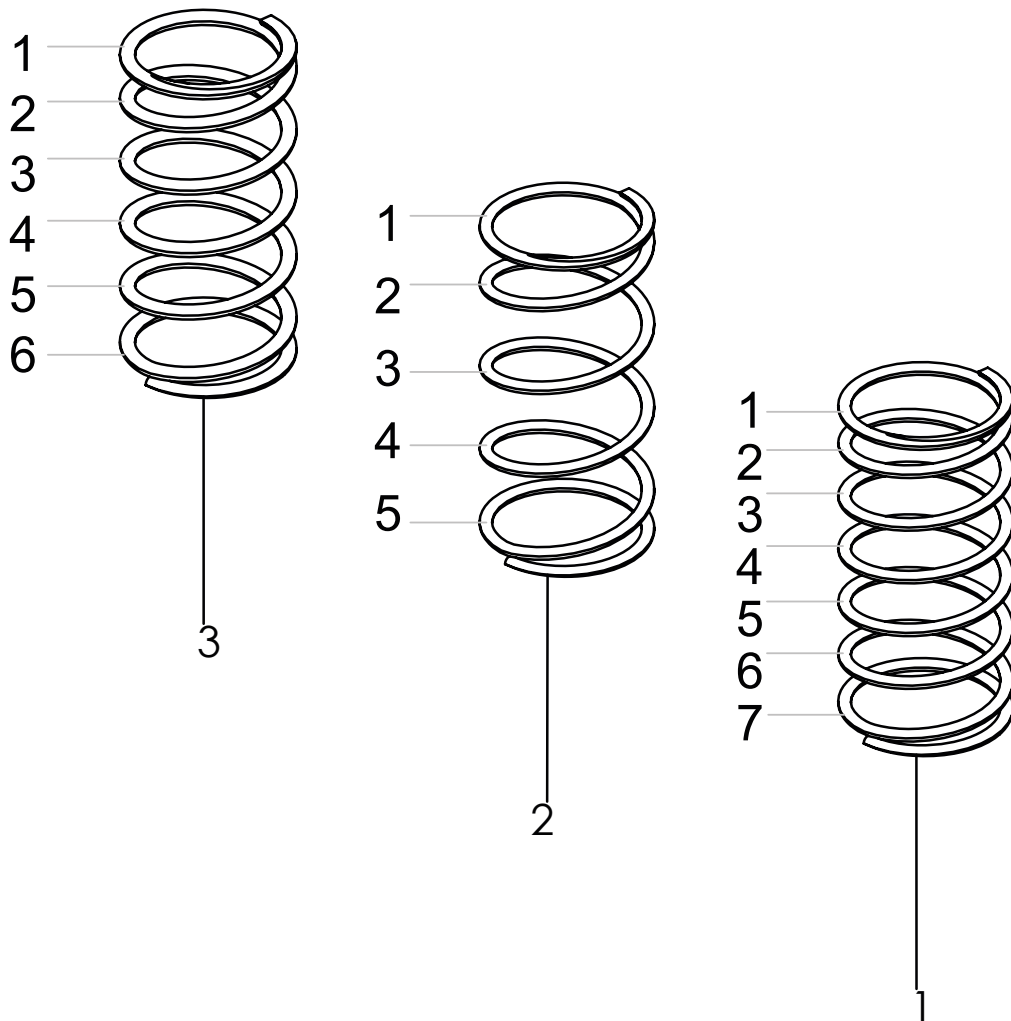

Pos	Article number	Articel description	Quantit	Unit
1	1131-9017E-022	Spring set for load up to 15kg, standard color, signal white	1	Set
	1131-9017E-xxx	Spring set for load up to 15kg, colour code for upholstery	1	Set
2	1131-9018E-022	Spring set for load up to 25kg, standard color, signal white	1	Set
	1131-9018E-xxx	Spring set for load up to 25kg, colour code for upholstery	1	Set
3	1131-9019E-022	Spring set for load from 25kg, standard color, signal white	1	Set
	1131-9019E-xxx	Spring set for load from 25kg, colour code for upholstery	1	Set


Upholstery color *With order replace xxx with colour code!*

352+349	351+348	350+347	346	345	344	340	339	338
Pastel turquoise	Siver	Light pink	Black dull matt	Red	Pearl copper	Violet	Turquoise blue	cream white
RAL6034	Silber	RAL3015	RAL 9005	RAL3002	RAL8029	RAL4003	RAL5018	RAL9001

Frame color


002	032	060
Silver	Black dull matt	Copper gold

Spare parts

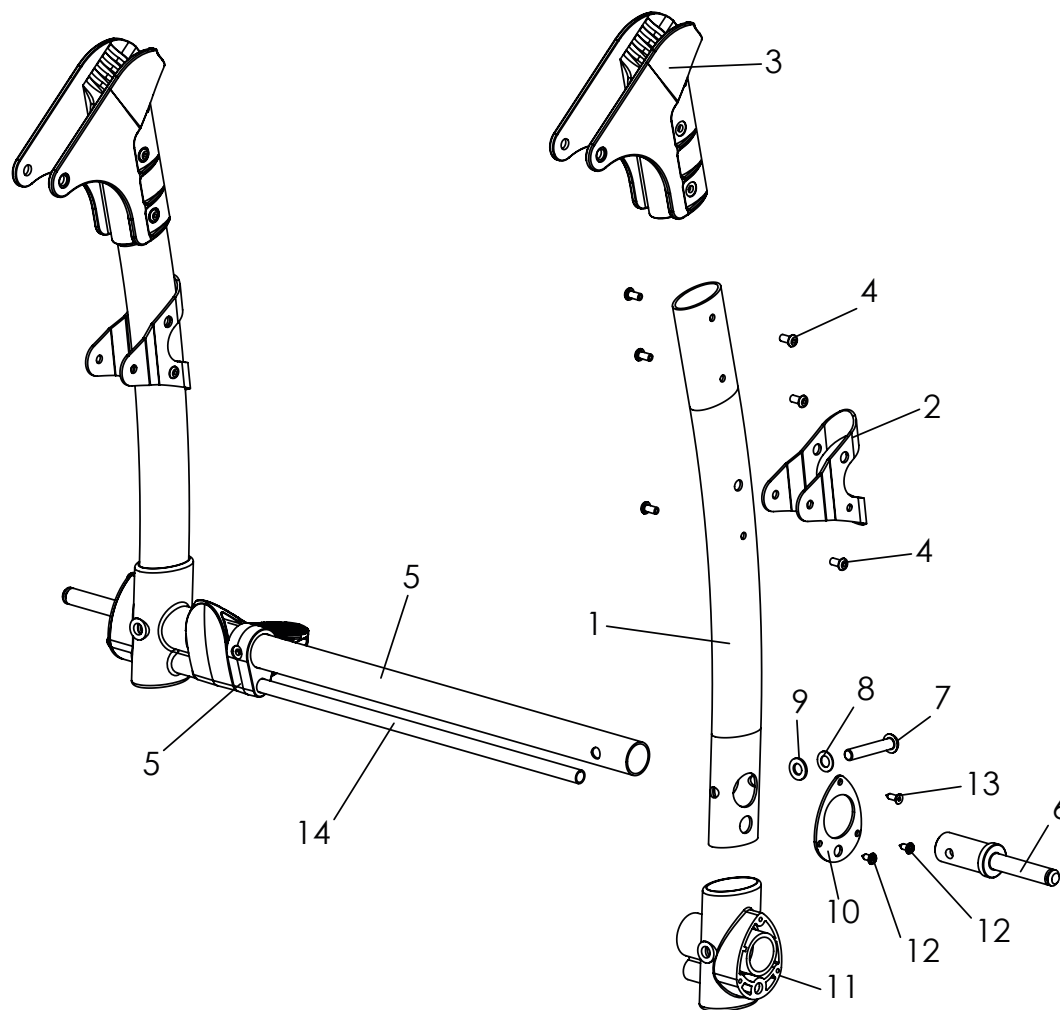


Pos	Article number	Articel description	Quantity	Unit
1-13	3231-1200E-xxx	Rear frame, size 1	1	Pc.
	3233-1200E-xxx	Rear frame, size 2	1	Pc.
1-4	3231-1211E-xxx	Side bar rear, incl. joint bandage	2	Pc.
1	3231-1211-xxx	Side bar rear	2	Pc.
2	3231-1212-xxx	Joint bandage	2	Pc.
3	3231-1213	Rear frame hinge	2	Pc.
4	1069-0004	Blind rivet F4,8x10 black	12	Pc.
5	3201-1220E-xxx	Axle tube size 1, with mounted brake rod retainer	1	Pc.
	3203-1220E-xxx	Axle tube size 2, with mounted brake rod retainer	1	Pc.
6*	3201-1240	Axle stump	2	Pc.
7	1029-0845	Button head socket screw M8x45	2	Pc.
8	1052-0008	Spring washer B8	2	Pc.
9	1051-0008	Washer A8,4	2	Pc.
10-12	3201-1280E	Connector with fasteners	2	Pc.
10	3201-1281	Stabilizer	2	Pc.
11	3201-1280	Connector	2	Pc.
12	1039-0001	Screw for plastic, cross rcessed pan head w. flange 4x10	2	Pc.
13	1039-0006	Tapping screw with countersunk head	1	Pc.
	320X-1301-xxx	Break, see next page!	1	Pc.
*6-9	3201-1240E	Axle stump as spare part	1	Pc.

Spare parts

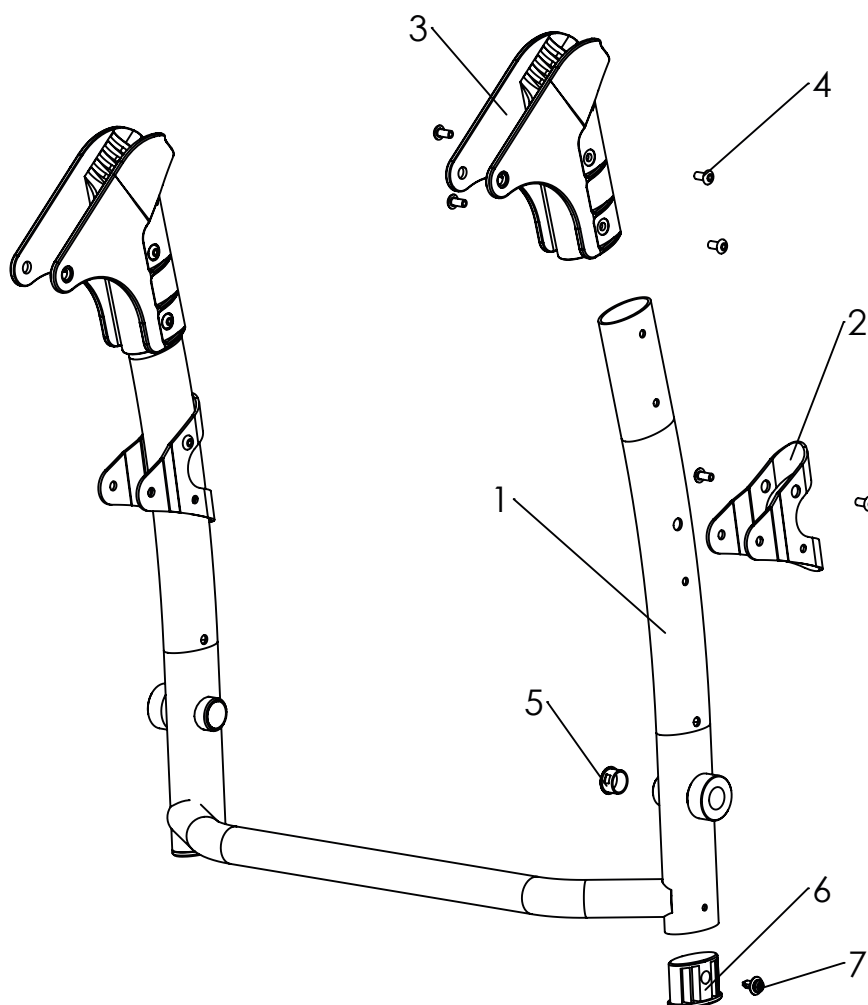


Pos	Article number	Articel description	Quantity	Unit
1-7	3232-1200E-xxx	Rear frame CROSS, size 1	1	Pc.
	3234-1200E-xxx	Rear frame CROSS, size 2	1	Pc.
1	3232-1210-xxx	Rear frame Bingo CROSS, size 1	1	Pc.
	3234-1210-xxx	Rear frame Bingo CROSS, size 2	1	Pc.
2	3231-1212-032	Joint bandage, black dull matt	2	Pc.
3	3231-1213	Rear frame hinge	2	Pc.
4	1069-0004	Blind rivet F4,8x10 black	12	Pc.
5	1141-0039	Cover plug	2	Pc.
6	3201-1270	Plug	2	Pc.
7	1149-0008	Screw rivet	2	Pc.

Spare parts

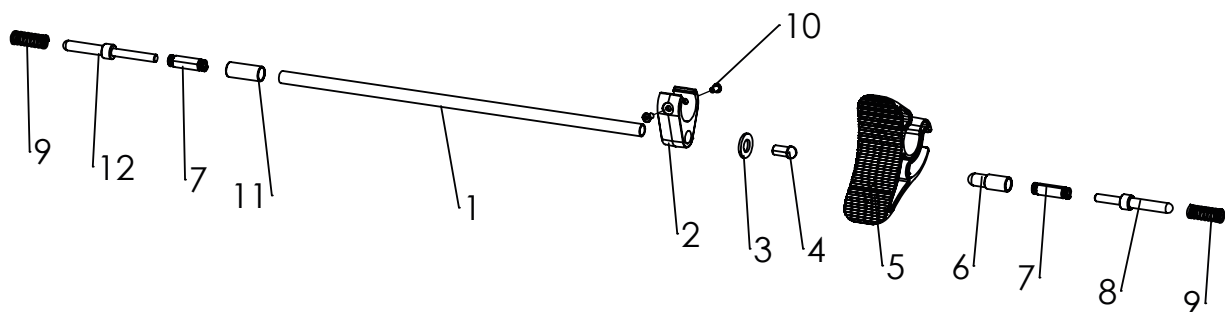

Pos	Article number	Articel description	Quantity	Unit
1-12	3232-1300E-xxx	Brake size 1, complete	1	Pc.
	3234-1300E-xxx	Brake size 2, complete	1	Pc.
1	3201-1330-xxx	Wheel lock rod size 1	1	Pc.
	3203-1330-xxx	Wheel lock rod size 2	1	Pc.
2	3232-1320	Wheel lock rod retainer open	1	Pc.
3	1113-0036	Washer PA black 20x10,5x2	1	Pc.
4	3201-1340	Slide insert	1	Pc.
5	3232-1310	Wheel lock lever open	1	Pc.
6	3232-1361	Slider brake right	1	Pc.
7	1131-0087	Compression D-198	2	Pc.
8	3232-1360-008	Lock pin right	1	Pc.
9	1131-0034	Compression spring D-180Q-03	2	Pc.
10	1062-0408	Blind rivet F4x8	2	Pc.
11	3232-1351-007	Brake sleeve black	1	Pc.
12	3232-1350-008	Lock pin, left	1	Pc.


Spare parts

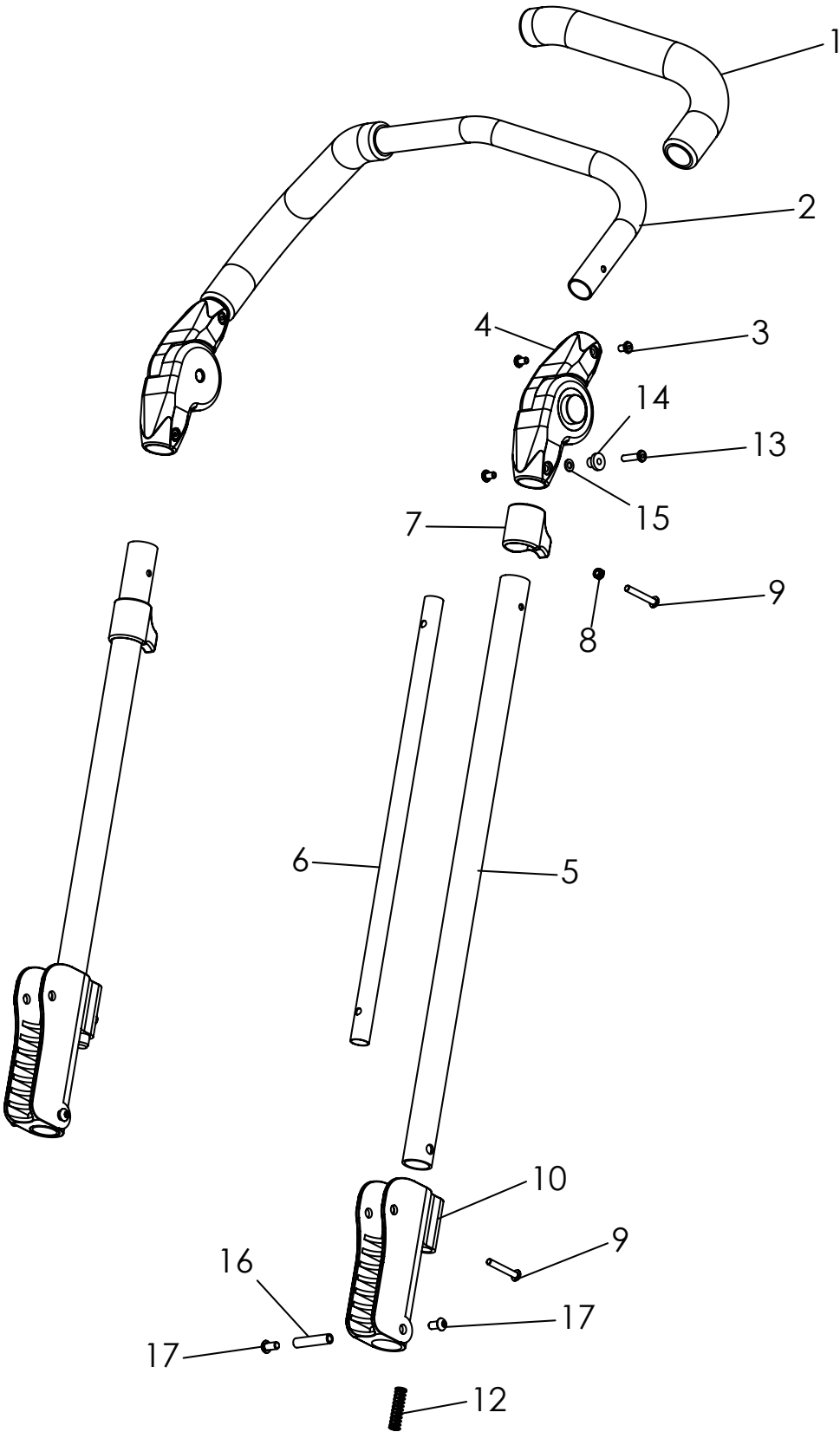

Spare parts


Pos	Article number	Articel description	Quantity	Unit
1-17	3231-1400E-xxx	Push bar size 1	1	Pc.
	3233-1400E-xxx	Push bar size 2	1	Pc.
	3235-1400E-xxx	Push bar size 2XL	1	Pc.
1	1143-0001	Handle grip, black	2	Pc.
2+3	3231-1421E-xxx	Upper part push handle size 1, with fastemers	1	Pc.
	3233-1421E-xxx	Upper part push handle size 2, with fastemers	1	Pc.
	3233-1421E-xxx	Upper part push handle size 2XL, with fastemers	1	Pc.
2	3231-1421-xxx	Push bar size 1	1	Pc.
	3233-1421-xxx	Push bar size 2	1	Pc.
	3233-1421-xxx	Push bar size 2 XL	1	Pc.
3	1069-0003	Blind rivet F5x8 black	7	Pc.
4	1129-0009	Ratchet joint 25x22	1	Pair
5	3231-1412-xxx	Push bar bottom part size 1 and 2	2	Pc.
	3235-1412-xxx	Push bar bottom part size 2 XL	2	Pc.
6	3231-1415	Guide rail size 1 and 2	2	Pc.
	3235-1415	Guide rail size 3	2	Pc.
7	3231-1414	Activator	2	Pc.
8	1042-0005	Hexagon nut self-locking zp DIN/ISO 985 M5	2	Pc.
9	1034-0535	Torxscrew M5x35	4	Pc.
10	3231-1411	Push handle joint	2	Pc.
11	3231-1413	Folding-security bolt	2	Pc.
12	1131-0043	Compression spring D-143F	2	Pc.
13	1062-0520	Blind rivet F5x20	1	Pc.
14	3231-1432	Shoulder bushing black	1	Pc.
15	1051-0005	Washer A5,3	1	Pc.
16	1114-0028	Threaded bushing 8xM6x38	2	Pc.
17	1029-0612-033	Buttonheadsocketscrew black, Tufflock M6x12	4	Pc.

Spare parts

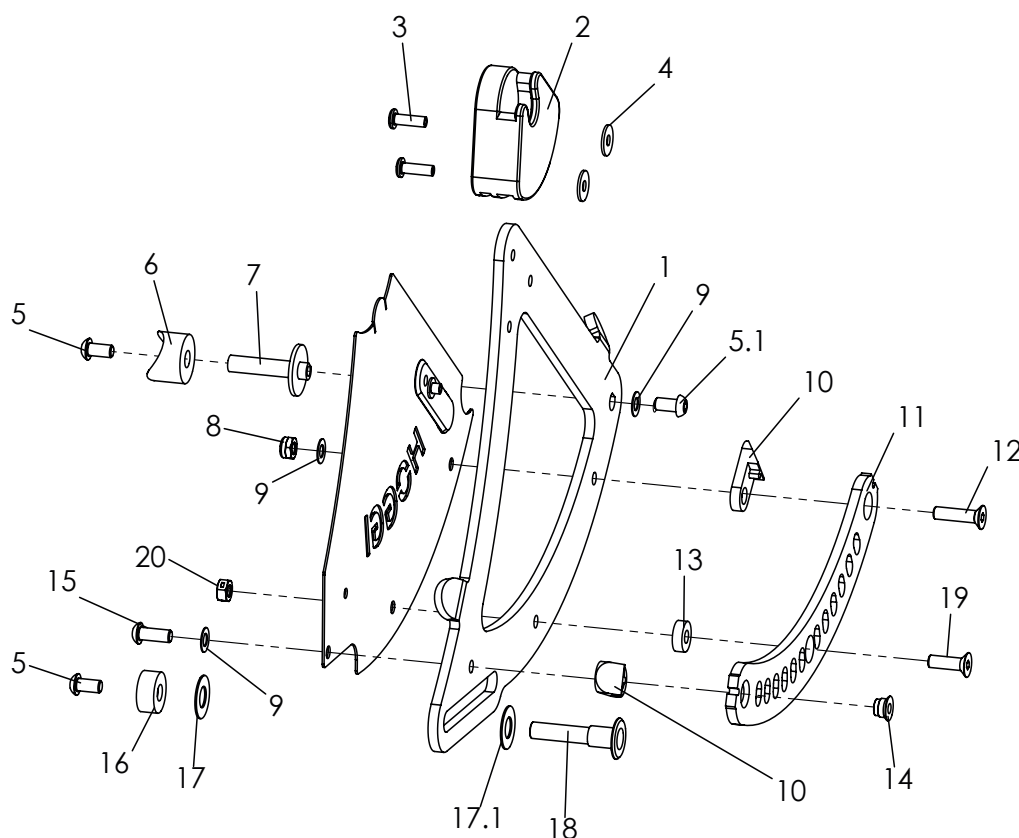


Pos	Article number	Articel description	Quantity	Unit
1-20	3231-1510E-032	Seat Bearing left size 1 and 2, complete, displayed	1	Pc.
	3231-1520E-032	Seat Bearing right size 1 and 2, complete	1	Pc.
1-4	3231-1511E-032	Seat Bearing left, with fasteners	1	Pc.
	3231-1512E-032	Seat Bearing right, with fasteners	1	Pc.
1	3231-1511-032	Seat Bearing left	1	Pc.
	3231-1512-032	Seat Bearing right	1	Pc.
2	3231-1516	Scanteling bearing	1	Pc.
3	1062-0516	Blind rivet F5x16	2	Pc.
4	1055-0005	Washer A5,3	2	Pc.
5	1029-0612-033	Button head socket screw black M6x12	2	Pc.
5.1	1029-6001-033	Button head socket screw black with lock, M6x12	1	Pc.
6	1111-0013	Saddle washer PA black 24x8.4x10	1	Pc.
7	3231-1515	Bearing shaft	1	Pc.
8	1042-0006-033	Hexagon nut self-locking, black M6	1	Pc.
9	1052-0006-033	Spring washer black B6	3	Pc.
10	3231-1514	Latch ramp	2	Pc.
11	3231-1525-006	Scantling rail left	1	Pc.
	3231-1526-006	Scantling rail right	1	Pc.
12	1024-0625-033	Hexagon socket countersunk head screw black M6x25	1	Pc.
13	1113-0074	Washer PA 6,4x15x5	1	Pc.
14	3231-1527	Threded insert	1	Pc.
15	1029-0616-033	Button head socket screw black M6x16	1	Pc.
16	3231-1518	Distance bearing	1	Pc.
17	1113-0056	Washer PA natural 10,5x21,0x0,8	1	Pc.
17.1	1113-0076	Washer PA natural d10,0xD18,0x1,0	1	Pc.
18	3231-1517	Gudie shaft seat bearing	1	Pc.
19	1024-0620-033	Hexagon socket countersunk head screw black M6x20	1	Pc.
20	1049-0070-033	BILOC-nut VM6	1	Pc.

Spare parts

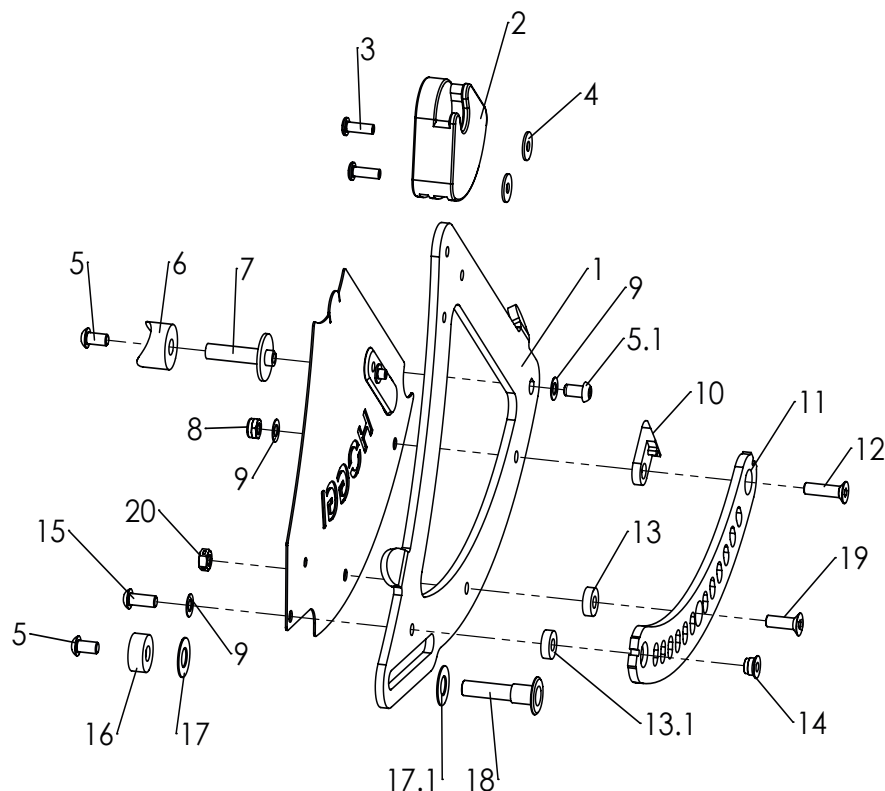


Pos	Article number	Articel description	Quantity	Unit
1-20	3235-1510E-032	Seat bearing, left complete, shown only for size 2XL	1	Pc.
	3235-1520E-032	Seat bearing, right complete, shown only for size 2XL	1	Pc.
1-4	3231-1511E-032	Seat Bearing left, with fasteners	1	Pc.
	3231-1512E-032	Seat Bearing right, with fasteners	1	Pc.
1	3231-1511-032	Seat Bearing left	1	Pc.
	3231-1512-032	Seat Bearing right	1	Pc.
2	3231-1516	Scanteling bearing	1	Pc.
3	1062-0516	Blind rivet F5x16	2	Pc.
4	1055-0005	Washer A5,3	2	Pc.
5	1029-0612-033	Button head socket screw black M6x12	2	Pc.
5.1	1029-6001-033	Button head socket screw black with lock, M6x12	1	Pc.
6	1111-0013	Saddle washer PA black 24x8.4x10	1	Pc.
7	3231-1515	Bearing shaft	1	Pc.
8	1042-0006-033	Hexagon nut self-locking, black M6	1	Pc.
9	1052-0006-033	Spring washer black B6	3	Pc.
10	3231-1514	Latch ramp	1	Pc.
11	3231-1525-006	Scantling rail left	1	Pc.
	3231-1526-006	Scantling rail right	1	Pc.
12	1024-0625-033	Hexagon socket countersunk head screw black M6x25	1	Pc.
13	1113-0074	Washer PA 6,4x15x5	1	Pc.
13.1	1113-0031	Washer PA 8,4x15x5	1	Pc.
14	3231-1527	Threded insert	1	Pc.
15	1029-0616-033	Button head socket screw black M6x16	1	Pc.
16	3231-1518	Distance bearing	1	Pc.
17	1113-0056	Washer PA natural 10,5x21,0x0,8	1	Pc.
17.1	1113-0076	Washer PA natural d10,0xD18,0x1,0	1	Pc.
18	3231-1517	Gudie shaft seat bearing	1	Pc.
19	1024-0620-033	Hexagon socket countersunk head screw black M6x20	1	Pc.
20	1049-0070-033	BILOC-nut zp DIN 934-8 VM6	1	Pc.

Spare parts

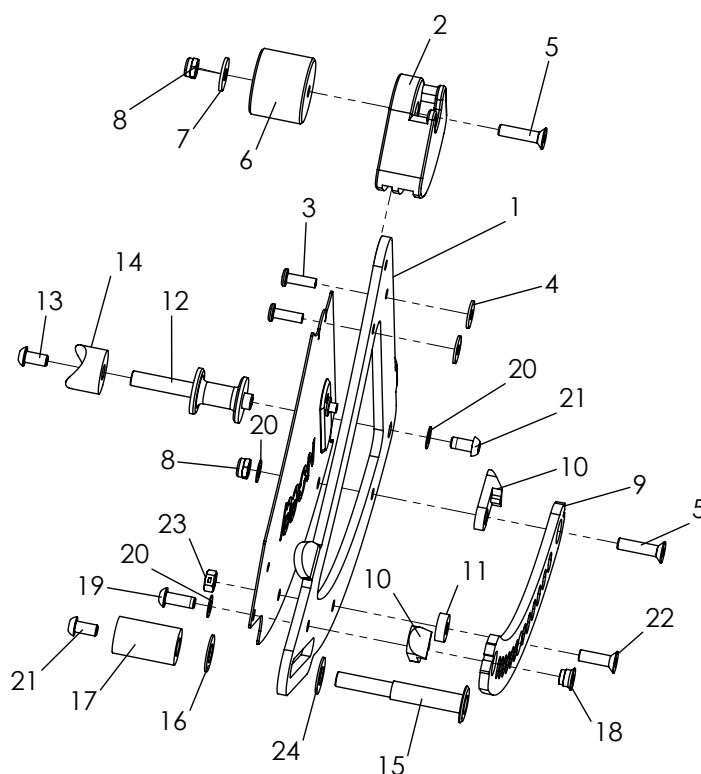

Pos	Article number	Articel description	Quantity	Unit
1-24	3230-1510E-032	Seat Bearing left, complete, displayed only for Mini	1	Pc.
	3230-1520E-032	Seat Bearing right, complete, only for Mini	1	Pc.
1-4	3230-1511E-032	Seat Bearing Mini left, with fasteners	1	Pc.
	3230-1512E-032	Seat Bearing Mini right, with fasteners	1	Pc.
1	3231-1511-032	Seat bearing, left	1	Pc.
	3231-1512-032	Seat bearing, right	1	Pc.
2	3230-1516	Scantlings bearing Mini	1	Pc.
3	1062-0516	Blind rivet F5x16	2	Pc.
4	1055-0005	Washer A5,3	2	Pc.
5	1024-0625-033	Hexagon socket countersunk head screw black M6x25	2	Pc.
6	3230-1519	Distance piece for Mini	1	Pc.
7	1055-0006	Washer A6,4	1	Pc.
8	1042-0006-033	Hexagon nut self-locking black M6	2	Pc.
9	3231-1525-006	Scantling rail left	1	Pc.
	3231-1526-006	Scantling rail right	1	Pc.
10	3231-1514	Latch ramp	2	Pc.
11	1113-0074	Washer PA 6,4x12,5x5	1	Pc.
12	3230-1515-033	Bearing shaft mini black	1	Pc.
13	1029-6001-033	Button head socket screw,Tufflock M6x12	2	Pc.
14	1111-0013	Saddle washer PA black 24x8.4x10	1	Pc.
15	3230-1517-008	Guided shaft seat bearing for Mini	1	Pc.
16	1113-0056	Washer PA natural 10,5x21,0x0,8 mm	1	Pc.
17	3230-1518-007	Distance piece, Guided shaft for Mini	1	Pc.
18	3231-1527	Threaded insert	1	Pc.
19	1029-0616-033	Button head socket screw black M6x16	1	Pc.
20	1052-0006-033	Spring washer black B 6	3	Pc.
21	1029-0612	Button head socket screw zp DIN/ISO 7380 M6x12	1	Pc.
22	1024-0620-033	Hexagon socket countersunk head screw black M6x20	1	Pc.
23	1049-0070-033	BILOC-nut VM6	1	Pc.
24	1113-0076	Washer PA natural d10,0xD18,0x1,0	1	Pc.

Spare parts

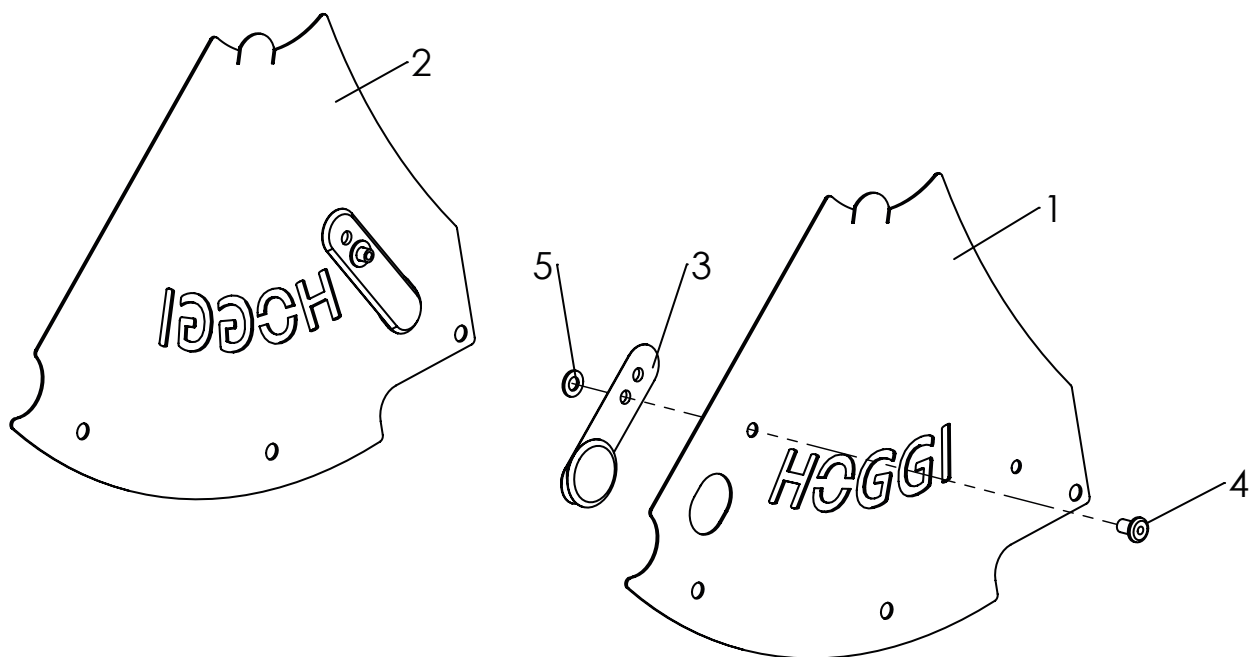

Pos	Article number	Articel description	Quantity	Unit
1-5	3231-9500-xxx	Side covers	1	Pc.
1	3231-9511-xxx	Side covers Hoggi left	1	Pc.
2	3231-9521-xxx	Side covers Hoggi right	1	Pc.
3	3231-9523	Button	2	Pc.
4	1051-0005	Washer A5,3	2	Pc.
5	1069-0003	Blind rivet F5x8 black	2	Pc.


Upholstery color *With order replace xxx with colour code!*

352+349	351+348	350+347	346	345	344	340	339	338
Pastel turquoise	Siver	Light pink	Black dull matt	Red	Pearl copper	Violet	Turquoise blue	cream white
RAL6034	Silber	RAL3015	RAL 9005	RAL3002	RAL8029	RAL4003	RAL5018	RAL9001

Frame color


002	032	060
Silver	Black dull matt	Copper gold

Spare parts

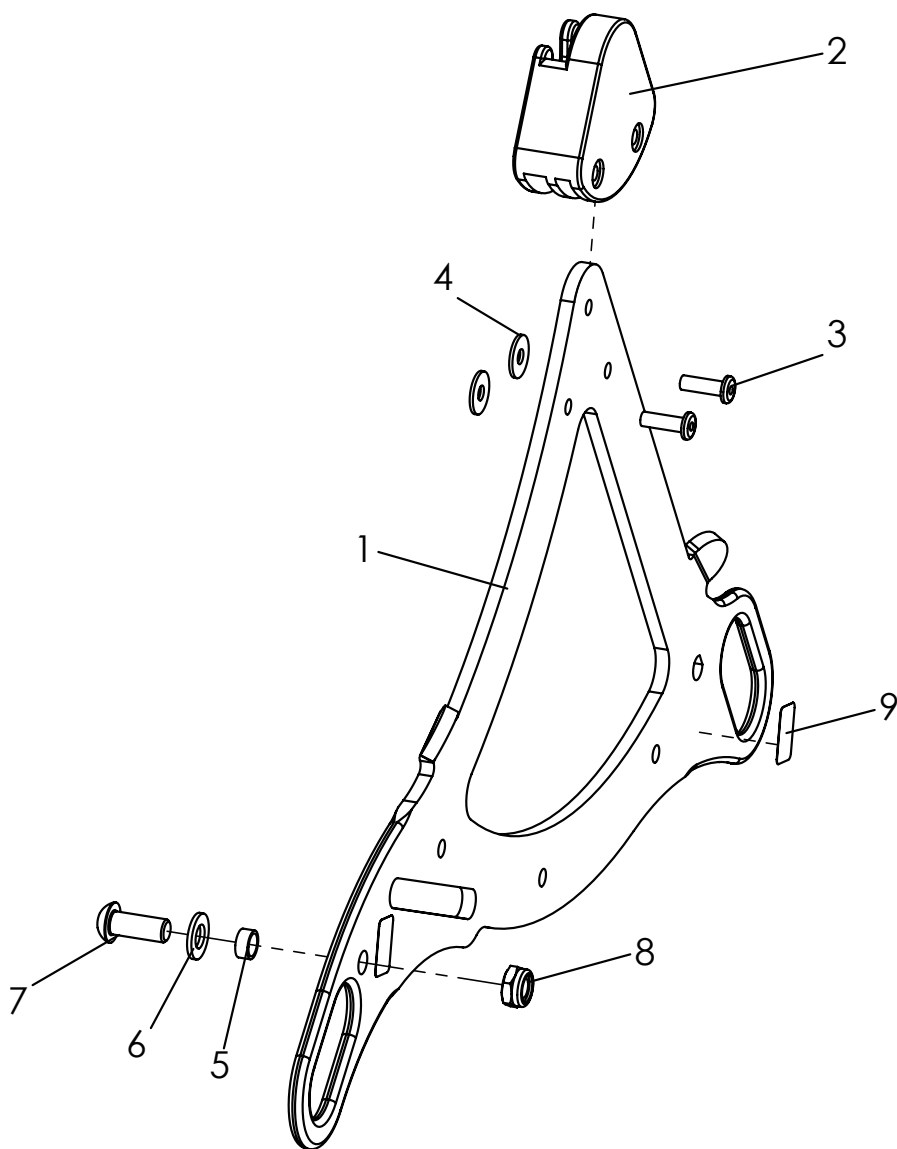


Pos	Article number	Articel description	Quantity	Unit
1-9	3231-7300E-xxx	Tie down kit	1	Pc.
1-4	3231-7312E-xxx	Seat bearing plate with lashing eyes, right	1	Pc.
	3231-7311E-xxx	Seat bearing plate with lashing eyes, left	1	Pc.
2	3231-1516	Scantlings bearing Mini	2	Pc.
3	1062-0516	Blind rivet F5x16	4	Pc.
4	1055-0005	Washer A5,3	4	Pc.
5	3201-7350	Slide bushing	2	Pc.
6	1051-0008	Washer A8,4	2	Pc.
7	1029-0820	Button head socket screw M8x20	2	Pc.
8	1042-0008	Hexagon nut self-locking M8	2	Pc.
9	1914-0007	Adhesive lable with hook symbol (4 pcs.)	1	Pc.

Spare parts

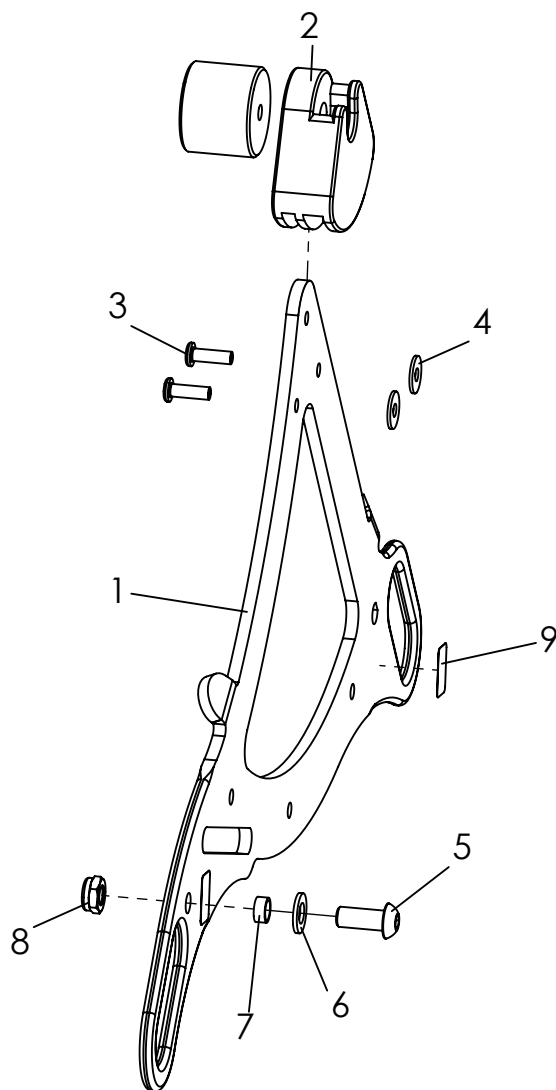


Pos	Article number	Articel description	Quantity	Unit
1-9	3230-7300E-032	Tie down kit Mini	1	Pc.
1-4	3230-7311E-032	Seat bearing plate with lashing eyes, Scantling bearing Mini, left	1	Pc.
	3230-7312E-032	Seat bearing plate with lashing eyes, Scantling bearing Mini, right	1	Pc.
2	3230-1516	Scantlings bearing Mini	2	Pc.
3	1062-0516	Blind rivet F5x16	4	Pc.
4	1055-0005	Washer A5,3	4	Pc.
5	1029-0820	Button head socket screw M8x20	2	Pc.
6	1051-0008	Washer A8,4	2	Pc.
7	3201-7350	Slide bushing	2	Pc.
8	1042-0008	Hexagon nut self-locking M8	2	Pc.
9	1914-0007	Adhesive lable with hook symbol (4 pcs.)	1	Pc.

Spare parts

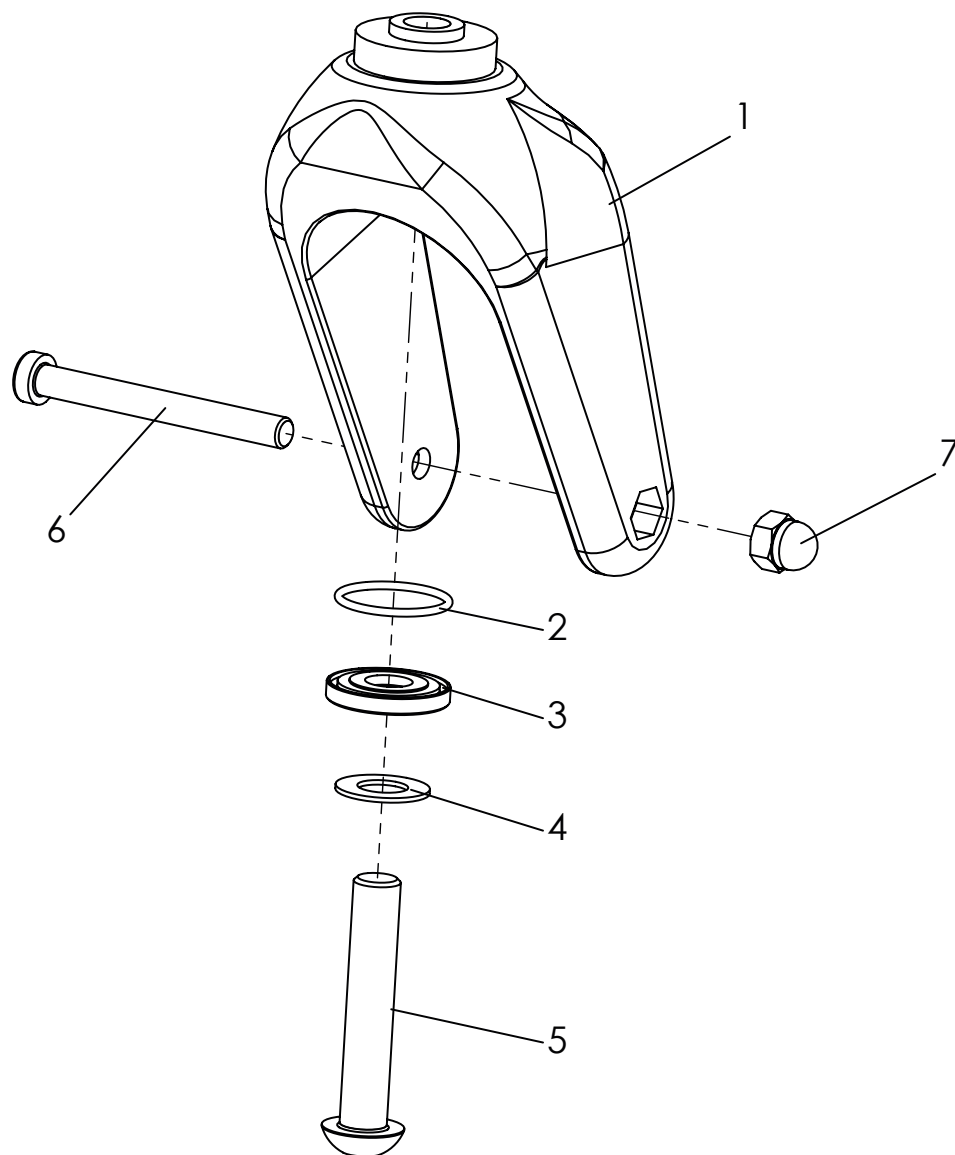


Pos	Article number	Articel description	Quantity	Unit
1-7	1281-0007E	Castor fork with fasteners	2	Pc.
1	1281-0007	Castor fork LG-K-200-12 schw 200x60	2	Pc.
2	1150-0023	O-ring 26x2,2	2	Pc.
3	3224-1174-008	Sealing disc caster fork	2	Pc.
4	1052-0012	Spring washer B12	2	Pc.
5	1029-1265	Button head socket screw M12x65	2	Pc.
6	1022-0875-033	Hexagon socket head cap screw M8x75	2	Pc.
7	1044-0008-033	Hexagonal domed cap nut self-locking M8	2	Pc.

Spare parts

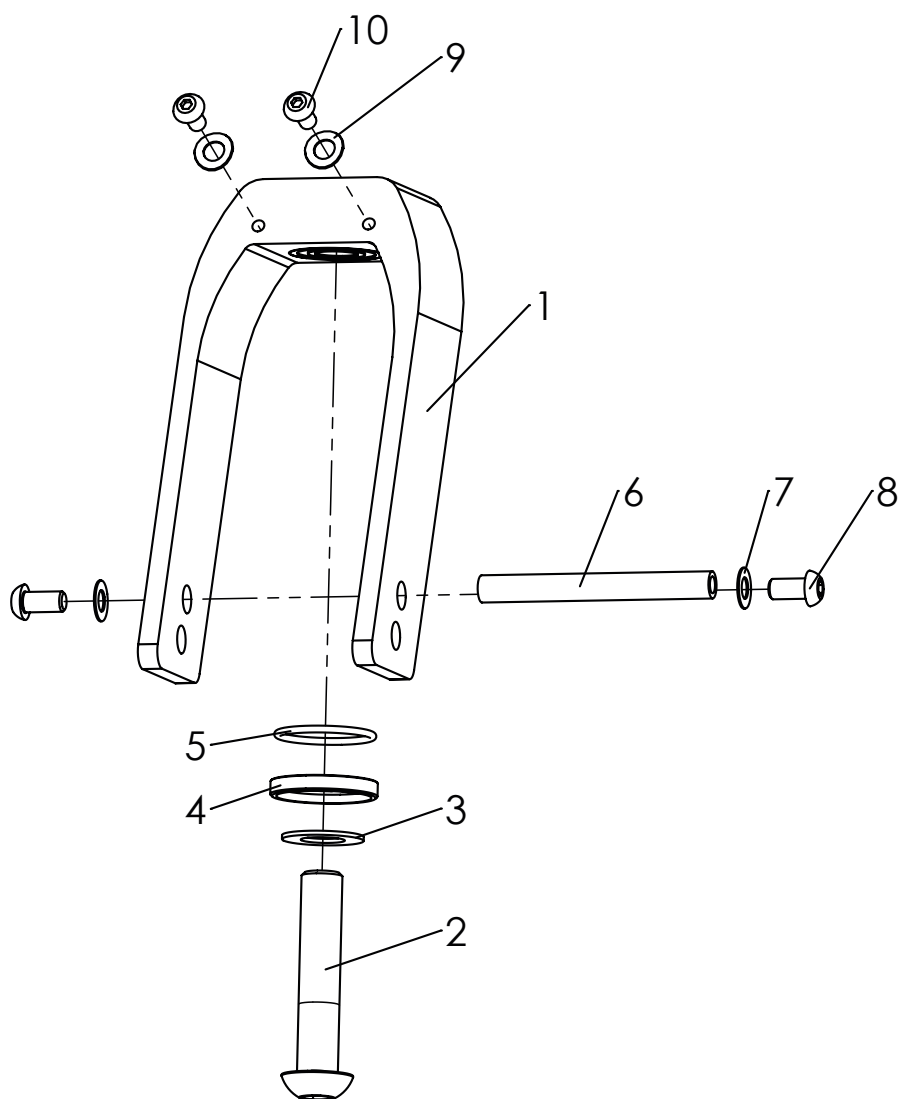


Pos	Article number	Articel description	Quantity	Unit
1-10	1281-1028E	Aluminum front forks	1	Pair
1	1281-0026LS-007	Castor fork LG-A-200-60-6001S	2	Pc.
2	1029-1260	Button head socket screw M12x60	2	Pc.
3	1052-0012	Spring washer B12	2	Pc.
4	3224-1174-008	Sealing disc caster fork	2	Pc.
5	1150-0018	O-ring 26x2	2	Pc.
6	1114-0057	Threaded bushing 8xM6x71	2	Pc.
7	1052-0006-033	Spring washer B6	4	Pc.
8	1029-6001-033	Button head socket screw M6x12	4	Pc.
9	1052-0005-033	Spring washer B5	4	Pc.
10	1029-0512-033	Button head socket screw M5x10	4	Pc.

Spare parts

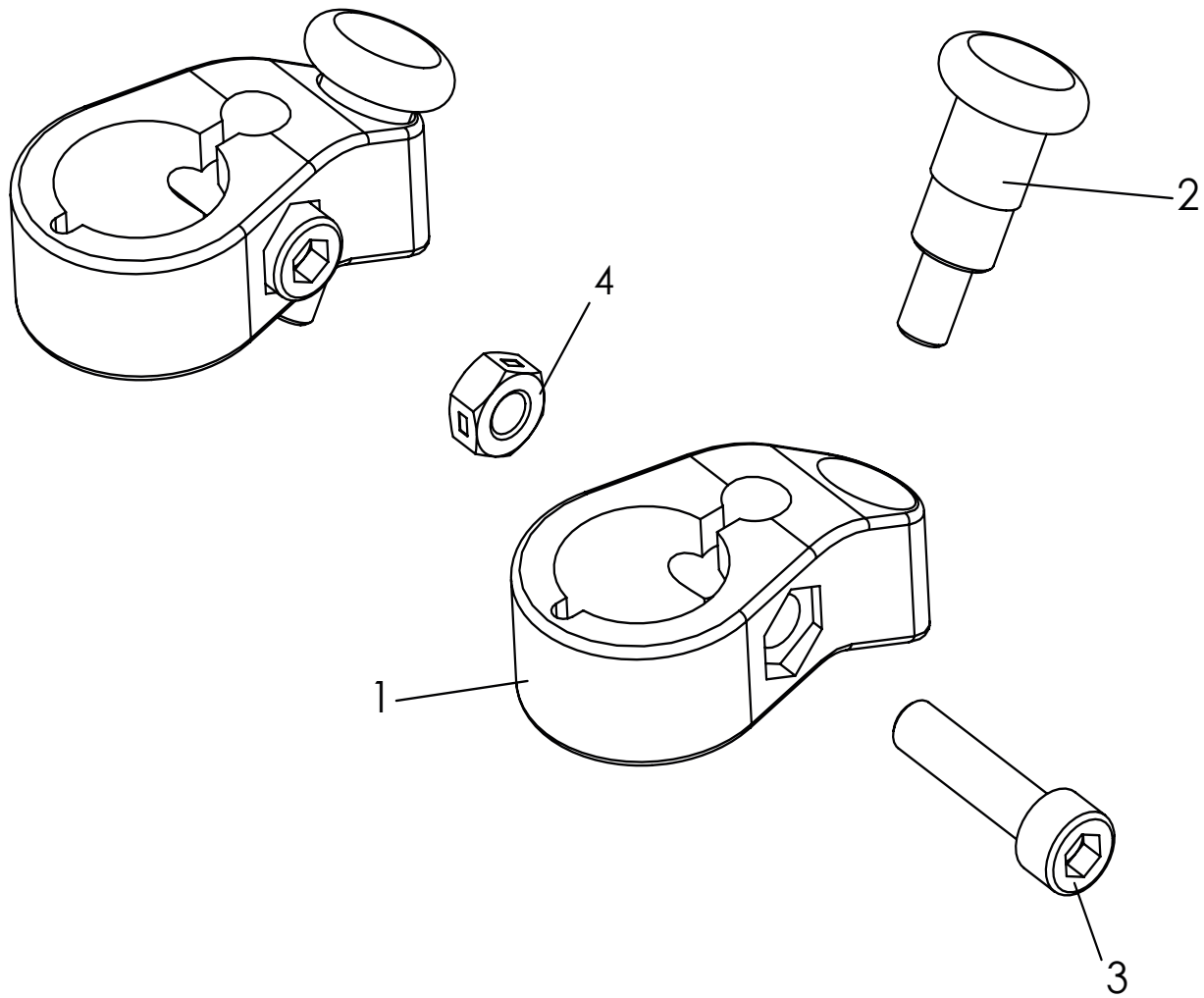


Pos	Article number	Articel description	Quantity	Unit
1-4	3202-7100	Swivel lock, for two castor fork`s	1	Pc.
1	3202-7110	Swivel lock housing	2	Pc.
2	1127-0009	Lock pin, with interlocksystem	2	Pc.
3	1021-0625-033	Hexagon socket head cap screw black M6x25	2	Pc.
4	1049-0070-033	BILOC-nut VM6	2	Pc.

Spare parts

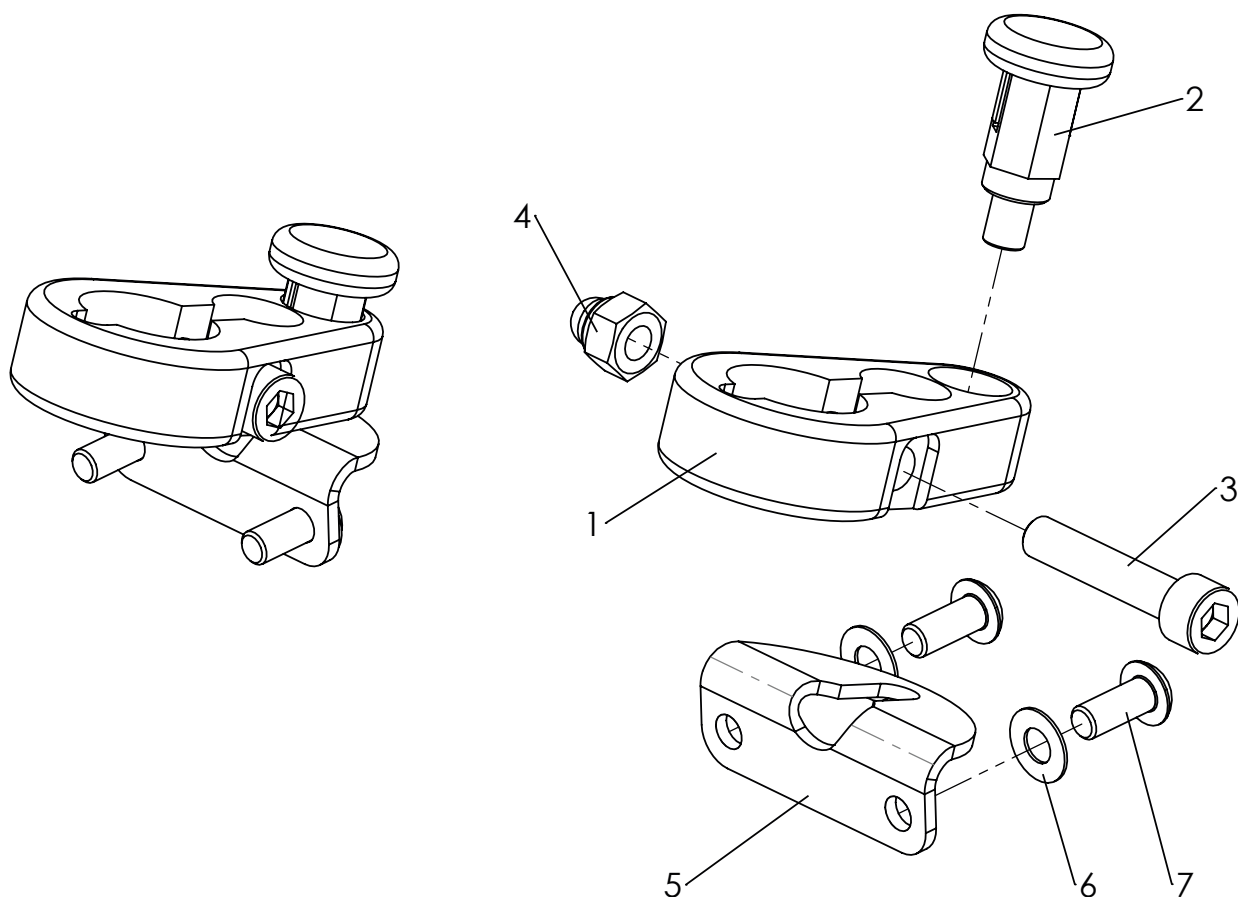


Pos	Article number	Articel description	Quantity	Unit
1-7	3232-7100	Swivel lock	1	Set
1	3232-7111	Swivel lock housing, black	2	Pc.
2	1127-0013	Lock pin with lock	2	Pc.
3	1021-0630	Hexagon socket head cap screw M6x30	2	Pc.
4	1044-0006	Hexagonal domed cap nut self-locking M6	2	Pc.
5	3232-7121-015	Catch plate black	2	Pc.
6	1052-0005	Spring washer B5	4	Pc.
7	1029-0512-033	Button head socket screw M5x10	4	Pc.

Spare parts

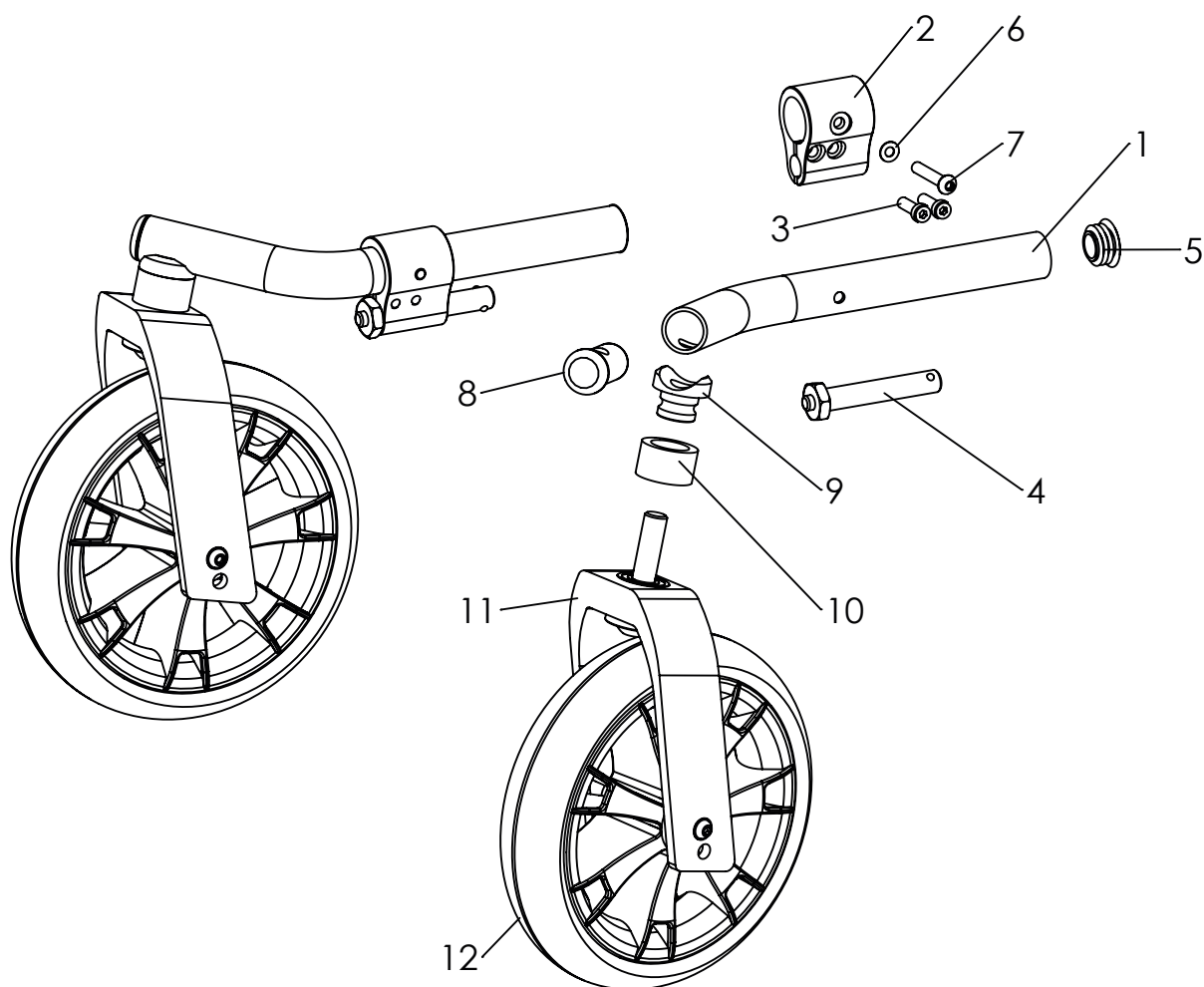


Pos	Article number	Articel description	Quantity	Unit
1-12	3032-1300E	Steering wheels Interchangeable attachment Switch +castor fork wheel	1	Pc.
1-10	3032-1301E	Steering wheels Interchangeable attachment Switch	1	Pc.
1	3032-1121-xxx	Steering wheel holder links	1	Pc.
	3032-1122-xxx	Steering wheel holder right	1	Pc.
2	3032-1131-007	Latching housing left, black anodized	1	Pc.
	3032-1132-007	Latching housing right, black anodized	1	Pc.
3	1022-0620	Hexagon socket head cap screw M6x20	4	Pc.
4	1205-0077	Quick release axle 12 x 60 mm	2	Pc.
5	1141-0045	Ribbed insert SFL 26 26x0,8-3 black	2	Pc.
6	1052-0005-033	Spring washer B5 black	2	Pc.
7	1029-0635	Button head socket screw M6x35	2	Pc.
8	3202-1130	Block	2	Pc.
9	3202-1140	Saddle bushing for front caster	2	Pc.
10	3202-1151	Cover ring	2	Pc.
11	1281-1027	Plastik front forks (EVO)	2	Pc.
	1281-1028	Aluminum front forks (EVO)	2	Pc.
12	1219-0055	Steering wheels 190x50 with pneumatic tires on black five-spoke rims	2	Pc.
	1229-0055	Steering wheels 190x50 with PU tires on black five-spoke rims	2	Pc.
	1229-0054	Steering wheels 200x50 with PU tires on "HOGGI" rims	2	Pc.

Spare parts

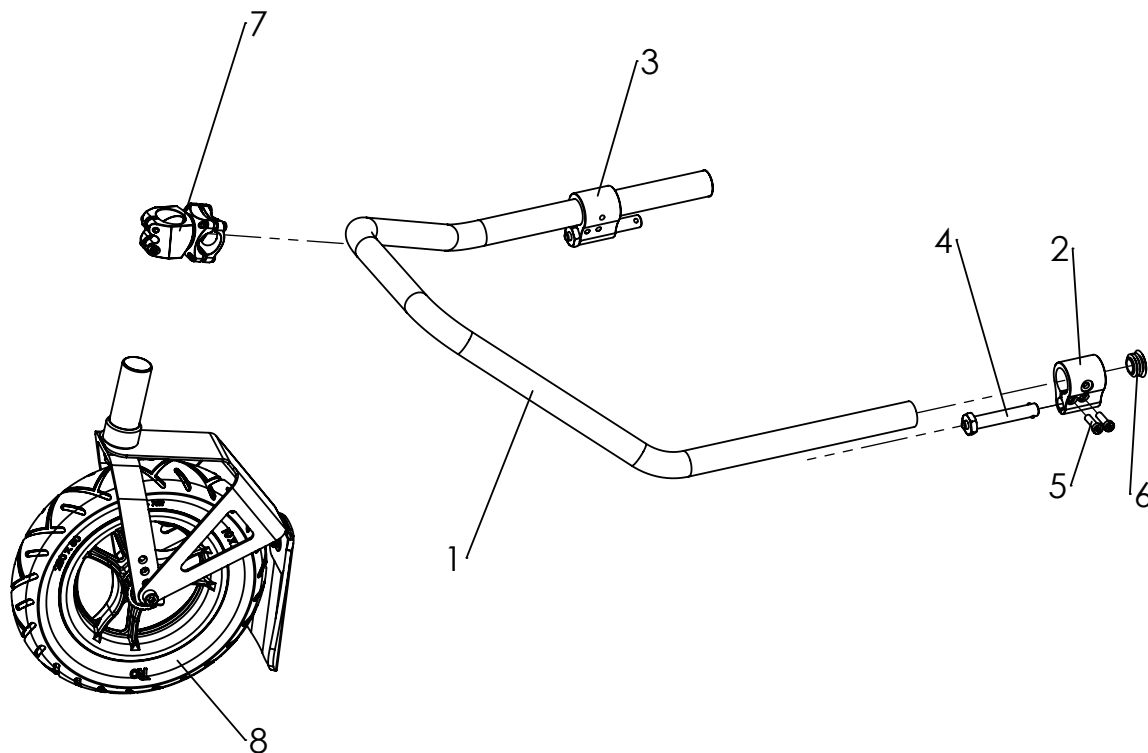


Pos	Article number	Articel description	Quantity	Unit
1-8	3032-1501E-xxx	Center steering wheel attachment (Mini/1), pneumatic tires	1	Pc.
	3034-1501E-xxx	Center steering wheel attachment (2/2XL), pneumatic tires	1	Pc.
	3032-1502E-xxx	Center steering wheel attachment (Mini/1), PU tires	1	Pc.
	3034-1502E-xxx	Center steering wheel attachment (2/2XL), PU tires	1	Pc.
1-7	3032-1500E-xxx	Center wheel attachment BINGO SWITCH size 1	1	Pc.
	3034-1500E-xxx	Center wheel attachment BINGO SWITCH size 2	1	Pc.
1	3032-1511	Tubular bracket outdoor attachment Bingo Switch size 1	1	Pc.
	3034-1511	Tubular bracket outdoor attachment Bingo Switch size 2	1	Pc.
2	3032-1131-007	Latching housing left, black anodized	1	Pc.
3	3032-1132-007	Latching housing right, black anodized	1	Pc.
4	1205-0077	Quick release axle 12 x 60 mm	2	Pc.
5	1022-0620	Hexagon socket head cap screw M6x20	4	Pc.
6	1141-0045	Ribbed insert SFL black	2	Pc.
7	7000-0242	Handlebar stem	1	Pc.
8	3032-1540	Center wheel for SWITCH, pneumatic tyres	1	Pc.
	3032-1530	Center steering wheel for SWITCH, PU tyres	1	Pc.

Spare parts

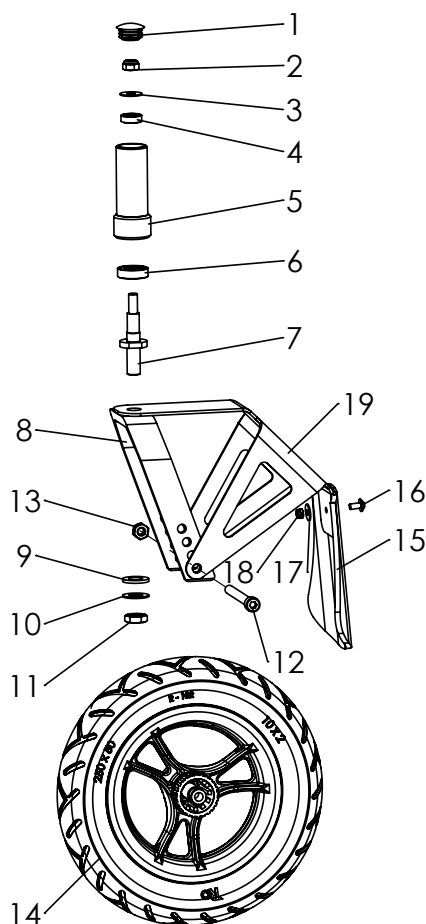


Pos	Article number	Articel description	Quantity	Unit
1-19	3032-1540	Center wheel BG for SWITCH, pneumatic tyres	1	Pc.
	3032-1530	Center steering wheel BG for SWITCH, PU tyres	1	Pc.
1	1141-0045	Ribbed insert black	1	Pc.
2	1042-0008	Hexagon nut self-locking M8	1	Pc.
3	1113-0071	Washer PA black 24x5,3x5x	1	Pc.
4	1090-0001	Grooved ball bearing 619-8 ZZ D19xd8x6	1	Pc.
5	3032-1532-007	Oxygen bottle holder short version (d=120 mm)	1	Pc.
6	1090-0005	Grooved ball bearing 6001 2RS D28xd12x8	1	Pc.
7	1205-0007	Wheel fork axle	1	Pc.
8	1281-0013	Wheel fork aluminum	1	Pc.
9	1051-0012	Washer A13	1	Pc.
10	1052-0012	Spring washer B12	1	Pc.
11	1043-0012	Hexagon nut M12	1	Pc.
12	1022-0880	Hexagon socket head cap screw M8x80	1	Pc.
13	1049-0067	BILOC-nut VM8	1	Pc.
14	1211-0052	Pneumatic wheel 250x50 out door frame	1	Pc.
	1221-0023	PU-wheel 250x50 out door frame	1	Pc.
15	7000-0169	Spoiler XXL 53 mm	1	Pc.
16	1030-0510	Button head socket screw with flange M5x10	1	Pc.
17	1055-0005	Washer A5,3	1	Pc.
18	1042-0005	Hexagon nut self-locking M5	1	Pc.
19	3246-9512	Wheel guard	1	Pc.

Spare parts

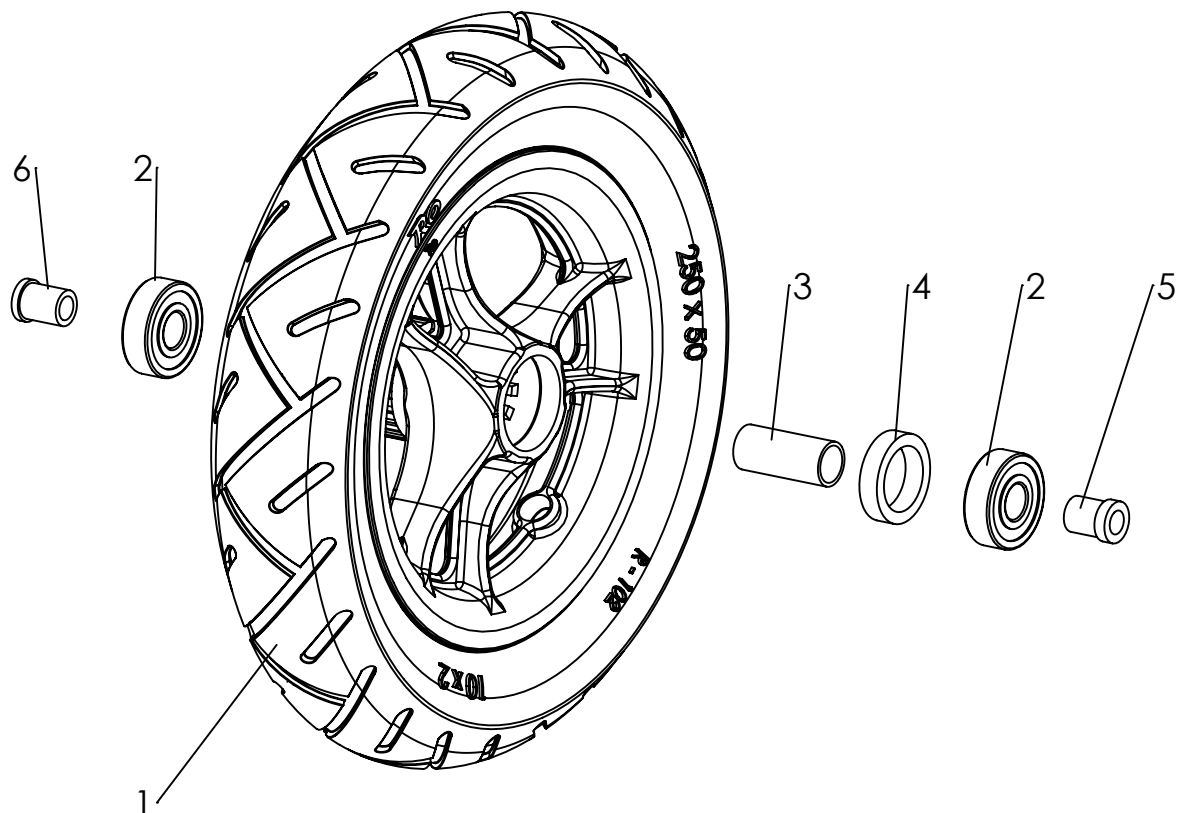


Pos	Article number	Articel description	Quantity	Unit
1-6	1211-0052	Pneumatic wheel 250x50 out door frame	1	Pc.
	1221-0023	PU-wheel 250x50 out door frame	1	Pc.
1	1211-0041	Pneumatic wheel 250x50 on black 6-spoke rim, with quick release	1	Pc.
	1221-0041	PU-wheel 250x50 on black 6-spoke rim, with quick release	1	Pc.
2	1090-0002	Grooved ball bearing 6201 2RS D32xd12x10	1	Pc.
3	1205-0039	Distance tube 15x1.3x34 DIN 2393 BK	1	Pc.
4	1209-0011	Spacer ring 30x7	1	Pc.
5	1205-0037	Reducing bush 12/8 with collar 2.5 mm	1	Pc.
6	1205-0036	Reducing bush 12/8 with collar 4 mm	1	Pc.

Spare parts

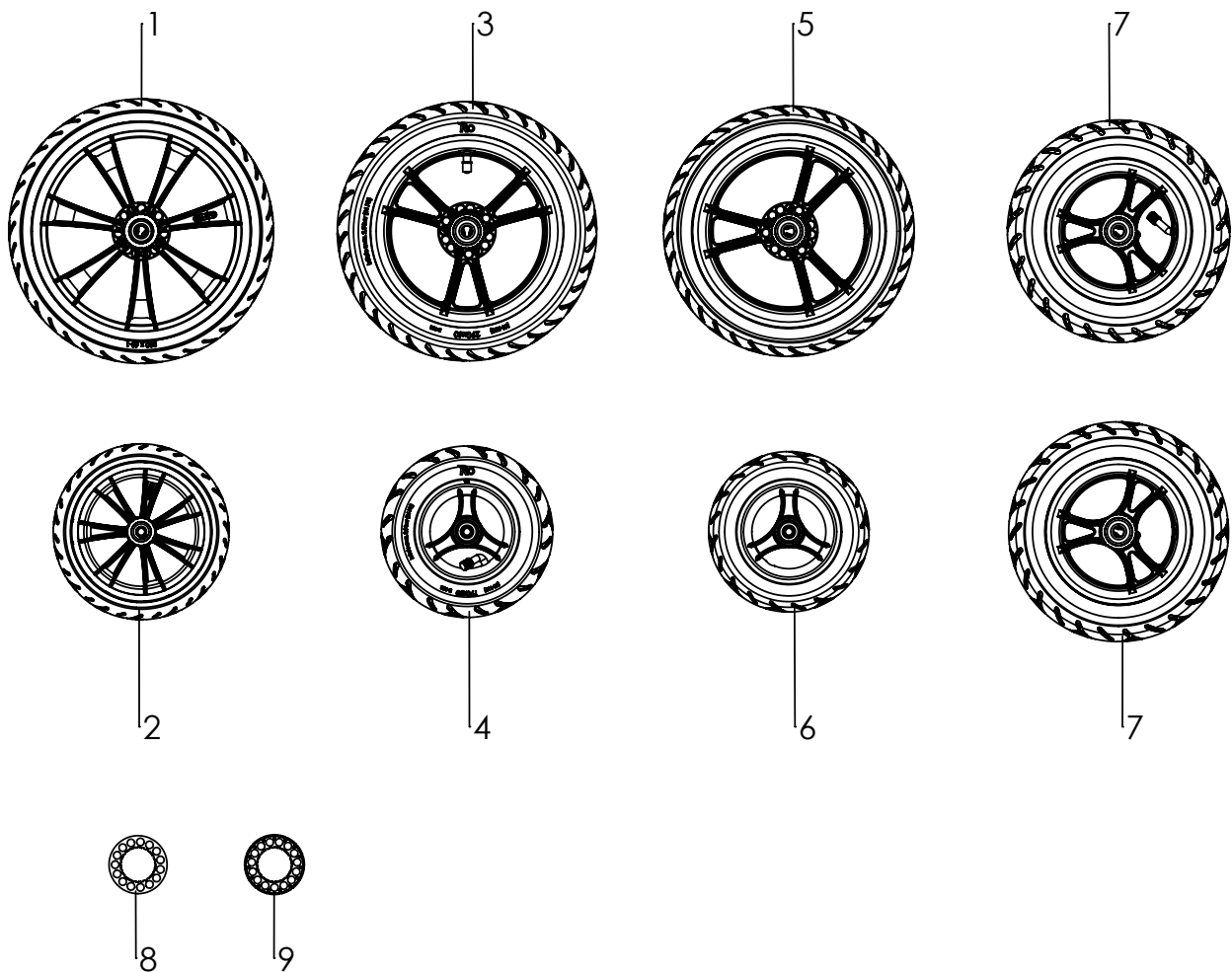


Pos	Article number	Articel description	Quantity	Unit
1	1221-3063	PU-Wheel R300/60, ABS black	2	Pc.
	1221-3064	PU-Wheel R300/60, ABS black, brake disk with aluminium ring	2	Pc.
2	1221-2049	PU-Wheel R200/48, ABS black	2	Pc.
3	1211-0036	Pneumatic wheel R290/60, ABS black	2	Pc.
	1211-2962E	Pneumatic wheel R290/60, ABS black, brake disk with aluminium ring	2	Pc.
4	1211-0017	pneumatic wheel 190x50, ABS black	2	Pc.
5	1221-0037	PU-wheel 29x60, ABS black	2	Pc.
	1221-0037-AS	PU-wheel 29x60, ABS black, brake disk with aluminium ring	2	Pc.
6	1221-0017	PU-wheel 190x50, ABS black	2	Pc.
7	1211-0041	pneumatic wheel 250x50, ABS black	2	Pc.
	1221-0041	PU-wheel 250x50, ABS black	2	Pc.
8	1204-0002	Brake disc, plastic	2	Pc.
9	1204-0005	Brake disc with aluminium-ring	2	Pc.
	1200-0001	tire for wheel 290x60	1	Pc.
	1200-0002	tire for wheel 190 x 50	1	Pc.
	1201-0001	tube for wheel 290 x 60	1	Pc.
	1201-0002	tube for wheel 190 x 50	1	Pc.

Spare parts

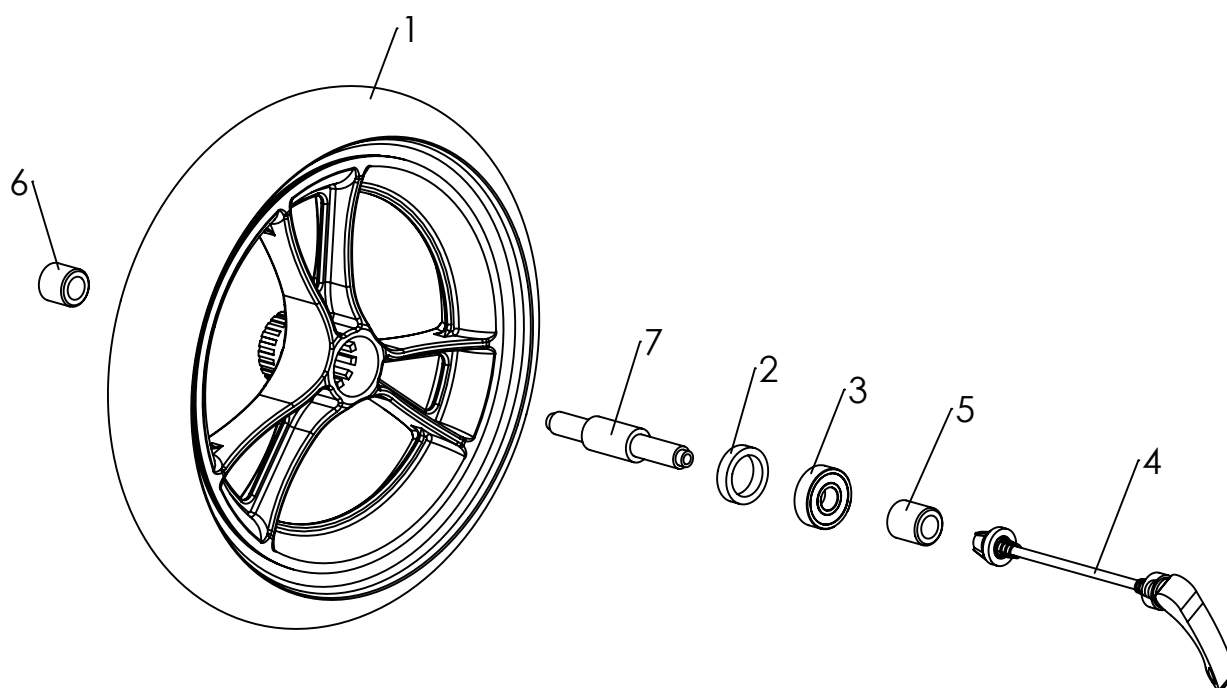

Pos	Article number	Articel description	Quantity	Unit
1-3	1211-0021	Pneumatic wheel 290x60 on black rims, TRIKE front wheel	1	Pc.
	1221-0021	PU-wheel 290x60 on black rims, TRIKE front wheel	1	Pc.
	1211-0026	Pneumatic wheel 290x60 on 5-spokes black, TRIKE front wheel	1	Pc.
	1221-0038	PU wheel 290x60 on 5-spokes black, TRIKE front wheel	1	Pc.
2	1209-0011	Spacer ring, black plastic	1	Pc.
3	1090-0002	Grooved ball bearing 6201 2RS D32xd12x10	1	Pc.
4	1205-0004	Quick release 111 mm for hollow axle	1	Pc.
5	3206-1160	Pressure bushing L 23	1	Pc.
6	3206-1150	Pressure bushing L 19	1	Pc.
7	3206-1140	Hollow axle	1	Pc.


Spare parts

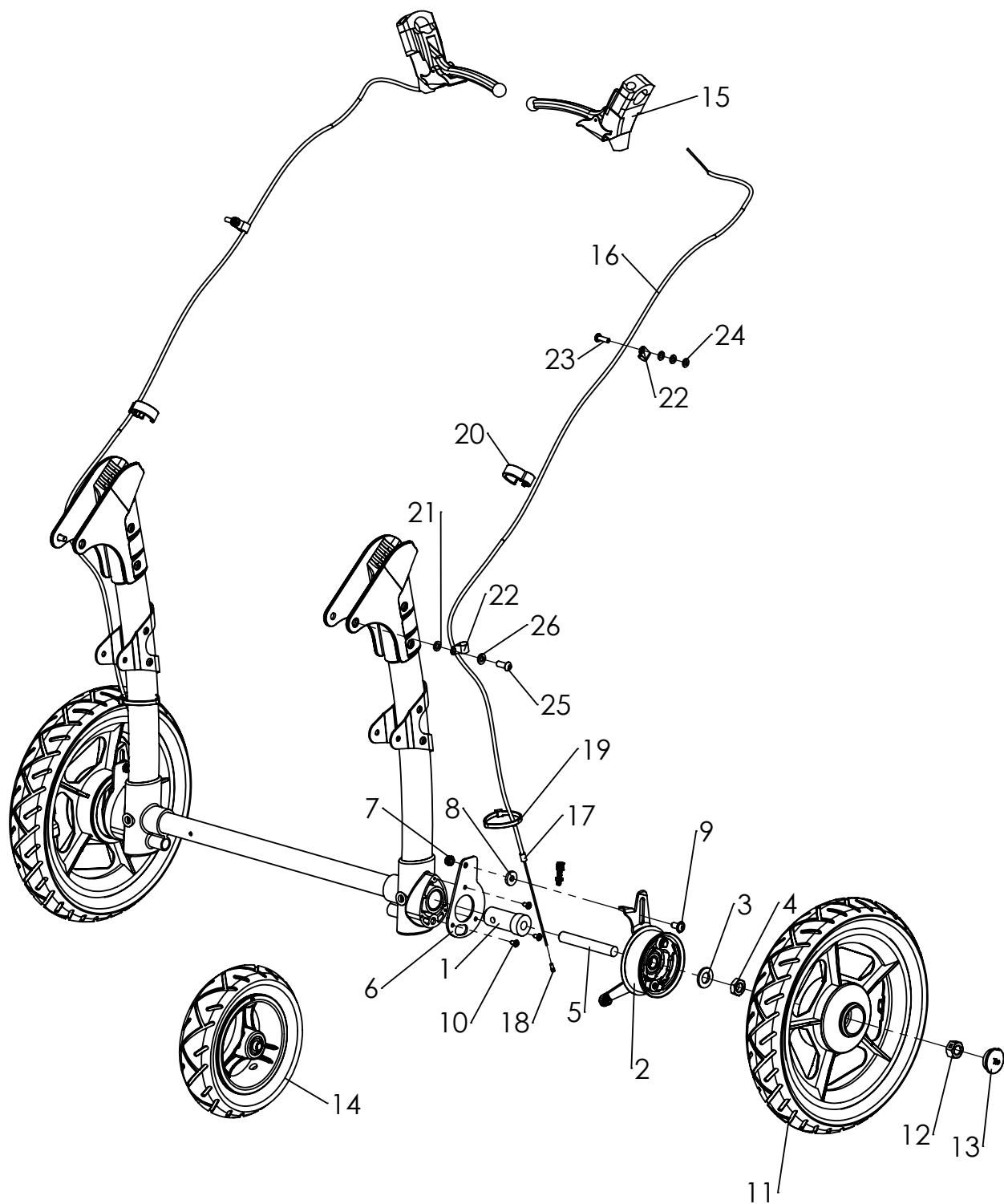

Spare parts


Pos	Article number	Articel description	Quantity	Unit
1-24	1219-0029E	Drum brake wheels, pneumatic tyres, 5-spoke rim, rigid front wheels	1	Set
	1229-0029E	Drum brake wheels, PU tyres, 5-spoke rim, rigid front wheels	1	Set
	1219-0030E	Drum brake wheels, pneumatic tyres, 5-spoke rim, swivelling front wheels	1	Set
	1229-0030E	Drum brake wheels, PU tyres, 5-spoke rim, swivelling front wheels	1	Set
	1219-0031E	Drum brake wheels, pneumatic tyres, 5-spoke rims, TRIKE	1	Set
	1229-0031E	Drum brake wheels, PU tyres, 5-spoke rim, TRIKE	1	Set
	1219-0057E	Drum brake wheels, pneumatic tyres, 5-spoke rim, Switch with central steering wheel	1	Set
	1229-0057E	Drum brake wheels, PU tyres, black 5-spoke rim, Switch with central steering wheel	1	Set
1	3201-1271	Bracket	2	Pc.
2	1205-0041	Brake plate steel left with 12 mm hole	1	Pc.
	1205-0042	Brake plate steel right with 12 mm hole	1	Pc.
3	1051-0012	Washer A 13	2	Pc.
4	1043-0012	Hexagon nut M12	2	Pc.
5	3201-1242	Axle bolt	2	Pc.
6	3201-1281	Stabilizer		
7	1042-0006-033	Hexagon nut self-locking black M6		
8	1059-0003	Washer M6	2	Pc.
9	1030-0612	button head socket screw with flange M6x12	2	Pc.
10	1039-0001	screw for plastic, cross pan head w. flange 4x10	2	Pc.
11	1219-0032	Pneumatic wheel 310x55, black 5 spoke plastic rim, drum brake left side	1	Set
	1219-0033	Pneumatic wheel 310x55, 5 spoke plastic rim, drum brake right side	1	Set
	1229-0025	PU-wheel 290x60, 5 spoke plastic rim, drum brake left side	1	Set
	1229-0026	PU-wheel 290x60, 5 spoke plastic rim, drum brake right side	1	Set
12	1049-0065	BILOC-nut M12	2	Pc.
13	1209-0004	Hub cap KN-S-32/12	2	Pc.
14	1211-0017	Pneumatic wheel 190x50 with black 3 spoke plastic rim	2	Pc.
	1221-0017	PU-wheel 190x50 with black 3 spoke plastic rim	2	Pc.
	1211-0025	Pneumatic wheel 250x50, black 5 spoke plastic rim with quick release	2	Pc.
	1221-0036	PU-wheel 250x50, black 5 spoke plastic rim with quick release	2	Pc.
	1211-0026	Pneumatic 5 spoke wheel black, 290x60, TRIKE front wheel	1	Pc.
	1211-0038	PU 5 spoke wheel black, 290x60, TRIKE front wheel	1	Pc.
15	1209-0030	Brake lever 36 ° manually detent 22 mm	2	Pc.
16-18	1209-0001	Bowden wire set 1,5m cable	2	Set
19	1089-0004	Cable tie, black, 5x180	2	Pc.
20	1149-0001	Cable clip D 25	2	Pc.
21	1050-0006	Washer A6,4	2	Pc.
22	1149-0003	Cable clip d 6,4 / 8 mm, black	4	Pc.
23	1062-0516	Blind rivet F5x16	2	Pc.
24	1051-0005	Washer A5,3	6	Pc.
25	1029-0616	Button head socket screw M6x16	2	Pc.

Spare parts

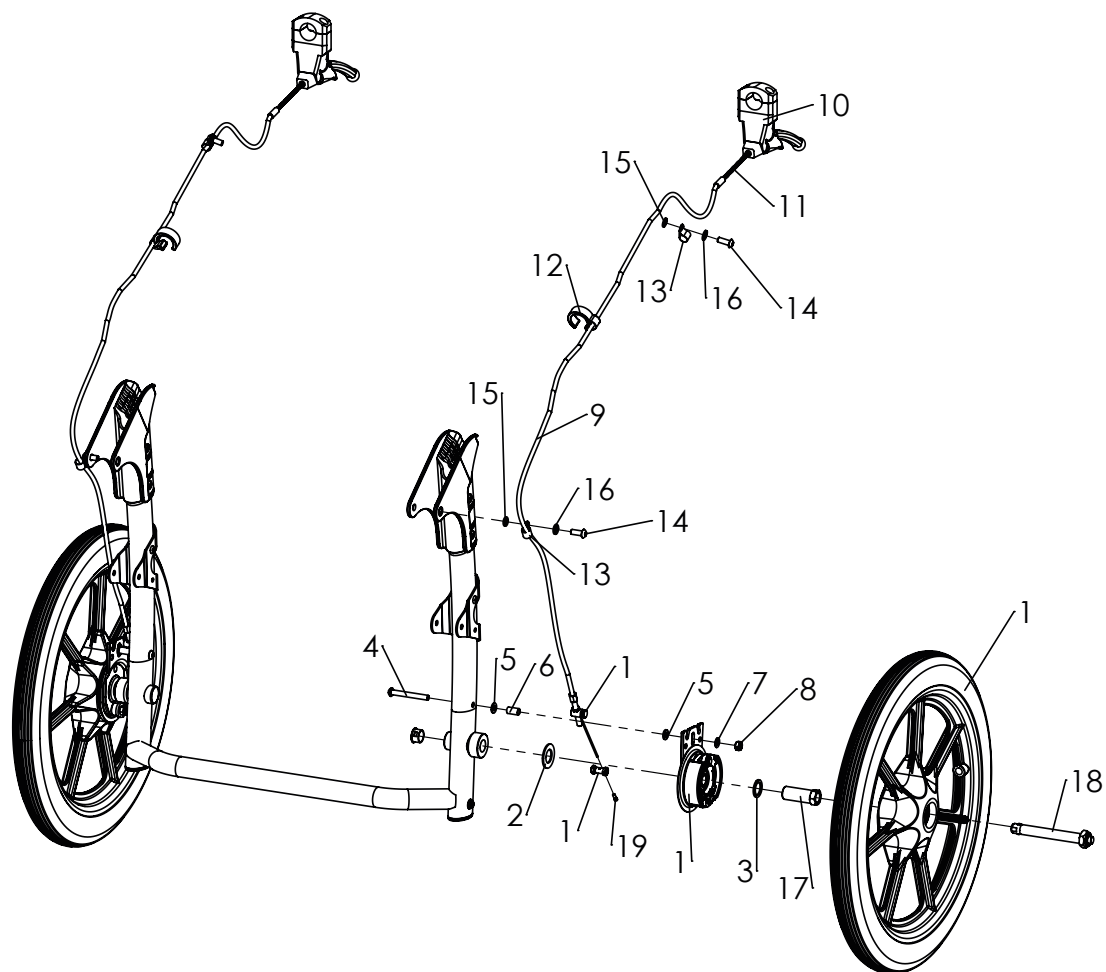


Pos	Article number	Articel description	Quantity	Unit
1	1219-0059	Pneumatic wheel black 16 x 1,75" drum brake left side	1	Pc.
	1219-0060	Pneumatic wheel black 16 x 1,75" drum brake right side	1	Pc.
	1229-0059	PU-wheel black 400x40, black drum brake left side	1	Pc.
	1229-0060	PU-wheel black 400x40, black drum brake right side	1	Pc.
2-19	3232-1290E	Brake components EVO CROSS complete	1	Set
	3235-1290E	Brake components EVO CROSS 2XL complete	1	Set
2	1052-0016	Spring washer B16	2	Pc.
3	1059-0002	Shim ring 16x22x1,5	2	Pc.
4	1029-0650-033	Button head socket screw M6x50, black	2	Pc.
5	1051-0006	Washer A6,4	6	Pc.
6	1114-0018	Bushing 8x0,8x14,5	2	Pc.
7	1052-0006-033	Spring washer B6 black	4	Pc.
8	1049-0070-033	BILOC-nut VM6	2	Pc.
9	1209-0001	Bowden wire set 1,5m cable with 1,3m cable housing, end cap for cable housing, end cap for cabel end	2	Pc.
10	1209-0008	Brake lever 80° manually locking 22 mm (for push-bar)	2	Pc.
11	1205-0054	Bowden cable guide, flexible	2	Pc.
12	1149-0001	Cable clip D 25	2	Pc.
13	1149-0003	Cable clip d 6,4 / 8 mm, black	2	Pc.
14	1029-0620-033	button head socket screw black M6x20	2	Pc.
15	1050-0006	Washer A6,4	2	Pc.
16	1051-0006-033	Washer A6,4, black	2	Pc.
17	1205-0010	Adapter with nut M16x1 for quick-release-axle 12/45 mm	2	Pc.
18	1205-0068	Quick release axle 12 x 85 mm	2	Pc.
19	1209-0007	End cap aluminium	2	Pc.

Spare parts

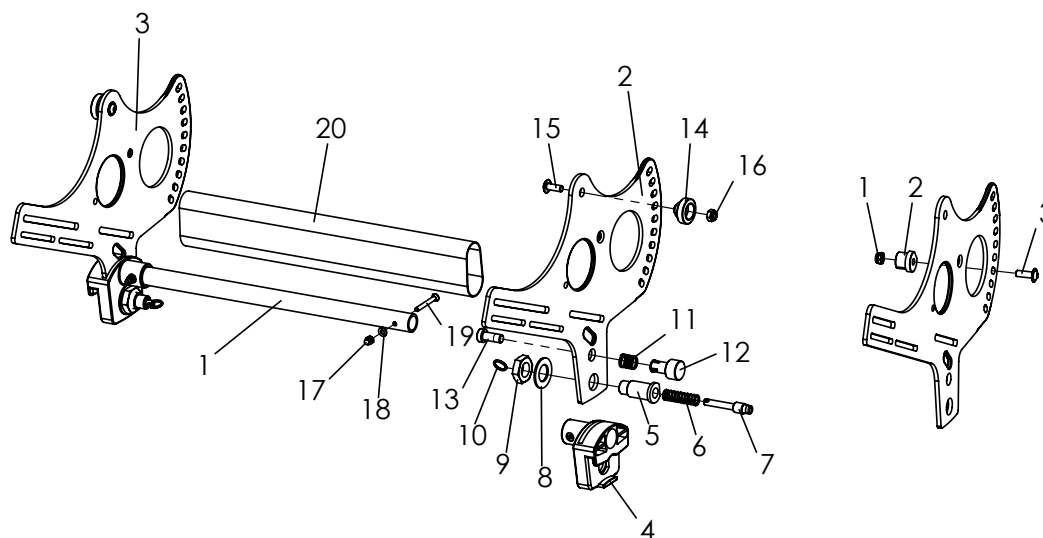


Pos	Article number	Articel description	Quantity	Unit
1-20	3230-2100E-032	Seat base Mini, complete	1	Pc.
	3232-2100E-xxx	Seat base size 1, complete	1	Pc.
	3234-2100E-xxx	Seat base size 2, complete	1	Pc.
	3235-2100E-xxx	Seat base size 2XL, complete	2	Pc.
1	3230-2119-xxx	Cross strut, seat Mini	1	Pc.
	3231-2119-032	Cross strut, seat size 1	1	Pc.
	3233-2119-xxx	Cross strut, seat size 2	1	Pc.
	3230-2121-007	Side plate seat base Mini, left	1	Pc.
2	3231-2121-007	Side plate, seat base size 1 and 2, left	1	Pc.
	3235-2121-007	Side plate, seat base size 2XL, left	1	Pc.
3	3230-2122-007	Side plate seat base Mini, right	1	Pc.
	3231-2122-007	Side plate, seat base size 1 and 2, right	1	Pc.
	3235-2122-007	Side plate, seat base size 2XL, right	1	Pc.
4	3231-2114	Scantlings guide	2	Pc.
5	3231-2115	Locking bolt housing	2	Pc.
6	1131-0070	Compression D-162	2	Pc.
7	3240-2422	Locking bolt	2	Pc.
8	1052-0016	Spring washer B16	2	Pc.
9	1040-0016	Hexagon nut 16x1,5	2	Pc.
10	1129-0036	Key ring 16x1,6	2	Pc.
11	1131-0065	Compression spring D-117J	1	Pc.
12	3231-2124	Lock button	1	Pc.
13	1022-0820	Hexagon socket head cap screw M8x20	1	Pc.
14	3231-2113	Scantlings shaft	2	Pc.
15	1030-0616	Button head socket screw with flange M6x16	2	Pc.
16	1042-0006	Hexagon nut self-locking M6	2	Pc.
17	1044-0004	Hexagonal domed cap nut self-locking M4	2	Pc.
18	1051-0004	Washer A4,3	2	Pc.
19	1024-0430	Hexagon socket countersunk head screw M4x30	2	Pc.
20	3230-2140	Cover for tilt mechanism mini	1	Pc.
	3231-2140	Cover for tilt mechanism size 1	1	Pc.
	3233-2140	Cover for tilt mechanism size 2	1	Pc.
1-3	<i>Limitation of back tilt</i>			
	<i>Size 2XL, to be mounted fourth hole from the bottom!</i>			
1	1042-0006	Hexagon nut self-locking M6	2	Pc.
2	3240-1180	Bearing plug	2	Pc.
3	1030-0618	Button head socket screw with flange M6x18	2	Pc.

Spare parts

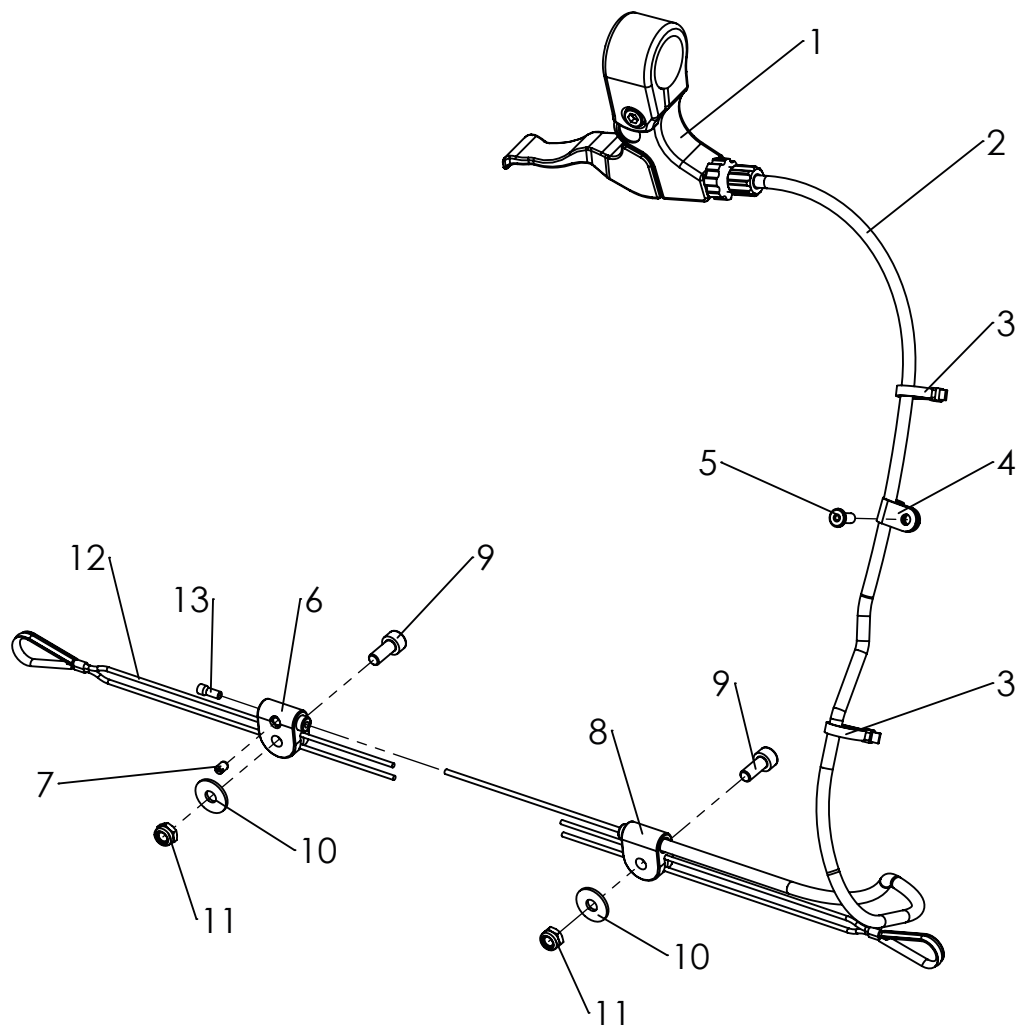


Pos	Article number	Articel description	Quantity	Unit
2-13	3231-1900E	Releaser, tiltingmechanism	1	Pc.
1	1209-0017	Brake lever with splittet bracket, d 22 mm black	1	Pc.
2	1209-0029	Cable BZ-670x780	1	Pc.
3	1089-0007	Cable tie 3,3 x 145	2	Pc.
4	1119-0003	Wire clamp	1	Pc.
5	1062-0410	Blind rivit F4x10	1	Pc.
6	3243-1932	Nipple bearing	1	Pc.
7	1099-0004	Screw nipple 1100, D5x8	1	Pc.
8	3243-1931	Bowden bearing	1	Pc.
9	1021-0512	Cylinder head screw, M5x12	2	Pc.
10	1055-0005	Washer A5,3	2	Pc.
11	1042-0005	Hexagon nut self-locking M5	2	Pc.
12	1129-1001	Universal wire-rope with rope thimble for seat angle adjustment	2	Pc.
13	1209-0007	End cap aluminium U-BA-S for Bowden cable wire	1	Pc.

Spare parts

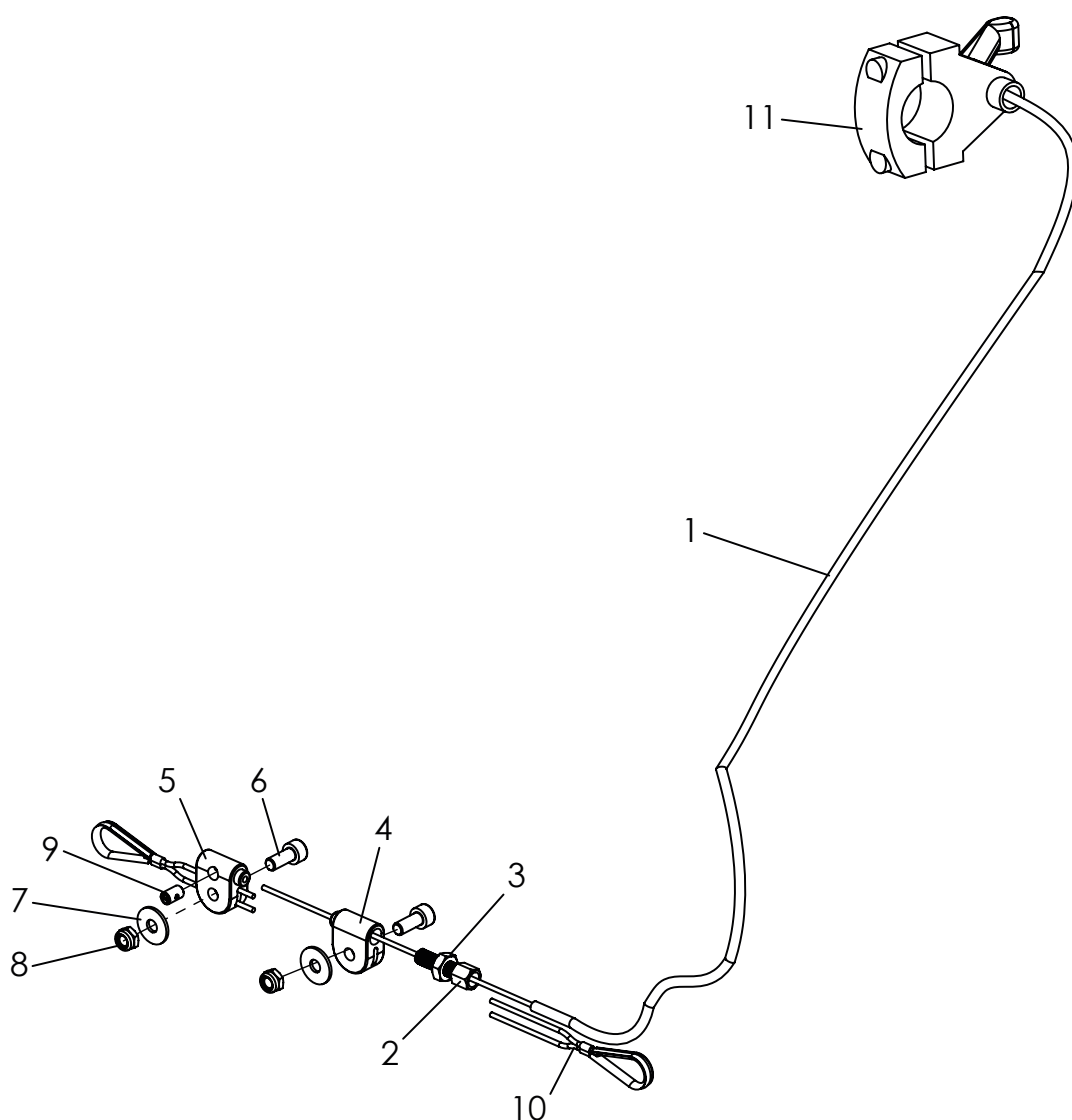


Pos	Article number	Articel description	Quantity	Unit
1-10	3230-1900E	Releaser, tiltingmechanism, Mini	1	Pc.
1	1209-0042	Bowden cable BZ-600x750	1	Pc.
2	1099-0003	Adjusting screw	1	Pc.
3	1042-0006	Hexagon nut self-locking M6	1	Pc.
4	3007-1931	Bowden bearing prepared for adj. Screw	1	Pc.
5	3243-1932	Nipple bearing	1	Pc.
6	1021-0512	Cylinder head screw M5x12	2	Pc.
7	1055-0005	Washer A5,3	2	Pc.
8	1042-0005	Hexagon nut self-locking M5	2	Pc.
9	1099-0004	Screw nipple 1100, D5x8	1	Pc.
10	3230-1910	Wire-rope with rope thimble, for seat angle adjustment, Mini	2	Pc.
11	1209-0036	Release lever D22	1	Pc.

Spare parts

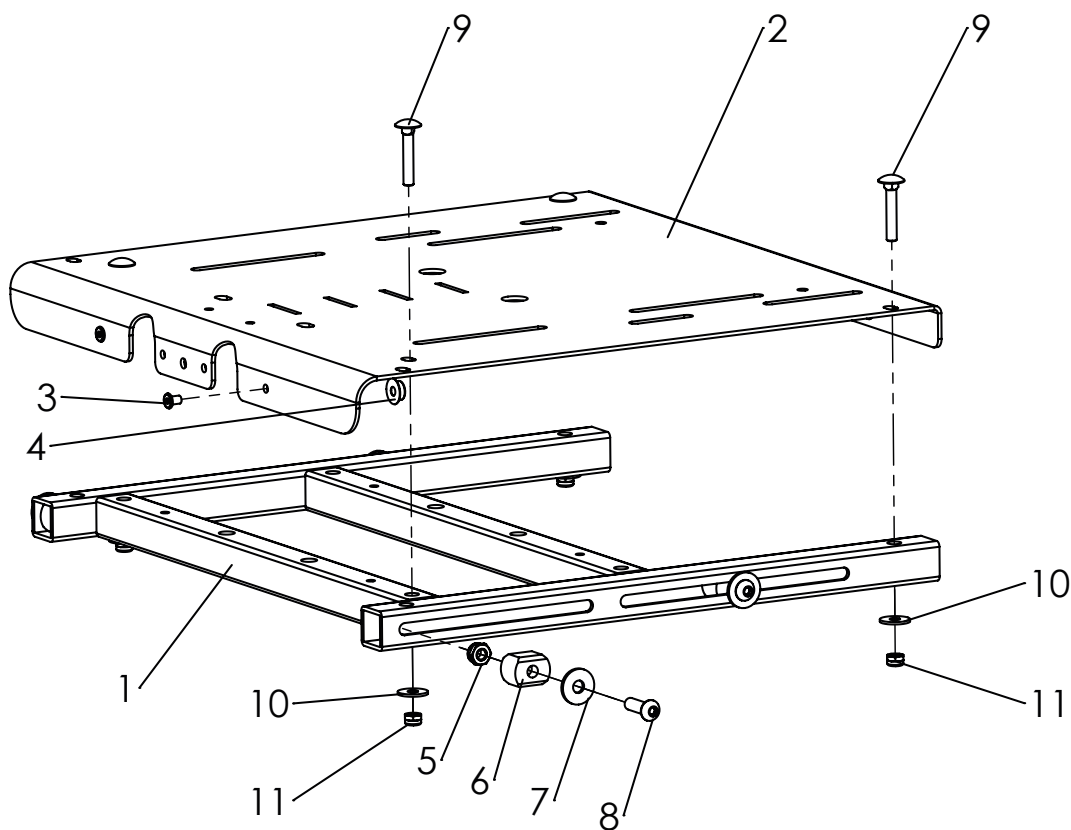


Pos	Article number	Articel description	Quantity	Unit
1-11	3232-2200E-xxx	Seat surface size 1	1	Pc.
	3234-2200E-xxx	Seat surface size 2	1	Pc.
	3235-2200E-xxx	Seat surface size 2 XL	1	Pc.
1	3231-2250E-xxx	Adjustable seat frame size 1	1	Pc.
	3233-2250E-xxx	Adjustable seat frame size 2	1	Pc.
	3235-2250E-xxx	Adjustable seat frame size 2 XL	1	Pc.
2-4	3232-2221E-xxx	Seat plate size 1	1	Pc.
	3234-2221E-xxx	Seat plate size 2	1	Pc.
	3236-2221E-xxx	Seat plate size 2 XL	1	Pc.
3	1061-0406	Blind rivet F4x6	4	Pc.
4	1533-0005	Ring-ball part DRK 95B	4	Pc.
5	1042-0006	Hexagon nut self-locking M6	4	Pc.
6	3231-2262	Clamping block	4	Pc.
7	1055-0006-033	Washer A6,4 black	4	Pc.
8	1029-0620-033	Button head socket screw black M6x20	4	Pc.
9	1026-0530	Mushroom head square neck bolt M5x30	4	Pc.
10	1055-0005	Washer A5,3	4	Pc.
11	1042-0005	Hexagon nut self-locking M5	4	Pc.

Spare parts

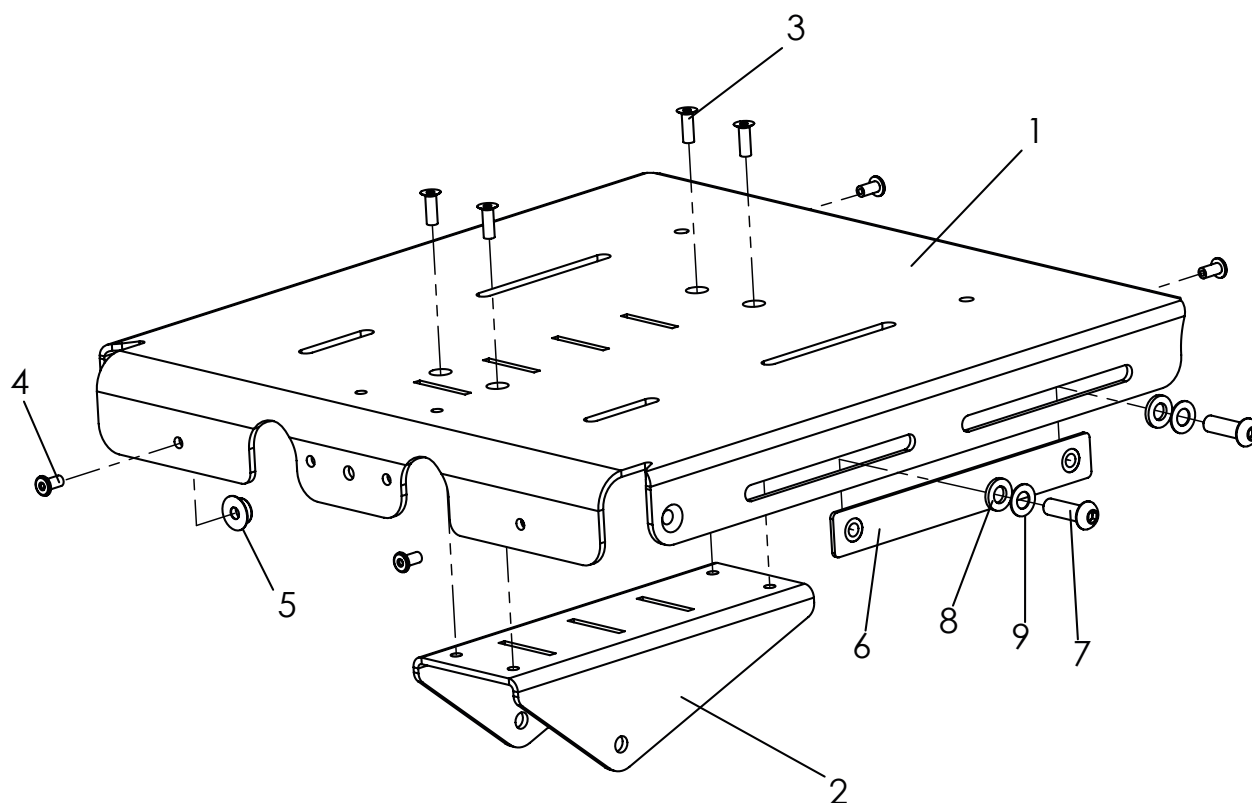


Pos	Article number	Articel description	Quantity	Unit
1-9	3230-2200E-032	Seat surface Mini, complete	1	Pc.
1-5	3230-2211E-032	Seat surface Mini	1	Pc.
2	3230-2212-032	Retainer for Footrest hanger	1	Pc.
3	1063-0412	Blind rivet S4x12	4	Pc.
4	1061-0406	Blind rivet F4x6	4	Pc.
5	1533-0005	Ring-ball part	4	Pc.
6	3230-2231	Threadplate M6x110	2	Pc.
7	1029-0620-033	Button head socket screw black M6x20	4	Pc.
8	1055-0006-033	Washer A6,4	4	Pc.
9	1052-0006-033	Spring washer B6	4	Pc.

Spare parts

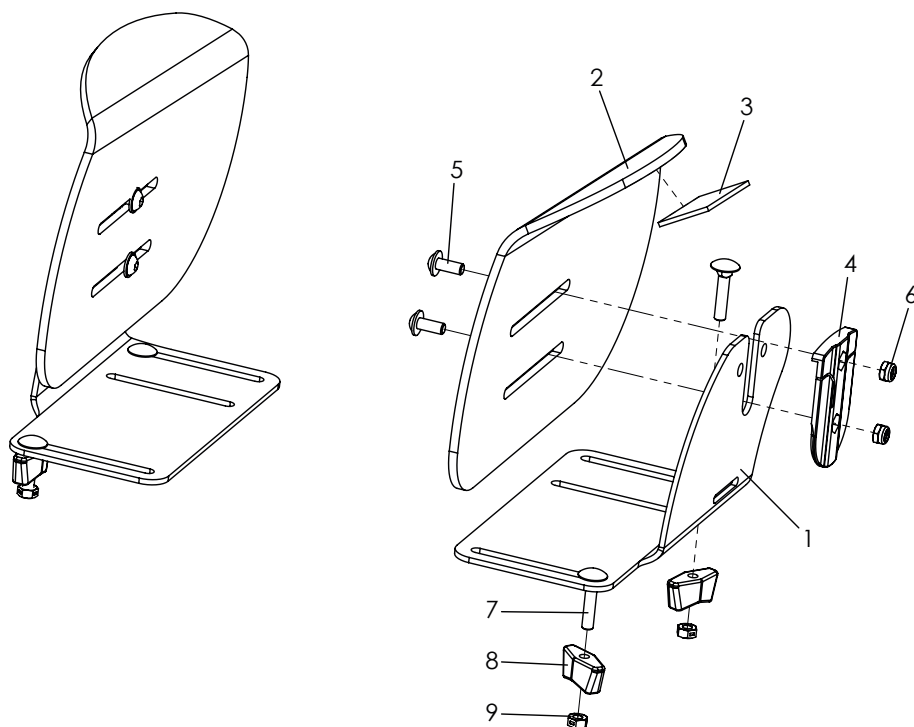


Pos	Article number	Articel description	Quantity	Unit
1-9	3230-2602E	Hip support right Mini	1	Pc.
	3230-2601E	Hip support left Mini	1	Pc.
	3231-2602E	Hip support right size 1	1	Pc.
	3231-2601E	Hip support left size 1	1	Pc.
	3233-2602E	Hip support right size 2	1	Pc.
	3233-2601E	Hip support left size 2	1	Pc.
	3235-2602E	Hüftpelotte right Gr. 2XL	1	Pc.
	3235-2601E	Hüftpelotte left Gr. 2XL	1	Pc.
1	3230-2612-032	Bracket for hip supports, right, Mini, black dull matt	1	Pc.
	3230-2611-032	Bracket for hip supports, left, Mini, black dull matt	1	Pc.
	3231-2612-032	Bracket for hip supports, right, size 1, black dull matt	1	Pc.
	3231-2611-032	Bracket for hip supports, left, size 1, black dull matt	1	Pc.
	3233-2612-032	Bracket for hip supports, right, size 2, black dull matt	1	Pc.
	3233-2611-032	Bracket for hip supports, leftt, size 2, black dull matt	1	Pc.
2-3	3230-2625E-032	Hip support formed part, right, Mini with velcro	1	Pc.
	3230-2624E-032	Hip support formed part, left, Mini with velcro	1	Pc.
	3231-2625E-032	Hip support formed part, right, size 1 with velcro	1	Pc.
	3231-2624E-032	Hip support formed part, left, size 1 with velcro	1	Pc.
	3233-2625E-032	Hip support formed part, right, size 2 with velcro	1	Pc.
	3233-2624E-032	Hip support formed part, left, size 2 with velcro	1	Pc.
	3235-2625E-032	Hip support formed part, right, size 2XL with velcro	1	Pc.
	3235-2624E-032	Hip support formed part, left, size 2XL with velcro	1	Pc.
3	1501-0003	Hook strap black 25mm	0,2	m
4	3231-2622	Locking hinge	2	Pc.
5	1030-0510	Button head socket screw with flange M5x10	4	Pc.
6	1042-0005	Hexagon nut self-locking M5	4	Pc.
7	1026-0525	Mushroom head square neck bolt M5x25	4	Pc.
8	1123-0500	Wing nut PA black M5	4	Pc.
9	1049-0066	BILOC-nut M5	4	Pc.

Spare parts

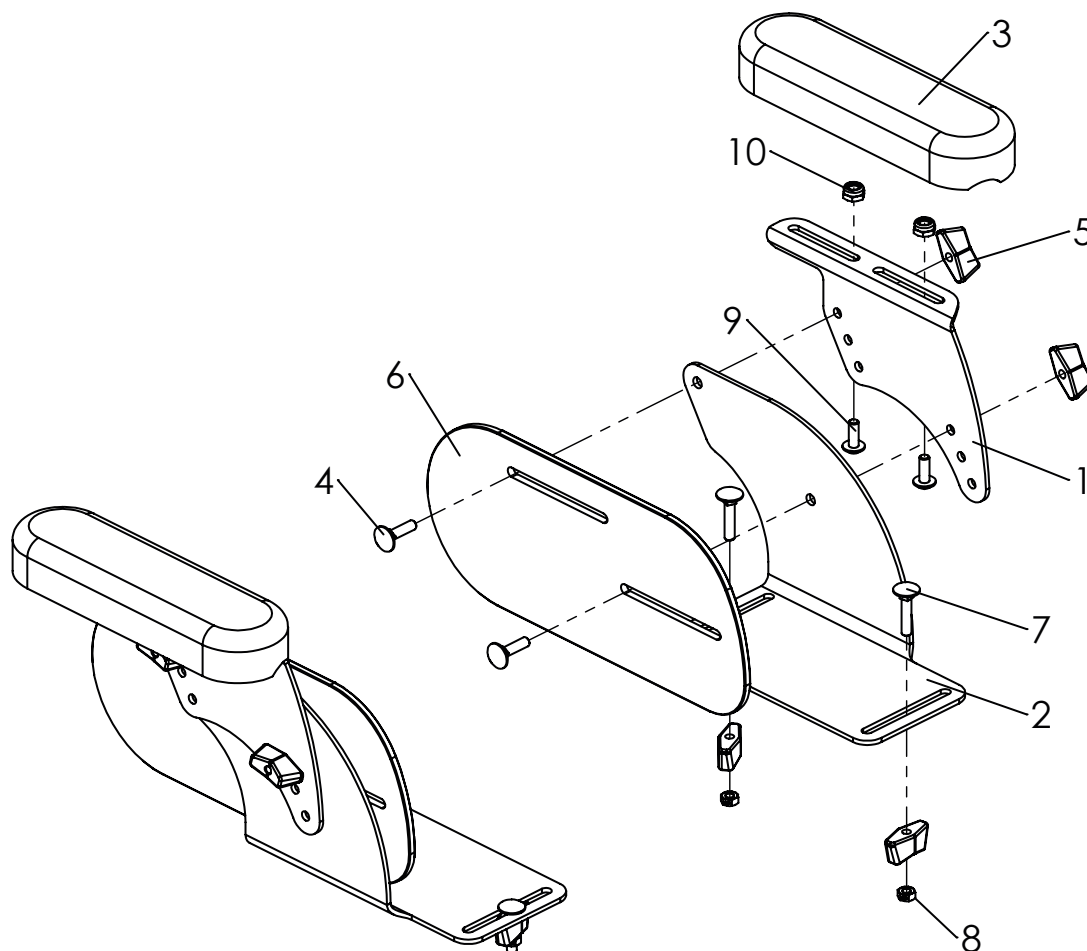


Pos	Article number	Articel description	Quantity	Unit
1-10	3231-3202E	Adaptive hight adjustable arm rests with hip support plate size 1	1	Pc.
	3233-3202E	Adaptive hight adjustable arm rests with hip support plate size 2	1	Pc.
1	3201-3332-032	Armrest support right, black dull matt	1	Pc.
	3201-3232-032	Armrest support left, black dull matt	1	Pc.
2	3201-3312-032	Support right size 1, black dull matt	1	Pc.
	3203-3312-032	Support right size 2, black dull matt	1	Pc.
	3201-3212-032	Support left size 1, black dull matt	1	Pc.
	3203-3212-032	Support left size 2, black dull matt	1	Pc.
3	7000-0023	Armrest pads PU 175 mm	2	Pc.
4	1026-0520	Mushroom head square neck bolt M5x20	4	Pc.
5	1123-0500	Wing nut PA black M5	8	Pc.
6	3201-3220	Hip support plate size 1	2	Pc.
7	1026-0525	Mushroom head square neck bolt M5x25	4	Pc.
8	1049-0066	BILOC-nut VM5	4	Pc.
9	1030-0514	Button head socket screw with flange M5x14	4	Pc.
10	1042-0006	Hexagon nut self-locking M6	4	Pc.

Spare parts

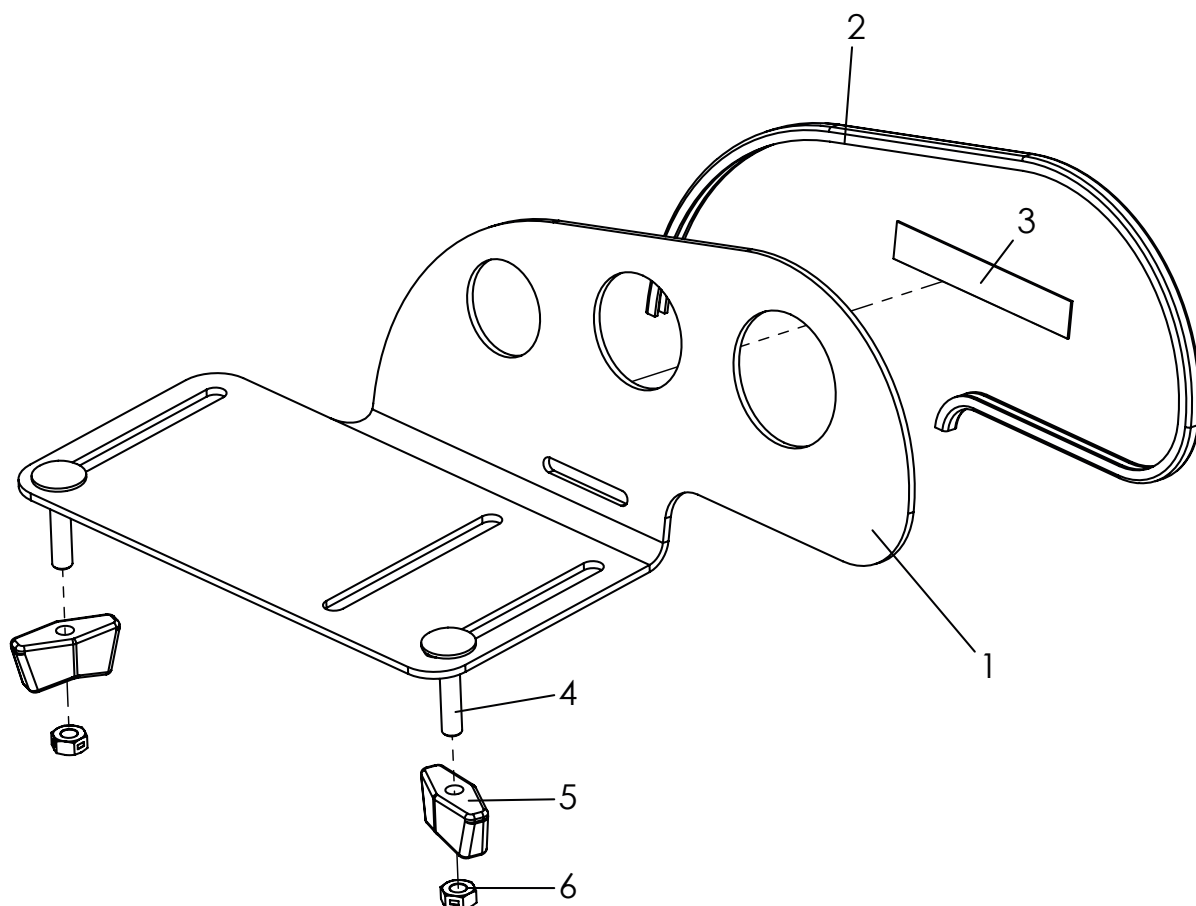


Pos	Article number	Articel description	Quantity	Unit
1-6	3230-2901E	Pelotte thigh support Mini, left, picture	1	Pc.
	3230-2902E	Pelotte thigh support Mini, right	1	Pc.
	3231-2901E	Pelotte thigh support size 1, left	1	Pc.
	3231-2902E	Pelotte thigh support size 1, right	1	Pc.
	3233-2901E	Pelotte thigh support size 2, left, picture	1	Pc.
	3233-2902E	Pelotte thigh support size 2, right	1	Pc.
	3235-2901E	Pelotte thigh support size 2XL, left	1	Pc.
	3235-2902E	Pelotte thigh support size 2XL, right	1	Pc.
	1-3 3230-2911E	Pelotte thigh support Mini, left, picture	1	Pc.
	3230-2912E	Pelotte thigh support Mini, right	1	Pc.
1-3	3231-2911E	Pelotte thigh support size 1, left	1	Pc.
	3231-2912E	Pelotte thigh support size 1, right	1	Pc.
	3233-2911E	Pelotte thigh support size 2, left, picture	1	Pc.
	3233-2912E	Pelotte thigh support size 2, right	1	Pc.
		Pelotte thigh support size 2XL, left	1	Pc.
	3235-2912E	Pelotte thigh support size 2XL, right	1	Pc.
2	1144-0001	KS-Profil PVC, size 1 and mini 0,36m, size 2= 0,43m, size 2XL= 0,52m		mtr.
3	1501-0003	Hook strap black 25mm	0,4	m
		Mushroom head square neck bolt M5x25	2	Pc.
		Wing nut PA black M5	2	Pc.
6	1049-0066	BILOC-nut M5	2	Pc.

Spare parts

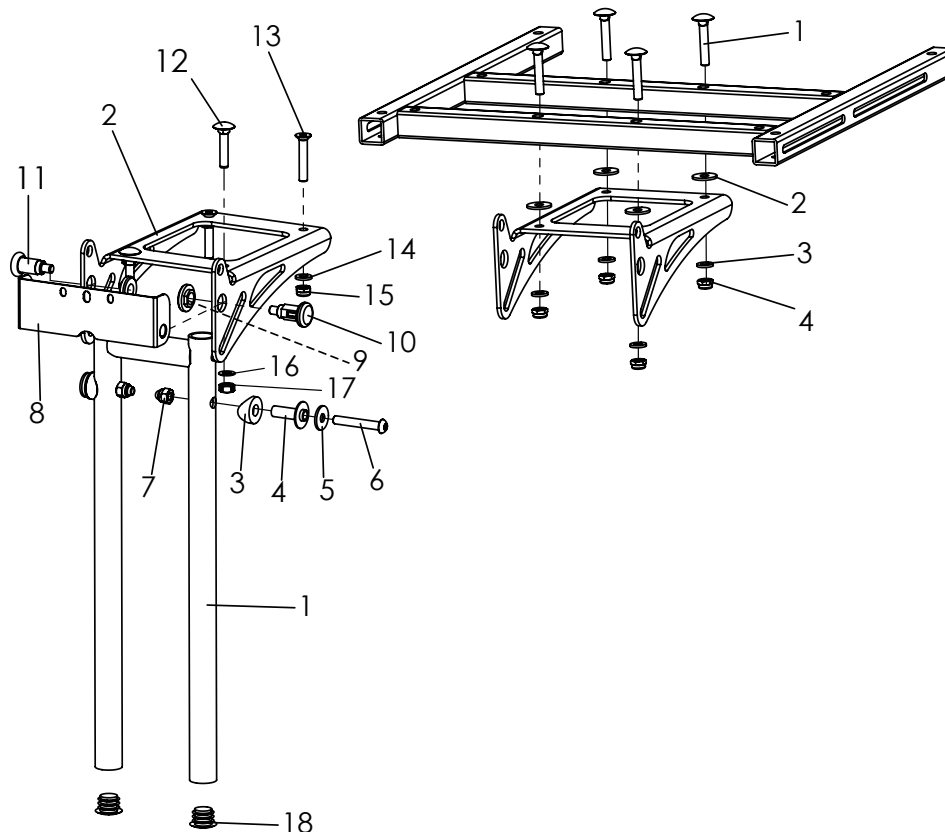

Pos	Article number	Articel description	Quantity	Unit
1-18	3231-2501E-xxx	Foot rest hanger 90° fix size 1	1	Pc.
	3233-2501E-xxx	Foot rest hanger 90° fix size 2	1	Pc.
1	3231-2520-xxx	Bracket for footrest hanger size 1	1	Pc.
	3233-2520-xxx	Bracket for footrest hanger size 2	1	Pc.
2	3231-2411-007	Console for footrest black anodized	1	Pc.
3	1111-0016	Saddle washer PA black 20x8.4x3	2	Pc.
4	3231-2532	Pivot bearing	2	Pc.
5	1055-0006	Washer A6,4	2	Pc.
6	1029-0640	Button head socket screw M6x40	2	Pc.
7	1044-0006	Hexagonal domed cap nut self-locking 986 M6	2	Pc.
8	3231-2545	Fold stop	1	Pc.
9	3231-2544	Nut for lock pin	2	Pc.
10	1127-0011	Lock pin with lock funktion	1	Pc.
11	1127-0010	Lock pin	1	Pc.
12	1026-0635	Mushroom head square neck bolt M6x30	2	Pc.
13	1024-0630	Hexagon socket countersunk head screw M6x35	2	Pc.
14	1051-0006	Washer A6,4	2	Pc.
15	1042-0006	Hexagon nut self-locking M6	2	Pc.
16	1052-0006	Spring washer B6	2	Pc.
17	1049-0068	BILOC-nut M6	2	Pc.
18	1141-0006	Ribbed insert SFL 20x0,8-2,5	2	Pc.
	for size 2 and 2XL	Pos. 12-13-14-15-16-17 must be replaced with 1-2-3-4 See extra explosion for assembly!		
1	1026-0635	Mushroom head square neck bolt M6x35	4	Pc.
2	1055-0006	Washer zp A6,4	4	Pc.
3	1051-0006	Washer A6,4	4	Pc.
4	1042-0006	Hexagon nut self-locking M6	4	Pc.


Spare parts

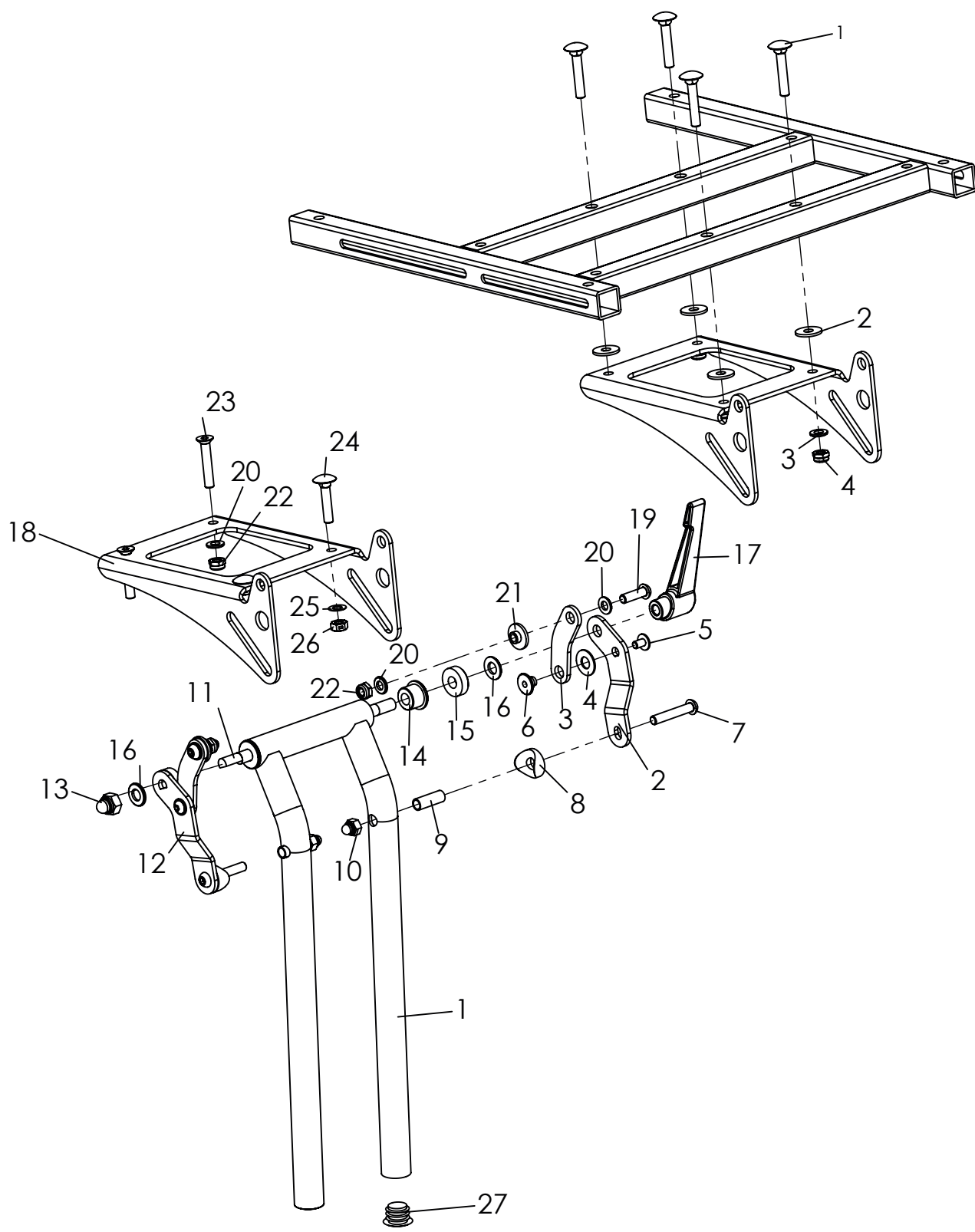

Spare parts


Pos	Article number	Articel description	Quantity	Unit
1-27	3231-2403E-xxx	Foot rest hanger size 1 angle adjustable 85°-170°	1	Pc.
	3233-2403E-xxx	Foot rest hanger size 2 angle adjustable 85°-170°	1	Pc.
1	3240-2870-xxx	Brackets for footrest hanger, angle adjustable size 1	1	Pc.
	3233-2410-xxx	Brackets for footrest hanger, angle adjustable size 2	1	Pc.
2	3231-2434-032	Swing lever, left	1	Pc.
3	3231-2433-032	Fold lever short	2	Pc.
4	1113-0003	Washer PA 18x8,4x1 black	2	Pc.
5	1030-0508	Button head socket screw with flange M5x8	2	Pc.
6	3231-2437	Swing lever axel	2	Pc.
7	1029-0635	Button head socket screw M6x35	2	Pc.
8	1111-0016	Saddle washer PA black 20x8.4x3	2	Pc.
9	1114-0012	Bushing 8x0,8x22,5	2	Pc.
10	1044-0006	Hexagonal domed cap nut self-locking M6	2	Pc.
11	3231-2438-008	Thread rod	1	Pc.
12	3231-2435-032	Swing lever, right	1	Pc.
13	1044-0008	Hexagonal domed cap nut self-locking M8	1	Pc.
14	3240-2864	Clamp bushing	2	Pc.
15	3231-2436	Trust piece	2	Pc.
16	1051-0008	Washer A8,4	2	Pc.
17	1121-0800	Locking lever PA black M8	1	Pc.
18	3231-2411-007	Console for footrest black anodized	1	Pc.
19	1029-0620	Button head socket screw M6x20	2	Pc.
20	1051-0006	Washer A6,4	6	Pc.
21	3210-1238	Shoulder bushing	2	Pc.
22	1042-0006	Hexagon nut self-locking M6	4	Pc.
23	1024-0635	Hexagon socket countersunk head screw M6x35	2	Pc.
24	1026-0630	Mushroom head square neck bolt M6x30	2	Pc.
25	1052-0006	Spring washer B6	2	Pc.
26	1049-0068	BILOC-nut M6	2	Pc.
27	1141-0006	Ribbed insert SFL 20x0,8-2,5	2	Pc.
For size 2 and 2XL		Pos. 20-22-23-24-25-26 must be replaced with 1-2-3-4		
		See extra explosion for assembly!		
1	1026-0635	Mushroom head square neck bolt M6x35	4	Pc.
2	1055-0006	Washer zp A6,4	4	Pc.
3	1051-0006	Washer A6,4	4	Pc.
4	1042-0006	Hexagon nut self-locking M6	4	Pc.

Spare parts

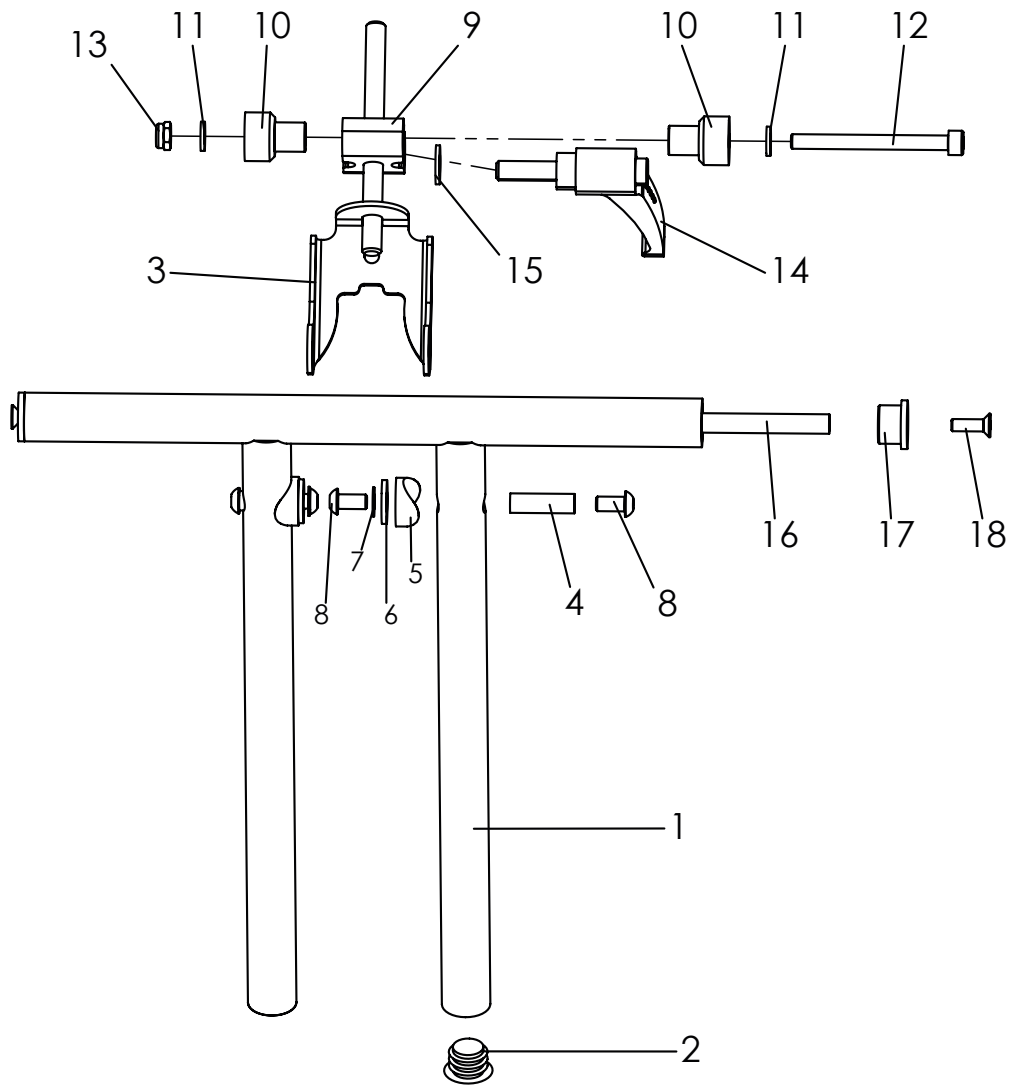

Pos	Article number	Articel description	Quantity	Unit
1-18	3230-2400E-032	Foot rest hanger Mini angle adjustable	1	Pc.
1	3230-2420-032	Brackets for footrest hanger, angle adjustable Mini	1	Pc.
2	1141-0006	Ribbed insert SFL 20x0,8-2,5	2	Pc.
3	3230-2430-033	Suporte retainer for footrest hanger	1	Pc.
4	1114-0062	Threaded bushing 8xM6x26,5	2	Pc.
5	1111-0016	Saddle washer PA black 20x8.4x3	2	Pc.
6	1113-0035	Washer PA 18x8,4x2 black	2	Pc.
7	1052-0006-033	Spring washer B6 black B6	2	Pc.
8	1029-0612-033	Button head socket screw M6x12	4	Pc.
9	3230-2411-007	Clamping block	1	Pc.
10	3230-2413	Axel bushing	2	Pc.
11	1051-0006	Washer A6,4 zp DIN/ISO 125 A6,4	2	Pc.
12	1021-0665	Hexagon socket head cap screw M6x65	1	Pc.
13	1042-0006	Hexagon nut self-locking M6	1	Pc.
14	1121-0825	Locking lever PA black M8x25	1	Pc.
15	1051-0008	Washer A8,4 zp DIN/ISO 125 A8,4	1	Pc.
16	3230-2441-008	Bushing tube with M6 thread	1	Pc.
17	3230-2442	Bushing for Bracket footrest hanger	2	Pc.
18	1024-0516	Hexagon socket countersunk head screw M5x16	2	Pc.


Spare parts

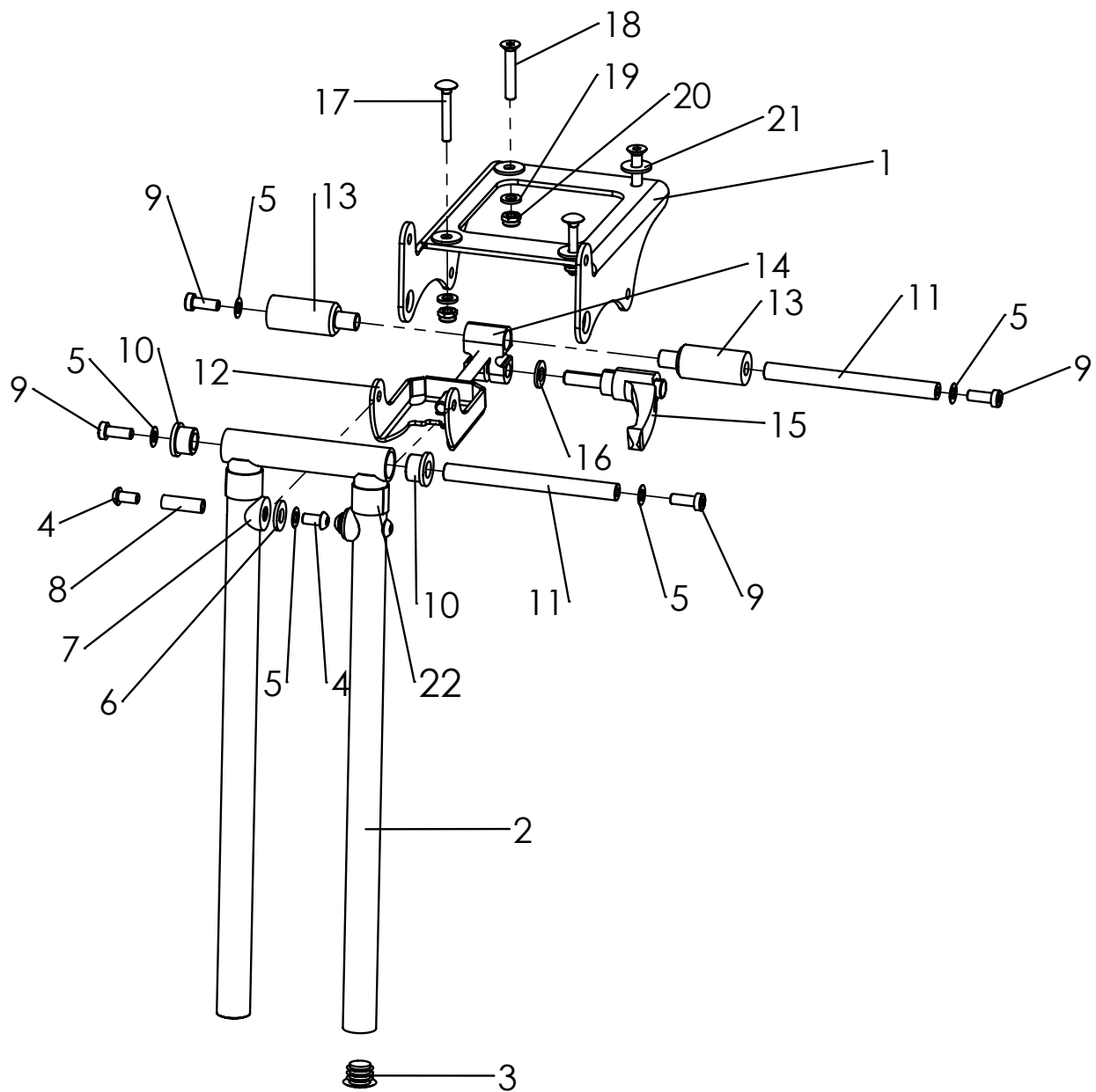

Spare parts


Pos	Article number	Articel description	Quantity	Unit
1-22	3232-2403E-xxx	Foot rest hanger, angle adjust. 85-175°, size 1	1	Pc.
	3234-2403E-xxx	Foot rest hanger, angle adjust. 85-175°, size 2	1	Pc.
1	3232-2412-007	Console for footrest black	1	Pc.
2	3232-2420-xxx	Brackets for footrest hanger, size 1	1	Pc.
	3234-2420-xxx	Brackets for footrest hanger, size 2	1	Pc.
3	1141-0006	Ribbed insert SFL 20x0,8-2,5	2	Pc.
4	1029-0612-033	Button head socket screw M6x12	4	Pc.
5	1052-0006	Spring washer B6	6	Pc.
6	1113-0035	Washer PA 18x8,4x2 black	2	Pc.
7	1111-0016	Saddle washer PA black 20x8.4x3	2	Pc.
8	1114-0064	Threaded bushing 8xM6x25,2	2	Pc.
9	1022-0616	Hexagon socket head cap screw M6x16	4	Pc.
10	3232-2446	Bushing for foot rest hanger	2	Pc.
11	3232-2444	Bushing	2	Pc.
12	3230-2430-030	Suporte retainer for footrest hanger	1	Pc.
13	3232-2413	Axle alves Evo	2	Pc.
14	3230-2411-007	Clamping block	1	Pc.
15	1121-0825	Locking lever PA black M8x25	1	Pc.
16	1051-0008	Washer A8,4	1	Pc.
17	1026-0635	Mushroom head square neck bolt M6x35 (front srew`s)	2	Pc.
18	1024-0635	Hexagon socket countersunk head screw M6x35 (rear screw`s)	2	Pc.
19	1051-0006	Washer A6,4	4	Pc.
20	1042-0006	Hexagon nut self-locking M6	4	Pc.
21	1055-0006	Washer A6,4 (to be instaled between console for foot rest and seat frame)	4	Pc.
22	1500-0010	Loop strap black 16 mm self adhesive A glue	0,1	m
	for size 2 and 2XL	<i>Pos. 17-18-19-20-21 must be replaced with 1-2-3-4 See extra explosion for assembly!</i>		
1	1026-0635	Mushroom head square neck bolt M6x35	4	Pc.
2	1055-0006	Washer zp A6,4	4	Pc.
3	1051-0006	Washer A6,4	4	Pc.
4	1042-0006	Hexagon nut self-locking M6	4	Pc.

Spare parts

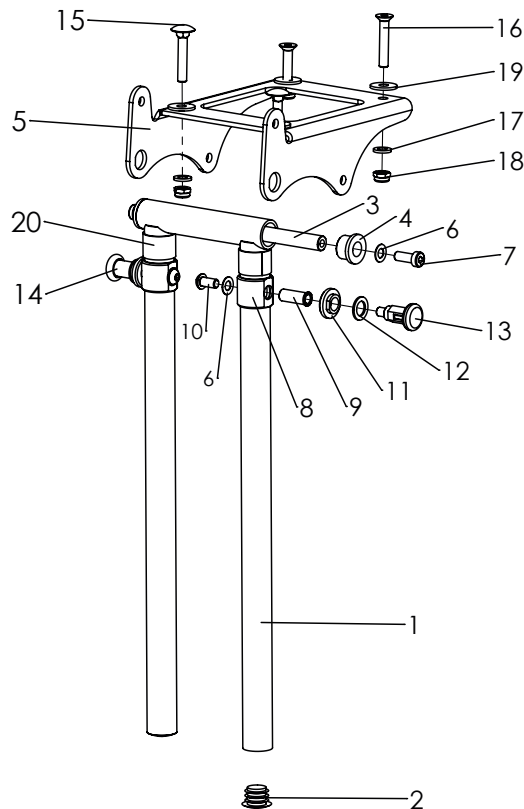

Pos	Article number	Articel description	Quantity	Unit
1-20	3232-2501E-xxx	Foot rest hanger size 1, 90° fix	1	Pc.
	3234-2501E-xxx	Foot rest hanger size 2, 90° fix	1	Pc.
1	3232-2420-xxx	Bracket for footrest hanger size 1	1	Pc.
	3234-2420-xxx	Bracket for footrest hanger size 2	1	Pc.
2	1141-0006	Ribbed insert 20x0,8-2,5	2	Pc.
3	3232-2444	Axle	1	Pc.
4	3232-2446	Bushing for foot rest hanger	2	Pc.
5	3234-2412-032	Console for foot rest black anodized	1	Pc.
6	1052-0006	Spring washer B6	4	Pc.
7	1022-0616	Hexagon socket head cap screw M6x16	2	Pc.
8	3232-2445	Slide ring Evo	2	Pc.
9	3232-2443	Bolt catch	2	Pc.
10	1029-0612	Button head socket screw M6x12	2	Pc.
11	3231-2544	Nut for lock pin	2	Pc.
12	1059-0001	Shim ring 10x16x1,2	2	Pc.
13	1127-0011	Lock pin M10x1-KS with lock	1	Pc.
14	1127-0010	Lock pin M-10x1	1	Pc.
15	1026-0635	Mushroom head square neck bolt M6x35 (front srew`s)	2	Pc.
16	1024-0635	Hexagon socket countersunk head screw M6x35 (rear screw`s)	2	Pc.
17	1051-0006	Washer A6,4	4	Pc.
18	1042-0006	Hexagon nut self-locking M6	4	Pc.
19	1055-0006	Washer A6,4 (to be instaled between console for foot rest and seat frame)	4	Pc.
20	1500-0010	Loop strap black 16 mm self adhesive A glue	0,1	m
for size 2 and 2XL				
Pos. 12-13-14-15-16-17 must be replaced with 1-2-3-4				
See extra explosion for assembly!				
1	1026-0635	Mushroom head square neck bolt M6x35	4	Pc.
2	1055-0006	Washer zp A6,4	4	Pc.
3	1051-0006	Washer A6,4	4	Pc.
4	1042-0006	Hexagon nut self-locking M6	4	Pc.

Spare parts

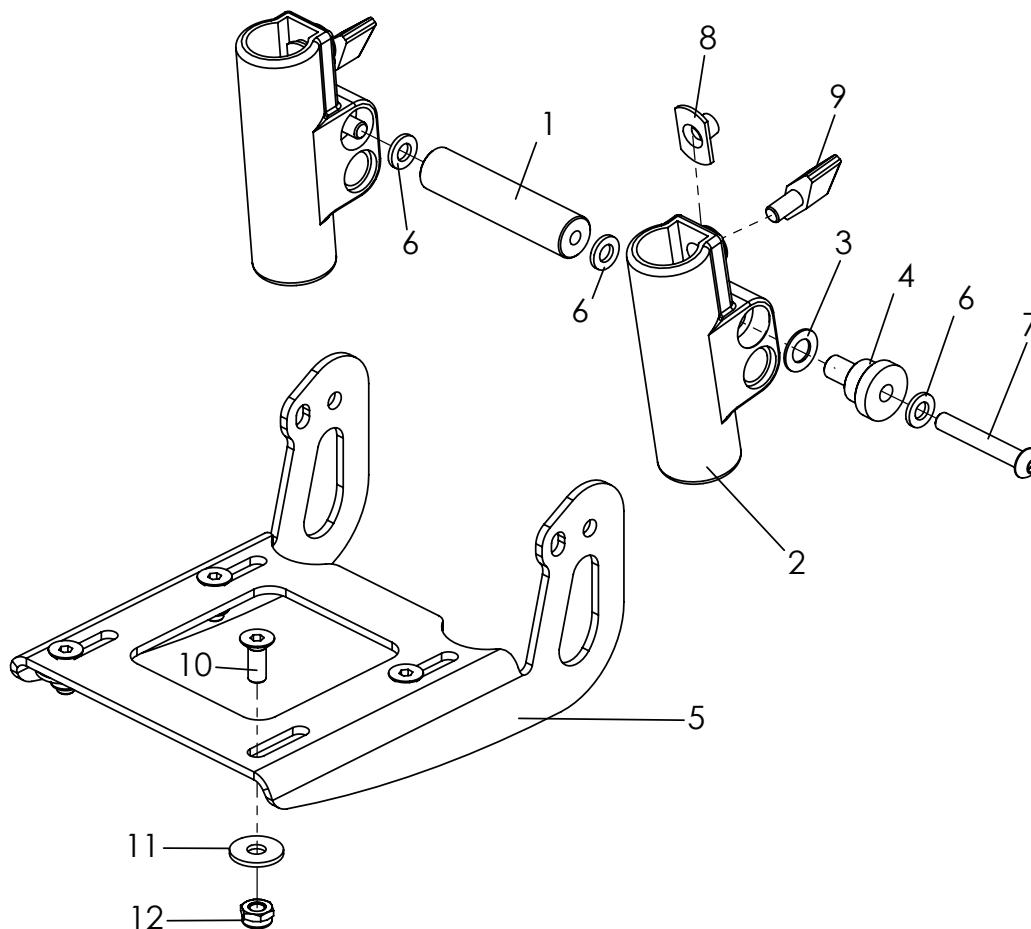

Pos	Article number	Articel description	Quantity	Unit
1-12	3240-2901E-032	Foot board assembly angle adjustable and foldable	1	Pc.
	3240-2902E-032	Foot board assembly reinforced angle adjustable and foldable	1	Pc.
1	3240-2913	Stabilizer	1	Pc.
2	3201-2520	Footboard holder	2	Pc.
3	1052-0008	Spring washer B8	2	Pc.
4	3201-2530	Bushing	2	Pc.
5	3240-2911-032	Foot rest bracket black	1	Pc.
	3240-2912-032	Foot rest bracket steel, black dull matt	1	Pc.
6	1051-0006	Washer A6,4	4	Pc.
7	1029-0640	Button head socket screw M6x40	2	Pc.
8	1049-0052	T-nut L-769 M6x8,5	2	Pc.
9	1123-0612	Wing screw Pa black M6x12 M6x BPF 20	2	Pc.
10	1024-0616-033	Hexagon socket countersunk head screw black M6x16	4	Pc.
11	1055-0006	Washer A 6,4	4	Pc.
12	1042-0006	Hexagon nut self-locking M6	4	Pc.

This exploded view diagram illustrates the assembly of a mechanical device. The components are numbered as follows:


- 1**: A vertical cylindrical housing or tube.
- 2**: A small cylindrical component, possibly a pin or a small shaft.
- 3**: A small cylindrical component, possibly a pin or a small shaft.
- 4**: A circular plate with four holes, likely a base or a cover.
- 5**: A small cylindrical component, possibly a pin or a small shaft.
- 6**: A small cylindrical component, possibly a pin or a small shaft.
- 7**: A small cylindrical component, possibly a pin or a small shaft.
- 8**: A small cylindrical component, possibly a pin or a small shaft.

The diagram shows the following assembly sequence and relationships:

- Part **4** is the base component.
- Part **3** is inserted into part **4**.
- Part **2** is inserted into part **3**.
- Part **1** is inserted into part **2**.
- Part **5** is inserted into part **1**.
- Part **6** is inserted into part **1**.
- Part **7** is inserted into part **6**.
- Part **8** is inserted into part **5**.






69

Spare parts

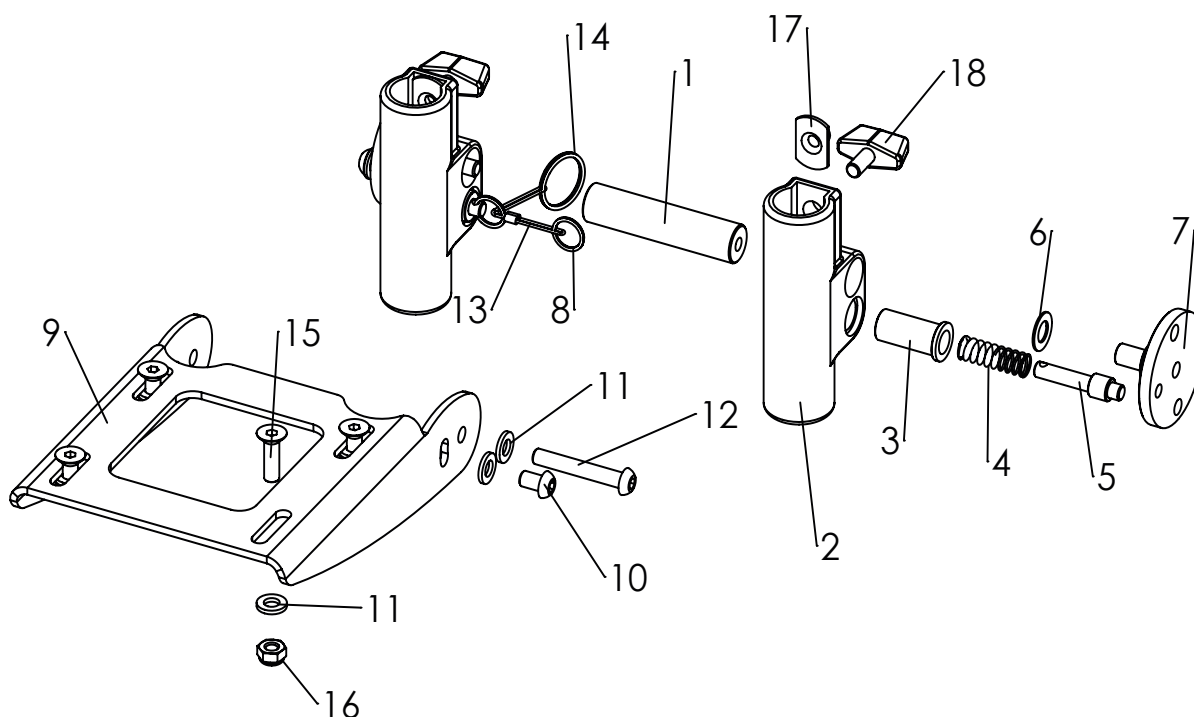


Pos	Article number	Articel description	Quantity	Unit
1-18	3240-2903E-032	Foot board assembly angle adjustable and foldable	1	Pc.
1	3240-2913	Stabilizer	1	Pc.
2	3201-2520	Footboard holder	2	Pc.
3	3201-7520	Spring housing	2	Pc.
4	1131-0036	Compression spring D-127d 0,8x8,3x29	2	Pc.
5	3201-7530	Lock pin	2	Pc.
6	1052-0008	Spring washer B8	2	Pc.
7	3201-7510	Lock plate	2	Pc.
8	1129-0036	Key ring 16x1,6	2	Pc.
9	3240-2914-032	Foot rest bracket black dull matt	1	Pc.
10	1030-0608	Button head socket screw with flange zp M6x8	2	Pc.
11	1051-0006	Washer A6,4	6	Pc.
12	1029-0640	Button head socket screw M6x40	2	Pc.
13	3201-7560	Wire rope for foot rest lock, EVO	1	Pc.
14	1129-0003	Key ring 24x1,5	1	Pc.
15	1024-0616	Hexagon socket countersunk head screw M6x16	4	Pc.
16	1042-0006	Hexagon nut self-locking M6	4	Pc.
17	1049-0052	T-nut	2	Pc.
18	1123-0612	Wing screw Pa M6x12	2	Pc.

Spare parts

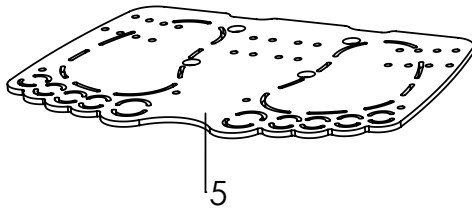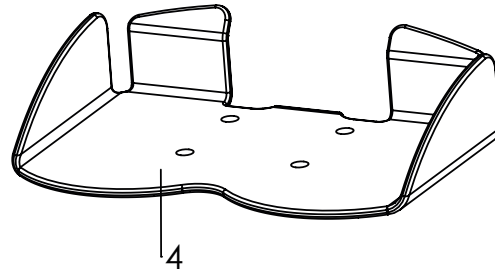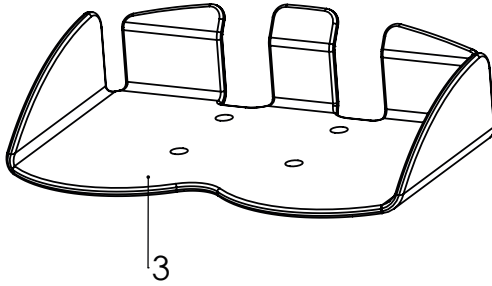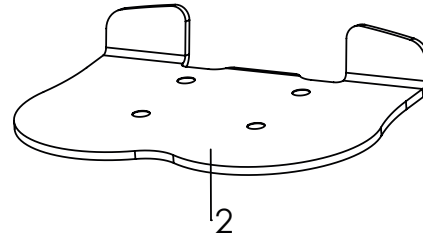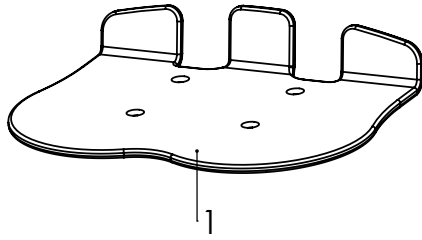

Pos	Article number	Articel description	Quantity	Unit
1	3231-2491E	Evo foot rest with heel stop, width 27cm	1	Pc.
	3231-2492E	Evo foot rest with heel stop, width 29cm	1	Pc.
	3231-2493E	Evo foot rest with heel stop, width 31cm	1	Pc.
	3231-2494E	Evo foot rest with heel stop, width 33cm	1	Pc.
	3231-2495E	Evo foot rest with heel stop, width 35cm	1	Pc.
2	3231-2481E	Evo foot rest with heel stop for short LLL, width 27 cm	1	Pc.
	3231-2482E	Evo foot rest with heel stop for short LLL, width 29 cm	1	Pc.
	3231-2483E	Evo foot rest with heel stop for short LLL, width 31 cm	1	Pc.
	3231-2484E	Evo foot rest with heel stop for short LLL, width 33 cm	1	Pc.
	3231-2485E	Evo foot rest with heel stop for short LLL, width 35 cm	1	Pc.
3	3231-2551E	Bingo-Evo foot plate in boat form size 1, width 30cm	1	Pc.
	3231-2552E	Bingo-Evo alu foot plate in boat form, FS black, size 1, width 30cm	1	Pc.
	3233-2551E	Bingo-Evo foot plate in boat form size 2, width 35cm	1	Pc.
	3233-2552E	Bingo-Evo alu foot plate in boat form, FS black, size 2, width 35cm	1	Pc.
4	3231-2553E	Bingo-Evo foot plate in boat form size 1, for short LLL	1	Pc.
	3233-2553E	Bingo-Evo foot plate in boat form size 2, for short LLL	1	Pc.
5	3231-2463E-015	Evo alu-footplate "footprint"with heel stop, width 30cm	1	Pc.
	3231-2465E-015	Evo alu-footplate "footprint"with heel stop, width 34cm	1	Pc.


Spare parts

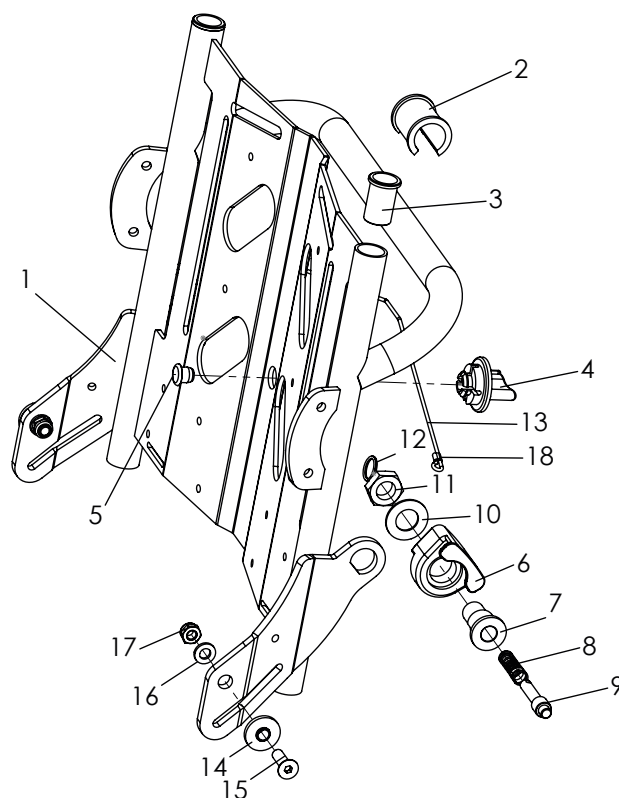

Pos	Article number	Articel description	Quantity	Unit
1-18	3231-2300E-xxx	Back base size 1 complete	1	Pc.
	3233-2300E-xxx	Back base size 2 complete	1	Pc.
	3235-2300E-xxx	Back base size 2XL complete	1	Pc.
1	3231-2310-xxx	Back base size 1	1	Pc.
	3233-2310-xxx	Back base size 2	1	Pc.
	3235-2310-xxx	Back base size 2XL	1	Pc.
2	3231-2324	Balance bushing	1	Pc.
3	3231-2323	Guiding bush	2	Pc.
4	3240-2441	Turning knob	1	Pc.
5	1141-0028	Decorative plug	1	Pc.
6	3231-2322	Back guidance	2	Pc.
7	3231-2321	Locking bolt housing	2	Pc.
8	1131-0036	Compression spring D-127d 0,8x8,3x29	2	Pc.
9	3201-7530	Lock pin	2	Pc.
10	1052-0012	Spring washer B12	2	Pc.
11	1040-0012	Hexagon nut M12(fine pitch thread)	2	Pc.
12	1129-0036	Key ring 16x1,6	2	Pc.
13	3231-2325	Release cable for backrest recline size 1	1	Pc.
	3233-2325	Release cable for back angle adjustment size 2 and 2XL	1	Pc.
14	3231-2131	Bushing	2	Pc.
15	1024-0620-033	Hexagon socket countersunk head screw black M6x20	2	Pc.
16	1051-0006	Washer A6,4	2	Pc.
17	1042-0006	Hexagon nut self-locking M6	2	Pc.
18	1089-0005	Cable clamp	2	Pc.
19	1089-0007	Cable tie 3,3x145	1	Pc.
20	1119-0003	Wire clamp	1	Pc.
21	1062-0410	Blind rivet F4x10	1	Pc.

19-21 are not shown, are supplied loose.


Spare parts

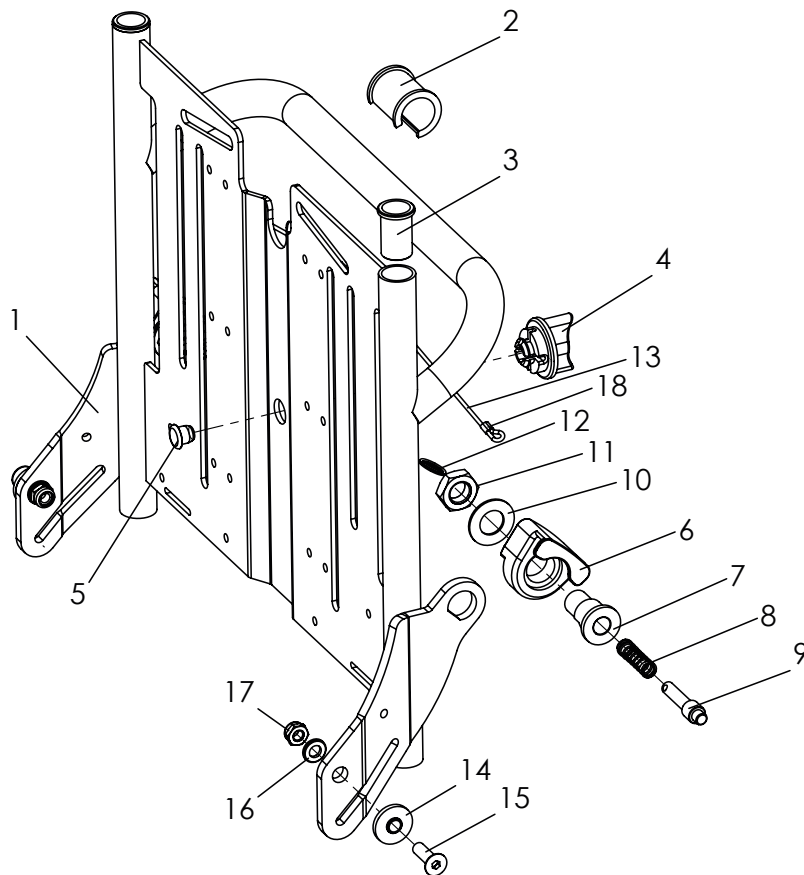

Pos	Article number	Articel description	Quantity	Unit
1-18	3231-3300E-xxx	Back base OT size 1 complete	1	Pc.
	3233-3300E-xxx	Back base OT size 2 complete	1	Pc.
1	3231-3310-xxx	Back base size 1	1	Pc.
	3233-3310-xxx	Back base size 2	1	Pc.
2	3231-2324	Balance bushing	1	Pc.
3	3231-2323	Guiding bush	2	Pc.
4	3240-2441	Turning knob	1	Pc.
5	1141-0028	Decorative plug	1	Pc.
6	3231-2322	Back guidance	2	Pc.
7	3231-2321	Locking bolt housing	2	Pc.
8	1131-0036	Compression spring D-127d 0,8x8,3x29	2	Pc.
9	3201-7530	Lock pin	2	Pc.
10	1052-0012	Spring washer B12	2	Pc.
11	1040-0012	Hexagon nut M12(fine pitch thread)	2	Pc.
12	1129-0036	Key ring 16x1,6	2	Pc.
13	3231-2325	Release cable for backrest recline size 1	1	Pc.
	3233-2325	Release cable for back angle adjustment size 2	1	Pc.
14	3231-2131	Bushing	2	Pc.
15	1024-0620-033	Hexagon socket countersunk head screw black M6x20	2	Pc.
16	1051-0006	Washer A6,4	2	Pc.
17	1042-0006	Hexagon nut self-locking M6	2	Pc.
18	1089-0005	Cable clamp	2	Pc.
19	1089-0007	Cable ties	1	Pc.
20	1119-0003	Wire clamp	1	Pc.
21	1062-0410	Blind rivet F4x10	1	Pc.


19-21 are not shown, are supplied loose.

Spare parts

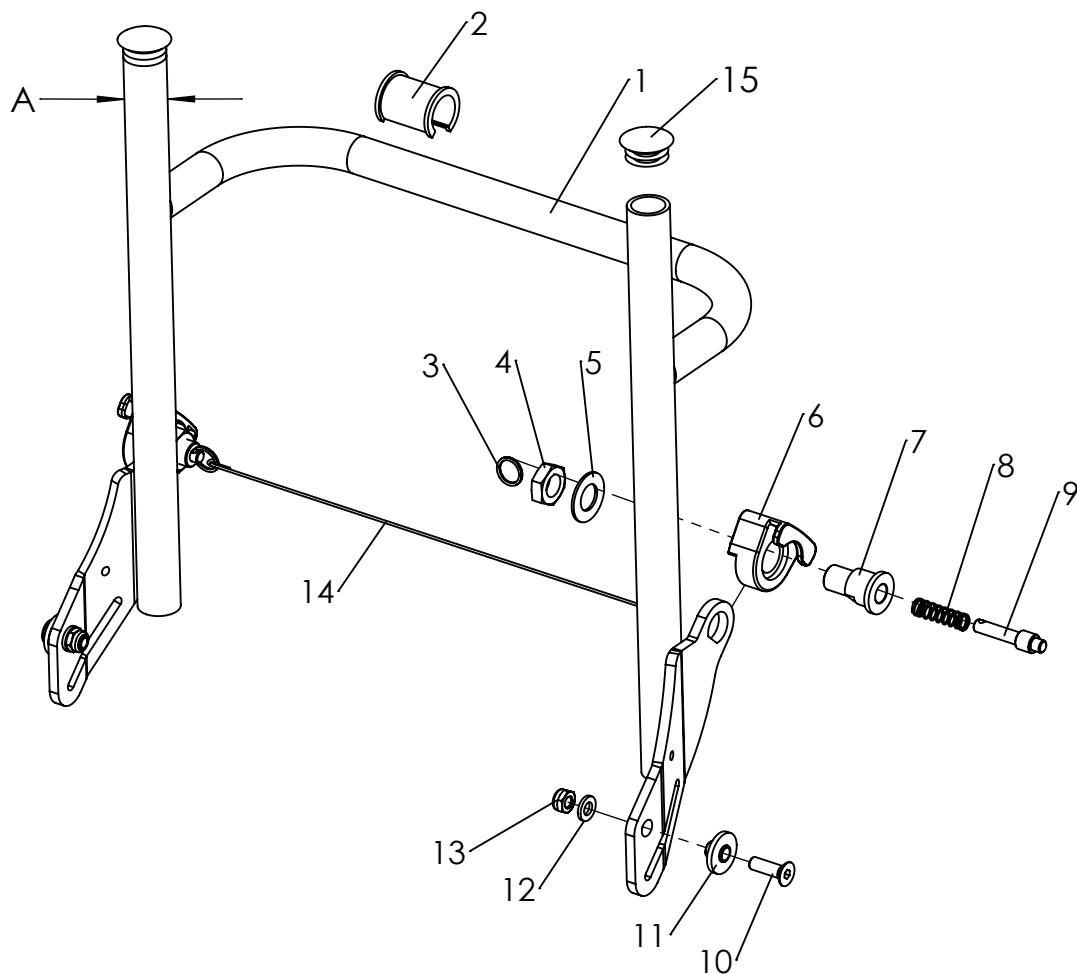


Pos	Article number	Articel description	Quantity	Unit
1-15	3231-3600E	Back base OT Plus size 1, A = 25mm outer diameter	1	Pc.
	3233-3600E	Back base OT Plus size 2	1	Pc.
	3231-3400E-xxx	Back base OT professionnel size 1 A = 18mm outer diameter	1	Pc.
	3233-3400E-xxx	Back base OT professionnel size 2	1	Pc.
1	3231-3610-xxx	Back base OT Plus, size 1	1	Pc.
	3233-3610-xxx	Back base OT Plus, size 2	1	Pc.
	3231-3410-xxx	Back base OT professionnel size 1	1	Pc.
	3233-3410-xxx	Back base OT professionnel size 2	1	Pc.
2	3231-2324	Balance bushing	1	Pc.
3	1129-0036	Key ring 16x1,6	2	Pc.
4	1040-0012	Hexagon nut M12 (fine pitch thread)	2	Pc.
5	1052-0012	Spring washer B12	2	Pc.
6	3231-2322	Back guidance	2	Pc.
7	3231-2321	Locking bolt housing	2	Pc.
8	1131-0036	Compression spring D-127	2	Pc.
9	3201-7530	Lock pin	2	Pc.
10	1024-0620	Hexagon socket countersunk head screw M6x20	2	Pc.
11	3231-2131	Bushing	2	Pc.
12	1051-0006	Washer A6,4	2	Pc.
13	1042-0006	Hexagon nut self-locking M6	2	Pc.
14	3231-3699	Release cable for backrest size 1	1	Pc.
	3233-3699	Release cable for backrest size 2	1	Pc.
15	1141-0009	Ribbed insert SFL 25x1-2,5	2	Pc.

Spare parts

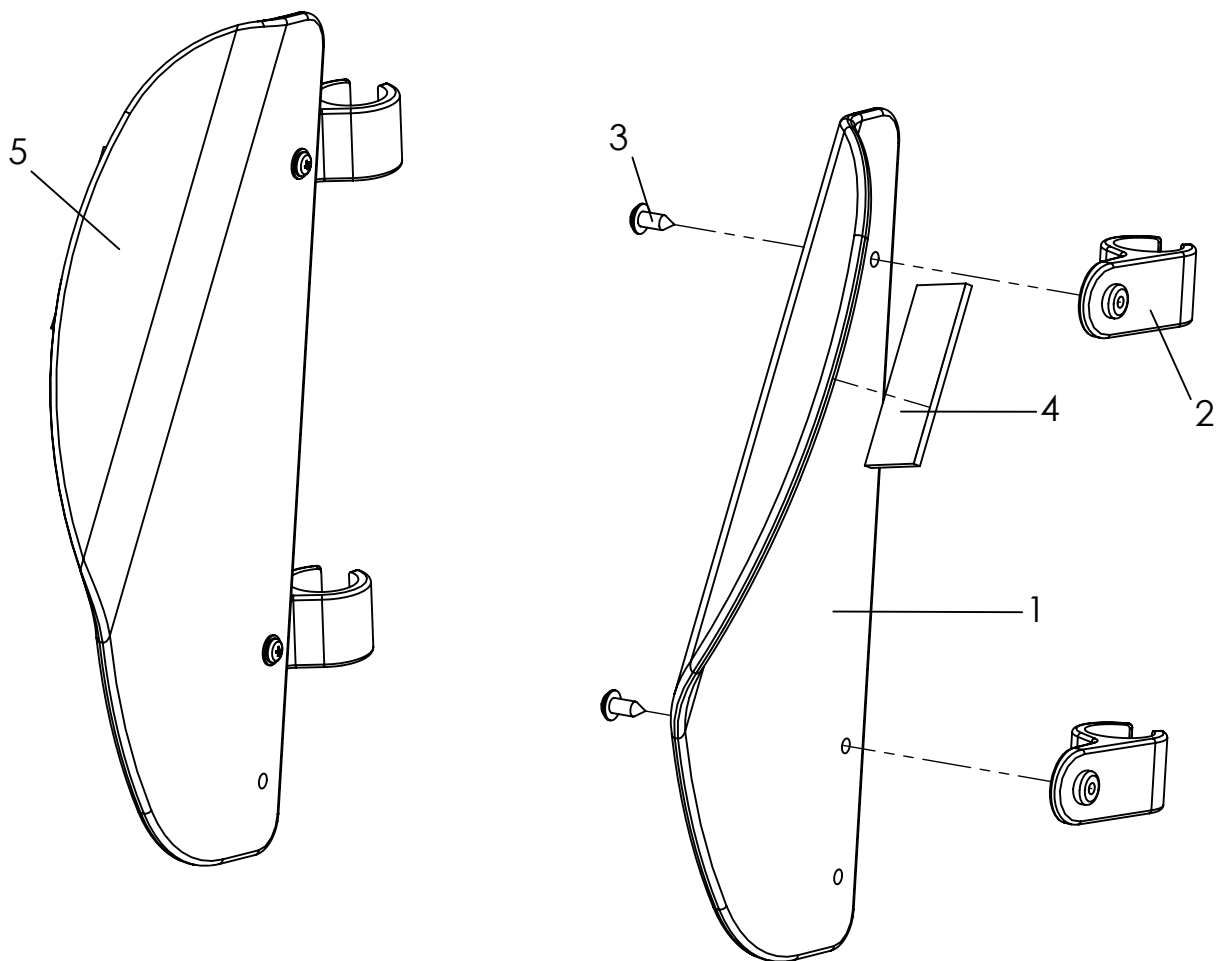


Pos	Article number	Articel description	Quantity	Unit
1-4	3231-2710E	Lateral support left size 1 complete	1	Pc.
	3233-2710E	Lateral support left size 2 complete	1	Pc.
1	3201-2711	Lateral support formed part left size 1	1	Pc.
	3203-2711	Lateral support formed part left size 2	1	Pc.
2-5	3231-2720E	Lateral support right size 1 complete	1	Pc.
	3233-2720E	Lateral support right size 2 complete	1	Pc.
2	3231-2712	Pad hinge	2	Pc.
3	1039-0001	Screw for plastic, cross pan head w. flange zp 4x10	2	Pc.
4	1501-0003	Hook strap black 25mm self adhesive glue	0,1	Pc.
5	3201-2721	Lateral support formed part right size 1	1	Pc.
	3203-2721	Lateral support formed part right size 2	1	Pc.

Spare parts

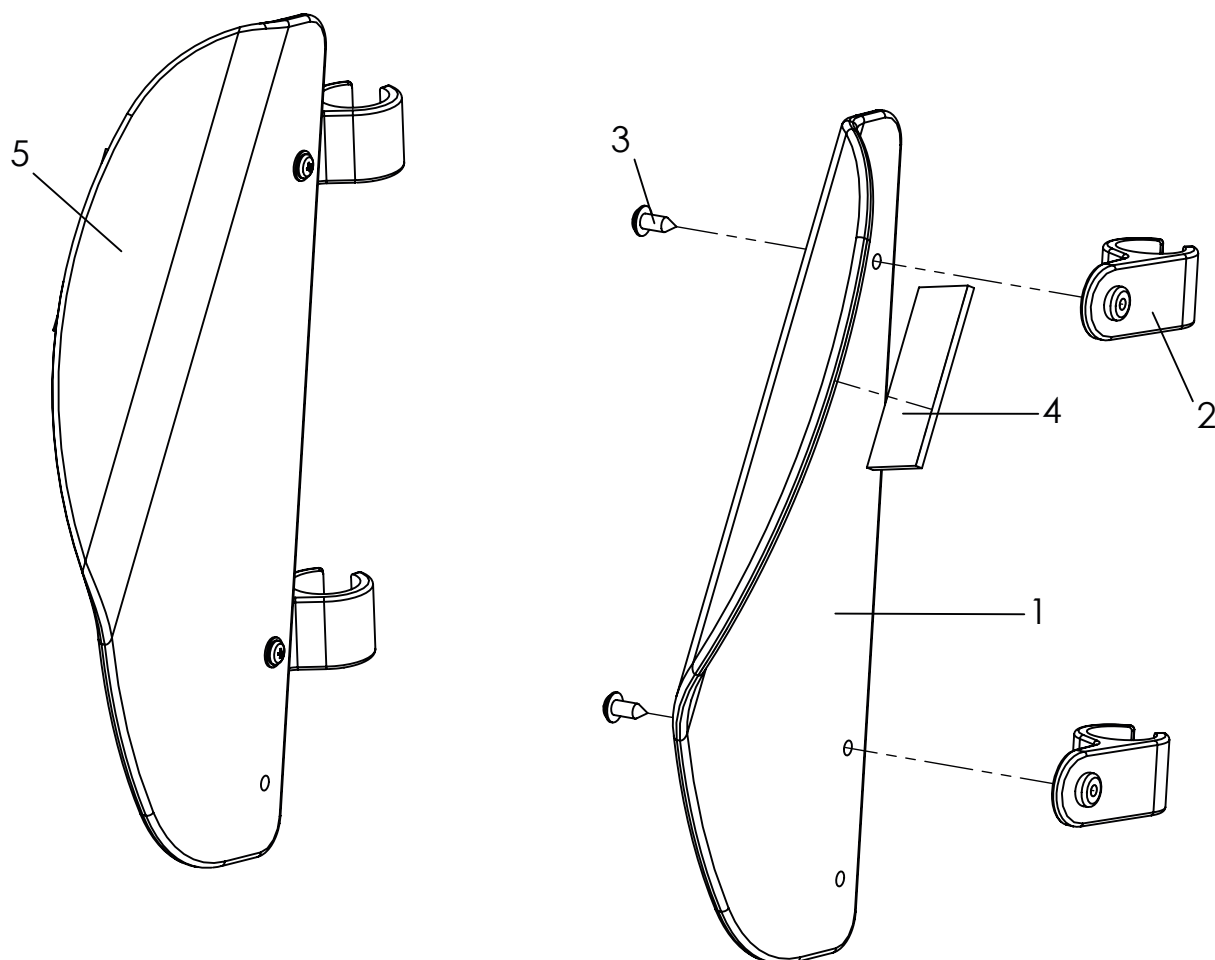


Pos	Article number	Articel description	Quantity	Unit
1-4	3231-2730E	Lateral support left size 1 complete, in alloy	1	Pc.
	3233-2730E	Lateral support left size 2 complete, in alloy	1	Pc.
	3235-2710E	Lateral support left size 2XL complete, in alloy	1	Pc.
1	3201-2714-032	Lateral support formed part left size 1, in alloy	1	Pc.
	3203-2714-032	Lateral support formed part left size 2,, in alloy	1	Pc.
	3235-2711-032*	Lateral support formed part left size 2XL, in alloy	1	Pc.
2-5	3231-2740E	Lateral support right size 1 complete, in alloy	1	Pc.
	3233-2740E	Lateral support right size 2 complete, in alloy	1	Pc.
	3235-2720E	Lateral support right size 2XL complete, in alloy	1	Pc.
2	3231-2712	Pad hinge	2	Pc.
3	1039-0003	Screw for plastic, pan head with flange 4x10	2	Pc.
4	1501-0003	Hook strap black 25mm self adhesive glue	0,1	mtr.
5	3201-2724-032	Lateral support formed part right size 1,, in alloy	1	Pc.
	3203-2724-032	Lateral support formed part right size 2, in alloy	1	Pc.
		Lateral support formed part right size 2XL, in alloy	1	Pc.
	*1039-0001	when 2XL is needed, Screw 4x10, Pos.3	3	Pc.
	*3231-2712	when 2XL is needed ,Pad hinge, Pos. 2	3	Pc.

Spare parts

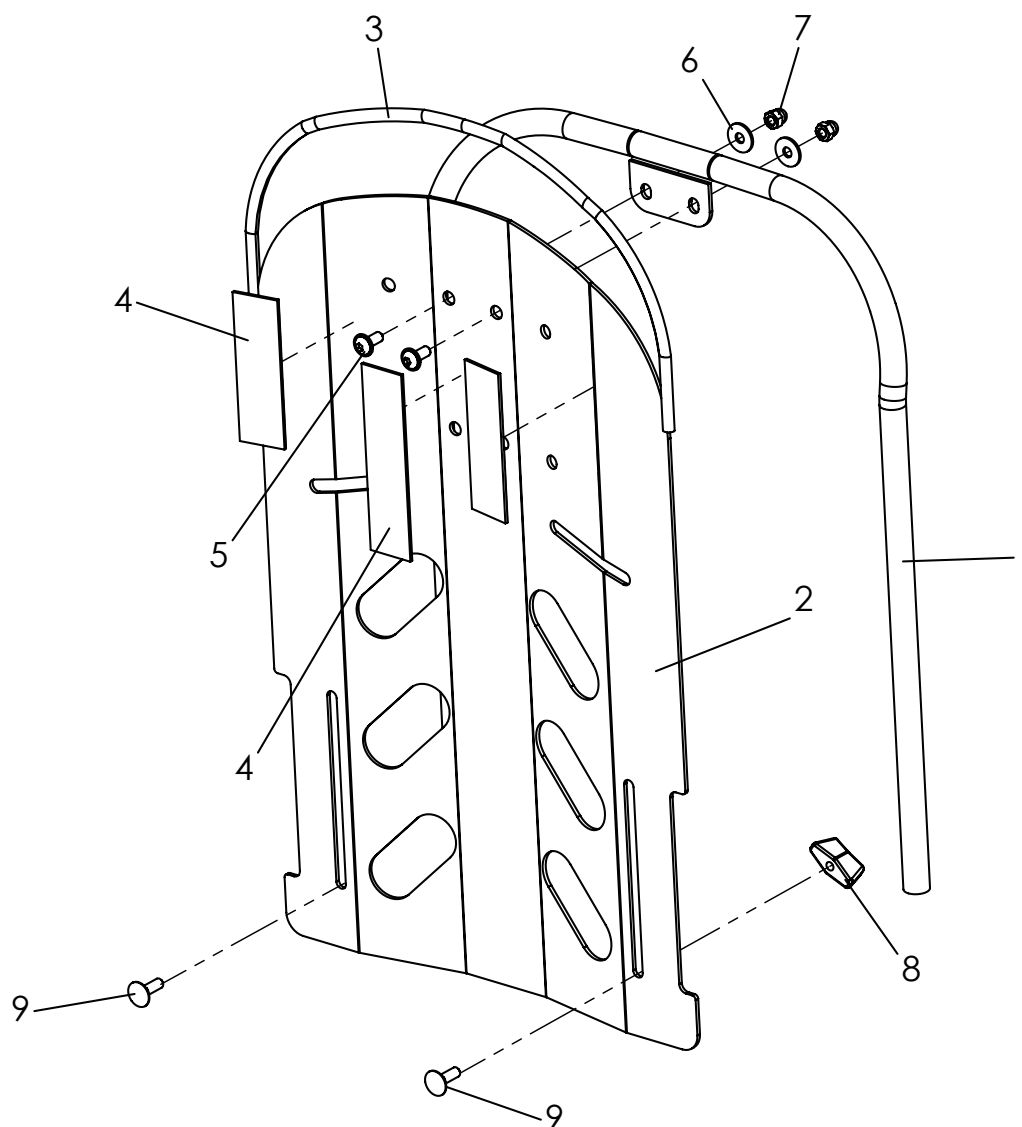


Pos	Article number	Articel description	Quantity	Unit
1-9	3201-2800E-xxx	Back insert size 1, complete	1	Pc.
	3203-2800E-xxx	Back insert size 2 and 2XL, complete	1	Pc.
1	3201-2810-xxx	Back insert size 1	1	Pc.
	3203-2810-xxx	Back insert size 2 and 2XL	1	Pc.
2-5	3201-2822E-xxx	Back panel size 1, with conectors	1	Pc.
	3203-2822E-xxx	Back panel size 2, with conectors	1	Pc.
2	3201-2822-xxx	Back panel size 1	1	Pc.
	3203-2822-xxx	Back panel size 2	1	Pc.
3	1144-0001	KS-Profil PVC	0,5	m.
4	1501-0003	Hook strap self adhesive (2x 0,05m, 1x 0,1m)	0,2	m.
5	1030-0512	Button head socket screw with flange M5x12	2	Pc.
6	1055-0005	Washer A5,3	2	Pc.
7	1044-0005	Hexagonal domed cap nut self-locking M5	2	Pc.
8	1123-0500	Wing nut PA black M5 M5 BPF 20	2	Pc.
9	1026-0516-033	Mushroom head square neck bolt black M5x16	2	Pc.

Spare parts

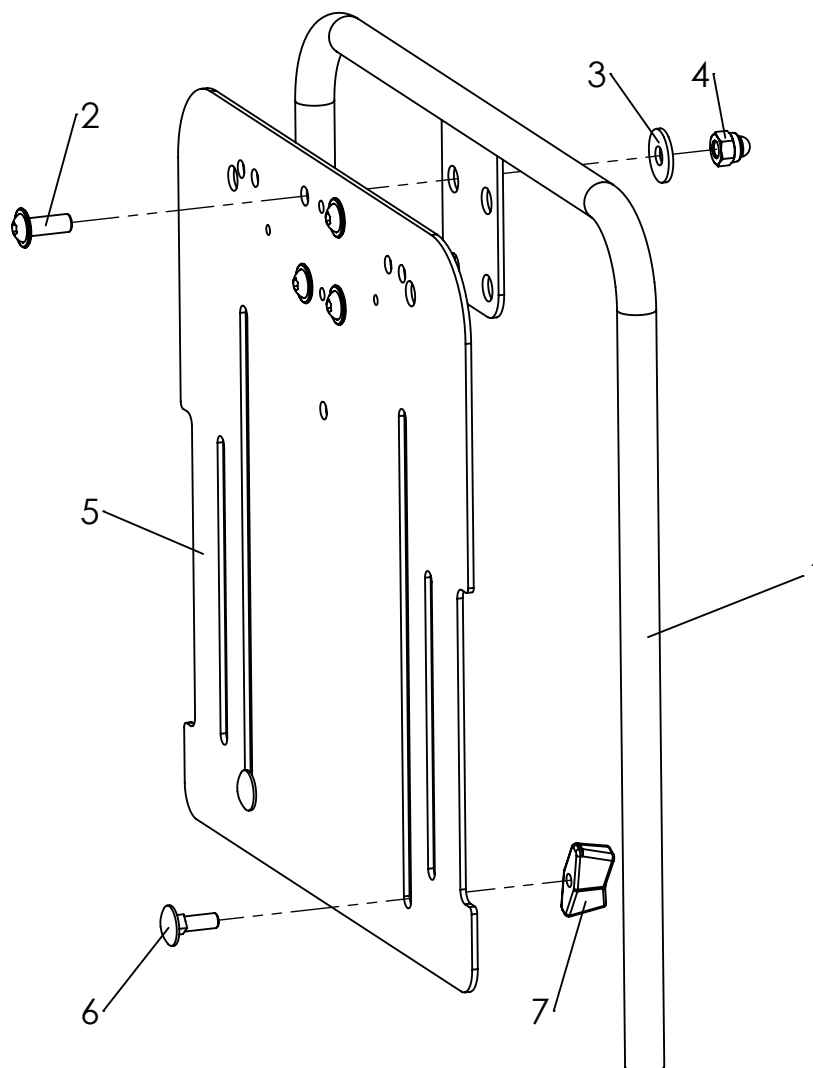


Pos	Article number	Articel description	Quantity	Unit
1-7	3201-3800E-xxx	OT Back insert size 1	1	Pc.
	3203-3800E-xxx	OT Back insert size 2	1	Pc.
1	3201-3810-xxx	OT Back frame size 1	1	Pc.
	3203-3810-xxx	OT Back frame size 2	1	Pc.
2	1030-0616	Button head socket screw with flange M6x16	4	Pc.
3	1055-0006	Washer A6,4	4	Pc.
4	1044-0006	Hexagonal domed cap nut self-locking M6	4	Pc.
5-7	3201-3822E-xxx	OT Back panel size 1 with fasteners	1	Pc.
	3203-3822E-xxx	OT Back panel size 2 with fasteners	1	Pc.
6	1026-0516-033	Mushroom head square neck bolt black M5x16	2	Pc.
7	1123-0500	Wing nut PA black M5	2	Pc.

Spare parts

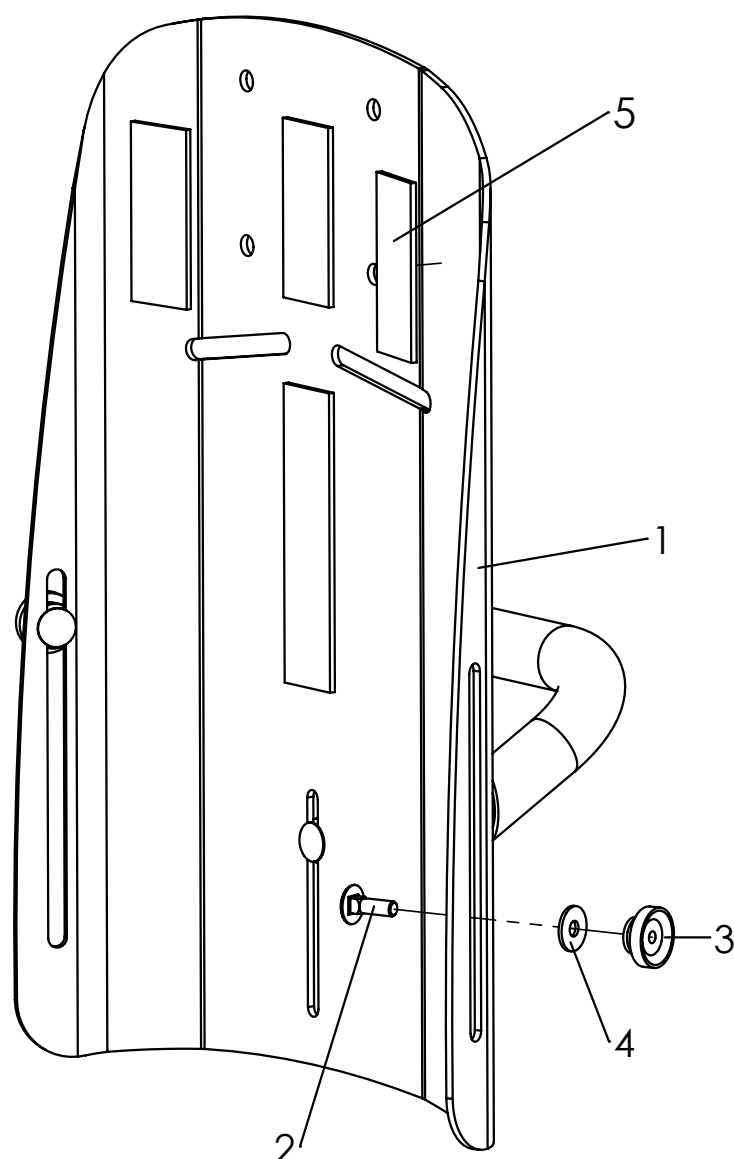

Pos	Article number	Articel description	Quantity	Unit
1-5	3230-2302E-032	Back insert Mini, complete	1	Pc.
1	3230-2331-032	Back insert Mini,	1	Pc.
2	1026-0516-033	Mushroom head square neck bolt black M5x16	3	Pc.
3	1124-0500	Knurled nut PA black M5	3	Pc.
4	1055-0005-033	Washer A5,3	3	Pc.
5	1501-0003	Hook strap self adhesive, 25mm, 6 cm long, 3 pc.	0,18	m
	1501-0003	Hook strap self adhesive, 25mm, 10 cm long, 1 pc.	0,12	m

Spare parts

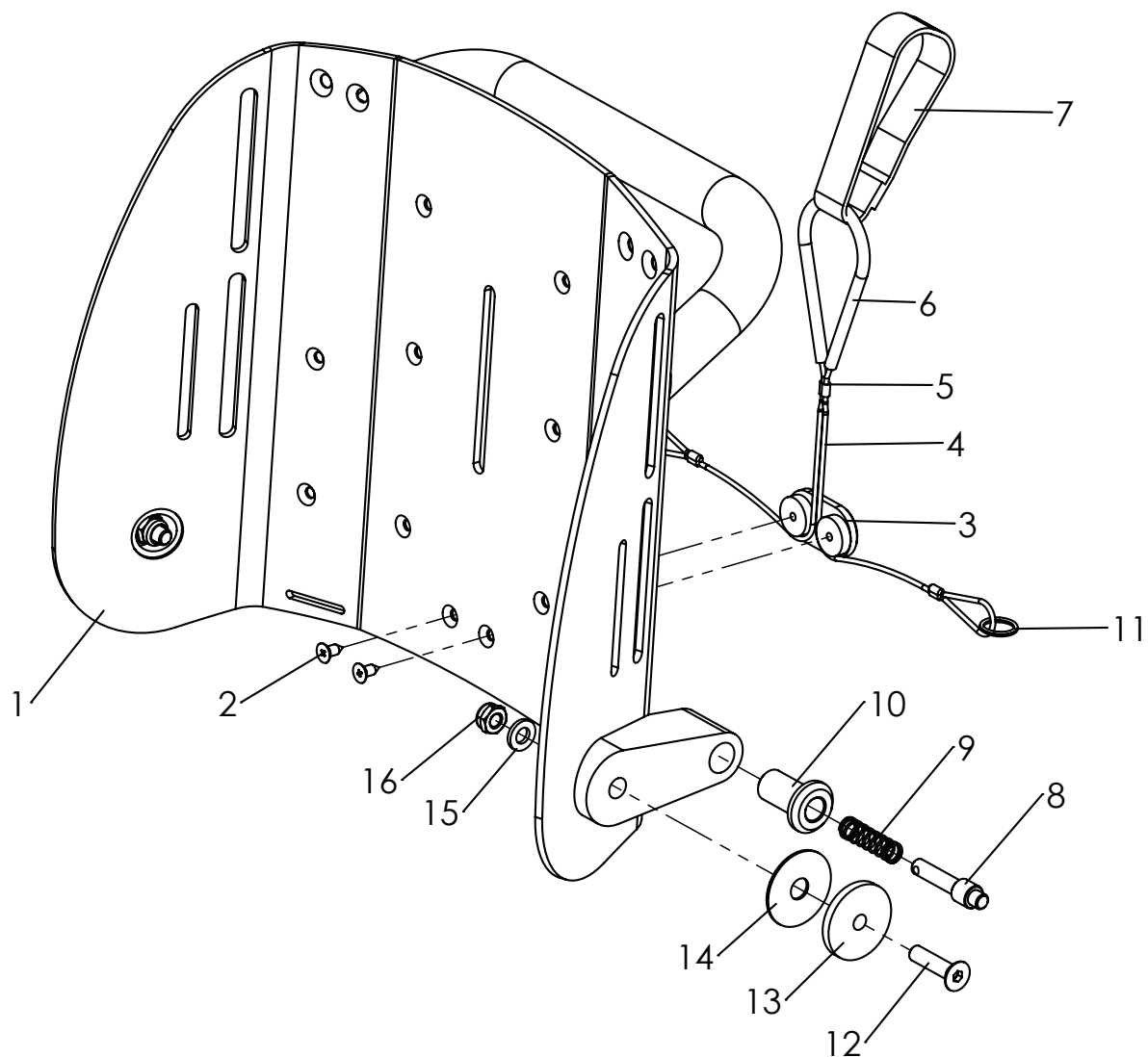

Pos	Article number	Articel description	Quantity	Unit
1-16	3230-2301E-032	Back base Mini, complete	1	Pc.
1	3230-2310-032	Back panel Mini	1	Pc.
2-7	3230-2325	Release cable for backrest recline size Mini	1	Set
8	3201-7530	Lock pin	2	Pc.
9	1131-0036	Compression spring D-127d 0,8x8,3x29	2	Pc.
10	3230-2322	Bolt socket	2	Pc.
11	1129-0036	Key ring 16x1,6	2	Pc.
12	1024-0625-033	Hexagon socket countersunk head screw M6x25	2	Pc.
13	3230-2321-033	Axel bushing	2	Pc.
14	1113-0065	Washer PA nature 8,4x30x1 mm	2	Pc.
15	1051-0006	Washer A6,4	2	Pc.
16	1042-0006	Hexagon nut self-locking M6	2	Pc.

Spare parts




Pos	Article number	Articel description	Quantity	Unit
1-3	3230-5200E-xxx	Upholstery size Mini	1	Pc.
	3232-5100E-xxx	Upholstery size 1	1	Pc.
	3234-5100E-xxx	Upholstery size 2	1	Pc.
	3235-5200E-xxx	Upholstery size 2XL	2	Pc.
1	3230-5120-xxx	Back upholstery size Mini	1	Pc.
	3231-5120-xxx	Back upholstery size 1	1	Pc.
	3233-5120-xxx	Back upholstery size 2	1	Pc.
	3235-5120-xxx	Back upholstery size 2XL	1	Pc.
2	3230-5160E-xxx	Lateral upholstery size Mini	1	Pc.
	3201-5160E-xxx	Lateral upholstery size 1	1	Pc.
	3203-5160E-xxx	Lateral upholstery size 2	1	Pc.
	3235-5160E-xxx	Lateral upholstery size 2XL	1	Pc.
3	3230-5140-xxx	Seat upholstery size mini	1	Pc.
	3232-5140-xxx	Seat upholstery size 1	1	Pc.
	3234-5140-xxx	Seat upholstery size 2	1	Pc.
	3235-5140-xxx	Seat upholstery size 2XL	1	Pc.
4	3231-2920-xxx	Upholstery for Bingo-Evo thigh support size 1, iron grey	1	Pc.
	3233-2920-xxx	Upholstery for Bingo-Evo thigh support size 2, iron grey	1	Pc.
	3235-2920-xxx	Upholstery for Bingo-Evo thigh support size 2XL, iron grey	1	Pc.
5	3230-5150-xxx	Hip upholstery size Mini	1	Pc.
	3201-5150E-xxx	Hip upholstery size 1	1	Pc.
	3203-5150E-xxx	Hip upholstery size 2	1	Pc.
	3235-5150E-xxx	Hip upholstery size 2XL	1	Pc.

With order replace **xxx** with colour code!


346	347	348	349	350	351	352
black	berry soft	grey soft	turquoise soft	berry soft/ black	grey soft/ black	turquoise soft/ black

Spare parts

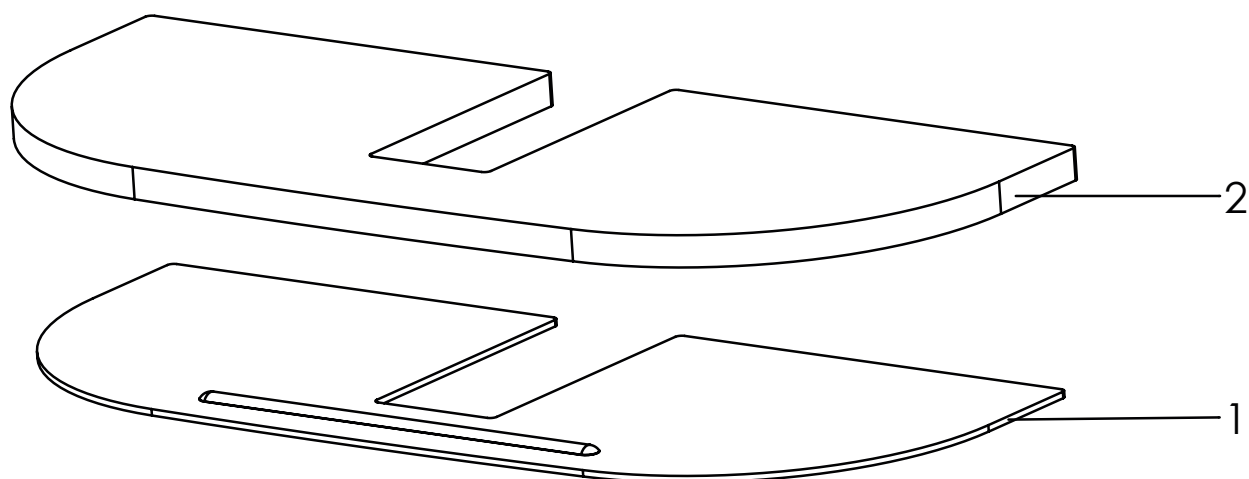


Pos	Article number	Articel description	Quantity	Unit
1-2	3232-5190	Calf pad support plate size 1 with foam insert	1	Set
	3234-5190	Calf pad support plate size 2 with foam insert	1	Set
1	3231-5149-032	Calf pad plate size 1	1	Pc.
	3233-5149-032	Calf pad plate size 2	1	Pc.
2	3232-5173	Foam for Calf pad plate size 1	1	Pc.
	3234-5173	Foam for Calf pad plate size 2	1	Pc.

Spare parts

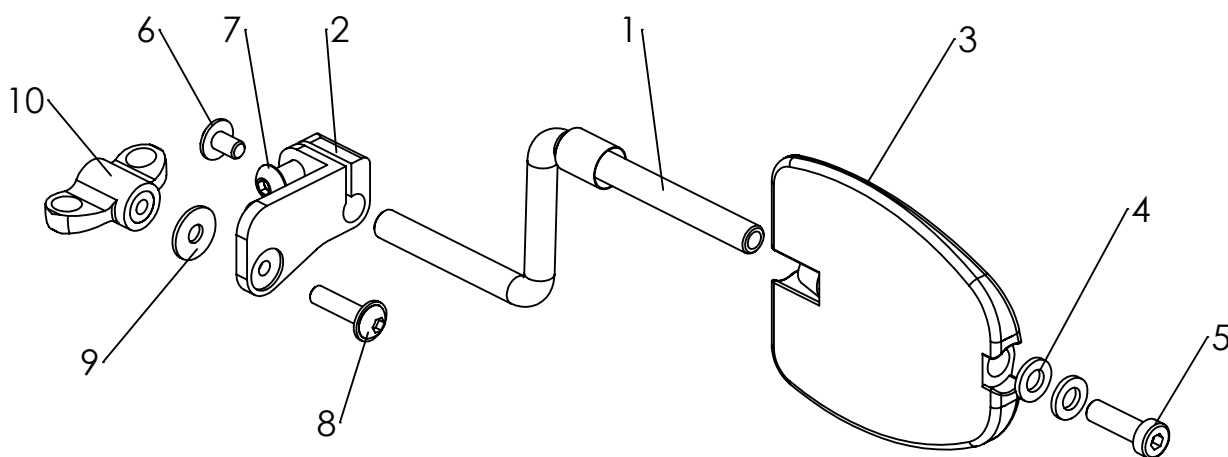


Pos	Article number	Articel description	Quantity	Unit
1-10	3232-7701	Trunk supports size 1 without cover	1	Set
	3232-7702	Trunk supports size 2 without cover	1	Set
	3232-7703	Trunk supports size 3 without cover	1	Set
1	3232-7710	Pad crank	2	Pc.
2	3232-7741	Base Trunk supports left	1	Pc.
	3232-7742	Base Trunk supports right	1	Pc.
3	3231-7730	Pad size 1	2	Pc.
	3233-7730	Pad size 2	2	Pc.
	3235-7730	Pad size 3	2	Pc.
4	1051-0006	Washer A6.4	4	Pc.
5	1022-0620	Hexagon socket head cap screw M6x20	2	Pc.
	1022-0640	Hexagon socket head cap screw M6x40	2	Pc.
	1022-0650	Hexagon socket head cap screw M6x50	2	Pc.
6	1030-0512	Button head socket screw with flange M5x12	2	Pc.
7	1029-0612	Button head socket screw M6x12	2	Pc.
8	1030-0516	button head socket screw with flange M5x16	2	Pc.
9	1055-0005	Washer A5,3	2	Pc.
10	1123-0800-DOR	Wing nut with brass bush orange	2	Pc.
11	3232-7771	Adhesive pad trunk support pad size 1	2	Pc.
	3232-7781	Adhesive pad trunk support pad size 2	2	Pc.
	-----	-----	-	-

Spare parts

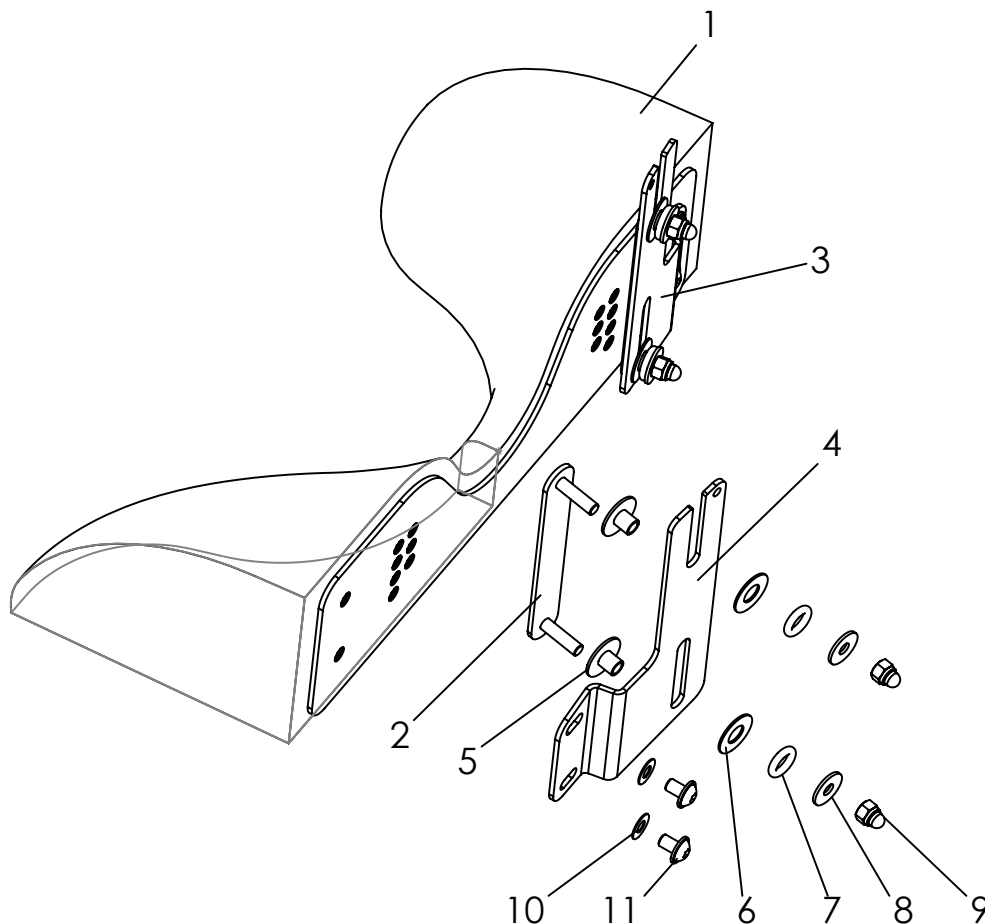


Pos	Article number	Articel description	Quantity	Unit
1-11	3230-6900	Anatomical neck-headrest with length compensation, size mini	1	Pc.
	3231-6900	Anatomical neck-headrest with length compensation, size 1	1	Pc.
	3233-6900	Anatomical neck-headrest with length compensation, size 2	1	Pc.
1	3210-6930-306	Neck headrest size 1	1	Pc.
2	3230-6921	Threaded plate size mini	2	Pc.
	3231-6921	Threaded plate size 1	2	Pc.
	3233-6921	Threaded plate size 2	2	Pc.
3	3230-6911-052	Guide size mini, left	1	Pc.
	3231-6911-052	Guide size 1, left	1	Pc.
	3233-6911-052	Guide size 2, left	1	Pc.
4	3230-6912-052	Guide size 1mini right	1	Pc.
	3231-6912-052	Guide size 1, right	1	Pc.
	3233-6912-052	Guide size 2, right	1	Pc.
5	3231-6923	Guide bushing	4	Pc.
6	1113-0004	Washer PA	4	Pc.
7	1150-0009	O-Ring NBR70	4	Pc.
8	1055-0005-033	Washer A5,3	4	Pc.
9	1044-0005-033	Hexagonal domed cap nut self-locking black M5	4	Pc.
10	1052-0005	Spring washer B5	4	Pc.
11	1030-0508	Button head socket screw with flange M5x8	4	Pc.

Spare parts

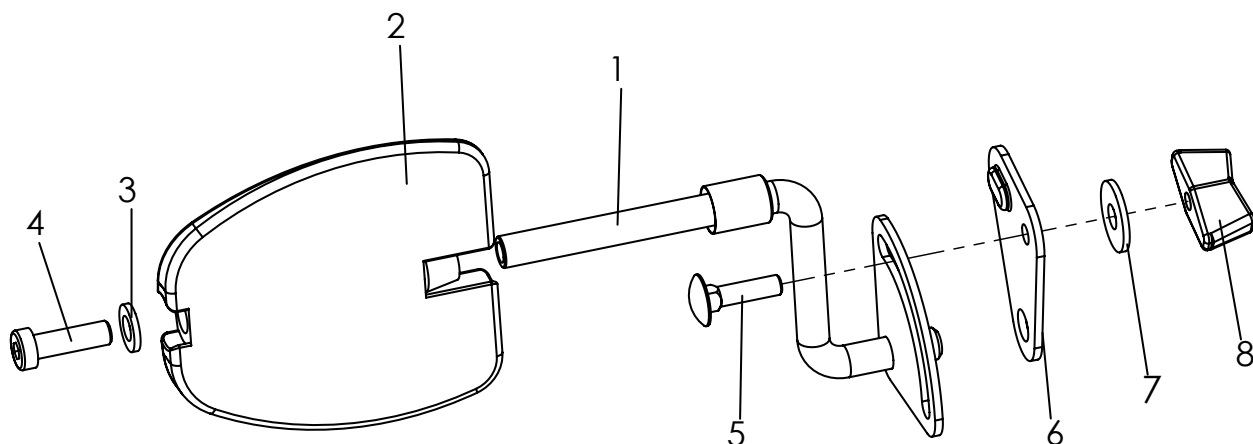


Pos	Article number	Articel description	Quantity	Unit
1-8	3231-7700	Trunk suport size 1, without upholstery	1	Pair
	3233-7700	Trunk suport size 2, without upholstery	1	Pair
	3235-7700	Trunk suport size 3, without upholstery	1	Pair
1	3201-7720-008	Pad crank right size 1	1	Pc.
	3201-7710-008	Pad crank left size 1	1	Pc.
2	3231-7730	Pad size 1	2	Pc.
	3233-7730	Pad size 2	2	Pc.
	3234-7730	Pad size 3	2	Pc.
3	1051-0006	Washer A6,4	2	Pc.
4	1022-0620	Hexagon socket head cap screw M6x20, for size 1	2	Pc.
	1022-0640	Hexagon socket head cap screw M6x40, for size 2	2	Pc.
	1022-0650	Hexagon socket head cap screw M6x50, for size 3	2	Pc.
5	1026-0520	Mushroom head square neck bolt M5x20	2	Pc.
6	3201-7717-008	Bracket of trunk support right	1	Pc.
	3201-7716-008	Bracket of trunk support left	1	Pc.
7	1055-0006	Washer A6,4	2	Pc.
8	1123-0500	Wing nut PA black M5	2	Pc.
9	3231-7760-346	Upholstery size 1, black	2	Pc.
	3233-7760-346	Upholstery size 2, black	2	Pc.
	3203-7760-346	Upholstery size 3, black	2	Pc.

Spare parts

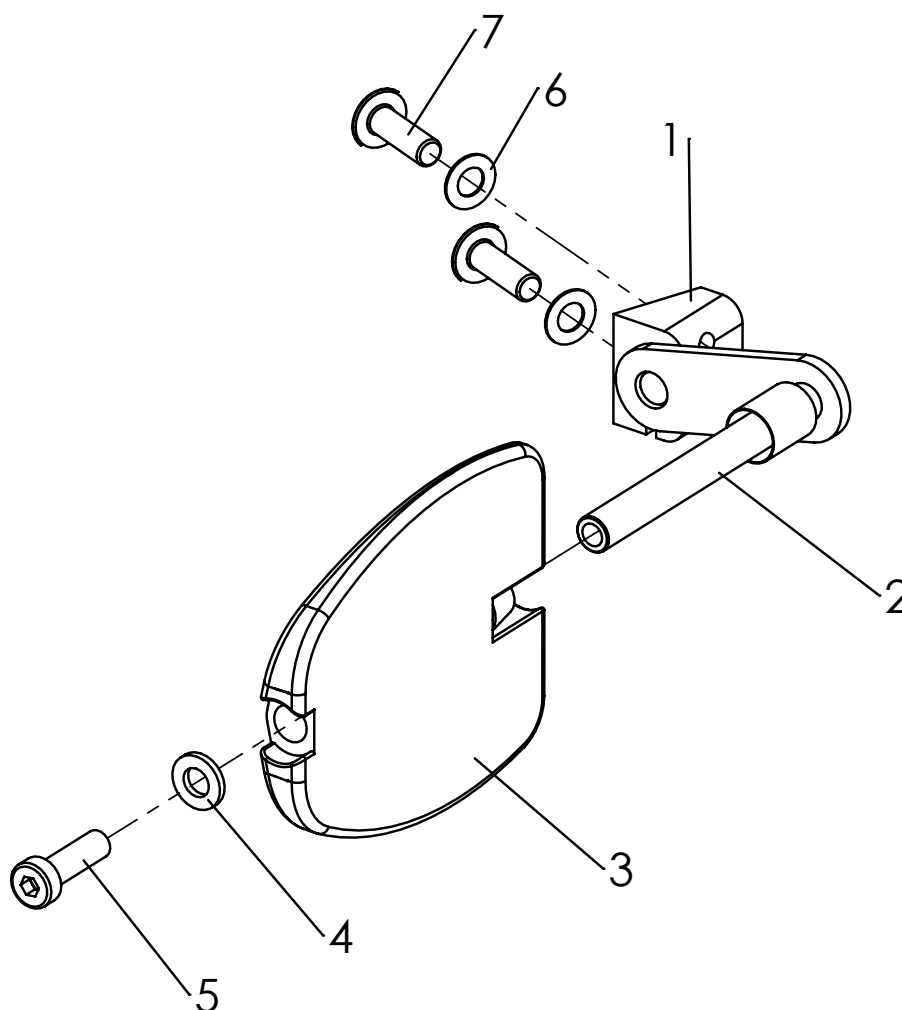

Pos	Article number	Articel description	Quantity	Unit
1-7	3230-7700	Trunk support, size Mini	1	Set
1	3230-7721-007	Clamp base Mini	2	Pc.
2	3230-7710-030	Swivel lever Thoraxpelotte Mini, black	2	Pc.
3	3231-7730	Pad size 1	2	Pc.
4	1051-0006	Washer A6,4	2	Pc.
5	1022-0620	Hexagon socket head cap screw M6x20	2	Pc.
6	1052-0006	Spring washer B6	4	Pc.
7	1030-0620	Button head socket screw with flange M6x20	4	Pc.
8	3231-7760-346	Upholstery size 1, black	2	Pc.


Spare parts

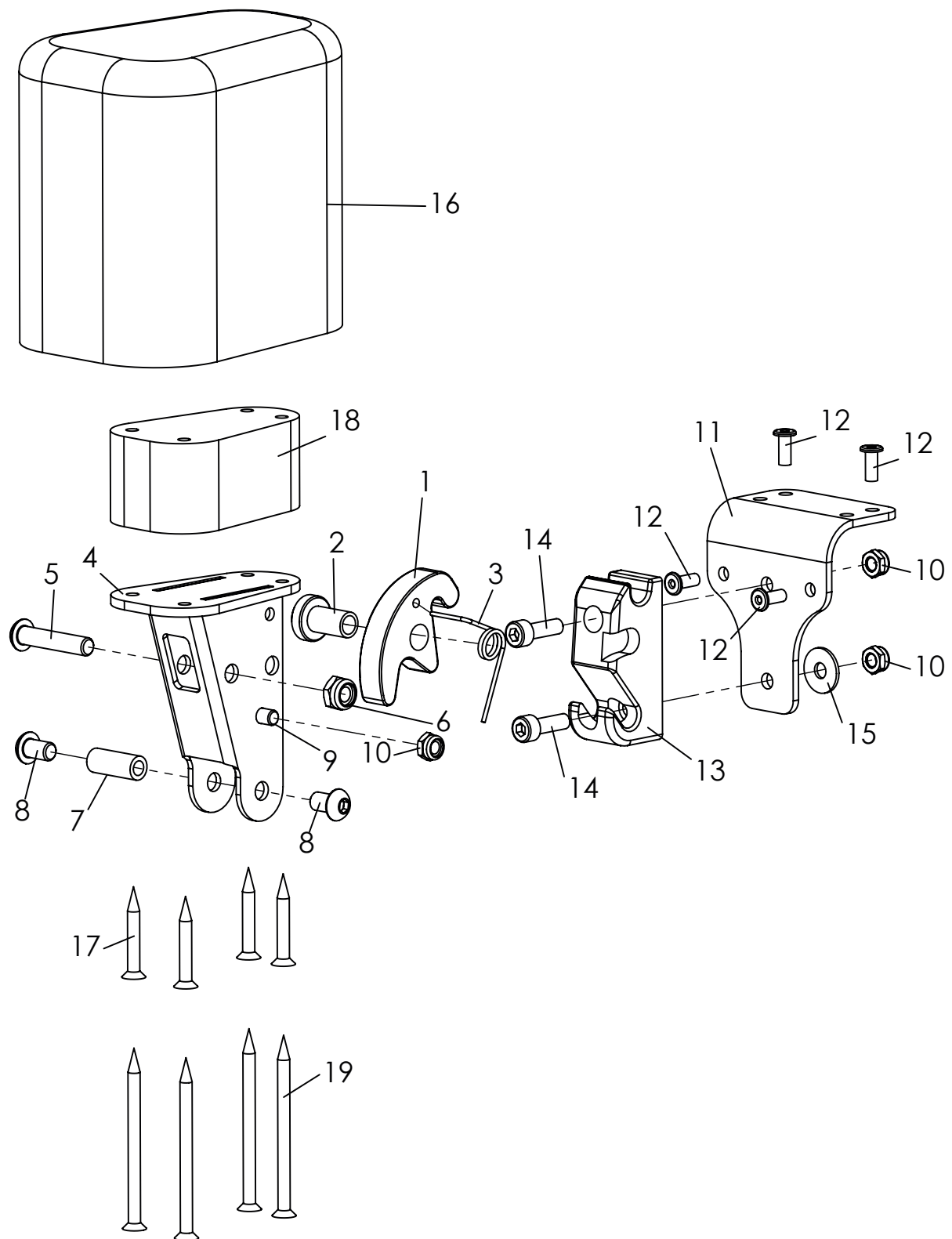

Spare parts


Pos	Article number	Articel description	Quantity	Unit
1-17	3231-7800	Abduction block size 1 swing&detachable, without cover	1	Pc.
	3231-7900	Abduction block size 2 swing&detachable, without cover	1	Pc.
	3233-7800	Abduction block size 3 swing&detachable, without cover	1	Pc.
1	3231-7841	Arresting clamp	1	Pc.
2	3231-7842	Bearing arresting clamp	1	Pc.
3	3231-7843	Leg spring 1,25 x 9,25 x 2,25	1	Pc.
4	3231-7820-032	A-wedge support	1	Pc.
5	1029-0630	Button head socket screw M6x30	1	Pc.
6	1042-0006	Hexagon nut self-locking 985 M6	1	Pc.
7	1114-0067	Threaded bushing 8xM6x20	1	Pc.
8	1030-0608	button head socket screw with flange zp M6x8	2	Pc.
9	1021-0508	Cylinder head screw M5x8	1	Pc.
10	1042-0005	Hexagon nut self-locking M5	3	Pc.
11	3231-7831-032	Base plate abduction block receiver	1	Pc.
12	1062-0410	Blind rivet F4x10	4	Pc.
13	3231-7832	Arresting part for A-wedge	1	Pc.
14	1021-0516	Hexagon socket head cap screw M5x16	2	Pc.
15	1055-0005	Washer A5,3	1	Pc.
16	3201-7810	Wood wedge size 1 with PU-foam	1	Pc.
	3201-7910	Wood wedge size 1 with PU-foam	1	Pc.
	3203-7810	Wood wedge size 1 with PU-foam	1	Set
17	1028-0430	Pressboard screw countersunk head 4x30	4	Pc.
18	3231-7821_SA-0286/2	Spacer plate for A-wedge 30 mm	1	Pc.
19	1028-0460	Pressboard screw countersunk head 4x60	4	Pc.
	3201-7840-346	Cover for abduction block size 1 black	1	Pc.
	3201-7940-346	Cover abduction block size 2 black	1	Pc.
	3203-7840-346	Cover for abduction block size 3 black	1	Pc.

Spare parts

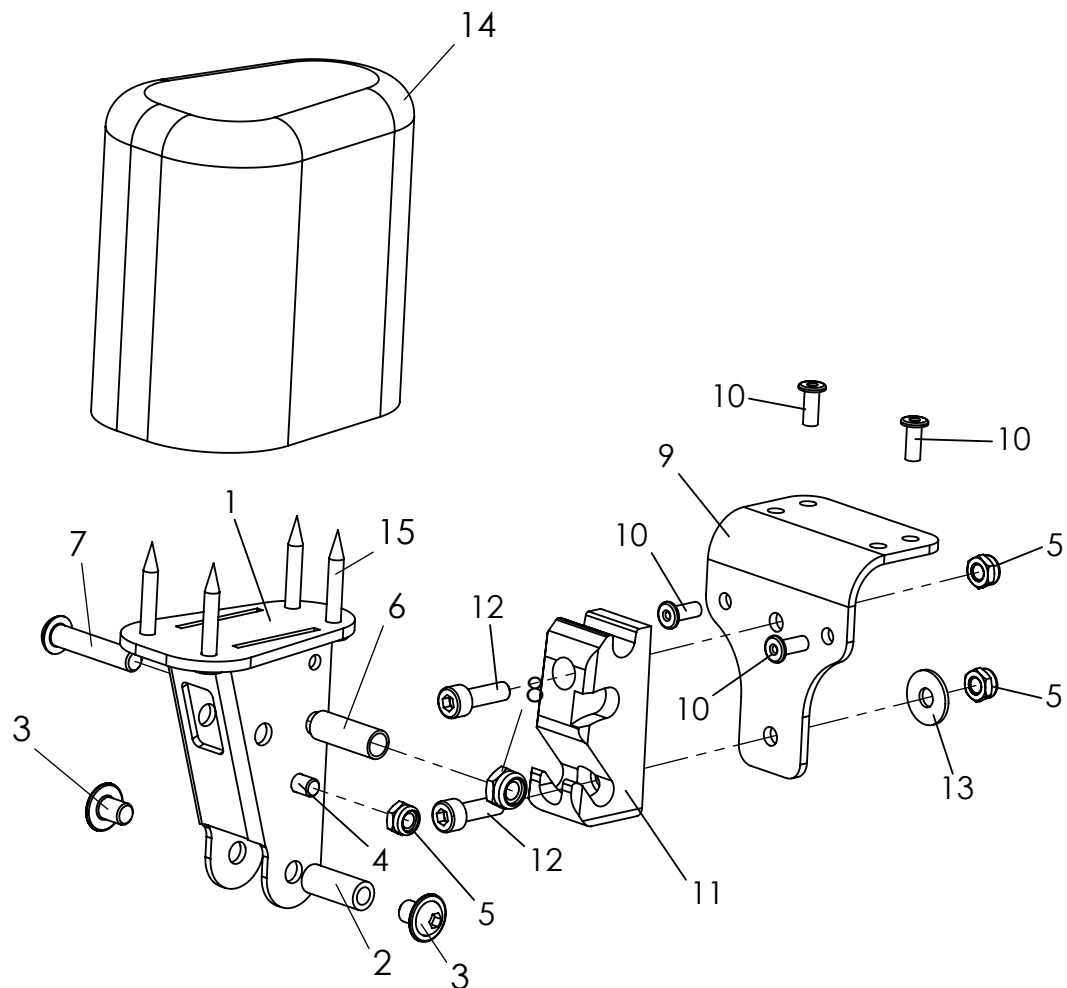


Pos	Article number	Articel description	Quantity	Unit
1-15	3231-7801	Bingo-Evo abduction block size 1 fixed, with out cover	1	Pc.
	3231-7901	Bingo-Evo abduction block size 2 fixed, with out cover	1	Pc.
	3233-7801	Bingo-Evo abduction block size 3 fixed, with out cover	1	Pc.
1	3231-7820-032	A-wedge support	1	Pc.
2	1114-0067	Threaded bushing 8xM6x20	1	Pc.
3	1030-0608	button head socket screw with flange M6x8	2	Pc.
4	1021-0508	Cylinder head screw M5x8	1	Pc.
5	1042-0005	Hexagon nut self-locking M5	3	Pc.
6	1114-0012	Bushing zp 8x0,8x22,5	1	Pc.
7	1029-0630	button head socket screw M6x30	1	Pc.
8	1042-0006	Hexagon nut self-locking M6	1	Pc.
9	3231-7831-032	Base plate abduction block receiver	1	Pc.
10	1062-0410	Blind rivet F4x10	4	Pc.
11	3231-7832	Arresting part for A-wedge	1	Pc.
12	1021-0516	Hexagon socket head cap screw M5x16	2	Pc.
13	1055-0005	Washer A5,3	1	Pc.
14	3201-7810	Wood wedge size 1 with PU-foam	1	Pc.
	3201-7910	Wood wedge size 2 with PU-foam	1	Pc.
	3203-7810	Wood wedge size 3 with PU-foam	1	Pc.
15	1028-0430	Pressboard screw countersunk head 4x30	4	Pc.
	3201-7840-346	Cover for abduction block size 1 black	1	Pc.
	3201-7940-346	Cover abduction block size 2 black	1	Pc.
	3203-7840-346	Cover for abduction block size 3 black	1	Pc.

Spare parts

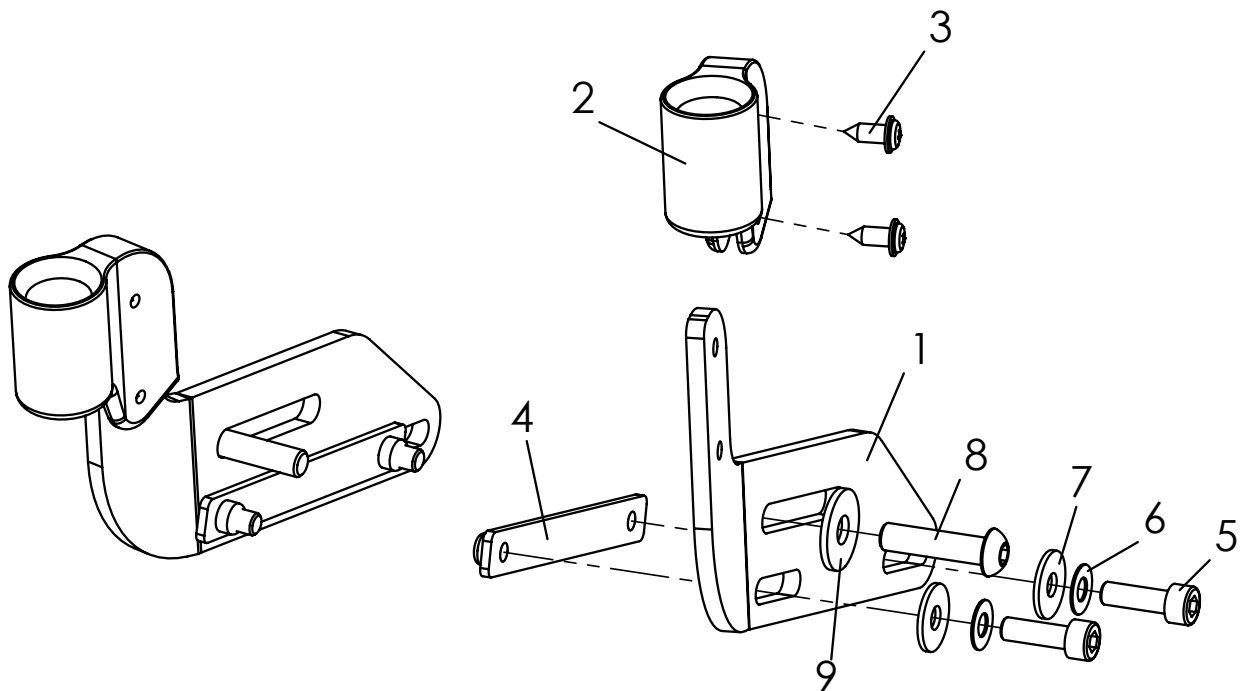


Pos	Article number	Articel description	Quantity	Unit
1-9	3232-2203-032	Retainer for accessories, like grab rail or therapy tray	1	Pc.
	3232-2204-032	Retainer long for accessories, like grab rail or therapy tray	1	Pc.
1	3232-2232-032	Holder for accessories retainer left side	1	Pc.
	3232-2242-032	Holder for accessories retainer right side	1	Pc.
	3232-2233-032	Holder long for accessories retainer left side	1	Pc.
	3232-2243-032	Holder long for accessories retainer right side	1	Pc.
2	3231-2232	Retainer for accessories mould part	2	Pc.
3	1039-0001	Screw for plastic, cross rcessed pan head w. flange 4x10	4	Pc.
4	3232-2235	Threaded plate black galvanized	2	Pc.
5	1021-0516-033	Hexagon socket head cap screw black M5x16	4	Pc.
6	1052-0005-033	Spring washer B5 black	4	Pc.
7	1055-0005-033	Washer A5,3 black	4	Pc.
8	1029-0625-033	Button head socket screw black M6x25	2	Pc.
9	1055-0006-033	Washer A6,4	2	Pc.

Spare parts

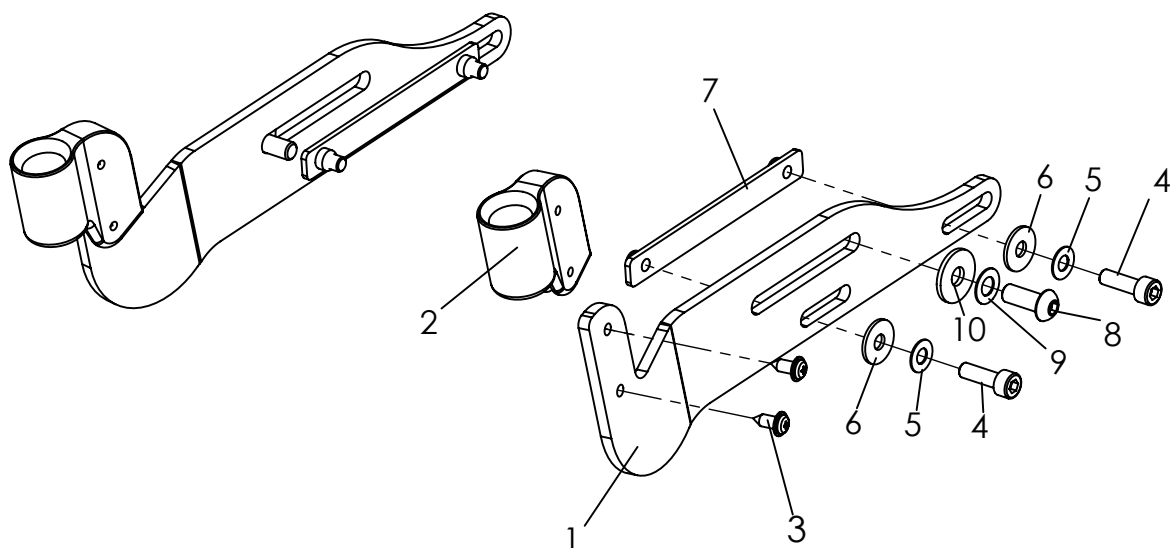


Pos	Article number	Articel description	Quantity	Unit
1-10	3230-2201-032	Retainer Mini, for accessories like grab rail or therapy tray	1	Pc.
1	3230-2232-032	Holder for accessories retainer Mini, left side	1	Pc.
	3230-2242-032	Holder for accessories retainer Mini, right side	1	Pc.
2	3231-2232	Retainer for accessories mould part	2	Pc.
3	1039-0001	Screw for plastic, cross rcessed pan head w. flange 4x10	4	Pc.
4	1021-0516-033	Hexagon socket head cap screw black M5x16	4	Pc.
5	1052-0005-033	Spring washer B5	4	Pc.
6	1055-0005-033	Washer A5,3	4	Pc.
7	3232-2235	Threaded plate black galvanized	2	Pc.
8	1029-0625-033	Button head socket screw black M6x25	2	Pc.
9	1052-0006-033	Spring washer B6 black	2	Pc.
10	1055-0006-033	Washer A6,4 black	2	Pc.

Spare parts

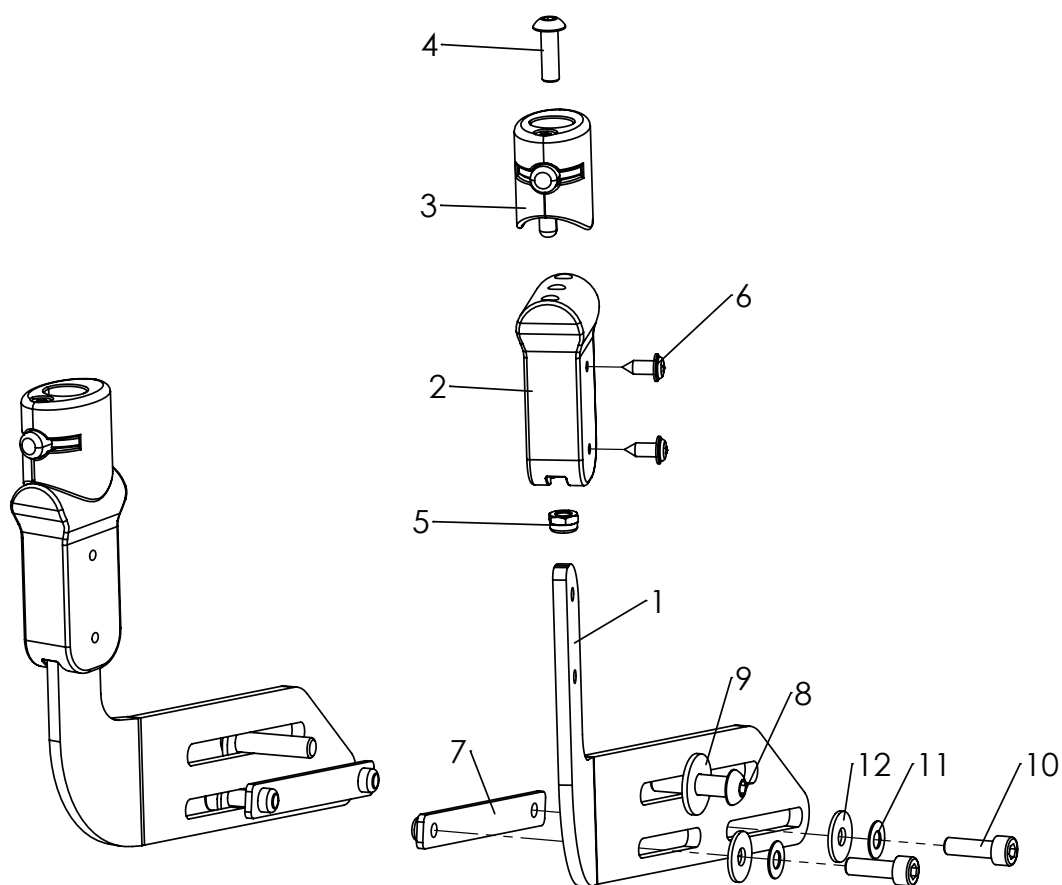


Pos	Article number	Articel description	Quantity	Unit
1-12	3232-2205-032	Bingo-Evo retainer	1	Pc.
1	3232-2251-032	Bracket accessory holder left black dull matt	1	Pc.
	3232-2261-032	Bracket accessory holder right black dull matt	1	Pc.
2	3102-2221	Holder for adapter	2	Pc.
3	3231-8130	Grab rail adapter	2	Pc.
4	1029-0516	Button head socket screw M5x16	2	Pc.
5	1044-0005	Hexagonal domed cap nut self-locking M5	2	Pc.
6	1039-0001	Screw for plastic, cross pan head w. flange 4x10	4	Pc.
7	3232-2235	Threaded plate black	2	Pc.
8	1029-0625-033	Button head socket screw black M6x25	2	Pc.
9	1055-0006-033	Washer A6,4	2	Pc.
10	1021-0516-033	Hexagon socket head cap screw black M5x16	4	Pc.
11	1052-0005-033	Spring washer B5	4	Pc.
12	1055-0005-033	Washer A5,3	4	Pc.

Spare parts

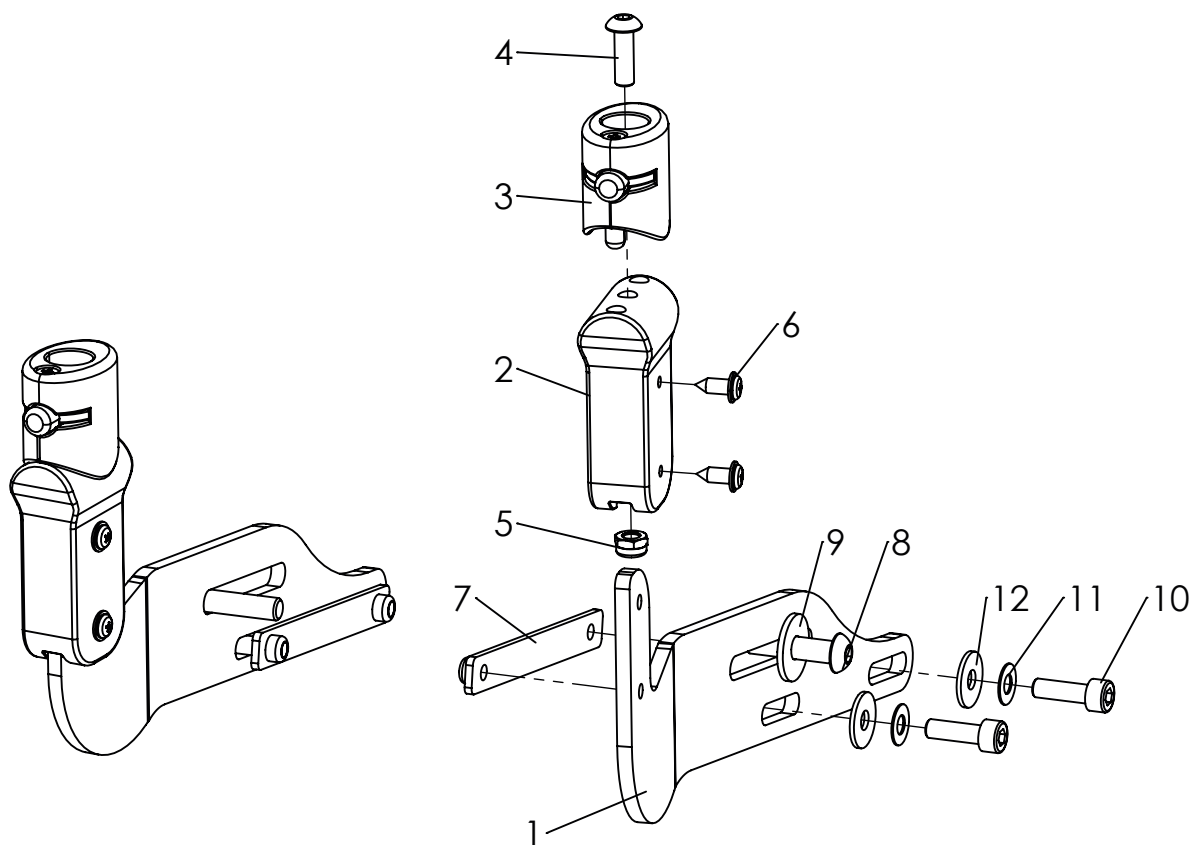


Pos	Article number	Articel description	Quantity	Unit
1-12	3230-3202-032	Bingo-Evo retainer size Mini	1	Pc.
1	3230-2234-032	Bracket accessory holder Bingo Evo mini left black dull matt	1	Pc.
	3230-2244-032	Bracket accessory holder Bingo Evo mini right black dull matt	1	Pc.
2	3102-2221	Holder for adapter	2	Pc.
3	3231-8130	Grab rail adapter	2	Pc.
4	1029-0516	Button head socket screw M5x16	2	Pc.
5	1044-0005	Hexagonal domed cap nut self-locking M5	2	Pc.
6	1039-0001	Screw for plastic, cross pan head w. flange 4x10	4	Pc.
7	3232-2235	Threaded plate black	2	Pc.
8	1029-0625-033	Button head socket screw black M6x25	2	Pc.
9	1055-0006-033	Washer black A6,4	2	Pc.
10	1021-0516-033	Hexagon socket head cap screw black M5x16	4	Pc.
11	1052-0005-033	Spring washer B5	4	Pc.
12	1055-0005-033	Washer A5,3	4	Pc.

Spare parts

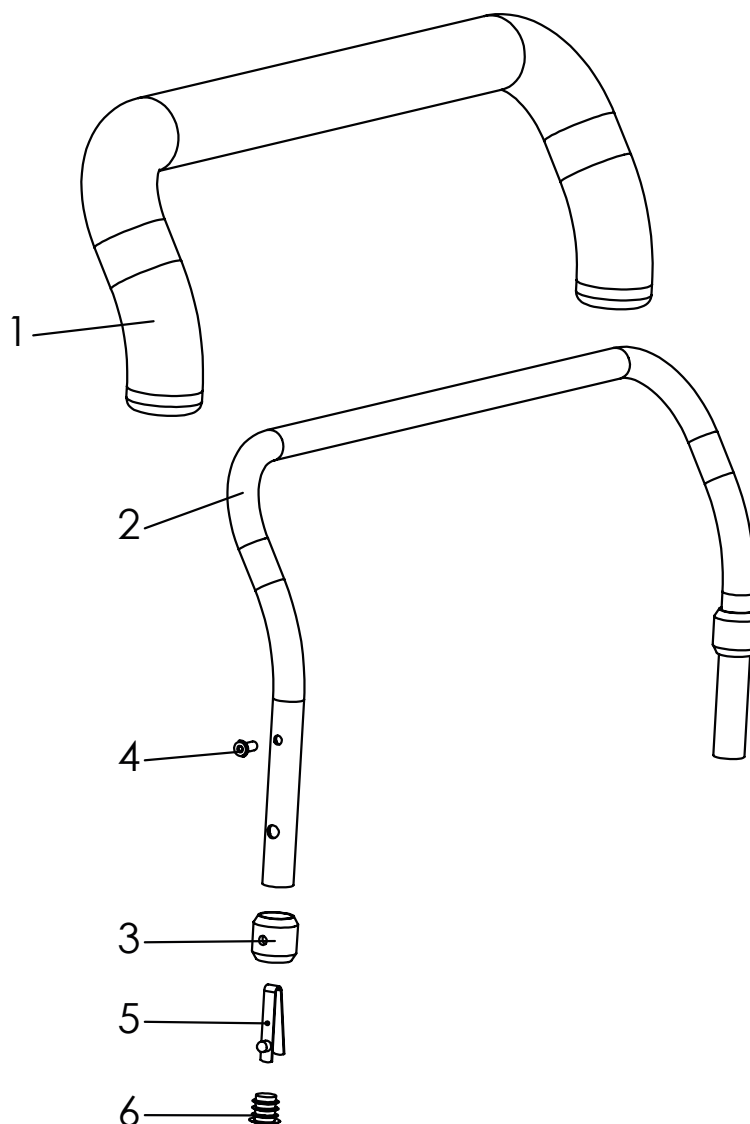


Pos	Article number	Articel description	Quantity	Unit
1-7	3230-8100	Grab rail with upholstery Mini	1	Pc.
	3231-8100	Grab rail with upholstery size 1	1	Pc.
	3233-8100	Grab rail with upholstery size 2	1	Pc.
1	3230-8130	Bend Tube for grab rail size Mini	1	Pc.
	3201-8130	Bend Tube for grab rail size 1	1	Pc.
	3203-8130	Bend Tube for grab rail size 2	1	Pc.
2-4	3230-8110	Upholstery foam material size Mini, (55 cm)	1	Pc.
	3231-8110	Upholstery foam material size 1, (60 cm)	1	Pc.
	3233-8110	Upholstery foam material size 2 (70 cm)	1	Pc.
3	3201-8120	Bushing	2	Pc.
4	1062-0408	Blind rivet St/St DIN/ISO 7337 F4x8	2	Pc.
5	1134-0005	Snap button A 102	2	Pc.
6	1141-0001	Ribbed insert SFL 14x0,8-2	2	Pc.
7	3230-8140-346	Cover of grab rail Mini, black	1	Pc.
	3201-8140-346	Cover of grab rail size 1, black	1	Pc.
	3203-8140-346	Cover of grab rail size 2, black	1	Pc.

Spare parts

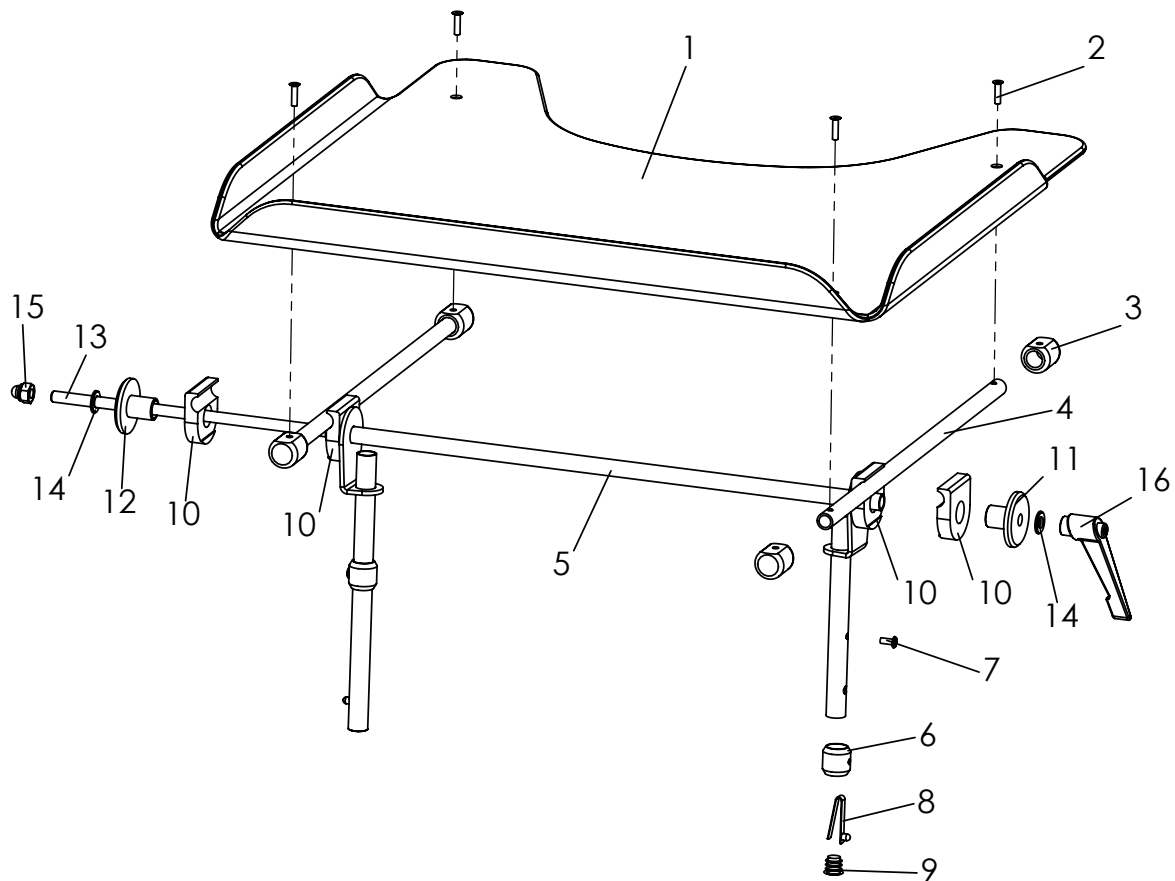


Pos	Article number	Articel description	Quantity	Unit
1-16	3230-8200	Therapy tray size Mini, clear	1	Pc.
	3231-8200	Therapy tray size 1, clear	1	Pc.
	3233-8200	Therapy tray size 2, clear	1	Pc.
	3230-8201	Therapy tray size Mini, gray	1	Pc.
	3231-8201	Therapy tray size 1, gray	1	Pc.
	3233-8201	Therapy tray size 2, gray	1	Pc.
1	3231-8290	Tray plate only, clear	1	Pc.
	3231-8291	Tray plate only, grey	1	Pc.
1-4	3231-8280	Tray plate clear, <i>with rail</i>	1	Pc.
	3231-8281	Tray plate gray, <i>with rail</i>	1	Pc.
2	1063-0416	Blind rivet S4x10	4	Pc.
3	3201-8280	Bearing block	4	Pc.
4	3201-8270-016	Rail	2	Pc.
5-9	3230-8210E-032	Table base frame, mini	1	Pc.
	3231-8210E-032	Table base frame, size 1	1	Pc.
	3233-8210E-032	Table base frame, size 2	1	Pc.
6	3201-8120	Bushing	2	Pc.
7	1062-0408	Blind rivet F4x8	2	Pc.
8	1134-0005	Snap button A 102	2	Pc.
9	1141-0001	Ribbed insert SFL 14x0,8-2	2	Pc.
10	3201-8230	Clamp half	4	Pc.
11	3201-8240	Clamp disk D 8,5	1	Pc.
12	3201-8250	Clamp disk M8	1	Pc.
13	3231-8260	Thread M8x47,2mm	0,47	m
14	1051-0008	Washer A8,4	2	Pc.
15	1044-0008	Hexagonal domed cap nut self-locking M8	1	Pc.
16	1121-0800	Locking lever PA black M8	1	Pc.

Spare parts

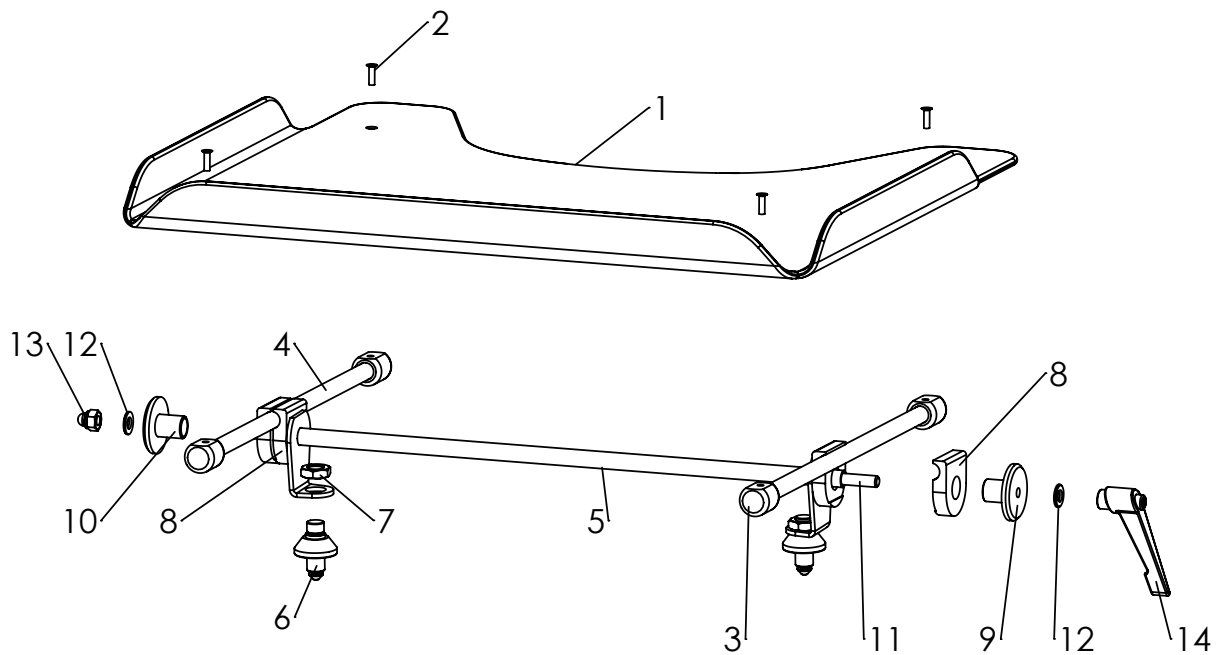


Pos	Article number	Articel description	Quantity	Unit
1-14	3230-8400	Therapy tray size mini, transparent	1	Pc.
	3231-8400	Therapy tray size 1, transparent	1	Pc.
	3233-8400	Therapy tray size 2, transparent	1	Pc.
	3230-8401	Therapy tray size mini, grey	1	Pc.
	3231-8401	Therapy tray size 1, grey	1	Pc.
	3233-8401	Therapy tray size 2, grey	1	Pc.
1	3231-8290	Tray plate clear	1	Pc.
	3231-8291	Tray plate grey	1	Pc.
1-4	3231-8280	Tray plate clear, <i>with rail</i>	1	Pc.
	3231-8281	Tray plate grey, <i>with rail</i>	1	Pc.
2	1063-0416	Blind rivet S4x16	4	Pc.
3	3201-8280	Bearing block	4	Pc.
4	3201-8270-016	Rail	2	Pc.
5	3230-8410-032	Adapter V2 adjustable mini	1	Pc.
	3231-8410-032	Adapter V2 adjustable size 1	1	Pc.
	3233-8410-032	Adapter V2 adjustable size 2	1	Pc.
6	3231-8413	Locking bolt therapy table simple	2	Pc.
7	1040-0012	Hexagon nut M12x1,25	2	Pc.
8	3201-8230	Clamp half	4	Pc.
9	3201-8250	Clamp disk M8	1	Pc.
10	3201-8240	Clamp disk D 8,5	1	Pc.
11	3231-8260	Threaded rod mtr 8x474	1	Pc.
12	1051-0008	Washer A8,4	2	Pc.
13	1044-0008	Hexagonal domed cap nut self-locking M8	1	Pc.
14	1121-0800	Locking lever PA black M8	1	Pc.

Spare parts

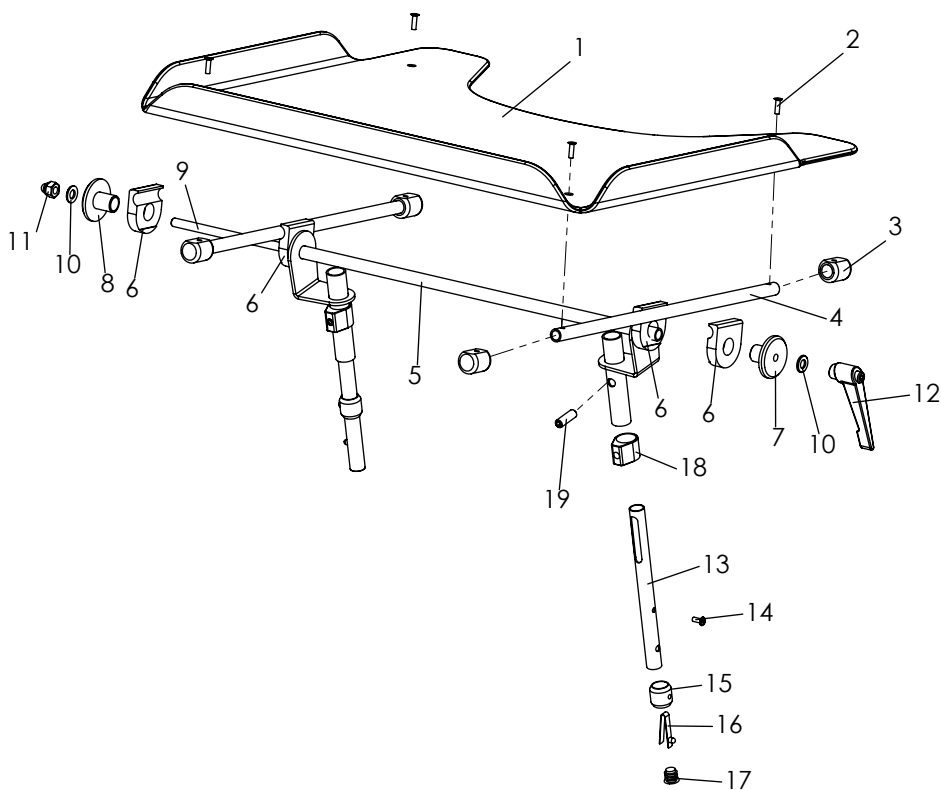


Pos	Article number	Articel description	Quantity	Unit
1-19	3230-8202	Therapy tray Mini, clear, height adjustable	1	Pc.
	3231-8202	Therapy tray size 1, clear, height adjustable	1	Pc.
	3233-8202	Therapy tray size 2, clear, height adjustable	1	Pc.
	3230-8203	Therapy tray Mini, gray, height adjustable	1	Pc.
	3231-8203	Therapy tray size 1, gray, height adjustable	1	Pc.
	3233-8203	Therapy tray size 2, gray, height adjustable	1	Pc.
1	3231-8290	Tray plate only, clear	1	Pc.
	3231-8291	Tray plate only, grey	1	Pc.
1-4	3231-8280	Tray plate clear, with rails	1	Pc.
1-4	3231-8281	Tray plate gray, with rails	1	Pc.
2	1063-0416	Blind rivet S4x10	4	Pc.
3	3201-8280	Bearing block	4	Pc.
4	3201-8270-016	Rail	2	Pc.
5	3230-8220-032	Table base frame, Mini, height adjustable	1	Pc.
	3231-8220-032	Table base frame, size 1, height adjustable	1	Pc.
	3233-8220-032	Table base frame, size 2, height adjustable	1	Pc.
6	3201-8230	Clamp half	4	Pc.
7	3201-8250	Clamp disk M8	1	Pc.
8	3201-8240	Clamp disk D 8,5	1	Pc.
9	3231-8260	Threaded rod 8x474	0,47	m
10	1051-0008	Washer A8,4	2	Pc.
11	1044-0008	Hexagonal domed cap nut self-locking M8	1	Pc.
12	1121-0800	Locking lever PA black M8	1	Pc.
13-17	3231-8215E-032	Support tube, adjustablt left	1	Pc.
	3231-8216E-032	Support tube, adjustablt right	1	Pc.
14	1062-0408	Blind rivet F4x8	2	Pc.
15	3201-8120	Bushing	2	Pc.
16	1134-0005	Snap button A 102	2	Pc.
17	1141-0001	Ribbed insert SFL 14x0,8-2	2	Pc.
18	3201-8221	Clamp ring for height adjustable tray	2	Pc.
19	1031-0825	Hexagon socket set screw M8x25	2	Pc.

Spare parts

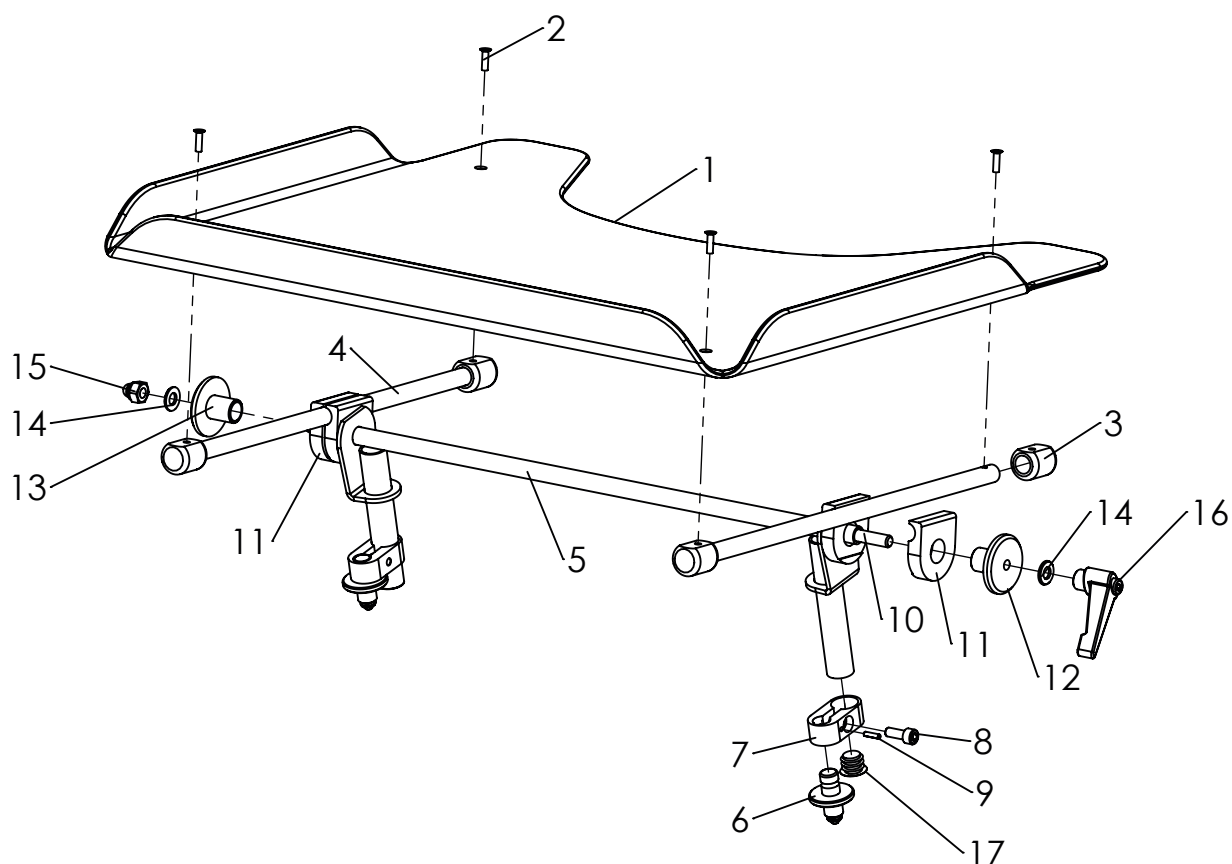

Pos	Article number	Articel description	Quantity	Unit
1-17	3230-8402	therapy tray size mini, transparent, height adjustable	1	Pc.
	3231-8402	therapy tray size 1, transparent, height adjustable	1	Pc.
	3233-8402	therapy tray size 2, transparent, height adjustable	1	Pc.
	3230-8403	therapy tray size mini, grey, height adjustable	1	Pc.
	3231-8403	therapy tray size 1, grey, height adjustable	1	Pc.
	3233-8403	therapy tray size 2, grey, height adjustable	1	Pc.
1	3231-8290	tray plate clear	1	Pc.
	3231-8291	tray plate grey	1	Pc.
1-4	3231-8280	tray plate clear with mounted rails	1	Pc.
	3231-8281	tray plate grey with mounted rails	1	Pc.
2	1063-0416	Blind rivet St/St DIN/ISO 7337 S4x16	1	Pc.
3	3201-8280	Bearing block	4	Pc.
4	3201-8270-016	rail	2	Pc.
5	3230-8430-032	adapter height adjustable mini	1	Pc.
	3231-8430-032	adapter height adjustable size 1	1	Pc.
	3233-8430-032	adapter height adjustable size 2	1	Pc.
6	3231-8422	Locking bolt Therapy table	2	Pc.
7	3231-8421-007	Clamp	2	Pc.
8	1021-0616	Hexagon socket head cap screw M6x16	2	Pc.
9	1070-0312	Spring pin 3x12	2	Pc.
10	3231-8260	Threaded rod 8x474	1	Pc.
11	3201-8230	clamp half	4	Pc.
12	3201-8250	clamp disk M8	1	Pc.
13	3201-8240	clamp disk D 8,5	1	Pc.
14	1051-0008	Washer A8,4	2	Pc.
15	1044-0008	Hexagonal domed cap nut self-locking M8	1	Pc.
16	1121-0800	Locking lever M8	1	Pc.
17	1141-0005	Ribbed insert	2	Pc.

Spare parts

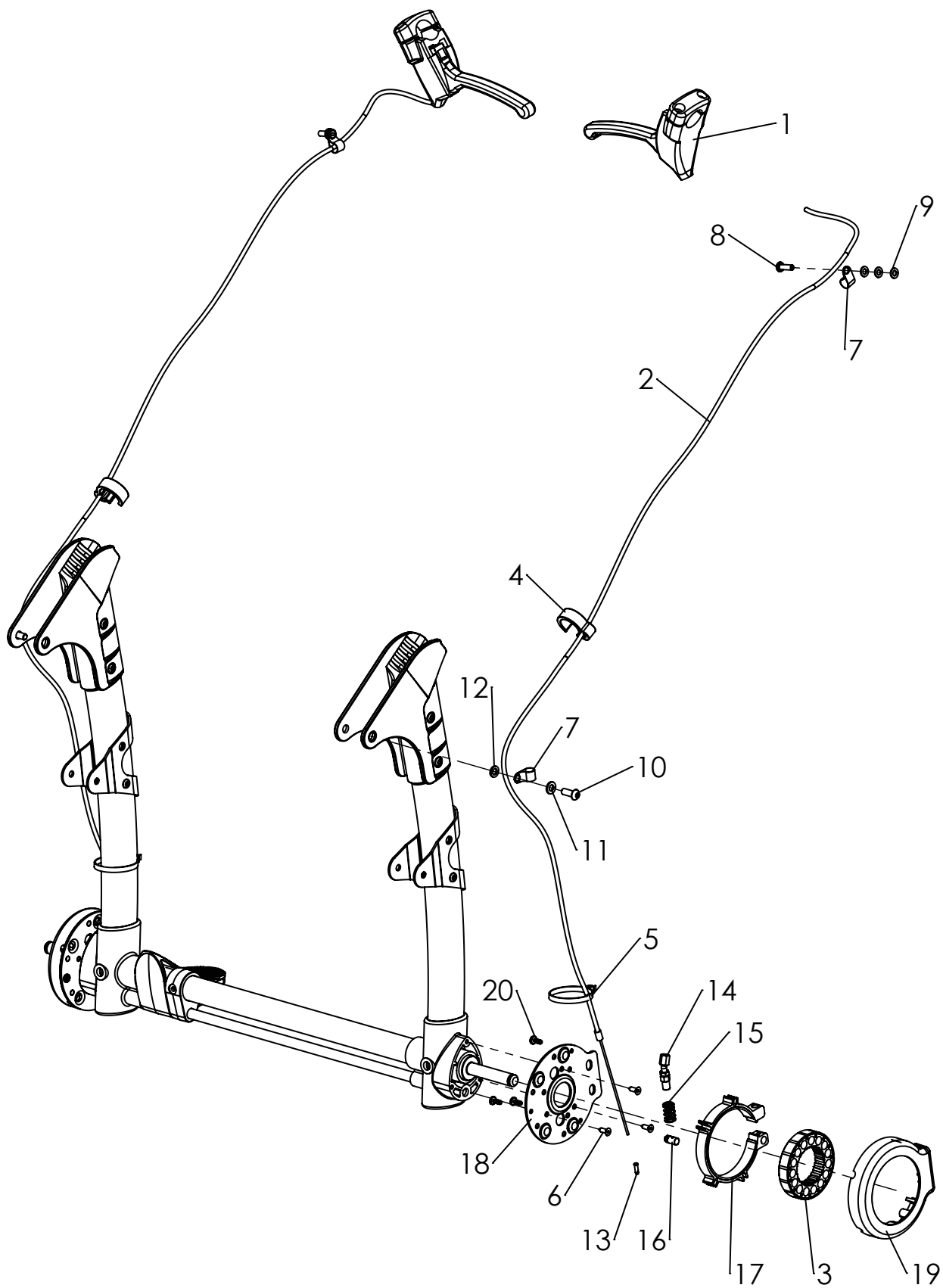

Spare parts


Pos	Article number	Articel description	Quantity	Unit
1-20	3231-7201	Attendant brake, two hand use,mounted at the Push bar	1	Pc.
	3231-7202	Attendant brake, two hand use, mounted at the ratchet joints	1	Pc.
	3235-7201	Attendant brake, two hand use,mounted at the Push bar 2XL	1	Pc.
	3235-7202	Attendant brake, two hand use, mounted at the ratchet joints 2XL	1	Pc.
1	1209-0021	Hand brake 80° no lock 22 mm (323X-7202)	2	Pc.
	1209-0022	Hand brake 36° no lock 22 mm (323X-7201)	2	Pc.
2	1209-0001	Break wire set 1,5m cable	2	Pc.
3	1204-0005	Brake disc with aluminium-ring	2	Pc.
4	1149-0001	Cable clip D 25	2	Pc.
5	1089-0004	Cable tie, black, 5x180	2	Pc.
6	1039-0006	Tapping screw with countersunk head	6	Pc.
7	1149-0003	Cable clip d 6,4 / 8 mm, black	4	Pc.
8	1062-0516	Blind rivet F5x16	2	Pc.
9	1051-0005	Washer A5,3 zp DIN/ISO 125 A5,3	6	Pc.
10	1029-0616	button head socket screw M6x16	2	Pc.
11	1050-0006	Washer A6,4	2	Pc.
12	1051-0006	Washer A6,4	2	Pc.
13	1209-0007	End cap aluminium	2	Pc.
14-20		Spare parts, Attendant brake		
14	3231-7217	Setscrew for cable, 160° angle	1	Pc.
15	1131-0073	Compression spring for hub brake	1	Pc.
16	1099-0015	Screw nipple	1	Pc.
17	3231-7213	Alu-profile bracket with brake pad	1	Pc.
18	3231-7214	Base plate for hub brake	1	Pc.
19	3231-7212	Cover for hub brake	1	Pc.
20	1039-0001	Screw for plastic	3	Pc.

Spare parts




Pos	Article number	Articel description	Quantity	Unit
1-4	3230-9300-323	Canopy incl. adapter for Evo Mini and rain protection, iron grey	1	Set
	3201-9300-323	Canopy for size 1 and 2, incl. adapter and rain protection, iron gray	1	Set
	3235-9300-323	Canopy size 2XL, incl. adapter and rain protection, iron gray	1	Set
1	7000-0007-323	Canopy with rain cover, gray	1	Pc.
2	7000-0008	Rain cover for canopy, mini, size 1 and 2	1	Pc.
	7000-0037	Rain cover for canopy, size 2XL	1	Pc.
3	7000-0004	Ratchet joint for canopy with fasteners	2	Pc.
4	3201-9310E	Canopy adapter with fastener complete (for size 1, 2 and 2XL)	1	Set
	3230-9310E	Canopy adapter for mini, with fastener complete	1	Set

Spare parts

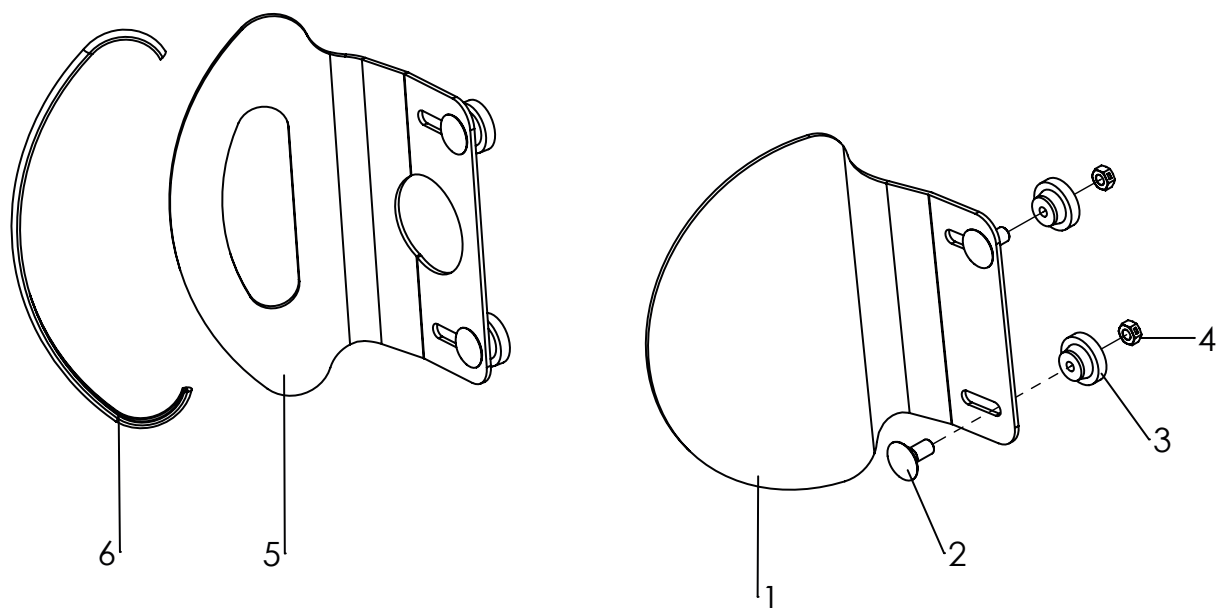


Pos	Article number	Articel description	Quantity	Unit
1-4	3230-7602-xxx	Headrest pads size Mini	1	Pair
	3201-7600-xxx	Headrest pads size 1	1	Pair
	3203-7600-xxx	Headrest pads size 2	1	Pair
1	3230-7631-032	Headrest pads mini, black dull matt	2	Pc.
	3201-7610-032	Headrest pads size 1 black dull matt	2	Pc.
	3203-7610-032	Headrest pads size 2 black dull matt	2	Pc.
2	1026-0520	Mushroom head square neck bolt M5x20	4	Pc.
3	1124-0500	Knurled nut PA black M5	4	Pc.
4	1049-0066	BILOC-nut VM5	4	Pc.
2-6	3230-7601-xxx	Bingo-Evo headrest pads low version size Mini	1	Pair
	3231-7600-xxx	Bingo-Evo headrest pads low version size 1	1	Pair
	3233-7600-xxx	Bingo-Evo headrest pads low version size 2	1	Pair
5	3230-7611-032	Headrest pads mini, black dull matt	2	Pc.
	3231-7611-032	Headrest pad low profile size 1 black dull matt	2	Pc.
	3233-7611-032	Headrest pad low profile size 2 black dull matt	2	Pc.
6	1144-0001	KS-Profil PVC, Mini 2x 0,19m	0,4	mtr
		KS-Profil PVC, size1 2x 0,22m	0,4	mtr
		KS-Profil PVC, size 2 2x 0,26m	0,5	mtr

Spare parts

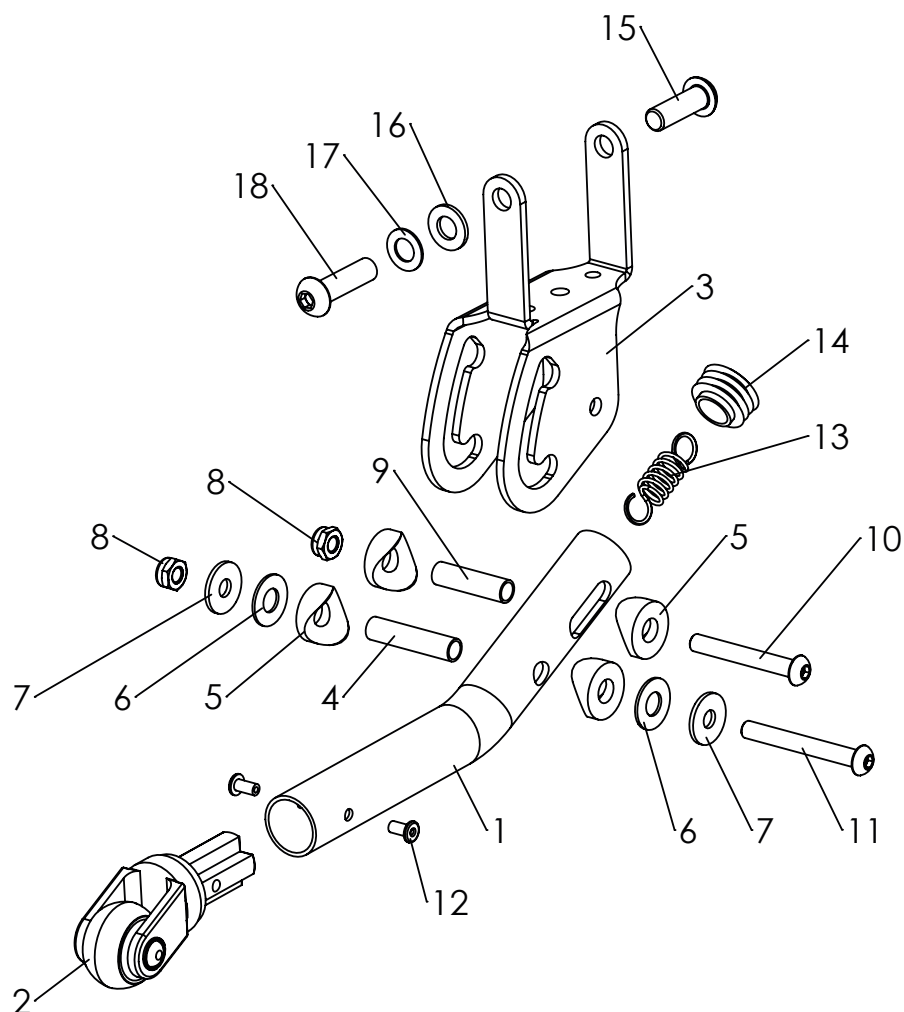


Pos	Article number	Articel description	Quantity	Unit
1-18	3231-7100	Evo anti tip left and right	1	Pc.
1	3231-7121-007	Black anti-tilt tube	1	Pc.
2	1249-0007	Insertion fork and castor	1	Pc.
3	3231-7111-032	Blunt matt black anti-tilt bracket	1	Pc.
4	1114-0047	Bushing steel zp 8x0,9x35	1	Pc.
5	1111-0012	Saddle washer PA black 24x8.4x3	4	Pc.
6	1113-0003	Washer PA 18x8,4x1 black	2	Pc.
7	1055-0006	Washer zp A6,4 zp DIN/ISO 9021 A6,4	2	Pc.
8	1042-0006	Hexagon nut self-locking zp DIN/ISO 985 M6	2	Pc.
9	1114-0037	Bushing steel zp 8x0,9x29,5	1	Pc.
10	1029-0640	Button head socket screw zp DIN/ISO 7380 M6x40	1	Pc.
11	1029-0645	Button head socket screw zp DIN/ISO 7380 M6x45	1	Pc.
12	1062-0408	Blind rivet St/St DIN/ISO 7337 F4x8	2	Pc.
13	1132-0001	Tension spring (1.25 x 11.20 x 30.20)	1	Pc.
14	1141-0008	Ribbed insert SFL 22x0,8-3,0 black	1	Pc.
15	1029-0825	Button head socket screw zp DIN/ISO 7380 M8x25	1	Pc.
16	1051-0008	Washer A8,4 zp DIN/ISO 125 A8,4	1	Pc.
17	1052-0008	Spring washer B8 zp DIN/ISO 137 B8	1	Pc.
18	1029-0830	Button head socket screw zp DIN/ISO 7380 M8x30	1	Pc.

Spare parts

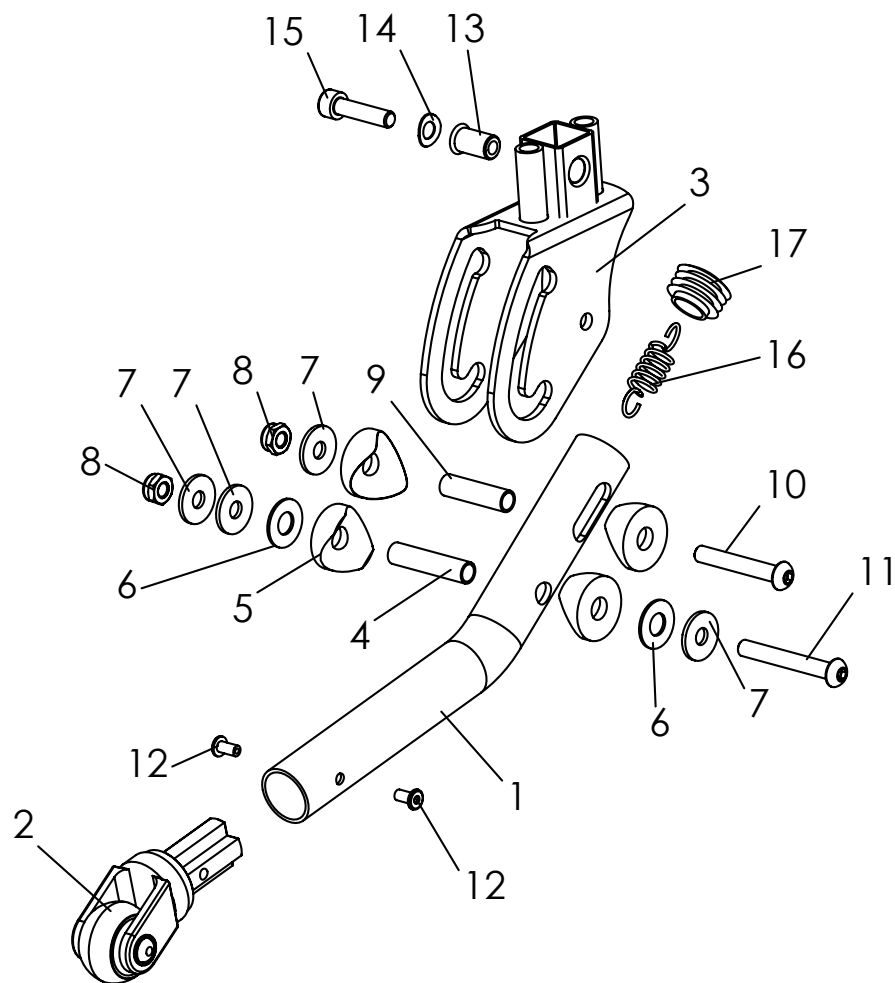


Pos	Article number	Articel description	Quantity	Unit
1-17	3234-7100	EVO-CROSS anti tip left and right	1	Pc.
1	3231-7151-007	Anti-tilt tube	1	Pc.
2	1249-0007	Insertion fork and castor	1	Pc.
3	3231-7140-052	Insert fork and roller	1	Pc.
4	1114-0047	Bushing steel	1	Pc.
5	1111-0012	Saddle washer PA	4	Pc.
6	1113-0003	Washer PA	2	Pc.
7	1055-0006	Washer A6,4	4	Pc.
8	1042-0006	Hexagon nut self-locking M6	2	Pc.
9	1114-0037	Bushing steel	1	Pc.
10	1029-0645	Button head socket screw M6x45	1	Pc.
11	1029-0650	Button head socket screw M6x50	1	Pc.
12	1062-0408	Blind rivet F4x8	2	Pc.
13	1049-0061	Rivet nut steel countersunk head 90°	1	Pc.
14	1052-0006	Spring washer B6	1	Pc.
15	1021-0625	Hexagon socket head cap screw M6x25	1	Pc.
16	1132-0001	Tension spring	1	Pc.
17	1141-0008	Ribbed insert	1	Pc.

Spare parts

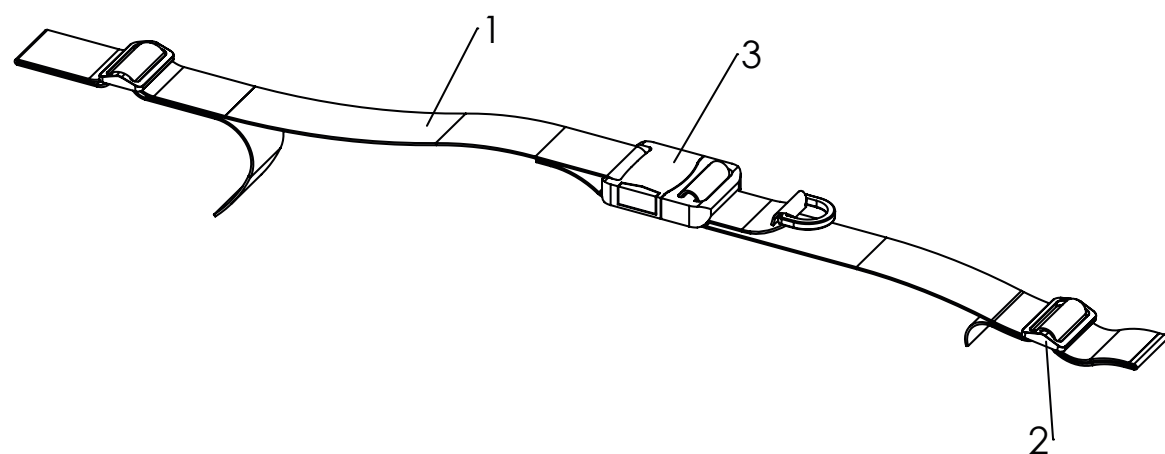


Pos	Article number	Articel description	Quantity	Unit
1-3	3201-6100	Hip belt	1	Pc.
2	1530-0003	Buckle TG 40/N	2	Pc.
3	1531-0002	Buckle SS40/HC	1	Pc.

Spare parts

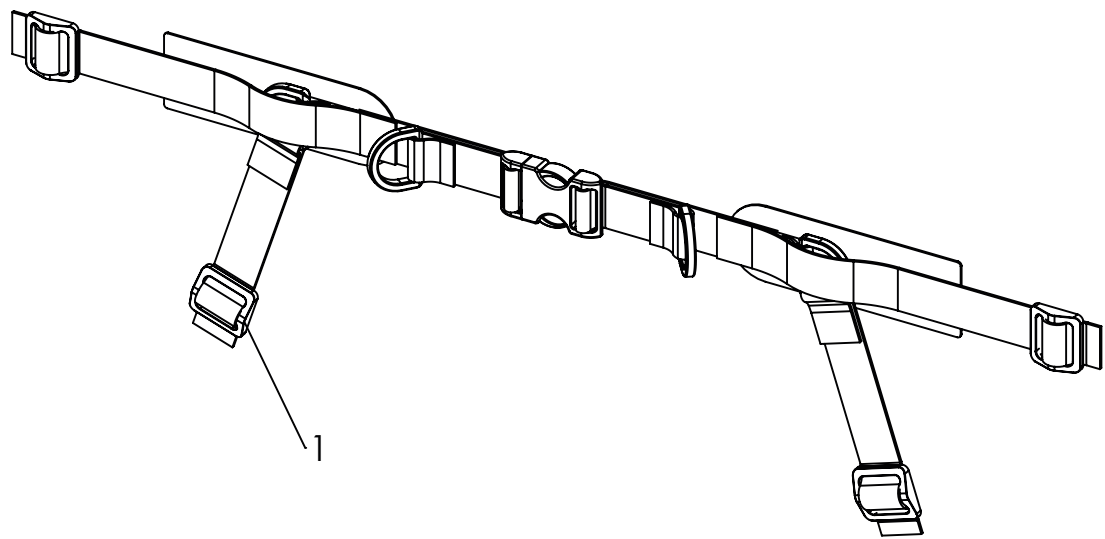


Pos	Article number	Articel description	Quantity	Unit
1-1	3231-6701	Bingo-Evo 4-point lap belt size XS (for seat width 18 - 24)	1	Set
	3231-6702	Bingo-Evo 4-point lap belt size S (for seat width 22 - 30)	1	Set
	3231-6703	Bingo-Evo 4-point lap belt size M (for seat width 28 - 36)	1	Set
	3231-6704	Bingo-Evo 4-point lap belt size L (for seat width 34 - 42)	1	Set
1	1530-0004	Buckle	4	Pc.

Spare parts

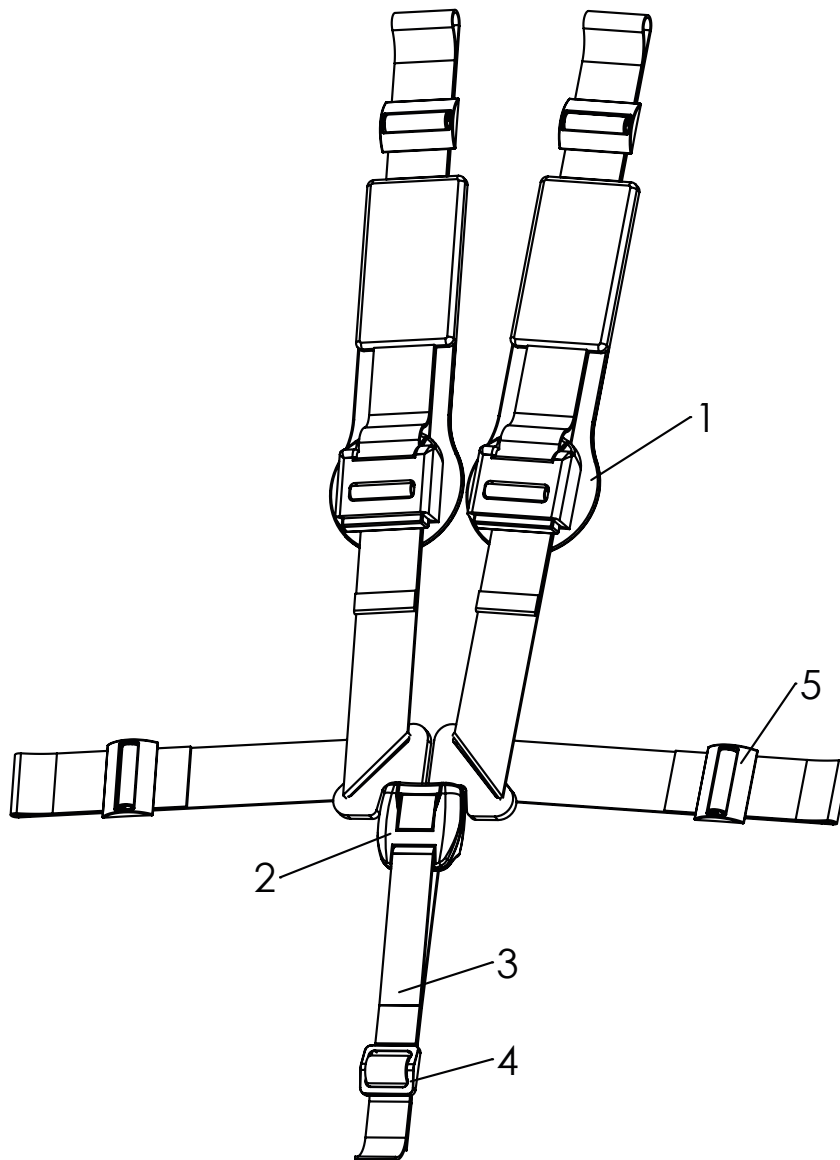


Pos	Article number	Articel description	Quantity	Unit
1-5	3201-6200-346	Five-point harness	1	Set
1	3201-6210-346	Shoulder pads, black	1	Set
2	7000-0025	Five point harness	1	Pc.
3	1512-0013	PP Belt 25x600 black twice sewn	1	Pc.
4	1530-0004	Buckle TG 25/N	1	Pc.
5	1530-0006	Buckle ETG 40	4	Pc.

Spare parts

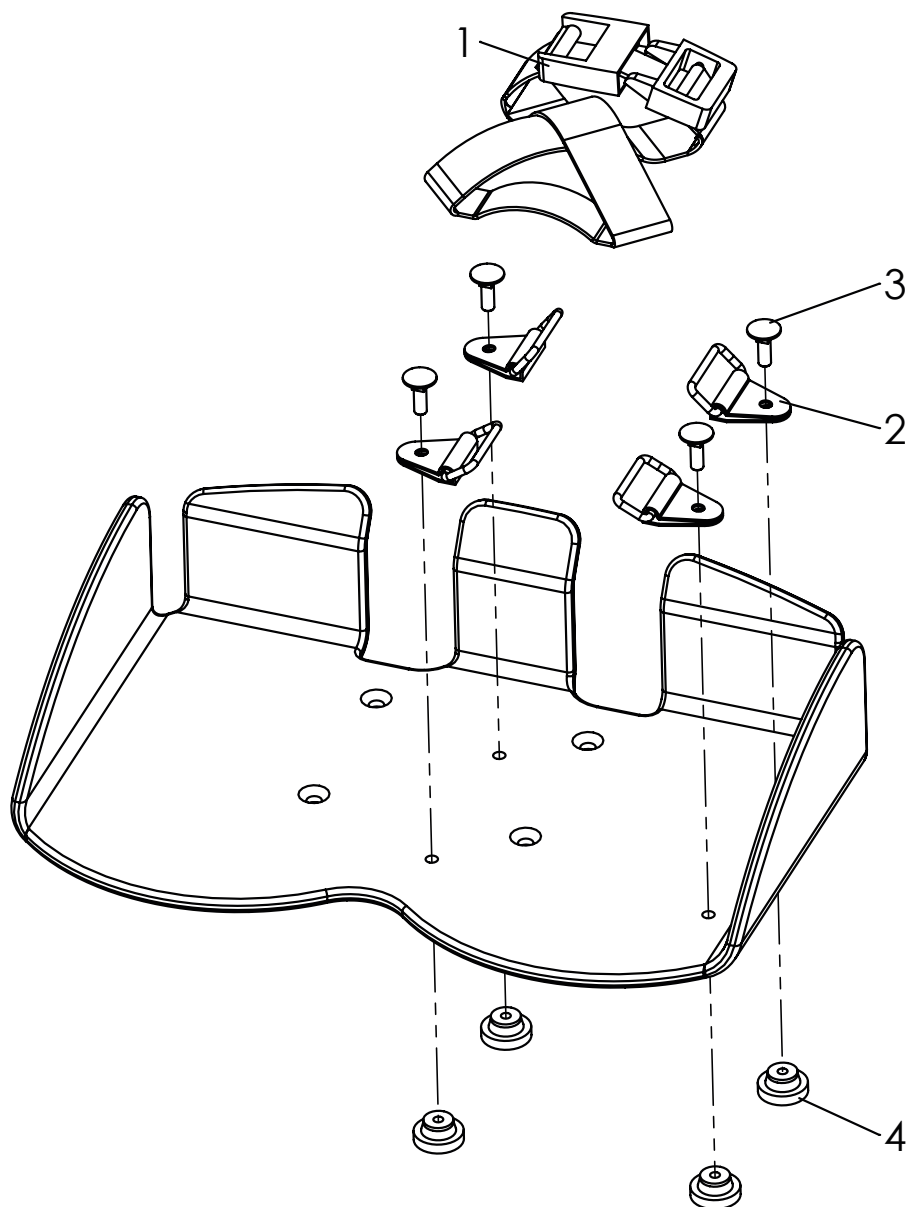

Pos	Article number	Articel description	Quantity	Unit
1-4	3201-6300	Foot straps	1	Set
1	1531-0001	Buckle QCR 725 STD	2	Pc.
2	1532-0003	Rectangle-ring with plastic tab NY 140 NP/25 black	8	Pc.
3	1026-0516	Mushroom head square neck bolt zp DIN/ISO 603 M5x16	8	Pc.
4	1124-0500	Knurled nut PA black M5	8	Pc.

Spare parts

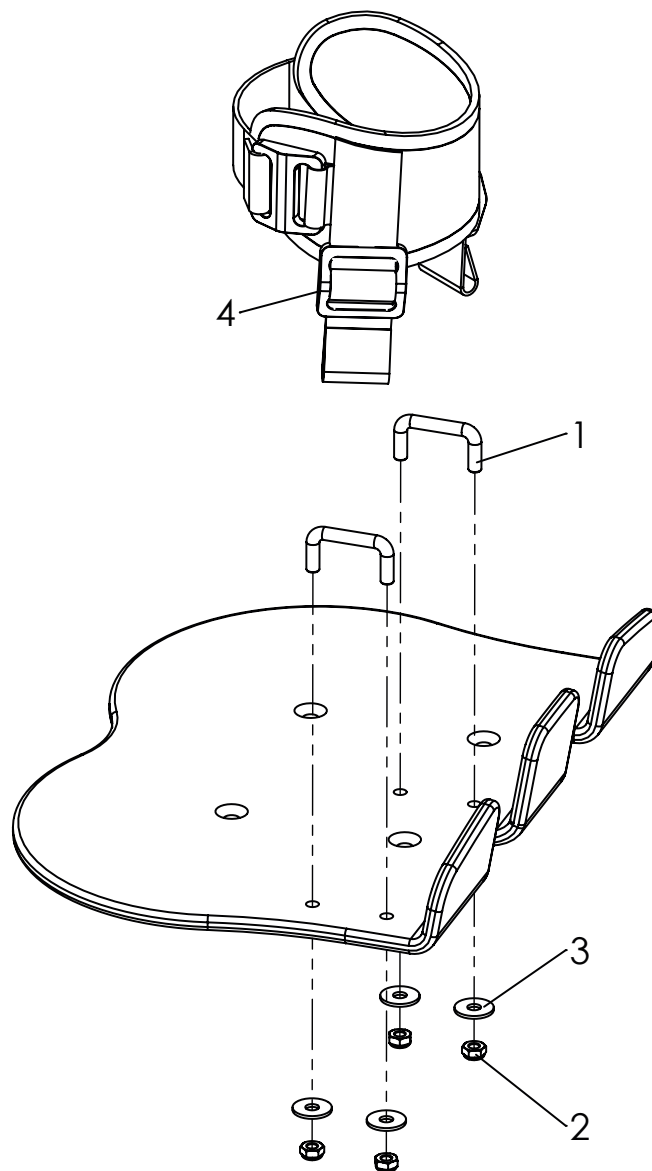


Pos	Article number	Articel description	Quantity	Unit
1-4	3201-6400	Ankle huggers size 1 (diameter approx. 4 - 6.5 cm)	1	Set
	3203-6400	Ankle huggers size 2 (diameter approx. 6 - 8.5 cm)	1	Set
1	3201-6410	Foot strap anchor Gr.1	4	Pc.
	3203-6410	Foot strap anchor size 2	4	Pc.
2	1042-0005	Hexagon nut self-locking zp DIN/ISO 985 M5	8	Pc.
3	1055-0005	Washer A5,3	8	Pc.
4	1530-0004	Buckle TG 25/N	4	Pc.

Spare parts




Pos	Article number	Articel description	Quantity	Unit
1-6	3230-5300-346	Groin strap Bingo-Evo size mini, black	1	Set
	3231-5300-346	Groin strap Bingo-Evo size 1, black	1	Set
	3233-5300-346	Groin strap Bingo-Evo size 2, black	1	Set
1	1531-0001	Buckle	2	Pc.
2	1530-0004	Buckle	2	Pc.
3	1532-0003	Rectangle-ring with plastic tab black	2	Pc.
4	1030-0512	Button head socket screw M5x12	2	Pc.
5	1055-0005	Washer A5,3	2	Pc.
6	1042-0005	Hexagon nut self-locking M5	2	Pc.

Spare parts

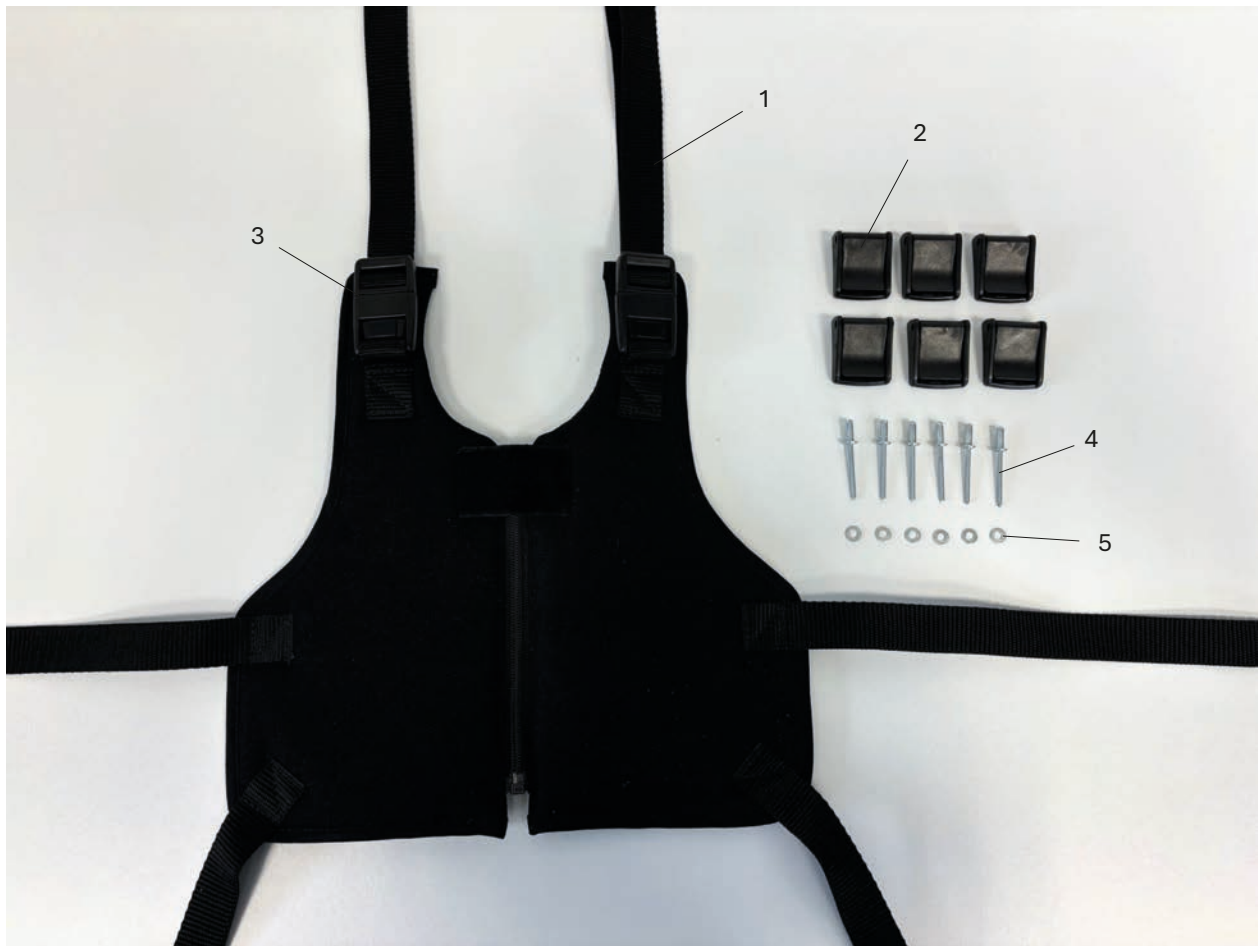


Pos	Article number	Articel description	Quantity	Unit
1-5	3201-5800-346	Fixation jacket size 0, black	1	Set
	3201-5400-346	Fixation jacket size 1, black	1	Set
	3201-5500-346	Fixation jacket size 2, black	1	Set
	3203-5400-346	Fixation jacket size 3, black	1	Set
1	1511-2560	PE-strap 25x600 black	6	Pc.
2	1530-0002	Cam buckle with drilling 4,2	6	Pc.
3	1531-0001	Buckle	2	Pc.
4	1062-0410	Blind rivet F4x10	6	Pc.
5	1051-0004	Washer A4,3	6	Pc.

Spare parts

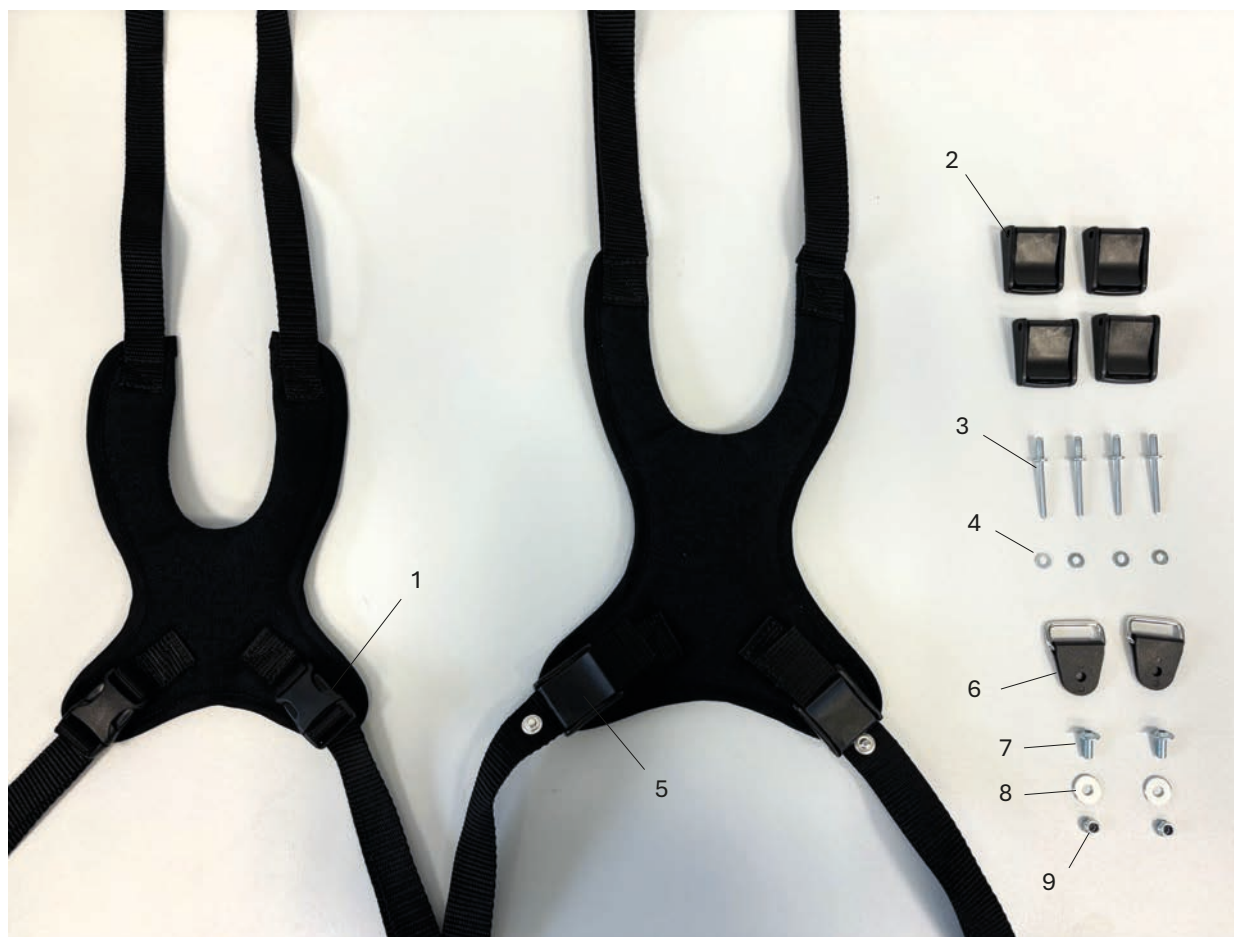

Pos	Article number	Articel description	Quantity	Unit
1-4	3230-5600-346	Chest-shoulder harness size mini, black	1	Set
2-9	3201-5600-346	Chest-shoulder harness size 1, black	1	Set
	3201-5700-346	Chest-shoulder harness size 2, black	1	Set
	3203-5600-346	Chest-shoulder harness size 3, black	1	Set
1	1531-0006	Side Release Buckles	2	Pc.
2	1530-0002	Cam buckle with drilling 4,2	4	Pc.
3	1062-0410	Blind rivet F4x10	4	Pc.
4	1051-0004	Washer A4,3	4	Pc.
5	1530-0001	Cam buckle	2	Pc.
6	1532-0003	Rectangle-ring with plastic tab black	2	Pc.
7	1030-0514	Button head socket screw with flange M5x14	2	Pc.
8	1055-0005	Washer A5,3	2	Pc.
9	1044-0005	Hexagonal domed cap nut self-locking M5	2	Pc.

Spare parts

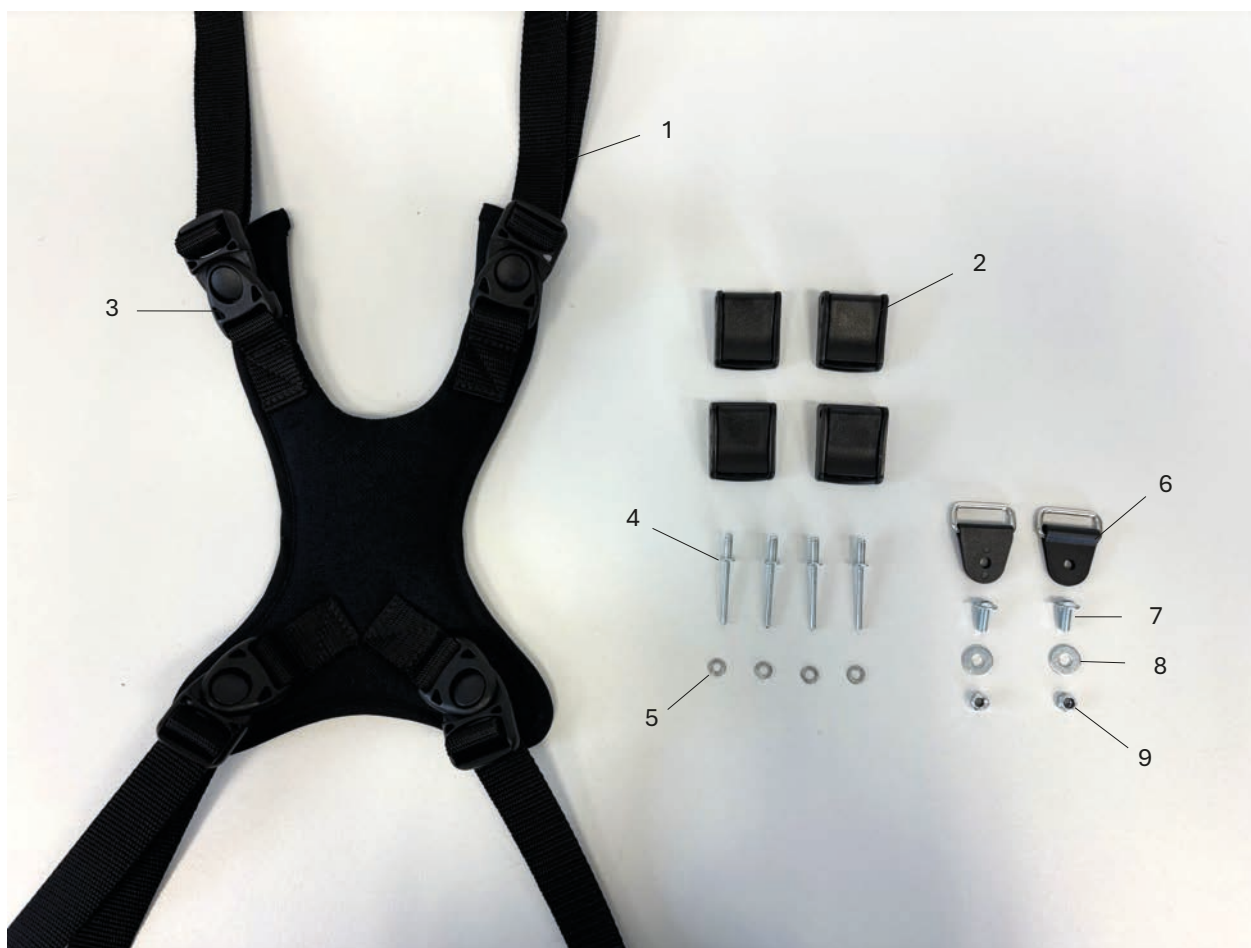


Pos	Article number	Articel description	Quantity	Unit
1-9	3201-5601-346	Chest-shoulder harness black size 1 with 4 buckels	1	Set
	3201-5701-346	Chest-shoulder harness black size 2 with 4 buckels	1	Set
	3203-5601-346	Chest-shoulder harness black size 3 with 4 buckels	1	Set
1	1511-2560	PE-strap 25x600 black	4	Pc.
2	1530-0002	Cam buckle with drilling 4,2	4	Pc.
3	1531-0001	Buckle	4	Pc.
4	1062-0410	Blind rivet F4x10	4	Pc.
5	1051-0004	Washer A4,3	4	Pc.
6	1532-0003	Rectangle-ring with plastic tab black	2	Pc.
7	1030-0514	Button head socket screw M5x14	2	Pc.
8	1055-0005	Washer A5,3	2	Pc.
9	1044-0005	Hexagonal domed cap nut M5	2	Pc.

Spare parts




Pos	Article number	Articel description	Quantity	Unit
1-1	3232-9401E	Bingo-Evo "soft" accessories bag size 1, max. 3kg	1	Pc.
	3234-9401E	Bingo-Evo "soft" accessories bag size 2, max. 3kg	1	Pc.
1	3201-9461	Pocket welt size 1 (390x20x3)	2	Pc.
	3203-9461	Pocket welt size 2 (440 x 20 x 3)	2	Pc.

Spare parts

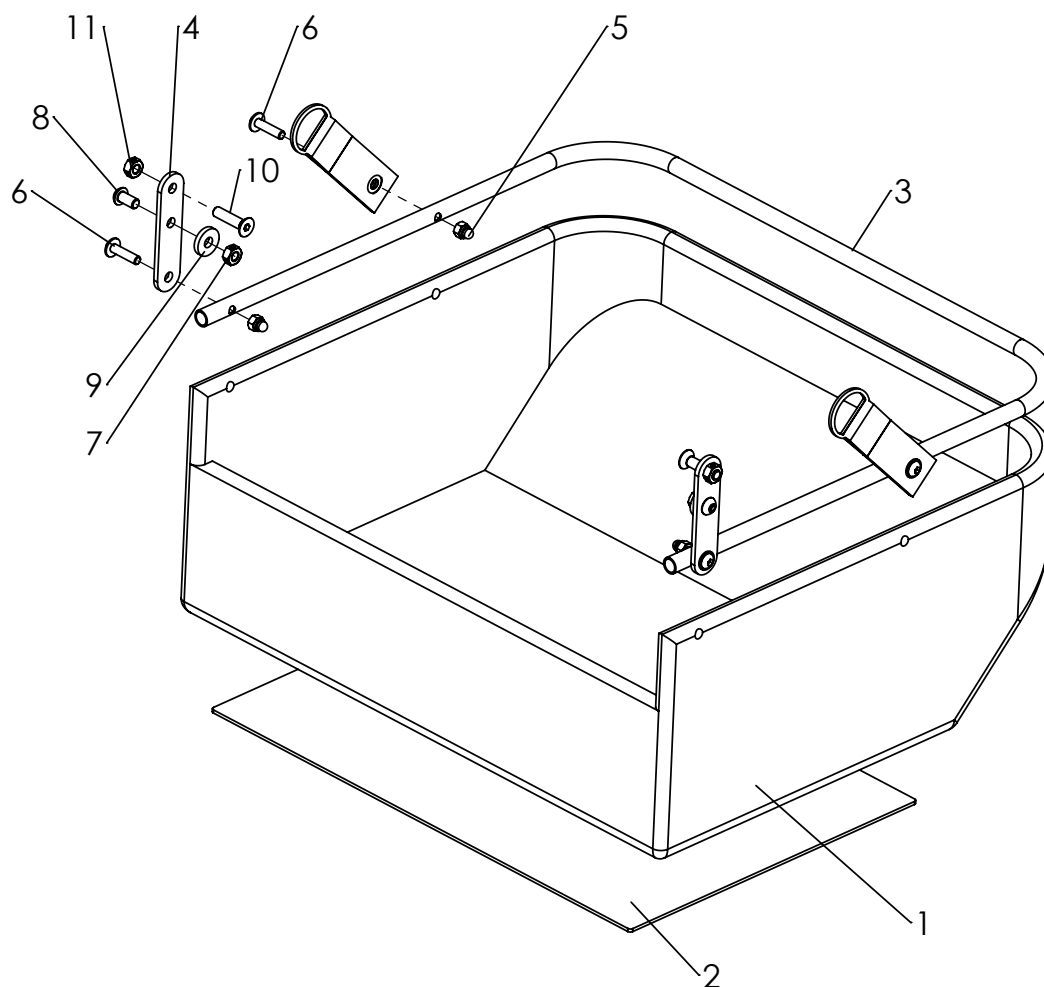


Pos	Article number	Articel description	Quantity	Unit
1-11	3231-9402E	large accessories bag (max. 8kg) size 1	1	Pc.
	3233-9402E	large accessories bag (max. 8kg) size 2	1	Pc.
1	3231-9450	accessories bag EVO size 1	1	Pc.
	3233-9450	accessories bag EVO size 2	1	Pc.
2	3201-9440	PE cut-out, pocket insert 230x340x2.0 mm	1	Pc.
3	3231-9410-001	storage bag holder size 1	1	Pc.
	3233-9410-001	storage bag holder size 2	1	Pc.
4	3231-9411-032	Retaining strap for pocket clip	2	Pc.
5	1044-0005	Hexagonal domed cap nut self-locking M5	4	Pc.
6	1030-0520	button head socket screw with flange M5x20	4	Pc.
7	1049-0068	BILOC-nut VM6	4	Pc.
8	1029-0612	button head socket screw M6x12	2	Pc.
9	1059-0003	Washer 6,4	2	Pc.
10	1024-0625	Hexagon socket countersunk head screw M6x25	2	Pc.
11	1042-0006	Hexagon nut self-locking M6	2	Pc.

Spare parts

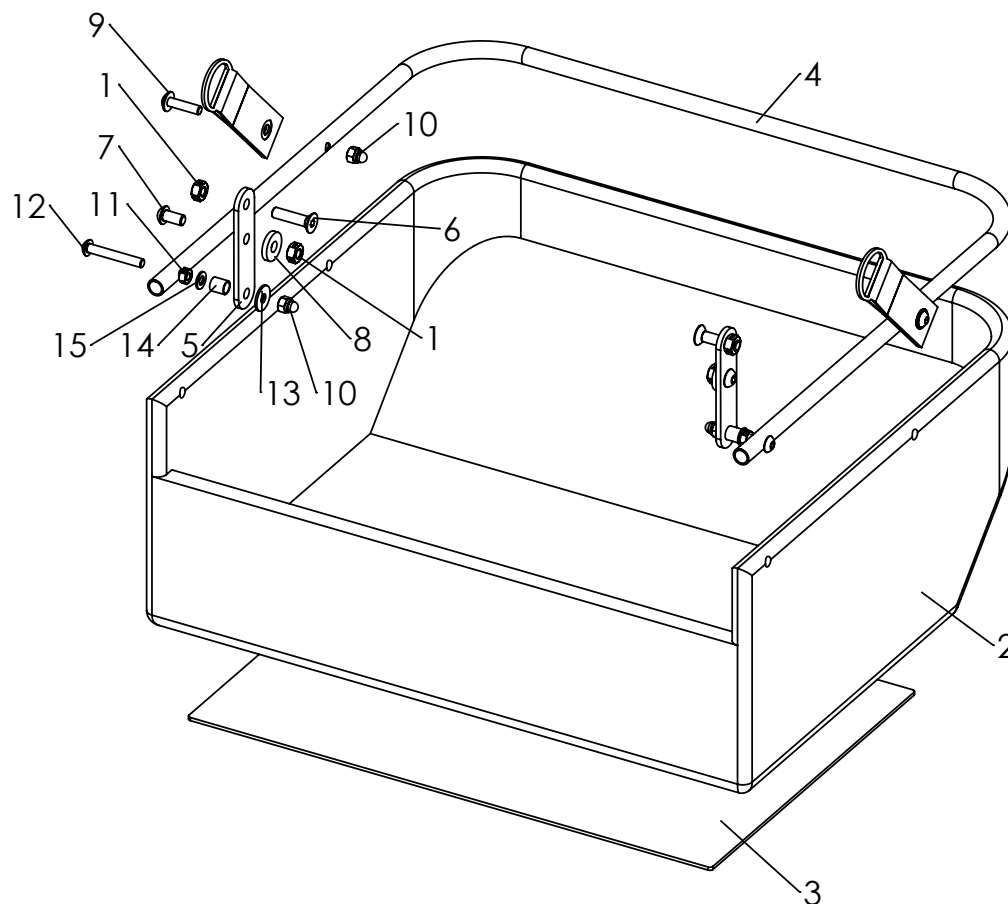


Pos	Article number	Articel description	Quantity	Unit
1-15	3230-9402E	Bingo-Evo large accessories bag (max. 8kg) for "mini"	1	Pc.
1	1049-0070-033	BILOC-nut VM6	4	Pc.
2	3231-9450	accessories bag EVO size 1	1	Pc.
3	3201-9440	PE cut-out, pocket insert 230x340x2.0 mm	1	Pc.
4	3231-9410-001	storage bag holder size 1 Bingo Evo, black	1	Pc.
5	3231-9411-032	Retaining strap for pocket clip	2	Pc.
6	1024-0625	Hexagon socket countersunk head screw M6x25	2	Pc.
7	1029-0612	button head socket screw M6x12	2	Pc.
8	1059-0003	Washer 6,4	2	Pc.
9	1030-0520	button head socket screw with flange M5x20	4	Pc.
10	1044-0005	Hexagonal domed cap nut self-locking M5	4	Pc.
11	1049-0066	BILOC-nut VM5	2	Pc.
12	1029-0535	button head socket screw M5x35	2	Pc.
13	1055-0005	Washer A5,3	2	Pc.
14	1114-0055	Bushing 8x0,8x8,7	2	Pc.
15	1051-0005	Washer A5,3	2	Pc.

Spare parts

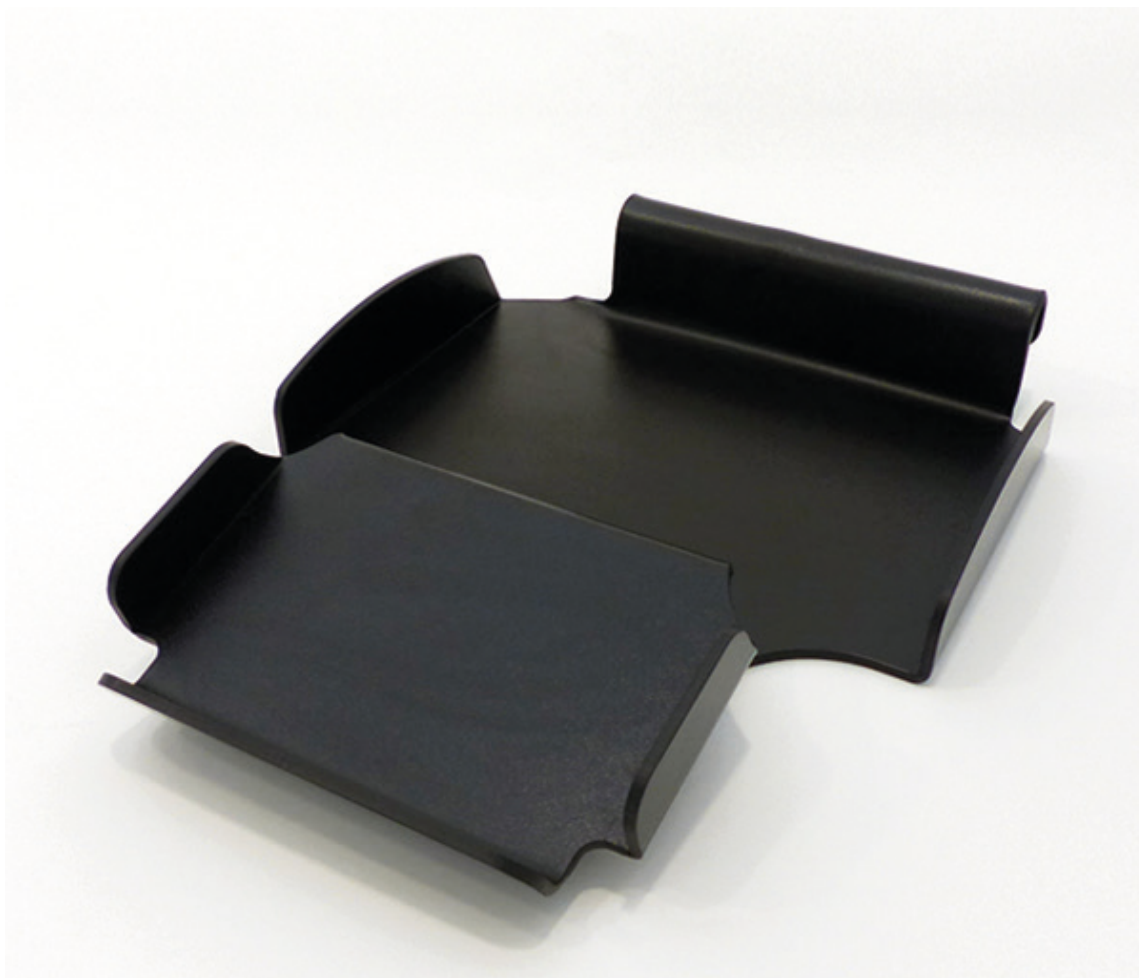


Pos	Article number	Articel description	Quantity	Unit
1-1	3202-4100E	Tray for additional equipment size 1 (max. 15kg)	1	Pc.
	3204-4100E	Tray for additional equipment size 2 (max. 15kg)	1	Pc.
1	3202-4120	Shelf for additional appliances size 1	1	Pc.
	3204-4120	Shelf for additional appliances size 2	1	Pc.

Spare parts

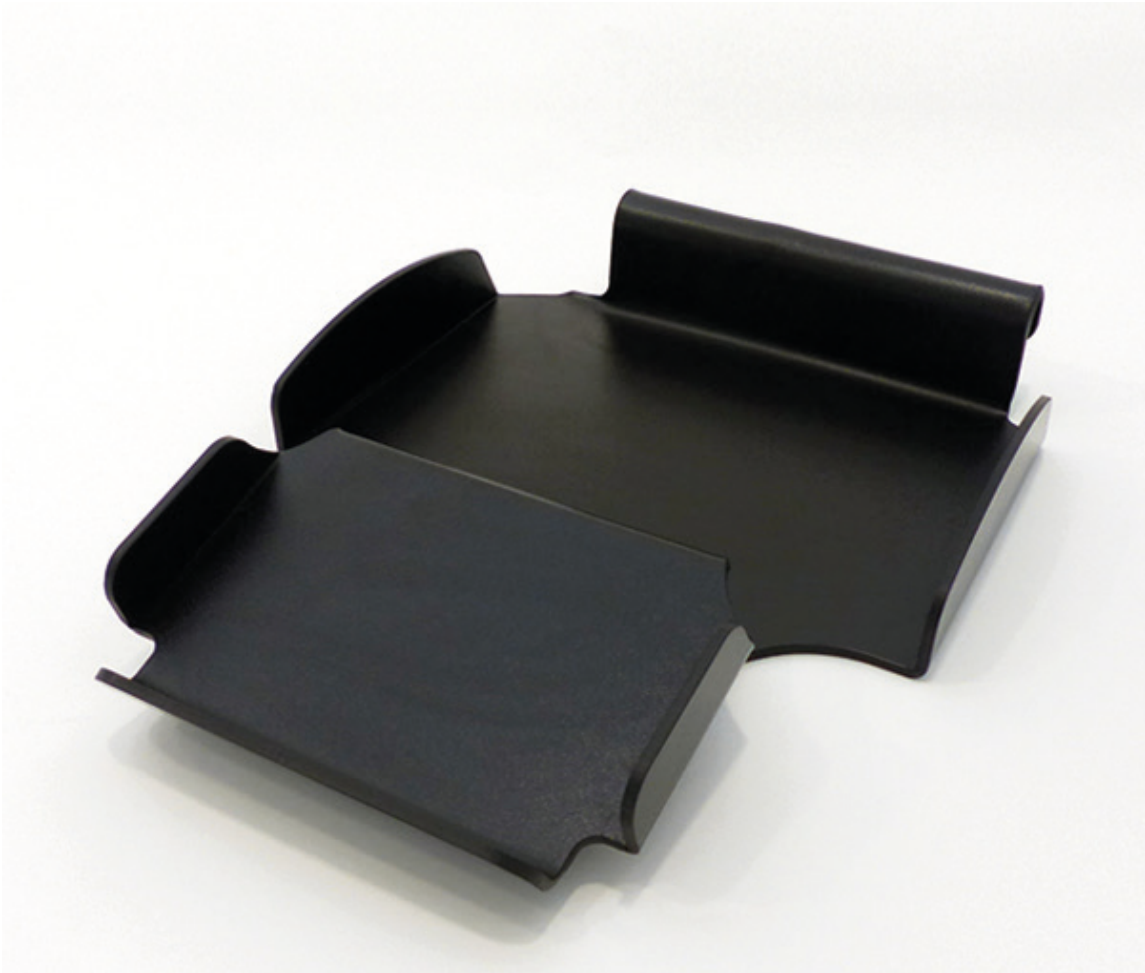


Pos	Article number	Articel description	Quantity	Unit
1-1	3238-4400E	Evo CROSS tray for additional equipment size 1 (max. 25kg)	1	Pc.
	3239-4400E	Evo CROSS tray for additional equipment size 2 (max. 25kg)	1	Pc.
1	3232-4420-015	Aluminum shelf for additional equipment EVO-CROSS size 1, FS black	1	Pc.
	3234-4420-015	Aluminum shelf for additional equipment EVO-CROSS size 2, FS black	1	Pc.

Spare parts

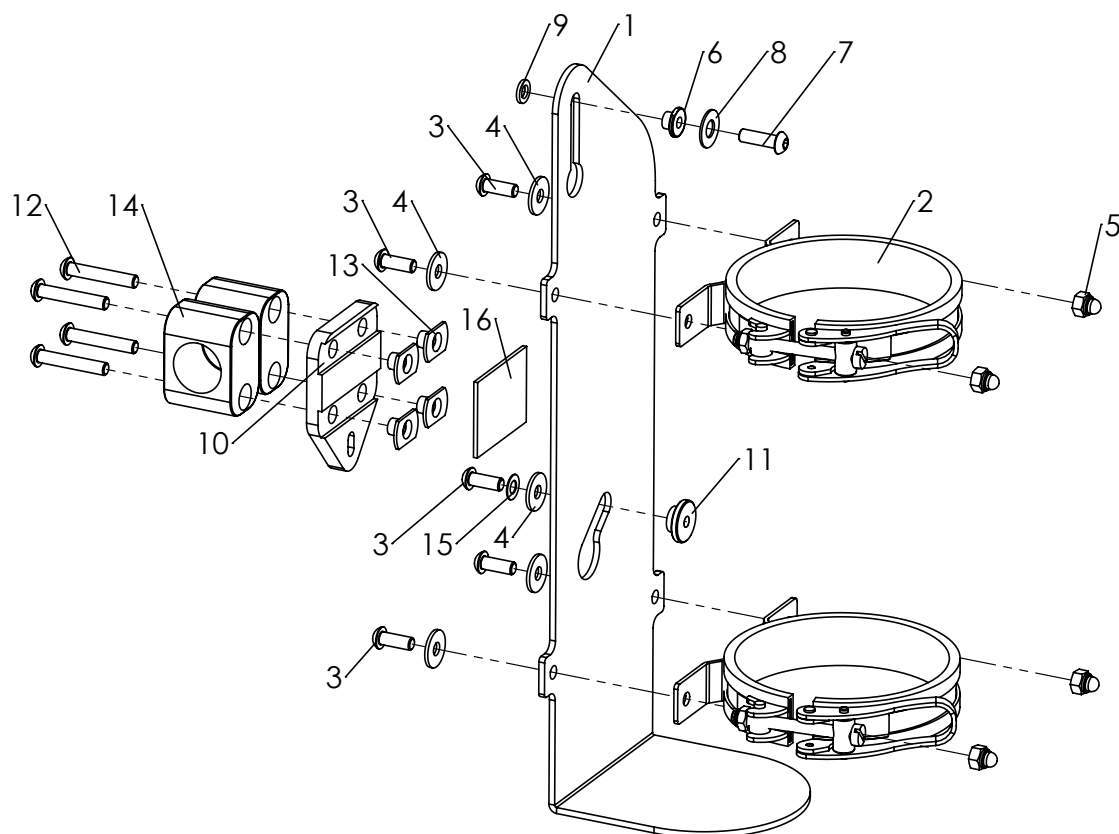


Pos	Article number	Articel description	Quantity	Unit
1-16	3231-4200	Oxygen bottle holder (d=100 mm) long, right hand attachment	1	Pc.
	3231-4204	Oxygen bottle holder (d=100 mm) long, left hand attachment	1	Pc.
	3231-4201	Oxygen bottle holder short version (d=100 mm), right hand attachment	1	Pc.
	3231-4205	Oxygen bottle holder short version (d=100 mm), left hand attachment	1	Pc.
1	3201-4211-015	bottle holder, structural black, right	1	Pc.
	3201-4213-015	bottle holder, structural black, left	1	Pc.
	3201-4212-015	bottle holder short, FS black, right	1	Pc.
	3201-4214-015	bottle holder short, FS black, left	1	Pc.
2	7000-0049	bottle clamps (d=100 mm)	1	Pc.
3	1029-0616	button head socket screw M6x16	4	Pc.
4	1055-0006	Washer A6,4	5	Pc.
5	1044-0006	Hexagonal domed cap nut self-locking M6	5	Pc.
6	3201-7360	belt safety knob	1	Pc.
7	1029-0620	button head socket screw M6x20	1	Pc.
8	1113-0003	Washer PA 18x8,4x1 black	1	Pc.
9	1050-0006	Washer A6,4	1	Pc.
10	3231-4214	adapter plate, black	1	Pc.
11	3201-4220	Shoulder axle M6	1	Pc.
12	1029-0635	button head socket screw M6x35	4	Pc.
13	1049-0052	T-nut 22x14xM6x8,5	4	Pc.
14	1116-0002	tube clamp PA black D25	2	Pc.
15	1052-0006	Spring washer B6	1	Pc.
16	1500-0013	velcro self-adhesive 38 mm	0,1	mtr.

Spare parts

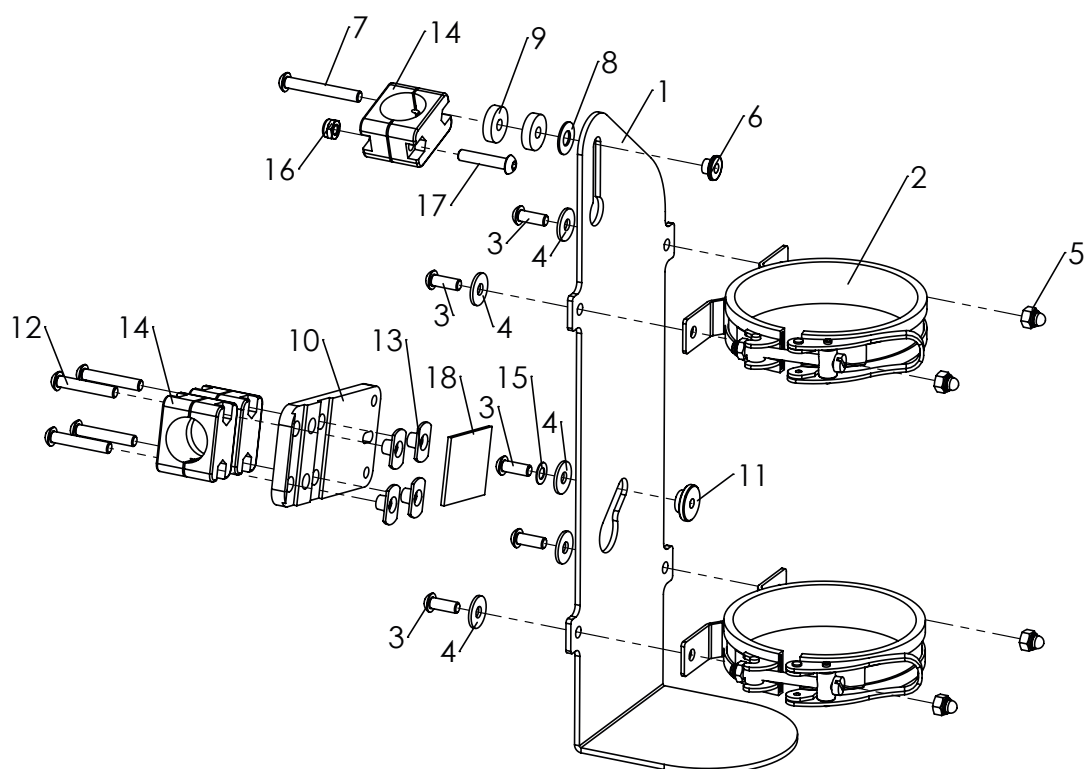


Pos	Article number	Articel description	Quantity	Unit
1-18	3231-4200_CROSS	Oxygen bottle holder (d=100 mm) long, right hand attachment	1	Pc.
	3231-4204_CROSS	Oxygen bottle holder (d=100 mm) long, left hand attachment	1	Pc.
	3231-4201_CROSS	Oxygen bottle holder (d=100 mm) short, right hand attachment	1	Pc.
	3231-4205_CROSS	Oxygen bottle holder (d=100 mm) short, left hand attachment	1	Pc.
1	3201-4211-015	bottle holder long right, structural black	1	Pc.
	3201-4213-015	bottle holder long left, structural black	1	Pc.
	3201-4212-015	bottle holder short right, structural black	1	Pc.
	3201-4214-015	bottle holder short left, structural black	1	Pc.
2	7000-0049	bottle clamps (d=100 mm)	1	Pc.
3	1029-0616	button head socket screw M6x16	4	Pc.
4	1055-0006	Washer A6,4	4	Pc.
5	1044-0006	Hexagonal domed cap nut self-locking M6	4	Pc.
6	3201-7360	belt safety knob	1	Pc.
7	1029-0645	button head socket screw M6x45	1	Pc.
8	1113-0003	Washer PA 18x8,4x1 black	1	Pc.
9	3274-7314	Spacer bushing 20 x 6.2 x 5	1	Pc.
10	3231-4215	adapter plate, black	1	Pc.
11	3201-4220	Shoulder axle M6	1	Pc.
12	1029-0635	button head socket screw M6x35	4	Pc.
13	1049-0052	T-nut L-769 zp 22x14xM6x8,5	4	Pc.
14	1116-0009	tube clamp PA black D20	3	Pc.
15	1052-0006	Spring washer B6	1	Pc.
16	1042-0006	Hexagon nut self-locking M6	1	Pc.
17	1029-0630	button head socket screw M6x30	1	Pc.
18	1500-0013	velcro self-adhesive 38 mm	0,1	mtr.

Spare parts

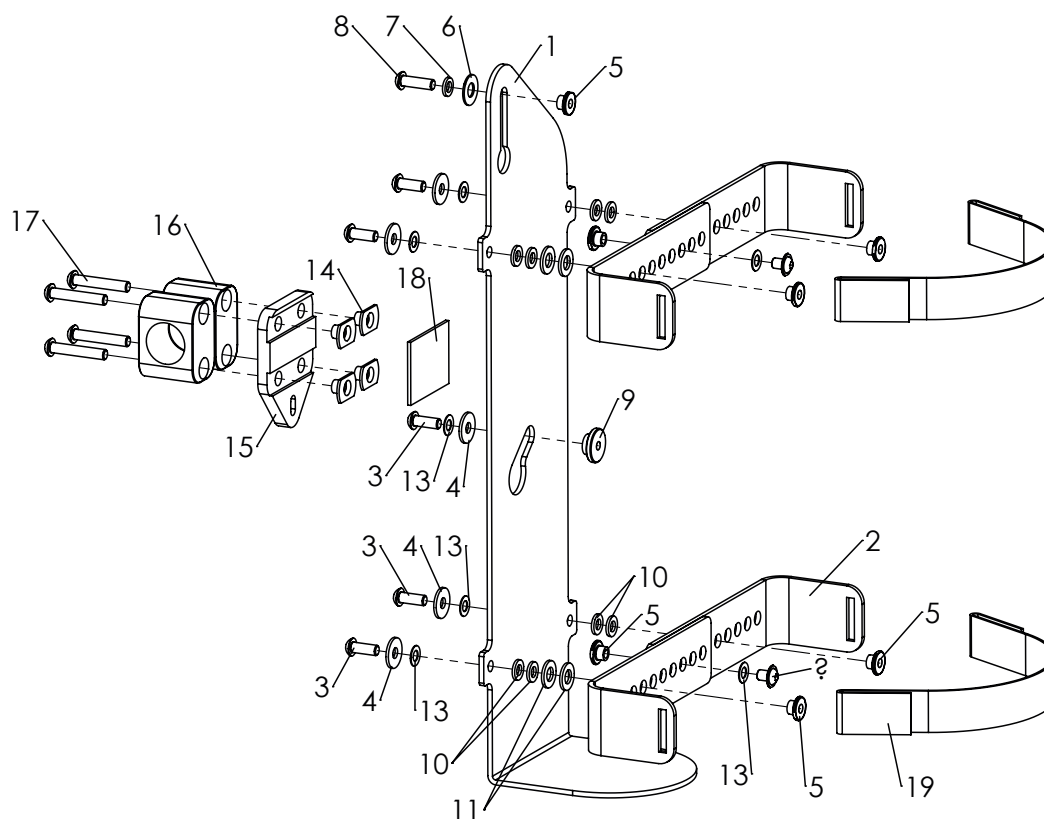


Pos	Article number	Articel description	Quantity	Unit
1-19	3231-4300	Oxygen bottle holder long, adjustable angle holders for contoured casing, right hand attachment	1	Pc.
	3231-4301	Oxygen bottle holder short, adjustable angle holders for contoured casing, right hand attachment	1	Pc.
	3231-4302	Oxygen bottle holder long, adjustable angle holders for contoured casing, left hand attachment	1	Pc.
	3231-4303	Oxygen bottle holder short, adjustable angle holders for contoured casing, left hand attachment	1	Pc.
1	3201-4211-015	Bottle holder vertical long right	1	Pc.
	3201-4212-015	Bottle holder vertical short right	1	Pc.
	3201-4213-015	Bottle holder vertical long left	1	Pc.
	3201-4214-015	Bottle holder vertical short left	1	Pc.
2	3231-4251	Angle for adjustable holder (SA-0225)	4	Pc.
3	1029-0616	Button head socket screw zp DIN/ISO 7380 M6x16	5	Pc.
4	1055-0006	Washer zp A6,4 zp DIN/ISO 9021 A6,4	5	Pc.
5	3201-7360	Belt safety knob	7	Pc.
6	1113-0003	Washer PA 18x8,4x1 black	1	Pc.
7	1050-0006	Washer A6,4 zp DIN/ISO 433 A6,4	1	Pc.
8	1029-0620	Button head socket screw zp DIN/ISO 7380 M6x20	1	Pc.
9	3201-4220	Shoulder axle M6	1	Pc.
10	1051-0006	Washer A6,4 zp DIN/ISO 125 A6,4	12	Pc.
11	1051-0008	Washer A8,4 zp DIN/ISO 125 A8,4	4	Pc.
12	1030-0608	Button head socket screw with flange zp M6x8	2	Pc.
13	1052-0006	Spring washer B6 zp DIN/ISO 137 B6	6	Pc.
14	1049-0052	T-nut L-769 zp 22x14xM6x8,5	4	Pc.
15	3231-4214	Adapter plate, black	1	Pc.
16	1116-0002	Tube clamp PA black D25	2	Pc.
17	1029-0635	Button head socket screw zp DIN/ISO 7380 M6x35	4	Pc.
18	1500-0013	Velcro self-adhesive 38 mm	0,1	mtr.
19	1500-0014	Back to Back on roll black 25 mm	2	Pc.

Spare parts

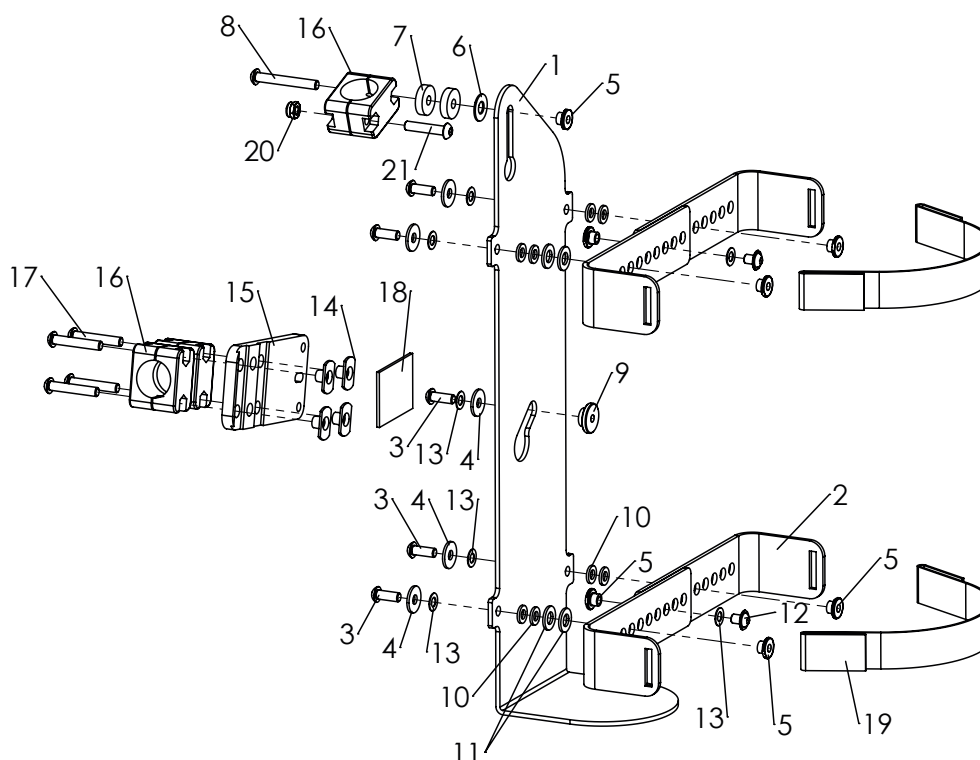

Pos	Article number	Articel description	Quantity	Unit
1-19	3231-4300_CROSS	Oxygen bottle holder long, adjustable angle holders for contoured casing, right hand attachment	1	Pc.
	3231-4301_CROSS	Oxygen bottle holder short, adjustable angle holders for contoured casing, right hand attachment	1	Pc.
	3231-4302_CROSS	Oxygen bottle holder long, adjustable angle holders for contoured casing, left hand attachment	1	Pc.
	3231-4303_CROSS	Oxygen bottle holder short, adjustable angle holders for contoured casing, left hand attachment	1	Pc.
1	3201-4211-015	bottle holder long right, structural black	1	Pc.
	3201-4213-015	bottle holder long left, structural black	1	Pc.
	3201-4212-015	bottle holder short right, structural black	1	Pc.
	3201-4214-015	bottle holder short left , structural black	1	Pc.
2	3231-4251	Bracket for adjustable holder (SA-0225)	1	Pc.
3	1029-0616	button head socket screw M6x16	5	Pc.
4	1055-0006	Washer A6,4	5	Pc.
5	3201-7360	belt safety knob	7	Pc.
6	1113-0003	Washer PA 18x8,4x1 black	1	Pc.
7	3274-7314	Spacer bushing 20 x 6.2 x 5	1	Pc.
8	1029-0645	button head socket screw M6x45	1	Pc.
9	3201-4220	Shoulder axle M6	1	Pc.
10	1051-0006	Washer A6,4	12	Pc.
11	1051-0008	Washer A8,4	4	Pc.
12	1030-0608	button head socket screw with flange M6x8	2	Pc.
13	1052-0006	Spring washer B6	6	Pc.
14	1049-0052	T-nut L-769 22x14xM6x8,5	4	Pc.
15	3231-4215	adapter plate, black	1	Pc.
16	1116-0009	Locking lever M10	3	Pc.
17	1029-0635	button head socket screw M6x35	4	Pc.
18	1500-0013	velcro self-adhesive 38 mm	0,1	Pc.
19	1500-0014	Back to Back on roll black 25 mm	2	Pc.
20	1042-0006	Hexagon nut self-locking M6	1	Pc.
21	1029-0630	button head socket screw M6x30	1	Pc.

Spare parts

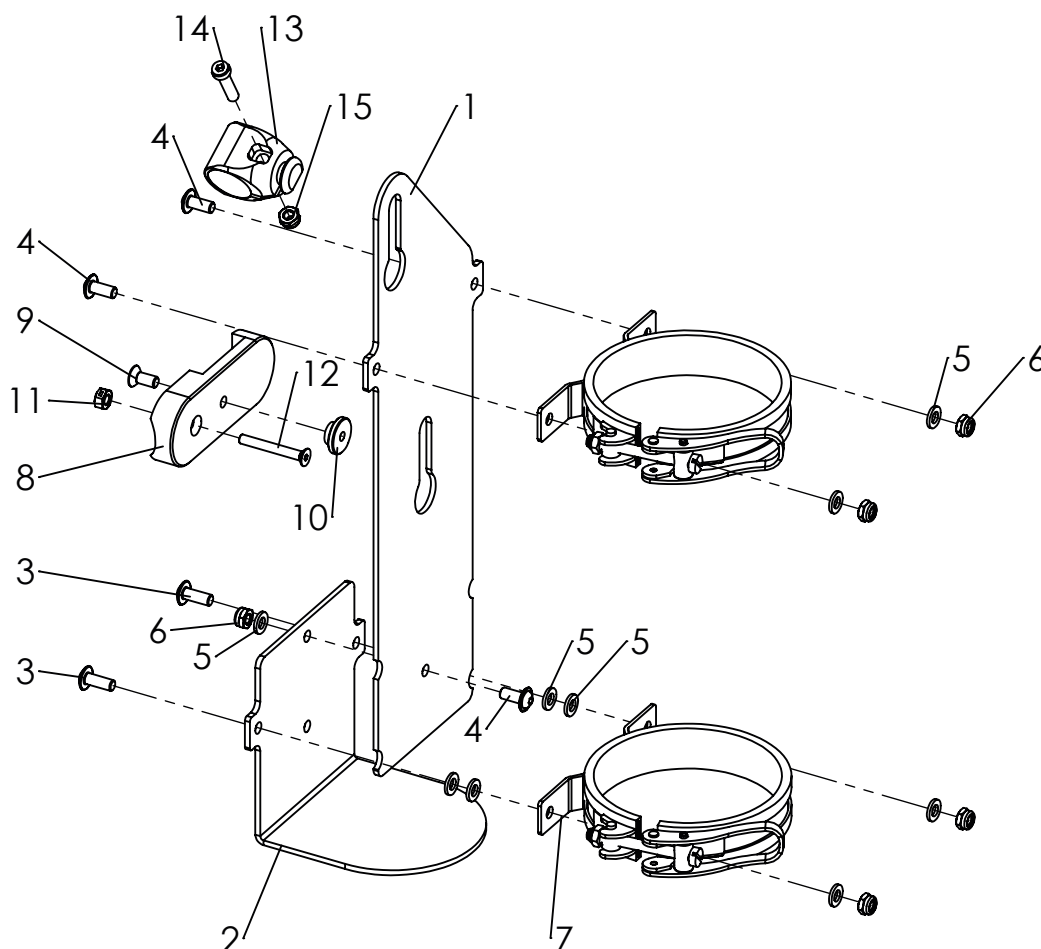

Pos	Article number	Articel description	Quantity	Unit
1-15	3030-4200	Oxygen bottle holder (d=100 mm) long, right hand attachment	1	Pc.
	3030-4204	Oxygen bottle holder (d=100 mm) long, left hand attachment	1	Pc.
	3030-4201	Oxygen bottle holder short vers. (d=100 mm),right hand attachment	1	Pc.
	3030-4205	Oxygen bottle holder short vers. (d=100 mm), left hand attachment	1	Pc.
1	3032-4211	Bottle holder	1	Pc.
2	3032-4212	Bottle holder Support bracket	1	Pc.
3	1030-0616	Button head socket screw with flange M6x16	2	Pc.
4	1030-0614	Button head socket screw with flange M6x14	3	Pc.
5	1051-0006	Washer A6,4	9	Pc.
6	1042-0006	Hexagon nut self-locking M6	5	Pc.
7	7000-0049	Bottle clamps (d=100 mm)	2	Pc.
8	3032-4213	Base plate, black	1	Pc.
9	1024-5002	Hexagon socket countersunk head screw M6x14 with clamp-tight	1	Pc.
10	3201-4220	Shoulder bushing M6	1	Pc.
11	1049-0066	BILOC-nut VM5	1	Pc.
12	1024-0545	Hexagon socket countersunk head screw M5x40	1	Pc.
13	3032-4215	Upper mount black	1	Pc.
14	1022-0625	Hexagon socket head cap screw M6x25	1	Pc.
15	1042-0006	Hexagon nut self-locking black M6	1	Pc.

HOGGI GmbH
Eulerstraße 27 • 56235 Ransbach-Baumbach • Deutschland
Telefon: +49 (0) 2623 / 92 499-0
E-mail: info@hoggi.de • www.hoggi.de
