
HOGGI[®]



SWINGBO VTi XL

Spare parts catalogue

en

Index

1 Model identification and basic equipment

1.1 Model identification	4
1.2 Basic equipment	4

2 Assemblies

2.1 Assemblée overview	5
------------------------	---

3 Spare parts

3.1 Base frame	6
3.2 Central unit	7
3.3 Pressure plates	8
3.4 Cross tube	9
3.5 Anti tip	10
3.6 Tip assist	11
3.7 Seat adapter	12
3.8 Tilt adjustment	13
3.9 Seat frame	14
3.10 Seat panel	15
3.11 Seat cushion	16-17
3.12 Contoured seat cushion	18
3.13 Hip support	19
3.14 Hip support SW44	20
3.15 Straight hip pads	21
3.14 Abduction block	22-23
3.15 Push bar, detachable	24-25
3.16 Push handle with handle grip	26-27
3.17 Push handle adapter	28
3.18 Push bar, foldable and angle adjustable	29-30
3.19 Push handles, foldable	31-32
3.20 Push bar, foldable	33-34
3.20 Cross bail	35
3.21 Push bar upper part	36-37
3.22 Push bar „wide“ upper part	38
3.23 Fender	39
3.24 Fender with integrated brake	40
3.25 Fender aluminum	41
3.26 Brake lock lever	42
3.27 Brake	43
3.28 Locking of the integrated brake	44
3.29 Wheel fork	45
3.30 Front wheels	46
3.30 Rear wheels	47
3.31 Rear wheel with drumbrake	48-49
3.32 Hand rim	50
3.33 Closed ergonomic hand rim	51
3.34 Maxgrepp push rim	52
3.35 Push rim cover	53
3.36 Spoke guard	54
3.37 Knee lever brakes	55
3.38 HOGGI light sport brakes	56
3.39 Back panel angle and height adjustable	57
3.40 Back panel	58
3.41 Backrest base	59

Index


3 Spare parts

3.42	Cross member	60
3.43	Back panel trough shaped	61
3.44	Head rest panel	62
3.45	Universal headrest bracket	63
3.46	Head rest with mounting bracket	64
3.47	Belt routing as a T-strap	65
3.48	Chest-/thoracic support	66
3.49	Thoracic support trough shaped	67
3.50	Thoracic support angle and height adjustable	68
3.51	MDP pad holder swing away	69
3.52	MDP pad holder fixed	70
3.53	MDP trunk support	71
3.54	Hip belt	72
3.55	4-point lap belt	73
3.56	Foot rest hanger	74
3.57	Foot rest hanger angle adjustable	75
3.58	Single knee angle system	76-77
3.59	Calf upholstery suitable	78
3.60	Foot board assembly angle adjustable and foldable	79
3.61	Foot rest bracket angle adjustable, with interlocking	80
3.62	Footboard holder for single footrest	81
3.63	Foot rest	82
3.64	Heel plate	83
3.65	Ankle huggers	84
3.66	Tie down kit	85
3.67	Armrest	86
3.68	Armrest with tilt balancing	87
3.69	Therapy tray	88
3.70	Clamping unit	89
3.71	Bumper wheels	90
3.72	Conversion kit to one hand drive	91
3.72	Canopy incl. adapter and rain protection	92

Model identification

The label with the serial number is placed on the back of the axle tube.


Please give us the Serial number with every spare part order, to ensure a correct delivery.

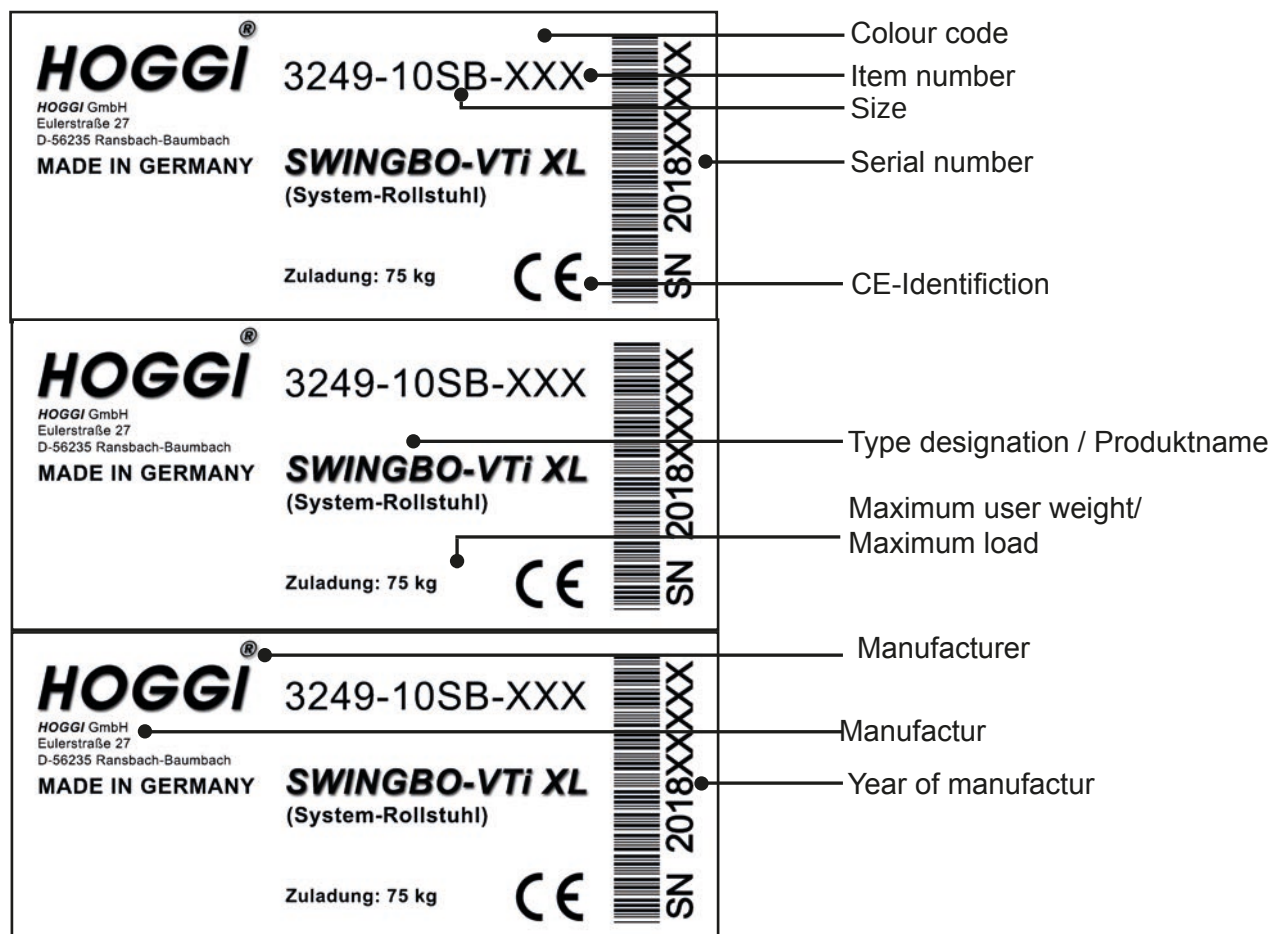

Basic configuration:


- SWINGBO VTi XL-Wheelchair with seat-system
- Front and rear wheels
- Fender with integrated break
- Push bar or Push handle
- Anti tip

Assemblie overview

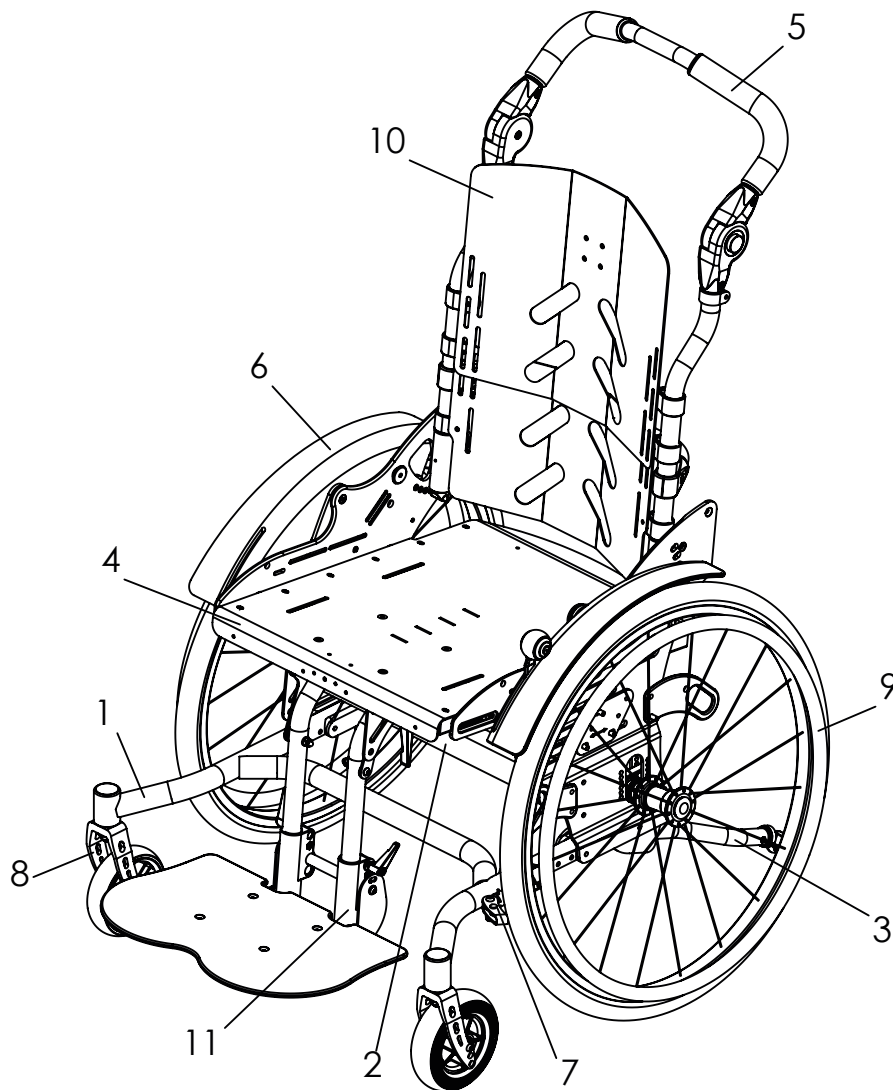


Pos	Articel description	from page
1	Base frame	6
2	Cross tube	9
3	Anti tip	10
4	Seat	14
5	Push bar	24
6	Fender	39
7	Brake	42
8	Steering wheels	45
9	Wheels and hand rim	47
10	Back	57
11	Foot rest hanger	74

Spare parts

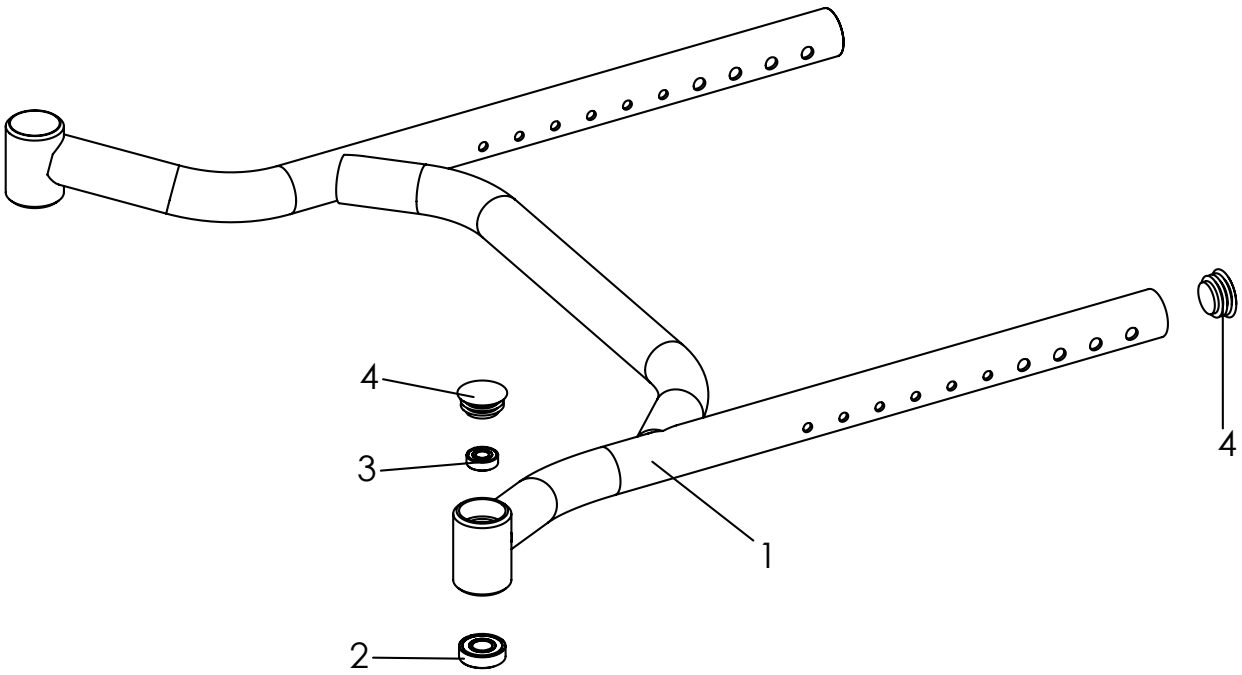



Pos	Article number	Article description	Quantity	Unit
1-4	3249-11SWE-000	Base Frame, in frame colour	1	Pc.
2	1090-0005	Grooved ball bearing D28xd12x8	2	Pc.
3	1090-0001	Grooved ball bearing D19xd8x6	2	Pc.
4	1141-0016	Ribbed insert SFL 30 30x0,8-2,5	4	Pc.

Spare parts

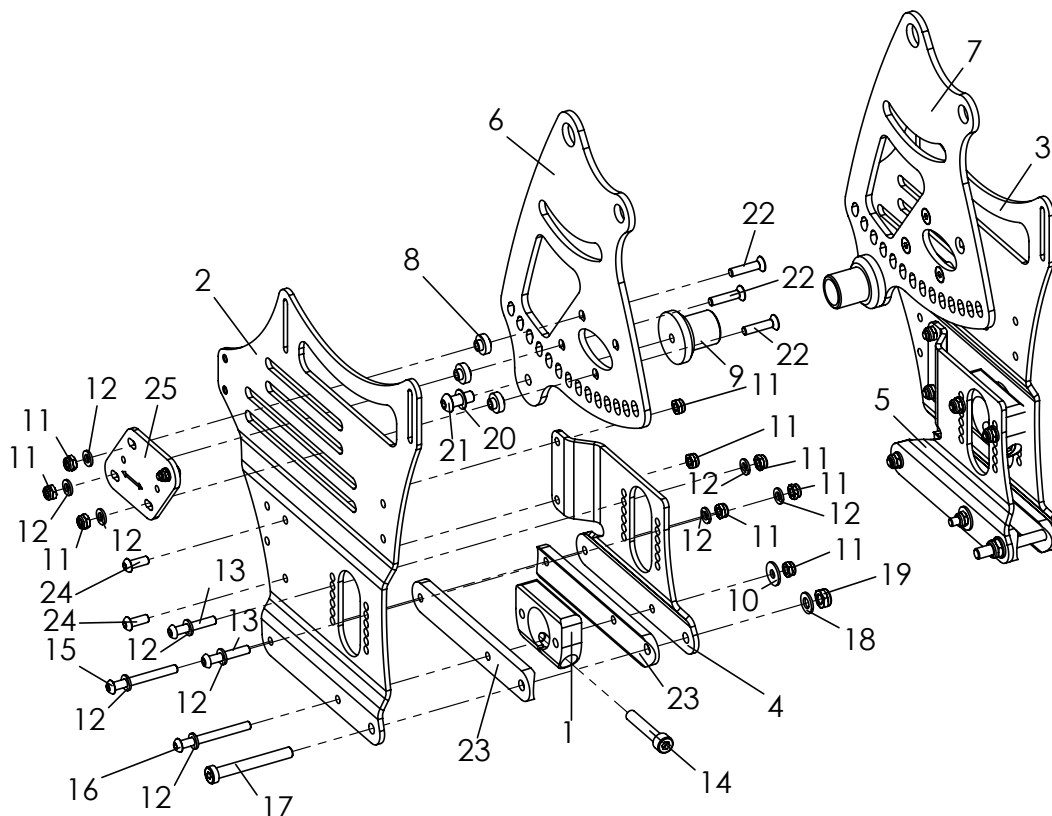


Pos	Article number	Articel description	Quantity	Unit
1-24	3249-1200E	Central unit, complete, without pressure plates	1	Set
1	3240-1340-007	Position adapter, black adodised	2	Pc.
2	3249-1251-015	Base plate left size 2, black	1	Pc.
3	3249-1252-015	Base plate right size 2, black	1	Pc.
4	3249-1255-015	Adapterplate left size 2, black	1	Pc.
5	3249-1256-015	Base plate right size 2, black	1	Pc.
6	3249-1261-015	Tilt bearing left size 2, black	1	Pc.
7	3249-1262-015	Tilt bearing right size 2, black	1	Pc.
8	3248-1264	Guiding bushing tilt bearing	6	Pc.
9	3245-1211-006	Clamping adapter, anodised	2	Pc.
10	1055-0006	Washer A6,4	2	Pc.
11	1042-0006	Hexagon nut self-locking M6	18	Pc.
12	1051-0006	Washer A6,4	20	Pc.
13	1029-0635	Button head socket screw M6x35	4	Pc.
14	1021-0845	Cylinder head screw M8x45	2	Pc.
15	1029-0655	Button head socket screw M6x55	2	Pc.
16	1029-0660	Button head socket screw M6x60	2	Pc.
17	1022-0865	Hexagon socket head cap screw M8x65	2	Pc.
18	1051-0008	Washer A8,4	2	Pc.
19	1042-0008	Hexagon nut self-locking M8	2	Pc.
20	1052-0008	Spring washer B8	2	Pc.
21	1029-0820	Button head socket screw M8x20	2	Pc.
22	1024-0627	Hexagon socket countersunk head screw M6x27	6	Pc.
23	3249-1156	Seat rail 150	4	Pc.
24	1029-0616	Button head socket screw M6x16	4	Pc.
25	3248-1265E-000	Pressure plates, in frame colour	1	Pair

Spare parts

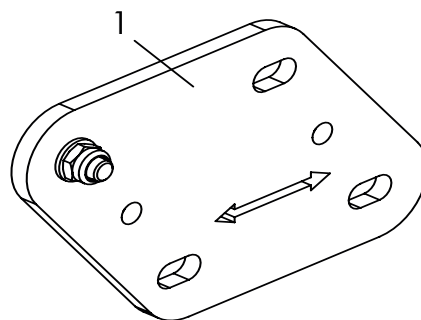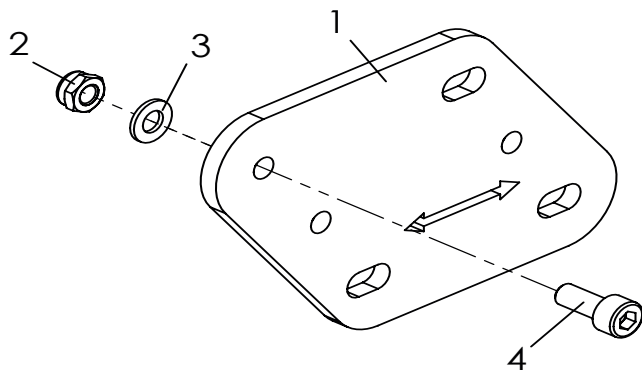

Pos	Article number	Articel description	Quantity	Unit
1-4	3248-1265E-000	Pressure plates, in frame colour	1	Pair
2	1051-0005	Washer A5,3	2	Pc.
3	1021-0512	Cylinder head screw M5x12	2	Pc.
4	1042-0005	Hexagon nut self-locking M5	2	Pc.

Spare parts

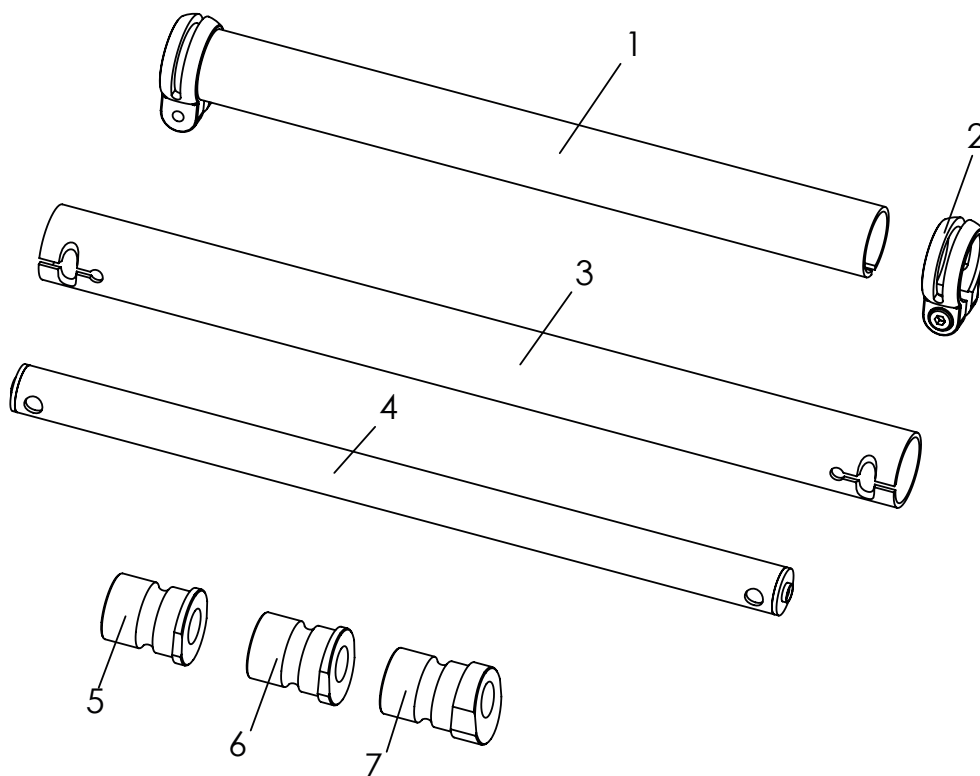


Pos	Article number	Article description	Quantity	Unit
1*	3246-12SW-007	Cross tube	1	Pc.
2	1129-0010	Saddle clamp d 29,8 mm black	2	Pc.
3**	3240-14SW-007	Axle tube	1	Pc.
4*	3247-19SW-015	Distance brace, fine structure black	1	Pc.
5	3240-1352-006	Adapter housing 6°, anodised	2	Pc.
6	3240-1362-006	Adapter housing 9°, anodised	2	Pc.
7	3240-1372-006	Adapter housing 3°, anodised	2	Pc.

1. Cross tube and distance brace must be completed with SW of Wheel chair

3. Axle tube must be added with 2 for correct SW (SW 28 + 2=30)


Spare parts

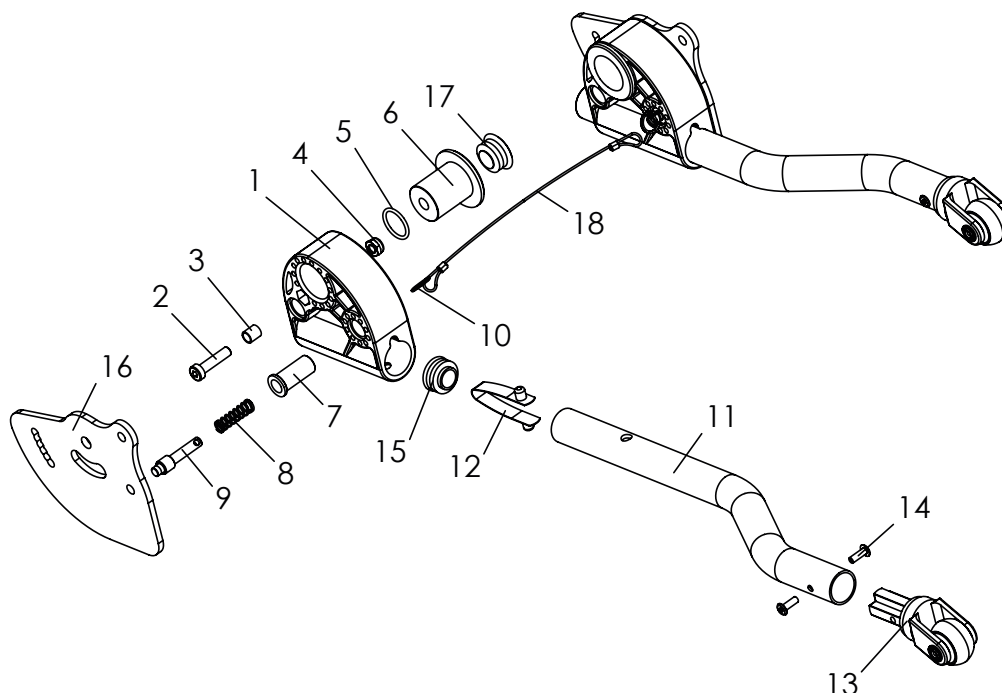

Pos	Article number	Articel description	Quantity	Unit
1-17	3248-1801E	Anti tip left	1	Pc.
	3248-1802E	Anti tip right	1	Pc.
1	3248-1821	Anti tip holder	1	Pc.
2	1022-0625	Hexagon socket head cap screw M6x25	1	Pc.
3	1114-0055	Bushing 8x0,8x8,7	1	Pc.
4	1042-0006	Hexagon nut self-locking M6	1	Pc.
5	1150-0001	O-ring NBR D18x2	1	Pc.
6	3248-1812	Bearing shaft	1	Pc.
7	3201-7520	Spring housing	1	Pc.
8	1131-0036	Compression spring D-127d 0,8x8,3x29	1	Pc.
9	3201-7530	Lock pin	1	Pc.
10	1129-0036	Key ring 16x1,6	1	Pc.
11-15	3248-1861E	SBO-VTi anti tip tube, left complete	1	Pc.
	3248-1862E	SBO-VTi anti tip tube, right complete	1	Pc.
11	3248-1831-007	Anti tip tube	1	Pc.
12	1134-0008	Snap button D-131	1	Pc.
13	1249-0007	Insertion fork and castor	1	Pc.
14	1062-0412-006	Blind rivit F4x8	2	Pc.
15	1141-0008	Ribbed insert SFL 22x0,8-3,0	1	Pc.
16	3248-1811-015	Guide plate anti-tilt	1	Pc.
17	1141-0009	Ribbed insert 25x1-2,5	2	Pc.
18	3248-188x	Wire-rope for VTi twin anti tippers, SW 24	1	Pc.


SW =	24	26	28	30	32	34	36	40	44
X =	1	2	3	4	5	6	7	8	9
Please specify the appropriate number w hen ordering!									

Spare parts

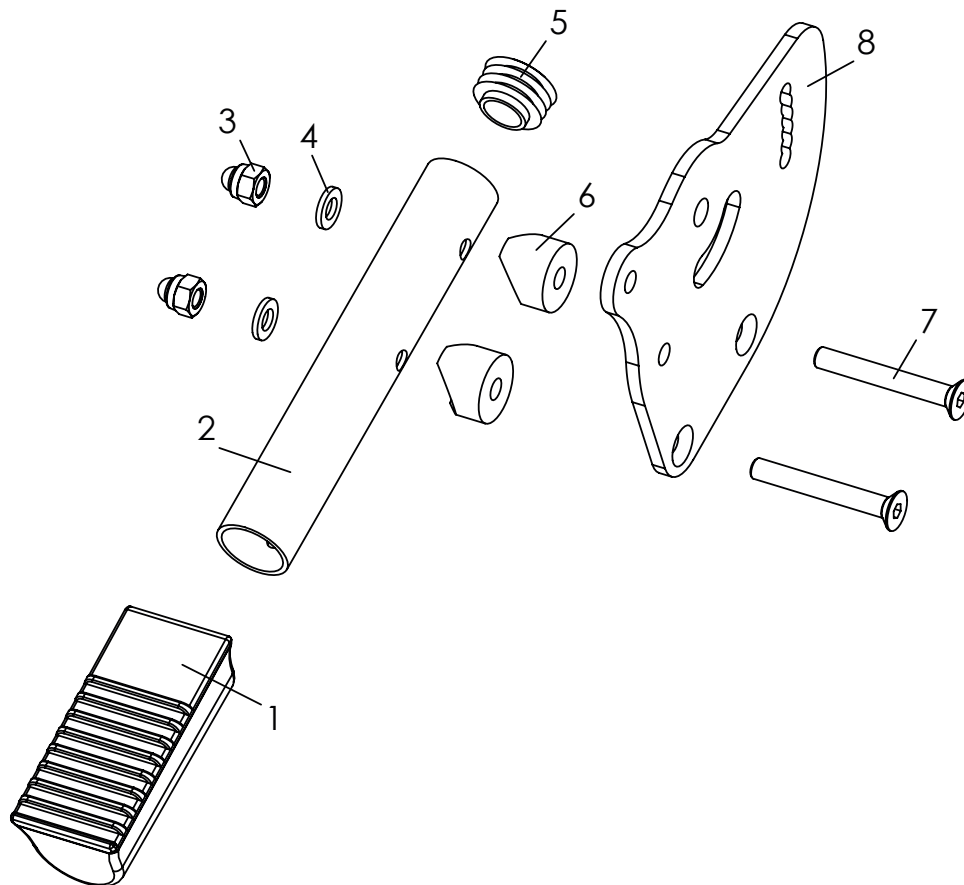


Pos	Article number	Articel description	Quantity	Unit
1-8	SA-0306/1-KP	Tip assist left	1	Pc.
	SA-0306/2-KP	Tip assist right	1	Pc.
1	7000-0068	Pedal D22 72x33 mm	1	Pc.
2	3245-5724_SA-0306	Cross tube SW24 with drilling for tip assist	1	Pc.
3	1044-0006	Hexagonal domed cap nut self-locking M6	2	Pc.
4	1051-0006	Washer A6,4 zp DIN/ISO 125 A6,4	2	Pc.
5	1141-0008	Ribbed insert SFL 22x0,8-3,0 black	1	Pc.
6	1111-0015	Saddle washer PA black 20x6,4x8	2	Pc.
7	1024-0645	Hexagon socket countersunk head screw M6x45	2	Pc.
8	3248-1811-015_SA-0306/1	Guide plate anti-tilt left	1	Pc.
	3248-1811-015_SA-0306/2	Guide plate anti-tilt right	1	Pc.

Spare parts

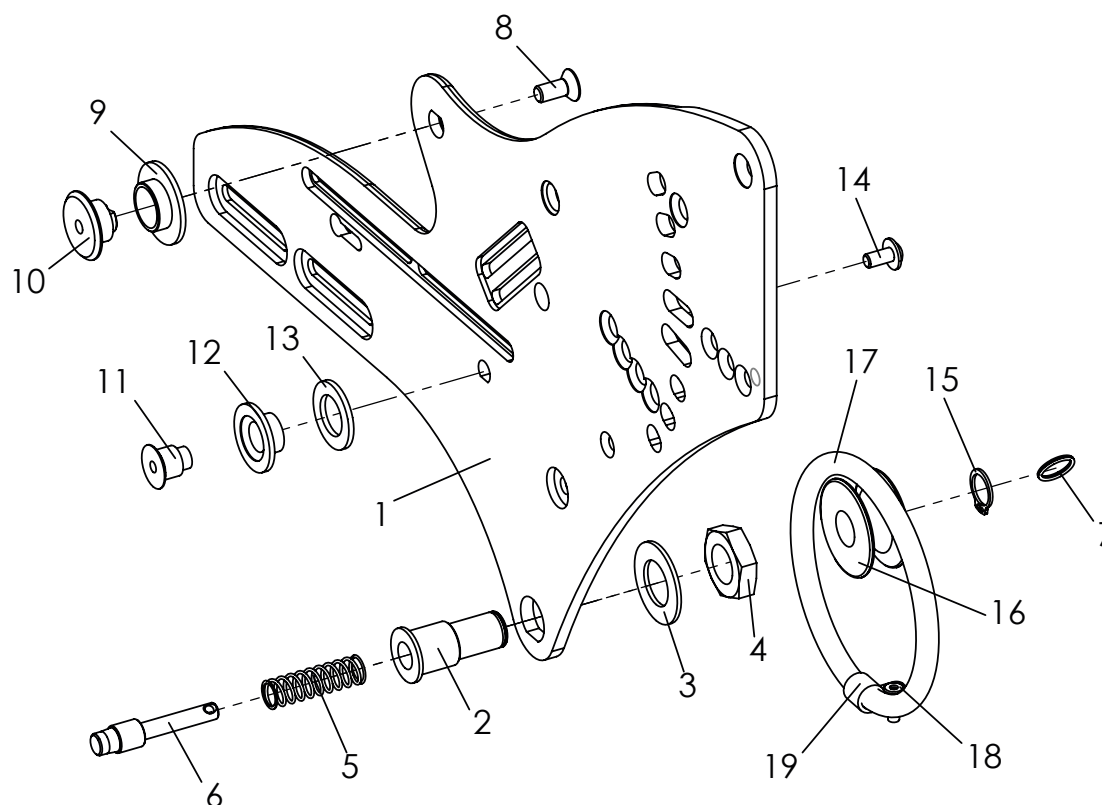

Pos	Article number	Articel description	Quantity	Unit
1-14	3249-1512E-015	Seat adapter left, complete	1	Pc.
	3249-1513E-015	Seat adapter right, complete	1	Pc.
1	3249-1512-015	Seat adapter left, black	1	Pc.
	3249-1513-015	Seat adapter right, black	1	Pc.
2	3246-1145	Bolt housing	1	Pc.
3	1052-0016	Spring washer B16	1	Pc.
4	1040-0016	Hexagon nut 16x 1,5	1	Pc.
5	1131-0070	Compression D-162	1	Pc.
6	3240-2422	Locking bolt	1	Pc.
7	1129-0036	Key ring 16x1,6	1	Pc.
8	1024-0614	Hexagon socket countersunk head screw M6x14	1	Pc.
9	3246-1146	Bushing 30x18x15x7,2	1	Pc.
10	3246-1147	Bearing axle tipping	1	Pc.
11	3246-1521	Stop pin	1	Pc.
12	3246-1522	Stop bushing	1	Pc.
13	1113-0052	Washer PA 15x25x2	1	Pc.
14	1030-0514	Button head socket screw with flange M5x14	1	Pc.
15-19	3249-1530	Tilt mechanism support kit (recommended for load over 40 kg)	1	Set
15	1080-0013	Retaining ring for shaft 13x1	1	Pc.
16	3247-1531	Rolle for rubber ring	1	Pc.
17	1150-0014	O-Ring 80x8	1	Pc.
18	1119-0004	Wire clamp D7,9 mm	1	Pc.
19	1062-0412	Blind rivet F4x12	1	Pc.


Spare parts

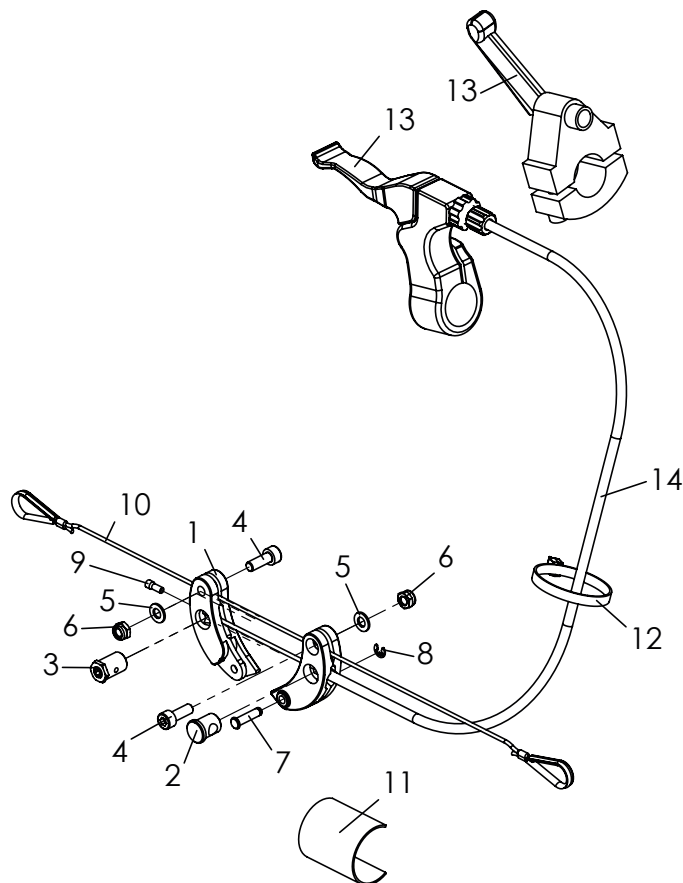

Pos	Article number	Articel description	Quantity	Unit
1-12	3246-19SWE	Tilt adjustment	1	Pc.
1-9	1209-1260E	Cable gearing with fasteners	1	Set.
1	1209-1261	Pull lever	2	Pc.
2	1209-1262	Bowden cable bracket	1	Pc.
3	1099-0025	Screw nipple D10x16 mm	1	Pc.
4	1021-0512	Hexagon socket head cap screw M5x12	2	Pc.
5	1051-0005	Washer A5,3	2	Pc.
6	1042-0005	Hexagon nut self-locking zp M5	2	Pc.
7	3257-1715	Security pin 15	1	Pc.
8	1081-0003	Security washer verz. 3,2	1	Pc.
9	1209-0007	End cap aluminium U-BA-S for Bowden cable wire	1	Pc.
10	3246-198X	Release wire rope with thimbles	1	Pc.
	1099-0018	Rope thimble	2	Pc.
	1089-0005	Cable clamp	2	Pc.
	1129-0011	Wire-rope 1,2x1,8 mm	1	Pc.
11	1500-0013	velcro self-adhesive 38mm	0,1	m
12	1089-0004	Cable tie, black, 5x180	1	Pc.
13	1209-0066	Brake lever with splittet bracket, d 22 mm black left	1	Pc.
	1209-0036	Release lever D22 (for push bar "wide")	1	Pc.
14	1209-0016	Bowden cable BZ-1100x1230-HOG for brake lever with splittet bracket	1	Pc.
	1209-0037	Bowden cable for release lever 1500/2000	1	Pc.

SW =	24	26	28	30	32	34	36
X =	1	2	3	4	5	6	7
Please specify the appropriate number w hen ordering!							

Spare parts

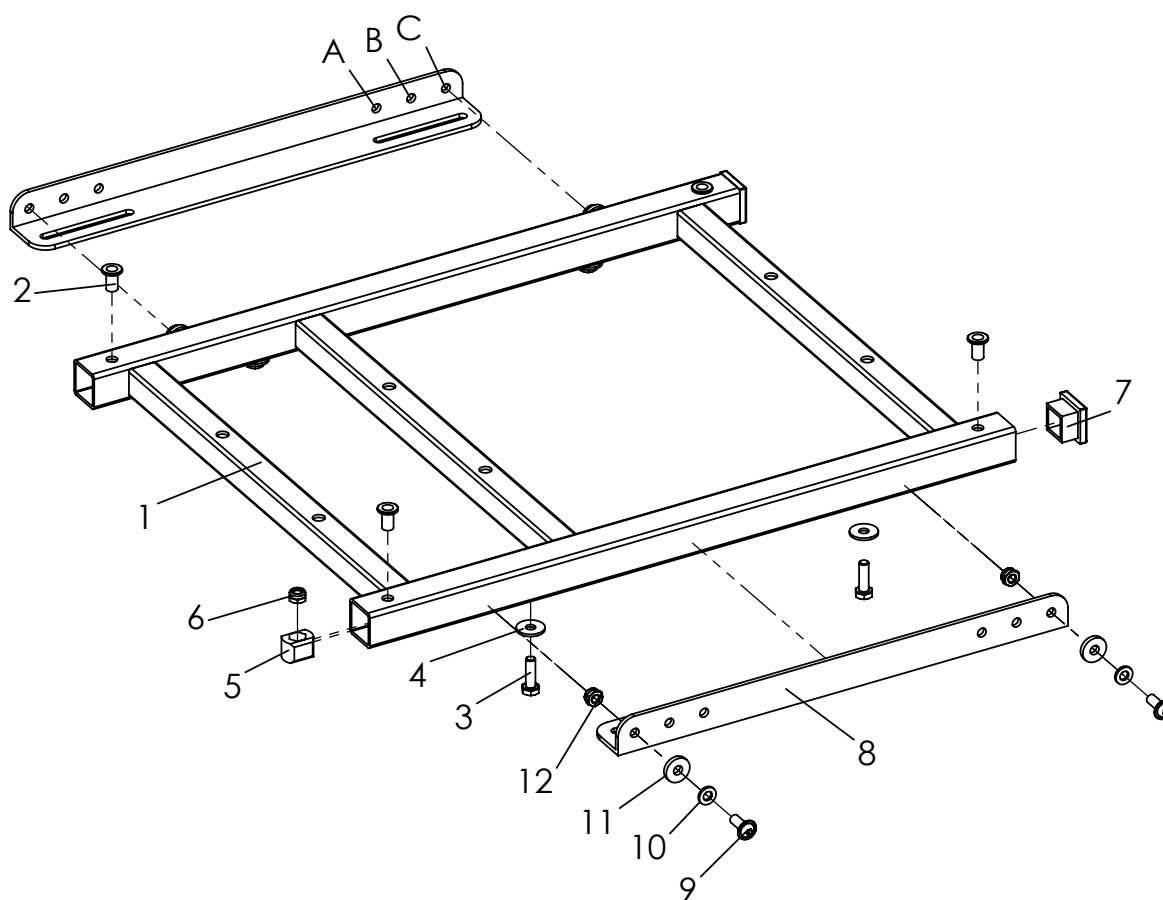


Pos	Article number	Articel description	Quantity	Unit
1-13	3249-23SWE-015	Seat frame complete with support brackets	1	Pc.
1-7	3249-22SWE-015	Seat frame with fasteners	1	Pc.
2	1049-0063	Rivet nut	4	Pc.
3	1011-0620	Hexagon head screw M6x20	4	Pc.
4	1055-0006	Washer A6,4	4	Pc.
5	3249-2262	Clamping block	4	Pc.
6	1044-0006	Hexagonal domed cap nut self-locking M6	4	Pc.
7	1141-0032	Ribbed insert VL 25	2	Pc.
8	3249-1551-015	Support bracket	2	Pc.
9	1030-0616	Button head socket screw with flange M6x16	4	Pc.
10	1051-0006	Washer A6,4	4	Pc.
11	1059-0003	Washer 17/6,4x3	4	Pc.
12	1042-0006-033	Hexagon nut self-locking black M6	4	Pc.
13	1030-0516	Button head socket screw with flange zp M5x16	4	Pc.

rear screw position support bracket (Pos. 3248-1551-015)

SW =	36	40	44
Pos. =	A	C	C

Article number 3249-22SWE-015 (Pos. 1-7) only for models built from 12/2023.
Article number 3248-23SWE-015 (Pos. 1-13) for Models built before 12/2023

Spare parts

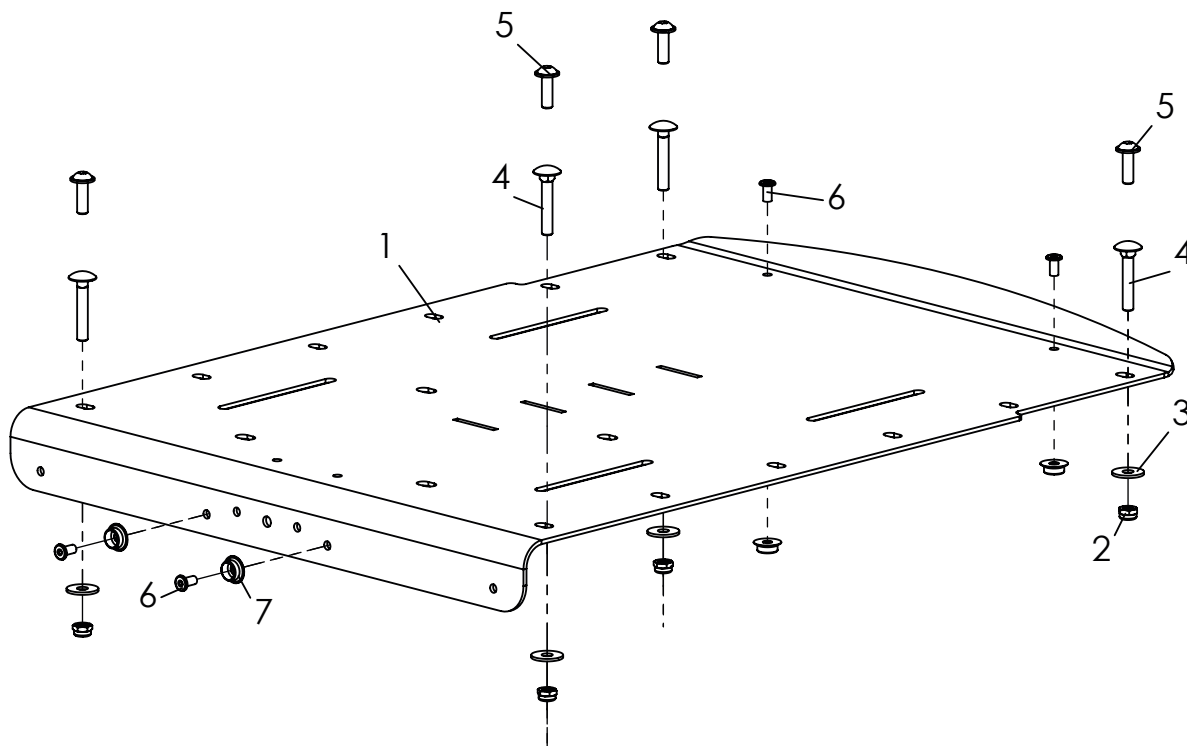

Pos	Article number	Articel description	Quantity	Unit
1-5	3245-24SW-015	Seat panel, complete	1	Pc.
1	3245-25SW-015	Seat panel, complete	1	Pc.
2	1042-0005	Hexagon nut self-locking M5	4	Pc.
3	1055-0005	Washer A5,3	4	Pc.
4	1026-0535	Mushroom head square neck bolt M5x30	4	Pc.
5	1030-0516	Button head socket screw with flange zp M5x16	4	Pc.
6*	1061-0406	Blind rivet F4x6	4	Pc.
7*	1533-0005	Ring-ball	4	Pc.

* Only when using Hoggi seat cushion.

Spare parts




Spare parts

Pos	Article number	Articel description	Quantity	Unit
1	3245-71SW-306	Seat cushion 3 cm thick	1	Pc.
	3245-71SW-346	Seat cushion 3 cm thick, black	1	Pc.
	3245-71SW-SA	Seat cushion 3 cm thick, special color	1	Pc.
2	3245-72SW-306	Back cushion, trough-shaped, 2,5 cm thick	1	Pc.
	3245-72SW-346	Back cushion, trough-shaped, 2,5 cm thick, black	1	Pc.
	3245-72SW-SA	Back cushion, trough-shaped, 2,5 cm thick, special color	1	Pc.
3-4	3245-73SW-306	Back cushion with side parts , 2,5 cm thick	1	Pc.
	3245-73SW-346	Back cushion with side parts , 2,5 cm thick, black	1	Pc.
	3245-73SW-SA	Back cushion with side parts , 2,5 cm thick, special color	1	Pc.
4	3245-7615-306	Side parts for back cushion, SW 24-44	1	Pair
5	3245-77SW-306	Seat cushion 3 cm thick, with cut out for hip support	1	Pc.
	3245-77SW-346	Seat cushion 3 cm thick, with cut out for hip support, black	1	Pc.
	3245-77SW-SA	Seat cushion 3 cm thick, with cut out for hip support, special design	1	Pc.

Color codes	306 =	black check
	346 =	black
	SA-0390 =	special colour
		321 = Spacer fabric, black
		347 = Berry Soft
		348 = Grey Soft
		349 = Turquoise Sof


Spare parts

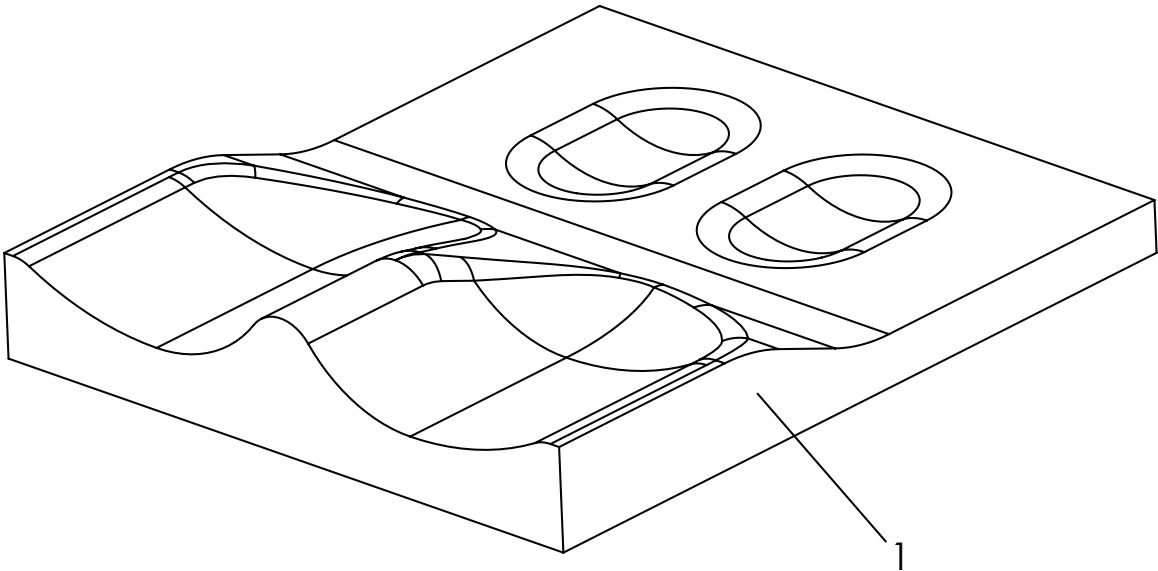

Pos	Article number	Article description	Quantity	Unit
1	3245-80SW-346	Contoured seat cushion with cover, black	1	Pc.
	3245-80SW-SA	Contoured seat cushion with cover, special color	1	Pc.


Color codes	306 =	black check
	346 =	black
	SA-0390 =	special colour
		321 = Spacer fabric, black
		347 = Berry Soft
		348 = Grey Soft
		349 = Turquoise Sof

Spare parts

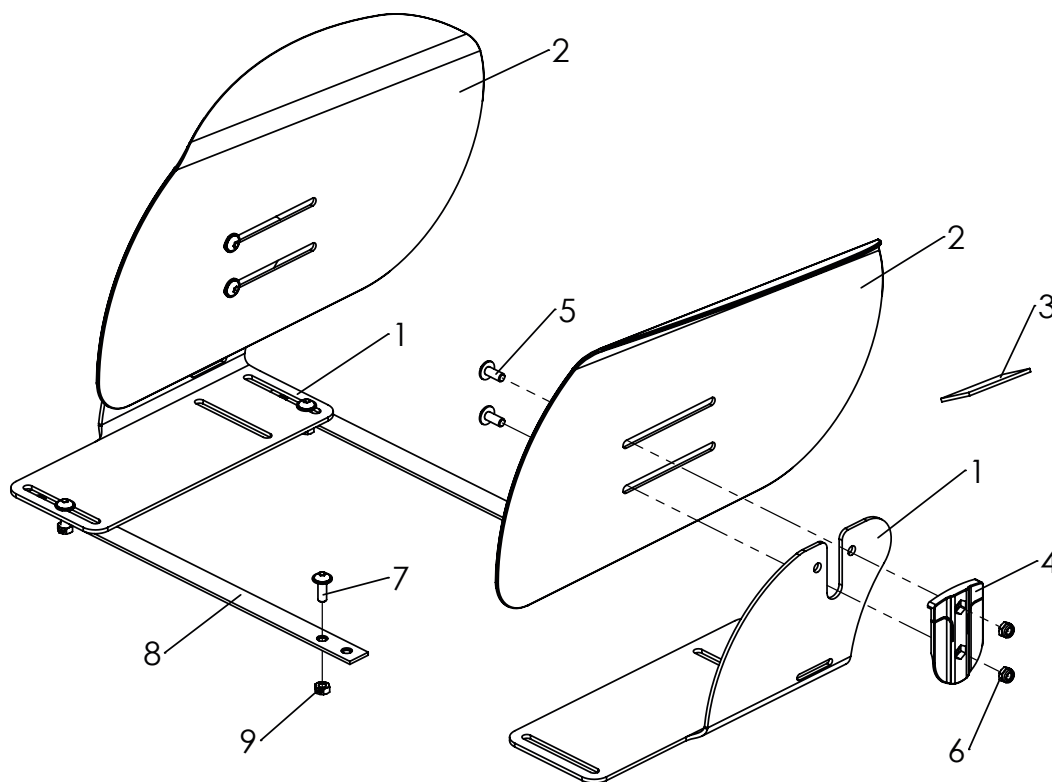

Pos	Article number	Articel description	Quantity	Unit
1-10	3245-6502-346	Hip supports size 2	1	Pair
1	3233-2612-032	Bracket for hip supports, right, size 2	1	Pc.
	3233-2611-032	Bracket for hip supports, left, size 2	1	Pc.
2	3235-2625-032	Hip support, right, size 2	1	Pc.
	3235-2624-032	Hip support, left, size 2	1	Pc.
3	1501-0003	Hook strap black 25mm self adhesive A glue	0,2	Pc.
4	3231-2622	Locking hinge	4	Pc.
5	1030-0512	Button head socket screw M5x12	4	Pc.
6	1042-0005	Hexagon nut self-locking M5	4	Pc.
7	3245-6520	Threaded strip size. 2	2	Pc.
8	1123-0500	Wing nut PA black M5	4	Pc.
9	1049-0066	BILOC-nut VM5	4	Pc.
10	3203-5150E-309	Hip upholstery size 2 (not shown)	1	Pair


Spare parts

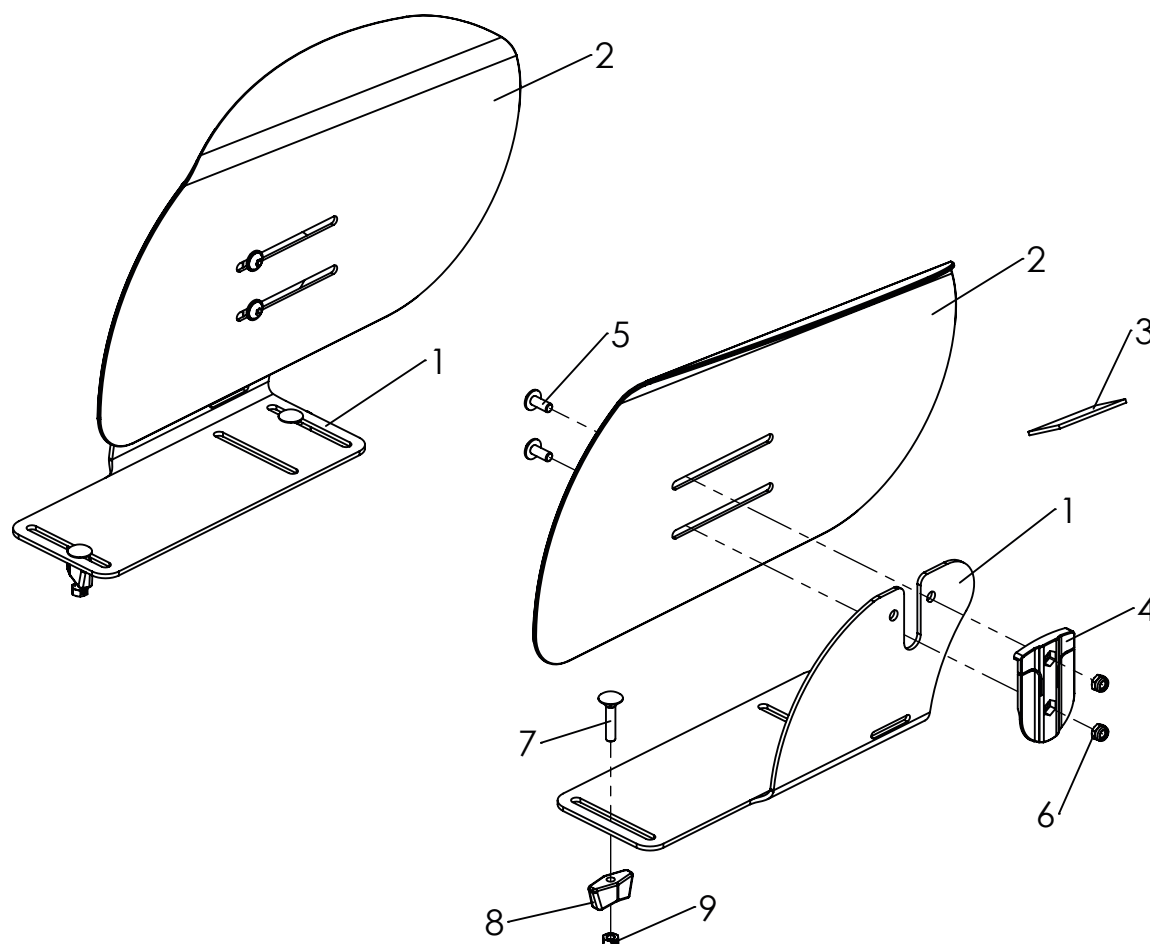

Pos	Article number	Articel description	Quantity	Unit
1-10	3247-6502-309	Hip supports size 2	1	Pair
1	3233-2612-032	Bracket for hip supports, right, size 2	1	Pc.
	3233-2611-032	Bracket for hip supports, left, size 2	1	Pc.
2	3235-2625-032	Hip support, right, size 2	1	Pc.
	3235-2624-032	Hip support, left, size 2	1	Pc.
3	1501-0003	Hook strap black 25mm self adhesive A glue	0,2	Pc.
4	3231-2622	Locking hinge	4	Pc.
5	1030-0512	Button head socket screw M5x12	4	Pc.
6	1042-0005	Hexagon nut self-locking M5	4	Pc.
7	1026-0525	Mushroom head square neck bolt M5x25	4	Pc.
8	1123-0500	Wing nut PA black M5	4	Pc.
9	1049-0066	BILOC-nut VM5	4	Pc.
10	3203-5150E-309	Hip upholstery size 2 (not shown)	1	Pair


Only for seat width 44!

Spare parts

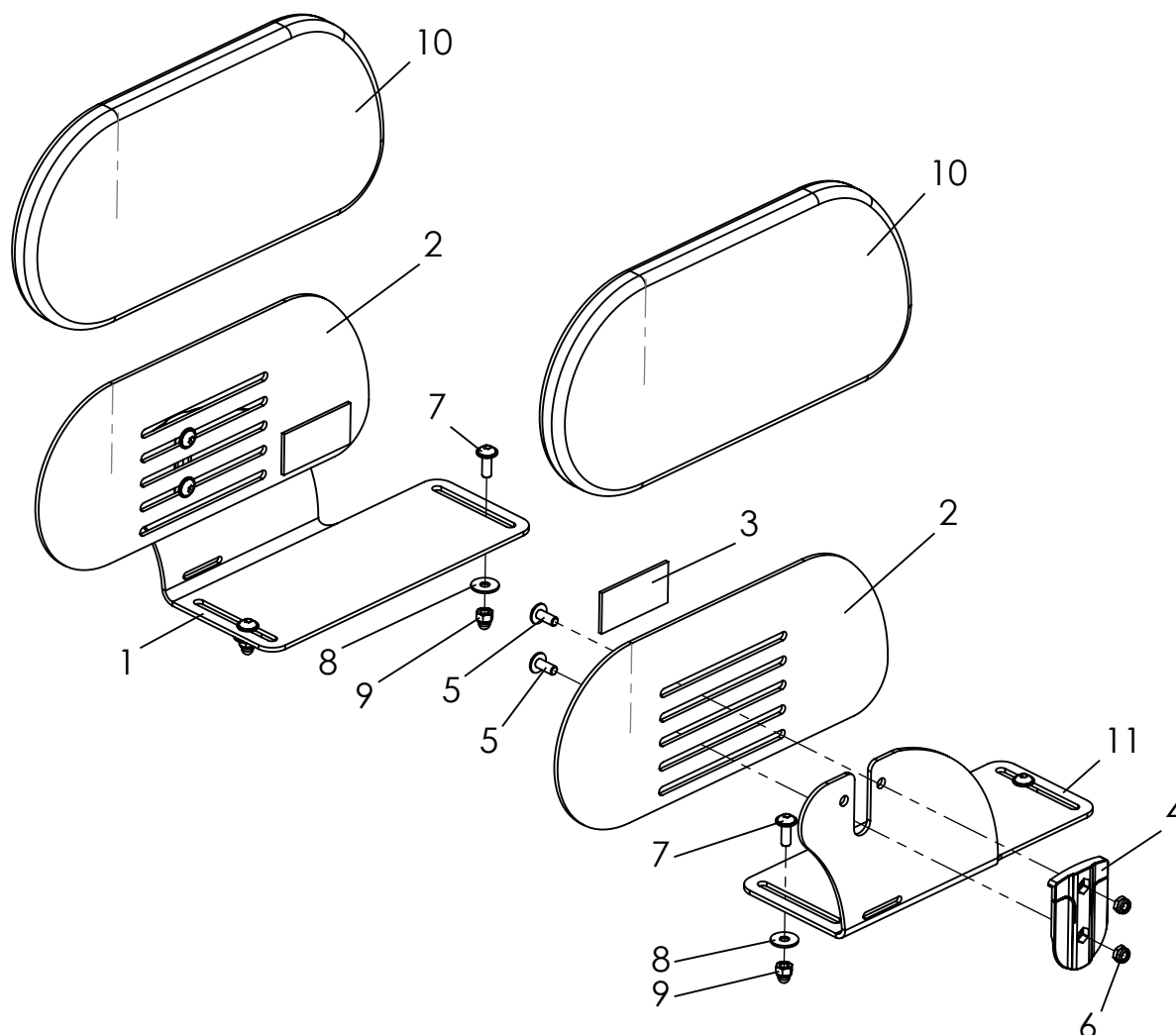


Pos	Article number	Articel description	Quantity	Unit
1-10	3249-6502-346	Hip support size 2, black	1	Pair
1	3249-6511-001	Bracket for hip supports size 1, left	1	Pc.
	3249-6512-001	Bracket for hip supports size 1, right	1	Pc.
	3249-6521-001	Straight hip pad size 2, black	2	Pc.
3	1501-0003	Hook strap black 25mm self adhesive	0,2	Pc.
4	3231-2622	Locking hinge	2	Pc.
5	1030-0512	Button head socket screw with flange M5x12	4	Pc.
6	1042-0005	Hexagon nut self-locking M5	4	Pc.
7	1030-0510	Button head socket screw with flange M5x10	4	Pc.
8	1055-0005	Washer A5,3	4	Pc.
9	1044-0005	Hexagonal domed cap nut self-locking M5	4	Pc.
10	3249-6550-346	Hip upholstery size 2 black	1	Pair

Spare parts

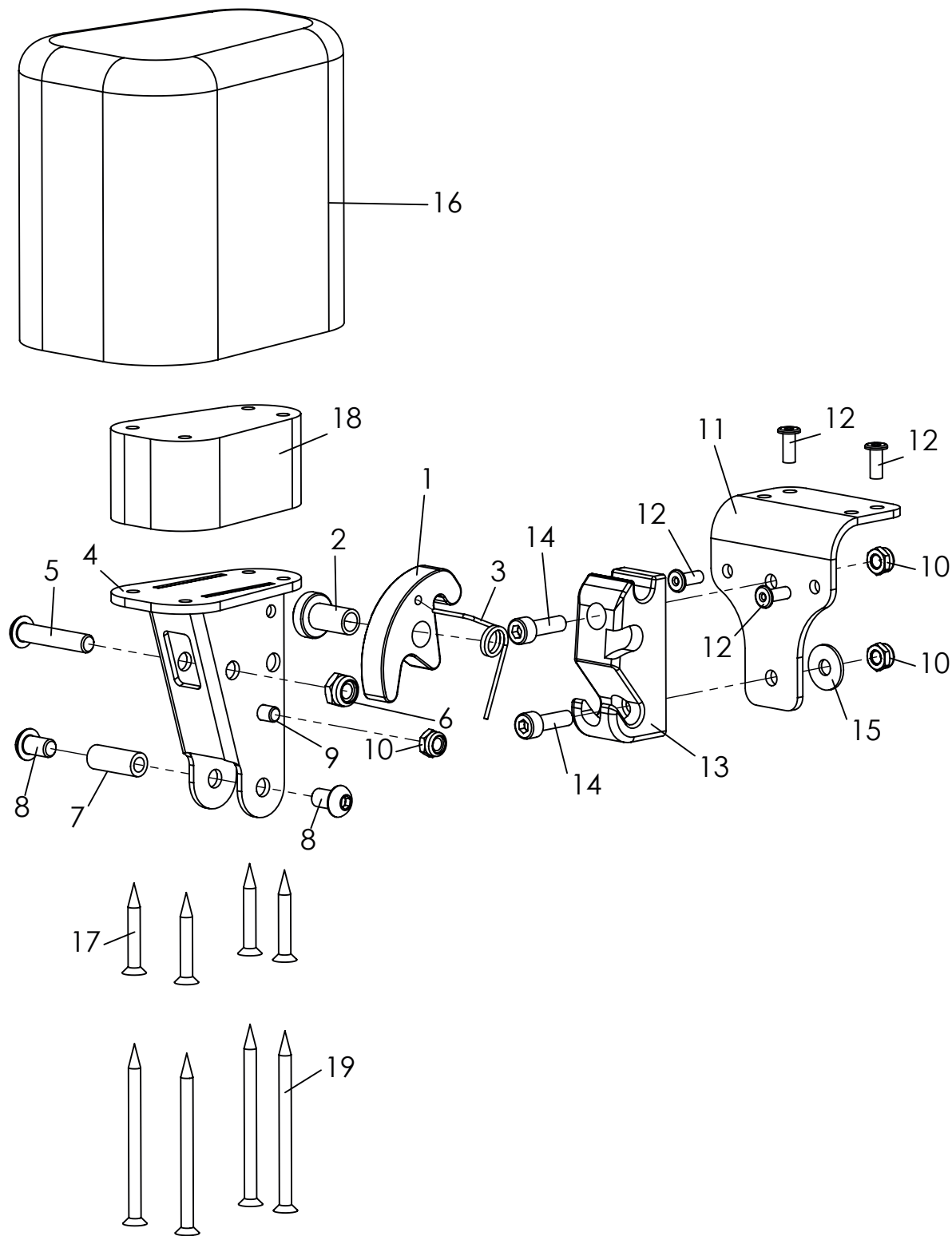


Spare parts

Pos	Article number	Articel description	Quantity	Unit
1-17	3231-7800	Abduction block size 1 swing&detachable, without cover	1	Pc.
	3231-7900	Abduction block size 2 swing&detachable, without cover	1	Pc.
	3233-7800	Abduction block size 3 swing&detachable, without cover	1	Pc.
1	3231-7841	Arresting clamp	1	Pc.
2	3231-7842	Bearing arresting clamp	1	Pc.
3	3231-7843	Leg spring 1,25 x 9,25 x 2,25	1	Pc.
4	3231-7820-032	A-wedge support	1	Pc.
5	1029-0630	Button head socket screw M6x30	1	Pc.
6	1042-0006	Hexagon nut self-locking 985 M6	1	Pc.
7	1114-0067	Threaded bushing 8xM6x20	1	Pc.
8	1030-0608	button head socket screw with flange zp M6x8	2	Pc.
9	1021-0508	Cylinder head screw M5x8	1	Pc.
10	1042-0005	Hexagon nut self-locking M5	3	Pc.
11	3231-7831-032	Base plate abduction block receiver	1	Pc.
12	1062-0410	Blind rivet F4x10	4	Pc.
13	3231-7832	Arresting part for A-wedge	1	Pc.
14	1021-0516	Hexagon socket head cap screw M5x16	2	Pc.
15	1055-0005	Washer A5,3	1	Pc.
16	3201-7810	Wood wedge size 1 with PU-foam	1	Pc.
	3201-7910	Wood wedge size 1 with PU-foam	1	Pc.
	3203-7810	Wood wedge size 1 with PU-foam	1	Set
17	1028-0430	Pressboard screw countersunk head 4x30	4	Pc.
18	3231-7821_SA-0286/2	Spacer plate for A-wedge 30 mm	1	Pc.
19	1028-0460	Pressboard screw countersunk head 4x60	4	Pc.
	3201-7840-346	Cover for abduction block size 1 black	1	Pc.
	3201-7940-346	Cover abduction block size 2 black	1	Pc.
	3203-7840-346	Cover for abduction block size 3 black	1	Pc.

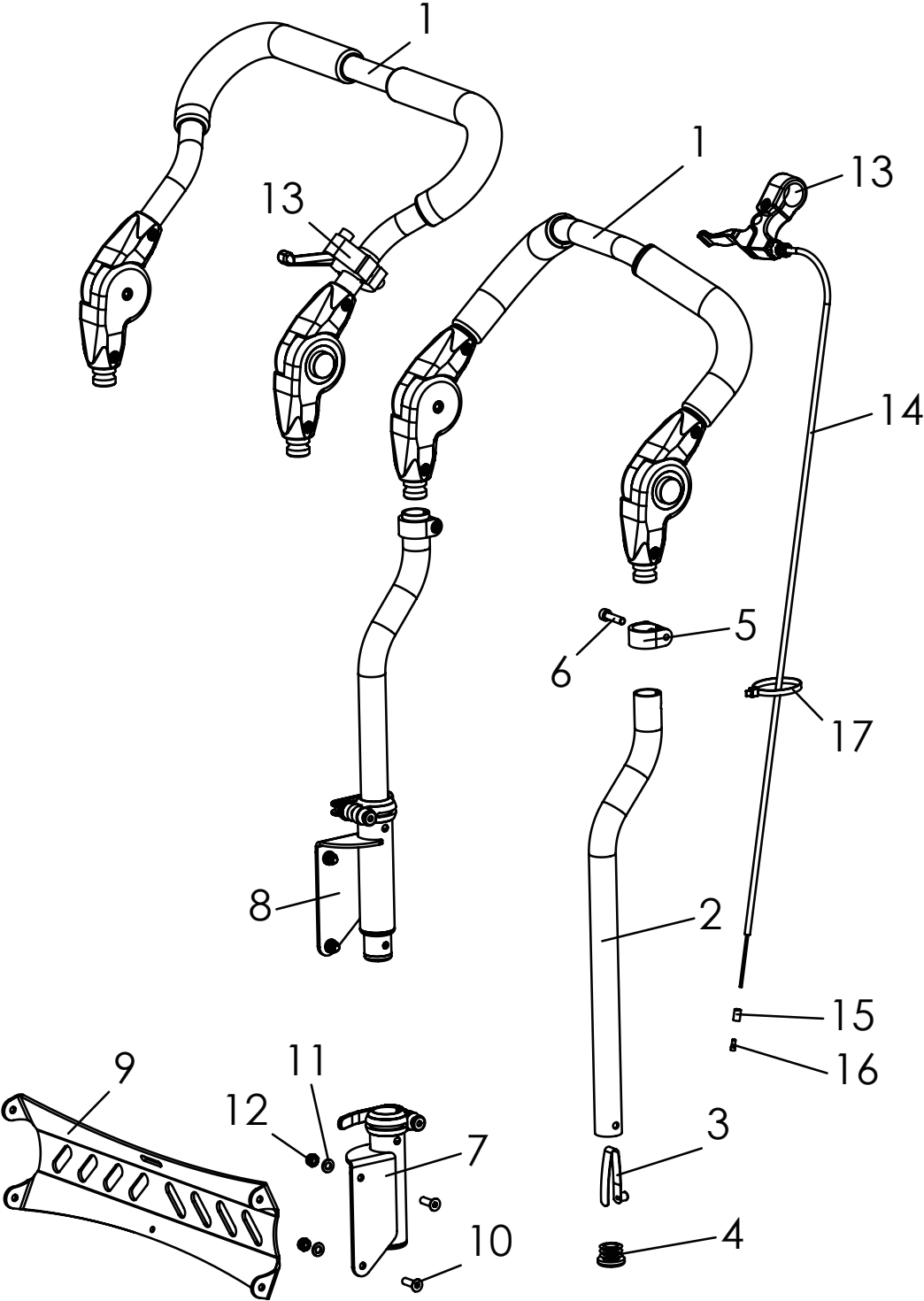

Spare parts

Pos	Article number	Articel description	Quantity	Uni
1-17	3245-55SWE	Push bar, detachable	1	Set
	3245-58SWE	Push bar "wide", detachable	1	Set
1	3240-5550E	Push bar upper part size 1, complete, from SW 24 up to SW 34	1	Pc.
	3240-5560E	Push bar upper part size 2, complete, from SW 36 up to SW 40	1	Pc.
	3240-5570E	Push bar upper part size 3, complete, for SW 44	1	Pc.
	3275-5450E	Push-bar "wide" upper part size 1, complete, from SW 24 up to SW 34	1	Pc.
	3275-5460E	Push-bar "wide" upper part size 2, complete, from SW 36 up to SW 40	1	Pc.
	3275-5470E	Push-bar "wide" upper part size 3, complete, for SW 44	1	Pc.
2	3245-3113-007	Push bar bottom part, anodised, black	2	Pc.
3	1134-0006	Snap button B 141	2	Pc.
4	1141-0015	Ribbed insert GL 25x1-2.5	2	Pc.
5	3240-5553-007	Clamping ring, anodised	2	Pc.
6	1021-0625	Hexagon socket head cap screw M6x25	2	Pc.
7	3245-5150E	Handle bracket left, complete	1	Pc.
8	3245-5170E	Handle bracket right, complete	1	Pc.
9	3245-52SW-015	Slide valve cross bar, structural black	1	Pc.
10	1024-0620	Hexagon socket countersunk head screw M6x20	4	Pc.
11	1051-0006	Washer A6,4	4	Pc.
12	1042-0006	Hexagon nut self-locking M6	4	Pc.
13	1209-0066	Brake lever with splittet bracket, d 22 mm black left	1	Pc.
	1209-0036	Release lever D22 (for Push bar "wide")	1	Pc.
14	1209-0016	Bowden cable BZ-1100x1230-HOG for brake lever with splittet bracket	1	Pc.
	1209-0037	Bowden cable for release lever 1500/2000	1	Pc.
15	1209-0006	End cap brass U-BA-H for Bowden cable cover	1	Pc.
16	1209-0007	End cap aluminium U-BA-S for Bowden cable wire	1	Pc.
17	1089-0004	Cable tie, black, 5x180	1	Pc.

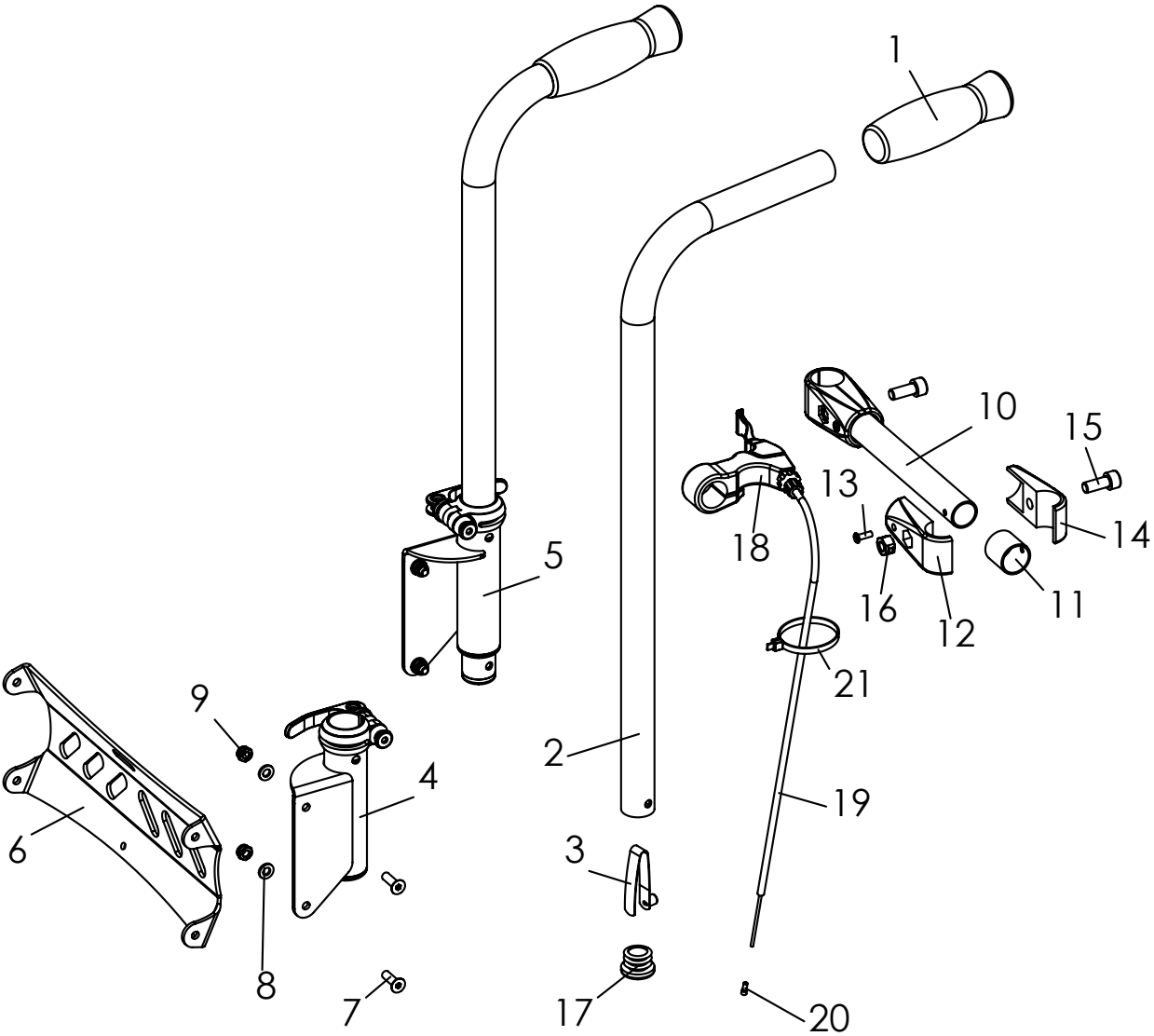


Spare parts

Pos	Article number	Articel description	Quantity	Unit
1-21	3245-56SWE	Push handle with handle grip, complete	1	Set.
1	1143-0005	Handle grip black d25 x 120	1	Set
2	3240-5311-007	Bent tube push handle, anodised black	1	Pc.
3	1134-0006	Snap button B 141 7,7x9,4	1	Pc.
4	3245-5150E	Push-handle adapter, left	1	Pc.
5	3245-5170E	Push-handle adapter, right	1	Pc.
6	3245-52SW-015	Slide valve cross bar	1	Pc.
7	1024-0620	Hexagon socket countersunk head screw M6x20	1	Pc.
8	1051-0006	Washer A6,4	4	Pc.
9	1042-0006	Hexagon nut self-locking M6	4	Pc.
10-16	3245-57SWE	Cross tube, complete	1	Pc.
10	3245-57SW-015	Cross tube	1	Pc.
11	3245-5711	Reducing bush	2	Pc.
12	1145-0001-015	Tube connector T-90° 25/25 with bore 4,2	2	Pc.
13	1062-0412	Blind rivet F4x12	2	Pc.
14	1145-0002-015	Tube connector T-90° 25/25 4,2	2	Pc.
15	1021-0820	Hexagon socket head cap screw M8x20	2	Pc.
16	1042-0008	Hexagon nut self-locking zp M8	2	Pc.
17	1141-0015	Ribbed insert GL 25x1-2.5	2	Pc.
18	1209-0066	Brake lever with splittet bracket, d 22 mm black left	1	Pc.
19	1209-0016	Bowden cable BZ-1100x1230-HOG	1	Pc.
20	1209-0007	End cap aluminium U-BA-S for Bowden cable wire	1	Pc.
21	1089-0004	Cable tie, black, 5x180	1	Pc.


Pos. 15 Tighten screw with max. 15 nm.

Spare parts

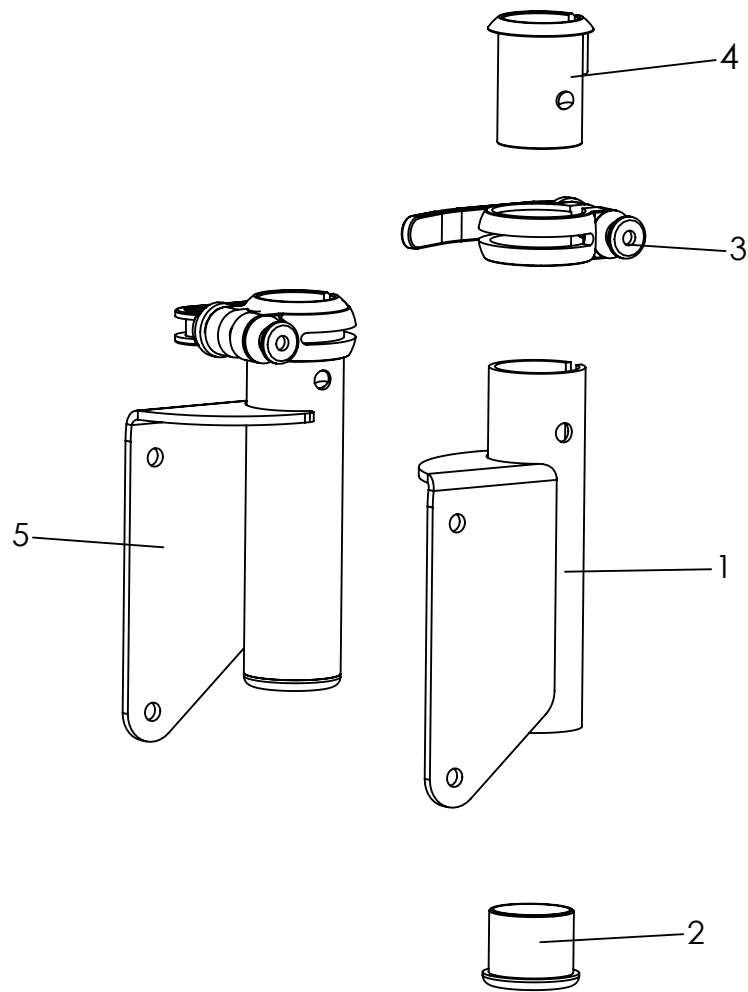

Pos	Article number	Articel description	Quantity	Unit
1-4	3245-5150E	Push-handle adapter left, complete	1	Pc.
2-5	3245-5170E	Push-handle adapter right, complete	1	Pc.
1	3245-5160-015	Push-handle adapter left	1	Pc.
2	3301-1530	Flange bushing	1	Pc.
3	1129-0007	Saddle clamp with quick-release D 31,8	1	Pc.
4	3301-1240	Clamp bushing	1	Pc.
5	3245-5170-015	Push-handle adapter right	1	Pc.


Spare parts

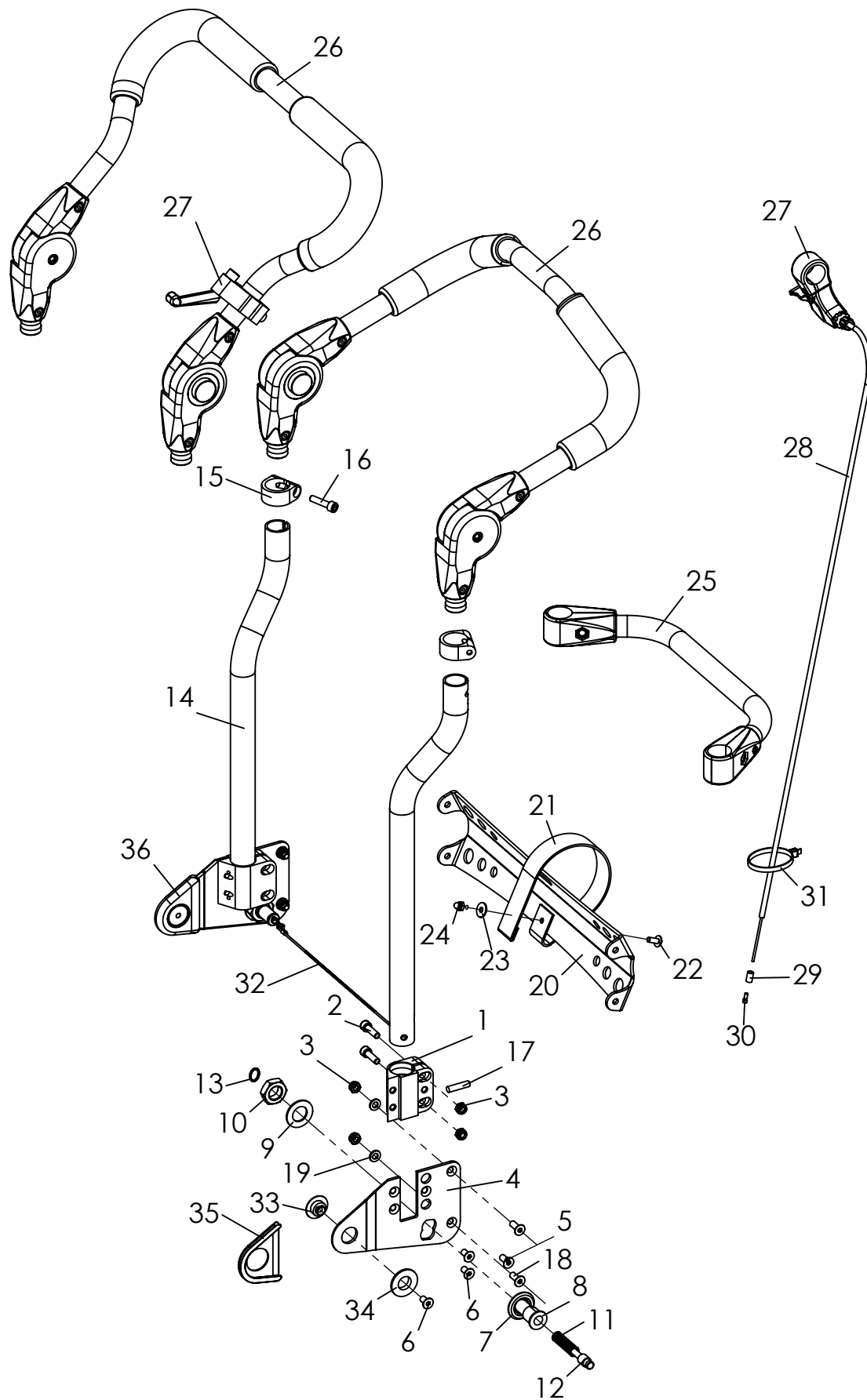

Spare parts


Pos	Article number	Articel description	Quantity	Unit
1-36	3248-30SWE	Push bar, foldable and angle adjustable	1	Set
	3248-38SWE	Push bar "wide", foldable and angle adjustable	1	Set
1	3248-3511-007	Push-bar adapter, anodised	2	Pc.
2	1021-0620	Hexagon socket head cap screw M6x20	4	Pc.
3	1042-0006	Hexagon nut self-locking M6	8	Pc.
4	3248-3512-015	Back base left	1	Pc.
	3248-3513-015	Back base right	1	Pc.
5	1024-0614	Hexagon socket countersunk head screw M6x14	2	Pc.
6	1024-0612	Hexagon socket countersunk head screw M6x12	6	Pc.
7	3248-3013	Thrust washer	2	Pc.
8	3246-1145	Bolt housing	2	Pc.
9	1052-0016	Spring washer B16	2	Pc.
10	1040-0016	Hexagon nut zp 16x 1,5	2	Pc.
11	1131-0070	Compression D-162 1,0x9,0x36x9,5	2	Pc.
12	3240-2422	Locking bolt	2	Pc.
13	1129-0036	Key ring 16x1,6	2	Pc.
14	3245-3113-007	Push bar bottom part	2	Pc.
15	3240-5553-007	Clamping ring	2	Pc.
16	1021-0625	Hexagon socket head cap screw M6x25	2	Pc.
17	1071-0624	Parallel pin m6x24	2	Pc.
18	1024-0616	Hexagon socket countersunk head screw zp DIN/ISO 7991 M6x16	4	Pc.
19	1051-0006	Washer A6,4	6	Pc.
20	3248-36SW-015	Slid valve cross bar	1	Pc.
21	1512-0012-1	PP strap 25x350 black with a sewn loop opposite the perforation	1	Pc.
22	1030-0512	Button head socket screw with flange M5x12	1	Pc.
23	1055-0005	Washer A5,3	1	Pc.
24	1044-0005	Hexagonal domed cap nut self-locking M5	1	Pc.
25	3245-36SWE	Cross bail, complete	1	Pc.
26	3240-5650E	Push bar upper part, 5 cm longer, size 1, from SW 24 up to SW 34	1	Pc.
	3240-5660E	Push bar upper part, 5 cm longer, size 2, from SW 36 up to SW 40	1	Pc.
	3240-5670E	Push bar upper part, 5 cm longer, size 2, complete, for SW 44	1	Pc.
	3275-5450E	Push-bar "wide" upper part size 1, complete, from SW 24 up to SW 34	1	Pc.
	3275-5460E	Push-bar "wide" upper part size 2, complete, from SW 36 up to SW 40	1	Pc.
	3275-5470E	Push-bar "wide" upper part size 3, complete, for SW 44	1	Pc.
27	1209-0066	Brake lever with splittet bracket, d 22 mm black left	1	Pc.
	1209-0036	Release lever D22 (for push bar "wide")	1	Pc.
28	1209-0016	Bowden cable BZ-1100x1230-HOG for brake lever with splittet bracket	1	Pc.
	1209-0037	Bowden cable for release lever 1500/2000	1	Pc.
29	1209-0006	End cap brass U-BA-H for Bowden cable cover	1	Pc.
30	1209-0007	End cap aluminium U-BA-S for Bowden cable wire	1	Pc.
31	1089-0004	Cable tie, black, 5x180	1	Pc.
32	3248-308X	Wire-rope for push handle angle adjustable (SW according to order)	1	Pc.
33	3245-2614	Bearing shaft	2	Pc.
34	3245-2613	Flange bushing	2	Pc.
35	3248-3515	Cover back base left	1	Pc.
36	1089-0004	Cover back base right	1	Pc.

SW =	24	26	28	30	32	34	36
X =	1	2	3	4	5	6	7

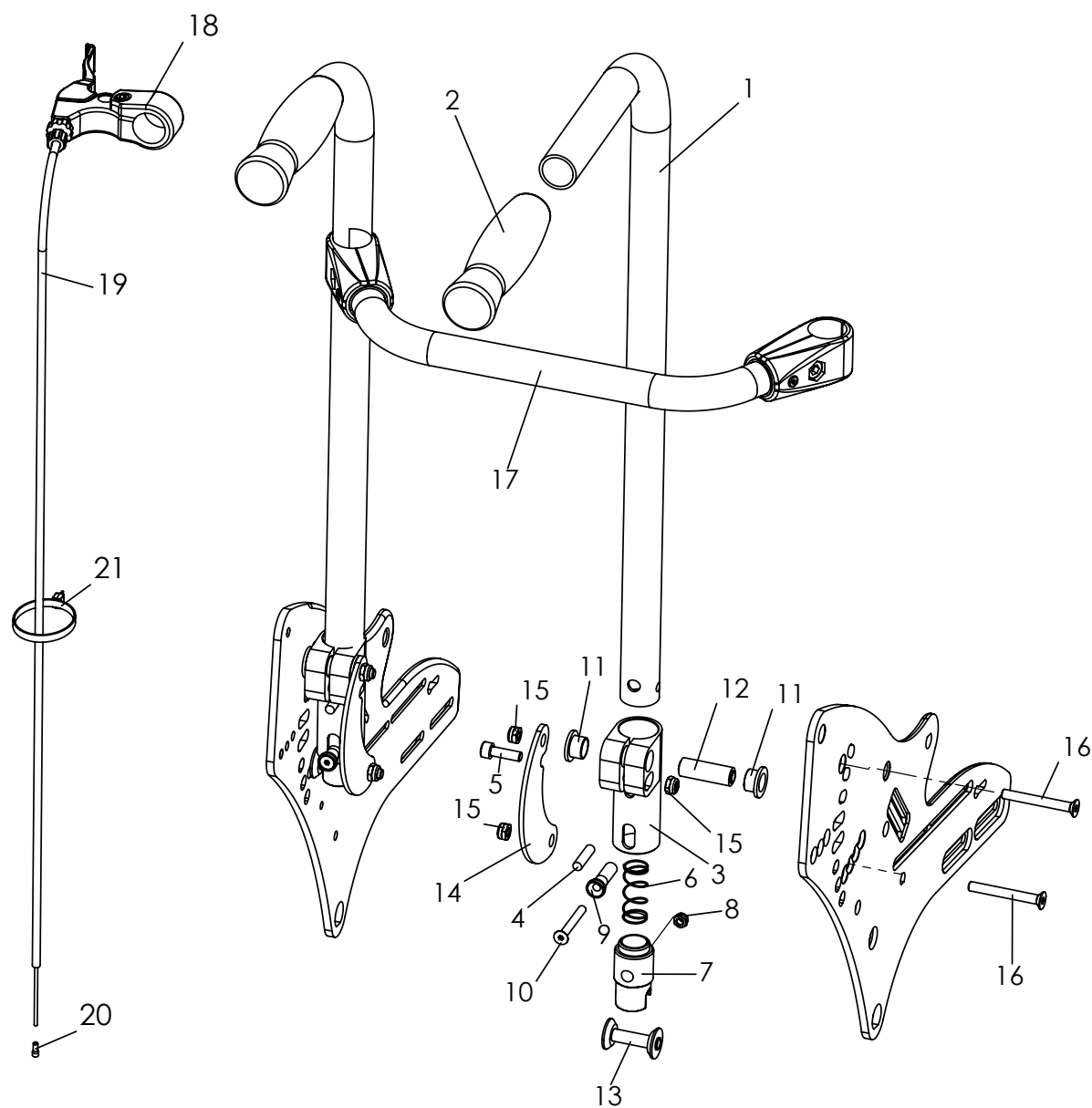

Spare parts


Pos	Article number	Articel description	Quantity	Unit
1-21	3245-35SWE	Push handles, foldable	1	Set
	3245-5610E	Push handle	1	Pc.
1	3240-5311-007	Bent tube push handle	2	Pc.
2	1143-0005	Handle grip black d22 x 120	2	Pc.
3-16	3245-3010E-007	Push bar holder, complete	2	Pc.
3	3245-3011-007	Push-bar adapter, anodised	2	Pc.
4	1071-0624	Parallel pin M6x24	2	Pc.
5	1021-0620	Hexagon socket head cap screw M6x20	2	Pc.
6	1131-0064	Compression spring D-142A 0,9x15x39,2x3,5	2	Pc.
7	3245-5112	Locking claw	2	Pc.
	3245-5113S	Locking claw, for tie down kit	2	Pc.
8	1049-0066	BILOC-nut zp VM5	1	Pc.
9	3245-3013	Activator	2	Pc.
10	1024-0540	Hexagon socket countersunk head screw M5x40	2	Pc.
11	3245-3019	Flanged bush	4	Pc.
12	3245-3015	Bearing shaft	2	Pc.
13	3245-3014	Latch shaft	2	Pc.
14	3245-3016-015	Backup bar, structural black	2	Pc.
15	1042-0006	Hexagon nut self-locking M6	6	Pc.
16	1024-0650	Hexagon socket countersunk head screw M6x50	4	Pc.
17	3245-36SWE	Cross bail, complete	1	Pc.
18	1209-0066	Brake lever with splittet bracket, d 22 mm black left	1	Pc.
19	1209-0016	Bowden cable BZ-1100x1230-HOG	1	Pc.
20	1209-0007	End cap aluminium U-BA-S for Bowden cable wire	1	Pc.
21	1089-0004	Cable tie, black, 5x180	1	Pc.

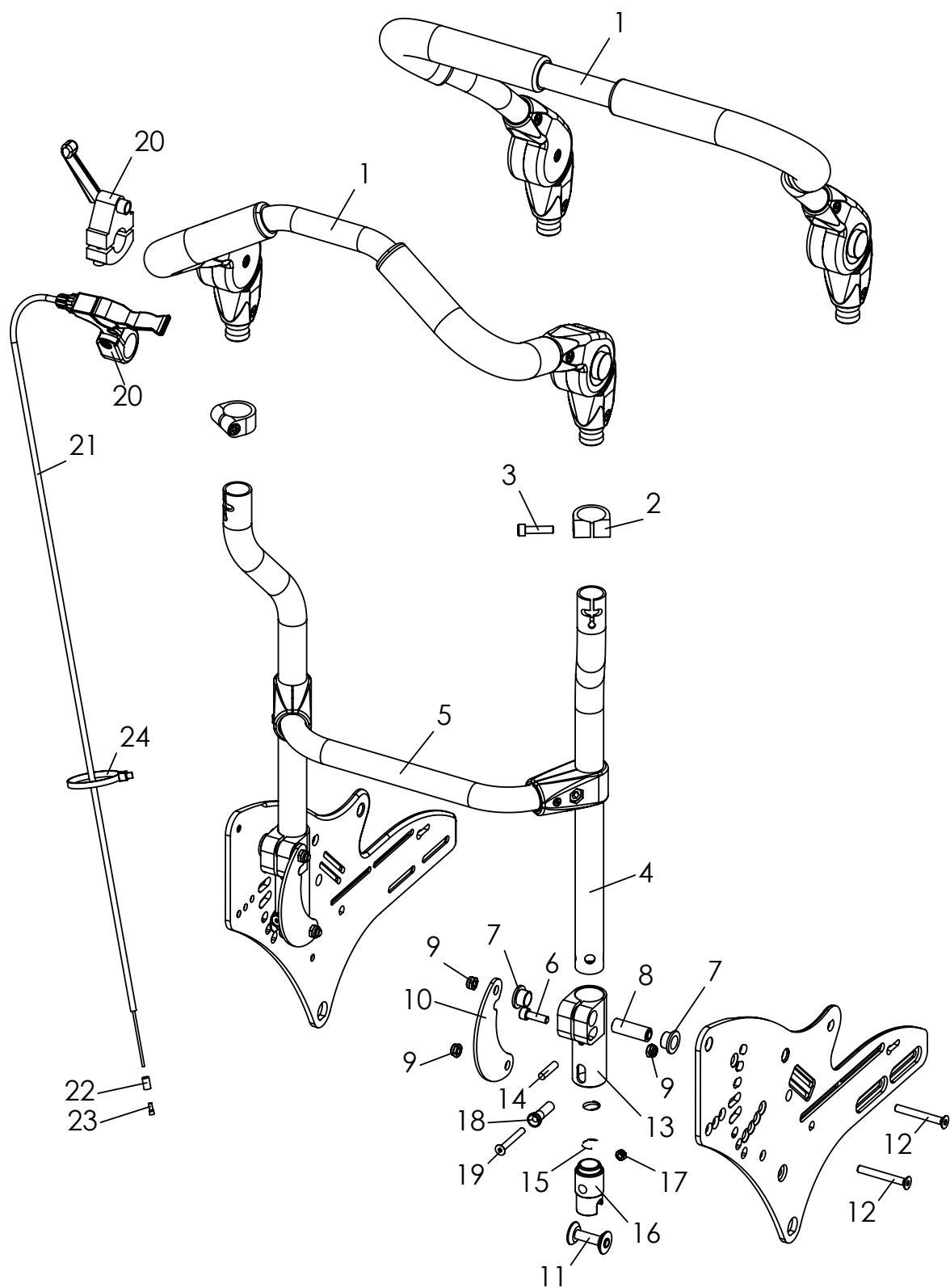

Spare parts

Pos	Article number	Articel description	Quantity	Unit
1-24	3245-30SWE	Push bar, foldable	1	Set
	3245-38SWE	Push bar "wide", foldable	1	Set
1	3240-5550E	Push bar upper part size 1, complete, from SW 24 up to SW 34	1	Pc.
	3240-5560E	Push bar upper part size 2, complete, from SW 36 up to SW 40	1	Pc.
	3240-5570E	Push bar upper part size 3, complete, for SW 44	1	Pc.
	3275-5450E	Push-bar "wide" upper part size 1, complete, from SW 24 up to SW 34	2	Pc.
	3275-5460E	Push-bar "wide" upper part size 2, complete, from SW 36 up to SW 40	1	Pc.
	3275-5470E	Push-bar "wide" upper part size 3, complete, for SW 44	1	Pc.
2	3240-5553-007	Clamping ring	2	Pc.
3	1021-0625	Hexagon socket head cap screw M6x25	2	Pc.
4	3245-3113-007	Push bar bottom part, anodised, black	2	Pc.
5	3245-36SWE	Cross bail, complete	1	Pc.
6-19	3245-3010E	Push bar holder, complete	2	Pc.
6	1021-0620	Hexagon socket head cap screw M6x20	2	Pc.
7	3245-3019	Flanged bush	4	Pc.
8	3245-3015	Bearing shaft	2	Pc.
9	1042-0006	Hexagon nut self-locking M6	6	Pc.
10	3245-3016-015	Backup bar	2	Pc.
11	3245-3014	Latch shaft	2	Pc.
12	1024-0650	Hexagon socket countersunk head screw M6x50	4	Pc.
13	3245-3011-007	Push-bar adapter	2	Pc.
14	1071-0624	Parallel pin M6x24	2	Pc.
15	1131-0064	Compression spring D-142A 0,9x15x39,2x3,5	2	Pc.
16	3245-5112	Locking claw	2	Pc.
	3245-5113S	Locking claw Alloy (for tie down kit)	2	Pc.
17	1042-0005	Hexagon nut self-locking M5	2	Pc.
18	3245-3013	Activator	2	Pc.
19	1024-0540	Hexagon socket countersunk head screw M5x40	2	Pc.
20	1209-0066	Brake lever with splittet bracket, d 22 mm black left	1	Pc.
	1209-0036	Release lever D22 (for push bar "wide")	1	Pc.
21	1209-0016	Bowden cable BZ-1100x1230-HOG for brake lever with splittet bracket	1	Pc.
	1209-0037	Bowden cable for release lever 1500/2000	1	Pc.
22	1209-0006	End cap brass U-BA-H for Bowden cable cover	1	Pc.
23	1209-0007	End cap aluminium U-BA-S for Bowden cable wire	1	Pc.
24	1089-0004	Cable tie, black, 5x180	1	Pc.

Spare parts

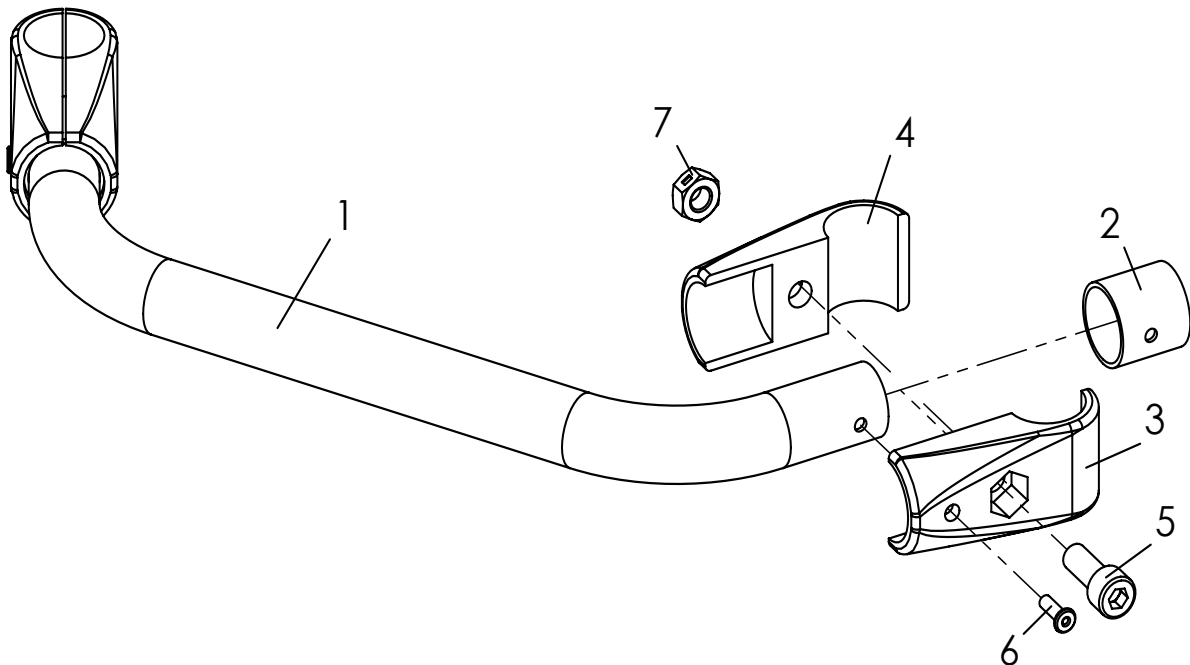


Pos	Article number	Article description	Quantity	Unit
1-7	3245-36SWE	Cross bail, complete	1	Pc.
1	3245-36SW-015	Cross bail, complete	1	Pc.
2	3245-5711	Reducing bush	2	Pc.
3	1145-0001-015	Tube connector T-90° 25/25 with bore 4,2, structural black	2	Pc.
4	1145-0002-015	Tube connector T-90° 25/25 4,2 structural black	2	Pc.
5	1021-0820	Hexagon socket head cap screw M8x20	2	Pc.
6	1062-0412	Blind rivet F4x12	2	Pc.
7	1049-0067	BILOC-nut VM8	2	Pc.

Pos. 5

Tighten screw with max. 15 nm.

Spare parts

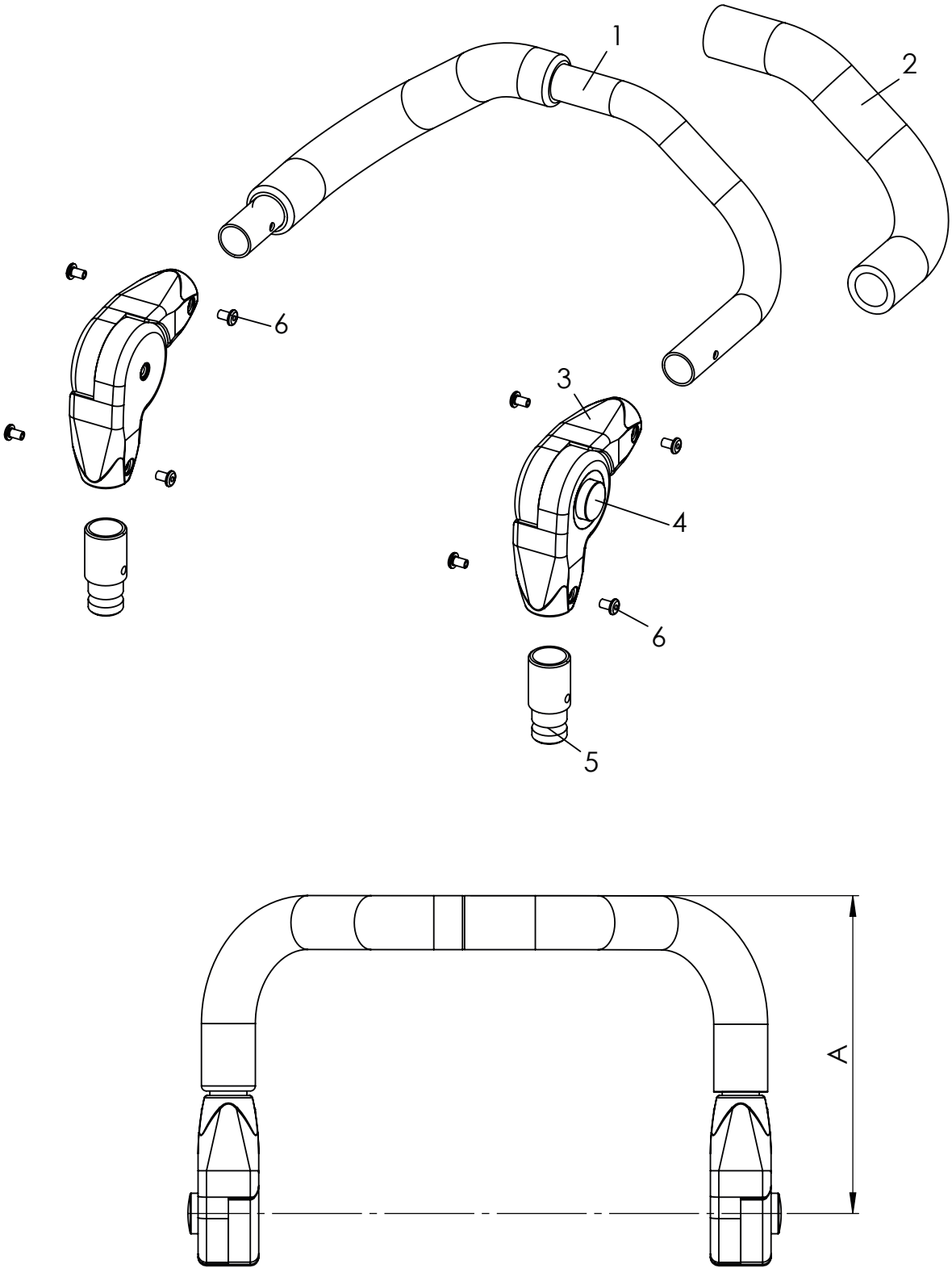


Spare parts

Pos	Article number	Articel description	Quantity	Unit
1-6	3240-5550E	Push bar upper part complete (A= 188 mm), size 1, SW 24 up to SW 34	1	Pc.
	3240-5560E	Push bar upper part complete (A= 188 mm), size 2, SW 36 up to SW 40	1	Pc.
	3240-5570E	Push bar upper part complete (A= 188 mm), size 3, SW 44	1	Pc.
	3240-5650E	Push bar upper part complete (A= 238 mm), size 1, SW 24 up to SW 34	1	Pc.
	3240-5660E	Push bar upper part complete (A= 238 mm), size 2, SW 36 up to SW 40	1	Pc.
	3240-5670E	Push bar upper part complete (A= 238 mm), size 3, SW 44	1	Pc.
	3240-5750E	Push bar upper part complete (A= 288 mm), size 1 SW 24 up to SW 34	1	Pc.
	3240-5760E	Push bar upper part complete (A= 288 mm), size 2, SW 36 up to SW 40	1	Pc.
	3240-5770E	Push bar upper part complete (A= 288 mm), size 3, SW 44	1	Pc.
	3240-5850E	Push bar upper part complete (A= 338 mm), size 1 SW 24 up to SW 34	1	Pc.
	3240-5860E	Push bar upper part complete (A= 338 mm), size 2, SW 36 up to SW 40	1	Pc.
	3240-5870E	Push bar upper part complete (A= 338 mm), size 3, SW 44	1	Pc.
1	3240-5552-001	Push bar upper part size 1 (151 mm), black	1	Pc.
	3240-5562-001	Push bar upper part size 2 (151 mm), black	1	Pc.
	3240-5572-001	Push bar upper part size 3 (151 mm), black	1	Pc.
	3240-5652-001	Push bar upper part size 1 (201 mm), black	1	Pc.
	3240-5662-001	Push bar upper part size 2 (201 mm), black	1	Pc.
	3240-5672-001	Push bar upper part size 3 (201 mm), black	1	Pc.
	3240-5752-001	Push bar upper part size 1 (251 mm), black	1	Pc.
	3240-5762-001	Push bar upper part size 2 (251 mm), black	1	Pc.
	3240-5772-001	Push bar upper part size 3 (251 mm), black	1	Pc.
	3240-5852-001	Push bar upper part size 1 (301 mm), black	1	Pc.
	3240-5862-001	Push bar upper part size 2 (301 mm), black	1	Pc.
	3240-5872-001	Push bar upper part size 3 (301 mm), black	1	Pc.
2	1143-0001	Handle grip EVA black d30 x 260	2	Pc.
3-4	1129-0009	Ratchet joint 25x22	1	Pair
4	7000-0054	Button for ratchet joint	2	Pc.
5	3240-5554	Coupling binder	2	Pc.
6	1062-0508	Blind rivet F5x8	8	Pc.


SA-0183	Length Push bar upper part			
	Standard	+ 5cm	+ 10cm	+ 15cm
3245-55SW	3240-5550E	3240-5650E	3240-5750E	3240-5850E
	3240-5560E	3240-5660E	3240-5760E	3240-5860E
	3240-5570E	3240-5670E	3240-5770E	3240-5870E
3245-30SW	3240-5550E	3240-5650E	3240-5750E	3240-5850E
	3240-5560E	3240-5660E	3240-5760E	3240-5860E
	3240-5570E	3240-5670E	3240-5770E	3240-5870E
3248-30SW	3240-5650E	3240-5750E	3240-5850E	-
	3240-5660E	3240-5760E	3240-5860E	-
	3240-5670E	3240-5770E	3240-5870E	-

Spare parts

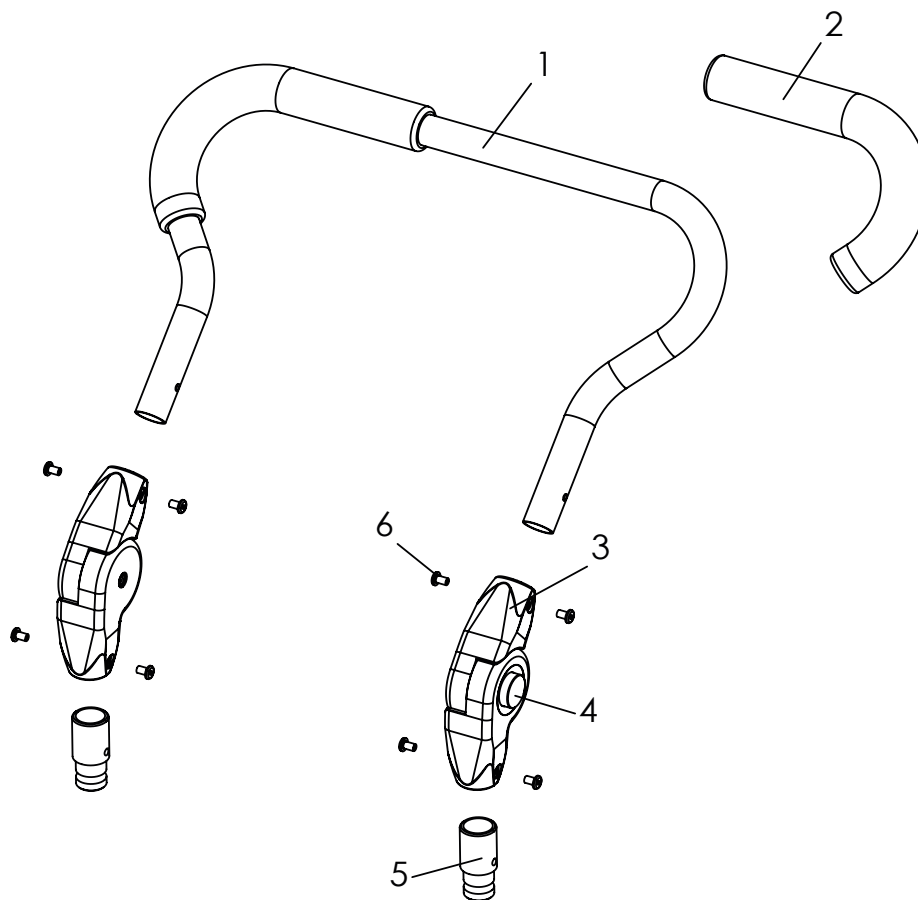


Pos	Article number	Articel description	Quantity	Unit
1-5	3275-5450E	Push-bar "wide" upper part size 1, complete	1	Pc.
	3275-5460E	Push-bar "wide" upper part size 2, complete	1	Pc.
	3275-5470E	Push-bar "wide" upper part size 3, complete	1	Pc.
1	3240-5552-001	Push bar upper part size 1, black	1	Pc.
	3240-5562-001	Push bar upper part size 2, black	1	Pc.
	3240-5572-001	Push bar upper part size 3, black	1	Pc.
2	1143-0001	Handle grip EVA black d30 x 260	2	Pc.
3-4	1129-0009	Ratchet joint 25x22	1	Pair
4	7000-0054	Button for ratchet joint	2	Pc.
5	3240-5554	Coupling binder	2	Pc.
6	1062-0508	Blind rivet F5x8	8	Pc.

Spare parts

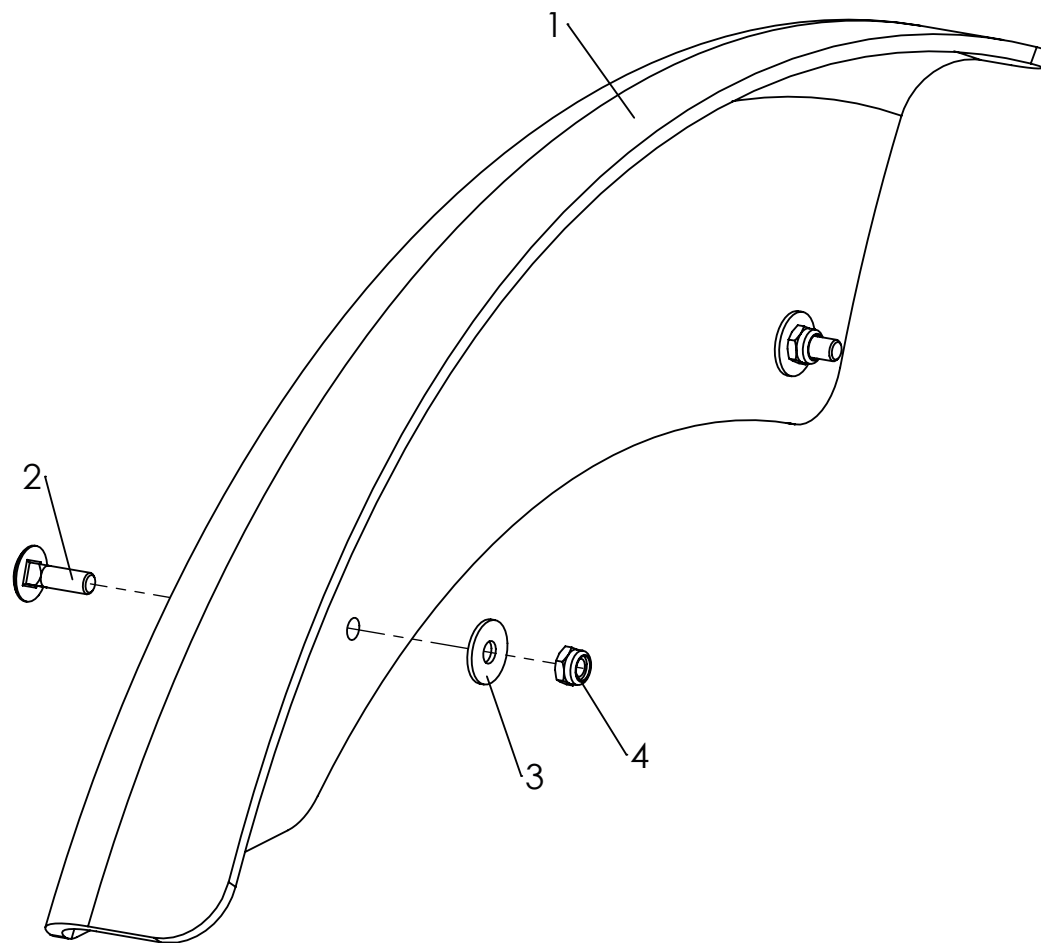


Pos	Article number	Articel description	Quantity	Unit
1-4	3248-1602E	Wheel guards 22"	1	Set
	3248-1604E	Wheel guards 24"	1	Set
	3248-1603E	Wheel guards 22" wide for 3° camber	1	Set
	3248-1605E	Wheel guards 24" wide for 3° camber	1	Set
1	3248-1632	Fender left 22"	1	Pc.
	3248-1642	Fender right 22"	1	Pc.
	3248-1652	Fender left 24"	1	Pc.
	3248-1662	Fender right 24"	1	Pc.
	3248-1634	Fender left 22", wide for 3° camber	1	Pc.
	3248-1644	Fender right 22" wide, wide for 3° camber	1	Pc.
	3248-1654	Fender left 24" wide, wide for 3° camber	1	Pc.
	3248-1664	Fender right 24" wide, wide for 3° camber	1	Pc.
2	1026-0516	Mushroom head square neck bolt M5x16	4	Pc.
3	1055-0005	Washer A5,3	4	Pc.
4	1042-0005	Hexagon nut self-locking M5	4	Pc.

Spare parts

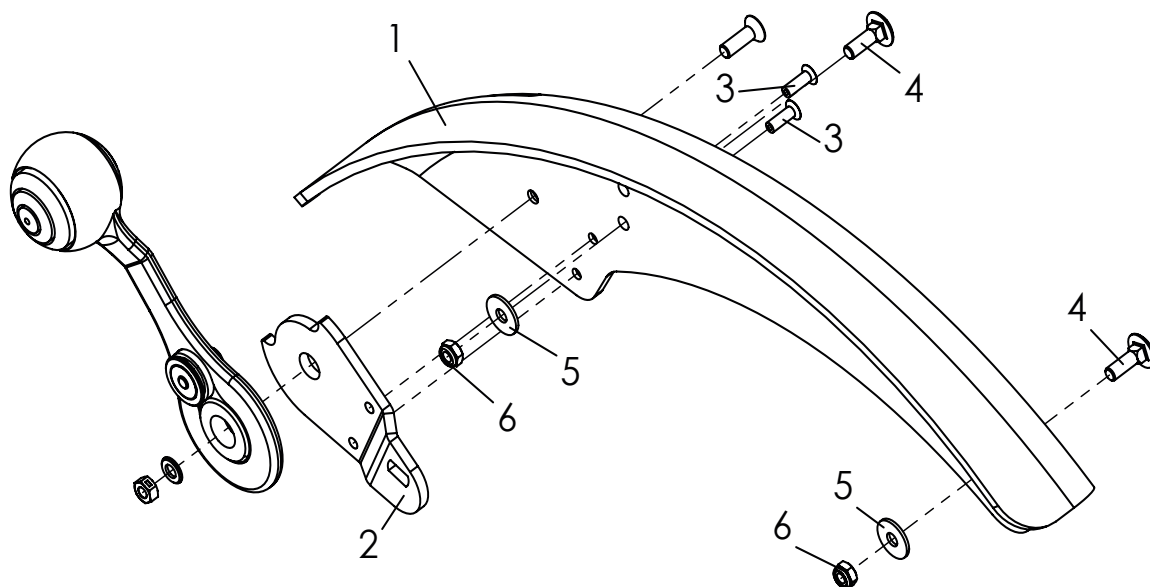


Pos	Article number	Articel description	Quantity	Unit
1-3	3248-1611E	Fender left 20" with base plate for break lever	1	Pc.
	3248-1621E	Fender right 20" with base plate for break lever	1	Pc.
	3248-1631E	Fender left 22" with base plate for break lever	1	Pc.
	3248-1641E	Fender right 22" with base plate for break lever	1	Pc.
	3248-1651E	Fender left 24" with base plate for break lever	1	Pc.
	3248-1661E	Fender right 24" with base plate for break lever	1	Pc.
	3248-1613E	Fender (for 3° camber) left 20" with base plate for break lever	1	Pc.
	3248-1623E	Fender (for 3° camber) right 20" with base plate for break lever	1	Pc.
	3248-1633E	Fender (for 3° camber) left 22" with base plate for break lever	1	Pc.
	3248-1643E	Fender (for 3° camber) right 22" with base plate for break lever	1	Pc.
	3248-1653E	Fender (for 3° camber) left 24" with base plate for break lever	1	Pc.
	3248-1663E	Fender (for 3° camber) right 24" with base plate for break lever	1	Pc.
2	3251-1741	Base plate brake lock lever, left	1	Pc.
	3251-1751	Base plate brake lock lever, right	1	Pc.
3	1063-0412	Blind rivet S4x12	2	Pc.
4	1026-0516	Mushroom head square neck bolt M5x16	2	Pc.
5	1055-0005	Washer A5,3	2	Pc.
6	1042-0005	Hexagon nut self-locking M5	2	Pc.

Spare parts

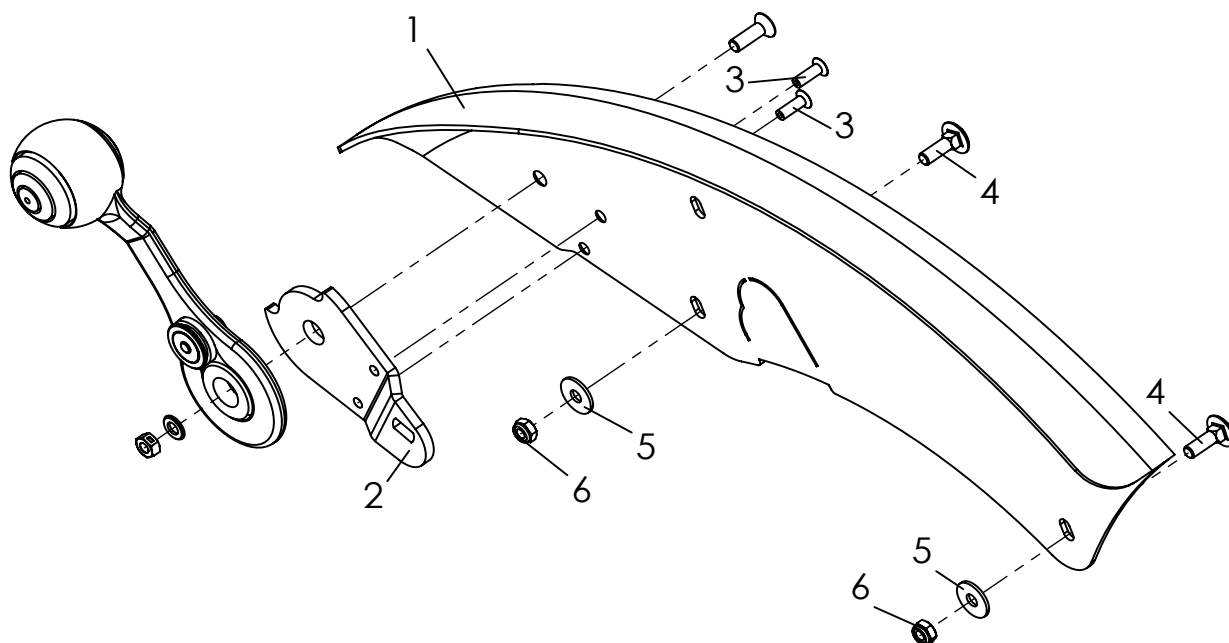


Pos	Article number	Articel description	Quantity	Unit
1-3	3278-1651E-000	Fender Aluminum left 22-24" in frame color with base plate for break lever	1	Pc.
	3278-1661E-000	Fender Aluminum right 22-24" in frame color with base plate for break lever	1	Pc.
	3278-1653E-000	Fender Aluminum (for 3° camber) left 22-24" in frame color with base plate for break lever	1	Pc.
	3278-1663E-000	Fender Aluminum (for 3° camber) left 22-24" in frame color with base plate for break lever	1	Pc.
2	3251-1741	Base plate brake lock lever, left	1	Pc.
	3251-1751	Base plate brake lock lever, right	1	Pc.
3	1063-0412	Blind rivet S4x12	2	Pc.
4	1026-0516	Mushroom head square neck bolt M5x16	2	Pc.
5	1055-0005	Washer A5,3	2	Pc.
6	1042-0005	Hexagon nut self-locking M5	2	Pc.

Spare parts

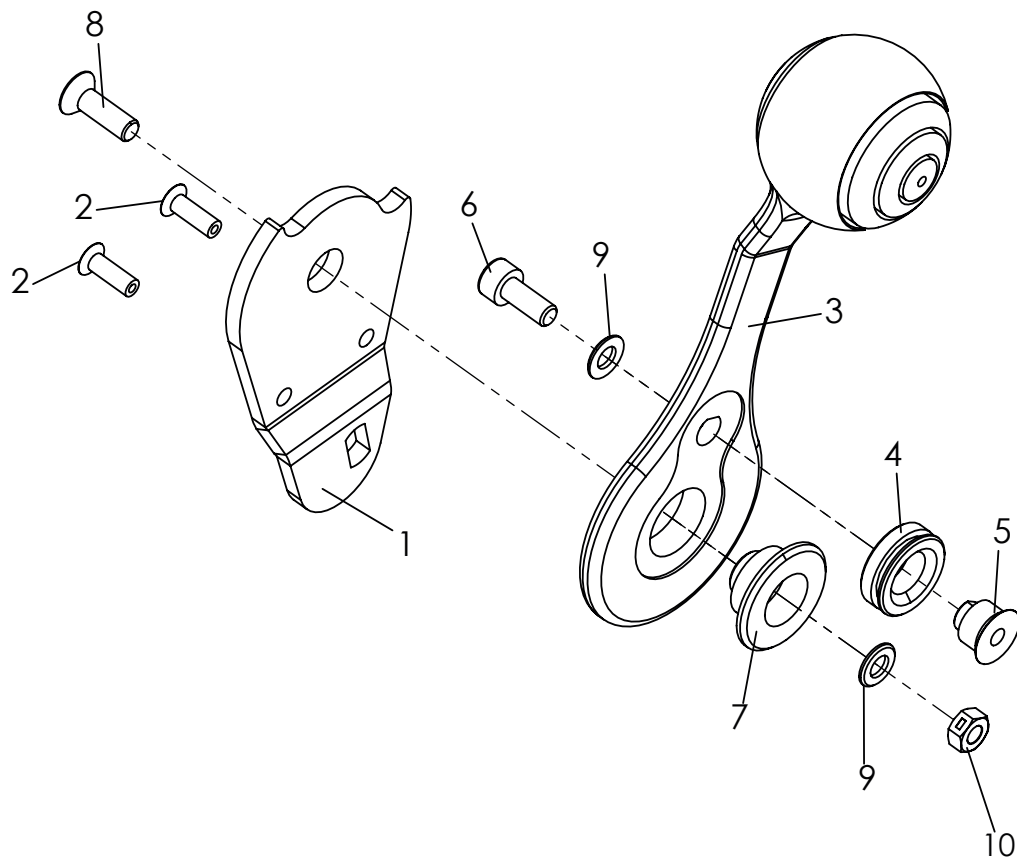


Pos	Article number	Articel description	Quantity	Unit
1-10	3240-1780E	Brake lock lever, left, displayed	1	Pc.
	3240-1790E	Brake lock lever, right	1	Pc.
1	3251-1741	Base plate brake lock lever, left	1	Pc.
	3251-1751	Base plate brake lock lever, right	1	Pc.
2	1063-0412	Blind rivet S4x12	2	Pc.
3	3240-1781	Brake lock lever	1	Pc.
4	3240-1747	Cable roll	1	Pc.
5	3240-1745	Cable support	1	Pc.
6	1021-0512	Cylinder head screw M5x12	1	Pc.
7	3240-1743	Lever bearing	1	Pc.
8	1024-0516	Hexagon socket countersunk head screw M5x16	1	Pc.
9	1051-0005	Washer A5,3	2	Pc.
10	1049-0066	BILOC-nut M5	1	Pc.

Spare parts

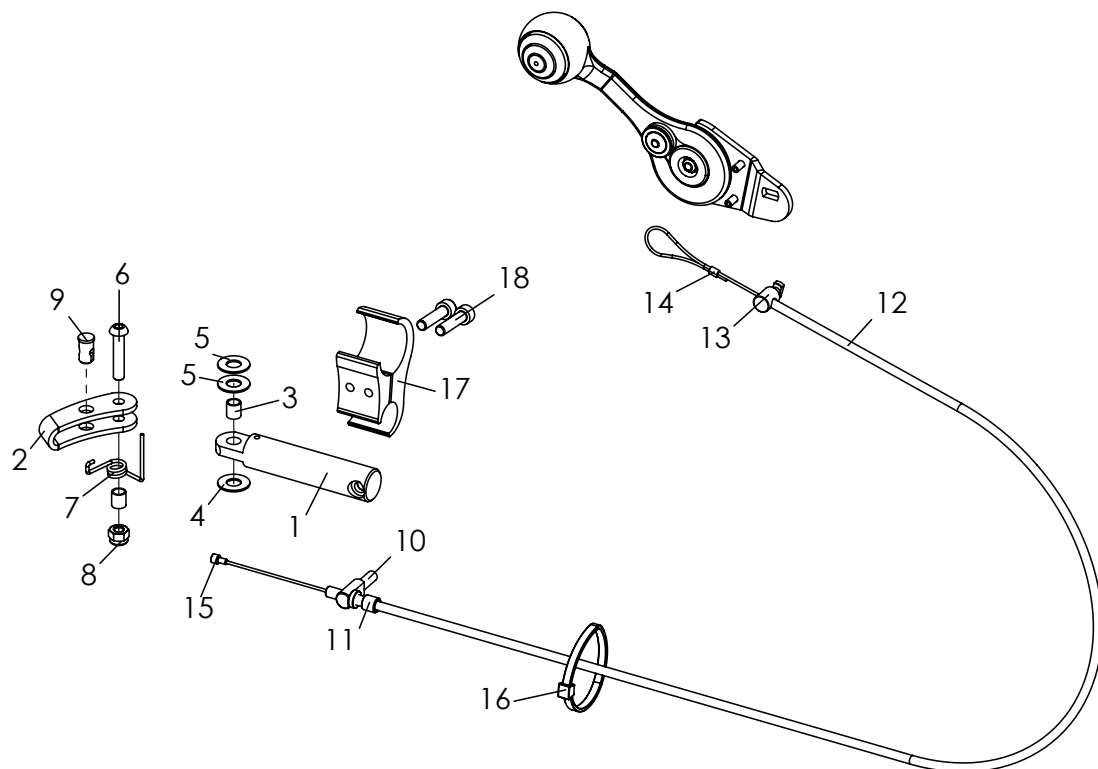


Pos	Article number	Articel description	Quantity	Unit
1-9	3240-1710E	Brake, left, displayed	1	Pc.
	3240-1720E	Brake, right	1	Pc.
	3248-1710E	Brake with long brake lever holder, left	1	Pc.
	3248-1720E	Brake with long brake lever holder, right	1	Pc.
1	3240-1711-007	Brake lever bracket	1	Pc.
	3240-1711-008	Brake lever braket long	1	Pc.
2	3240-1712-007	Brake lock lever, anodised	1	Pc.
3	1114-0055	Bushing	2	Pc.
4	1113-0035	Washer PA 18x8,4x2 black	1	Pc.
5	1113-0003	Washer PA 18x8,4x1 black	2	Pc.
6	1029-0630	Button head socket screw M6x30	1	Pc.
7	3240-1761	Retaining spring, left	1	Pc.
	3240-1762	Retaining spring, right	1	Pc.
8	1042-0006	Hexagon nut self-locking M6	1	Pc.
9	1099-0014	Ball end cable holder	1	Pc.
10-16	1209-0025E	Brake cable for Swingbo VTI	2	Pc.
	3240-1713	Cross bolt	1	Pc.
	1099-0003	Adjusting screw with nut M6x20/30	1	Pc.
	1209-0012	Break cable	1	Pc.
	3240-1748	Bearing	1	Pc.
	1089-0005	Cable clamp	1	Pc.
	1209-0007	End cap aluminium U-BA-S for Bowden cable wire	1	Pc.
	1089-0004	Cable tie, black, 5x180	1	Pc.
17-18	3251-1730E-007	Clamp 30-19, tube distance 42 mm, black	1	Set
	3251-1730E-007	Clamp 30-19, tube distance 42 mm, black	2	Pc.
	1021-0625	Hexagon socket head cap screw M6x25	2	Pc.

Spare parts

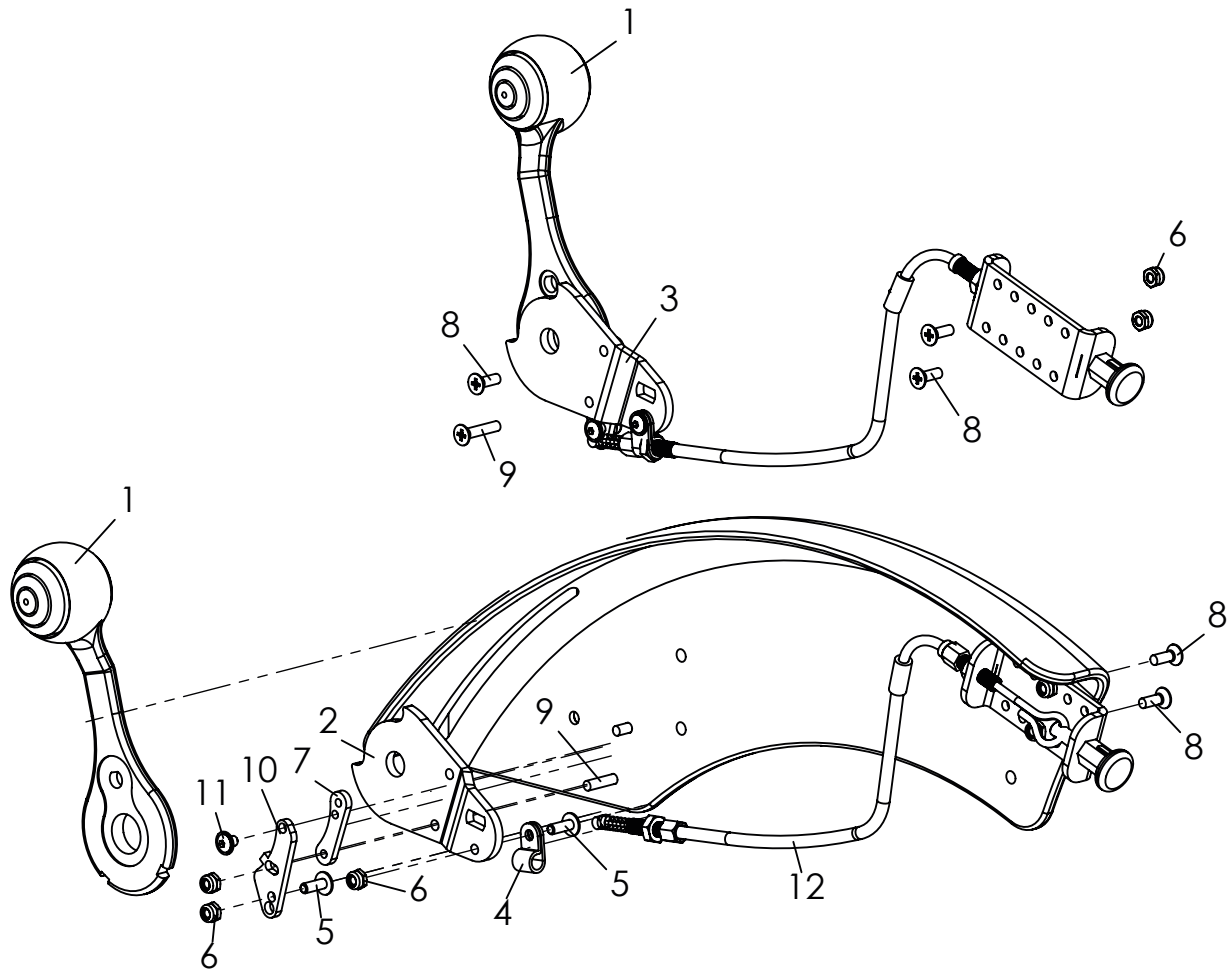


Pos	Article number	Article description	Quantity	Unit
1-12	SA-0505-KP	Locking of the integrated brake	1	Pc.
1	3240-1781_SA-0505	Brake lock lever with grooves for locking	2	Pc.
2	3251-1741_SA-0505	Base plate brake lock lever, left	1	Pc.
3	3251-1751_SA-0505	Base plate brake lock lever, right	1	Pc.
4	1119-0003	Wire clamp	2	Pc.
5	1030-0412	Button head socket screw M4x12	4	Pc.
6	1042-0004	Hexagon nut self-locking M4	10	Pc.
7	SA-0505-0001	Baseplate VA	2	Pc.
8	1023-0412	Countersunk flat head screw M4x12	6	Pc.
9	1023-0420	Countersunk flat head screw M4x20	2	Pc.
10	SA-0505-0002	Swivel lock VA	2	Pc.
11	1030-0406	Button head socket screw with flange zp M4x6	2	Pc.
12	SA-0505-1110	Release with Bowden cable, complete	2	Pc.

Spare parts

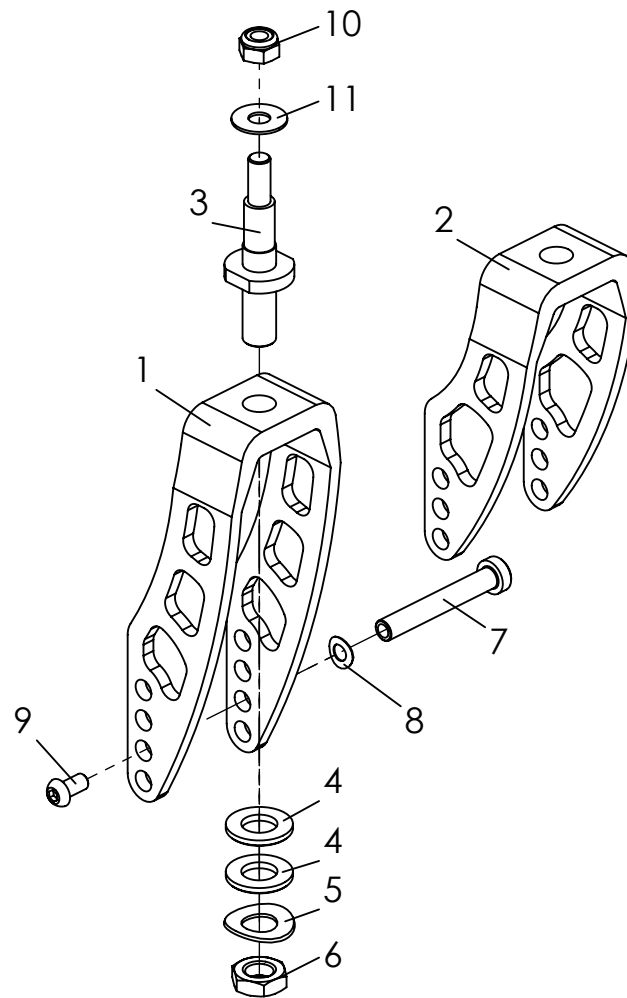

Pos	Article number	Articel description	Quantity	Unit
1	1281-0017-007	Wheel fork aluminum, long, black anodized	1	Pc.
	1281-0017-000	Wheel fork aluminum, long, Frame Colour	1	Pc.
2	1281-0016-007	Wheel fork aluminum, short, black anodized	1	Pc.
	1281-0016-000	Wheel fork aluminum, short, Frame Colour	1	Pc.
3	1205-0007	Wheel fork axle	1	Pc.
4	1051-0012	Washer A13	2	Pc.
5	1052-0012	Spring washer B12	1	Pc.
6	1043-0012	Hexagon nut M12	1	Pc.
7	1205-0009	Wheel axle L54	1	Pc.
8	1052-0008	Spring washer B8	1	Pc.
9	1029-6001	Button head socket screw,Tufflock M6x12	1	Pc.
10	1042-0008	Hexagon nut self-locking M8	1	Pc.
11	1113-0071	Friction washer	1	Pc.

Spare parts

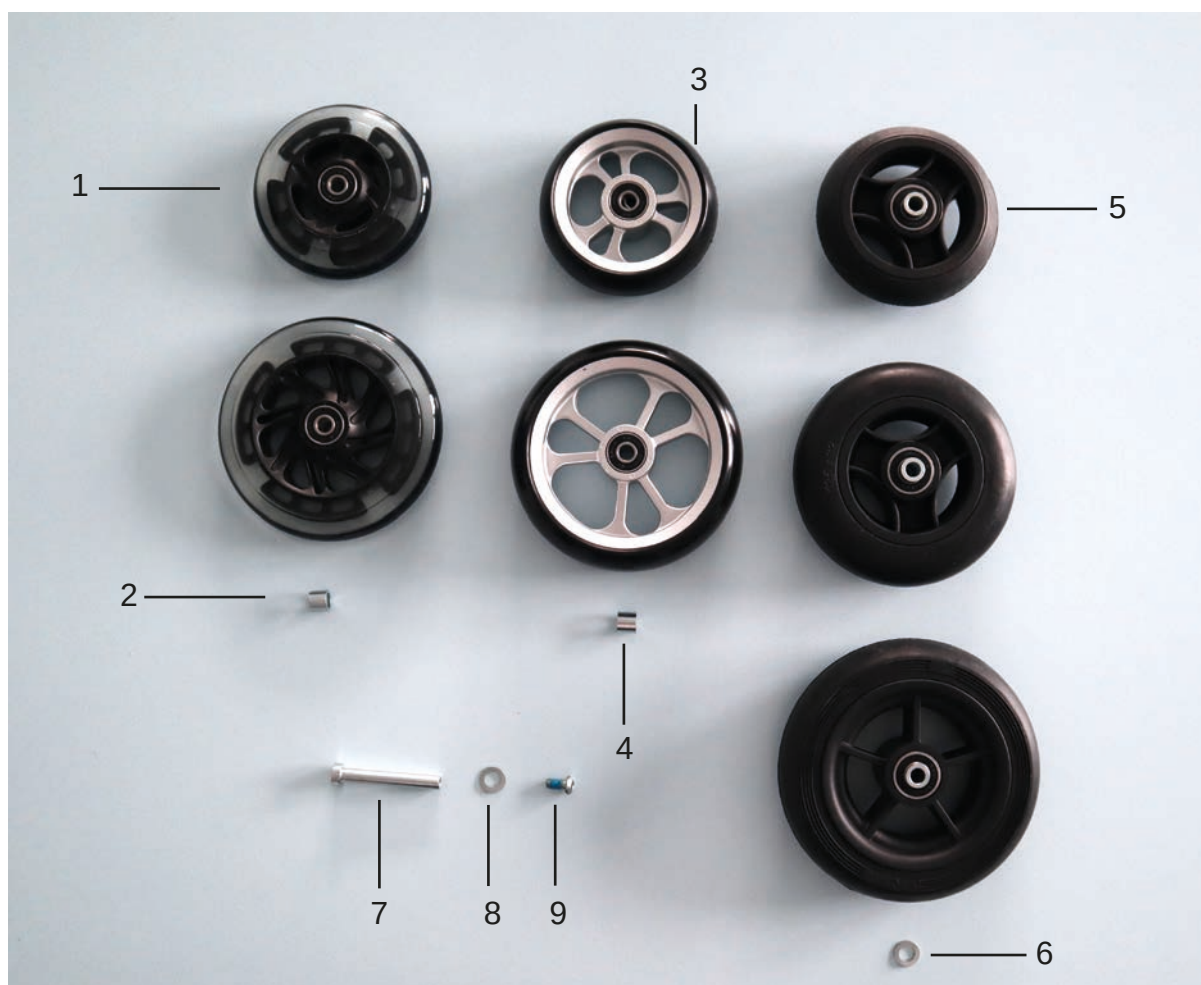


Pos	Article number	Articel description	Quantity	Unit
1	1249-0005	Skater wheels with illuminating diode 100x24 (max. 5 Nm)	2	Pc.
	1249-0006	Skater wheel with illuminating diode 125x24 (max. 5 Nm)	2	Pc.
2	1205-0047	spacer for Skater wheels 12x1.75x10.3 verz.	4	Pc.
3	1221-0024	PU-front castor with design plastic rim 100x34	2	Pc.
	1221-0025	PU-front castor with design plastic rim 125x34	2	Pc.
4	1205-0075	spacer for Skater wheels 12x1.75x9	4	Pc.
5	1250-0005	Front castor with solid rubber tires 100x32, black	2	Pc.
	1250-0006	Front castor with solid rubber tires 125x32, black	2	Pc.
	1250-1460	Front castor with solid rubber tires 140x40, black	2	Pc.
6	1052-0008	Washer A8,4 for front castor with solid rubber tires	2	Pc.
7	1205-0009	Wheel axle L54	2	Pc.
8	1052-0008	Spring washer B8 zp DIN/ISO 137 B8	2	Pc.
9	1029-6001	Button head socket screw,Tuflock M6x12	2	Pc.

Spare parts



Pos	Article number	Articel description	Quantity	Unit
1	1211-0007	Pneumatic wheel 20x1" HOGGI light with out push rims	2	Pc.
	1211-0008	Pneumatic wheel 22x1" Hoggi light w/o push rims	2	Pc.
	1211-0009	pneumatic wheel 24x1" Hoggi light w/oithout push rims	2	Pc.
	1221-0009	Rear wheels with PU-tires 20x1 3/8" HOGGI light w/o push rims	2	Pc.
	1221-0049	Rear wheels with PU-tires 22x1" HOGGI light w/o push rims, black	2	Pc.
	1221-0011	Rear wheels with PU-tires 24x1" HOGGI light w/o push rims	2	Pc.
	1221-0050	Rear wheels with PU-tires 24x1" HOGGI light w/o push rims, black	2	Pc.
2	1205-0010	Adapter with nut M16x1 for quick-release-axle 12/45 mm	2	Pc.
3	1205-0014	Quick release axle 12 x 98 mm	2	Pc.
4	1052-0016	Spring washer B16	2	Pc.
<i>Spare parts</i>				
	1201-0003	Tube for wheel 20x1	2	Pc.
	1201-0004	Tube for wheel 22x1	2	Pc.
	1201-0005	Tube for wheel 24x1	2	Pc.
	1201-0006	Rim band 20"	2	Pc.
	1201-0007	Rim band 22"	2	Pc.
	1201-0008	Rim band 24"	2	Pc.
	1200-0003	Tire for wheel 20x1" grey	2	Pc.
	1200-0019	Tire for wheel 22x1" RightRun HS387 Light	2	Pc.
	1200-0023	Tire for wheel 24x1" RightRun HS387 Light	2	Pc.
	1203-0005	PU-tire 20x1 3/8" grey	2	Pc.
	1203-0006S	PU-tire 22x1" black	2	Pc.
	1203-0007S	PU-tire 24x1" black	2	Pc.


Spare parts

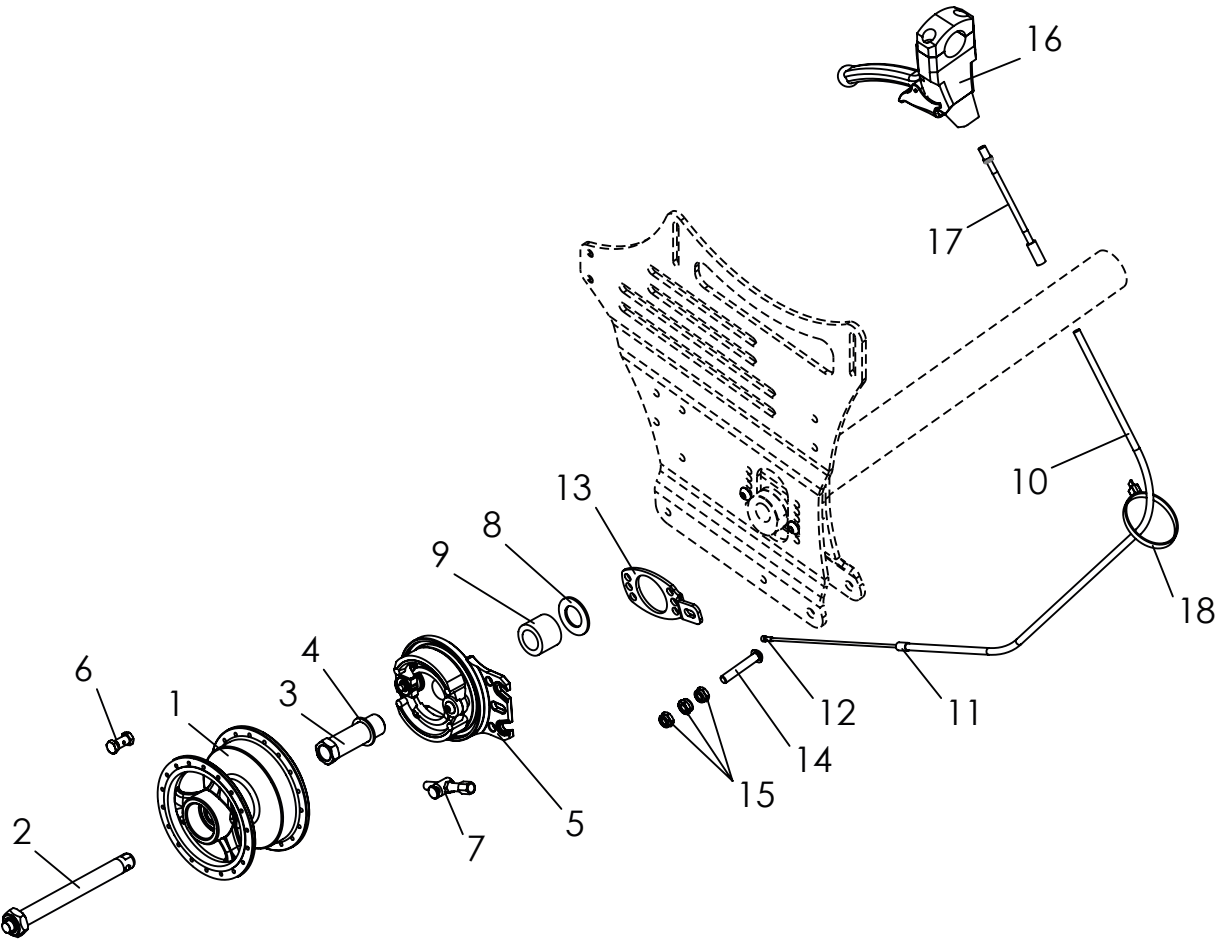

Spare parts

Pos	Article number	Articel description	Quantity	Unit
1-7	1219-0013	Pneumatic wheel 20x1" „HOGGI light“ with drumbrake, left, w/o push rims	1	Pc.
	1219-0014	Pneumatic wheel 20x1" „HOGGI light“ with drumbrake, right w/o push rim	1	Pc.
	1211-0045	Pneumatic wheel 22x1" „HOGGI light“ with drumbrake, left, w/o push rim	1	Pc.
	1211-0046	Pneumatic wheel 22x1" „HOGGI light“ with drumbrake, right, w/o push rim	1	Pc.
	1211-0048	Pneumatic wheel 24x1" „HOGGI light“ with drumbrake, left, w/o push rim	1	Pc.
	1211-0049	Pneumatic wheel 24x1" „HOGGI light“ with drumbrake, right w/o push rim	1	Pc.
	1221-0003	PU-wheel 20x1 3/8" „HOGGI light“ with drum brake, left, w/o push rim	1	Pc.
	1221-0004	PU-wheel 20x1 3/8" „HOGGI light“ with drum brake, right, w/o push rim	1	Pc.
	1221-0045	PU-wheel 22"x1" „HOGGI light“ with drum brake, left, w/o push rim	1	Pc.
	1221-0046	PU-wheel 22"x1" „HOGGI light“ with drum brake, right, w/o push rim	1	Pc.
	1221-0047	PU wheels 24x1" with drum brakes, left, w/o push rims	1	Pc.
	1221-0048	PU wheels 24x1" with drum brakes, right, w/o push rims	1	Pc.
2	1205-0022	Quick release axle 12 x 108 mm	2	Pc.
3	1205-0051	Adapter M16x1 for quick-release-axle 12/57 mm	2	Pc.
4	1059-0002	Shim ring 16x22x1,5	2	Pc.
5	1205-0031	Brake plate left	1	Pc.
	1205-0032	Brake plate right	1	Pc.
6	1205-0029	Pinch bolt d8x15 mm, without slot	2	Pc.
7	1205-0030	Brake adjuster M6x30 mm	2	Pc.
8	1052-0016	Spring washer B16	2	Pc.
9	1205-0015	Spacer bushing 25x16.2x15, for 3 and 6°	2	Pc.
	1205-0016	Spacer bushing 25x16.2x20, for 9°	2	Pc.
10-12	1209-0001	Bowden wire set 1,5m cable	2	Pc.
13	1205-0035-001	Brake anchor support plate SBO-VTi	2	Pc.
14	1029-0635	Button head socket screw M6x35, for 9°	2	Pc.
	1022-0625	Hexagon socket head cap screw M6x25, for 3 and 6°	2	Pc.
15	1042-0006	Hexagon nut self-locking M6	3	Pc.
16	1209-0026	Brake lever 36° manually locking 25 mm (for push-handle)	2	Pc.
	1209-0030	Brake lever 36 ° manually locking 22 mm (for push bar)	2	Pc.
17	1205-0054	Bowden cable guide, flexible	2	Pc.
18	1089-0004	Cable tie, black, 5x180	4	Pc.

Spare parts

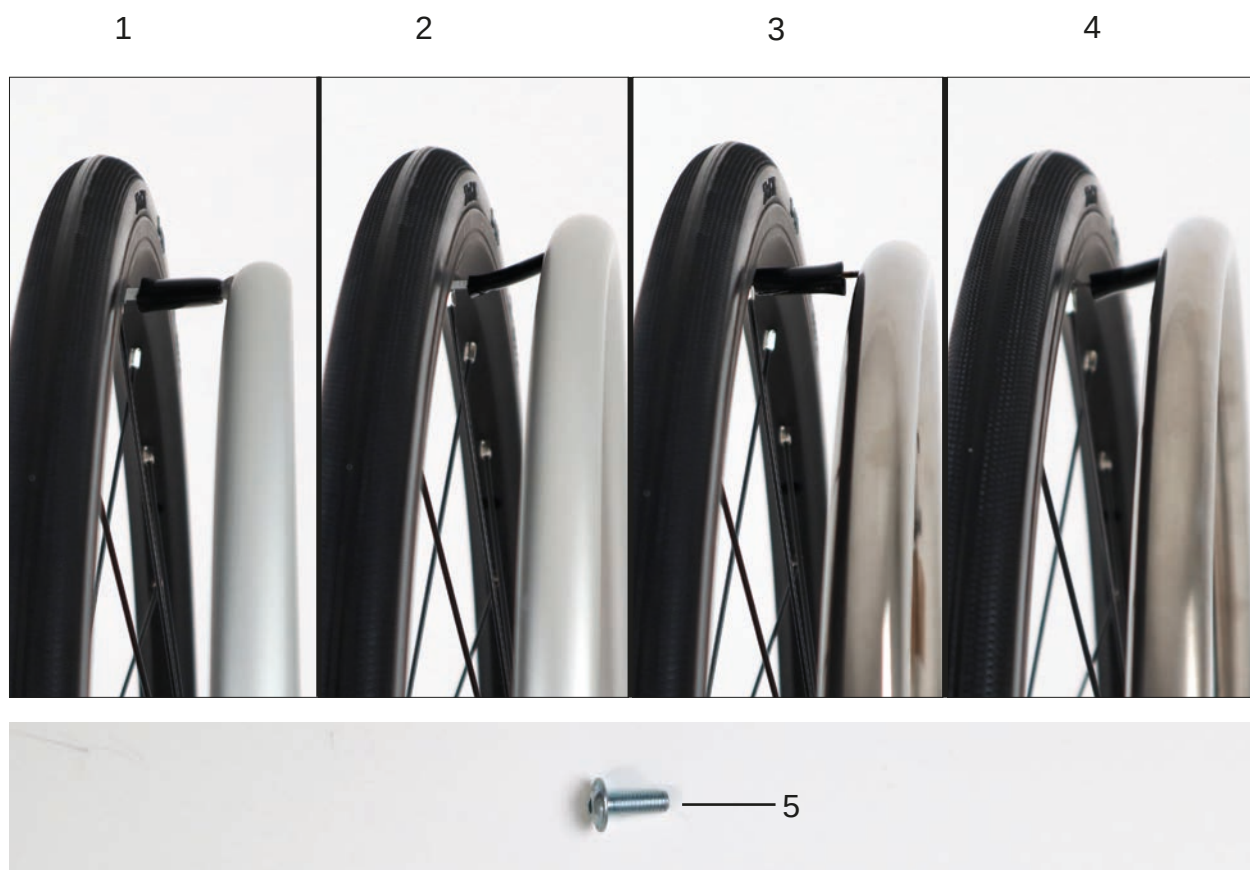


Pos	Article number	Articel description	Quantity	Unit
1	1204-0006	Aluminum hand rim 20", anodised - „standard“	2	Pc.
	1204-0006-007	Aluminum hand rim 20", black anodised - „standard“	2	Pc.
	1204-0007-007	Aluminum hand rim 22", anodised - „standard“	2	Pc.
	1204-0008	Aluminum hand rim 24", anodised - „standard“	2	Pc.
	1204-0008-007	Aluminum hand rim 24", black anodised - „standard“	2	Pc.
2	1204-0019	Aluminum hand rim 20", anodised - for mounting position „high“	2	Pc.
	1204-0020	Aluminum hand rim 22", anodised - for mounting position „high“	2	Pc.
	1204-0020-007	Aluminum hand rim 22", black anodised - for mounting position „high“	2	Pc.
	1204-0021	Aluminum hand rim 24", anodised - for mounting position „high“	2	Pc.
	1204-0021-007	Aluminum hand rim 24", black anodised - for mounting position „high“	2	Pc.
3	1204-0009	Stainless steel hand rim 20" - "standard"	2	Pc.
	1204-0010	Stainless steel hand rim 22" - „standard“	2	Pc.
	1204-0011	Stainless steel hand rim 24" - „standard“	2	Pc.
4	1204-0022	Stainless steel hand rim 20" - for mounting position „high“	2	Pc.
	1204-0023	Stainless steel hand rim 22" - for mounting position „high“	2	Pc.
	1204-0024	Stainless steel hand rim 24" - for mounting position „high“	2	Pc.

5 1030-0514 Button head socket screw with flange M5x14

(Handrims 20 "and 22" with 4 mounting points)

(Handrims 24" with 6 mounting points)

Spare parts

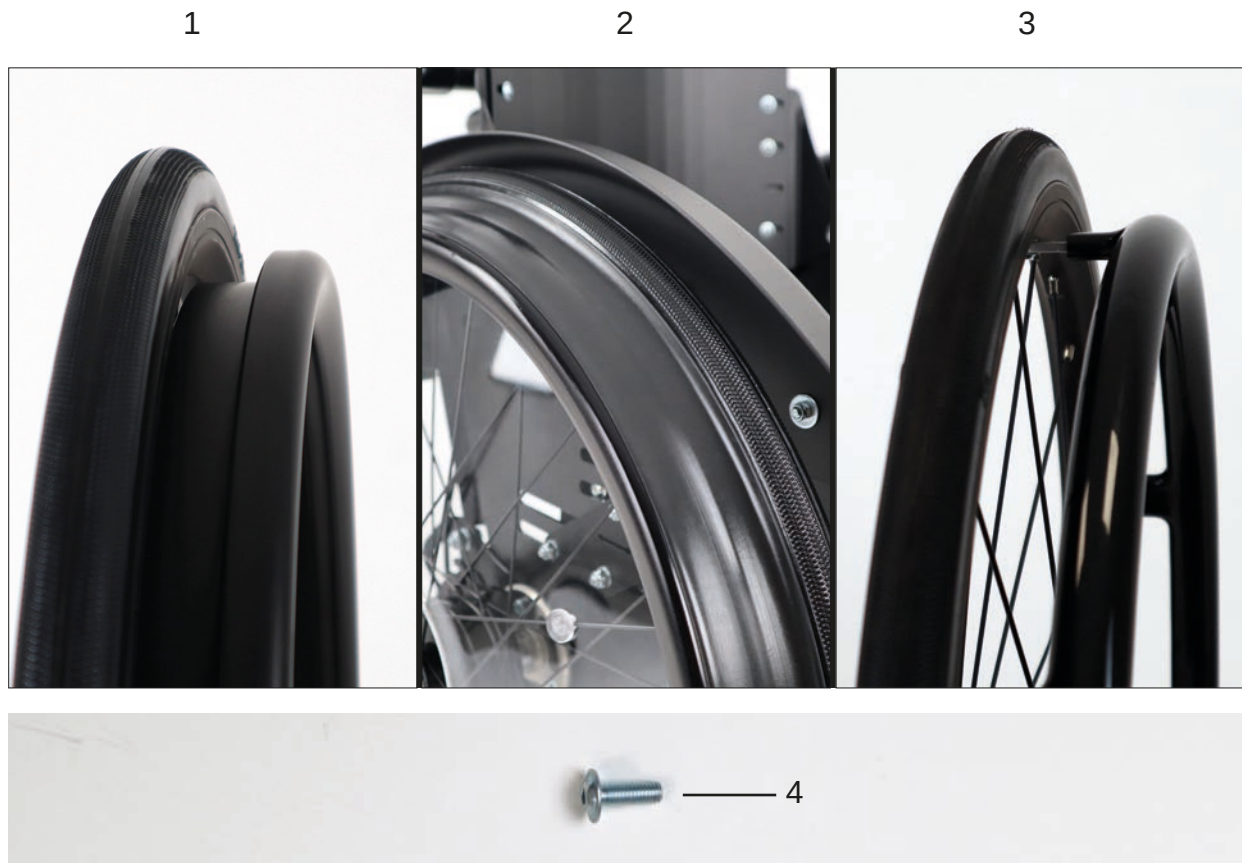


Pos	Article number	Articel description	Quantity	Unit
1	1204-0055-007	Closed ergonomic Aluminum hand rim 22", black anodised	2	Pc.
	1204-0056-007	Closed ergonomic Aluminum hand rim 24", black anodised	2	Pc.
2	1204-0060	Rubber profile 22" Grip+	2	Pc.
	1204-0061	Rubber profile 24" Grip+	2	Pc.
	1204-0042	Aluminum hand rim 20", black PVC coated - „standard“	2	Pc.
	1204-0045	Aluminum hand rim 20", black PVC coated - for mounting position „high“	2	Pc.
	1204-0043	Aluminum hand rim 22", black PVC coated - „standard“	2	Pc.
	1204-0046	Aluminum hand rim 22", black PVC coated - for mounting position „high“	2	Pc.
	1204-0044	Aluminum hand rim 24", black PVC coated - „standard“	2	Pc.
	1204-0047	Aluminum hand rim 24", black PVC coated - for mounting position „high“	2	Pc.
4	1030-0514	Button head socket screw with flange M5x14		
		(Handrims 20 "and 22" with 4 mounting points)		
		(Handrims 24" with 6 mounting points)		

Spare parts




Pos	Article number	Articel description	Quantity	Unit
1	7000-0150	Maxgrepp Original push rim 20", 4 L	2	Pc.
	7000-0151	Maxgrepp Original push rim 22", 4L	2	Pc.
	7000-0152	Maxgrepp Original push rim 24", 6L	2	Pc.
2	7000-0162	Maxgrepp Ergo Para push rim 20", 4 L	2	Pc.
	7000-0163	Maxgrepp Ergo Para push rim 22", 4 L	2	Pc.
	7000-0164	Maxgrepp Ergo Para push rim 24", 6 L	2	Pc.
3	7000-0165	Maxgrepp Ergo Para CP push rim 20", 4 L	2	Pc.
	7000-0166	Maxgrepp Ergo Para CP push rim 22", 4 L	2	Pc.
	7000-0167	Maxgrepp Ergo Para CP push rim 24", 6 L	2	Pc.

Spare parts

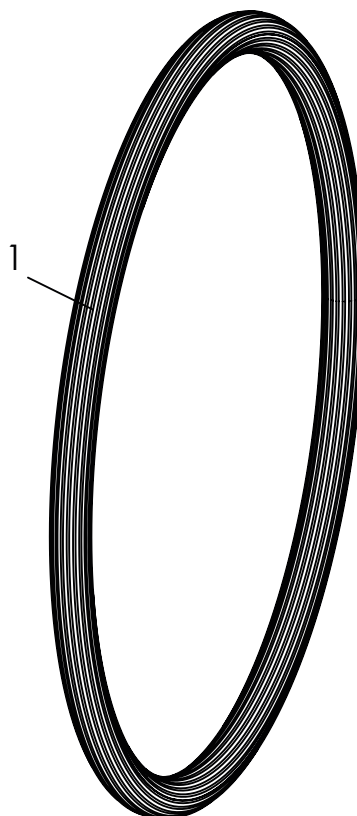

Pos	Article number	Articel description	Quantity	Unit
1	1204-0015	Push rim cover in silicone, 20", black fluted	1	Pc.
	1204-0016	Push rim cover in silicone, 22", black fluted	1	Pc.
	1204-0017	Push rim cover in silicone, 24", black fluted	1	Pc.


Spare parts

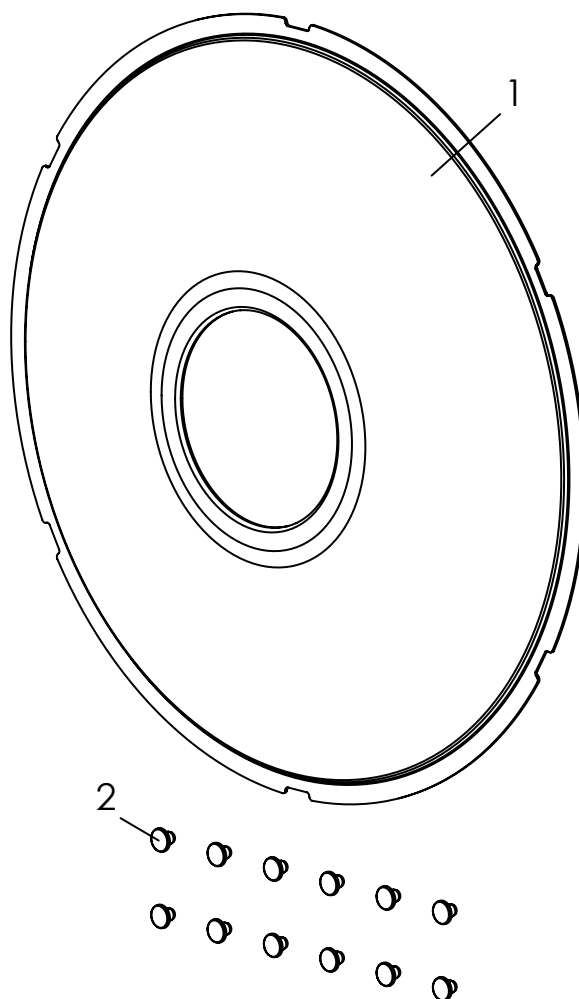

Pos	Article number	Articel description	Quantity	Unit
1-2	1204-1012	Spoke guard 20", transparent	1	Pc.
	1204-1013	Spoke guard 22", transparent	1	Pc.
	1204-1014	Spoke guard 24", transparent	1	Pc.
	1204-1020	Spoke guard 20", for drum brake wheels, transparent	1	Pc.
	1204-1018	Spoke guard 22", for drum brake wheels, transparent	1	Pc.
	1204-1019	Spoke guard 24", for drum brake wheels, transparent	1	Pc.
1	1204-0012	Spoke guard 20", transparent	2	Pc.
	1204-0031	Spoke guard 22", transparent	2	Pc.
	1204-0032	Spoke guard 24", transparent	2	Pc.
	1204-0013			
	1204-0033	Spoke guard 22", for drum brake wheels, transparent	2	Pc.
	1204-0034	Spoke guard 24", for drum brake wheels, transparent	2	Pc.
2	1205-0033	Fastening clips for Spoke guard transparent ABS	12	Pc.
3	1204-2001-xxx	Special inside foiling of spoke guards: Specify the motif when ordering	1	Set


The names of the motifs can be found at the following URL:
<https://hoggi.de/en/products/spoke-guards>

Spare parts

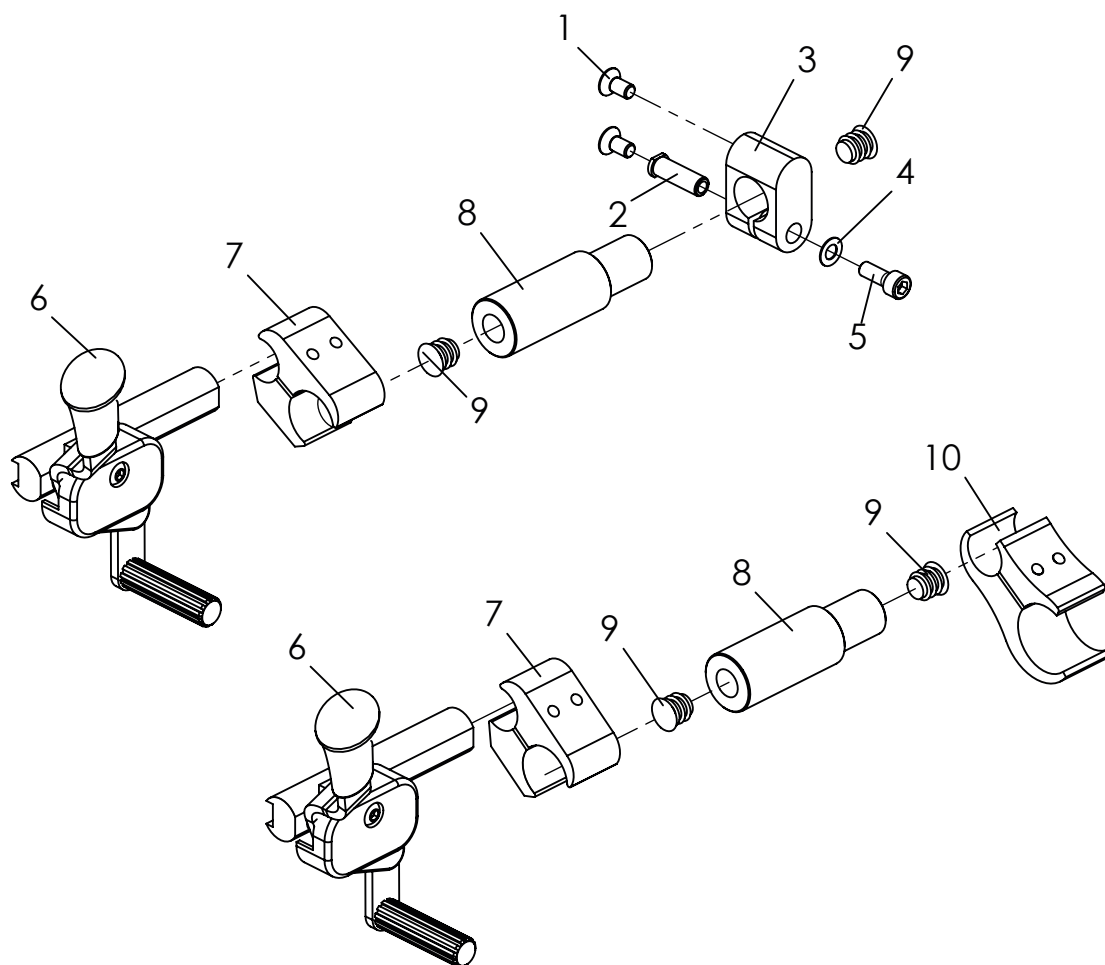


Pos	Article number	Articel description	Quan	Unit
1-9	3249-1750E	Standard knee lever brakes, mounting on base plate	1	Pair
6-10	3249-1751E	Standard knee lever brakes, mounting at the lower frame tube	1	Pair
1	1024-0612	Hexagon socket countersunk head screw zp DIN/ISO 7991 M6x12	4	Pc.
2	3275-1731	Brake clamp Bearing	2	Pc.
3	3249-1721-007	Clamp for brake mechanism	2	Pc.
4	1052-0006	Spring washer B6	2	Pc.
5	1021-0616	Hexagon socket head cap screw M6x16	2	Pc.
6	1207-0001	Hand brake push left	1	Pc.
	1207-0002	Hand brake push right	1	Pc.
7	1207-0003	Bracket 25/19 with 2 zylinder head srews	2	Pc.
8	3265-1832-006	Transitional shaft 25/19x80	2	Pc.
9	1141-0001	Ribbed insert SFL 14x0,8-2	4	Pc.
10	3251-1730E	Clamp 30-19, tube distance 42 mm, with screws	2	Pc.

Spare parts

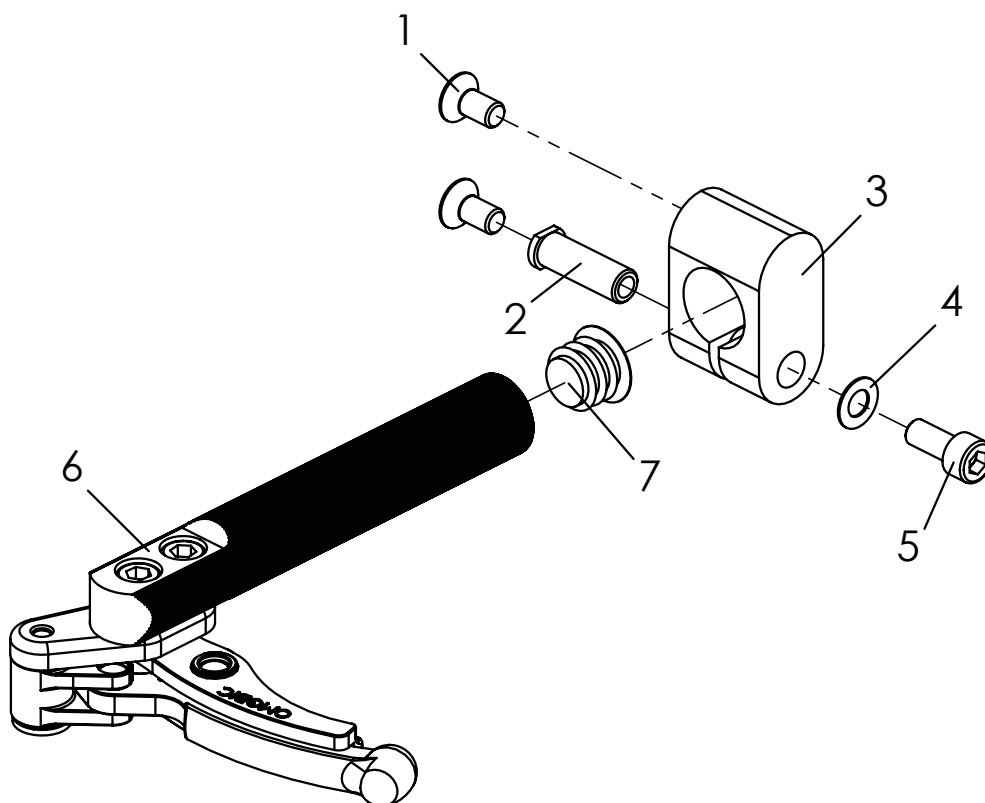


Pos	Article number	Articel description	Quan	Unit
1-7	3249-1760E	HOGGI light sport-brakes, complete	1	Pair
1	1024-0612	Hexagon socket countersunk head screw M6x12	4	Pc.
2	3275-1731	Brake clamp Bearing	2	Pc.
3	3249-1721-007	Clamp for brake mechanism black anodized	2	Pc.
4	1052-0006	Spring washer B6	2	Pc.
5	1021-0616	Hexagon socket head cap screw M6x16	2	Pc.
6	1207-0006	Brake sport left	1	Pc.
	1207-0007	Brake sport right	1	Pc.
7	1141-0005	Ribbed insert SFL 19x0,8-2,5	2	Pc.

Spare parts

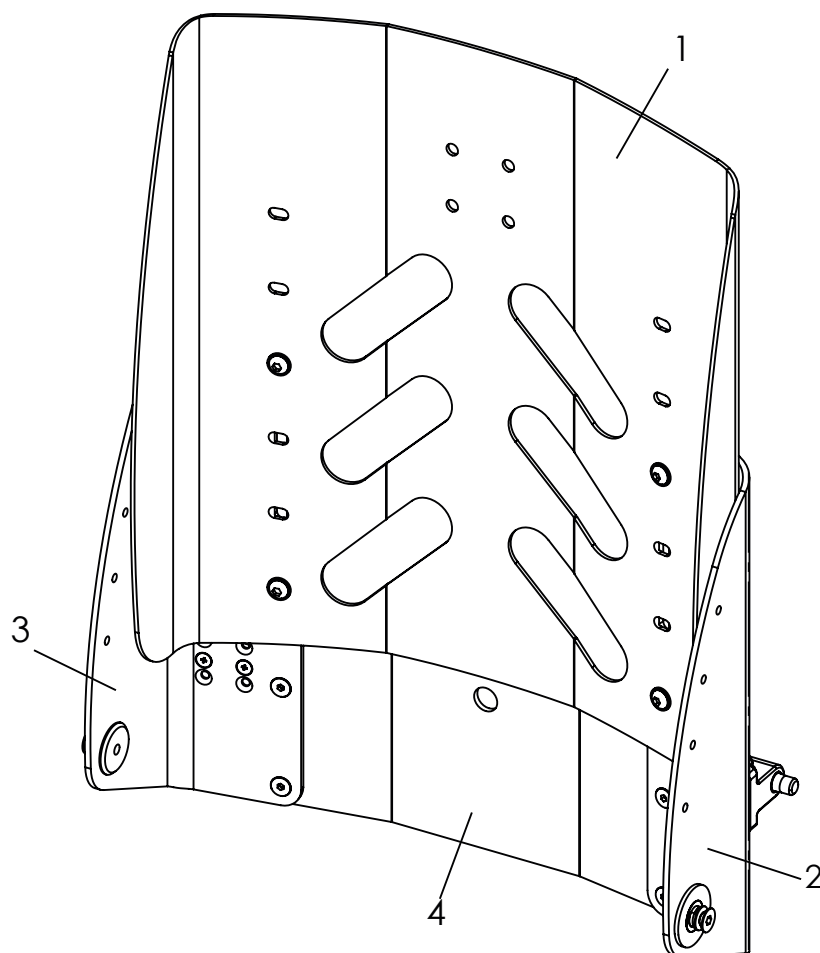


Pos	Article number	Articel description	Quantity	Unit
1-4	3245-29SW-000	Back panel angle- and height adjustable, in frame colour	1	Pc.
1	3245-28SWE-000	Back panel, in frame colour	1	Pc.
2	3245-2610E-000	Back rest base, left, in frame colour	1	Pc.
3	3245-2620E-000	Back rest base, right, in frame colour	1	Pc.
4	3245-27SWE-000	Cross member, in frame colour	1	Pc.

Spare parts

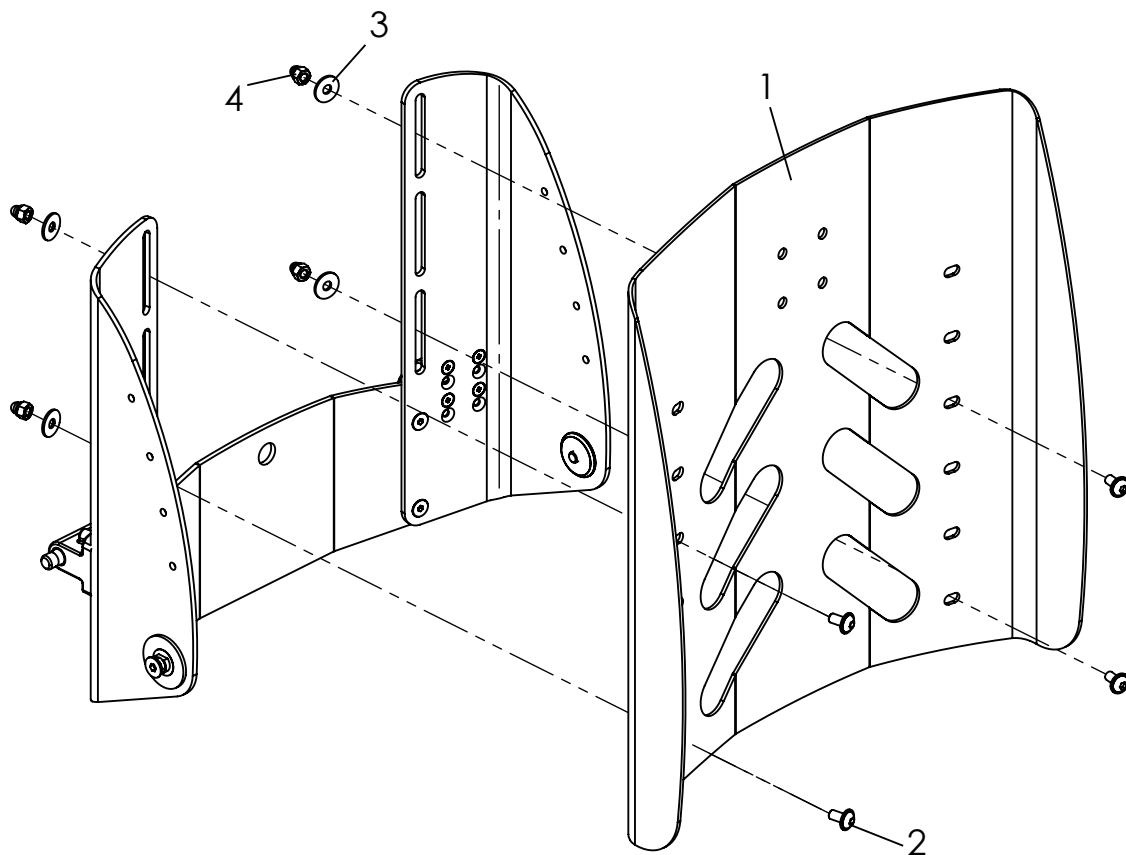


Pos	Article number	Articel description	Quantity	Unit
1-4	3245-28SWE-000	Back panel, complete	1	Pc.
2	1030-0514	Button head socket screw with flange M5x14	4	Pc.
3	1055-0005	Washer A5,3	4	Pc.
4	1044-0005	Hexagonal domed cap nut self-locking M5	4	Pc.

Spare parts

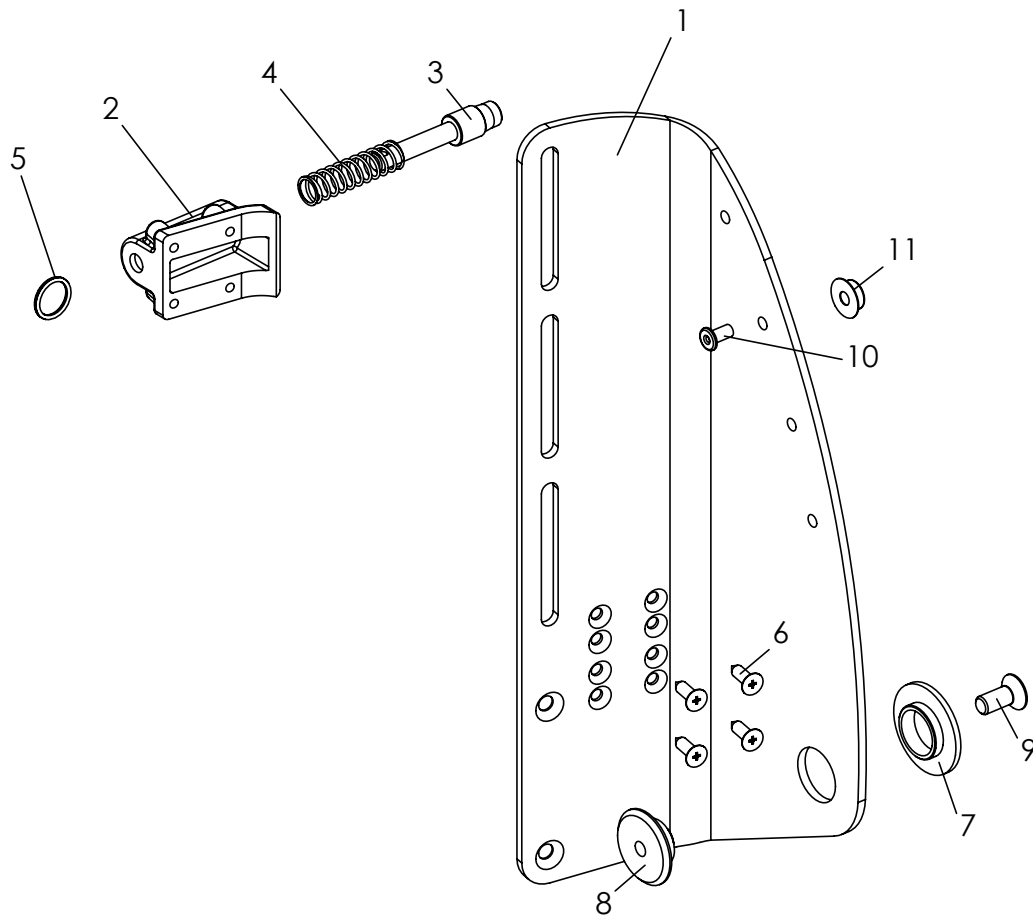


Pos	Article number	Articel description	Quantity	Unit
1-9	3245-2610E-000	Backrest base, left, complete	1	Pc.
	3245-2620E-000	Backrest base, right, complete	1	Pc.
1	3245-2611	Backrest base, left, size 1	1	Pc.
	3245-2621	Backrest base, right, size	1	Pc.
2*	3240-2431	Locking bolt housing	1	Pc.
		* High position 80° up to 120°		
		* Low position 75° up to 115°		
3	3240-2422	Locking bolt	1	Pc.
4	1131-0043	Compression spring D-143F	1	Pc.
5	1129-0036	Key ring 16x1,6	1	Pc.
6	1039-0006	Tapping screw with countersunk head	4	Pc.
7	3245-2613	Flange bushing	1	Pc.
8	3245-2614	Bearing shaft	1	Pc.
9	1024-0612	Hexagon socket countersunk head screw M6x12	1	Pc.
		<i>Pos. 10 and 11 only with upholstery</i>		
10	1062-0406	Blind rivet F4x6	1	Pc.
11	1533-0005	Ring-ball part	1	Pc.

Spare parts

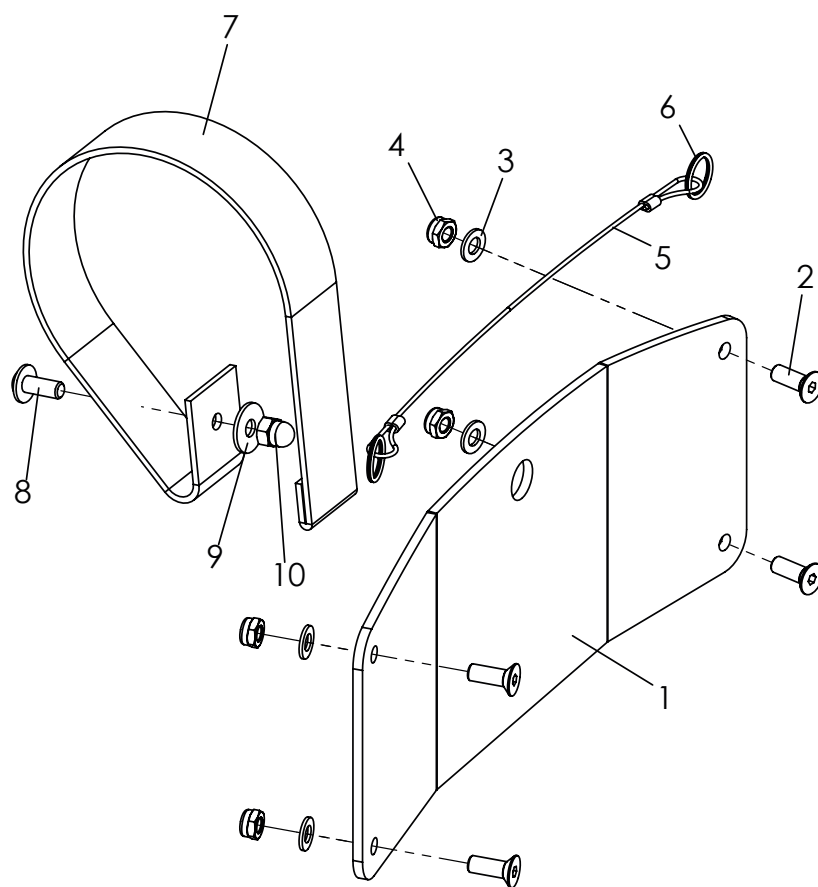

Pos	Article number	Articel description	Quantity	Unit
1-4	3245-27SWE-000	Cross member, complete	1	Pc.
1	3245-27SW-000	Cross member	1	Pc.
2	1024-0514	Hexagon socket countersunk head screw M5x14	4	Pc.
3	1051-0005	Washer A5,3	4	Pc.
4	1042-0005	Hexagon nut self-locking M5	4	Pc.
5-6	3245-26SWE	Wire-rope, for back rest recline	1	Pc.
6	1129-0036	Key ring 16x1,6	1	Pc.
7-10	1512-0012E	Release strap with fasteners	1	Pc.
7	1512-0012	PP strap 25x350, with a sewn loop opposite the perforation	1	Pc.
8	1030-0512	Button head socket screw with flange M5x12	1	Pc.
9	1055-0005	Washer A5,3	1	Pc.
10	1044-0005	Hexagonal domed cap nut self-locking 986 M5	1	Pc.


Spare parts

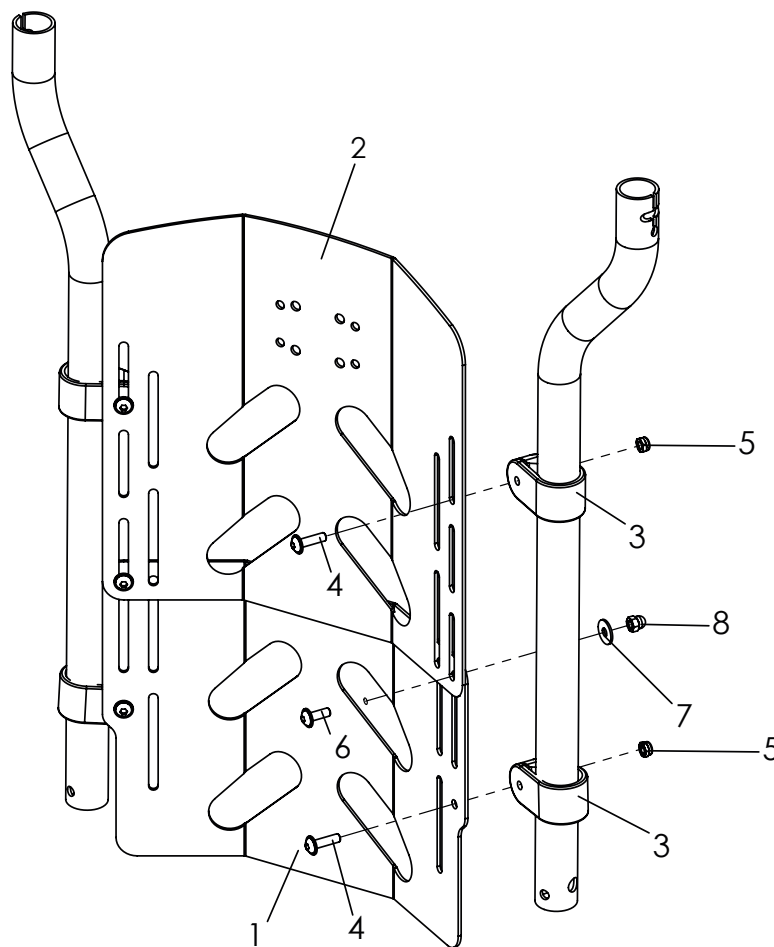

Pos	Article number	Articel description	Quantity	Unit
1-8	3245-32SW-000	Back panel trough shaped, two parts, height adjustable, in frame colour	1	Pc.
1	3245-33SW-000	Base backrest panel	1	Pc.
2	3245-34SW-000	Upper part backrest panel	1	Pc.
3	3245-3311	Gripper clamp	4	Pc.
4	1030-0518	Button head socket screw with flange M5x18	4	Pc.
5	1042-0005	Hexagon nut self-locking M5	4	Pc.
6	1030-0514	Button head socket screw with flange M5x14	2	Pc.
7	1055-0005	Washer A5,3	2	Pc.
8	1044-0005	Hexagonal domed cap nut self-locking M5	2	Pc.
3-5	3245-3310	Four gripper clamps D25 with fasteners	1	Set


* From SW 36, the back panel must be fastened with 6 gripper clamps!

Spare parts

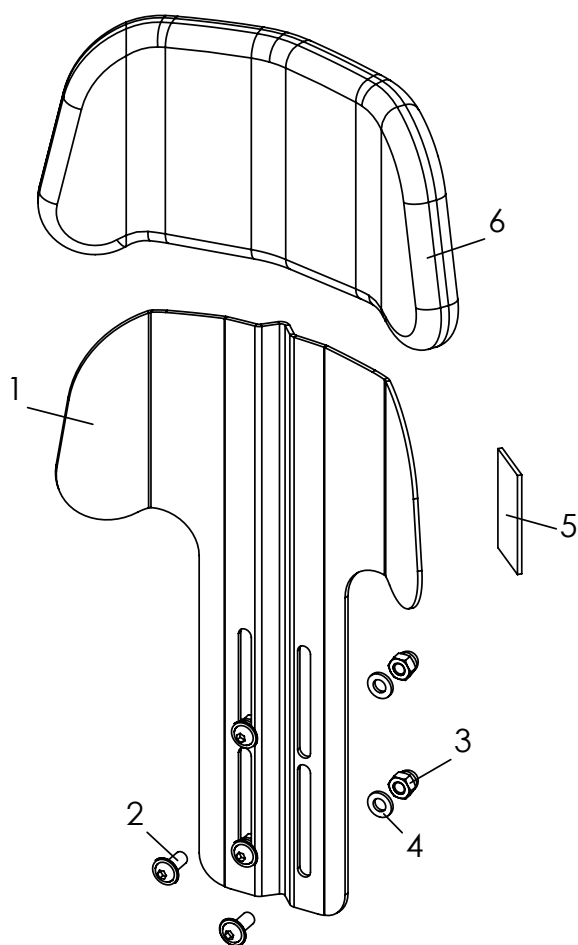


Pos	Article number	Articel description	Quantity	Unit
1-5	3243-7613E-000	Head rest panel size 3, from SW 28, in frame colour	1	Pc.
2	1030-0614	Button head socket screw with flange M6x14	4	Pc.
3	1044-0006	Hexagonal domed cap nut self-locking M6	4	Pc.
4	1051-0006	Washer A6,4	4	Pc.
5	1501-0003	Hook strap black 25mm self adhesive A glue	0,1	Pc.
6		Head cushion size 3, black	1	Pc.

Spare parts

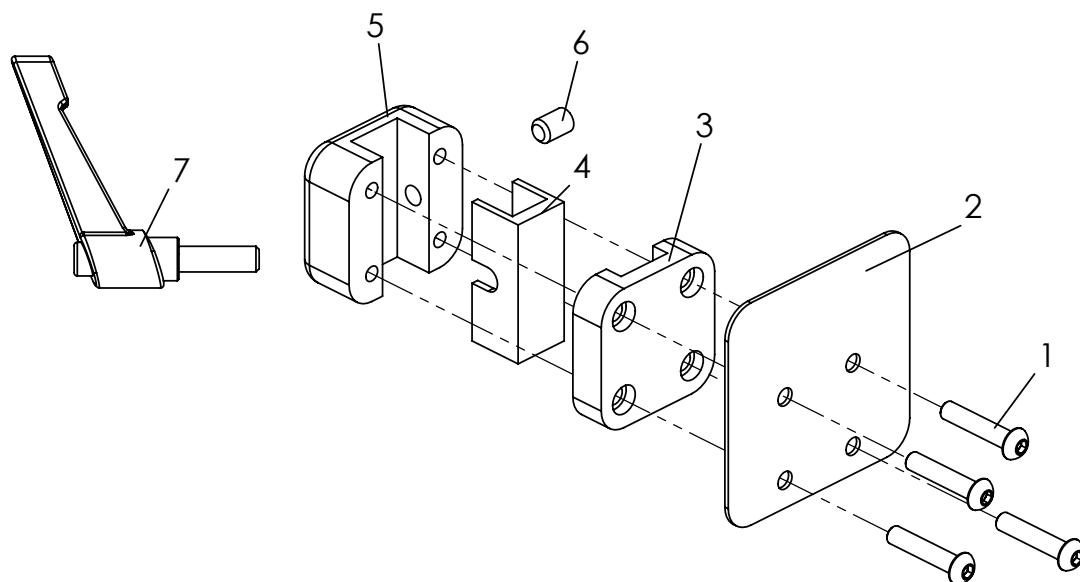


Pos	Article number	Articel description	Quantity	Unit
1-7	3201-3830	Universal headrest bracket	1	Pc.
1	1029-0630	Button head socket screw M6x30	4	Pc.
2	3201-3834-015	Reinforcing plate back, FS black	1	Pc.
3	3201-3831	Clamp bottom part	1	Pc.
4	3201-3833	Reducing piece	1	Pc.
5	3201-3832	Clamp upper part	1	Pc.
6	1031-0812	Hexagon socket set screw M8x12	1	Pc.
7	1121-0825	Locking lever PA black M8x25	1	Pc.

Spare parts

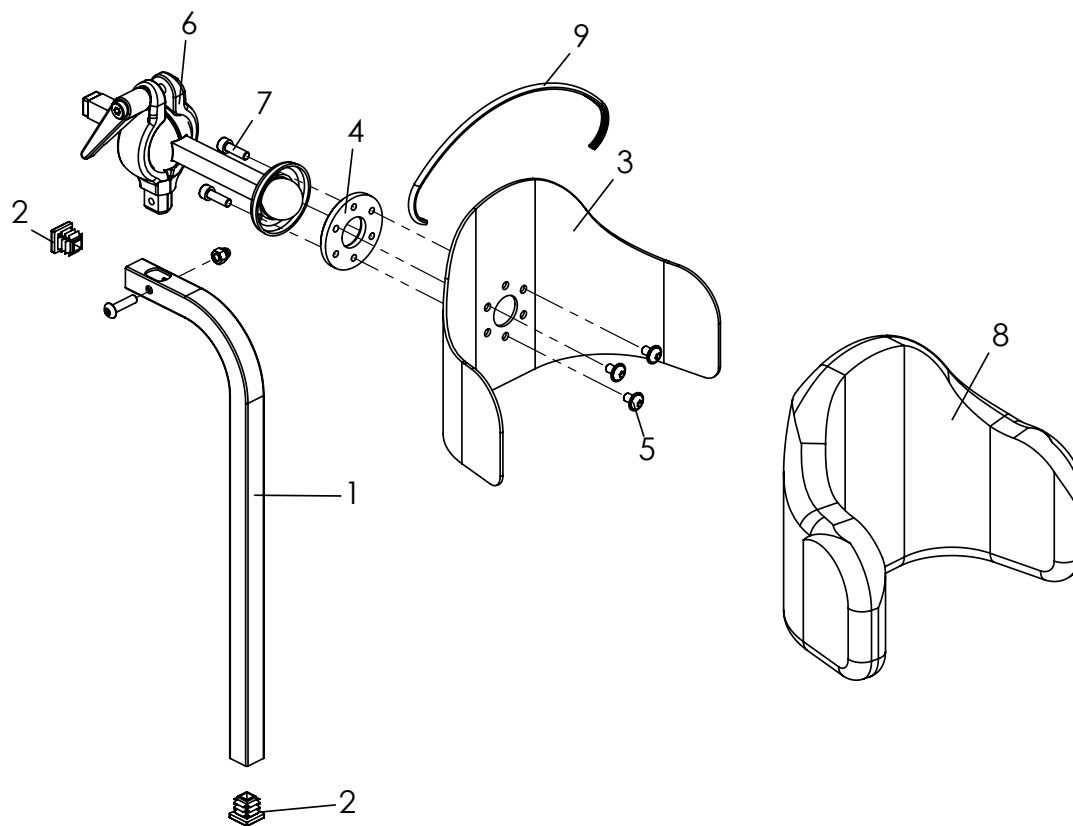


Pos	Article number	Articel description	Quantity	Unit
1-9	3245-7800-306	Head rest with mounting bracket, no cushion	1	Pc.
1	3245-7821-016	Pillar 90°	1	Pc.
2	1141-0030	Ribbed insert VL 15x15x0,8-2 black	2	Pc.
3	3245-7832	Pelotte	1	Pc.
4	3245-7831-008	Clamping ring	1	Pc.
5	1030-0508	Button head socket screw with flange M5x8	3	Pc.
6	3245-7810	Ball-head clamping complete	1	Pc.
7	1021-0516	Hexagon socket head cap screw M5x16	3	Pc.
8	3245-7850-306	Head rest cushion, black	1	Pc.
9	1144-0001	KS-Profil PVC	0,8	m

Spare parts

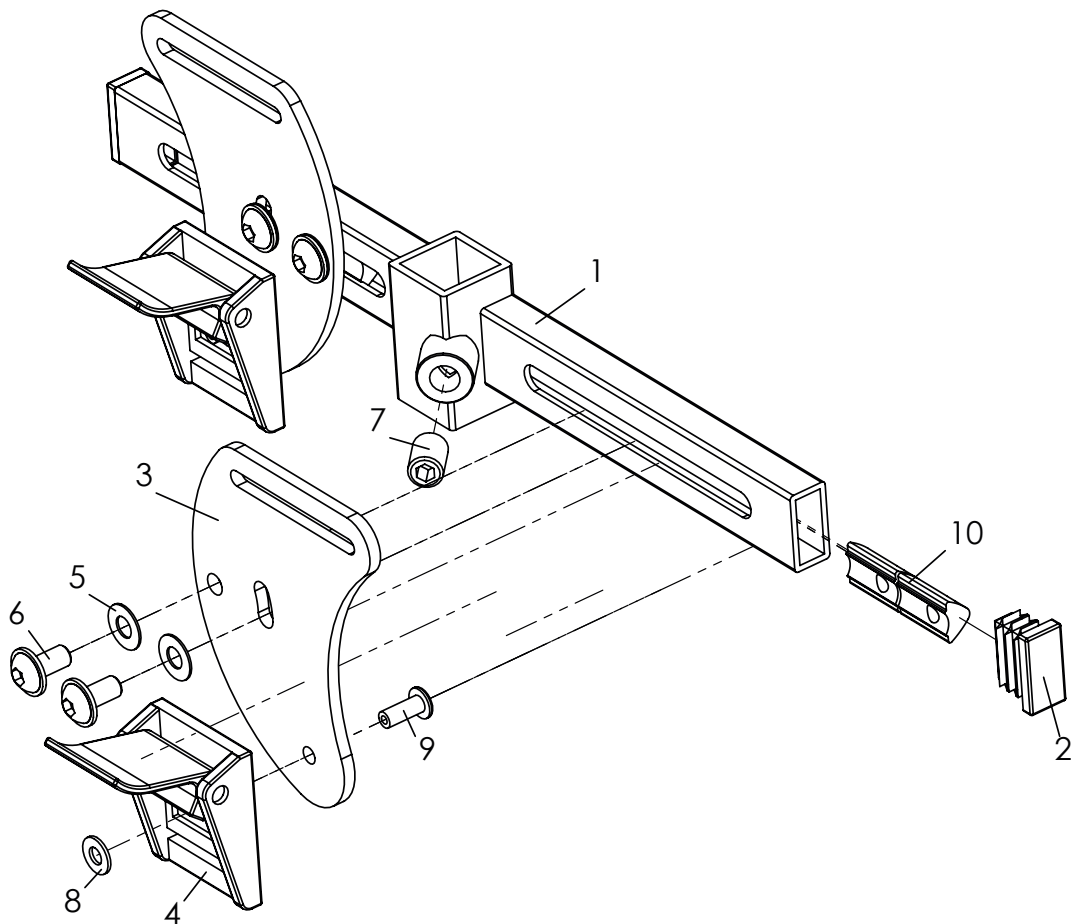


Pos	Article number	Articel description	Quantity	Unit
1-5	3200-0401	Belt routing as a T-strap above the shoulders	1	Pc.
1	3200-0410-015	T-beam	1	Pc.
2	1141-0040	Ribbed insert VL 10x20 x 0,8-2 black	2	Pc.
3	3200-0431-015	Shoulder strap guide	2	Pc.
4	1530-0001	Cam buckle HPCB 325	2	Pc.
5	1052-0005	Spring washer B5	4	Pc.
6	1030-0510	Button head socket screw M5x10	4	Pc.
7	1031-0812	Hexagon socket set screw M8x12	1	Pc.
8	1051-0004	Washer A4,3	2	Pc.
9	1062-0410	Blind rivet St/St DIN/ISO 7337 F4x10	2	Pc.
10	1049-0073	Nut insert for system profile, M5	4	Pc.

Spare parts

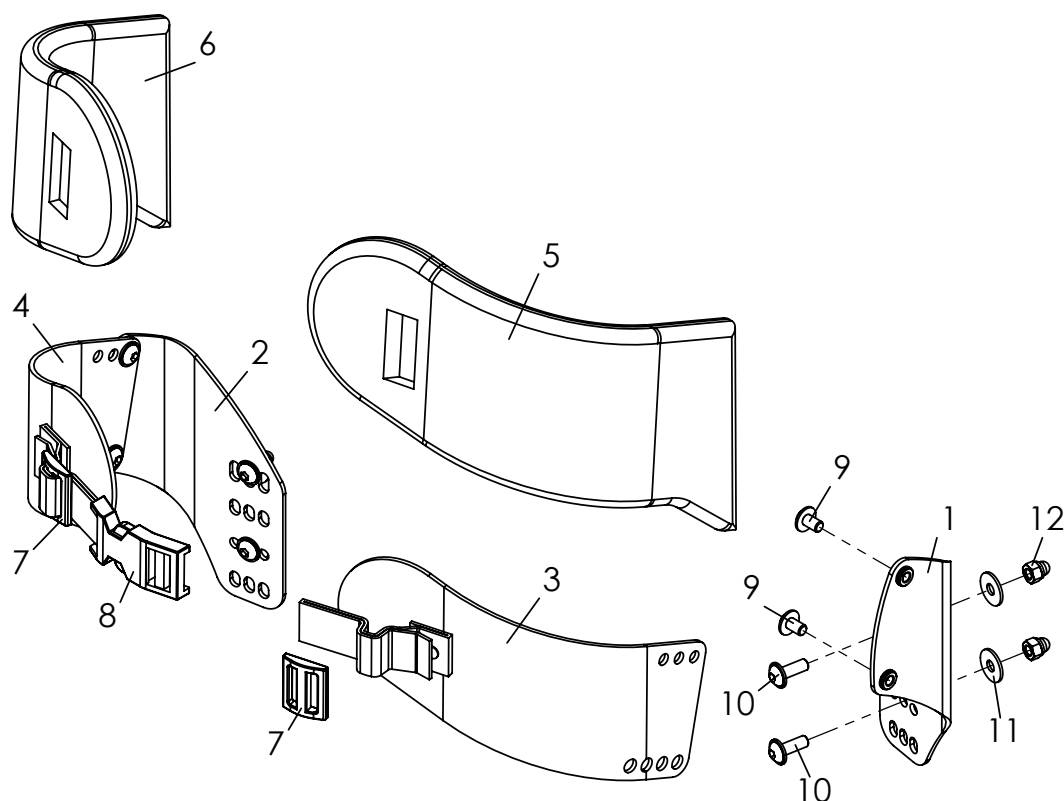


Pos	Article number	Article description	Quantity	Unit
1-12	3245-6401-346	Chest-/ thoracic support size 1	1	Pc.
	3245-6402-346	Chest- / thoracic support size 2	1	Pc.
	3245-6403-346	Chest / thoracic support size 3	1	Pc.
1	3240-6410	Support left	1	Pc.
2	3240-6420	Support right	1	Pc.
3	3240-6454	Pad with belt size 1, left	1	Pc.
	3240-6464	Pad with belt size 2, left	1	Pc.
	3240-6474	Pad with belt size 3, left	1	Pc.
4	3240-6455	Pad with belt size 1, right	1	Pc.
	3240-6465	Pad with belt size 2, right	1	Pc.
	3240-6475	Pad with belt size 3, right	1	Pc.
5	3240-6452-346	Pad upholstery size 1, left	1	Pc.
	3240-6462-346	Pad upholstery size 2, left	1	Pc.
	3240-6472-346	Pad upholstery size 3, left	1	Pc.
6	3240-6453-346	Pad upholstery size 1, right	1	Pc.
	3240-6463-346	Pad upholstery size 2, right	1	Pc.
	3240-6473-346	Pad upholstery size 3, right	1	Pc.
7	1530-0005	Buckle TG 20/N	2	Pc.
8	1531-0012	Side Release buckle for SBO chest/thoracic pads	1	Pc.
9	1030-0508	Button head socket screw with flange M5x10	4	Pc.
10	1030-0514	Button head socket screw with flange M5x14	4	Pc.
11	1055-0005	Washer A5,3	4	Pc.
12	1044-0005	Hexagonal domed cap nut self-locking M5	4	Pc.

Spare parts

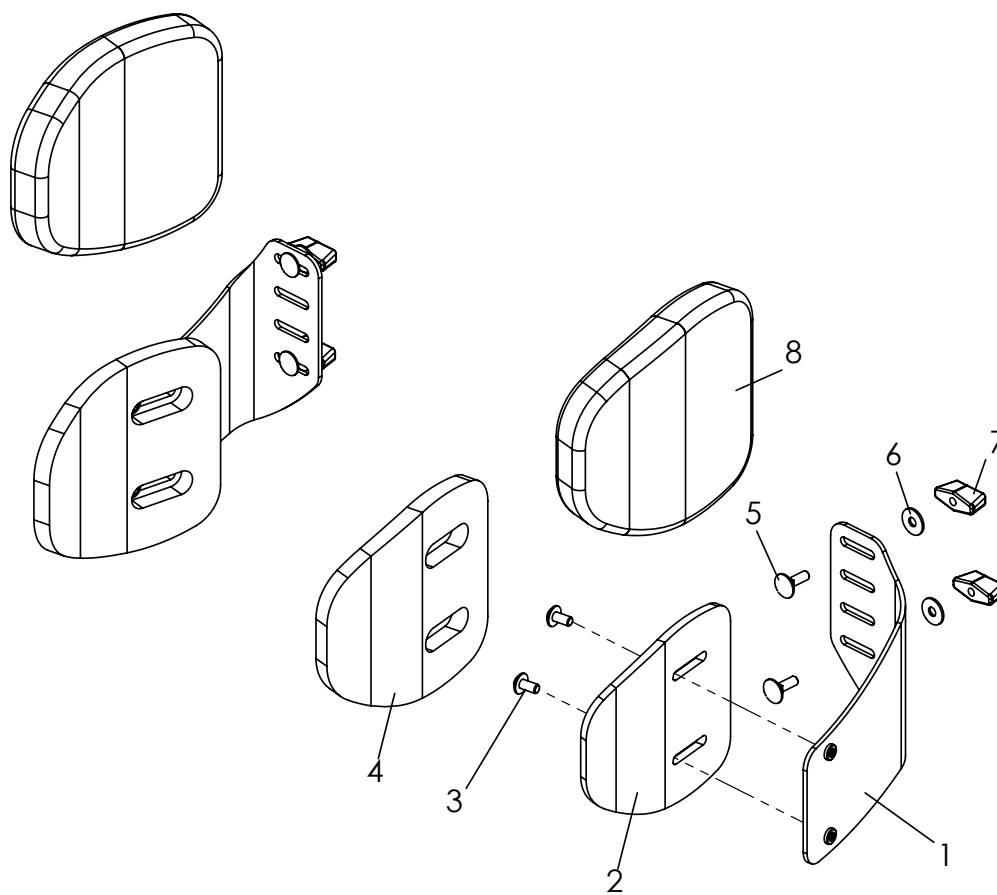


Pos	Article number	Articel description	Quantity	Unit
1-8	3246-68SW-306	Thoracic support, Back panel trough shaped	1	Pc.
1	3246-6820-015	Pad holder 20° left, 20° for SW 44	1	Pc.
	3246-6810-015	Pad holder 20° right, 20° for SW 44	1	Pc.
	3246-6840-015	Pad holder 25° left, 25° for SW 24, 26, 36, 40	1	Pc.
	3246-6830-015	Pad holder 25° right, 25° for SW 24, 26, 36, 40	1	Pc.
	3246-6860-015	Pad holder 30° left, 30° for SW 32, 34	1	Pc.
	3246-6850-015	Pad holder 30° right, 30° for SW 32, 34	1	Pc.
	3246-6880-015	Pad holder 35° left, 35° for SW 28, 30	1	Pc.
	3246-6870-015	Pad holder 35° left, 35° for SW 28, 30	1	Pc.
2	3220-2030	Pad	2	Pc.
3	1030-0510	Button head socket screw with flange M5x10	2	Pc.
4	3220-2040	Adhesive foam pad for RAC trunk support	2	Pc.
5	1026-0516	Mushroom head square neck bolt M5x16	4	Pc.
6	1055-0005	Washer A5,3	4	Pc.
7	1123-0500	Wing nut PA black M5	4	Pc.
8	3220-2020-306	Cover of trunk support, black	2	Pc.

Spare parts

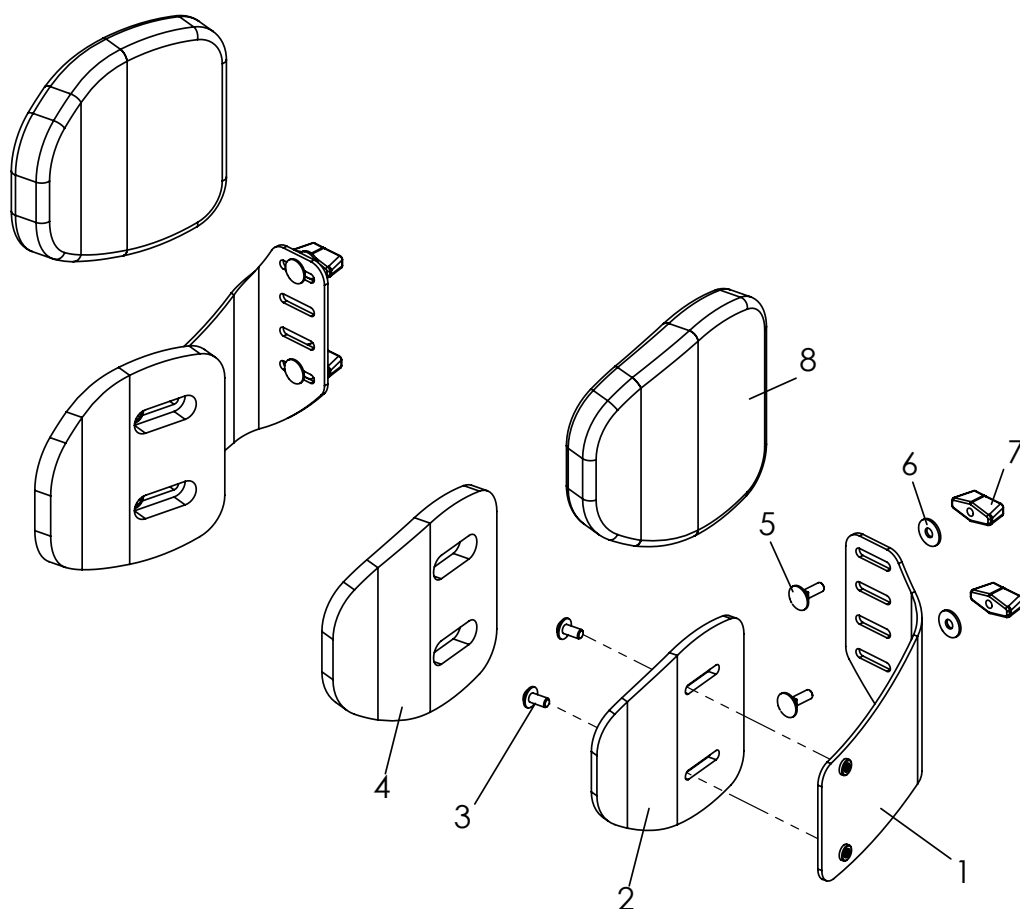


Pos	Article number	Articel description	Quantity	Unit
1-8	3245-6802-306	Thoracic support, 12x13x2, only for back, angle and height adjustable	1	Pair
1	3245-6810-015	Pelotte support right	1	Pc.
	3245-6820-015	Pelotte support left	1	Pc.
2	3220-2030	Pelotte	2	Pc.
3	1030-0510	Button head socket screw with flange M5x10	2	Pc.
4	3220-2040	Adhesive foam pad for RAC trunk support	2	Pc.
5	1026-0516	Mushroom head square neck bolt M5x16	4	Pc.
6	1055-0005	Washer A5,3	4	Pc.
7	1123-0500	Wing nut PA black M5	4	Pc.
8	3220-2020-306	Cover of trunk support, black	2	Pc.

Spare parts

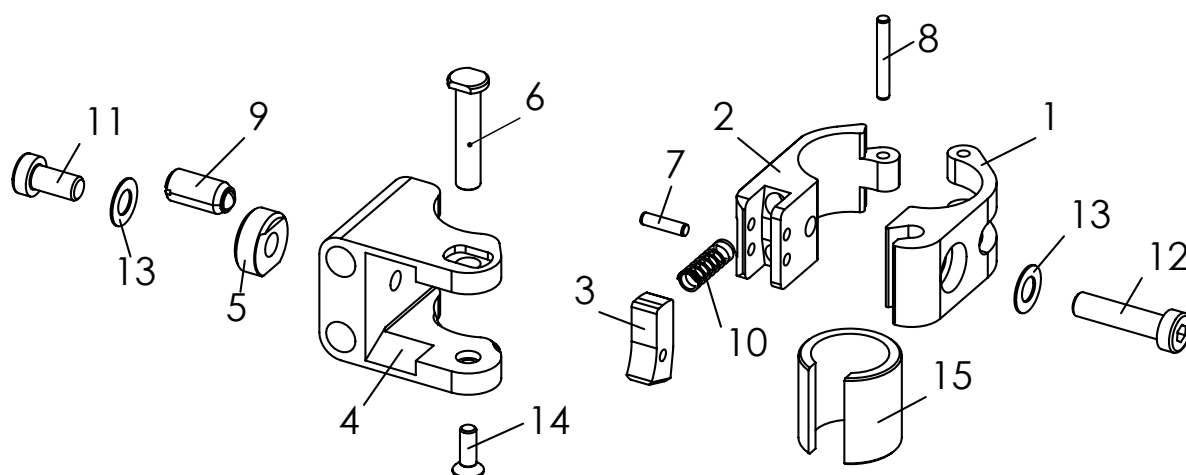


Pos	Article number	Articel description	Quantity	Unit
1-14	2240-1505	MDP pad holder swing away and detachable for attachment to round tube D25 mm, left side	1	Pc.
	2240-1506	MDP pad holder swing away and detachable for attachment to round tube D25 mm, right side	1	Pc.
1	2240-1421-007	Clamping half swing away Ø 25	1	Pc.
2	2240-1422-007	Clamping half swing away Ø 25 - M6	1	Pc.
3	2240-1423-007	Pawl	1	Pc.
4	2240-1431-007	Swing lever MDP pad holder	1	Pc.
5	2240-1433	Clamping disc on one side Ø18	1	Pc.
6	2240-1434	Locking shaft	1	Pc.
7	1071-0312	Parallel pin 3m6x12	1	Pc.
8	1071-0322	Parallel pin 3m6x22	1	Pc.
9	1139-0004	spring-loaded pressure piece with POM D=M08, L=16	1	Pc.
10	1131-0042	Compression spring D-071A 0.5x5.5x16.4x5.5	1	Pc.
11	1022-0612	Hexagon socket head cap screw zp M6x12	1	Pc.
12	1022-0625	Hexagon socket head cap screw zp M6x25	1	Pc.
13	1052-0006	Spring washer B6 zp	2	Pc.
14	1024-0410	Hexagon socket countersunk head screw zp M4x10	1	Pc.
15	2240-1441	Reducing bush Ø19	1	Pc.
	2240-1442	Reducing bush Ø22	1	Pc.

Spare parts

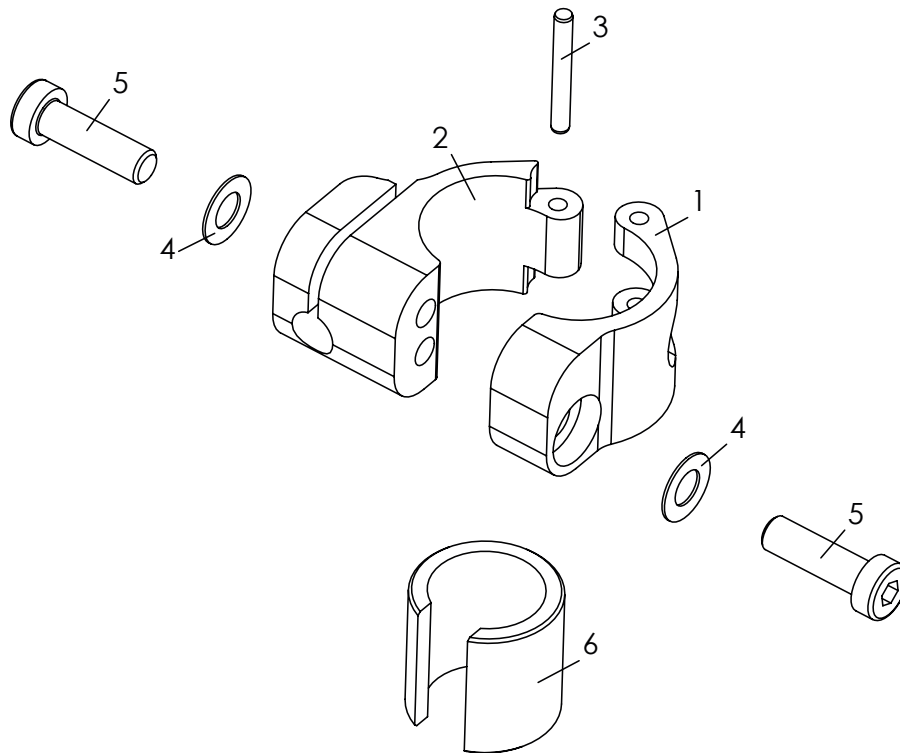


Pos	Article number	Articel description	Quanti	Unit
1-5	2240-1403	MDP pad holder fixed for attachment to round tube D25 mm, left/right	1	Pc.
1	2240-1411-007	Clamping half Ø 25	1	Pc.
2	2240-1412-007	Clamping half Ø 8-25	1	Pc.
3	1071-0324	Parallel pin DIN/ISO 6325 3m6x24	1	Pc.
4	1052-0006	Spring washer B6 zp DIN/ISO 137 B6	2	Pc.
5	1022-0620	Hexagon socket head cap screw zp DIN/ISO 7984 M6x20	2	Pc.
6	2240-1441	Reducing bush Ø19	1	Pc.
	2240-1442	Reducing bush Ø22	1	Pc.

Spare parts

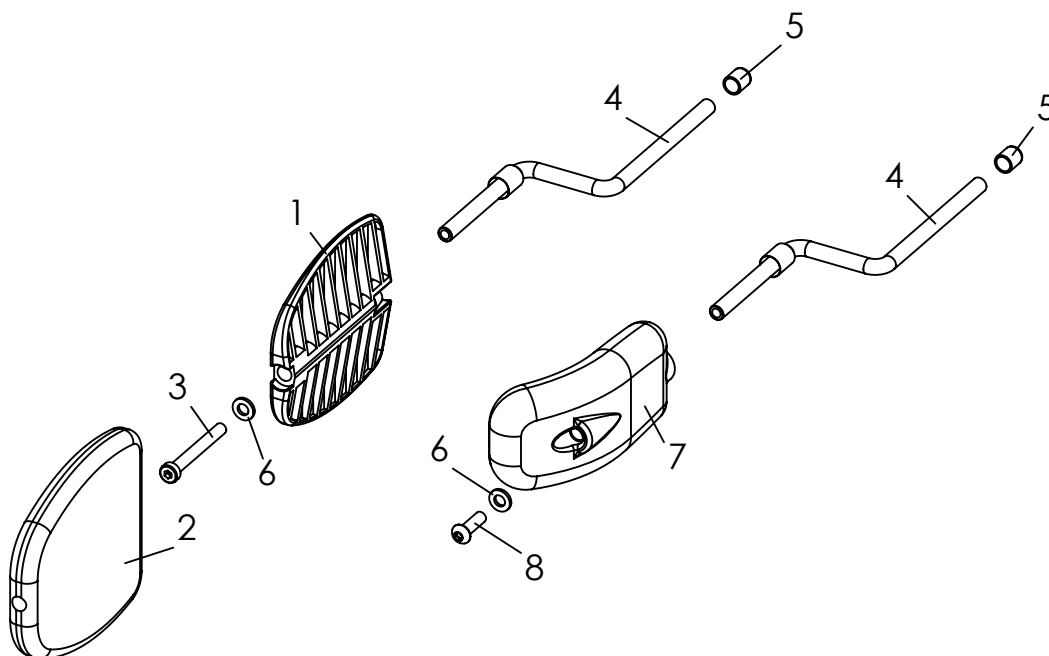


Pos	Article number	Articel description	Quantity	Unit
1-6	2240-1461-346	MDP trunk support size 1 (8 x 10 x 2 cm), cover black	1	Pc.
	2240-1462-346	MDP trunk support size 2 (10 x 12 x 2 cm), cover black	1	Pc.
	2240-1463-346	MDP trunk support size 3 (12 x 13 x 2 cm), cover black	1	Pc.
4-8	2240-1471	MDP trunk support curved size 1 (6 x 11,5 x 2,5 cm), PU w/o cover	1	Pc.
	2240-1472	MDP trunk support curved size 2 (8 x 14 x 2,5 cm), PU w/o cover	1	Pc.
1	3231-7730	Pad size 1	1	Pc.
	3233-7730	Pad size 2	1	Pc.
	3235-7730	Pad size 3	1	Pc.
2	3231-7760-346	Cover thorax pad size 1, black	1	Pc.
	3233-7760-346	Cover thorax pad size 2, black	1	Pc.
	3203-7760-346	Cover thorax pad size 3, black	1	Pc.
3	1022-0620	Hexagon socket head cap screw zp M6x20	1	Pc.
	1022-0640	Hexagon socket head cap screw zp M6x40	1	Pc.
	1022-0650	Hexagon socket head cap screw zp M6x50	1	Pc.
4	3006-7710-033	Swivel Lever Trunk support SK, black galvanized	1	Pc.
5	1142-0016	Cover K 8 PE black	1	Pc.
6	1051-0006	Washer A6,4 zp DIN/ISO 125 A6,4	1	Pc.
7	3401-7740-001	PU- trunk support with inserts, size 1, black	1	Pc.
	3401-7760-001	PU- trunk support with inserts, size 2, black	1	Pc.
8	1029-0620	Button head socket screw M6x20	1	Pc.

Spare parts

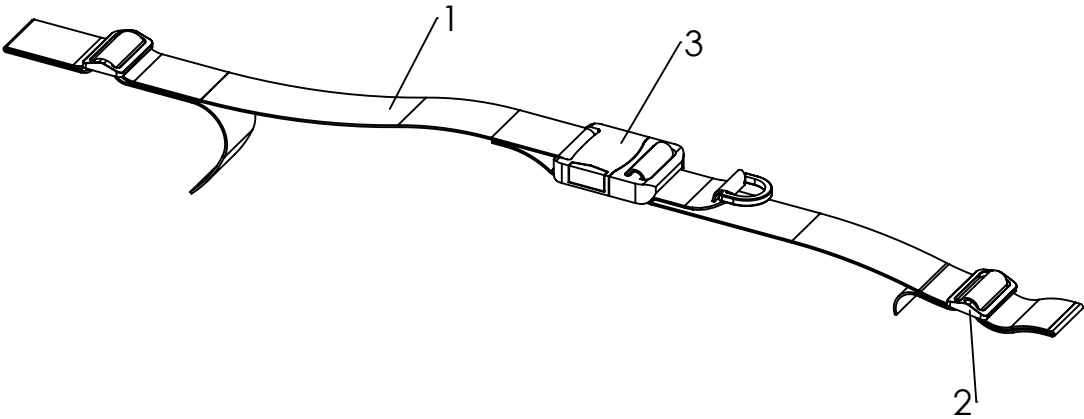


Pos	Article number	Articel description	Quantity	Unit
1-3	3201-6100	Hip belt	1	Pc.
2	1530-0003	Buckle TG 40/N	2	Pc.
3	1531-0002	Buckle SS40/HC	1	Pc.

Spare parts

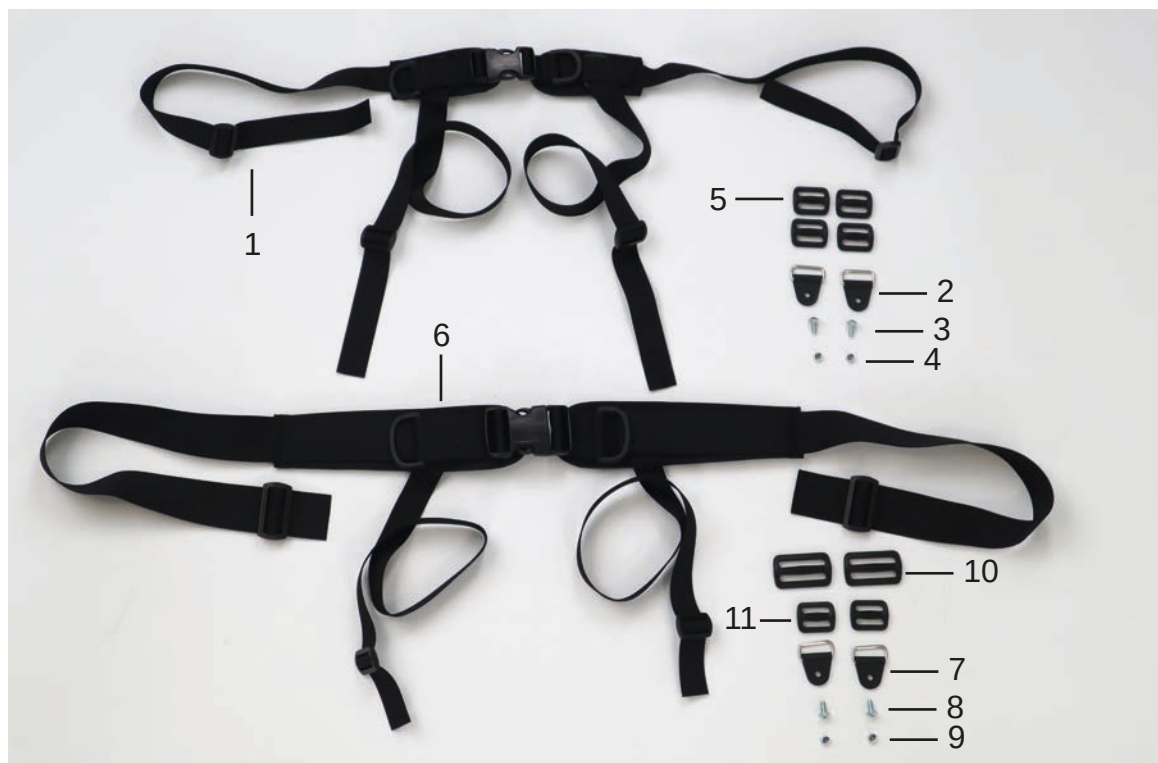


Pos	Article number	Articel description	Quantity	Unit
1-5	3240-6701	4-point lap belt size XS (for seat width 18 - 24)	1	Pc.
	3240-6702	4-point lap belt size S (for seat width 22 - 30)	1	Pc.
1	7000-0055	4-point spina strap size XS (for seat width 18 - 24)	1	Pc.
	7000-0056	4-point spina strap size S (for seat width 22 - 30)	1	Pc.
2	1532-0003	Rectangle-ring with plastic tab NY 140 NP/25 black	2	Pc.
3	1030-0510	Button head socket screw with flange M5x10	2	Pc.
4	1042-0005	Hexagon nut self-locking M5	2	Pc.
5	1530-0004	Buckle TG 25/N	4	Pc.
6-11	3240-6703	4-point lap belt size M (for seat width 28 - 36)	1	Pc.
	3240-6704	4-point lap belt size L (for seat width 34 - 42)	1	Pc.
6	7000-0057	4-point spina strap size M (for seat width 28 - 36)	1	Pc.
	7000-0067	4-point spina strap size L (for seat width 34 - 42)	1	Pc.
7	1532-0003	Rectangle-ring with plastic tab NY 140 NP/25 black	2	Pc.
8	1030-0510	Button head socket screw with flange M5x10	2	Pc.
9	1042-0005	Hexagon nut self-locking M5	2	Pc.
10	1530-0003	Buckle TG 40/N	2	Pc.
11	1530-0004	Buckle TG 25/N	2	Pc.

Spare parts

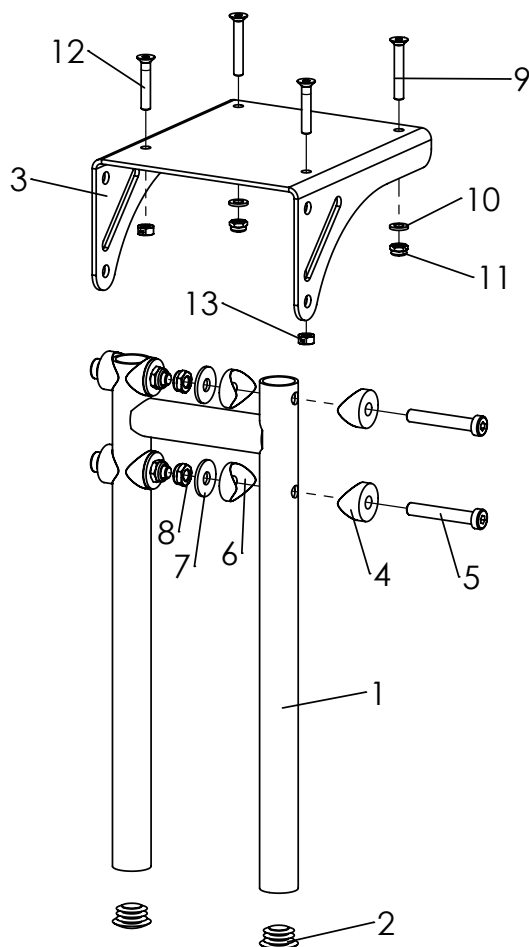


Pos	Article number	Articel description	Quantity	Unit
1-13	3249-4005E-015	Foot rest hanger 90°, lower leg length short 20-36cm	1	Set
	3249-4006E-015	Foot rest hanger 90°, lower leg length long 20-41cm	1	Set
1	3245-4250-015	Footrest adapter short	1	Pc.
	3245-4260-015	Footrest adapter long	1	Pc.
2	1141-0009	Ribbed insert SFL 25x1-2,5	2	Pc.
3	3245-4121-007	Console, black anodised	1	Pc.
4	1111-0014	Saddle washer PA black 24x8.4x6	4	Pc.
5	1022-0850	Hexagon socket head cap screw M8x50	4	Pc.
6	1111-0012	Saddle washer PA black 24x8.4x3	4	Pc.
7	1055-0008	Washer A8,4	4	Pc.
8	1042-0008	Hexagon nut self-locking M8	4	Pc.
9	1024-0635	Hexagon socket countersunk head screw M6x35	2	Pc.
10	1051-0006	Washer A6,4	2	Pc.
11	1042-0006	Hexagon nut self-locking M6	2	Pc.
12	1024-0630	Hexagon socket countersunk head screw M6x30	2	Pc.
13	1049-0068	BILOC-nut VM6	2	Pc.

Spare parts

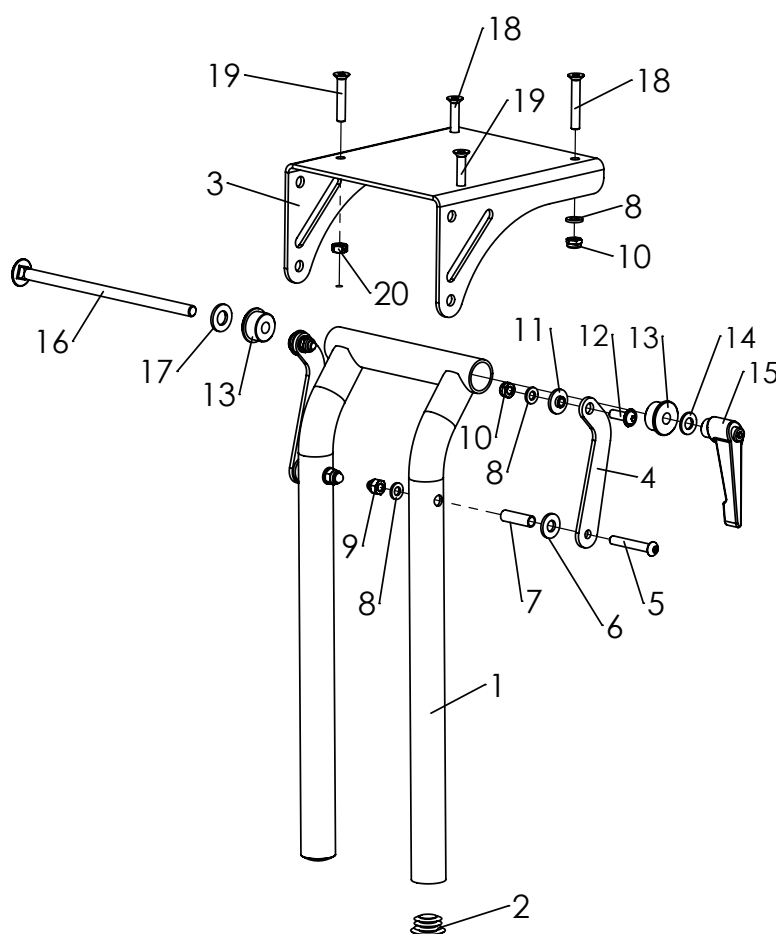


Pos	Article number	Articel description	Quantity	Unit
1-13	3245-4007E-015	Foot rest hanger angle adjustable 85-160°, lower leg short 20-36 cm	1	Set
	3245-4008E-015	Foot rest hanger angle adjustable 85-160°, lower leg length 20-41	1	Set
1	3245-4270-015	Footrest adapter short	1	Pc.
	3245-4280-015	Footrest adapter long	1	Pc.
2	1141-0009	ribbed insert SFL 25x1-2,5	2	Pc.
3	3245-4121-007	Console, black anodised	1	Pc.
4	3231-2432-032	Pivot lever strut, black dull matt	2	Pc.
5	1029-0640	Button head socket screw M6x40	2	Pc.
6	1113-0035	Washer PA 18x8,4x2 black	2	Pc.
7	1114-0025	Bushing zp 8x0,8x27	2	Pc.
8	1051-0006	Washer A6,4	6	Pc.
9	1044-0006	Hexagonal domed cap nut self-locking M6	2	Pc.
10	1042-0006	Hexagon nut self-locking M6	4	Pc.
11	3210-1238	Shoulder bushing	2	Pc.
12	1030-0618	Button head socket screw M6x18	2	Pc.
13	3245-4151	Clamping bush	2	Pc.
14	1051-0008	Washer A8,4	1	Pc.
15	1121-0800	Locking lever PA black M8M8	1	Pc.
16	1026-0817	Mushroom head square neck bolt M8x170	1	Pc.
17	1051-0010	Washer A10,5	1	Pc.
18	1024-0635	Hexagon socket countersunk head screw M6x35	2	Pc.
19	1024-0630	Hexagon socket countersunk head screw M6x30	2	Pc.

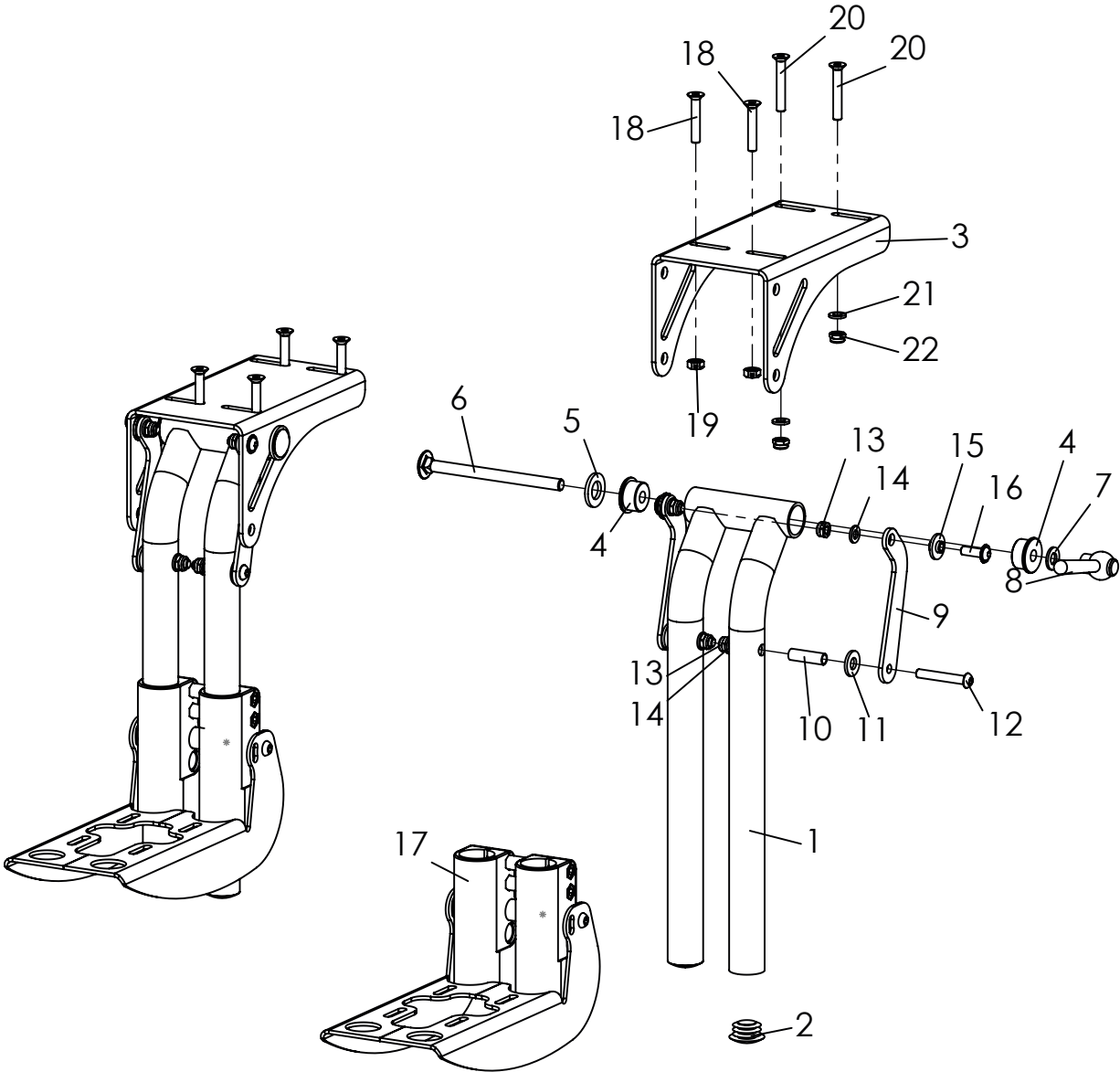

Spare parts


Pos	Article number	Articel description	Quantity	Unit
1-23	3006-4003E-015	Single foot rest hanger angle adjustable 85-160°, lower leg short 20-36 cm	1	Set
	3006-4004E-015	Single foot rest hanger angle adjustable 85-160°, lower leg long 20-41 cm	1	Set
1-16	3247-4007L	Single foot rest hanger angle adjustable, left, short	1	Set
	3247-4007R	Single foot rest hanger angle adjustable, right, short	1	Set
	3247-4008L	Single foot rest hanger angle adjustable, left, long	1	Set
	3247-4008R	Single foot rest hanger angle adjustable, right, long	1	Set
1	3247-4270-015	Footrest adapter short	1	Pc.
	3247-4280-015	Footrest adapter long	1	Pc.
2	1141-0009	Ribbed insert SFL 25x1-2,5	2	Pc.
3	3247-4121-007	Console, anodised	1	Pc.
4	3245-4151	Clamping bush	2	Pc.
5	1051-0012	Washer A13	1	Pc.
6	1026-0811	Mushroom head square neck bolt M8x110	1	Pc.
7	1051-0008	Washer A8,4	1	Pc.
8	1129-0013-033	Adjustable coniform lever M8, black galvanized	1	Pc.
9	3231-2432-032	Pivot lever strut, black dull matt	2	Pc.
10	1114-0025	Bushing zp 8x0,8x27	2	Pc.
11	1113-0035	Washer PA 18x8,4x2 black	2	Pc.
12	1029-0640	Button head socket screw M6x40	2	Pc.
13	1042-0006	Hexagon nut self-locking M6	4	Pc.
14	1051-0006	Washer A6,4	4	Pc.
15	3210-1238	Shoulder bushing	2	Pc.
16	1030-0618	Button head socket screw M6x18	2	Pc.
17	3247-4300E	Footrest holder for single footrests	2	Pc.
18	1024-0635	Hexagon socket countersunk head screw M6x35	4	Pc.
19	1049-0068	BILOC-nut VM6	4	Pc.
20	1024-0640	Hexagon socket countersunk head screw M6x40	4	Pc.
21	1051-0006	Washer A6,4	4	Pc.
22	1042-0006	Hexagon nut self-locking M6	4	Pc.

Spare parts

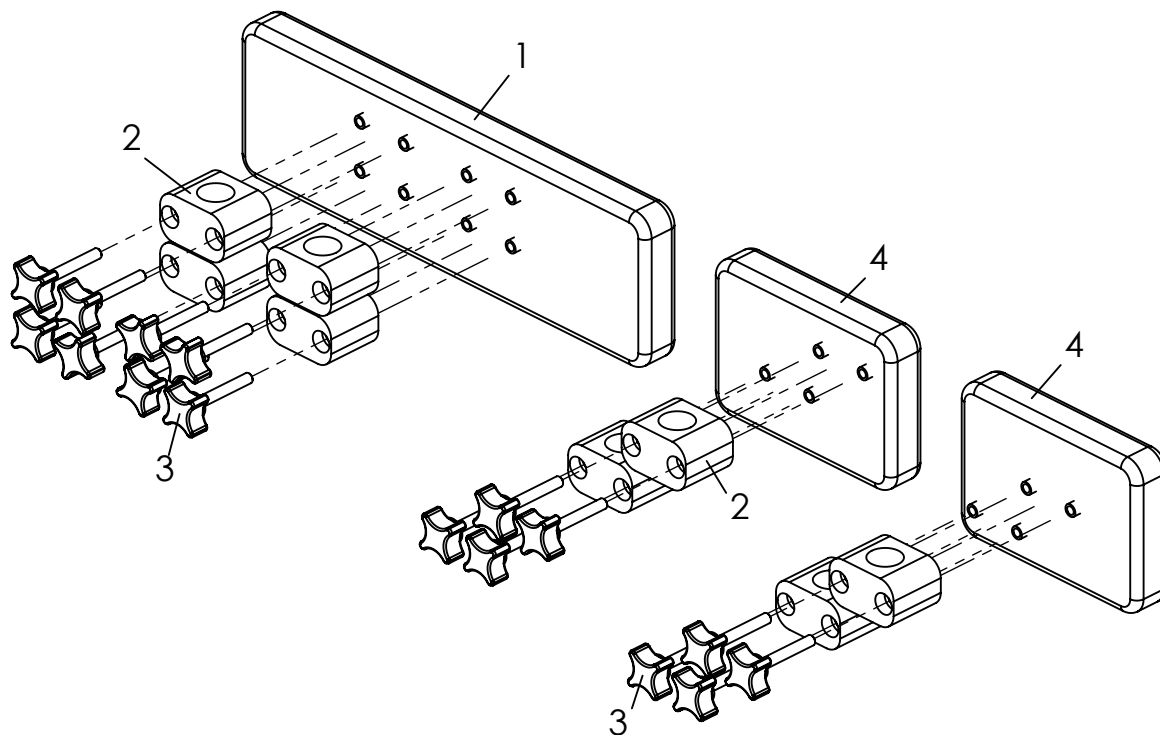


Pos	Article number	Articel description	Quantity	Unit
1-3	SA-0158-2/32-KP	Calf upholstery suitable to SW 32-34	1	Pc.
	SA-0158-2/36-KP	Calf upholstery suitable to SW 36-38	1	Pc.
	SA-0158-2/40-KP	Calf upholstery suitable to SW 40-42	1	Pc.
	SA-0158-2/44-KP	Calf upholstery suitable to SW 44-48	1	Pc.
2-4	SA-0158/4-KP	Calf upholstery for single foot rest hanger	1	Pc.
1	7000-0203	Calf upholstery for SA-0158-2 suitable to SW 32-34	1	Pc.
	7000-0204	Calf upholstery for SA-0158-2 suitable to SW 36-38	1	Pc.
	7000-0205	Calf upholstery for SA-0158-2 suitable to SW 40-42	1	Pc.
	7000-0206	Calf upholstery for SA-0158-2 suitable to SW 44-48	1	Pc.
2	1116-0002	Tube clamp PA black D25	4	Pc.
3	1125-0640	Cross handle screw KRGS 30 M6x40	8	Pc.
4	7000-0158	Calf upholstery for SA-0158-2 single foot rest hanger SW 36-44	1	Pair

Spare parts

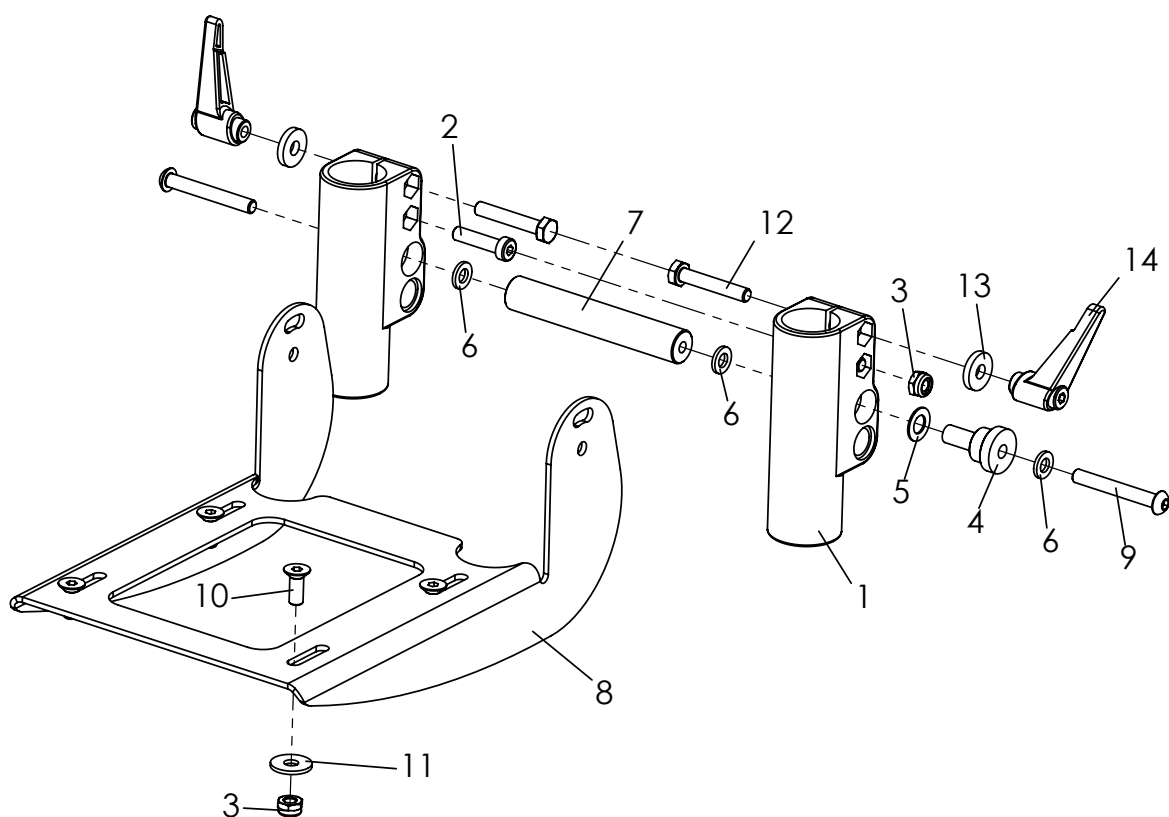


Pos	Article number	Articel description	Quantity	Unit
1-14	3245-4302E	foot rest bracket angle adjustable, flip up type	1	Set
1	3245-4321	Conductor	2	Pc.
2	1022-0625	Hexagon socket head cap screw M6x25	2	Pc.
3	1042-0006	Hexagon nut self-locking M6	6	Pc.
4	3245-4322	Bushing	2	Pc.
5	1052-0008	Spring washer B8	2	Pc.
6	1051-0006	Washer A6,4	4	Pc.
7	3245-4323	Stabiliser	1	Pc.
8	3245-4311-015	Footrest bearing, structural black	1	Pc.
9	1029-0645	Button head socket screw M6x45	2	Pc.
10	1024-0616-033	Hexagon socket countersunk head screw black M6x16	4	Pc.
11	1055-0006	Washer zp A6,4	4	Pc.
12	1011-0635	Hexagon head screw M6x35	2	Pc.
13	1059-0003	Washer M6 (17/6,4x3)	2	Pc.
14	1121-0006	Locking lever M6	2	Pc.

Spare parts

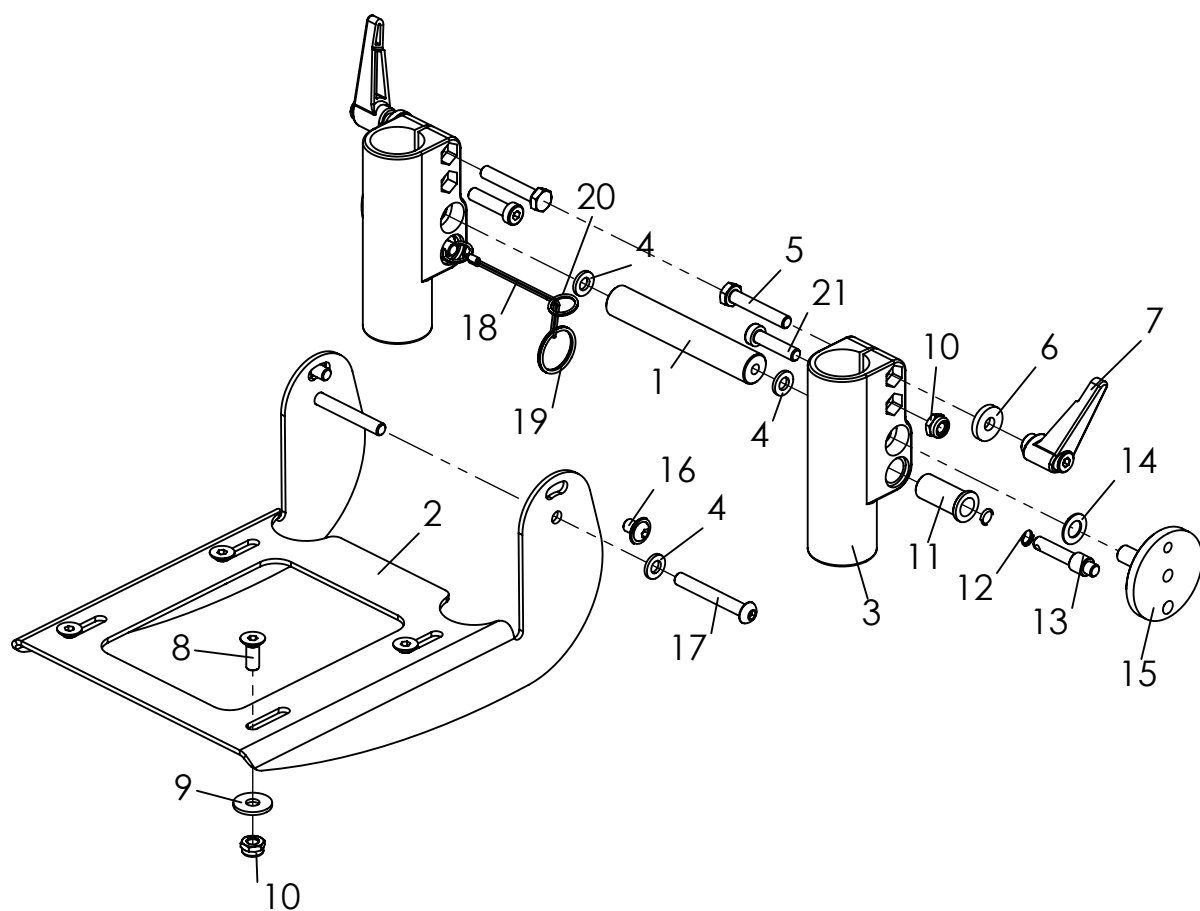




Pos	Article number	Articel description	Quantity	Unit
1-21	3245-4303E	Foot rest bracket angle adjustable, with interlocking	1	Set
1	3245-4323	Stabiliser	1	Pc.
2	3245-4311-015	Foot rest bracket	1	Pc.
3	3245-4321	Conductor	2	Pc.
4	1051-0006	Washer A6,4	4	Pc.
5	1011-0635	Hexagon head screw M6x35	2	Pc.
6	1059-0003	Washer M6 (17/6,4x3)	2	Pc.
7	1121-0006	Locking lever M6	2	Pc.
8	1024-0616-033	Hexagon socket countersunk head screw black M6x16	4	Pc.
9	1055-0006	Washer A6,4	4	Pc.
10	1042-0006	Hexagon nut self-locking M6	6	Pc.
11	3201-7520	Spring housing	2	Pc.
12	1131-0036	Compression spring D-127d 0,8x8,3x29	2	Pc.
13	3201-7530	Lock pin	2	Pc.
14	1052-0008	Spring washer B8	2	Pc.
15	3245-4341	Bearing	2	Pc.
16	1030-0608	Button head socket screw M6x8	2	Pc.
17	1029-0645	Button head socket screw M6x45	2	Pc.
18	3245-4342	Wire rope for foot rest lock	1	Pc.
19	1129-0003	Key ring 24x1,5	1	Pc.
20	1129-0012	Key ring 16x1,3	2	Pc.
21	1022-0625	Hexagon socket head cap screw M6x25	2	Pc.

Spare parts

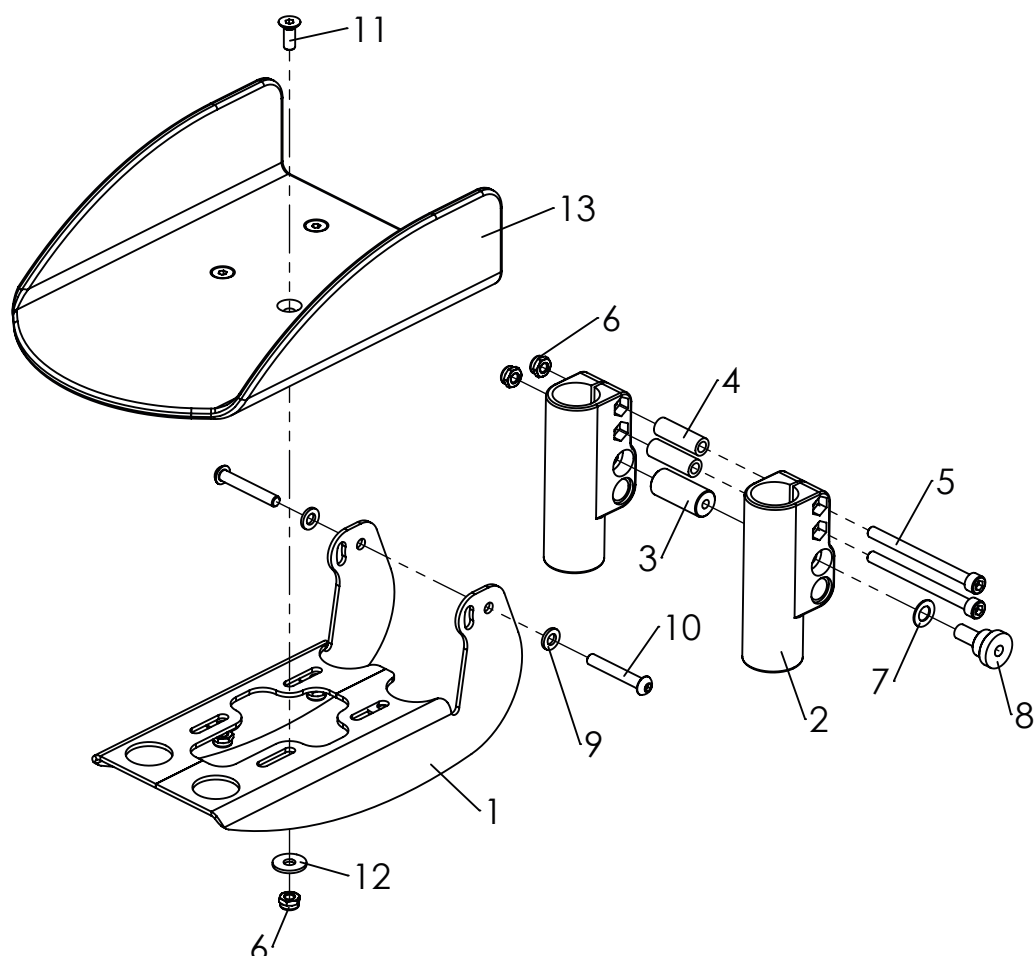


Pos	Article number	Articel description	Quantity	Unit
1-12	3247-4300E	Footrest holder for single footrests	1	Pc.
1	3247-4311-015	Footrest bearing, silver	2	Pc.
2	3245-4321	Conductor	1	Pc.
3	3247-4323	Stabilizer black anodized	2	Pc.
4	1114-0060	Bushing Alu 10x6x31	2	Pc.
5	1021-0675	Hexagon socket head cap screw M6x75	2	Pc.
6	1042-0006	Hexagon nut self-locking M6	6	Pc.
7	1052-0008	Spring washer B8	2	Pc.
8	3245-4322	Bushing	2	Pc.
9	1051-0006	Washer A6,4	2	Pc.
10	1029-0640	Button head socket screw M6x40	2	Pc.
11	1024-0616-033	Hexagon socket countersunk head screw M6x16	4	Pc.
12	1055-0006	Washer A6,4	4	Pc.
13	3245-4632	Two single footplates in trough shape, SW28 - SW32	1	Set
	3245-4640	Two single footplates in trough shape, SW36 - SW44	1	Set

Spare parts

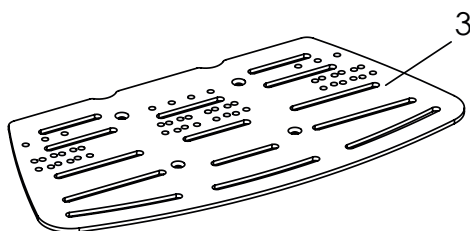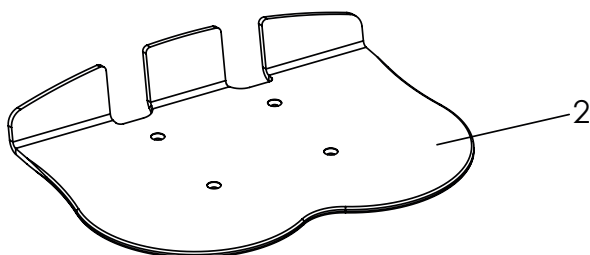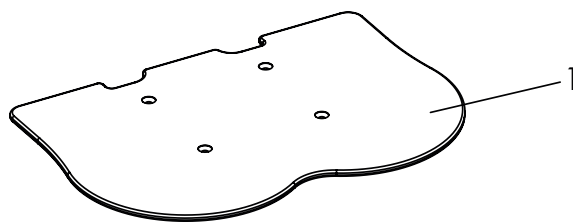


Pos	Article number	Articel description	Quanti	Unit
1	3245-44SW	Foot plate flat plastic	1	Pc.
2	3245-45SW	Foot rest with heel stop	1	Pc.
3	3249-46SW-000	Alu footrest, in frame colour	1	Pc.
	3249-46SW-015	Alu footrest, fine structure black	1	Pc.

Spare parts

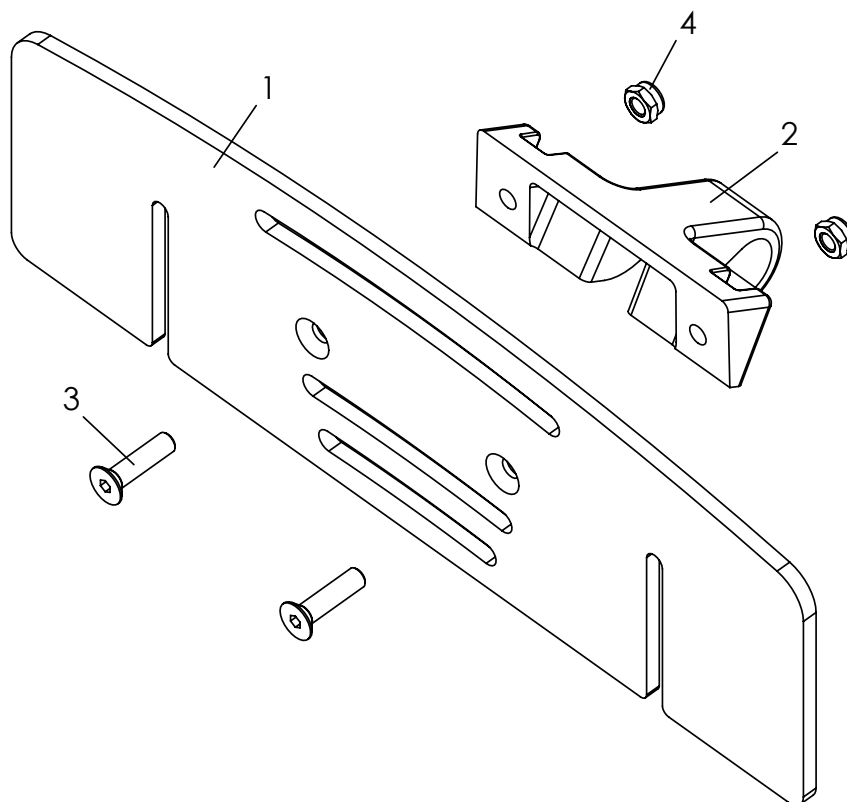


Pos	Article number	Articel description	Quantity	Unit
1-4	3249-45SW-015	Heel plate Aluminum, black, complete	1	Set
1	3290-46SW-015	Heel plate Aluminum, black	1	Pc.
2	3290-4611	Heel plate clamp	1	Pc.
3	1024-0520	Hexagon socket countersunk head screw M5x20	2	Pc.
4	1042-0005	Hexagon nut self-locking M5	2	Pc.

Spare parts

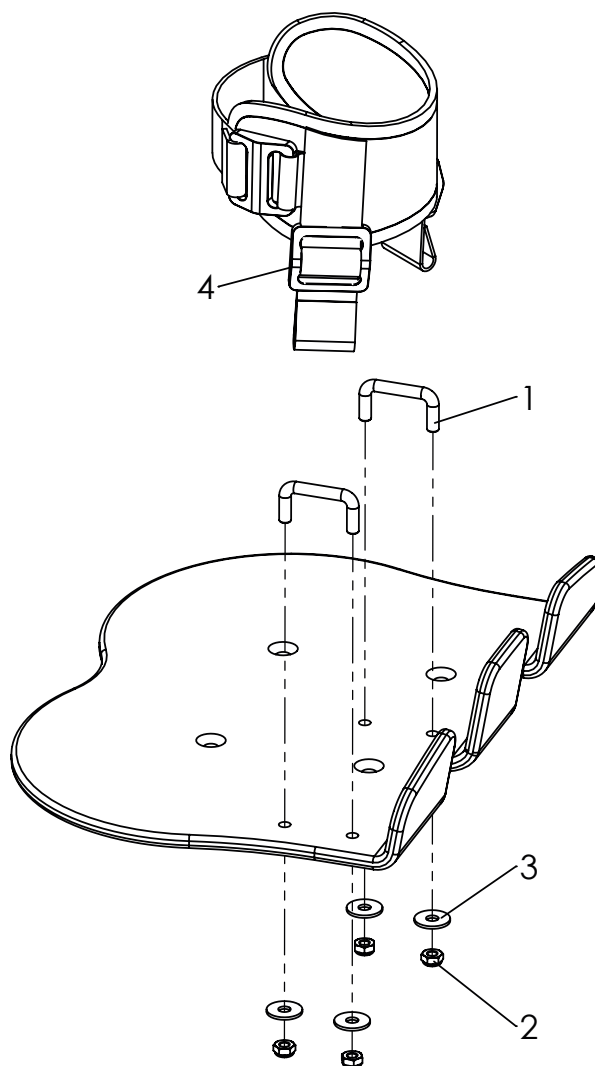


Pos	Article number	Articel description	Quantity	Unit
1-4	3201-6400	Ankle huggers size 1 (diameter approx. 4 - 6.5 cm)	1	Set
	3203-6400	Ankle huggers size 2 (diameter approx. 6 - 8.5 cm)	1	Set
1	3201-6410	Foot strap anchor Gr.1	4	Pc.
	3203-6410	Foot strap anchor size 2	4	Pc.
2	1042-0005	Hexagon nut self-locking zp DIN/ISO 985 M5	8	Pc.
3	1055-0005	Washer A5,3	8	Pc.
4	1530-0004	Buckle TG 25/N	4	Pc.

Spare parts

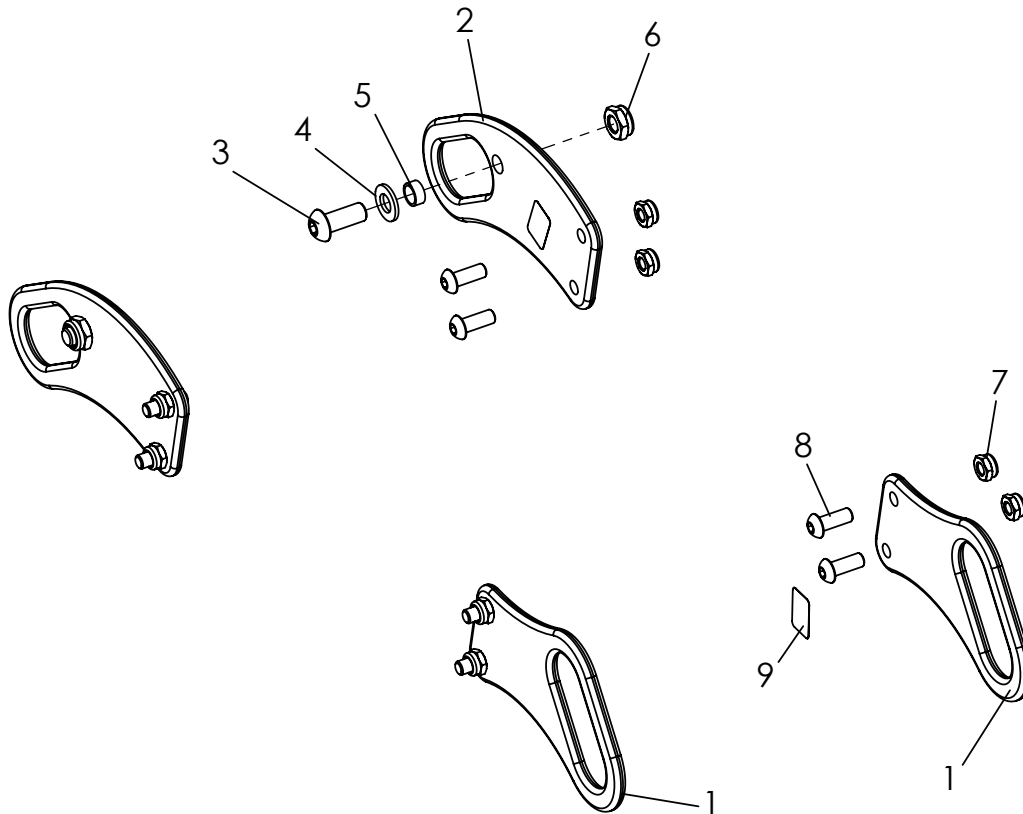


Pos	Article number	Articel description	Quantity	Unit
1-9	3249-7300	Tie down kit	1	Pc.
1	3249-7311-015	Lashing front left	2	Pc.
3	3248-7321-015	Lashing back side	2	Pc.
4	1029-0820	Button head socket screw M8x20	2	Pc.
5	1051-0008	Washer A8,4	2	Pc.
6	3201-7350	Slide bushing	2	Pc.
7	1042-0008	Hexagon nut self-locking M8	2	Pc.
8	1042-0006	Hexagon nut self-locking M6	8	Pc.
9	1029-0616	Button head socket screw M6x16	8	Pc.
10	1914-0007	Adhesive lable with hook symbol (4 pcs.)	1	Set

Spare parts

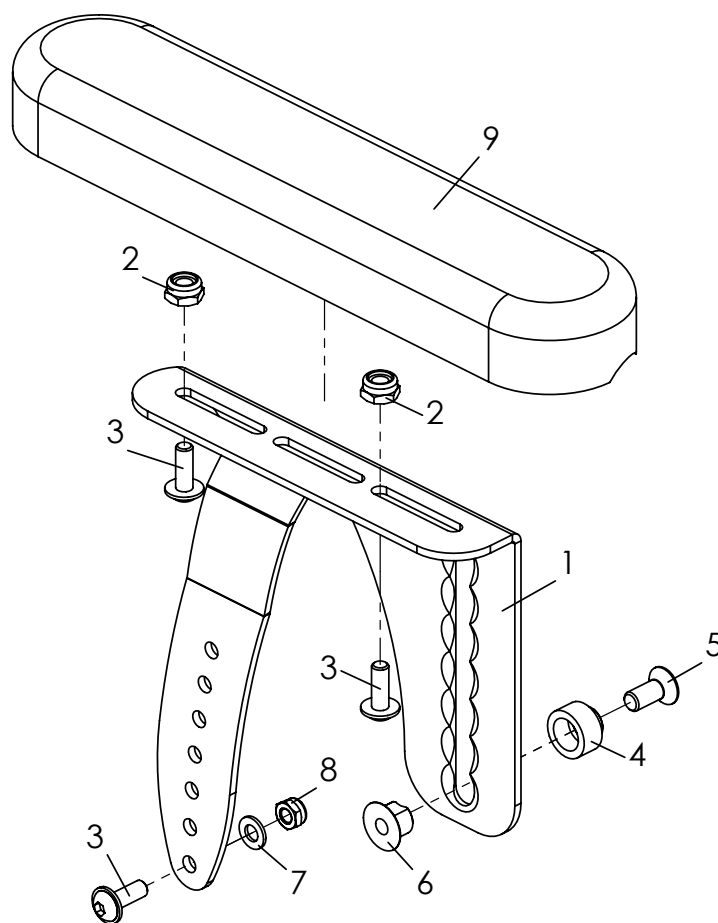

Pos	Article number	Articel description	Quantity	Unit
1-8	3246-6201E	Armrest left size 1, hight from 14-21cm	1	Pc.
	3246-6202E	Armrest right size 1, hight from 14-21cm	1	Pc.
	3246-6301E	Armrest left size 2, hight from 19-26cm	1	Pc.
	3246-6302E	Armrest right size 2, hight from 19-26cm	1	Pc.
1	3246-6211-015	Armrest carrier left size 1	1	Pc.
	3246-6212-015	Armrest carrier right size 1	1	Pc.
	3246-6311-015	Armrest carrier left size 2	1	Pc.
	3246-6312-015	Armrest carrier right size 2	1	Pc.
2	1042-0006	Hexagon nut self-locking zp M6	2	Pc.
3	1030-0514	Button head socket screw with flange M5x14	3	Pc.
4	3246-1149	Bearing axle for tipping armrest	1	Pc.
5	1024-0616	Hexagon socket countersunk head screw M6x16	1	Pc.
6	3246-6213	Clamping nut	1	Pc.
7	1051-0005	Washer A5,3	1	Pc.
8	1042-0005	Hexagon nut self-locking M5	1	Pc.
9	7000-0024	Armrest pads PU 235mm	1	Pc.
	7000-0047	Armrest pads PU 270mm	1	Pc.


Spare parts

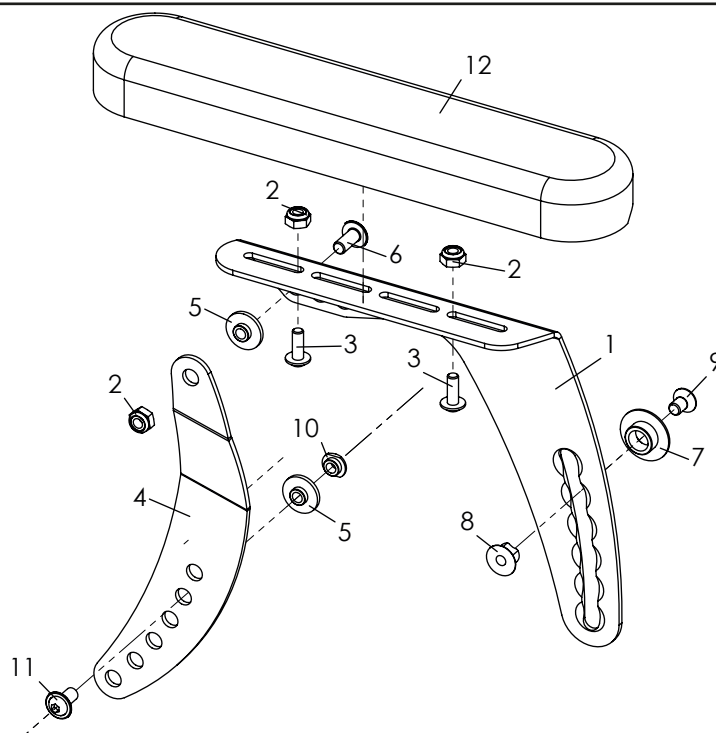

Pos	Article number	Articel description	Qua	Unit
1-12	3246-6401E	Armrest with tilt balancing, left size 1, higt from 15-21cm	1	Pc.
	3246-6402E	Armrest with tilt balancing, right size 1, higt from 15-21cm	1	Pc.
	3246-6501E	Armrest with tilt balancing, left size 2, higt from 18-26cm	1	Pc.
	3246-6502E	Armrest with tilt balancing, right size 2, higt from 18-26cm	1	Pc.
	3246-6501E-SA-0408	Armrest with tilt balancing, left size 2, reinforced	1	Pc.
	3246-6502E-SA-0408	Armrest with tilt balancing, right size 2, reinforced	1	Pc.
1	3246-6411-015	Armrest retainer left size 1, FS black	1	Pc.
	3246-6412-015	Armrest retainer right size 1, FS black	1	Pc.
	3246-6511-015	Armrest retainer left size 2, FS black	1	Pc.
	3246-6512-015	Armrest retainer right size 2, FS black	1	Pc.
	3246-6511_SA-0408-015	Armrest retainer left size 2, reinforced, FS black	1	Pc.
	3246-6512_SA-0408-015	Armrest retainer right size 2, reinforced, FS black	1	Pc.
2	1042-0006	Hexagon nut self-locking M6	3	Pc.
3	1030-0514	Button head socket screw with flange M5x14	2	Pc.
4	3246-6413-015	Pivot lever, arm rest, left, FS black	1	Pc.
	3246-6414-015	Pivot lever, arm rest, right, FS black	1	Pc.
	3246-6413_SA-0408-015	Pivot lever, arm rest, left reinforced, FS black	1	Pc.
	3246-6414_SA-0408-015	Pivot lever, arm rest, right reinforced, FS black	1	Pc.
5	3210-1238	Shoulder bushing	2	Pc.
6	1030-0614	Button head socket screw with flange M6x16	1	Pc.
7	3246-6215	Bearing for armrest	1	Pc.
8	3246-6213	Clamping nut	1	Pc.
9	1024-0610	Hexagon socket countersunk head screw M6x10	1	Pc.
10	3246-6216	Collar nut M6	1	Pc.
11	1030-0612	Button head socket screw with flange M6x12	1	Pc.
12*	3246-6421	Cover of tilt left	1	Pc.
	3246-6422	Cover of tilt right	1	Pc.
13	7000-0024	Armrest pads PU 235mm	1	Pc.
	7000-0047	Armrest pads PU 270mm	1	Pc.


* Pos. 12 only when using Armrest with tilt balancing size 1

Spare parts

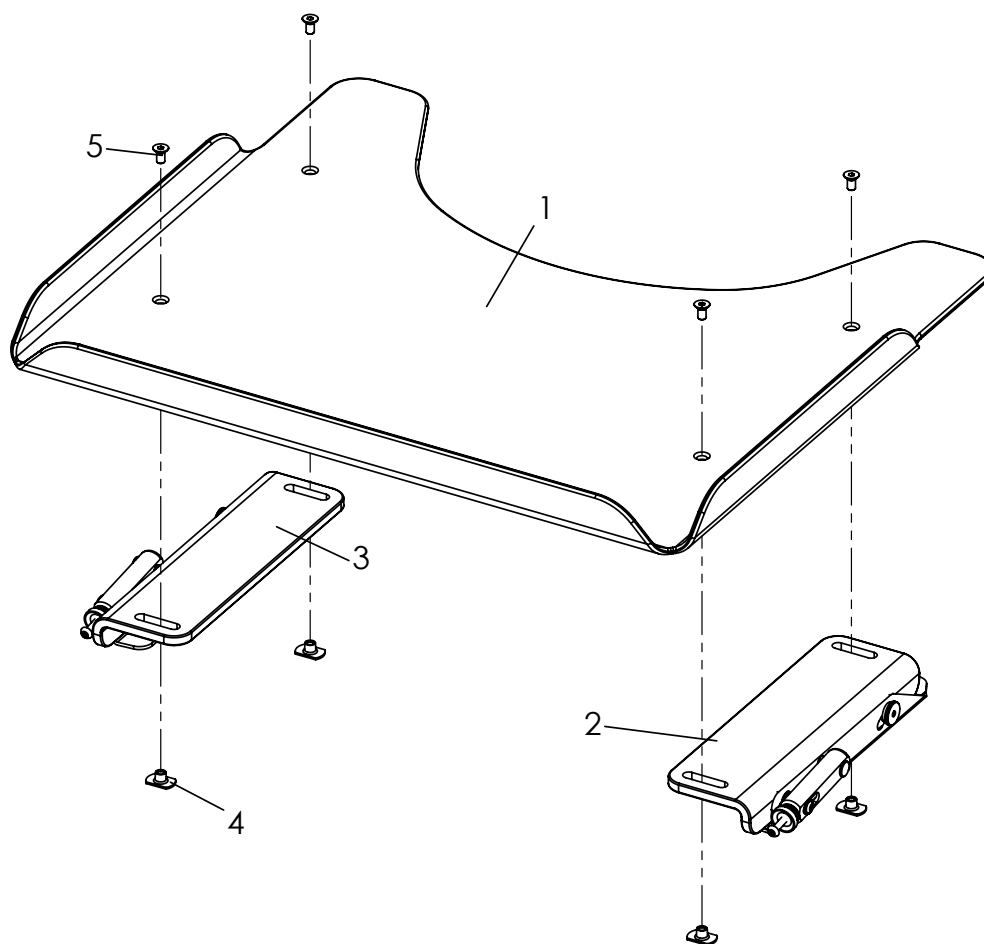


Pos	Article number	Articel description	Quantity	Unit
1-5	3240-8204	Therapy tray, size 4, SW 32-36	1	Pc.
	3240-8205	Therapy tray, size 5, SW 40-44	1	Pc.
1	3240-8294	Table board size 4	1	Pc.
	3240-8295	Table board size 5	1	Pc.
2	3240-8220E	Clamping unit, right	1	Pc.
3	3240-8210E	Clamping unit, left	1	Pc.
4	1049-0052	T-nut L-M6x8,5	4	Pc.
5	1024-0612	Hexagon socket countersunk head screw M6x12	4	Pc.

Spare parts

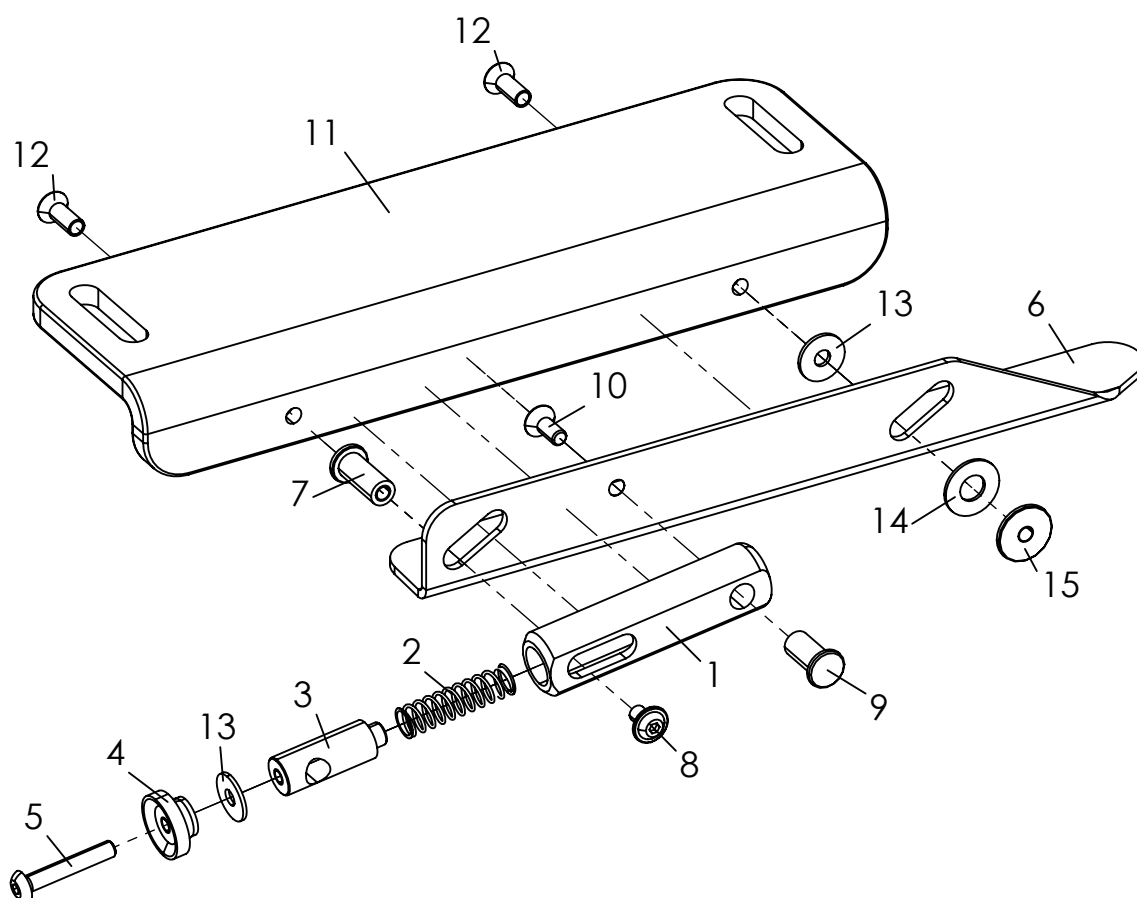


Pos	Article number	Articel description	Quantity	Unit
1-14	3240-8210E	Clamping unit, left	1	Pc.
	3240-8220E	Clamping unit, right	1	Pc.
	3240-8230E	Clamping unit long, left for SW 48	1	Pc.
	3240-8240E	Clamping unit long, right for SW 48	1	Pc.
1	3240-8214	Spring housing	1	Pc.
2	1131-0043	Compression spring D-143F	1	Pc.
3	3240-8215	Spring guide	1	Pc.
4	1124-0500	Knurled nut PA black M5	1	Pc.
5	1029-0530	Button head socket screw M5x30	1	Pc.
6	3240-8212-015	Terminal strip, left	1	Pc.
	3240-8222-015	Terminal strip, right	1	Pc.
7	3240-8217	Bearing bush	1	Pc.
8	1030-0508	Button head socket screw with flange M5x8	1	Pc.
9	3240-8216	Bearing bush	1	Pc.
10	1024-0512	Hexagon socket countersunk head screw M5x12	1	Pc.
11	3240-8211	Adjusting bracket	1	Pc.
	3240-8213	Adjusting bracket long for SW 48	1	Pc.
12	1024-0514	Hexagon socket countersunk head screw M5x14	2	Pc.
13	1055-0005	Washer A5,3	2	Pc.
14	1113-0003	Washer PA 18x8,4x1 black	1	Pc.
15	3240-8218	Bearing bush	1	Pc.

Spare parts

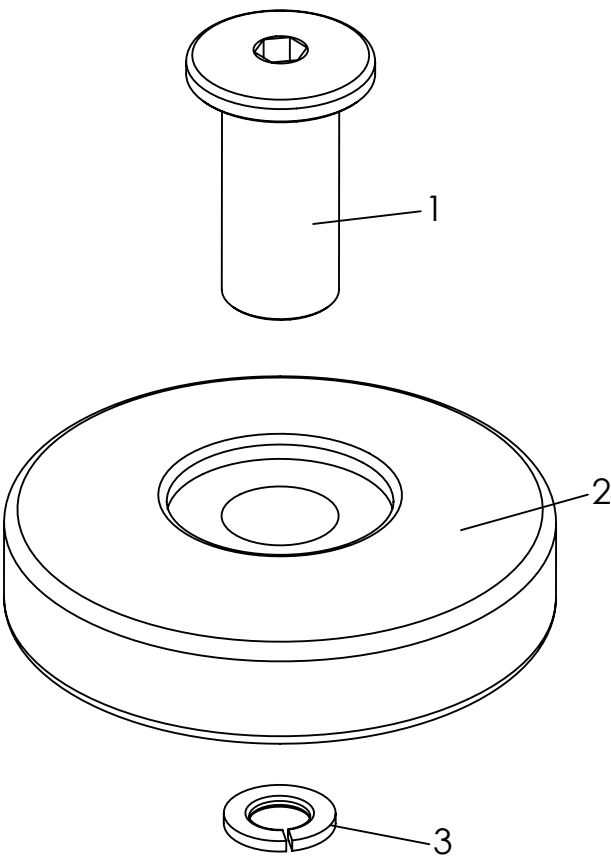


Pos	Article number	Articel description	Quantity	Unit
1-3	3275-1910	Bumper wheels	1	Set
1	3275-1911-007	Axle deflector role	2	Pc.
2	1249-0001	Deflector role	2	Pc.
3	1059-0010	Spring washer M8	2	Pc.

Spare parts

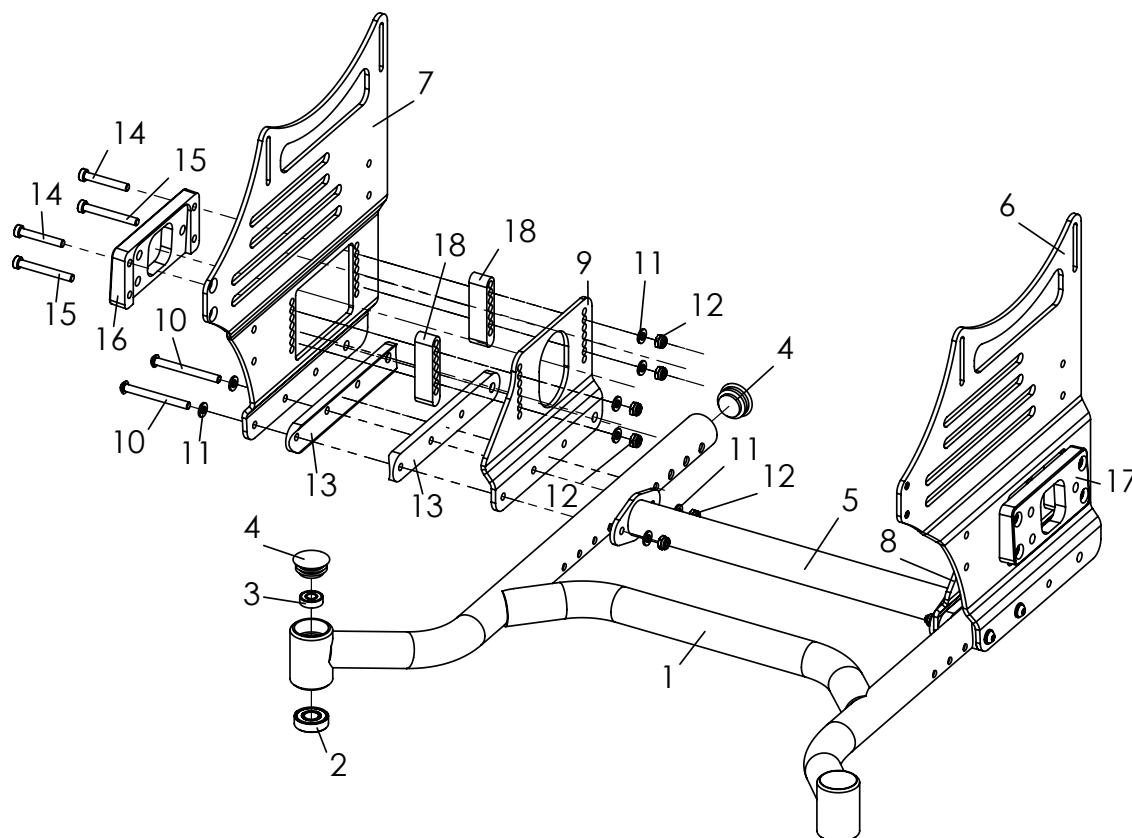

Pos	Article number	Article description	Quantity	Unit
1-18	SA-0315/3.1	Conversion kit to one-hand drive	1	Pc.
1	3249-11SW-BG_SA-0315	Base frame	1	Pc.
2	1090-0005	Grooved ball bearing D28xd12x8	2	Pc.
3	1090-0001	Grooved ball bearing D19xd8x6	2	Pc.
4	1141-0016	Ribbed insert SFL 30 30x0,8-2,5	4	Pc.
5	3248-SA-0315-11SW-000	Stabilizing tube in frame colour	1	Pc.
6	3248-1251-SA-0315	Base plate left	1	Pc.
7	3248-1252-SA-0315	Base plate right	1	Pc.
8	3248-1255-SA-0315	Adapterplate left	1	Pc.
9	3248-1256-SA-0315	Adapterplate right	1	Pc.
10	1029-0660	Button head socket screw zp M6x60	4	Pc.
11	1051-0006	Washer A6,4	16	Pc.
12	1042-0006	Hexagon nut self-locking zp M6	12	Pc.
13	3249-1156-SA-0315	Saddle rail 150	4	Pc.
14	1022-0640	Hexagon socket head cap screw zp M6x40 (for axis adapter 3°)	4	Pc.
	1022-0645	Hexagon socket head cap screw zp M6x45 (for axis adapter 6°)	4	Pc.
	1022-0650	Hexagon socket head cap screw zp M6x50 (for axis adapter 9°)	4	Pc.
15	1022-0640	Hexagon socket head cap screw zp M6x40 (for axis adapter 3°)	4	Pc.
	1022-0650	Hexagon socket head cap screw zp M6x50 (for axis adapter 6°)	4	Pc.
	1029-0655	Button head socket screw M6x55 (for axis adapter 9°)	4	Pc.
16	3248-SA-0315-1152	Axis adapter right 3°	1	Pc.
	3248-SA-0315-1162	Axis adapter right 6°	1	Pc.
	3248-SA-0315-1172	Axis adapter right 9°	1	Pc.
17	3248-SA-0315-1151	Axis adapter left 3°	1	Pc.
	3248-SA-0315-1161	Axis adapter left 6°	1	Pc.
	3248-SA-0315-1171	Axis adapter left 9°	1	Pc.
18	3248-SA-0315-1121	Strip	4	Pc.

Spare parts

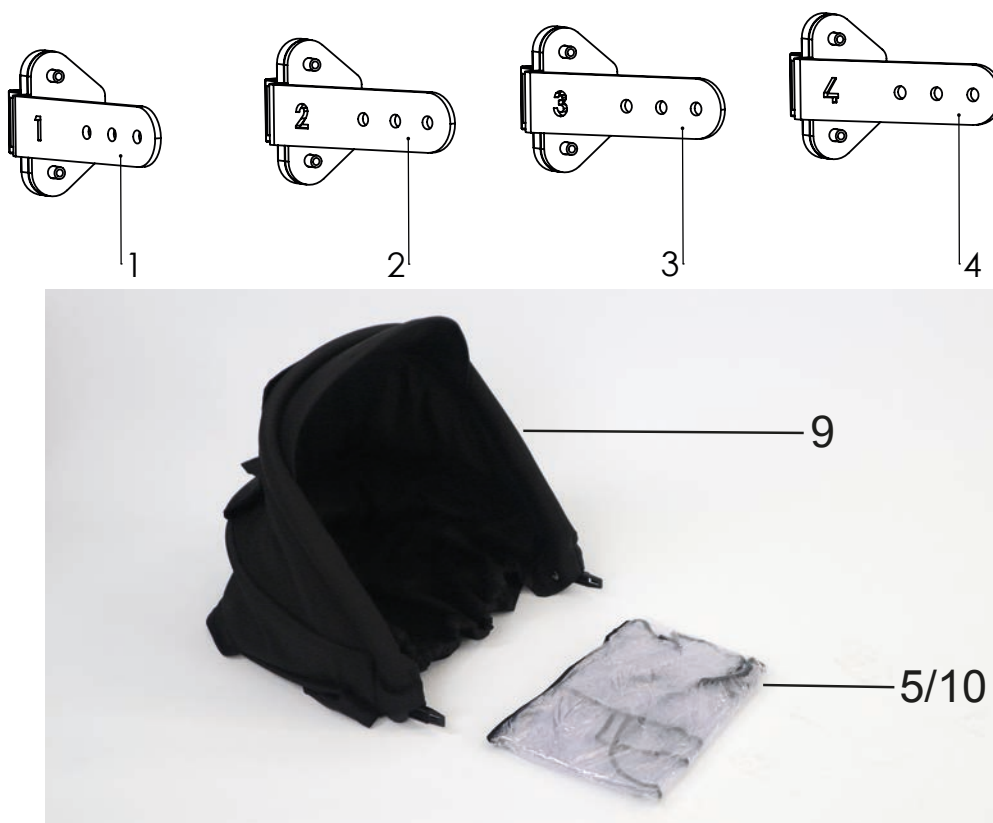

Pos	Article number	Articel description	Quantity	Unit
1, 5-9	3245-9300-306	Canopy incl. adapter and rain protection, black, suit to back 3245-29SW, SW 24-34	1	Pc.
1, 6-10	3245-9301-306	Canopy incl. adapter and ZIP 2 rain protection, black, suit to back 3245-29SW, SW 36-44	1	Pc.
2, 5-9	3245-9302-306	Canopy incl. adapter and rain protection, black, suit to back 3245-32SW, SW 24-26	1	Pc.
4, 5-9	3245-9303-306	Canopy incl. adapter and rain protection, black, suit to back 3245-32SW, SW 28-30	1	Pc.
3, 5-9	3245-9304-306	Canopy incl. adapter and rain protection, black, suit to back 3245-32SW, SW 32-34	1	Pc.
2, 6-10	3245-9305-306	Canopy incl. adapter and ZIP 2 rain protection, black, suit to back 3245-32SW, SW 36-44	1	Pc.
1	3245-9310E	Canopy bracket for back, angle adjustable and foldable: 3245-29SW	1	Set
2	3248-9310E	Canopy bracket with holder for mold form back 25°, SW 24, 26, 36, 40, 44	1	Set
3	3248-9330E	Canopy bracket with holder for mold form back 30°, SW 32 + 34	1	Set
4	3248-9320E	Canopy bracket with holder for mold form back 35°, SW 28 + 30	1	Set
<i>Fixing material not shown:</i>				
5	7000-0008	Rain cover for canopy	1	Pc.
6	1030-0514	Button head socket screw M5x14	4	Pc.
7	1044-0005	Hexagonal domed cap nut self-locking M5	4	Pc.
8	1055-0005	Washer A5,3	4	Pc.
9	7000-0083-306	Canopy with rain cover, black	1	Pc.
10	7000-0037	Rain cover for canopy Zip size 2, SW 36-44	1	Pc.

HOGGI GmbH
Eulerstraße 27 • 56235 Ransbach-Baumbach • Deutschland
Telefon: (+49) 2623 / 92 499-0
E-mail: info@hoggi.de • www.hoggi.de
