
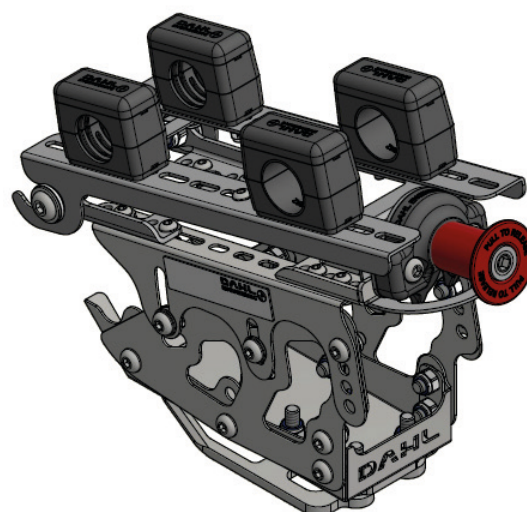
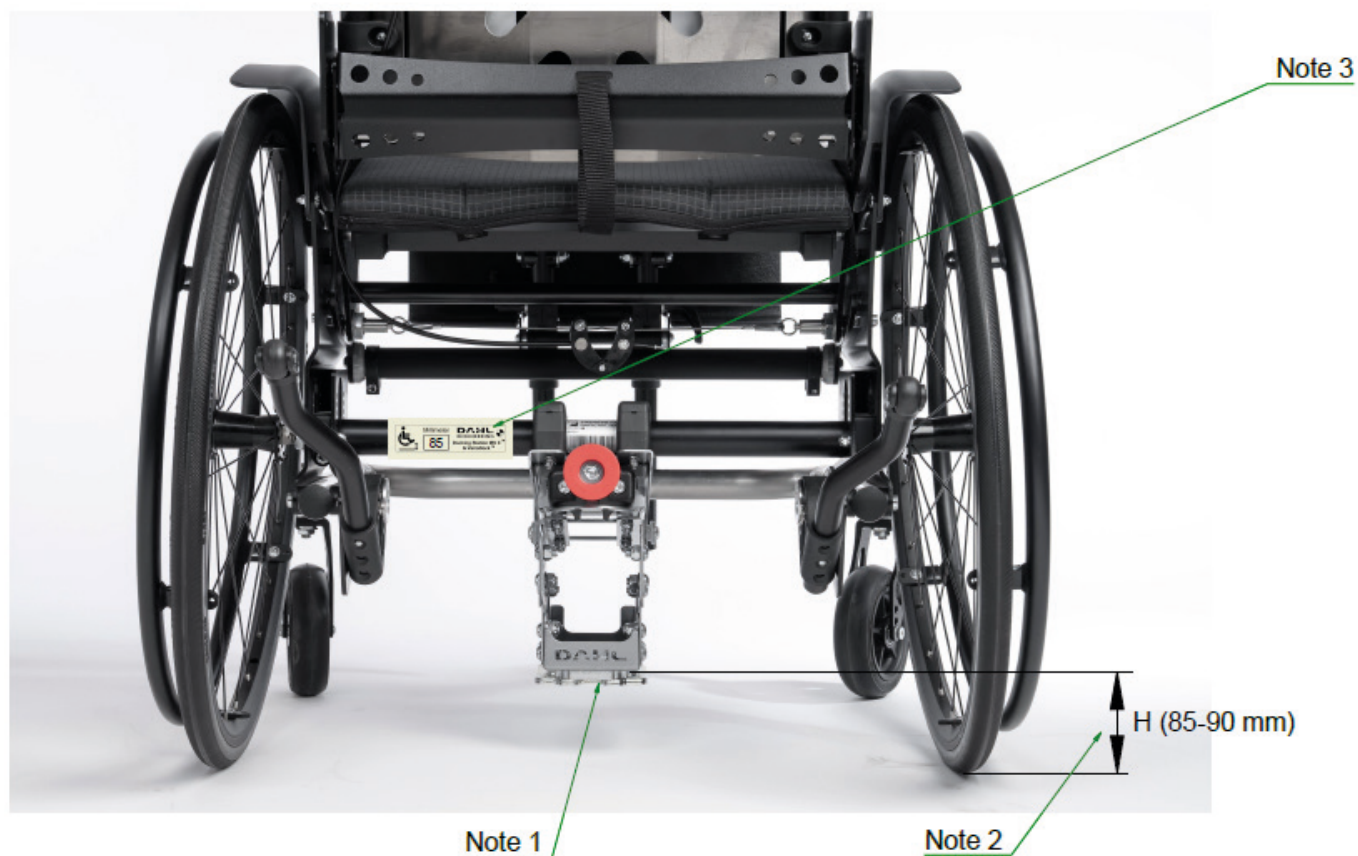
Docking-Adaptionsset 503044.
Docking Adaptation Kit 503044.

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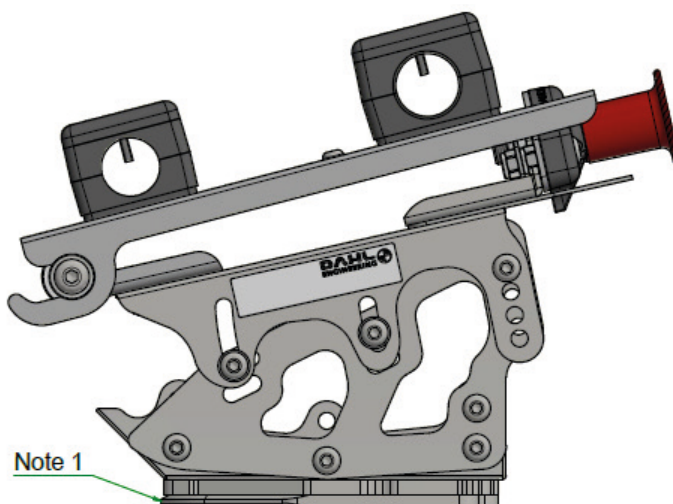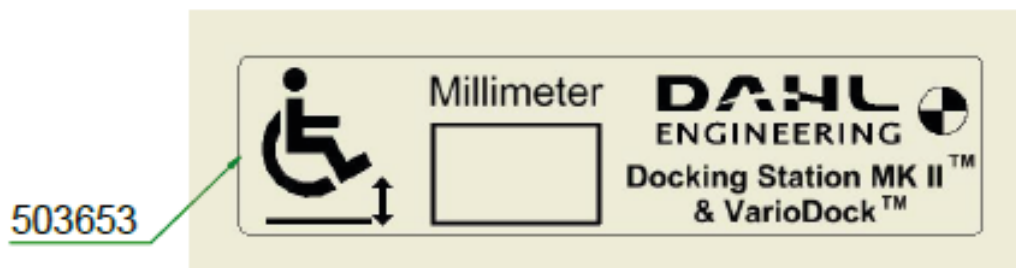
For the Docking Station to function properly the Locking Plate must be mounted horizontally.

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
When the entire assembly process is completed measure this height H with the user sitting in the wheelchair. Make sure the tire pressure is correct beforehand!

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

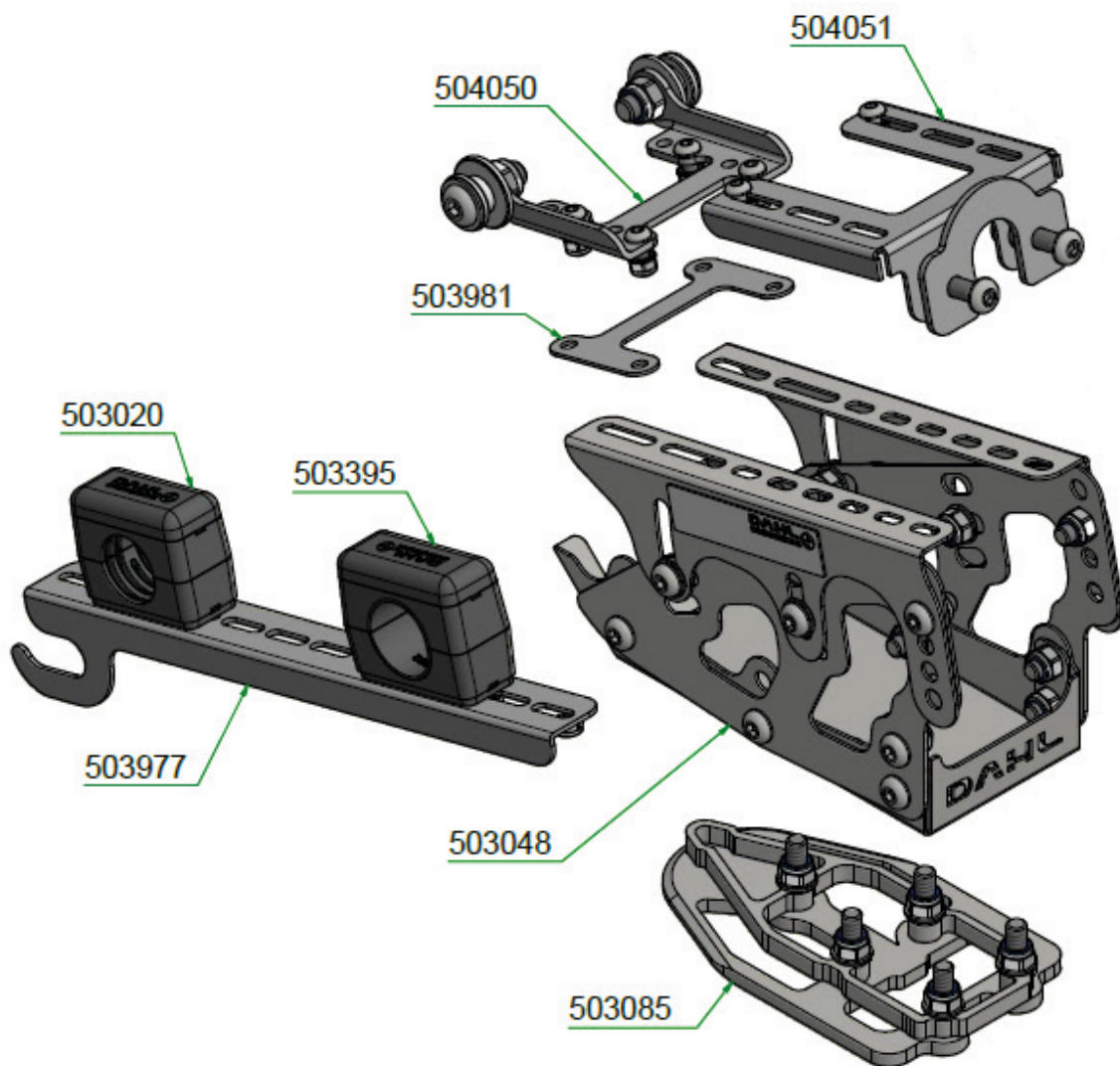
Record the measured height H minus 2 mm on the supplied sticker 503653 and place as shown

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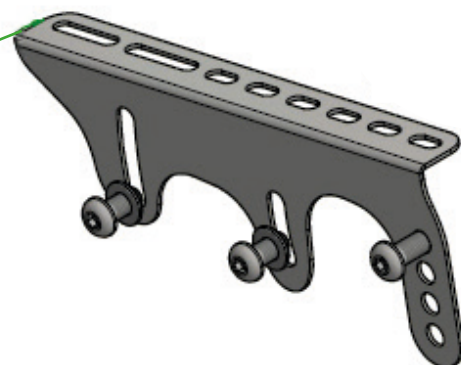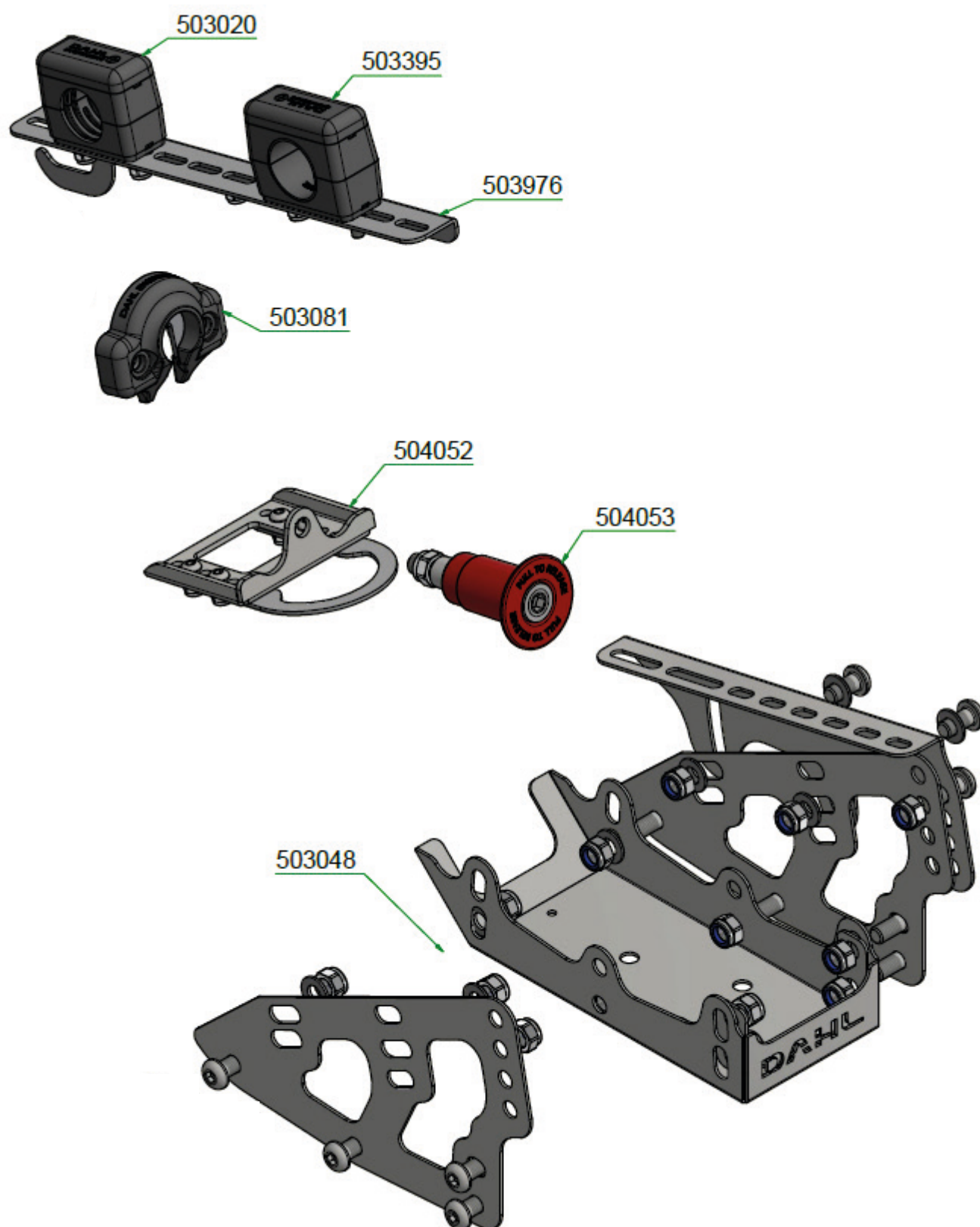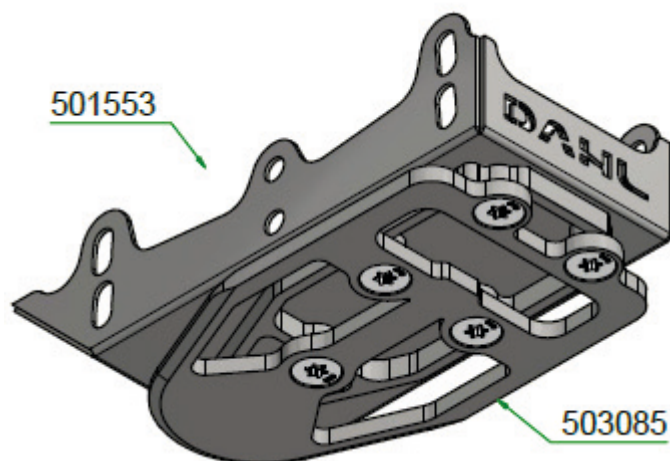
Considering the wheelchair's configuration and the resulting ground clearance of the locking plate, assemble 503048 and 503085 - see 501553 and 503044.

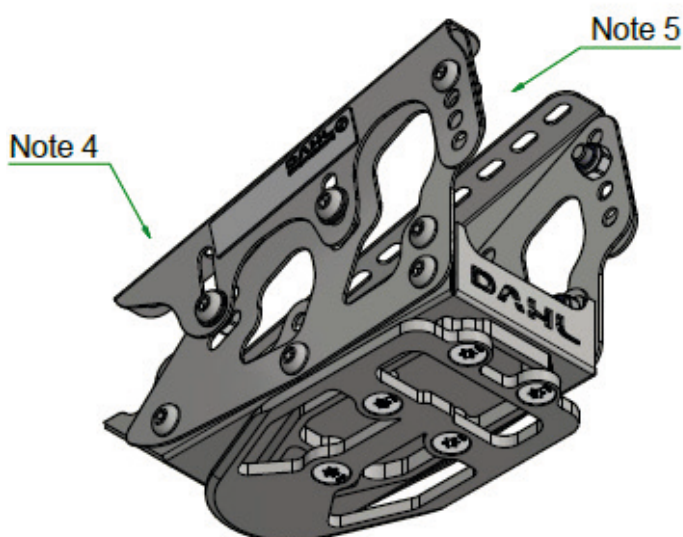
Do not completely tighten the screws before the final adjustment has been determined.

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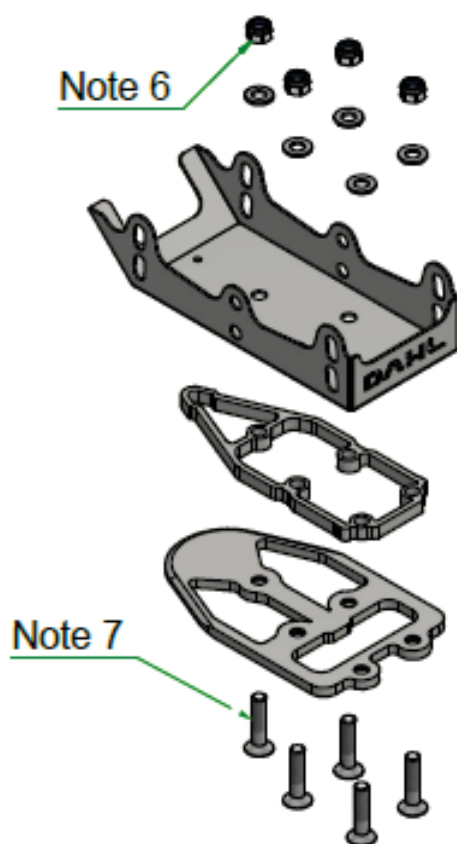
Do not use any other bolts for the locking plate than those supplied from Dahl Engineering (part no. 502800, quality 14.9, torx key size 27). Standard countersunk M8 bolts will not be strong enough in the event of a collision. A qualified and experienced technician must carry out the installation the Dahl MADSTTM adaptation system on your wheelchair.

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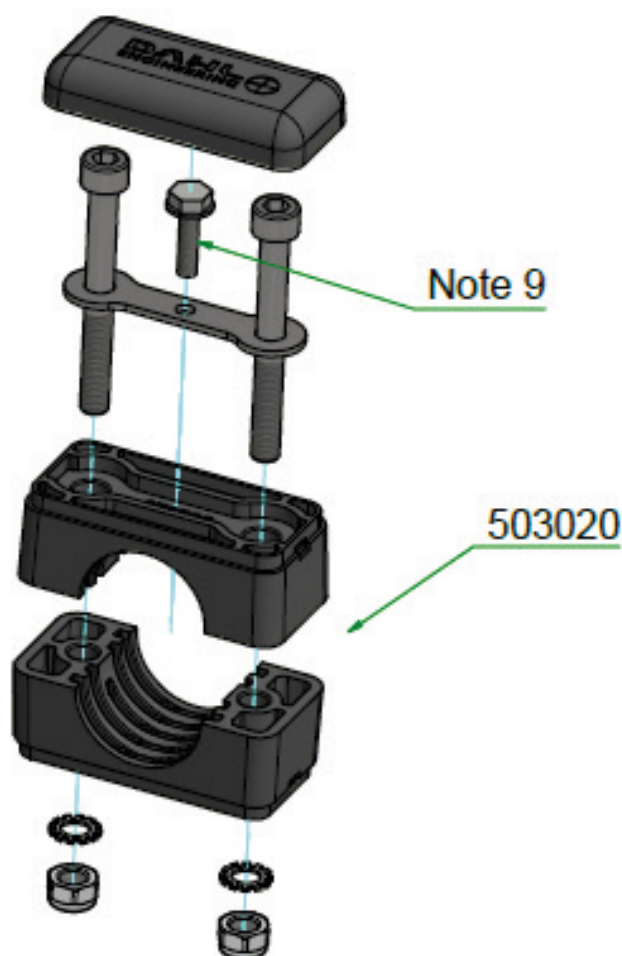Mount the clamps 503020 centered on the tube.

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

The thread forming screws 502729 (M5x16 mm) must be installed as locks against rotation. This is the LAST step of the installation process!

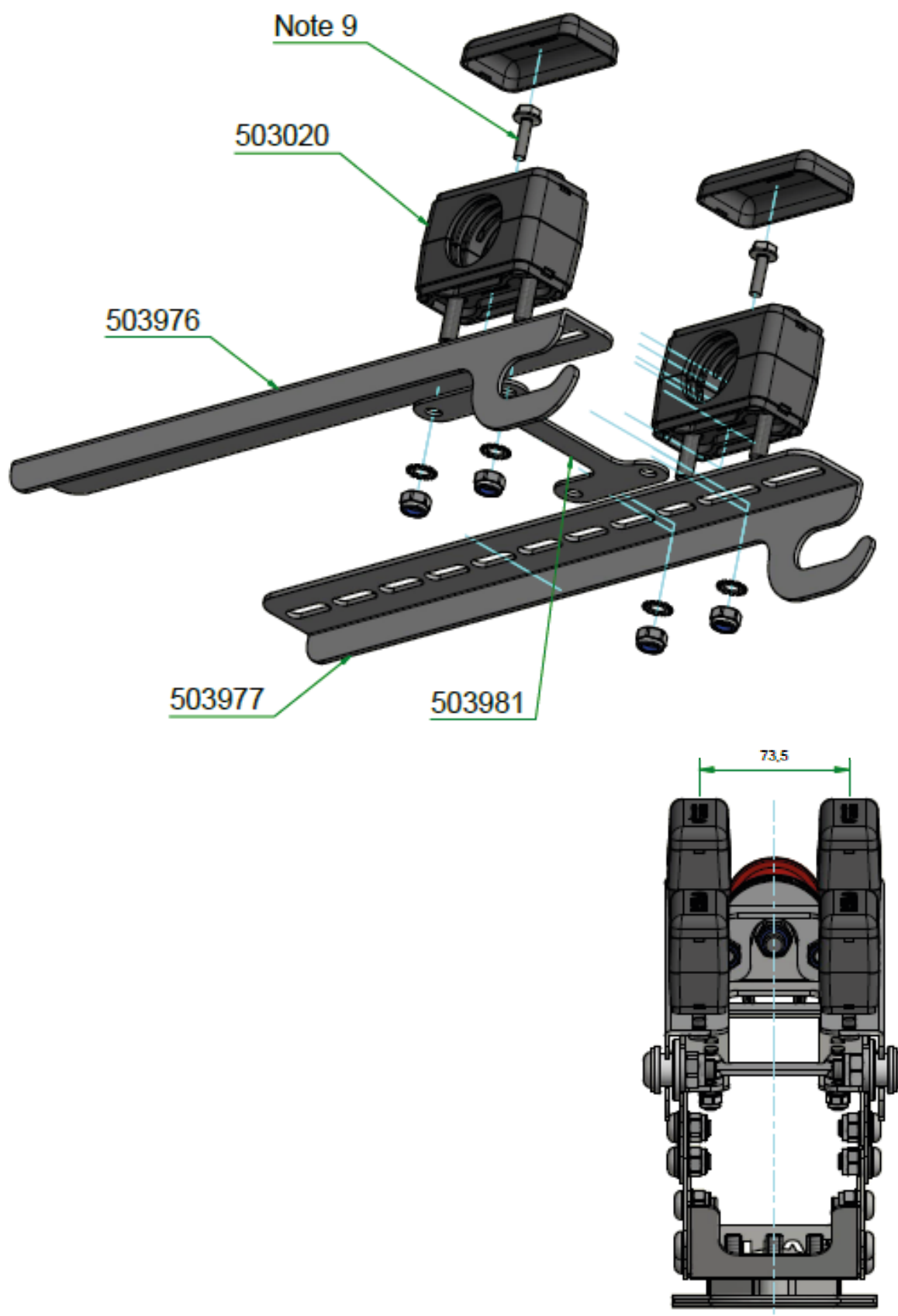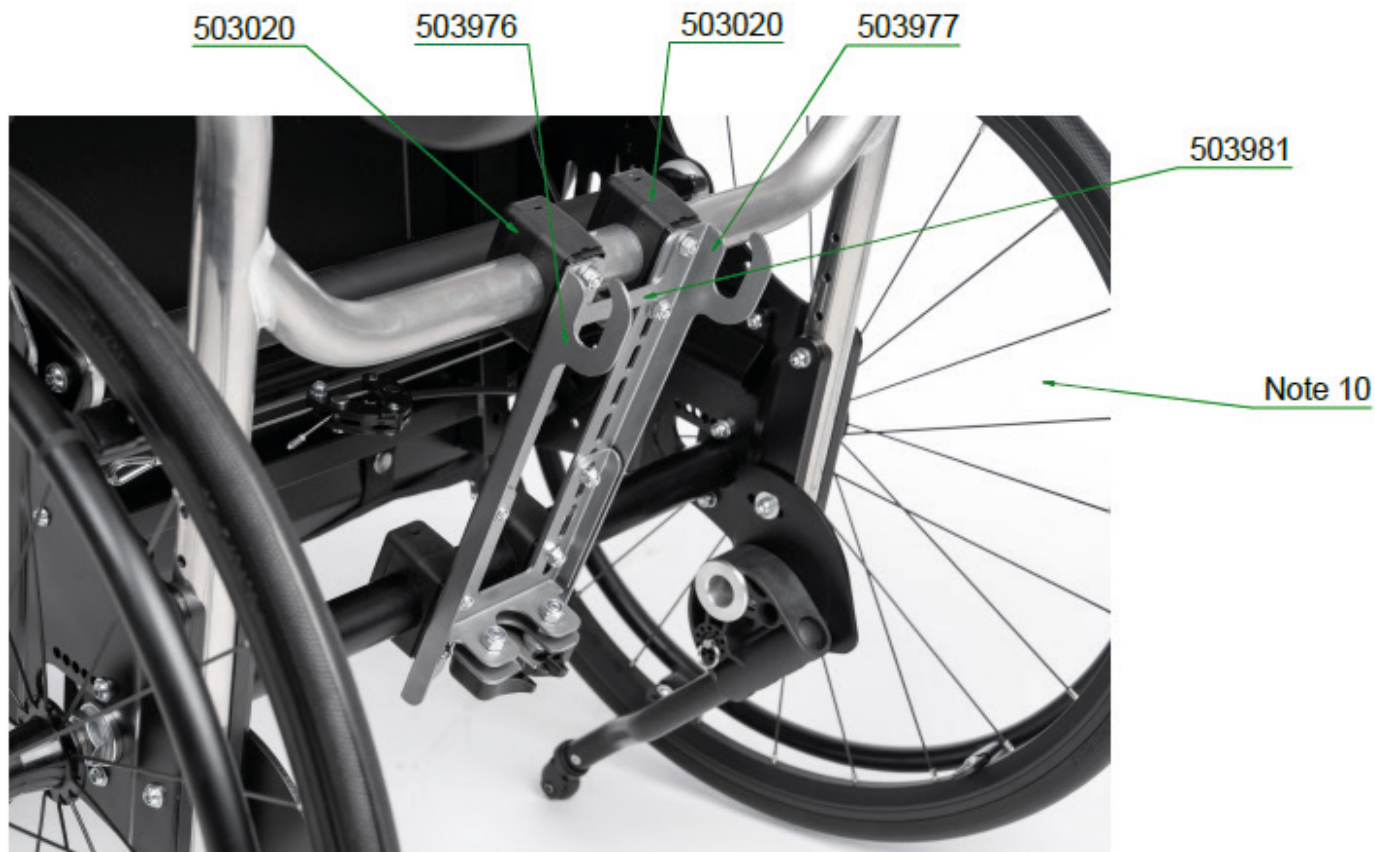After all elements have been assembled and adjusted, carry out a final functional test, whereafter Ø4 mm holes must be drilled through the plastic clamps and the wheelchair metal tube. Tighten screws with 4 Nm.

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
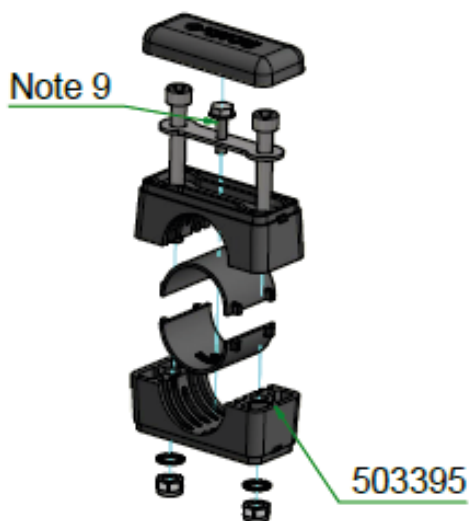
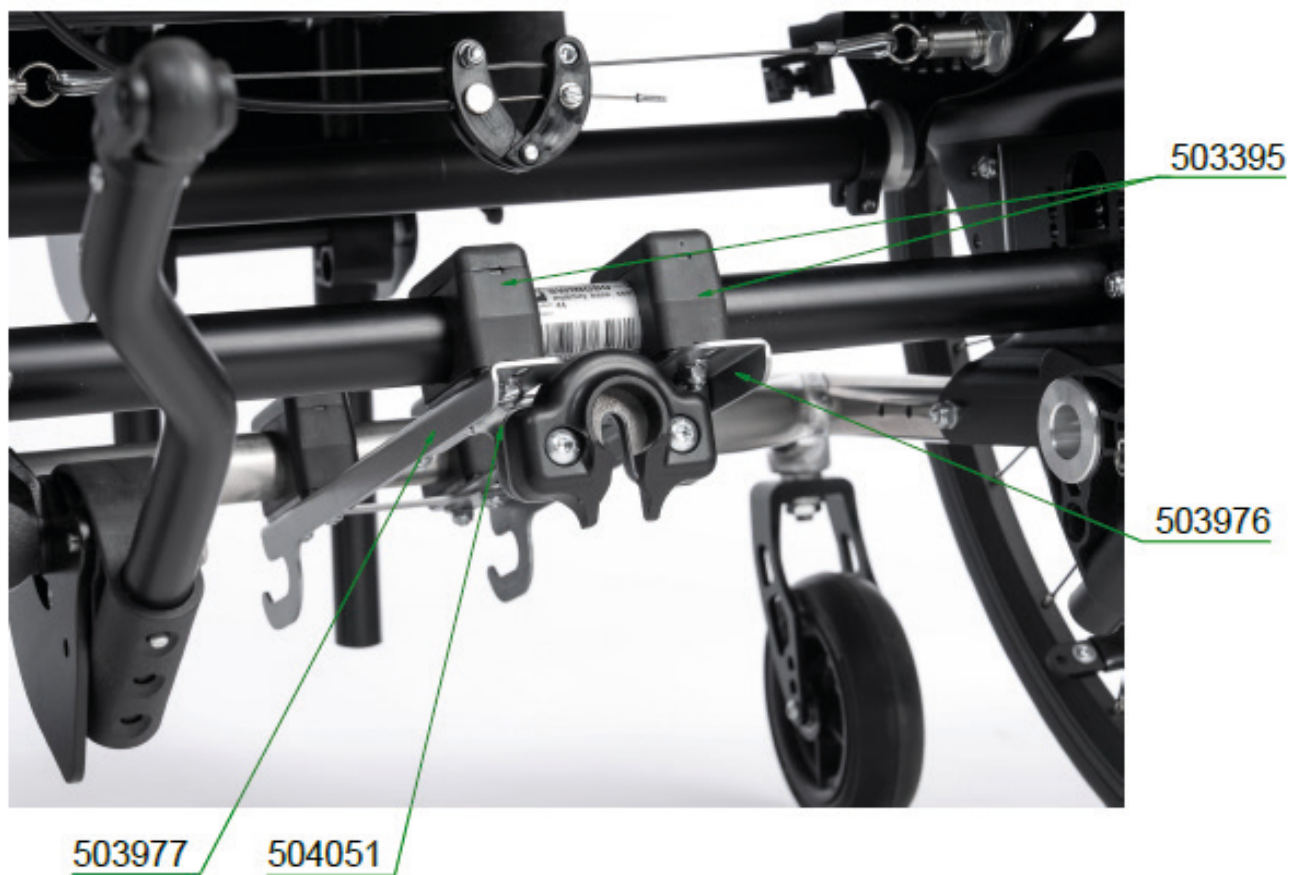
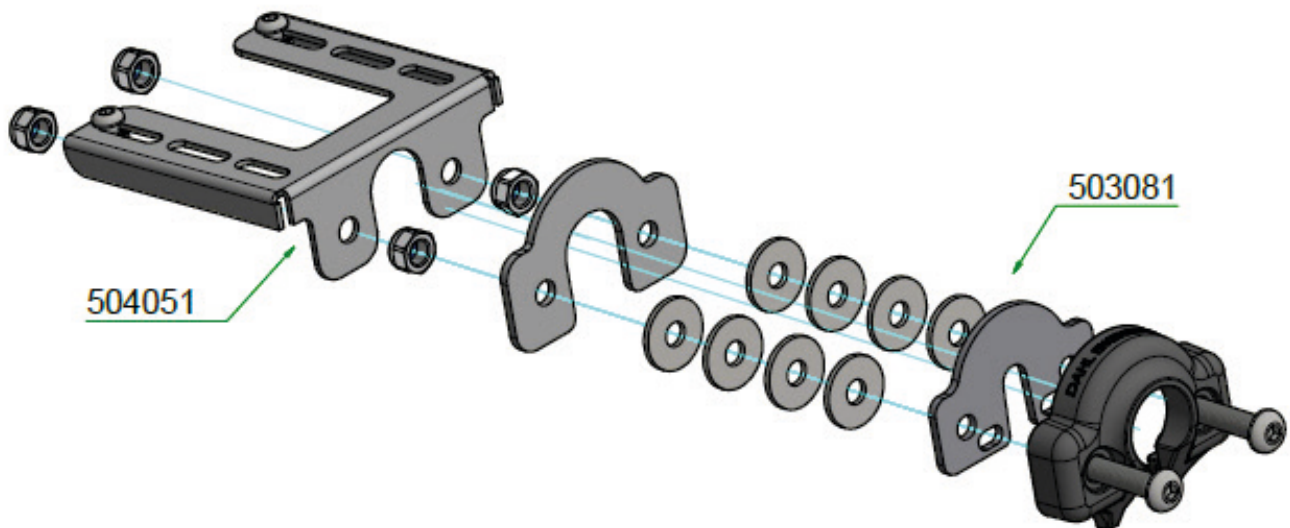
Mount the pipe holders 503395 centered on the tube.

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
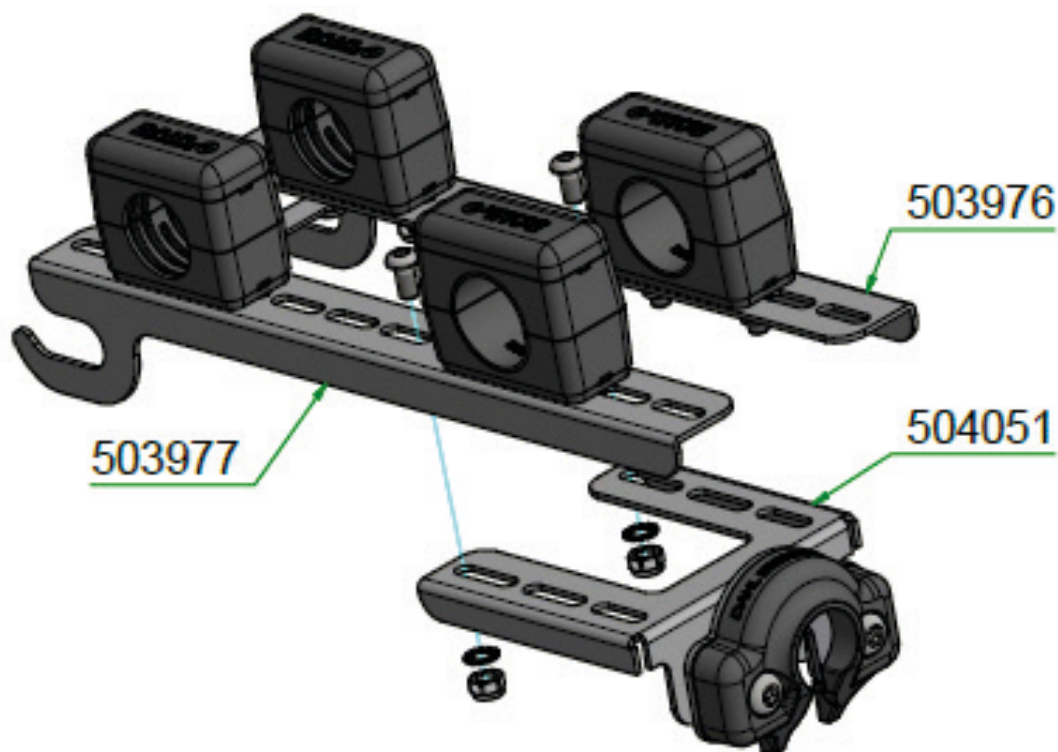
503081 is assembled with bracket 504051. Tightening Torque 5 Nm.

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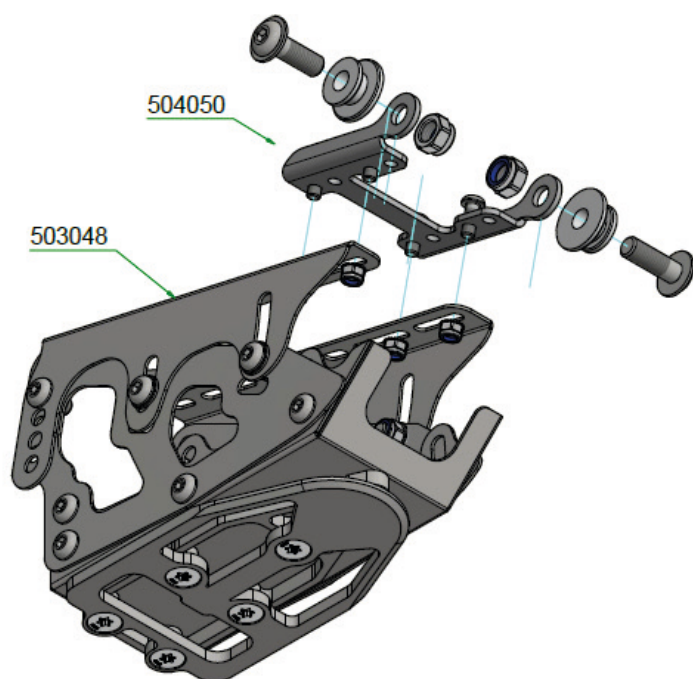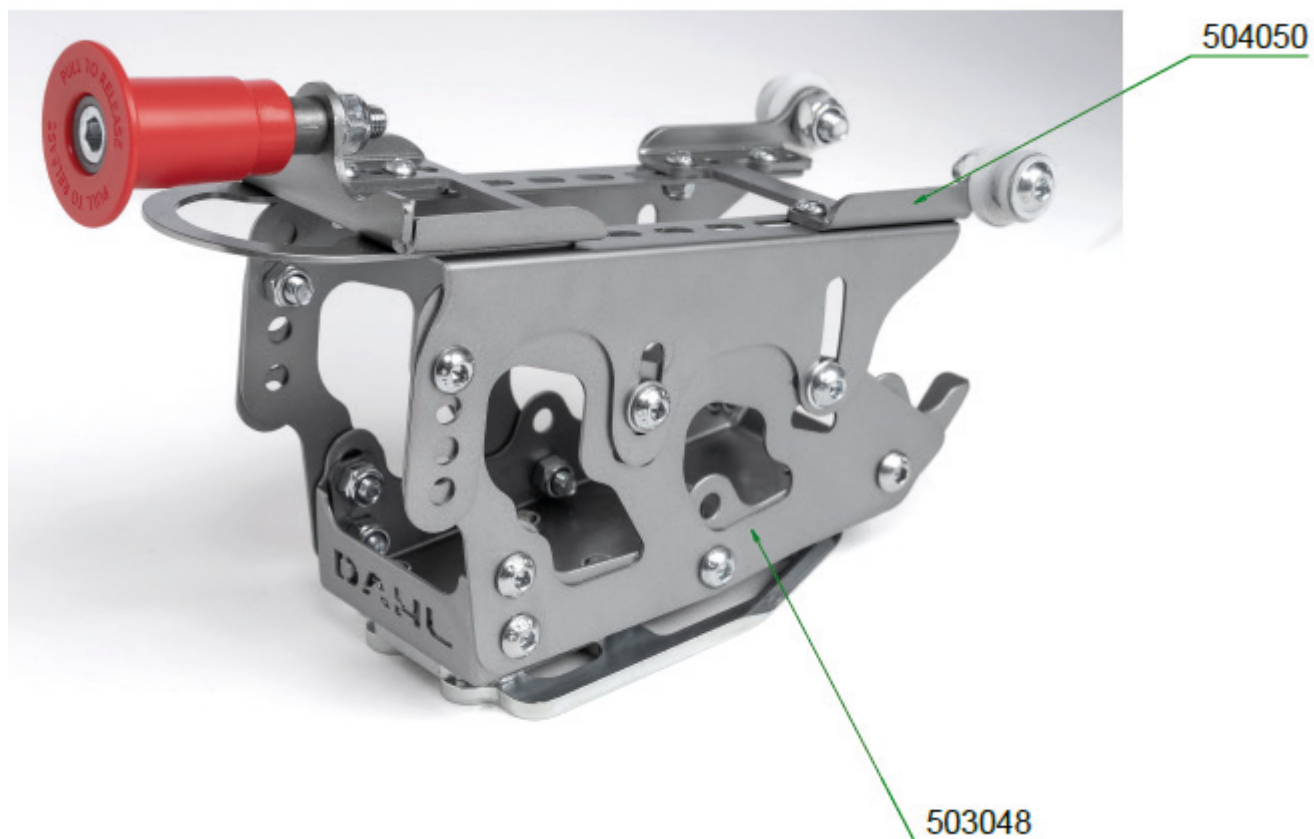
504051 is assembled with brackets 503976 and 503977.
Tightening Torque 5 Nm.

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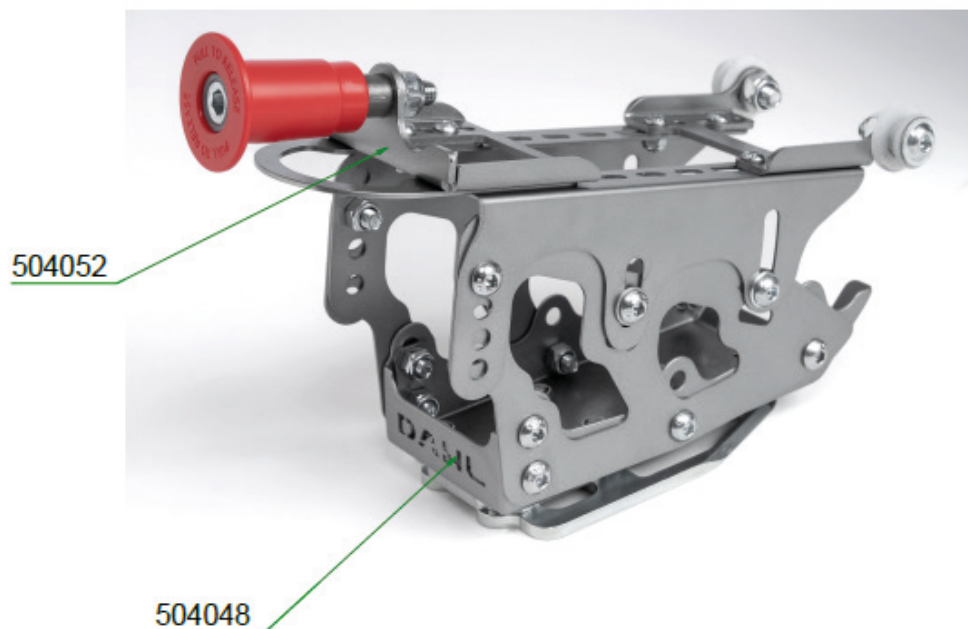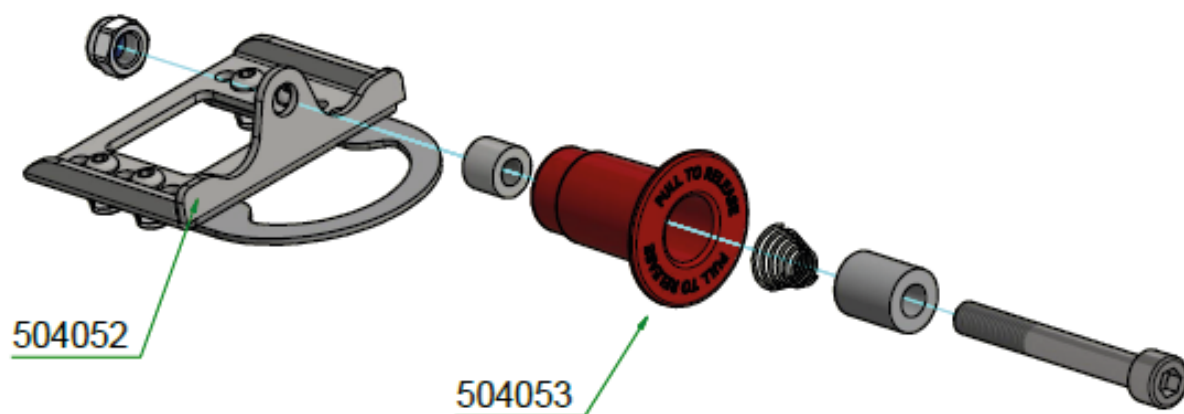
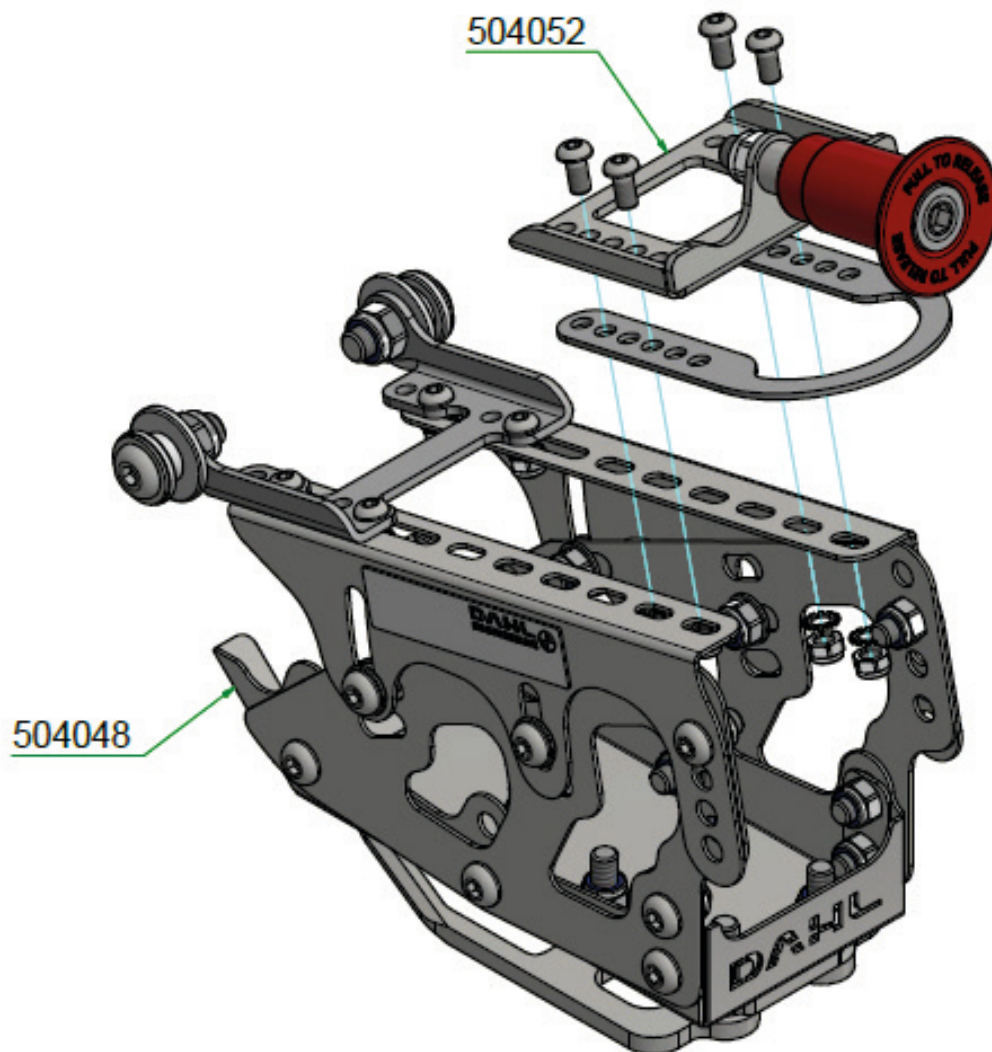
Locking Mechanism 504053 is assembled with bracket 504052.

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

Place the grooves of the plastic rollers 502914 into the hook brackets 503976 and 503977.

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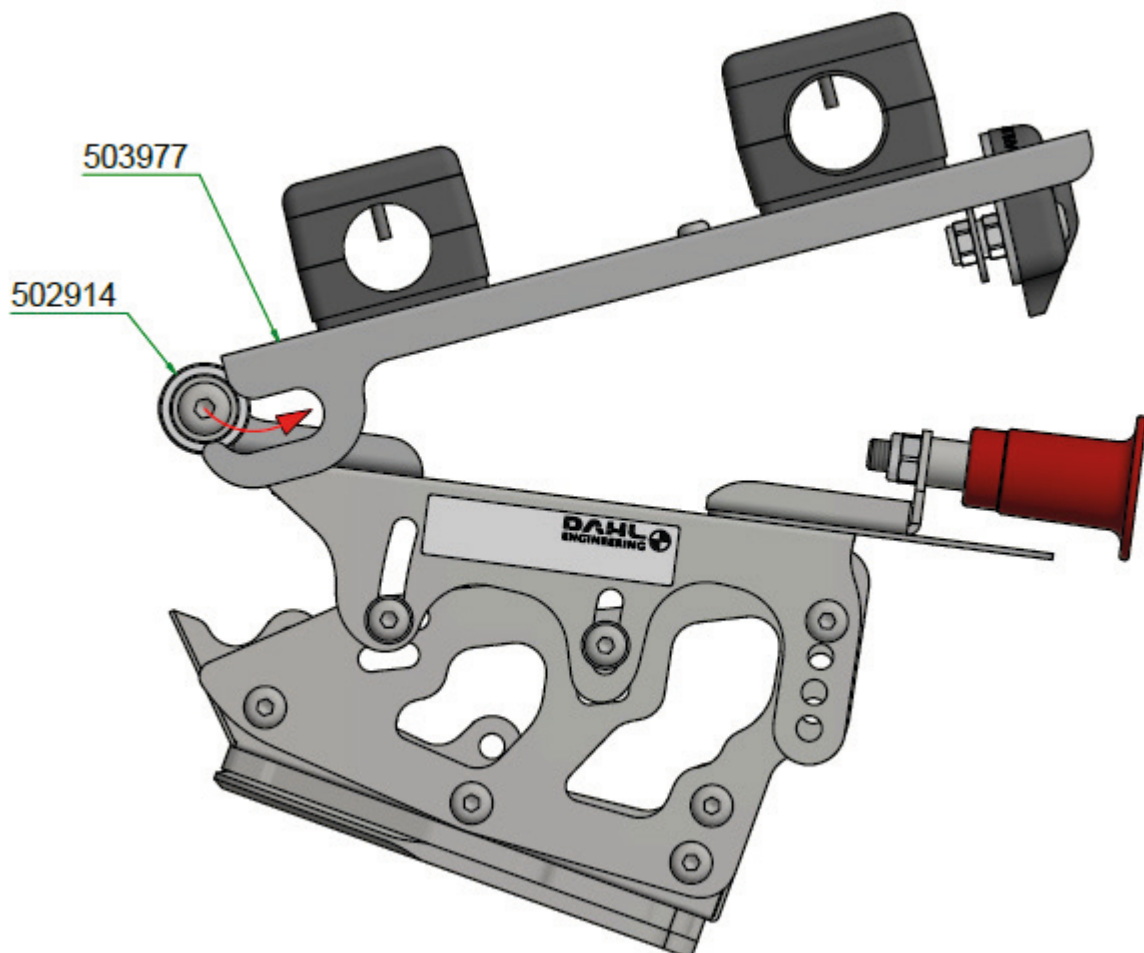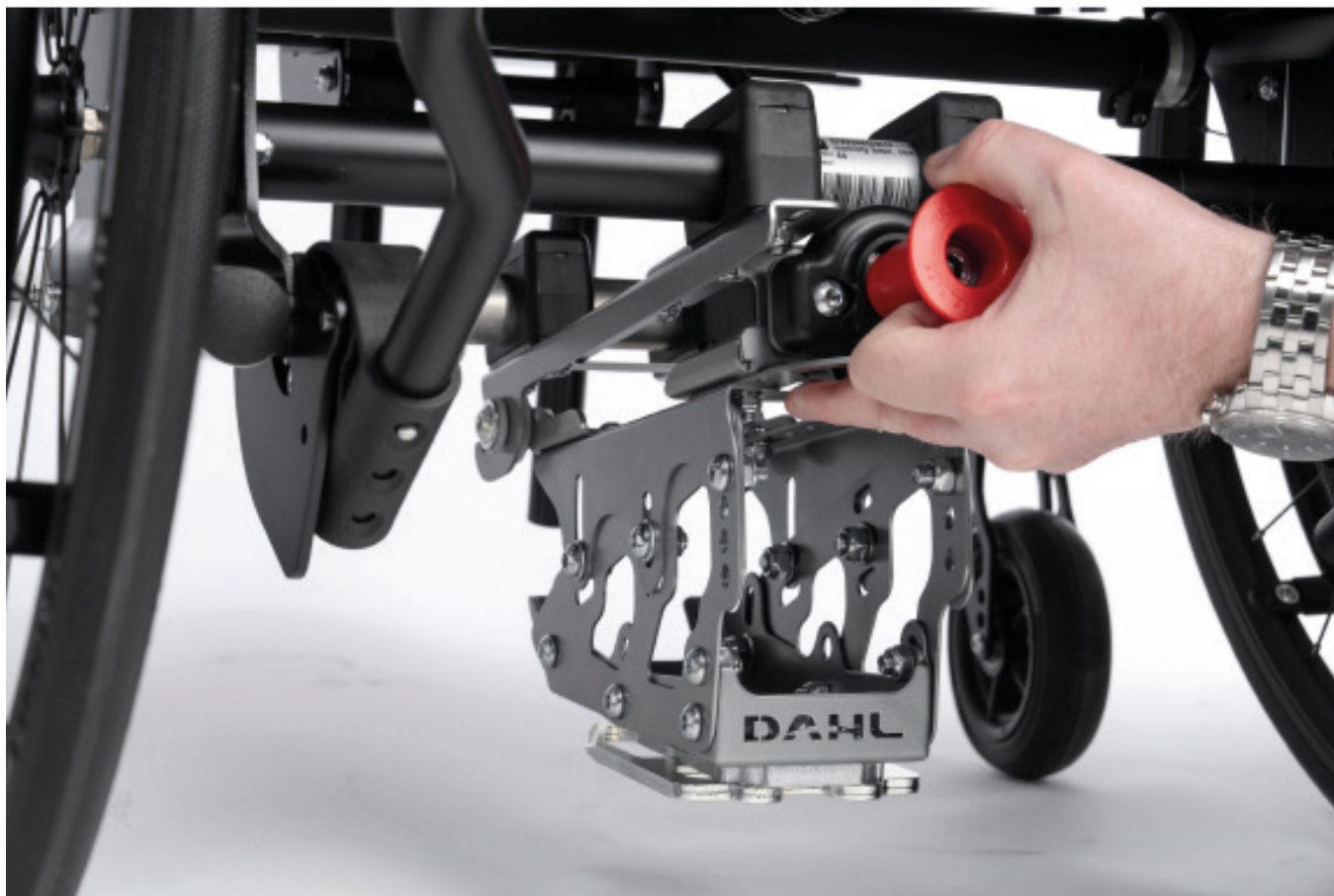Pull back console, until the plastic rollers 502914 are placed in the bottom of the hooks on both sides.

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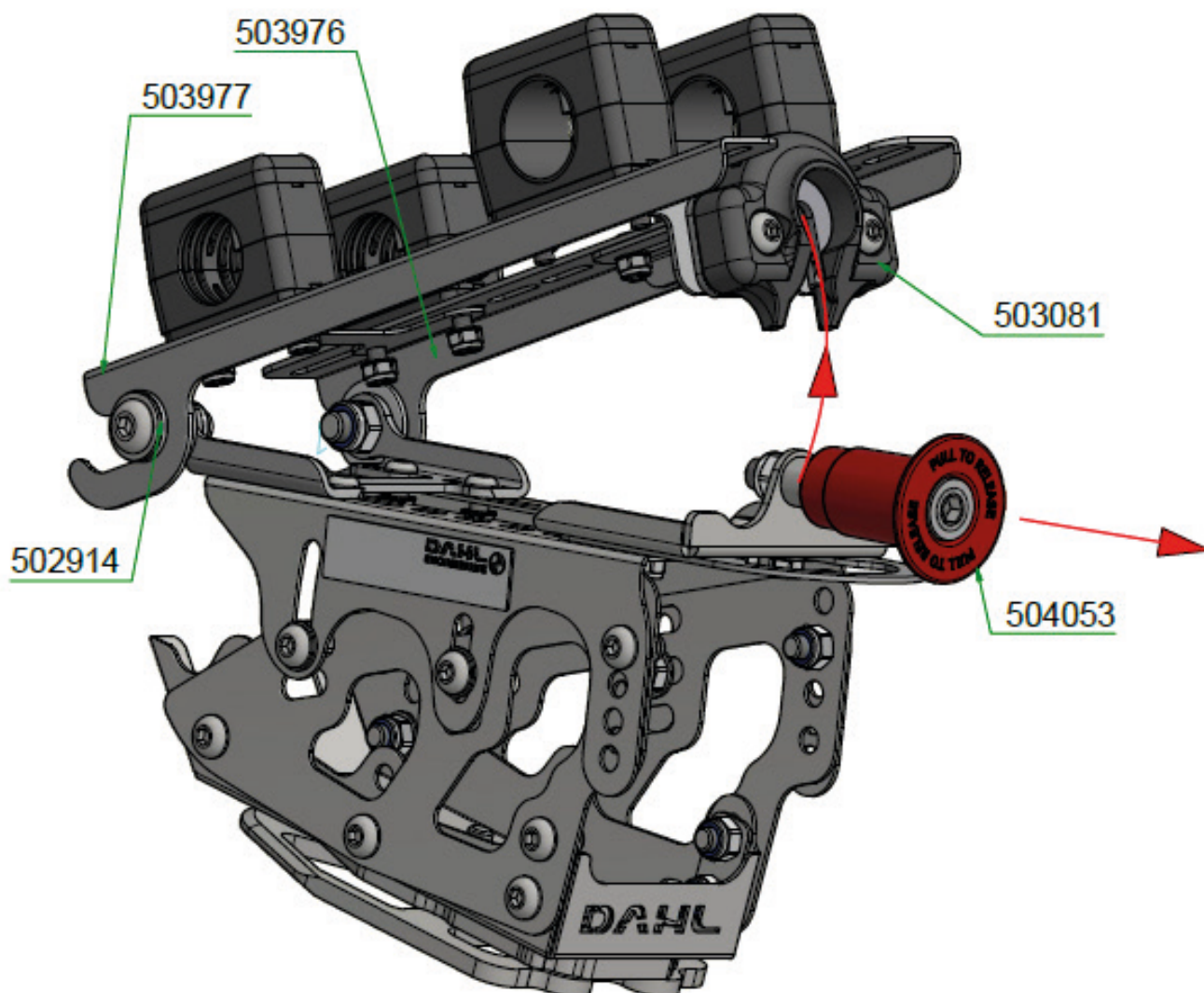
With the plastic rollers 502914 correctly placed in the hooks 503976 and 503977, pull out the red plunger 504053 and swing up the console toward the locking socket 503081. Release plunger when aligned with the hole in the socket.
To release the console, carry out the same steps in reversed order.

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The locking tube 504053 is not locked correctly, until the shiny tip (A) on the red locking plunger is fully inserted into the socket of the locking bracket 503081, whereafter the shiny tip (A) will no longer be visible.

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Incorrect installation of the Mads adaptation kit on the wheelchair could cause serious injury or death in a crash. Always check the specific wheelchair manufacturer's user and installation guides to assure the correct parts are properly installed on your specific wheelchair for use with the Dahl Docking Station. Do not drive the vehicle until wheelchair and user are correctly secured.