
HOGGI[®]



COBRA Spare parts catalogue

en

Index

1	Modellidentification and basic equipment	
1.1	Modelidentification	3
1.2	Basic equipment	3
2	Assemblies	
2.1	Assembly overview	4
3	Spare parts	
3.1	Base frame	5
3.2	Lower swivel arm	6
3.3	Upper swivel arm	7
3.4	Guid handle	8
3.5	Seat bearing plate, size 1	9
3.7	Reduction set	10
3.6	Seat bearing plate, size 2	11
3.8	Cross strut	12
3.9	Frame cover	13
3.10	Lift device with two gas spring`s	14
3.11	Part`s list for Lift device with two gas spring`s	15
3.12	Lift device with one gas spring	16

1 Modelidentification

1.1 Model identification

Please give us the Serial number with every spare part order, to ensure a correct delivery.


The diagram shows a product label for HOGGI COBRA Zimmeruntergestell Gr.1. The label is divided into three horizontal sections. Each section contains the following information:

- HOGGI** logo and company details: HOGGI GmbH, Eulerstraße 27, D-56235 Ransbach-Baumbach, MADE IN GERMANY.
- COBRA** logo and product name: Zimmeruntergestell Gr.1.
- Item number: 3004-0000-032.
- Maximum user weight / Maximum load: Max. load: 50 kg.
- CE-Identification.
- Barcode with SN (Serial Number) 201831975.

Labels on the right side of the diagram point to specific fields:


- Colour code (points to the top-left corner of the label).
- Item number (points to 3004-0000-032).
- Size (points to 3004-0000-032).
- Serial number (points to SN 201831975).
- CE-Identification identification (points to the CE mark).
- Type designation / Productname (points to COBRA Zimmeruntergestell Gr.1).
- Maximum user weight / Maximum load (points to Max. load: 50 kg).
- Manufacturer (points to HOGGI).
- Manufactur (points to HOGGI GmbH).
- Year of manufactur (points to 2018 in the SN).

2 Component overview

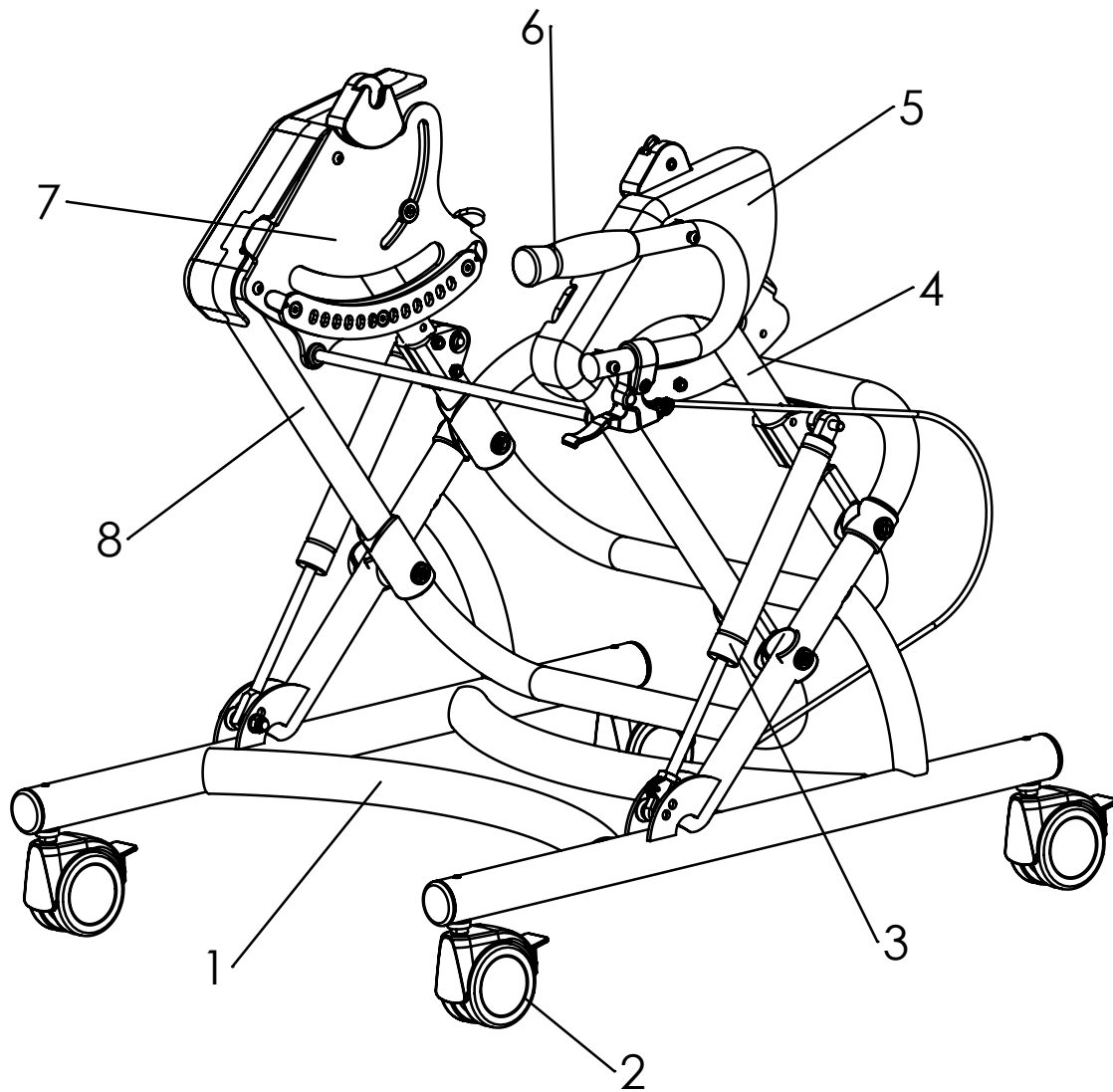


Pos	Articel number	Articel description	Page
1	3004-1100E	Base frame	5
2	124x-xxxx	Wheel typ`s	5
3	3004-1601	Lift device	ab 14
4	3004-1300E	Upper swivel arm	7
5	300X1500E	Frame cover	13
6	3004-1320E	Guid handle	8
7	300X-1410	Seat bearing plate	ab 9
8	3004-1200E	Lower swivel arm	6

Spare parts

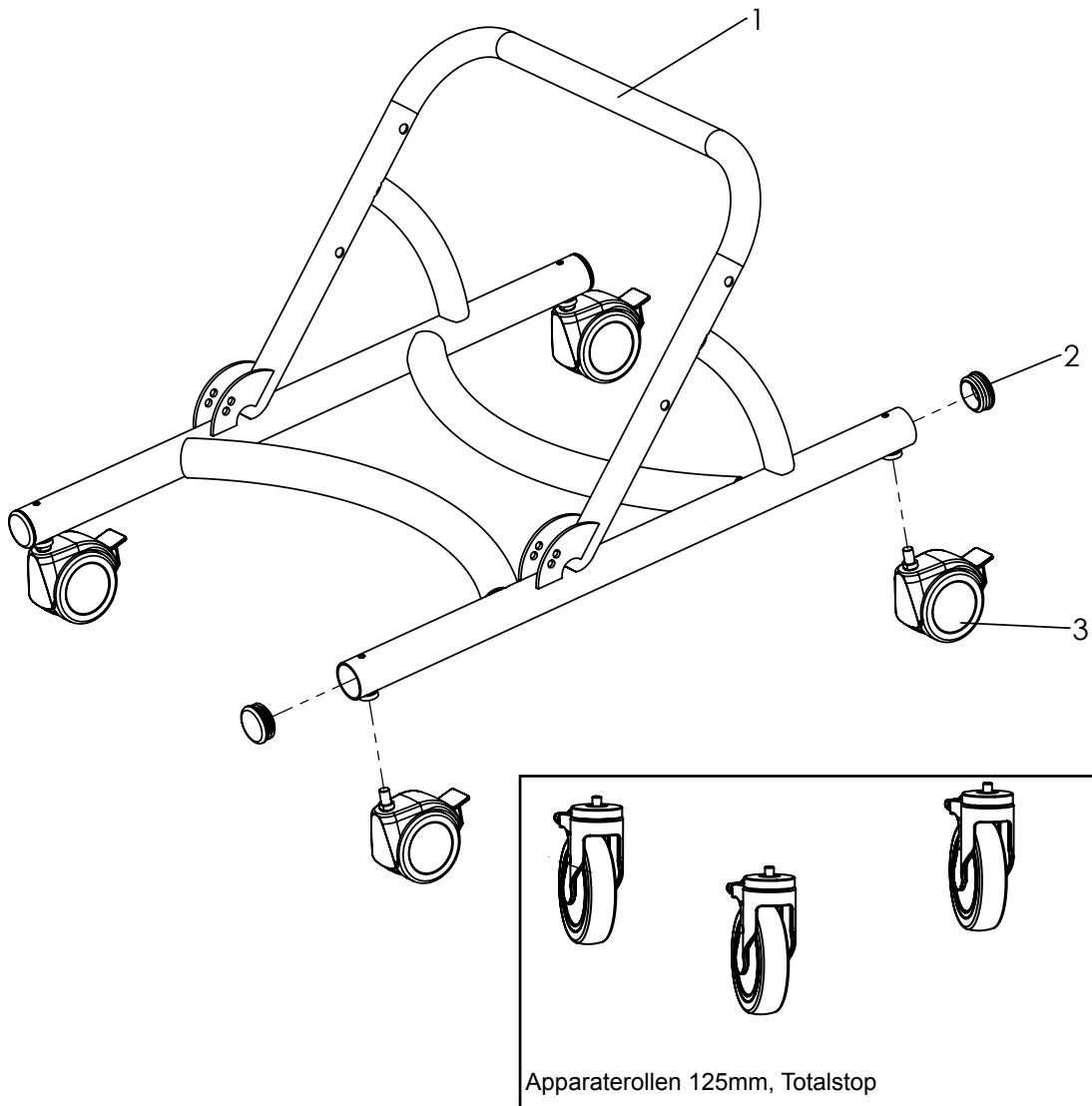


Pos	Article number	Articel description	Quantity	Unit
1-2	3004-1100E-000	COBRA base frame incl. Ribbed insert	1	Pc.
1	3004-1110-000	Base frame	1	Pc.
2	1141-0021	Ribbed insert 40x0,8-2,5	4	Pc.
3	1241-0001	Twin wheel castor, 75 mm, with wheel lock	4	Pc.
	1240-0002	Apparatus castor125 mm, total lock	4	Pc.

Spare parts

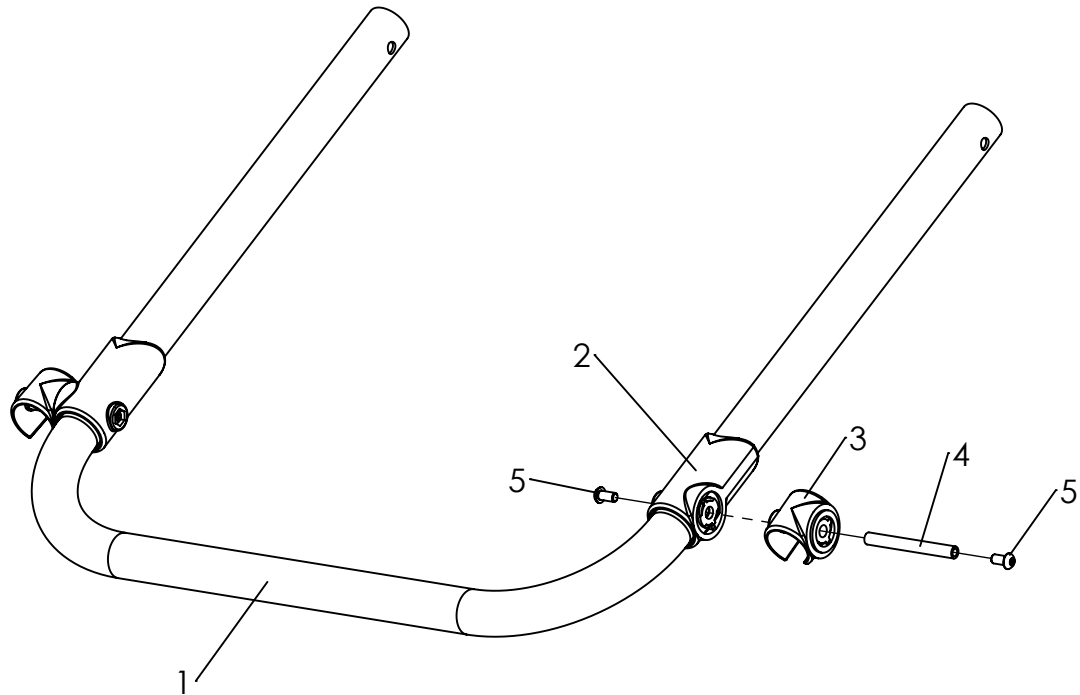


Pos	Article number	Articel description	Quantity	Unit
1-6	3004-1200E-000	Lower swivel arm, complete	1	Pc.
1	3004-1211-000	Lower swivel arm	1	Pc.
2	3313-1321	Support tube bearing open	2	Pc.
3	3313-1160	Hinge bandage D30	2	Pc.
4	1114-0057	Threaded bushing 8xM6x71	2	Pc.
5	1029-0612	Button head socket screw M6x12	4	Pc.

Spare parts

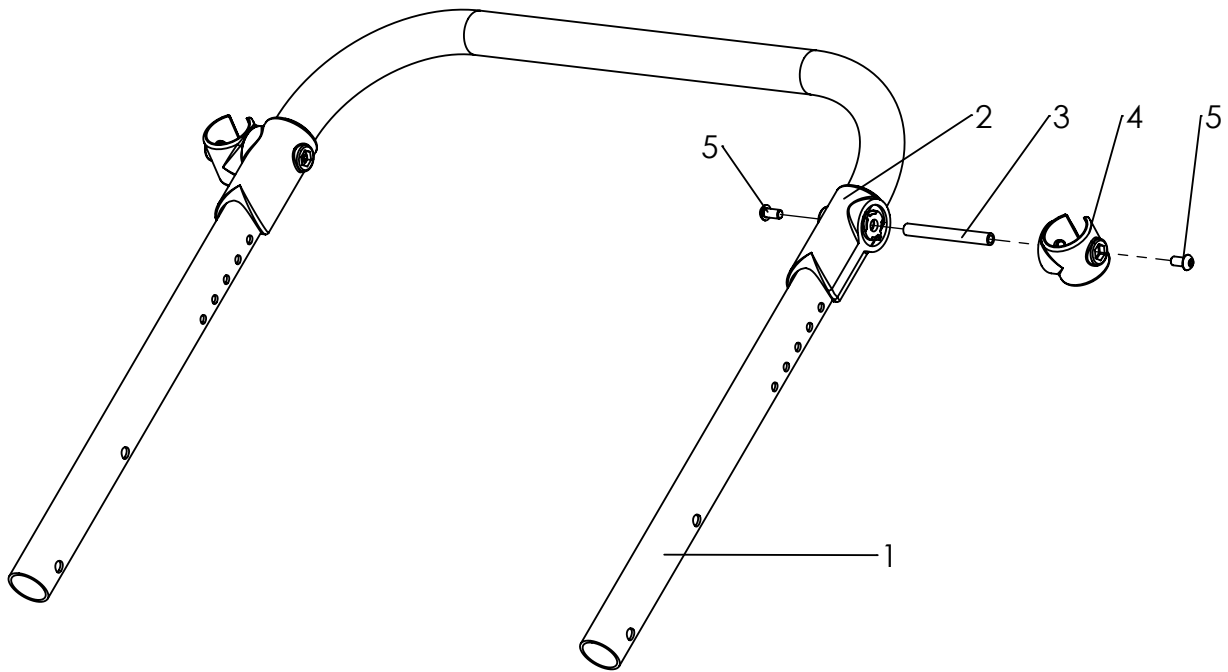


Pos	Article number	Articel description	Quantity	Unit
1-6	3004-1300E-000	Upper Swivel arm, complete	1	Pc.
1	3004-1311-000	Upper Swivel arm,	1	Pc.
2	3313-1321	Support tube bearing open	2	Pc.
3	1114-0057	Threaded bushing 8xM6x71	2	Pc.
4	3313-1160	Hinge bandage D30	2	Pc.
5	1029-0612	Button head socket screw M6x12	4	Pc.

Spare parts

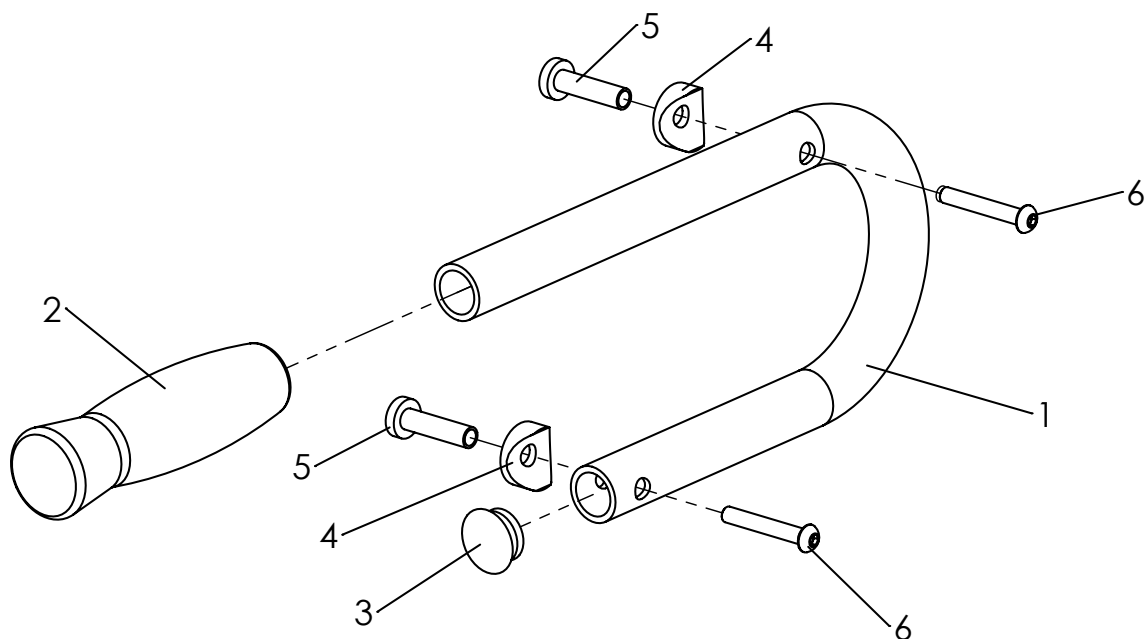


Pos	Article number	Articel description	Quantity	Unit
1-6	3004-1320E-000	Guide handle, complete	1	Pc.
1	3004-1321-000	Guide handle	1	Pc.
2	1143-0004	Handle grip black d22 x 120	1	Pc.
3	1141-0009	Ribbed insert 25x1-2,5	1	Pc.
4	1111-0012	Saddle washer PA black 24x8.4x3	2	Pc.
5	3004-1335	Spacer bushing guide handle	2	Pc.
6	1029-0640	Button head socket screw M6x40	2	Pc.

Spare parts

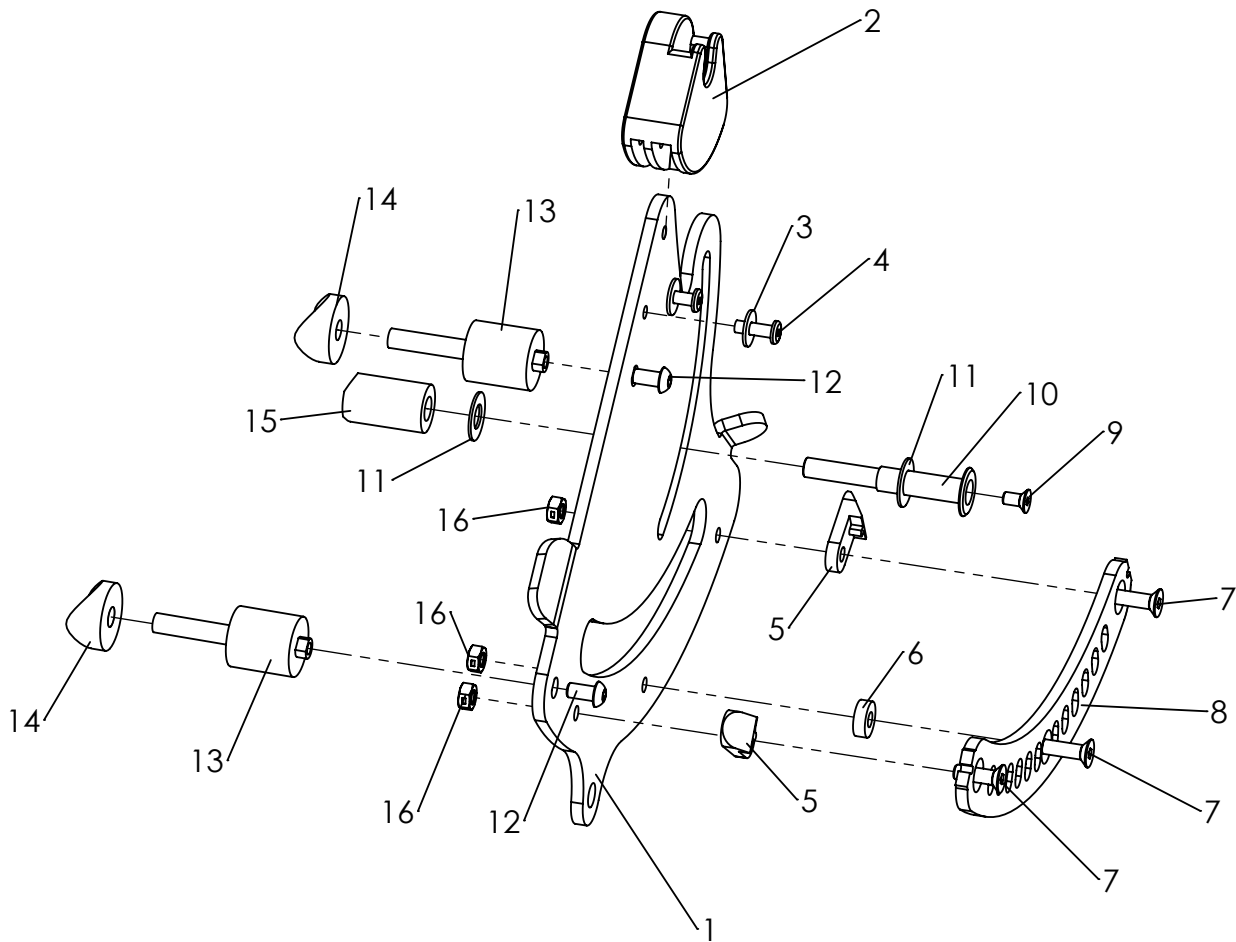

Pos	Article number	Articel description	Quantity	Unit
1-16	3004-1410E-000	Seat bearing plate, left, size 1, complete	1	Pc.
	3004-1420E-000	Seat bearing plate, right, size 1, complete	1	Pc.
1-4	3004-1411E-000	Seat bearing plate, left	1	Pc.
	3004-1421E-000	Seat bearing plate, right	1	Pc.
3	3231-1516	Scantlings bearing	1	Pc.
4	1055-0005	Washer A5,3	2	Pc.
5	1062-0516	Blind rivet F5x16	2	Pc.
5	3231-1514	Latch ramp	2	Pc.
6	1113-0051	Washer PA 6,4x15x5	1	Pc.
7	1024-0620	Hexagon socket countersunk head screw M6x20	3	Pc.
8	3231-1513-006	Scantlings rail	1	Pc.
9	1024-0510	Hexagon socket countersunk head screw M5x10	1	Pc.
10	3004-1345	Guide shaft seat bearing long	1	Pc.
11	1113-0050	Washer PA 10x21x1	2	Pc.
12	1029-6001	Button head socket screw,Tufflock M6x12	2	Pc.
13	3004-1412	Bearing axle swivel arm long	2	Pc.
14	1111-0006	Saddle washer PA black 28x8.4x5	2	Pc.
15	3004-1343	Bearing guide spacer, shaft long	1	Pc.
16	1049-0068	BILOC-nut M6	3	Pc.

Spare parts

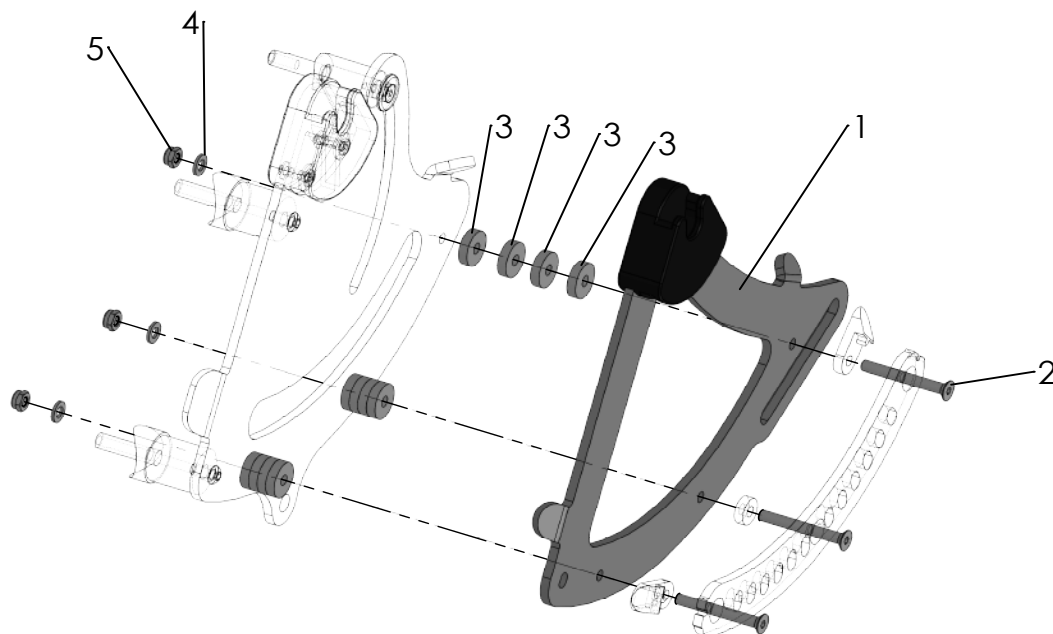


Pos	Article number	Articel description	Quantity	Unit
1-5	3004-1440E-032	Reduction set by one size	1	Set
1	3231-1511E-032	Seat Adapter plate left with scantling bearing	1	Pc.
	3231-1512E-032	Seat Adapter plate right with scantling bearing	1	Pc.
2	1024-0650	Hexagon socket countersunk head screw M6x50	6	Pc.
3	3274-7314	Spacer bushing 20 x 6.2 x 5	24	Pc.
4	1051-0006	Washer A6,4	6	Pc.
5	1042-0006	Hexagon nut self-locking M6	6	Pc.

Size 2 reduced to size 1


Size 1 reduced to Mini 1

Spare parts

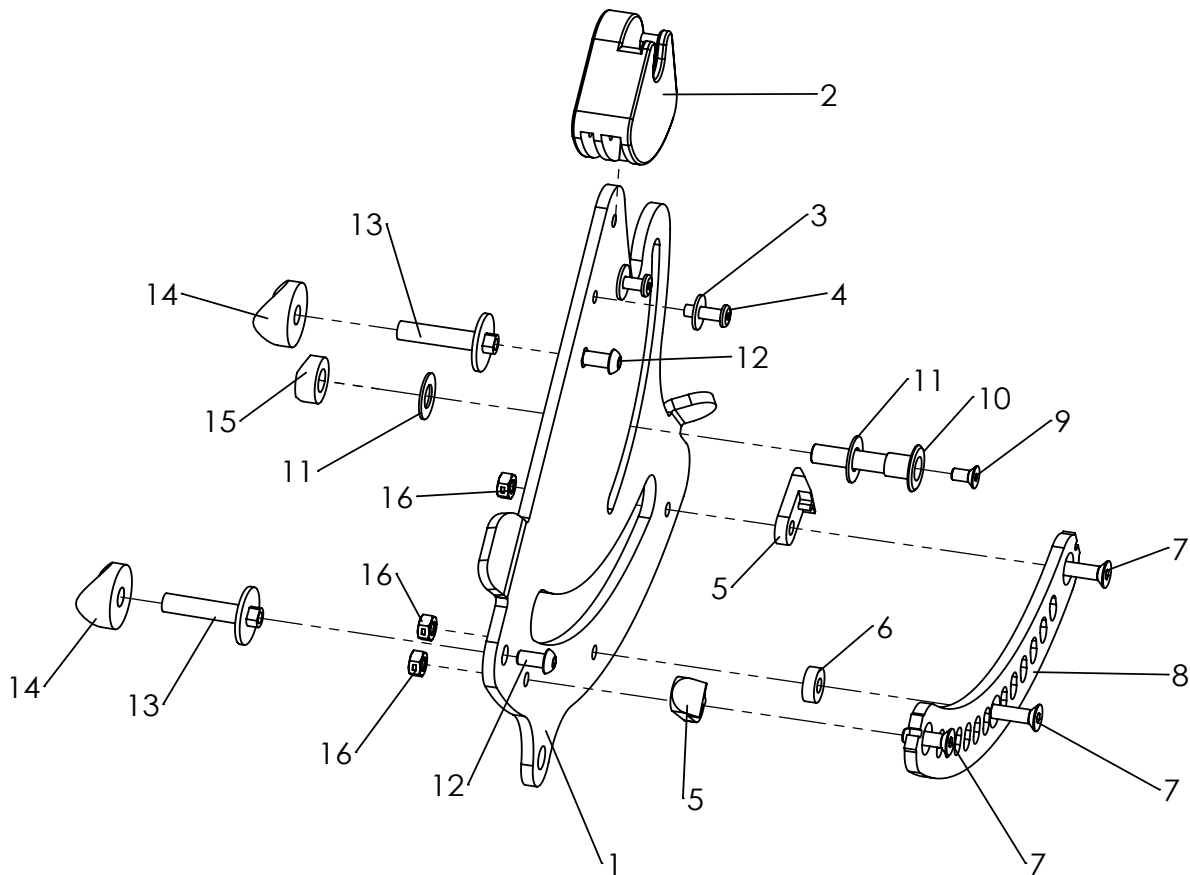


Pos	Article number	Articel description	Quantity	Unit
1-16	3005-1410E-000	Seat bearing plate, left, size 2, complete	1	Pc.
	3005-1420E-000	Seat bearing plate, right, size 2, complete	1	Pc.
1-4	3005-1411E-000	Seat bearing plate, left	1	Pc.
	3005-1421E-000	Seat bearing plate, right	1	Pc.
2	3231-1516	Scantlings bearing	1	Pc.
3	1055-0005	Washer A5,3	2	Pc.
4	1062-0516	Blind rivet F5x16	2	Pc.
5	3231-1514	Latch ramp	2	Pc.
6	1113-0051	Washer PA 6,4x15x5	1	Pc.
7	1024-0620	Hexagon socket countersunk head screw M6x20	3	Pc.
8	3231-1513-006	Scantlings rail	1	Pc.
9	1024-0510	Hexagon socket countersunk head screw M5x10	1	Pc.
10	3004-1344	Guide shaft seat bearing short	1	Pc.
11	1113-0050	Washer PA 10x21x1	2	Pc.
12	1029-6001	Button head socket screw,Tufflock M6x12	2	Pc.
13	3231-1515	Bearing axle swivel arm short	2	Pc.
14	1111-0006	Saddle washer PA black 28x8.4x5	2	Pc.
15	3004-1342	Bearing guide spacer, shaft short	1	Pc.
16	1049-0068	BILOC-nut M6	3	Pc.

Spare parts

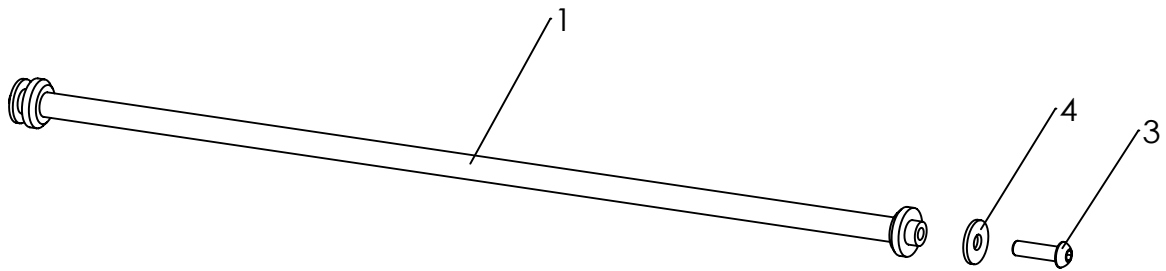


Pos	Article number	Articel description	Quantity	Unit
1-3	3004-1430E-032	Cross strut Cobra size 1, complete	1	Pc.
	3005-1430E-032	Cross strut Cobra size 2, complete	1	Pc.
1	3004-1430-032	Cross strut Cobra size 1	1	Pc.
2	3005-1430-032	Cross strut Cobra size 2	1	Pc.
3	1029-0620	Button head socket screw M6x20	2	Pc.
4	1055-0006	Washer A6,4	2	Pc.

Spare parts

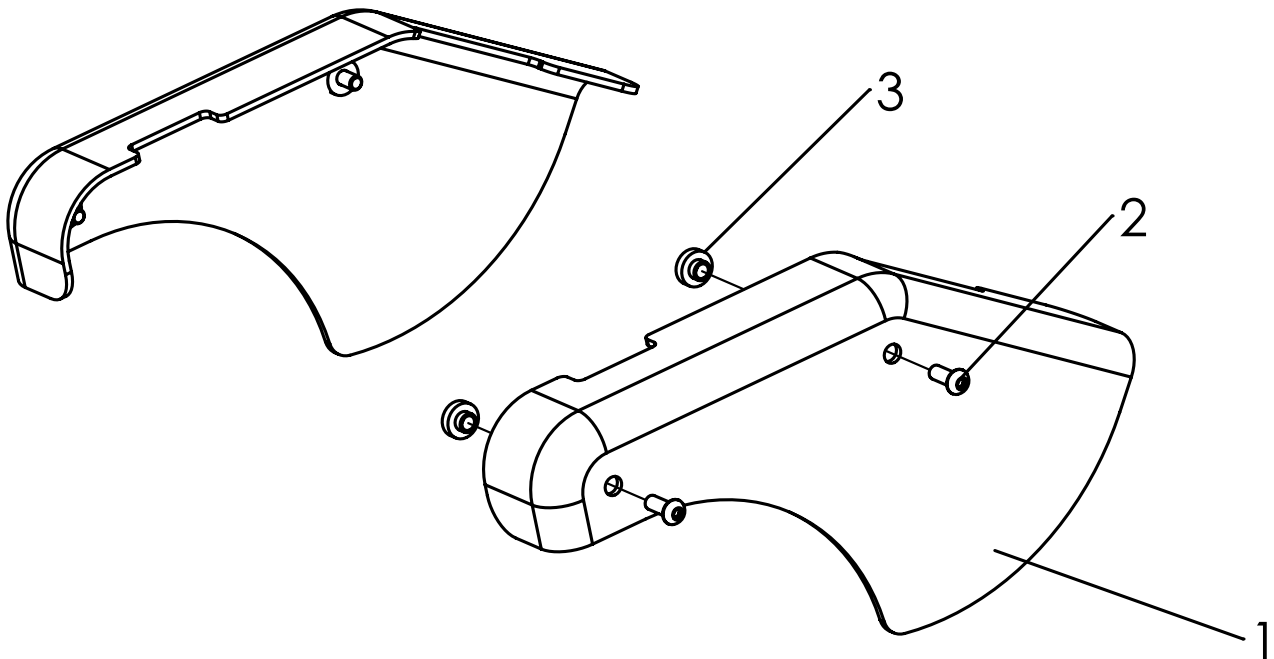

Pos	Article number	Articel description	Quantity	Unit
1-3	3004-1500E	Frame cover, wide for size 1, complete	1	Pc
	3005-1500E	Frame cover, narrow for size 2, complete	1	Pc.
1	3004-1331	Frame cover, narrow, left, for size 2	1	Pc.
	3004-1332	Frame cover, narrow, right, for size 2	1	Pc.
	3004-1333	Frame cover, wide, left, for size 1 and Mini	1	Pc.
	3004-1334	Frame cover, wide, right, for size 1 and Mini	1	Pc.
2	1029-0616	Button head socket screw M6x16	4	Pc.
3	3004-1336	Spacer bushing for cover	4	Pc.

Spare parts

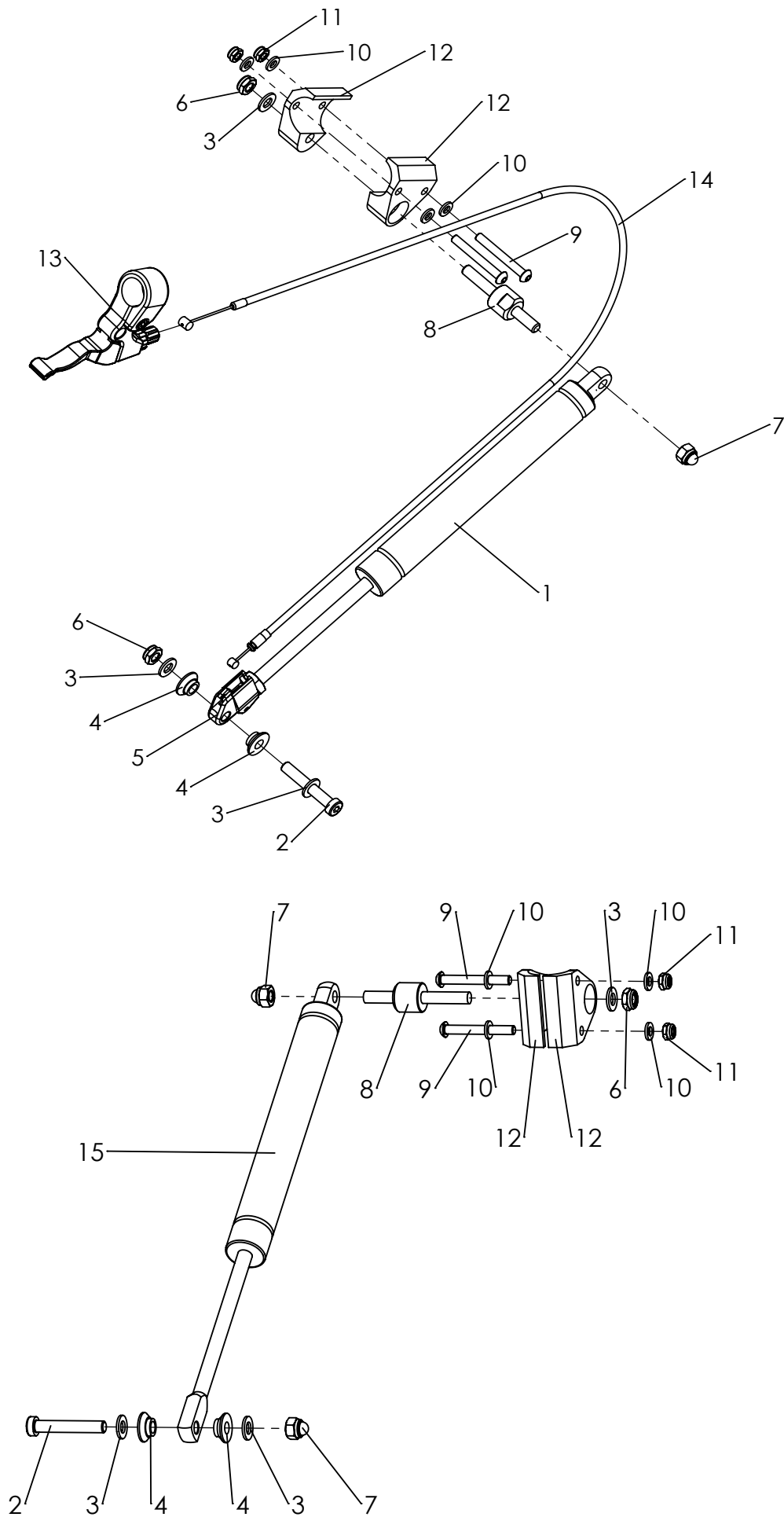


Spare parts

Pos	Article number	Articel description	Quantity	Unit
1-15	3004-1602E	Double Gas spring lift device, complete	1	Pc.
1	1136-0004	Gas strut 400N	1	Pc.
2	1022-0850	Hexagon socket head cap screw M8x50	2	Pc.
3	1051-0008	Washer A8,4	6	Pc.
4	3004-1611	Gas spring spacer	4	Pc.
5	1136-0002	Activator for gas strut	1	Pc.
6	1042-0008	Hexagon nut self-locking M8	3	Pc.
7	1044-0008	Hexagonal domed cap nut self-locking M8	3	Pc.
8	3004-1622	Gas spring bearing axle	2	Pc.
9	1029-0650	Button head socket screw M6x50	4	Pc.
10	1051-0006	Washer A6,4	8	Pc.
11	1042-0006	Hexagon nut self-locking M6	4	Pc.
12	3004-1621	Upper spring bearing black anodized	4	Pc.
13	1209-0015	Brake lever aluminium for 3 fingers use	1	Pc.
14	1209-0033	Bowden cable BZ 800x860x1,5	1	Pc.
15	1136-0005	Gas strut 400N	1	Pc.

For COBRA with two Gas springs (2x 400N)

Spare parts

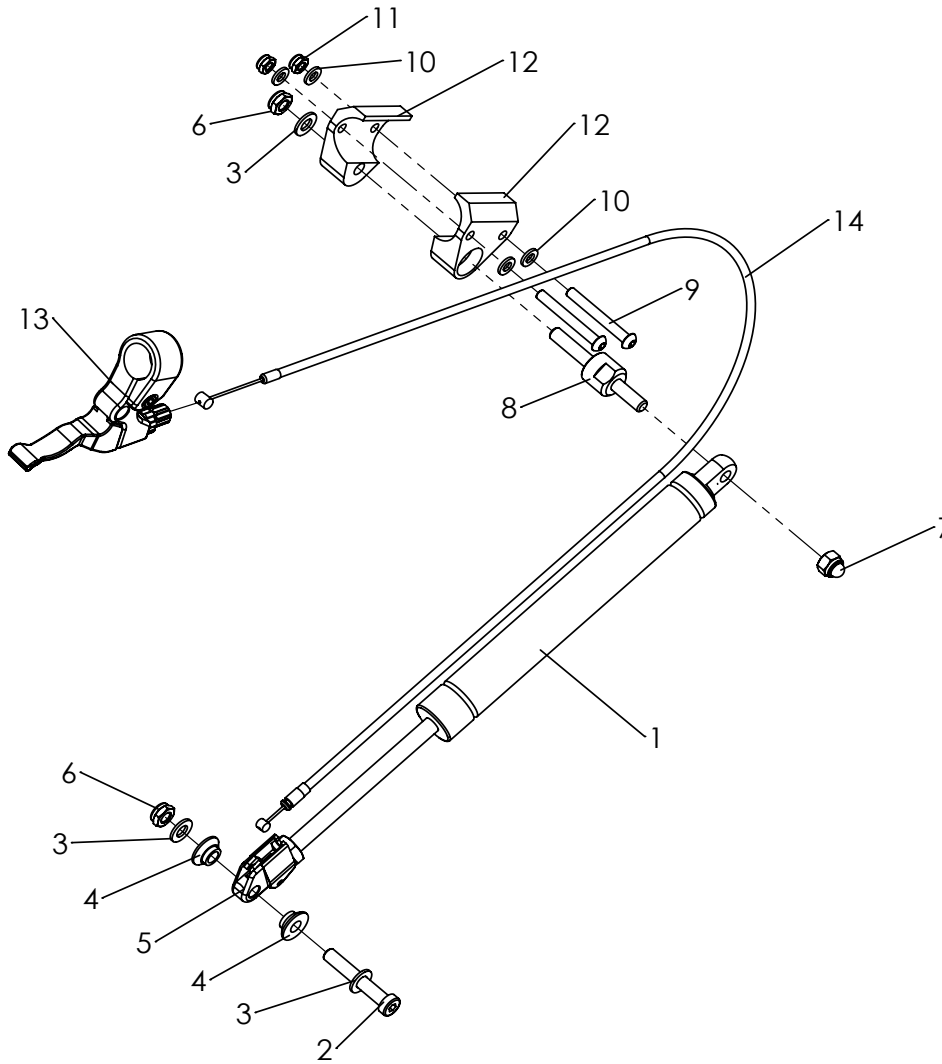

Pos	Article number	Articel description	Quantity	Unit
1-14	3004-1601E	Single Gas spring lift device, complete	1	Pc.
1	1136-0003	Gas strut 700N	1	Pc.
2	1022-0850	Hexagon socket head cap screw M8x50	2	Pc.
3	1051-0008	Washer A8,4	6	Pc.
4	3004-1611	Gas spring spacer	4	Pc.
5	1136-0002	Activator for gas strut	1	Pc.
6	1042-0008	Hexagon nut self-locking M8	3	Pc.
7	1044-0008	Hexagonal domed cap nut self-locking M8	3	Pc.
8	3004-1622	Gas spring bearing axle	2	Pc.
9	1029-0650	Button head socket screw M6x50	4	Pc.
10	1051-0006	Washer A6,4	8	Pc.
11	1042-0006	Hexagon nut self-locking M6	4	Pc.
12	3004-1621	Upper spring bearing black anodized	4	Pc.
13	1209-0015	Brake lever aluminium for 3 fingers use	1	Pc.
14	1209-0033	Bowden cable BZ 800x860x1,5	1	Pc.

For COBRA with one Gas spring (1x 700N)

HOGGI GmbH
Eulerstraße 27 • 56235 Ransbach-Baumbach • Deutschland
Telefon: (+49) 2623 / 92 499-0 • Telefax: (+49) 2623 / 92 499-99
E-mail: info@hoggi.de • www.hoggi.de
